


(Discontinued 1) Push and Pull to Lock Wheel Locks

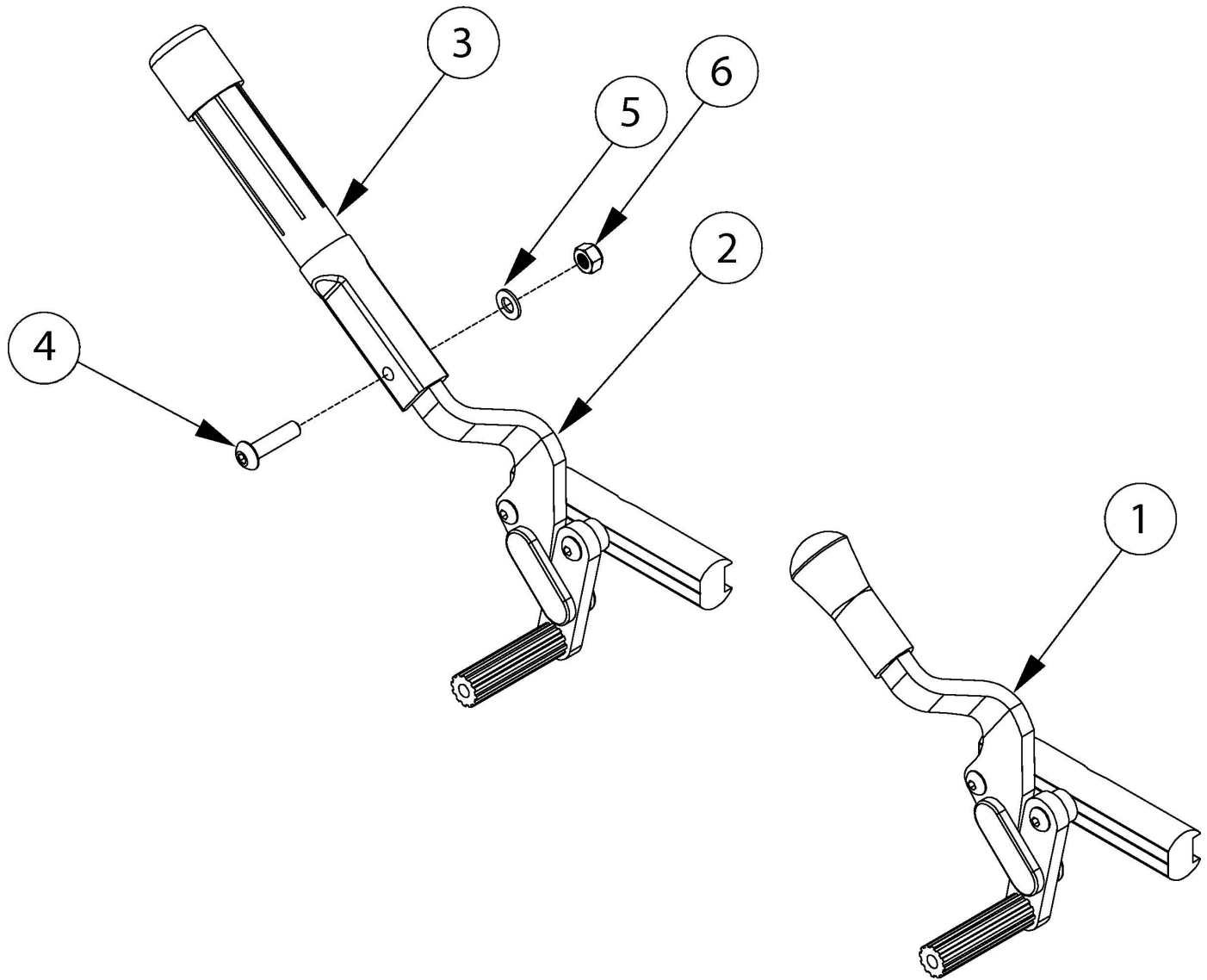

Item Number	Part Number	Description	Quantity
1a	100627	Wheel Lock Push Right <i>Used prior to 5/22/13</i>	1
1b	100628	Wheel Lock Push Left <i>Used prior to 5/22/13</i>	1
1c	100645	Wheel Lock Pull Right <i>Used prior to 5/31/13</i>	1
1d	100644	Wheel Lock Pull Left <i>Used prior to 5/31/13</i>	1
2a, 3-6	100647	Wheel Lock Push w/ Ext Right	1

Item Number	Part Number	Description	Quantity
		<i>Used prior to 6/22/13</i>	
2b, 3-6	100646	Wheel Lock Push w/ Ext Left <i>Used prior to 6/22/13</i>	1
2c, 3-6	100852	Wheel Lock Pull w/ Ext Right <i>Used prior to 5/11/13</i>	1
2d, 3-6	100853	Wheel Lock Pull w/ Ext Left <i>Used prior to 5/11/13</i>	1
3-6	100854	Brake Extension Replacement <i>Only available with 100646, 100647, 100852 and 100853 wheel locks</i>	1
4	101117	M5 x 22 BTN SHCS BLK ZC	1
5	101856	Flat Washer, M5, Blk Zn	1
6	100657	M5 Nylock Nut Blk Zn	1