



BATTERY REPLACEMENT WITH TILT REPLACEMENT INSTRUCTION SHEET







Please read these instructions carefully before beginning the assembly. Failure to understand and follow assembly instructions may result in injury to technician and/or end user and may void the warranty. If you have any questions call Sunrise Medical Technical Support at +1 (800) 333-4000.

Parts included:

1. 2 ea 50Ah Battery
2. 1 ea Female adhesive strip
3. 1 ea Male adhesive strip

Required tools:

- | | |
|------------------------------------|-----------------------|
| 1. 5mm hex key | 3. 4mm hex key |
| 2. 2x 13mm socket wrench or wrench | 4. 10mm socket wrench |
| | 5. Torque wrench |

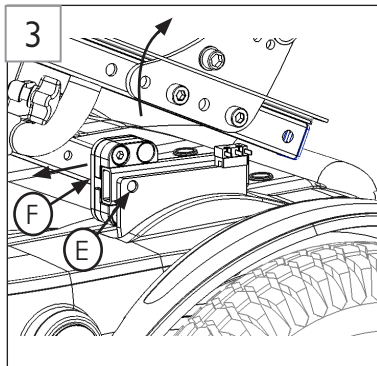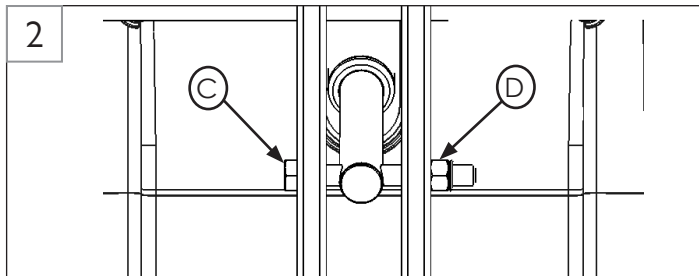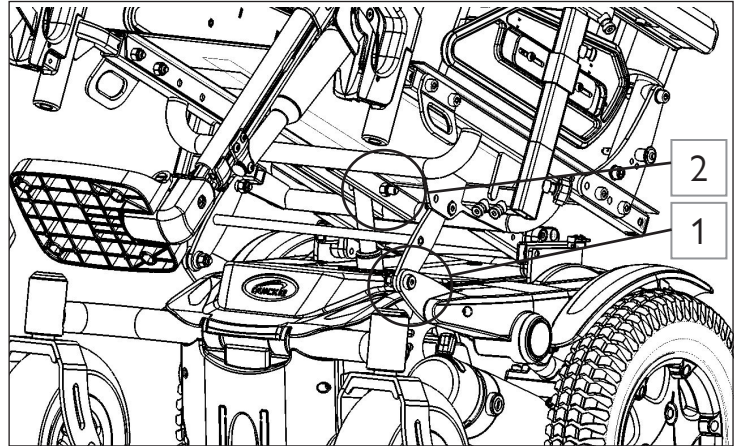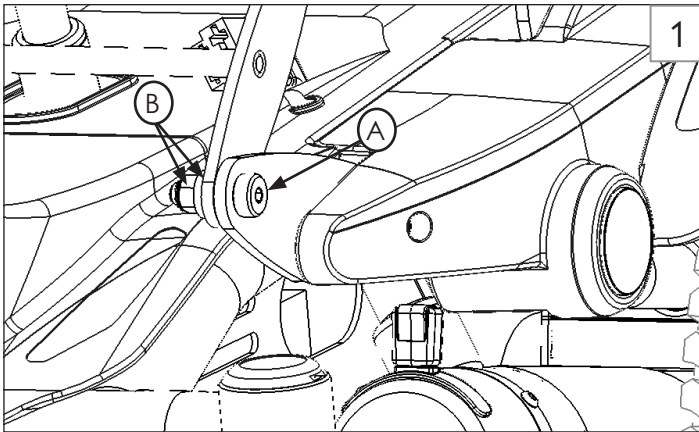

Note: place a cloth between the mudguard and the seat rails

A. Replacing the Batteries on a Q200R with Tilt

1. Remove Bolt (A), Washer and Nut (B) on both the left and right sides. (Fig 1)
2. Remove Bolt (C) and Nut (D) from tilt actuator. (Fig 2)
Note: extending the tilt will help with accessing the bolts.
3. Manually tilt the seat frame back. Place a cloth on top of the mudguard to protect the shroud and decal. Pull the seat forward on the rear slide block (F) until it reaches the rear mount stopping pin (E) as shown in figure 3.

⚠️ DEALER/TECHNICIAN WARNING

Attention dealers and qualified technicians, do not operate or service this wheelchair without first reading the owners manual. If you do not understand the instructions and warnings in the owners manual please contact the Sunrise Medical Technical Service Department before operating and/or servicing the Zippie wheelchair. Failure to do so may result in damage and/or injury.

Find more information and important warnings see the wheelchair owner's manual or at: www.SunriseMedical.com.

Sunrise Medical 2842 N. Business Park Ave Fresno, CA 93727 USA

Customer service: +1(800) 333-4000

or visit www.SunriseMedical.com





© 2021 Sunrise Medical (US) LLC
01/2022 255625 Rev.A Page 1 of 2



BATTERY REPLACEMENT WITH TILT REPLACEMENT INSTRUCTION SHEET








Please read these instructions carefully before beginning the assembly. Failure to understand and follow assembly instructions may result in injury to technician and/or end user and may void the warranty. If you have any questions call Sunrise Medical Technical Support at +1 (800) 333-4000.

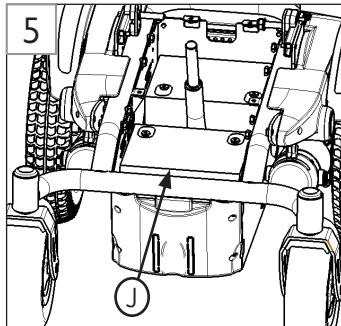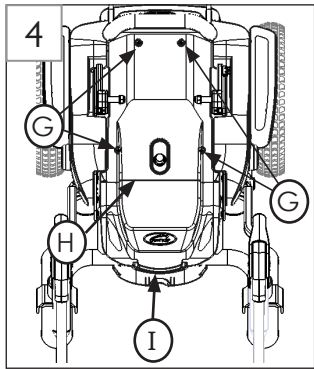

A. Replacing the Batteries on a Q200R with Tilt (continued)

4. Remove front battery cover (I) by lifting up and forward on the front hand grip. Remove the back battery cover (H) by removing the 4 screws (G). (Fig 4) Note: retracting the tilt actuator will help with removing the battery cover.
5. Disconnect the battery harnesses from the front and rear battery. Use the handles on the front battery (H) to lift it out of the battery box. (Fig 5)
6. Unplug actuator harness from controller or ism. The controller is located behind the rear shroud (L). To remove the rear shroud and access the controller, remove 4 screws (M) shown in figure 6B. The ism (N) is mounted on the front battery with adhesive hook and loop (P and O) shown in figure 6C. Remove the 6 bolts and nuts (K) shown in Figure 6A to remove the tilt actuator and lower mount.
7. Slide the rear battery (Q) forward to remove. (Fig 7)
8. Install the new rear battery (Q). Refer to figure 8 for the correct battery post position. Re-install the lower tilt mount, torque all 6 screws (K) to 2-4 Nm as shown in Figure 5. Re-route the actuator harness to the proper module.
9. Install the new front battery (J). Refer to figure 8 for the correct battery post position.
10. Re-install the battery harnesses. Refer to the battery schematic located inside of the battery shroud (H) removed in step 4. Re-route and connect the actuator harness.
11. For chairs with an ism, install new adhesive hook and loop (P and O) provided between the front battery and ism module as shown in figure 6C.
12. For chairs without an ism, re-install the rear shroud (L) after re-connecting the actuator. Torque all 4 screws to 4-6 Nm.
13. Re-install the battery cover (H) using the previous 4 screws (G). Torque to 4-6 Nm. (Fig 4)
14. Lower the seat and re-install the bolt (C) and nut (D) to the tilt actuator. Torque to 4-5 Nm. (Fig 2)
15. Re-install Bolt (A), Washer and Nut (B) on both the left and right sides as shown in figure 1. Torque to 6-8 Nm.
16. Slide the front battery cover (G) back, and push down on the front hand grip to latch. (Fig 4)

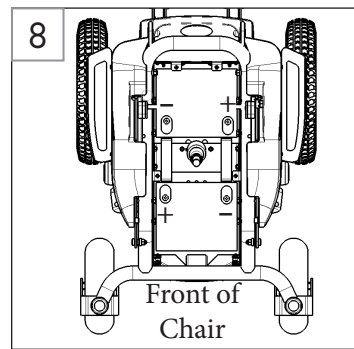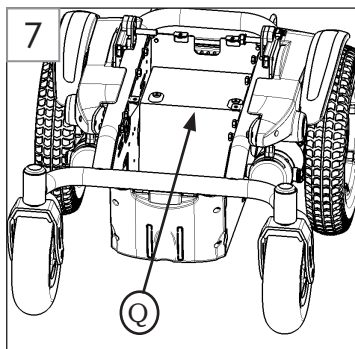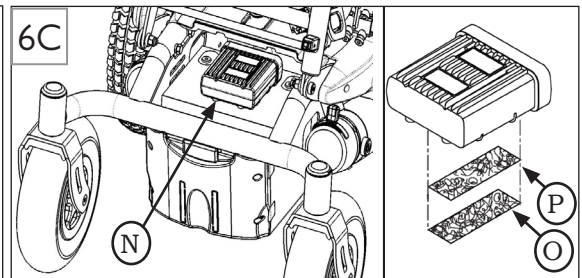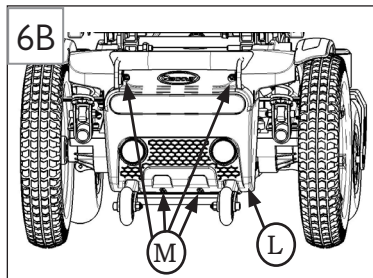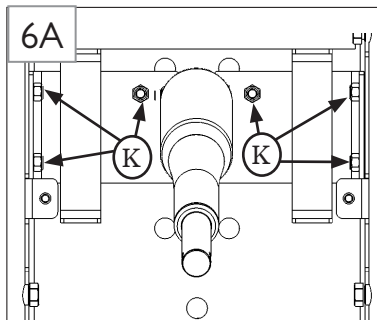

⚠ DEALER/TECHNICIAN WARNING

Attention dealers and qualified technicians, do not operate or service this wheelchair without first reading the owners manual. If you do not understand the instructions and warnings in the owners manual please contact the Sunrise Medical Technical Service Department before operating and/or servicing the Zippie wheelchair. Failure to do so may result in damage and/or injury. Find more information and important warnings see the wheelchair owner's manual or at: www.SunriseMedical.com.

Sunrise Medical 2842 N. Business Park Ave Fresno, CA 93727 USA

Customer service: +1(800) 333-4000
or visit www.SunriseMedical.com



© 2021 Sunrise Medical (US) LLC
01/2022 255625 Rev.A Page 2 of 2