


Catalyst E Wheel Locks - Attendant Lock

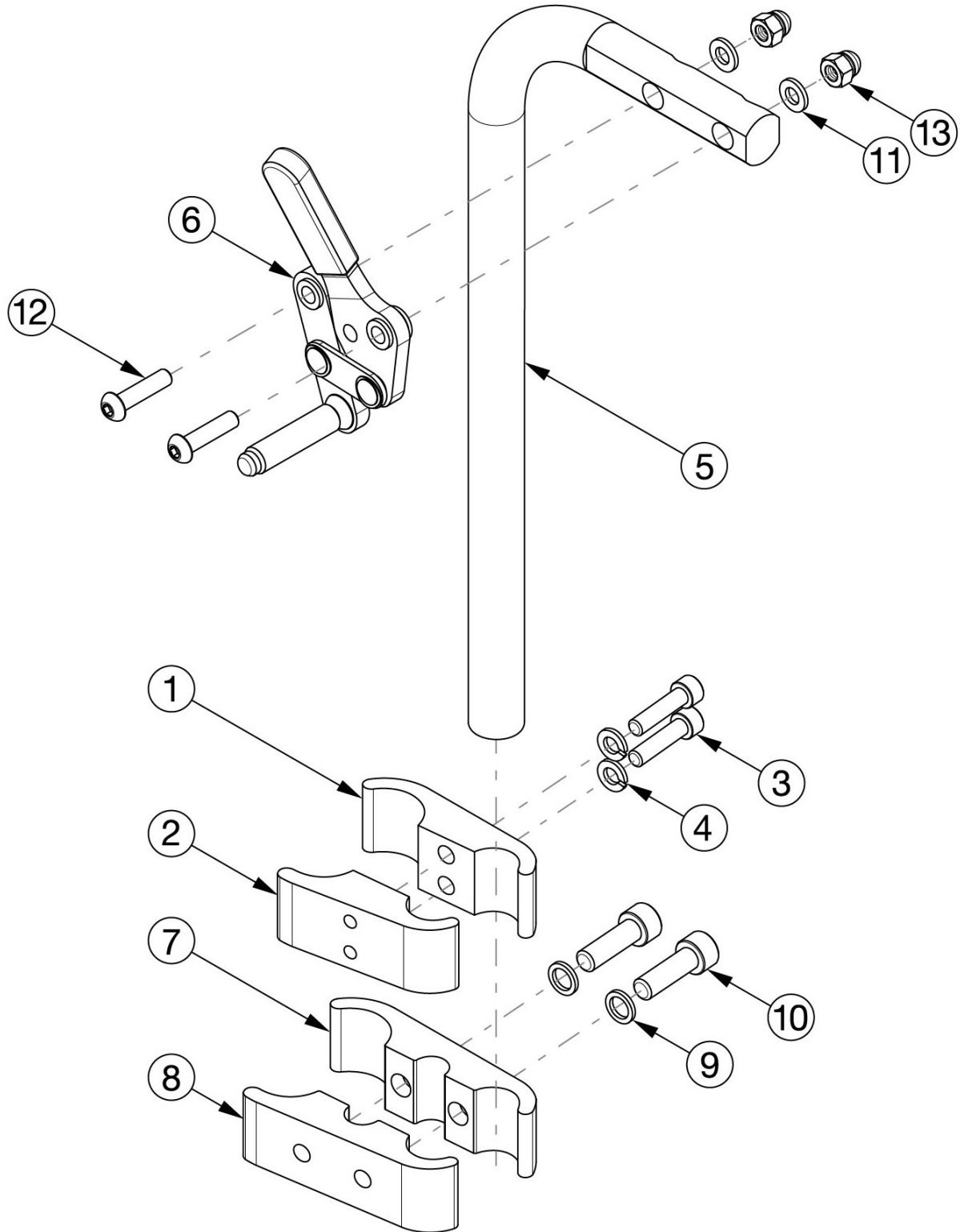

Standard Clamps utilized with 20" wheels in combination with any Center of Gravity positions and 22" wheels in combination with Center of Gravity position 1. <p>

Extended Clamps utilized with 22" wheels in combination with Center of Gravity positions 2, 3, or 4 and with 24" wheels in combination with any Center of Gravity position.

Item Number	Part Number	Description	Quantity	Retail
1-4, 5a,5b,6a,6b,12 -13	115840	Attendant Wheel Lock, Std Clamp	1	\$148.62
5a,5b,6a,6b,7- 13	115841	Attendant Wheel Lock, Ext Clamp	1	\$159.14
1-4	115842	Std Clamp	1	\$48.36
7-10	115843	Extended Clamp	1	\$58.88
1	002831	Wheel Lock Clamp 1.125 to .75 C'Bored Long Clk <i>Standard Clamp</i>	1	\$9.93
2	002832	Wheel Lock Clamp 1.125 to .75 Threaded Long Clk <i>Standard Clamp</i>	1	\$9.93
3	100780	M6 x 25 SHCS ZC	2	\$2.21
4	101835	M6 Lock Washer Blk Zn	2	\$0.56
5a	112029	Attendant Wheel Lock Mount Bar, Right	1	\$29.40
5b	112030	Attendant Wheel Lock Mount Bar, Left	1	\$29.40
6a	200252	Wheel Lock Push Left	1	\$37.15
6b	200251	Wheel Lock Push Right	1	\$37.15
7	002814	Wheel Lock Clamp 1.125 to .75 C'Bored Extra Long Clk <i>Extended Clamp</i>	1	\$11.04
8	002815	Wheel Lock Clamp 1.125 to .75 Threaded Extra Long Clk <i>Extended Clamp</i>	1	\$11.04
9	101701	Washer 3.315/.350 x .425/.525 x .059/.067 18-8 F/W	2	\$2.76
10	102864	M8 x 25mm SHCS, BLK ZC	2	\$1.66
11	100746	M6 Flat Washer Blk Zn	2	\$1.66
12	101593	M6 x 25 BTN SHCS BLKZC	2	\$1.66
13	100558	M6 NYL INS Domed Cap L/N ZC	2	\$1.10