


Flip Reclining Backrest - Growth

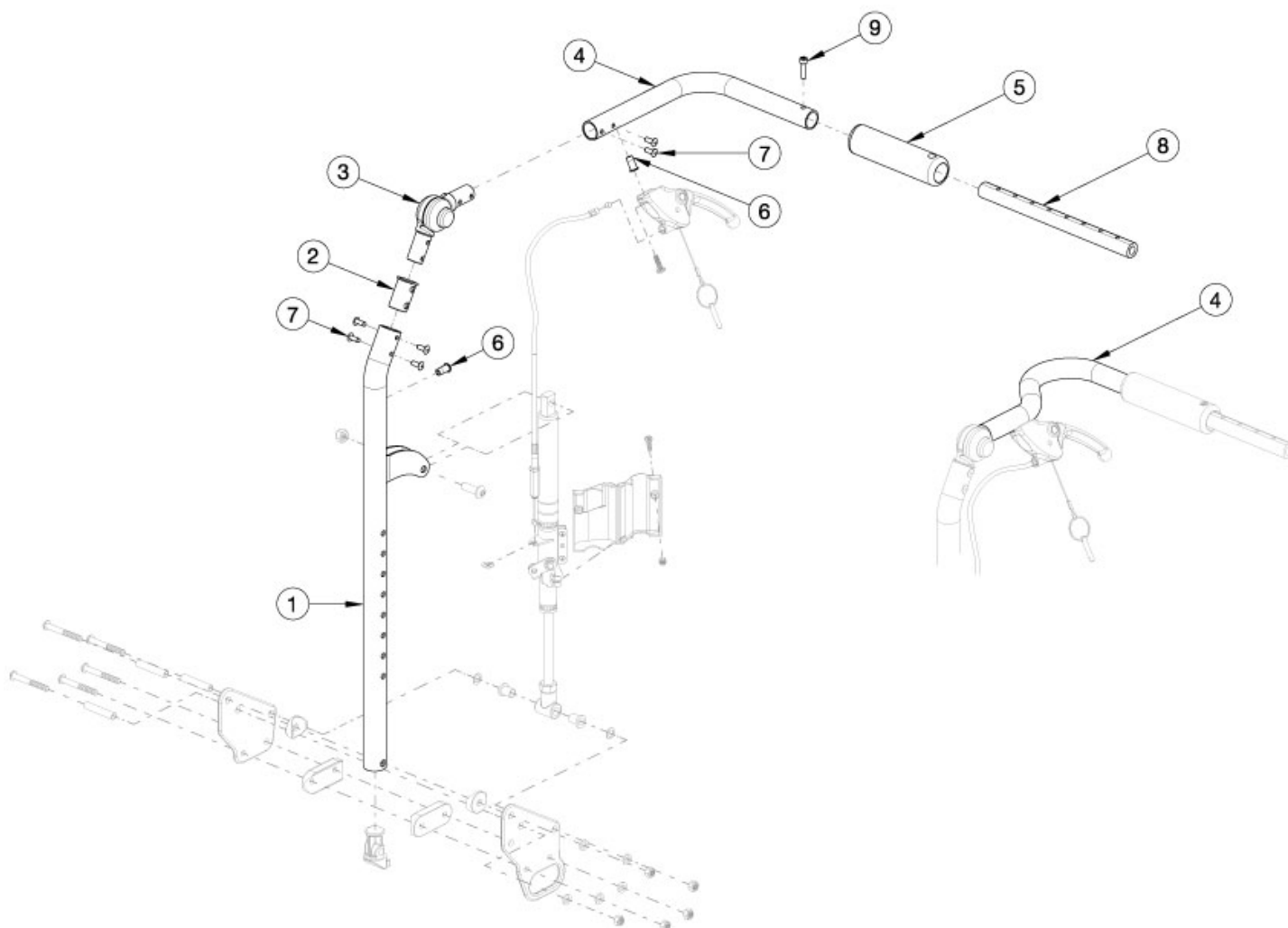

Has your width changed? Will the push handle (#4) accommodate your new width? If no, please select the appropriate back tube assembly. Callout #4: 10"-13" widths receive the curved handle and 14"-18" widths receive the "L" shaped handle.

Item Number	Part Number	Description	Quantity
1a, 2, 3, 4a, 5, 6, 7	107681	Recline Back Tube Subassembly, Short, 14-18" Wide, Right Flip	1
1a, 2, 3, 4b, 5, 6, 7	107682	Recline Back Tube Subassembly, Short, 14-18" Wide, Left Flip	1
1b, 2, 3, 4a, 5, 6, 7	107683	Recline Back Tube Subassembly, Tall, 14-18" Wide, Right Flip	1
1b, 2, 3, 4b, 5, 6, 7	107684	Recline Back Tube Subassembly, Tall, 14-18" Wide, Left Flip	1
1a, 2, 3, 4c, 5, 6, 7	107685	Recline Back Tube Subassembly, Ped, Short, 10-13" Wide, Right Flip	1
1a, 2, 3, 4d, 5, 6, 7	107686	Recline Back Tube Subassembly, Ped, Short, 10-13" Wide, Left Flip	1
1b, 2, 3, 4c, 5, 6, 7	107687	Recline Back Tube Subassembly, Ped, Tall, 10-13" Wide, Right Flip	1

Item Number	Part Number	Description	Quantity
1b, 2, 3, 4d, 5, 6, 7	107688	Recline Back Tube Subassembly, Ped, Tall, 10-13" Wide, Left Flip	1
8	002578	Growth Strut, Stroller, Short	1
9	101843	M5 x 20 SHCS Blk Zn	1