


Liberty Standard Elevating Leg Rest

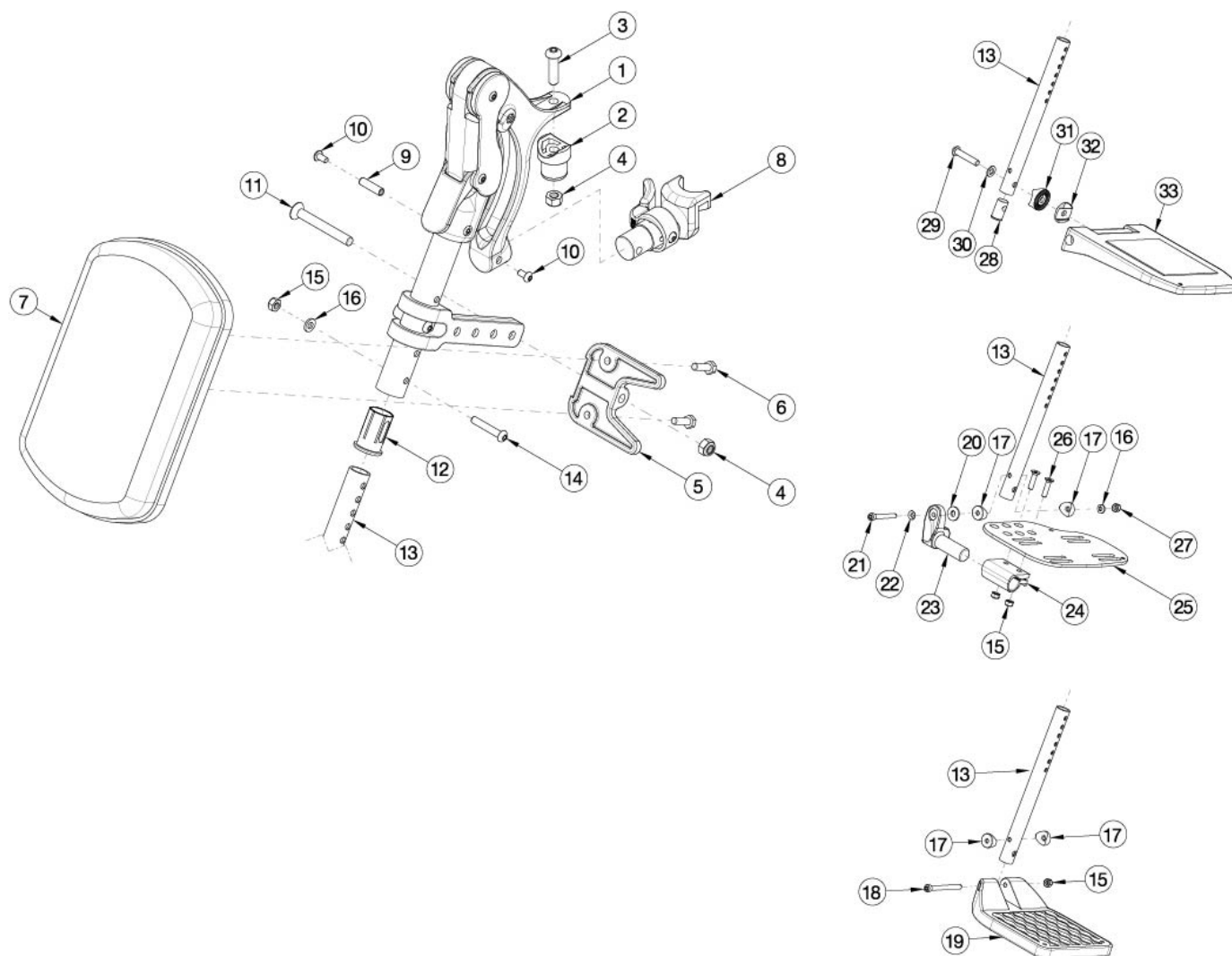

Item Number	Part Number	Description	Quantity
1-6, 7b, 8a, 8b, 9-12, 13a, 14-18, 19a	123742	Standard ELR Assembly 14, Composite Footplate, Short Ext, 4-Way Out Release, Pair	1
1-6, 7a, 8a, 8b, 9-12, 13a, 14-18, 19a	123743	Standard ELR Assembly 15, Composite Footplate, Short Ext, 4-Way Out Release, Pair	1
1-6, 7a, 8a, 8b, 9-12, 13a, 14-18, 19b	123744	Standard ELR Assembly 16-17, Composite Footplate, Short Ext, 4-Way Out Release, Pair	1
1-6, 7a, 8a, 8b, 9-12, 13a, 14-18, 19c	123745	Standard ELR Assembly 18-24, Composite Footplate, Short Ext, 4-Way Out Release, Pair	1
1-6, 7b, 8a, 9-12, 13a, 14-18, 19a	123746	Standard ELR Assembly 14, Composite Footplate, Short Ext, 4-Way Out Release, Right	1

Item Number	Part Number	Description	Quantity
1-6, 7a, 8a, 9-12, 13a, 14-18, 19a	123747	Standard ELR Assembly 15, Composite Footplate, Short Ext, 4-Way Out Release, Right	1
1-6, 7a, 8a, 9-12, 13a, 14-18, 19b	123748	Standard ELR Assembly 16-17, Composite Footplate, Short Ext, 4-Way Out Release, Right	1
1-6, 7a, 8a, 9-12, 13a, 14-18, 19c	123749	Standard ELR Assembly 18-24, Composite Footplate, Short Ext, 4-Way Out Release, Right	1
1-6, 7b, 8b, 9-12, 13a, 14-18, 19a	123750	Standard ELR Assembly 14, Composite Footplate, Short Ext, 4-Way Out Release, Left	1
1-6, 7a, 8b, 9-12, 13a, 14-18, 19a	123751	Standard ELR Assembly 15, Composite Footplate, Short Ext, 4-Way Out Release, Left	1
1-6, 7a, 8b, 9-12, 13a, 14-18, 19b	123752	Standard ELR Assembly 16-17, Composite Footplate, Short Ext, 4-Way Out Release, Left	1
1-6, 7a, 8b, 9-12, 13a, 14-18, 19c	123753	Standard ELR Assembly 18-24, Composite Footplate, Short Ext, 4-Way Out Release, Left	1
1-6, 7b, 8a, 8b, 9-12, 13b, 14-18, 19a	123754	Standard ELR Assembly 14, Composite Footplate, Medium Ext, 4-Way Out Release, Pair	1
1-6, 7a, 8a, 8b, 9-12, 13b, 14-18, 19a	123755	Standard ELR Assembly 15, Composite Footplate, Medium Ext, 4-Way Out Release, Pair	1
1-6, 7a, 8a, 8b, 9-12, 13b, 14-18, 19b	123756	Standard ELR Assembly 16-17, Composite Footplate, Medium Ext, 4-Way Out Release, Pair	1
1-6, 7a, 8a, 8b, 9-12, 13b, 14-18, 19c	123757	Standard ELR Assembly 18-24, Composite Footplate, Medium Ext, 4-Way Out Release, Pair	1
1-6, 7b, 8a, 9-12, 13b, 14-18, 19a	123758	Standard ELR Assembly 14, Composite Footplate, Medium Ext, 4-Way Out Release, Right	1
1-6, 7a, 8a, 9-12, 13b, 14-18, 19a	123759	Standard ELR Assembly 15, Composite Footplate, Medium Ext, 4-Way Out Release, Right	1
1-6, 7a, 8a, 9-12, 13b, 14-18, 19b	123760	Standard ELR Assembly 16-17, Composite Footplate, Medium Ext, 4-Way Out Release, Right	1
1-6, 7a, 8a, 9-12, 13b, 14-18, 19c	123761	Standard ELR Assembly 18-24, Composite Footplate, Medium Ext, 4-Way Out Release, Right	1
1-6, 7b, 8b, 9-12, 13b, 14-18, 19a	123762	Standard ELR Assembly 14, Composite Footplate, Medium Ext, 4-Way Out Release, Left	1
1-6, 7a, 8b, 9-12, 13b, 14-18, 19a	123763	Standard ELR Assembly 15, Composite Footplate, Medium Ext, 4-Way Out Release, Left	1
1-6, 7a, 8b, 9-12, 13b, 14-18, 19b	123764	Standard ELR Assembly 16-17, Composite Footplate, Medium Ext, 4-Way Out Release, Left	1
1-6, 7a, 8b, 9-12, 13b, 14-18, 19c	123765	Standard ELR Assembly 18-24, Composite Footplate, Medium Ext, 4-Way Out Release, Left	1

Item Number	Part Number	Description	Quantity
1-6, 7b, 8a, 8b, 9-12, 13c, 14-18, 19a	123766	Standard ELR Assembly 14, Composite Footplate, Long Ext, 4-Way Out Release, Pair	1
1-6, 7a, 8a, 8b, 9-12, 13c, 14-18, 19a	123767	Standard ELR Assembly 15, Composite Footplate, Long Ext, 4-Way Out Release, Pair	1
1-6, 7a, 8a, 8b, 9-12, 13c, 14-18, 19b	123768	Standard ELR Assembly 16-17, Composite Footplate, Long Ext, 4-Way Out Release, Pair	1
1-6, 7a, 8a, 8b, 9-12, 13c, 14-18, 19c	123769	Standard ELR Assembly 18-24, Composite Footplate, Long Ext, 4-Way Out Release, Pair	1
1-6, 7b, 8a, 9-12, 13c, 14-18, 19a	123770	Standard ELR Assembly 14, Composite Footplate, Long Ext, 4-Way Out Release, Right	1
1-6, 7a, 8a, 9-12, 13c, 14-18, 19a	123771	Standard ELR Assembly 15, Composite Footplate, Long Ext, 4-Way Out Release, Right	1
1-6, 7a, 8a, 9-12, 13c, 14-18, 19b	123772	Standard ELR Assembly 16-17, Composite Footplate, Long Ext, 4-Way Out Release, Right	1
1-6, 7a, 8a, 9-12, 13c, 14-18, 19c	123773	Standard ELR Assembly 18-24, Composite Footplate, Long Ext, 4-Way Out Release, Right	1
1-6, 7b, 8b, 9-12, 13c, 14-18, 19a	123774	Standard ELR Assembly 14, Composite Footplate, Long Ext, 4-Way Out Release, Left	1
1-6, 7a, 8b, 9-12, 13c, 14-18, 19a	123775	Standard ELR Assembly 15, Composite Footplate, Long Ext, 4-Way Out Release, Left	1
1-6, 7a, 8b, 9-12, 13c, 14-18, 19b	123776	Standard ELR Assembly 16-17, Composite Footplate, Long Ext, 4-Way Out Release, Left	1
1-6, 7a, 8b, 9-12, 13c, 14-18, 19c	123777	Standard ELR Assembly 18-24, Composite Footplate, Long Ext, 4-Way Out Release, Left	1
1-6, 7a, 8a, 8b, 9-12, 13a, 14-16, 28-32, 33a	123778	Standard ELR Assembly 15-16 Comp Angle Adj FP Short Ext 4-Way Out Release Pair	1
1-6, 7a, 8a, 8b, 9-12, 13a, 14-16, 28-32, 33b	123779	Standard ELR Assembly 17-18 Comp Angle Adj FP Short Ext 4-Way Out Release Pair	1
1-6, 7a, 8a, 8b, 9-12, 13a, 14-16, 28-32, 33c	123780	Standard ELR Assembly 19-22 Comp Angle Adj FP Short Ext 4-Way Out Release Pair	1
1-6, 7a, 8a, 9-12, 13a, 14-16, 28-32, 33a	123781	Standard ELR Assembly 15-16 Comp Angle Adj FP Short Ext 4-Way Out Release Right	1
1-6, 7a, 8a, 9-12, 13a, 14-16, 28-32, 33b	123782	Standard ELR Assembly 17-18 Comp Angle Adj FP Short Ext 4-Way Out Release Right	1
1-6, 7a, 8a, 9-12, 13a, 14-16, 28-32, 33c	123783	Standard ELR Assembly 19-22 Comp Angle Adj FP Short Ext 4-Way Out Release Right	1
1-6, 7a, 8b, 9-12, 13a, 14-16, 28-32, 33a	123784	Standard ELR Assembly 15-16 Comp Angle Adj FP Short Ext 4-Way Out Release Left	1

Item Number	Part Number	Description	Quantity
1-6, 7a, 8b, 9-12, 13a, 14-16, 28-32, 33b	123785	Standard ELR Assembly 17-18 Comp Angle Adj FP Short Ext 4-Way Out Release Left	1
1-6, 7a, 8b, 9-12, 13a, 14-16, 28-32, 33c	123786	Standard ELR Assembly 19-22 Comp Angle Adj FP Short Ext 4-Way Out Release Left	1
1-6, 7a, 8a, 8b, 9-12, 13b, 14-16, 28-32, 33a	123787	Standard ELR Assembly 15-16 Comp Angle Adj FP Medium Ext 4-Way Out Release Pair	1
1-6, 7a, 8a, 8b, 9-12, 13b, 14-16, 28-32, 33b	123788	Standard ELR Assembly 17-18 Comp Angle Adj FP Medium Ext 4-Way Out Release Pair	1
1-6, 7a, 8a, 8b, 9-12, 13b, 14-16, 28-32, 33c	123789	Standard ELR Assembly 19-22 Comp Angle Adj FP Medium Ext 4-Way Out Release Pair	1
1-6, 7a, 8a, 9-12, 13b, 14-16, 28-32, 33a	123790	Standard ELR Assembly 15-16 Comp Angle Adj FP Medium Ext 4-Way Out Release Right	1
1-6, 7a, 8a, 9-12, 13b, 14-16, 28-32, 33b	123791	Standard ELR Assembly 17-18 Comp Angle Adj FP Medium Ext 4-Way Out Release Right	1
1-6, 7a, 8a, 9-12, 13b, 14-16, 28-32, 33c	123792	Standard ELR Assembly 19-22 Comp Angle Adj FP Medium Ext 4-Way Out Release Right	1
1-6, 7a, 8b, 9-12, 13b, 14-16, 28-32, 33a	123793	Standard ELR Assembly 15-16 Comp Angle Adj FP Medium Ext 4-Way Out Release Left	1
1-6, 7a, 8b, 9-12, 13b, 14-16, 28-32, 33b	123794	Standard ELR Assembly 17-18 Comp Angle Adj FP Medium Ext 4-Way Out Release Left	1
1-6, 7a, 8b, 9-12, 13b, 14-16, 28-32, 33c	123795	Standard ELR Assembly 19-22 Comp Angle Adj FP Medium Ext 4-Way Out Release Left	1
1-6, 7a, 8a, 8b, 9-12, 13c, 14-16, 28-32, 33a	123796	Standard ELR Assembly 15-16 Comp Angle Adj FP Long Ext 4-Way Out Release Pair	1
1-6, 7a, 8a, 8b, 9-12, 13c, 14-16, 28-32, 33b	123797	Standard ELR Assembly 17-18 Comp Angle Adj FP Long Ext 4-Way Out Release Pair	1
1-6, 7a, 8a, 8b, 9-12, 13c, 14-16, 28-32, 33c	123798	Standard ELR Assembly 19-22 Comp Angle Adj FP Long Ext 4-Way Out Release Pair	1
1-6, 7a, 8a, 9-12, 13c, 14-16, 28-32, 33a	123799	Standard ELR Assembly 15-16 Comp Angle Adj FP Long Ext 4-Way Out Release Right	1
1-6, 7a, 8a, 9-12, 13c, 14-16, 28-32, 33b	123800	Standard ELR Assembly 17-18 Comp Angle Adj FP Long Ext 4-Way Out Release Right	1
1-6, 7a, 8a, 9-12, 13c, 14-16, 28-32, 33c	123801	Standard ELR Assembly 19-22 Comp Angle Adj FP Long Ext 4-Way Out Release Right	1
1-6, 7a, 8b, 9-12, 13c, 14-16, 28-32, 33a	123802	Standard ELR Assembly 15-16 Comp Angle Adj FP Long Ext 4-Way Out Release Left	1
1-6, 7a, 8b, 9-12, 13c, 14-16, 28-32, 33b	123803	Standard ELR Assembly 17-18 Comp Angle Adj FP Long Ext 4-Way Out Release Left	1

Item Number	Part Number	Description	Quantity
1-6, 7a, 8b, 9-12, 13c, 14-16, 28-32, 33c	123804	Standard ELR Assembly 19-22 Comp Angle Adj FP Long Ext 4-Way Out Release Left	1
1-6, 7b, 8a, 8b, 9-12, 13a, 14-17, 20-24, 25a, 25b, 26, 27	123805	Standard ELR Assembly 14 Angle Adj FP Short Ext 4-Way Out Release Pair	1
1-6, 7a, 8a, 8b, 9-12, 13a, 14-17, 20-24, 25a, 25b, 26, 27	123806	Standard ELR Assembly 15 Angle Adj FP Short Ext 4-Way Out Release Pair	1
1-6, 7a, 8a, 8b, 9-12, 13a, 14-17, 20-24, 25c, 25d, 26, 27	123807	Standard ELR Assembly 16-17 Angle Adj FP Short Ext 4-Way Out Release Pair	1
1-6, 7a, 8a, 8b, 9-12, 13a, 14-17, 20-24, 25e, 25f, 26, 27	123808	Standard ELR Assembly 18-24 Angle Adj FP Short Ext 4-Way Out Release Pair	1
1-6, 7b, 8a, 9-12, 13a, 14-17, 20-24, 25b, 26, 27	123809	Standard ELR Assembly 14 Angle Adj FP Short Ext 4-Way Out Release Right	1
1-6, 7a, 8a, 9-12, 13a, 14-17, 20-24, 25b, 26, 27	123810	Standard ELR Assembly 15 Angle Adj FP Short Ext 4-Way Out Release Right	1
1-6, 7a, 8a, 9-12, 13a, 14-17, 20-24, 25d, 26, 27	123811	Standard ELR Assembly 16-17 Angle Adj FP Short Ext 4-Way Out Release Right	1
1-6, 7a, 8a, 9-12, 13a, 14-17, 20-24, 25f, 26, 27	123812	Standard ELR Assembly 18-24 Angle Adj FP Short Ext 4-Way Out Release Right	1
1-6, 7b, 8b, 9-12, 13a, 14-17, 20-24, 25a, 26, 27	123813	Standard ELR Assembly 14 Angle Adj FP Short Ext 4-Way Out Release Left	1
1-6, 7a, 8b, 9-12, 13a, 14-17, 20-24, 25a, 26, 27	123814	Standard ELR Assembly 15 Angle Adj FP Short Ext 4-Way Out Release Left	1
1-6, 7a, 8b, 9-12, 13a, 14-17, 20-24, 25c, 26, 27	123815	Standard ELR Assembly 16-17 Angle Adj FP Short Ext 4-Way Out Release Left	1
1-6, 7a, 8b, 9-12, 13a, 14-17, 20-24, 25e, 26, 27	123816	Standard ELR Assembly 18-24 Angle Adj FP Short Ext 4-Way Out Release Left	1
1-6, 7b, 8a, 8b, 9-12, 13b, 14-17, 20-24, 25a, 25b, 26, 27	123817	Standard ELR Assembly 14 Angle Adj FP Medium Ext 4-Way Out Release Pair	1

Item Number	Part Number	Description	Quantity
1-6, 7a, 8a, 8b, 9-12, 13b, 14-17, 20-24, 25a, 25b, 26, 27	123818	Standard ELR Assembly 15 Angle Adj FP Medium Ext 4-Way Out Release Pair	1
1-6, 7a, 8a, 8b, 9-12, 13b, 14-17, 20-24, 25c, 25d, 26, 27	123819	Standard ELR Assembly 16-17 Angle Adj FP Medium Ext 4-Way Out Release Pair	1
1-6, 7a, 8a, 8b, 9-12, 13b, 14-17, 20-24, 25e, 25f, 26, 27	123820	Standard ELR Assembly 18-24 Angle Adj FP Medium Ext 4-Way Out Release Pair	1
1-6, 7b, 8a, 9-12, 13b, 14-17, 20-24, 25b, 26, 27	123821	Standard ELR Assembly 14 Angle Adj FP Medium Ext 4-Way Out Release Right	1
1-6, 7a, 8a, 9-12, 13b, 14-17, 20-24, 25b, 26, 27	123822	Standard ELR Assembly 15 Angle Adj FP Medium Ext 4-Way Out Release Right	1
1-6, 7a, 8a, 9-12, 13b, 14-17, 20-24, 25d, 26, 27	123823	Standard ELR Assembly 16-17 Angle Adj FP Medium Ext 4-Way Out Release Right	1
1-6, 7a, 8a, 9-12, 13b, 14-17, 20-24, 25f, 26, 27	123824	Standard ELR Assembly 18-24 Angle Adj FP Medium Ext 4-Way Out Release Right	1
1-6, 7b, 8b, 9-12, 13b, 14-17, 20-24, 25a, 26, 27	123825	Standard ELR Assembly 14 Angle Adj FP Medium Ext 4-Way Out Release Left	1
1-6, 7a, 8b, 9-12, 13b, 14-17, 20-24, 25a, 26, 27	123826	Standard ELR Assembly 15 Angle Adj FP Medium Ext 4-Way Out Release Left	1
1-6, 7a, 8b, 9-12, 13b, 14-17, 20-24, 25c, 26, 27	123827	Standard ELR Assembly 16-17 Angle Adj FP Medium Ext 4-Way Out Release Left	1
1-6, 7a, 8b, 9-12, 13b, 14-17, 20-24, 25e, 26, 27	123828	Standard ELR Assembly 18-24 Angle Adj FP Medium Ext 4-Way Out Release Left	1
1-6, 7b, 8a, 8b, 9-12, 13c, 14-17, 20-24, 25a, 25b, 26, 27	123829	Standard ELR Assembly 14 Angle Adj FP Long Ext 4-Way Out Release Pair	1
1-6, 7a, 8a, 8b, 9-12, 13c, 14-17, 20-24, 25a, 25b, 26, 27	123830	Standard ELR Assembly 15 Angle Adj FP Long Ext 4-Way Out Release Pair	1
1-6, 7a, 8a, 8b, 9-12, 13c, 14-17, 20-24, 25c, 25d, 26, 27	123831	Standard ELR Assembly 16-17 Angle Adj FP Long Ext 4-Way Out Release Pair	1

Item Number	Part Number	Description	Quantity
1-6, 7a, 8a, 8b, 9-12, 13c, 14-17, 20-24, 25e, 25f, 26, 27	123832	Standard ELR Assembly 18-24 Angle Adj FP Long Ext 4-Way Out Release Pair	1
1-6, 7b, 8a, 9-12, 13c, 14-17, 20-24, 25b, 26, 27	123833	Standard ELR Assembly 14 Angle Adj FP Long Ext 4-Way Out Release Right	1
1-6, 7a, 8a, 9-12, 13c, 14-17, 20-24, 25b, 26, 27	123834	Standard ELR Assembly 15 Angle Adj FP Long Ext 4-Way Out Release Right	1
1-6, 7a, 8a, 9-12, 13c, 14-17, 20-24, 25d, 26, 27	123835	Standard ELR Assembly 16-17 Angle Adj FP Long Ext 4-Way Out Release Right	1
1-6, 7a, 8a, 9-12, 13c, 14-17, 20-24, 25f, 26, 27	123836	Standard ELR Assembly 18-24 Angle Adj FP Long Ext 4-Way Out Release Right	1
1-6, 7b, 8b, 9-12, 13c, 14-17, 20-24, 25a, 26, 27	123837	Standard ELR Assembly 14 Angle Adj FP Long Ext 4-Way Out Release Left	1
1-6, 7a, 8b, 9-12, 13c, 14-17, 20-24, 25a, 26, 27	123838	Standard ELR Assembly 15 Angle Adj FP Long Ext 4-Way Out Release Left	1
1-6, 7a, 8b, 9-12, 13c, 14-17, 20-24, 25c, 26, 27	123839	Standard ELR Assembly 16-17 Angle Adj FP Long Ext 4-Way Out Release Left	1
1-6, 7a, 8b, 9-12, 13c, 14-17, 20-24, 25e, 26, 27	123840	Standard ELR Assembly 18-24 Angle Adj FP Long Ext 4-Way Out Release Left	1
1-6, 8a, 9-12, 14-16	123841	Standard ELR Upper Assembly with 4-Way Out Hanger Latch Release Right	1
1-6, 8b, 9-12, 14-17	123842	Standard ELR Upper Assembly with 4-Way Out Hanger Latch Release Left	1
2	000032	Footrest Hanger Pivot Saddle	2
3	103551	BHCS, M8 x 1.25 x 30 BLZ	2
4	100659	M8 Nylock Nut Blk Zn	2
5	003139	Swivel, Calf Pad	2
6	101823	M6 X 20 Hhcs Blk Zn	4
7a	101585	Calf Pad	2
7b	101470	Pediatric Calf Pad	2
8a	003416	Classic Hanger Latch Assembly, Right	1

Item Number	Part Number	Description	Quantity
8b	003417	Classic Hanger Latch Assembly, Left	1
9	000073	M5 x 24 Threaded Barrel	2
10	100669	M5 x 10 BTN SHCS BLKZC w/Patch	4
12	000034	Sleeve 3/4" x 1 1/2"	2
13a	118966	Standard ELR Extension Tube Short	2
13b	118967	Standard ELR Extension Tube, Medium	2
13c	118968	Standard ELR Extension Tube, Long	2
14	101811	M6 x 35 BHCS Blk Zn	2
15	100658	M6 Nylock Nut Blk Zn	6
16	100746	M6 Flat Washer Blk Zn	4
17	000053	Saddle 3/4"	2
18	101849	M6 x 65 SHCS Blk Zn	2
19a	100537	Footplate Composite 14-15" Wide CAT/TILT	2
19b	100540	Footplate Composite 16-17" Wide CAT/TILT	2
19c	100542	Footplate Composite 18-24" Wide CAT/TILT	2
20	000428	Wear Washer	2
21	101472	M6 x 45 SHCS BLK ZC	2
22	101455	M6 Belleville Washer (.252 x .551 x .050)	2
23	000047	Footplate Angle Adjustable Casting 3/4 Right	2
24	100502	Angle Adj Footplate Clamp 2"	2
25a	000105	Footplate Angle Adjustable 14-15" Wide CAT/TILT Left	2
25b	000106	Footplate Angle Adjustable 14-15" Wide CAT/TILT Right	2
25c	000107	Footplate Angle Adjustable 16-17" Wide CAT/TILT Left	2
25d	000108	Footplate Angle Adjustable 16-17" Wide CAT/TILT Right	2
25e	000109	Footplate Angle Adjustable 18-24" Wide CAT/TILT Left	2
25f	000110	Footplate Angle Adjustable 18-24" Wide CAT/TILT Right	2
26	100662	M6 x 25 FSHCS BLK ZC	2

Item Number	Part Number	Description	Quantity
27	120030	M6 Top Lock Nut	2
28	003161	Footplate Plug for Composite Angle Adjustable Footplate Extension Tube	2
29	003189	Bolt M8 x 45MM, Galvanized Steel	2
30	003188	Washer D8.4 x 1.6 MM	2
31	003187	Locking Wheel w/Radius, R12.5 & R10, D30 x 11.3, Cast Steel, Black	2
32	003186	Locking Wheel w/Hexagon, D30 x 6.8/6, Cast Steel, Black	2
33a	003173	Composite Angle Adjustable Footplate Small 15- 16" Wide	2
33b	003174	Composite Angle Adjustable Footplate Medium 17-18" Wide	2
33c	003175	Composite Angle Adjustable Footplate Large 19- 24" Wide	2