
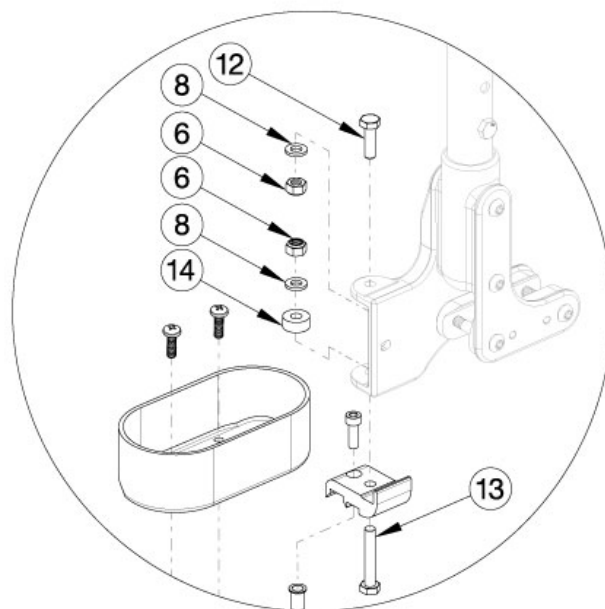


Liberty Cane and Crutch Holder

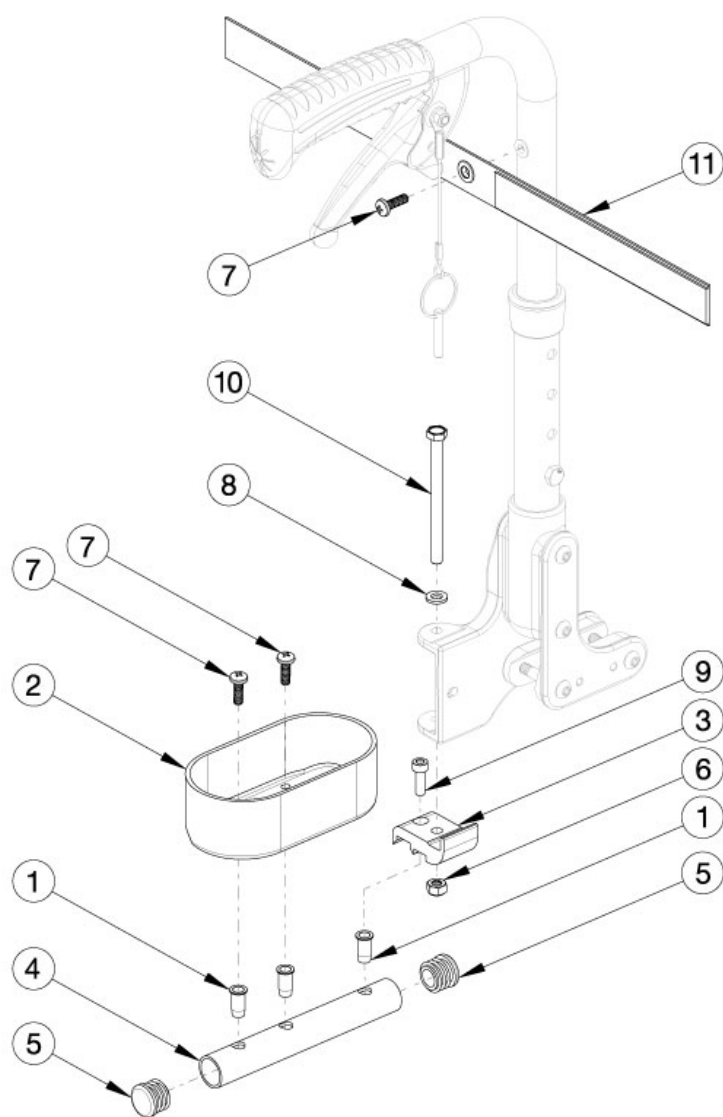

Rigid Frame Hardware

Item Number	Part Number	Description	Quantity
1-11	506201	Cane and Crutch Holder Assembly, Liberty	1
1, 4, 5	506336	Crutch Holder Tube, Liberty	1
6, 8, 12-14	118074	Cane and Crutch Holder Rigid Hardware Kit, Liberty <i>Used with Rigid Liberty Frame Only</i>	1
2	000917	Crutch Holder Cup	1
3	003698	Adapter, Crutch Holder	1
5	100638	Tube End Plug 3/4"	2

Item Number	Part Number	Description	Quantity
6	100658	M6 Nylock Nut Blk Zn	2
7	100734	M5 x 16 PHMS ZC	3
8	100746	M6 Flat Washer Blk Zn	2
9	101120	M5 x 16 SHCS BLZ	1
10	103027	M6 x 80 HHCS, Partial Thread, Blk Zn	1
11	101107	Crutch Holder Velcro Strap	1
12	101823	M6 X 20 Hhcs Blk Zn	1
13	102386	M6 x 35mm HHCS Blk Zc	1
14	116440	Spacer, .59 OD x .251 ID x .250 L	1