
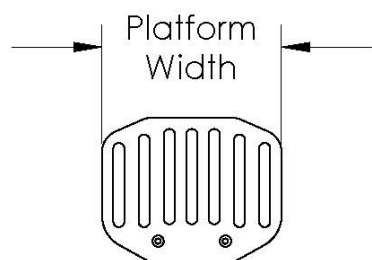


## Ethos Angle Adjustable Flip Under Footrest

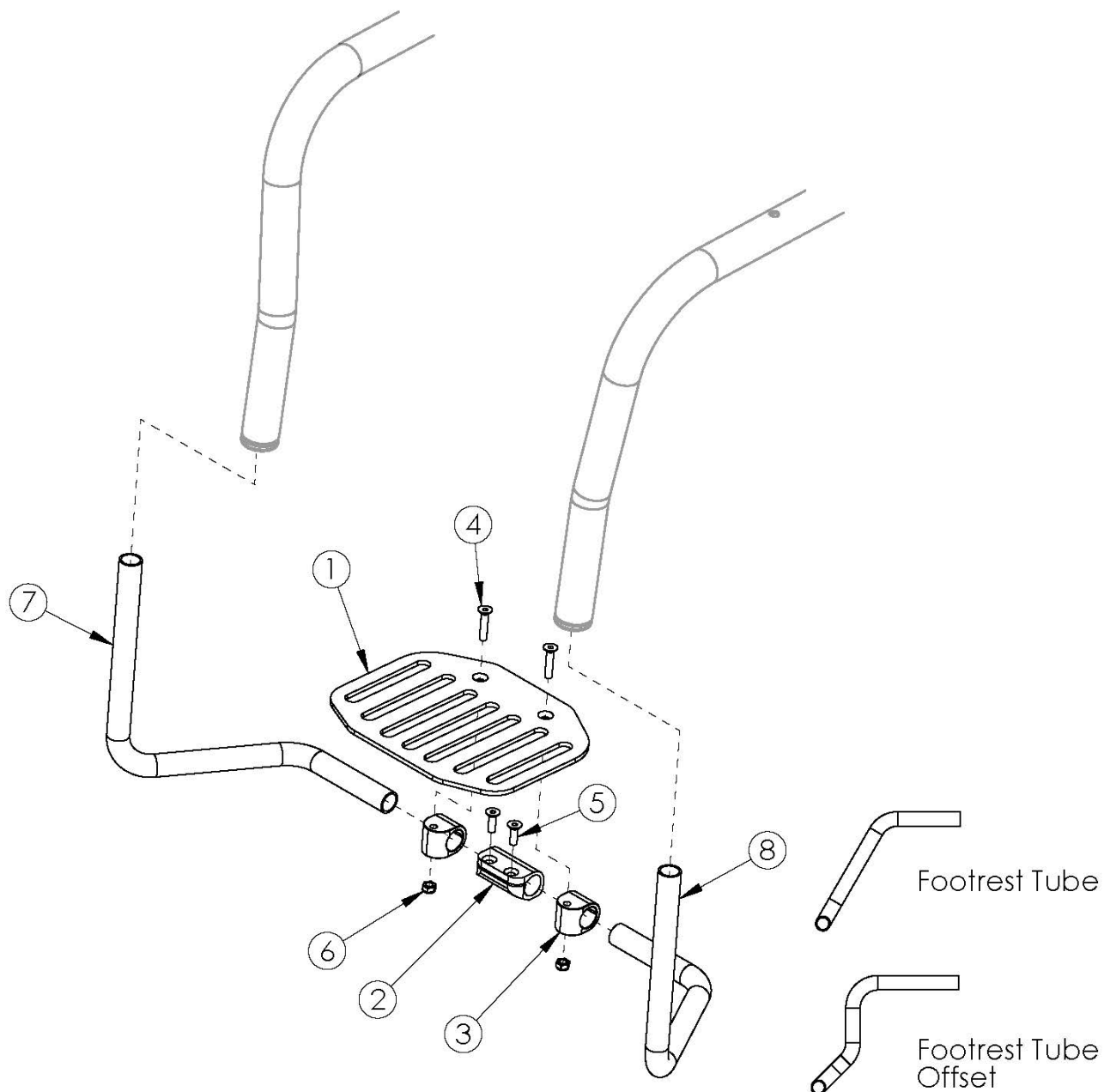

Part No.	Platform Width	For Seat Widths		
		Straight	1" Taper	2" Taper
000955	7.0"	N/A	14"	16"
000956	8.0"	12"-13"	15"	17"
001324	9.0"	14"	16"-17"	18"-19"
000957	10.0"	15"-18"	18"-20"	20"

Use Short Footrest if 0" taper and footrest length is 9" - 10.5" or if 1" or 2" taper and footrest length is 9" - 15.5" See Ethos Seat Frame page for hardware to attach footrest tubes to chair.

Item Number	Part Number	Description	Quantity
1b, 2 - 6, 7a, 8a	108148	Angle Adjustable Flip Under Footrest Assembly 12", Ethos <i>Used on 12" wide with 0" taper</i>	1
1b, 2 - 6, 7b, 8b	108149	Angle Adjustable Flip Under Footrest Assembly 13", Ethos <i>Used on 13" wide with 0" taper, 15" wide with 1" taper and 17" wide with 2" taper</i>	1
1c, 2 - 6, 7c, 8c	108150	Angle Adjustable Flip Under Footrest Assembly 14", Ethos <i>Used on 14" wide with 0" taper, 16" wide with 1" taper and 18" wide with 2" taper</i>	1
1a, 2 - 6, 7a, 8a	109683	Angle Adjustable Flip Under Footrest Assembly 14", 1" Taper, Ethos <i>Used on 14" wide with 1" taper and 16" wide with 2" taper</i>	1
1d, 2 - 6, 7d, 8d	108151	Angle Adjustable Flip Under Footrest Assembly 15", Ethos <i>Used on 15" wide with 0" taper</i>	1
1d, 2 - 6, 7e, 8e	108152	Angle Adjustable Flip Under Footrest Assembly 16", Ethos <i>Used on 16" wide with 0" taper, 18" wide with 1" taper and 20" wide with 2" taper</i>	1
1d, 2 - 6, 7f, 8f	108153	Angle Adjustable Flip Under Footrest Assembly 17", Ethos <i>Used on 17" wide with 0" taper and 19" wide with 1" taper</i>	1
1c, 2 - 6, 7d, 8d	109685	Angle Adjustable Flip Under Footrest Assembly 17", 1" Taper, Ethos <i>Used on 17" wide with 1" taper and 19" wide and 2" taper</i>	1
1d, 2 - 6, 7g, 8g	108154	Angle Adjustable Flip Under Footrest Assembly 18", Ethos <i>Used on 18" wide with 0" taper and 20" wide with 1" taper</i>	1
1b, 2 - 6, 7h, 8h	108155	Angle Adjustable Flip Under Footrest Assembly Short 12", Ethos <i>Used on 12" wide with 0" taper</i>	1
1b, 2 - 6, 7i, 8i	108156	Angle Adjustable Flip Under Footrest Assembly Short 13", Ethos <i>Used on 13" wide with 0" taper, 15" wide with 1" taper and 17" wide with 2" taper</i>	1
1c, 2 - 6, 7j, 8j	108157	Angle Adjustable Flip Under Footrest Assembly Short 14", Ethos <i>Used on 14" wide with 0" taper, 16" wide with 1" taper and 18" wide with 2" taper</i>	1
1a, 2 - 6, 7h, 8h	109686	Angle Adjustable Flip Under Footrest Assembly Short 14", 1" Taper, Ethos <i>Used on 14" wide with 1" taper and 16" wide with 2" taper</i>	1
1d, 2 - 6, 7k, 8k	108158	Angle Adjustable Flip Under Footrest Assembly Short 15", Ethos <i>Used on 15" wide with 0" taper</i>	1
1d, 2 - 6, 7l, 8l	108159	Angle Adjustable Flip Under Footrest Assembly Short 16", Ethos <i>Used on 16" wide with 0" taper, 18" wide with 1" taper and 20" wide with 2" taper</i>	1
1d, 2 - 6, 7m, 8m	108160	Angle Adjustable Flip Under Footrest Assembly Short 17", Ethos	1


Item Number	Part Number	Description	Quantity
		<i>Used on 17" wide with 0" taper and 19" wide with 1" taper</i>	
1c, 2 - 6, 7k, 8k	109688	Angle Adjustable Flip Under Footrest Assembly Short 17", 1" Taper, Ethos <i>Used on 17" wide with 1" taper and 19" wide and 2" taper</i>	1
1d, 2 - 6, 7n, 8n	108161	Angle Adjustable Flip Under Footrest Assembly Short 18", Ethos <i>Used on 18" wide with 0" taper and 20" wide with 1" taper</i>	1
1a	000955	Angle Adjustable Flip Under Footrest Platform 7" Rigid	1
1b	000956	Angle Adjustable Flip Under Footrest Platform 8" Rigid	1
1c	001324	Angle Adjustable Flip Under Footrest Platform 9" Rigid	1
1d	000957	Angle Adjustable Flip Under Footrest Platform 10" Rigid	1
2	000953	Footrest Flip Stop Clamp Rigid	1
3	000954	Footrest Flip Pivot Rigid	2
4	101733	M6 x 35 CLS10.9 DIN7991 FSHCS BLKZC	2
5	100866	M6 x 20 CLS12.9 DIN7991 FSHCS Blk Zc w/Patch	2
6	100658	M6 Nylock Nut Blk Zn	2
7a	004399	Angle Adjustable Flip Under Footrest Tube Offset Right, 5.312" <i>Used on 12" wide with 0" taper, 14" wide with 1" taper and 16" wide with 2" taper</i>	1
7b	004401	Angle Adjustable Flip Under Footrest Tube Offset Right, 5.812" <i>Used on 13" wide with 0" taper, 15" wide with 1" taper and 17" wide with 2" taper</i>	1
7c	004403	Angle Adjustable Flip Under Footrest Tube Offset Right, 6.312" <i>Used on 14" wide with 0" taper, 16" wide with 1" taper and 18" wide with 2" taper</i>	1
7d	004405	Angle Adjustable Flip Under Footrest Tube Offset Right, 6.812" <i>Used on 15" wide with 0" taper, 17" wide with 1" taper and 19" wide and 2" taper</i>	1
7e	004407	Angle Adjustable Flip Under Footrest Tube Offset Right, 7.312" <i>Used on 16" wide with 0" taper, 18" wide with 1" taper and 20" wide with 2" taper</i>	1
7f	004409	Angle Adjustable Flip Under Footrest Tube Offset Right, 7.812" <i>Used on 17" wide with 0" taper and 19" wide with 1" taper</i>	1
7g	004423	Angle Adjustable Flip Under Footrest Tube Right, 8.312" <i>Used on 18" wide with 0" taper and 20" wide with 1" taper</i>	1
7h	004411	Angle Adjustable Flip Under Footrest Tube Offset Short Right, 5.312" <i>Used on 12" wide with 0" taper, 14" wide with 1" taper and 16" wide with 2" taper</i>	1

Item Number	Part Number	Description	Quantity
7i	004413	Angle Adjustable Flip Under Footrest Tube Offset Short Right, 5.812" <i>Used on 13" wide with 0" taper, 15" wide with 1" taper and 17" wide with 2" taper</i>	1
7j	004415	Angle Adjustable Flip Under Footrest Tube Offset Short Right, 6.312" <i>Used on 14" wide with 0" taper, 16" wide with 1" taper and 18" wide with 2" taper</i>	1
7k	004417	Angle Adjustable Flip Under Footrest Tube Offset Short Right, 6.812" <i>Used on 15" wide with 0" taper, 17" wide with 1" taper and 19" wide and 2" taper</i>	1
7l	004419	Angle Adjustable Flip Under Footrest Tube Offset Short Right, 7.312" <i>Used on 16" wide with 0" taper, 18" wide with 1" taper and 20" wide with 2" taper</i>	1
7m	004421	Angle Adjustable Flip Under Footrest Tube Offset Short Right, 7.812" <i>Used on 17" wide with 0" taper and 19" wide with 1" taper</i>	1
7n	004429	Angle Adjustable Flip Under Footrest Tube Short Right, 8.312" <i>Used on 18" wide with 0" taper and 20" wide with 1" taper</i>	1
8a	004398	Angle Adjustable Flip Under Footrest Tube Offset Left, 5.312" <i>Used on 12" wide with 0" taper, 14" wide with 1" taper and 16" wide with 2" taper</i>	1
8b	004400	Angle Adjustable Flip Under Footrest Tube Offset Left, 5.812" <i>Used on 13" wide with 0" taper, 15" wide with 1" taper and 17" wide with 2" taper</i>	1
8c	004402	Angle Adjustable Flip Under Footrest Tube Offset Left, 6.312" <i>Used on 14" wide with 0" taper, 16" wide with 1" taper and 18" wide with 2" taper</i>	1
8d	004404	Angle Adjustable Flip Under Footrest Tube Offset Left, 6.812" <i>Used on 15" wide with 0" taper, 17" wide with 1" taper and 19" wide and 2" taper</i>	1
8e	004406	Angle Adjustable Flip Under Footrest Tube Offset Left, 7.312" <i>Used on 16" wide with 0" taper, 18" wide with 1" taper and 20" wide with 2" taper</i>	1
8f	004408	Angle Adjustable Flip Under Footrest Tube Offset Left, 7.812" <i>Used on 17" wide with 0" taper and 19" wide with 1" taper</i>	1
8g	004422	Angle Adjustable Flip Under Footrest Tube Left, 8.312" <i>Used on 18" wide with 0" taper and 20" wide with 1" taper</i>	1
8h	004410	Angle Adjustable Flip Under Footrest Tube Offset Short Left, 5.312" <i>Used on 12" wide with 0" taper, 14" wide with 1" taper and 16" wide with 2" taper</i>	1
8i	004412	Angle Adjustable Flip Under Footrest Tube Offset Short Left, 5.812" <i>Used on 13" wide with 0" taper, 15" wide with 1" taper and 17" wide with 2" taper</i>	1
8j	004414	Angle Adjustable Flip Under Footrest Tube Offset Short Left, 6.312"	1

Item Number	Part Number	Description	Quantity
		<i>Used on 14" wide with 0" taper, 16" wide with 1" taper and 18" wide with 2" taper</i>	
8k	004416	Angle Adjustable Flip Under Footrest Tube Offset Short Left, 6.812" <i>Used on 15" wide with 0" taper, 17" wide with 1" taper and 19" wide with 2" taper</i>	1
8l	004418	Angle Adjustable Flip Under Footrest Tube Offset Short Left, 7.312" <i>Used on 16" wide with 0" taper, 18" wide with 1" taper and 20" wide with 2" taper</i>	1
8m	004420	Angle Adjustable Flip Under Footrest Tube Offset Short Left, 7.812" <i>Used on 17" wide with 0" taper and 19" wide with 1" taper</i>	1
8n	004428	Angle Adjustable Flip Under Footrest Tube Short Left, 8.312" <i>Used on 18" wide with 0" taper and 20" wide with 1" taper</i>	1


Distance between Point A and Point B  
B = Footplate Length

**NOTE:** The yardstick on the top plane of the seat frame must lie flat against frame. The yardstick going from the footplate to the second yardstick must be square against the hanger tube.

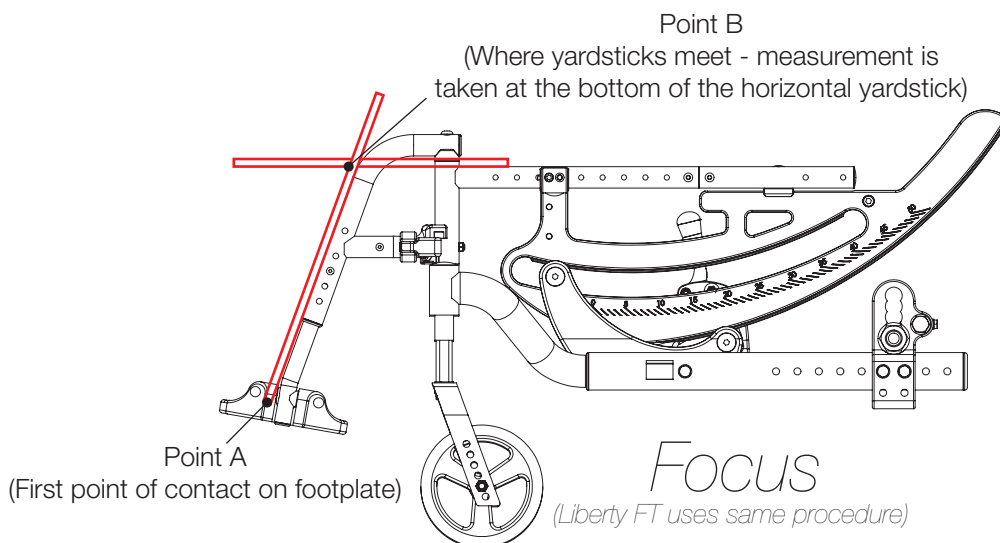

Distance between Point A and Point B  
B = Footplate Length

**NOTE:** The yardstick on the top plane of the seat frame must lie flat against frame. The yardstick going from the footplate to the second yardstick must be square against the front tube.

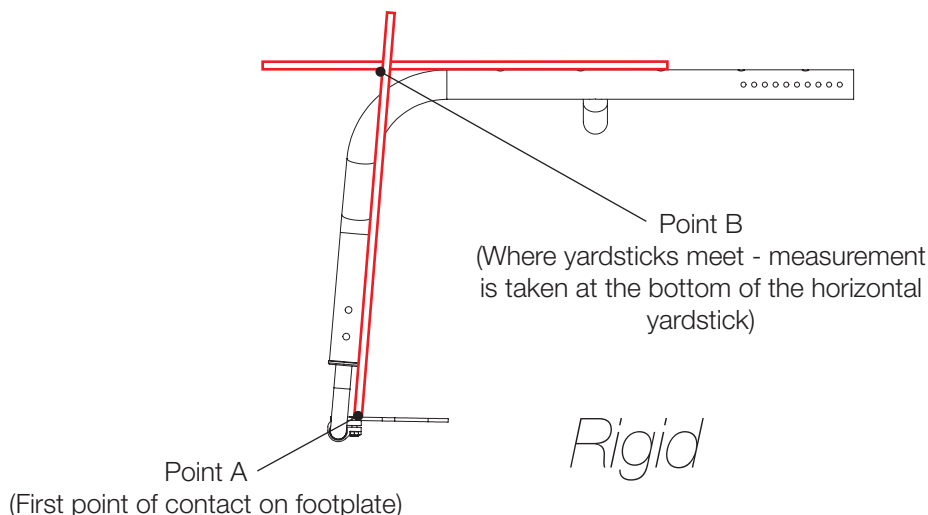

Distance between Point A and Point B  
B = Footplate Length

**NOTE:** The yardstick on the top plane of the hanger must lie flat against hanger. The yardstick going from the footplate to the second yardstick must be square against the hanger tube.

