


Rogue2 User Activated Anti-Tip

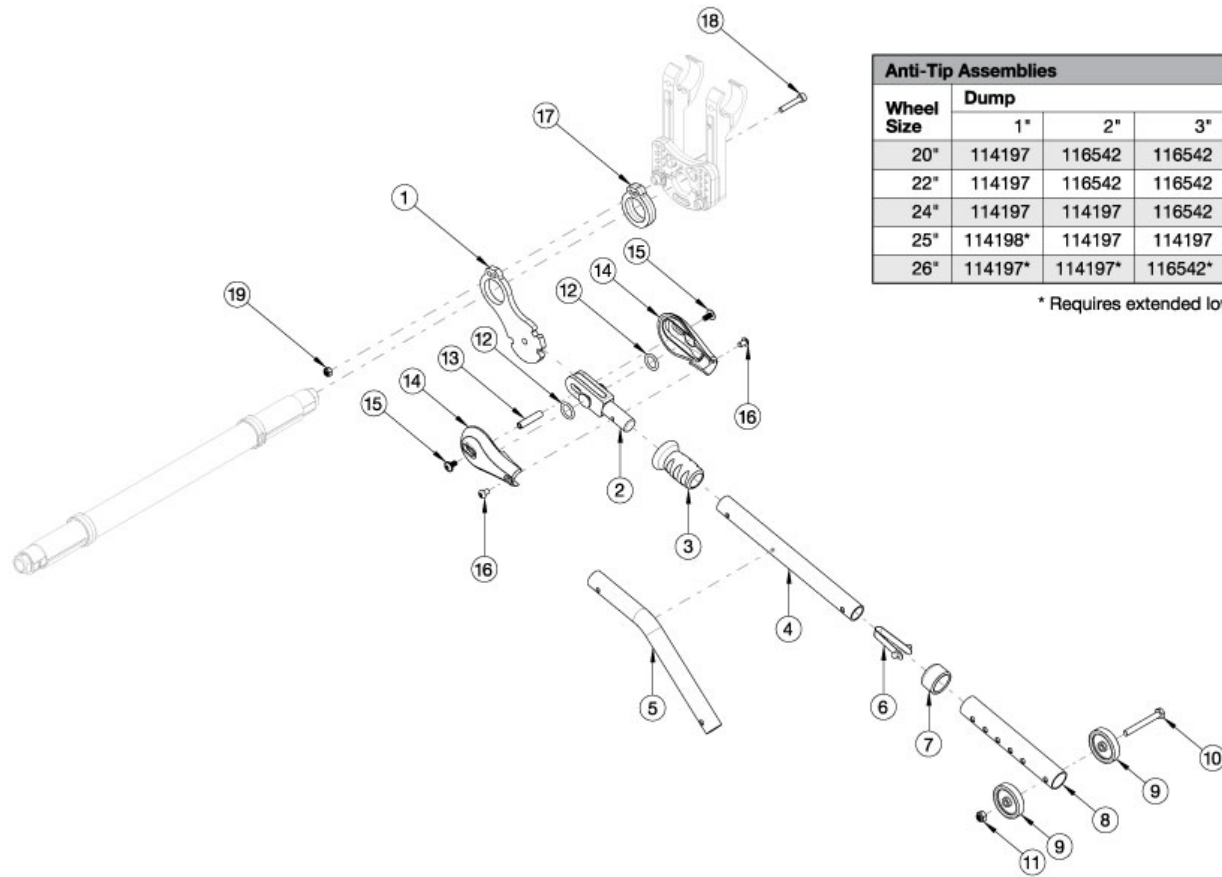

Anti-Tip Assemblies					
Wheel Size	Dump				
	1"	2"	3"	4"	5"
20"	114197	116542	116542	114198	114198
22"	114197	116542	116542	116542	114198
24"	114197	114197	116542	116542	116542
25"	114198*	114197	114197	116542	116542
26"	114197*	114197*	116542*	116542*	116542*

* Requires extended lower assembly 116544

Contact Customer Service for chairs with more than 2" of variance between Front and Rear Seat Height. Complete assemblies contain "one" complete user activated anti-tip. Quantity two is required for a pair.

Item Number	Part Number	Description	Quantity
1-3, 4, 6, 7, 8a, 9-19	117507	User Anti Tip Complete Assembly, Straight, Rogue2	1
1-3, 5a, 6, 7, 8a, 9-19	117508	User Anti Tip Complete Assembly, Bent, Rogue2	1
1-3, 5b, 6, 7, 8a, 9-19	117509	User Anti Tip Complete Assembly, Bent, 15 Degrees, Rogue2	1
1-3, 4, 6, 7, 8b, 9-19	117590	User Anti Tip Complete Assembly, X-Large, Straight, Rogue2	1
1-3, 5a, 6, 7, 8b, 9-19	117591	User Anti Tip Complete Assembly, X-Large, Bent, Rogue2	1
1-3, 5b, 6, 7, 8b, 9-19	117592	User Anti Tip Complete Assembly, X-Large, Bent, 15 Degrees, Rogue2	1
1-3, 4, 6, 12-16	114198	Anti-Tip Upper Assembly Straight, User Activated, Rogue2	1
1-3, 5a, 6, 12-16	114197	Anti-Tip Upper Assembly Bent, User Activated, Rogue2	1
1-3, 5b, 6, 12-16	116542	Anti-Tip Upper Assembly Bent, User Activated, 15 Degrees, Rogue2	1

Item Number	Part Number	Description	Quantity
8a-11	004691	Anti-Tip Lower Assembly Rogue2 Ethos	1
8b-11	116544	Anti-Tip Lower Assembly X-Large Rogue2	1
6	100534	Detent Button Lower	1
7	004306	Anti-Tip Anti Rattle Bushing	1
15	103323	M5 x 10 Flange BHCS Blk Zc	2
16	103324	M5 x 8 BHCS Blk Zc	2
17	113998	Anti Tip Spacer	1
18	114015	SHCS, M5X0.8X30, Cls12.9, Blz, DIN 912	1
19	100657	M5 Nylock Nut Blk Zn	1