


Catalyst Upper Pivot Assembly - Growth

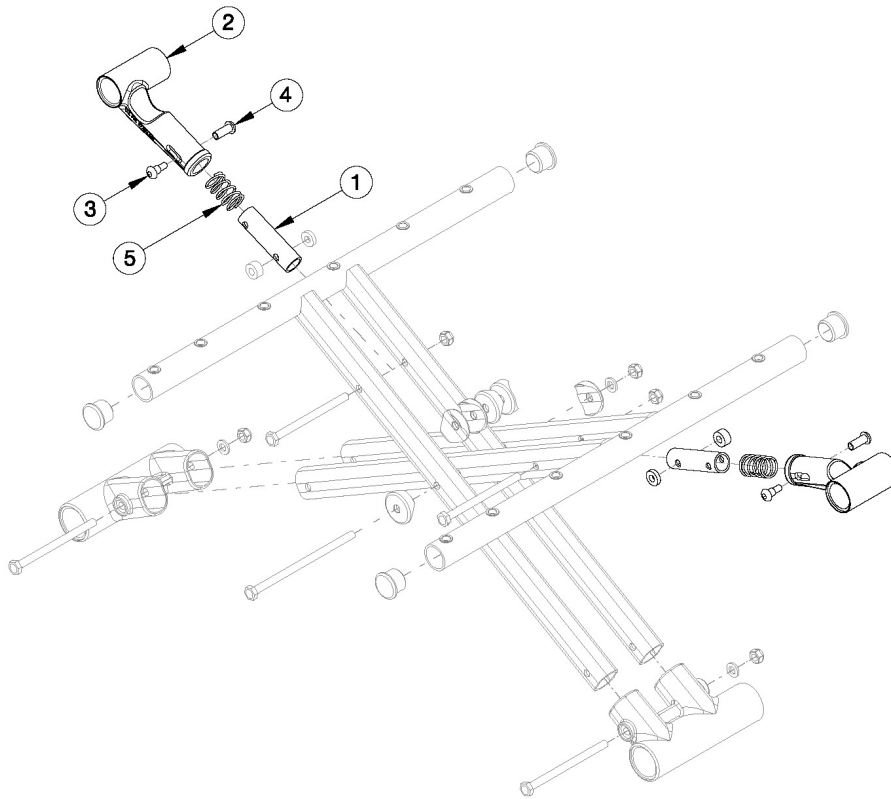

| Cross Brace Overall Width | |
|---------------------------|------------------------------------|
| Seat Width | Cross Brace Measurement |
| 14"-16" | Approx. 1" less than seat width |
| 17" | Approx. 1.25" less than seat width |
| 18"-21" | Approx. 1.5" less than seat width |
| 22" | Approx. 1.75" less than seat width |

| Cross Brace Overall Depth | |
|---------------------------|-------------------------------------|
| Seat Depth | Cross Brace Measurement |
| 14"-20" | Approx. 0.375" less than seat depth |

| Item Number | Part Number | Description | Quantity |
|-------------|-------------|--|----------|
| 1a, 2a, 3-5 | 500024 | Upper Pivot Assembly 14" Width <i>Measures Approximately 4.5"</i> | 2 |
| 1b, 2b, 3-5 | 500025 | Upper Pivot Assembly 15" Width <i>Measures Approximately 5"</i> | 2 |
| 1c, 2b, 3-5 | 500026 | Upper Pivot Assembly 16" Width <i>Measures Approximately 5.5"</i> | 2 |
| 1d, 2b, 3-5 | 500027 | Upper Pivot Assembly 17" Width <i>Measures Approximately 6"</i> | 2 |
| 1e, 2b, 3-5 | 500028 | Upper Pivot Assembly 18" Width <i>Measures Approximately 6.5"</i> | 2 |

| Item Number | Part Number | Description | Quantity |
|-------------|-------------|--|----------|
| 1f, 2b, 3-5 | 500029 | Upper Pivot Assembly 19" Width <i>Measures Approximately 7"</i> | 2 |
| 1g, 2b, 3-5 | 500030 | Upper Pivot Assembly 20" Width <i>Measures Approximately 7.25"</i> | 2 |
| 1h, 2b, 3-5 | 501479 | Upper Pivot Assembly 21/22" Width <i>Measures Approximately 8.25"</i> | 2 |