


# Liberty Side Guard

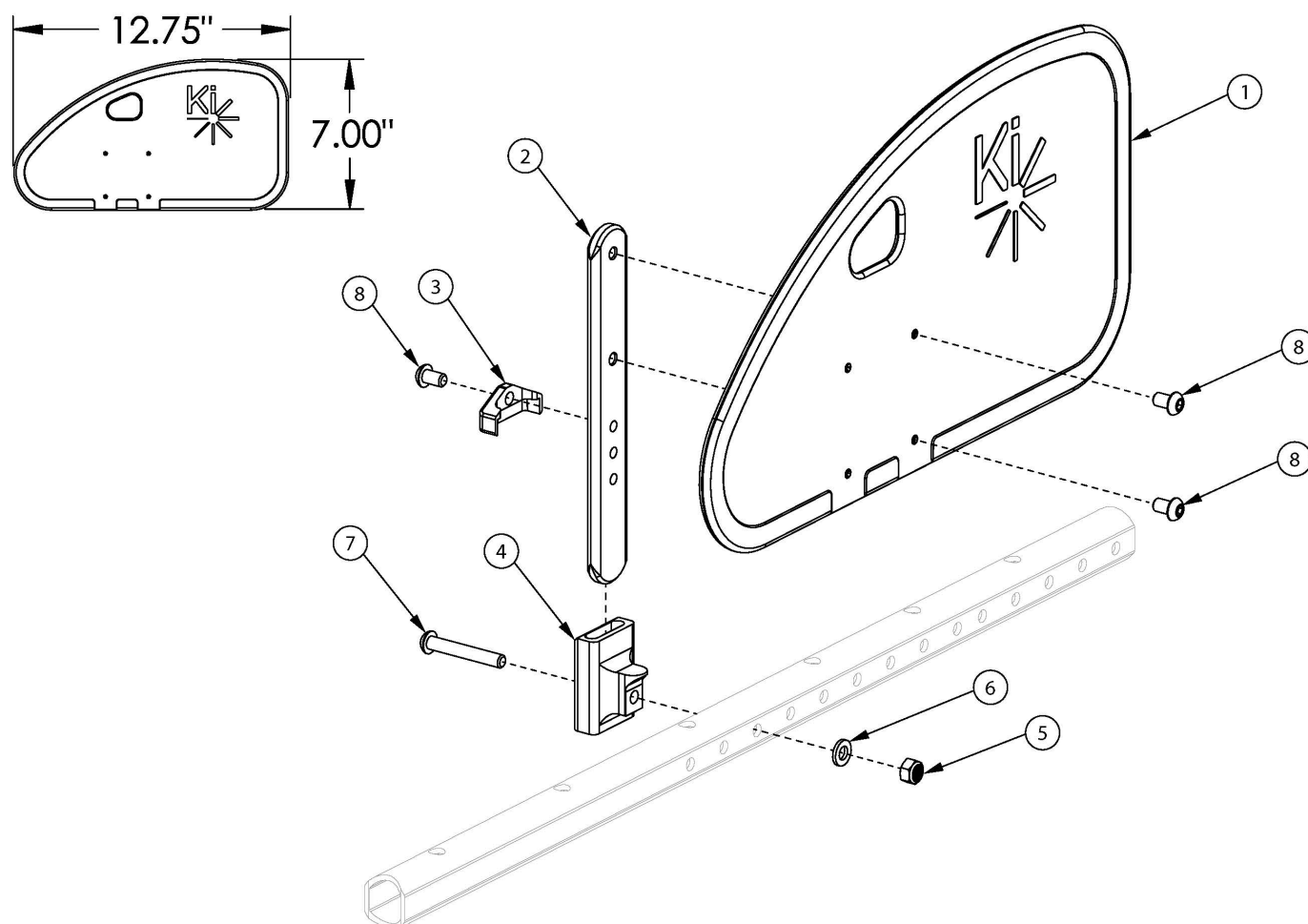

Rear holes on side guard panel (#1) are drilled for all chair configurations, unless otherwise specified. If retrofitting, side guard receivers may be required

Item Number	Part Number	Description	Quantity
1-3, 8	112578	Side Guard Assembly, Rear Drilled, Left	1
1-3, 8	112579	Side Guard Assembly, Rear Drilled, Right	1
1-3, 8	112580	Side Guard Assembly, Rear Drilled, PAIR	1
1-8	506197	Side Guard Assembly with Receiver, Right, Liberty	1
1-8	506198	Side Guard Assembly with Receiver, Left, Liberty	1
1-8	506199	Side Guard Assembly with Receiver, PAIR, Liberty	1
1	112574	Side Guard Panel Rear Drilled	1
2	000140	Side Guard Post	1

Item Number	Part Number	Description	Quantity
3	001342	Side Guard Post Stop	1
4	003439	Receiver, Sideguard, Liberty	1
5	100658	M6 Nylock Nut Blk Zn	1
6	100746	M6 Flat Washer Blk Zn	1
7	100797	M6 x 40 BHCS BLK ZC	1
8	101184	M6 x 10 BTN SHCS BLKZC w/Patch	3