


Flip for Leckey Push Wheel Lock 12" Wheels

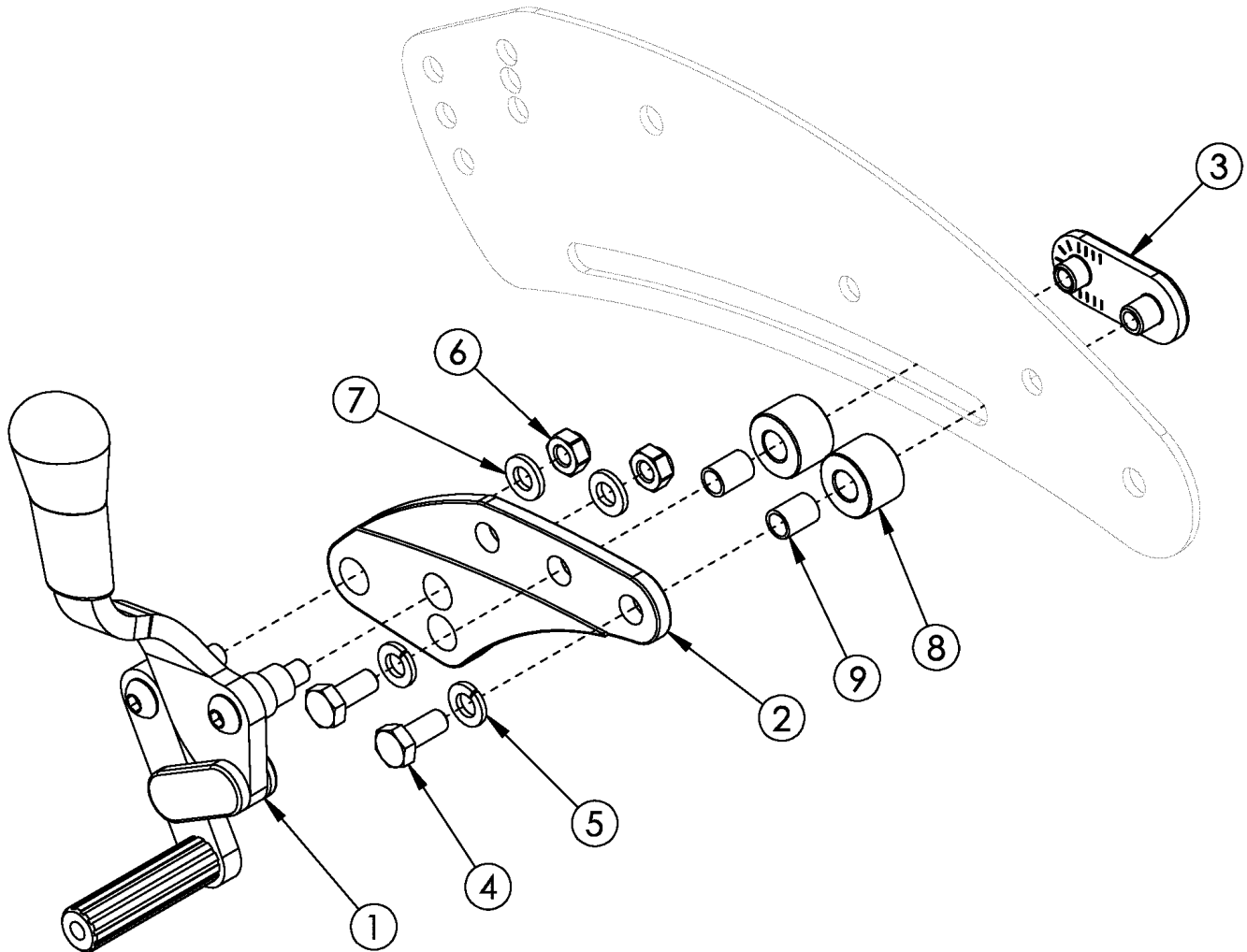

Item Number	Part Number	Description	Quantity
1a, 2 - 9	503417	Wheel Lock Assembly, Std Push, Right, Flip, 12" Wheels	1
1b, 2 - 9	503418	Wheel Lock Assembly, Std Push, Left, Flip, 12" Wheels	1
1a, 1b, 2 - 9	503419	Wheel Lock Assembly, Std Push, PAIR, Flip, 12" Wheels	1
3	001905	Nut, Slide, Flip	1
4	102792	M6 x 25, HHCS, Blk Zn	2
5	101835	M6 Lock Washer Blk Zn	2
6	100658	M6 Nylock Nut Blk Zn	2

Item Number	Part Number	Description	Quantity
7	100746	M6 Flat Washer Blk Zn	2
8	003156	Spacer .7"OD x .315"ID x .485"	2
9	001019	Bearing Shaft Upper	2