



QM-7 TRANSIT INSTALLATION



Please read these instructions carefully BEFORE beginning the installation. Failure to understand and follow installation instructions may result in injury to installer and/or end user and may void the warranty. If you have any questions call Sunrise Medical Technical support at 800-333-4000

Parts for installing this Transit kit.

- | | |
|-----------------------------------|------------------------------|
| 1. 1x Transit Bracket Front Right | 3. 4x M12 Hex head Cap Screw |
| 2. 1x Transit Bracket Front Left | 4. 4x Transit Decal |

Tools needed:


- | | |
|------------------|------------------|
| 1. 18mm wrench) | 3. Torque wrench |
| 2. 5/8 in socket | |

This instruction will take you thru the step by step process of installing a Transit system onto one side of your QM-7 series base.

REMOVING THE DRIVE WHEEL

1 2


1. Remove the 3 lugs holding the drive wheel using a 5/8" Socket (A).
2. Using a 18mm wrench or socket, Remove the 2 hex head cap screws (B) at the front suspension pivot points and discard the washers (C).

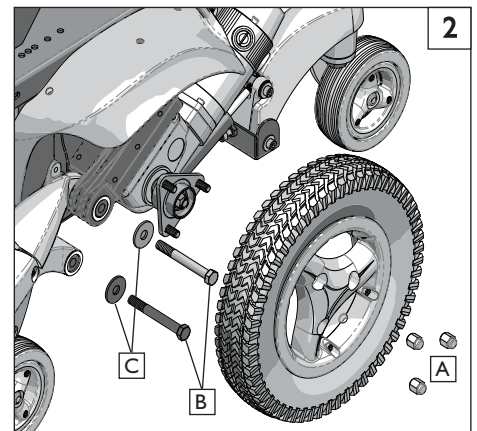

INSTALLING THE FRONT TRANSIT HOOK



3

1. Insert the two new hex head cap screws(B) through the front transit bracket(E) and start threading them into the suspension pivot points.
2. Hand-tighten the screws and transit hook assembly into the suspension pivot points (E) as shown.
3. Using a Torque wrench with an 18mm socket, tighten to a torque of 50 ft-lbs or 67.8 Nm

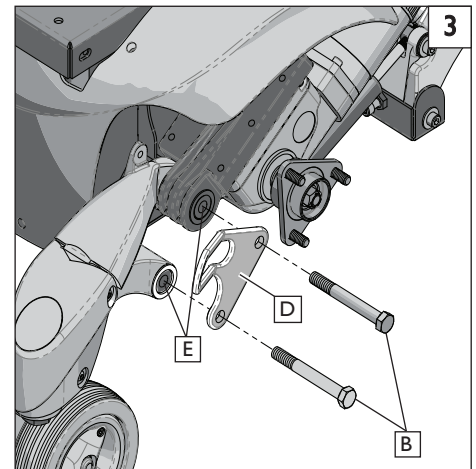

C. FRONT AND REAR TRANSIT STICKERS 4 5

- I. The placement of Transit hook location stickers is essential for the safety of the rider.
 - a. Peel the backing from the Front Transit Hook decals and place them on each side of the chair in the location shown in Figure 4.
 - b. Peel the backing from the Rear Transit Hook decals and place them on both sides of the chair in the location shown in Figure 5.

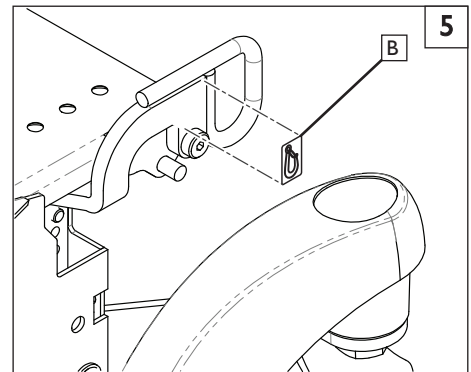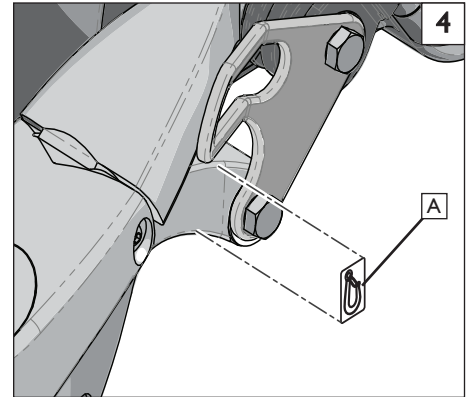**D. CHECKOUT**

- I. Double-check function and correct Torque on all Hardware.
 - a. Using a torque wrench and socket wrench to check all hardware for correct torque and tightness.
 - b. Reinstall the drive wheel, torque lugs (C) to 25 ft-lbs.

DEALER/TECHNICIAN WARNING**⚠ WARNING**

Attention dealers and qualified technicians, do not operate or service this device without first reading the owners manual. If you do not understand the instructions and warnings in the owners manual please contact the Quickie Technical Service Department before operating and/or servicing the Quickie device. Failure to do so may result in damage and/or injury.



Sunrise Medical • 2842 Business Park Ave. • Fresno, CA 93727 • USA
In Canada (800) 263-3390