


# Ethos Rear Anti-Tip

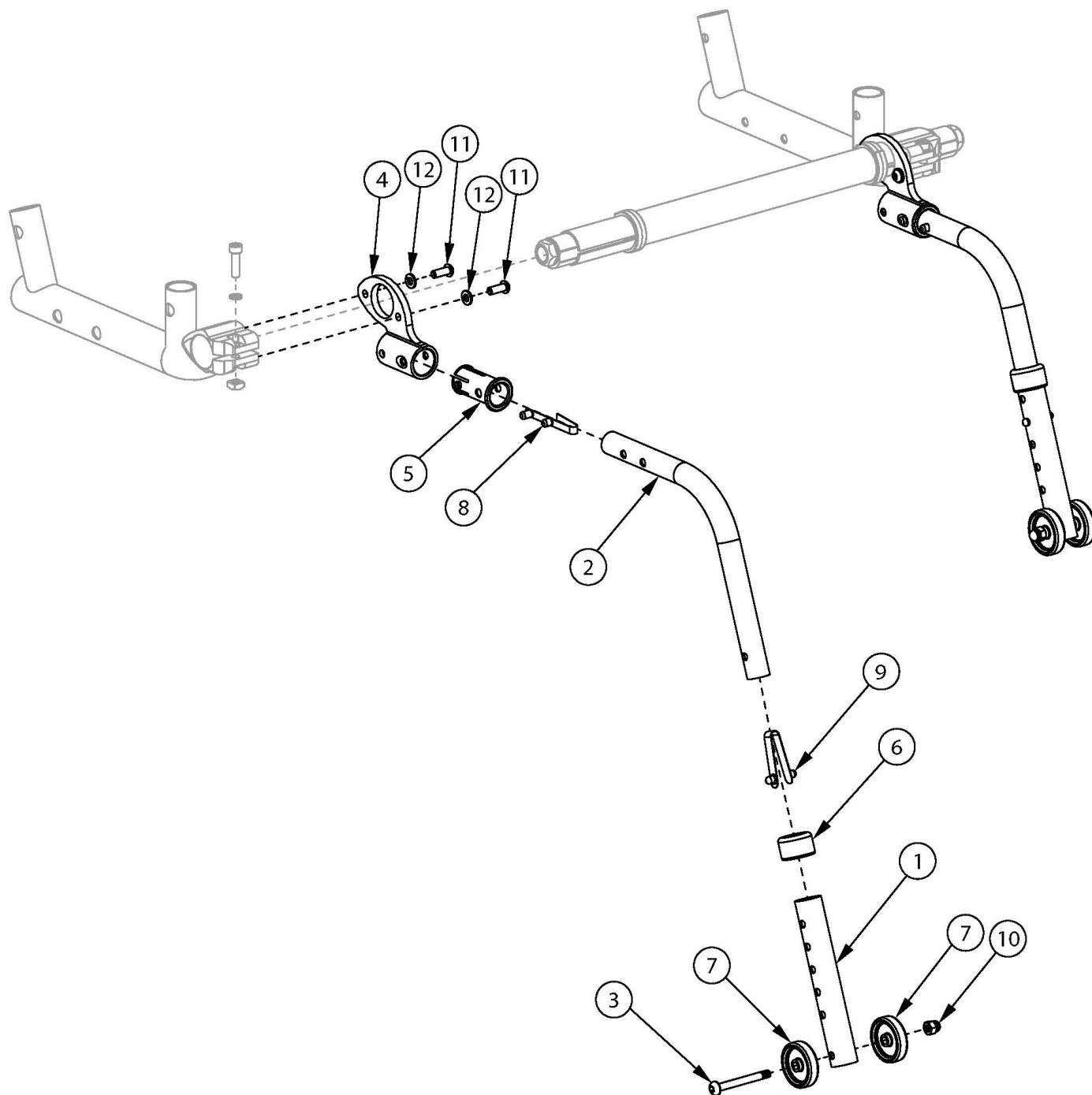

Item Number	Part Number	Description	Quantity
1, 2a, 3-12	108105	Rear Anti-Tip with Receiver Retrofit Kit, 22"-25" Rear Wheel, Ethos <i>This kit contains a pair of rear Anti-Tips with receivers and attaching hardware</i>	1
1, 2b, 3-12	108106	Rear Anti-Tip with Receiver Retrofit Kit, 26" Rear Wheel, Ethos	1

Item Number	Part Number	Description	Quantity
		<i>This kit contains a pair of rear Anti-Tips with receivers and attaching hardware</i>	
1, 2a, 3, 6-10	108102	Anti-Tip Assembly 22"-25" Wheel, Ethos	2
1, 2b, 3, 6-10	108103	Anti-Tip Assembly 26" Wheel, Ethos	2
1, 3, 7, 10	119668	Anti-Tip Lower Assembly	2
4, 5, 11, 12	108104	Rear Anti-Tip Receiver Kit, Ethos	2
2a	000077	Anti Tip Tube Upper Medium <i>Commonly used with 22" - 25" rear wheels</i>	2
2b	000078	Anti Tip Tube Upper Long <i>Commonly used with 26" rear wheels</i>	2
4	003982	Standard Anti-Tip Receiver	2
5	004187	Standard Anti Tip Sleeve	2
6	004306	Anti-Tip Anti Rattle Bushing	2
8	100533	Detent Button	2
9	100534	Detent Button Lower	2
11	101721	M6 X 16 Btn Shcs W/ Patch	4
12	101835	M6 Lock Washer Blk Zn	4