


Catalyst Height Adjustable T-Arm

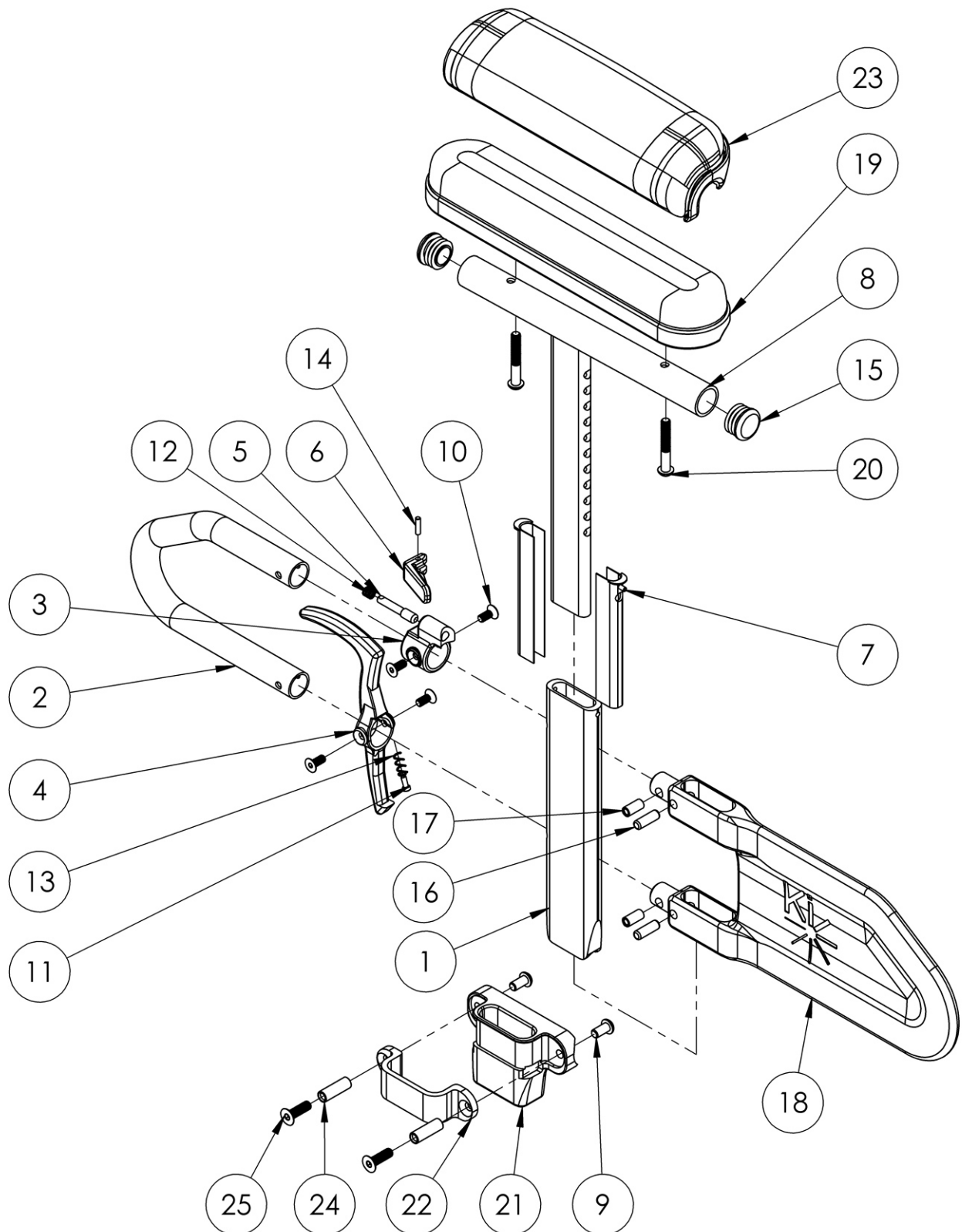

Short assemblies are used with 14-17 Seat Depths. Long assemblies are used with 18"-20" Seat Depths <p>

This design effective November 10, 2017 for long panels and December 6, 2017 for short panels <p>

If retrofitting to Height Adjustable T-Arms and One Arm Drive is present, a new drive shaft may be required. In addition to a new drive shaft, order the corresponding hardware kit: 109438 for Blade Style One Arm Drive, 503433 for Hex Style One Arm Drive or 112898 for Quick Release One Arm Drive

| Item Number | Part Number | Description | Quantity | Retail |
|---------------------|-------------|---|----------|---------|
| 1 | 000089 | Height Adj T Armrest Post | 1 | \$25.10 |
| 2 | 000095 | Height Adj T Armrest Transfer Tube | 1 | \$28.07 |
| 3, 5, 6, 10, 12, 14 | 505490 | Height Adj T-Arm Upper Latch Assembly | 1 | \$40.56 |
| 7 | 000090 | Height Adj T Armrest Shim | 2 | \$2.55 |
| 8a, 15 | 112906 | Height Adj T Armrest Upper Desk w/ Tube End Plug | 1 | \$76.67 |
| 8b, 15 | 112907 | Height Adj T Armrest Upper Full w/ Tube End Plug | 1 | \$76.67 |
| 9 | 100667 | M6 x 12 BTN SHCS BLKZC w/Patch | 2 | \$1.58 |
| 10 | 003458 | M5 X 12mm FHCS BLK ZN W/ Patch | 4 | \$1.05 |
| 11 | 000152 | Height Adj T Armrest Release Spring Button | 1 | \$1.09 |
| 13 | 000135 | Height Adj T Armrest Release Lever Spring | 1 | \$1.09 |
| 15 | 100803 | Tube End Plug 7/8" | 2 | \$1.58 |
| 16 | 003641 | .25" x .75" Dowel Pin | 2 | \$1.05 |
| 17 | 003457 | M5 x 14mm Barrel Nut | 2 | \$1.05 |
| 19a | 100500 | Arm Pad Full | 1 | \$23.64 |
| 19b | 100501 | Arm Pad Desk | 1 | \$23.64 |
| 20a | 100696 | M5 x 30mm PPH Capscrew <i>Used with Standard Arm Pads</i> | 2 | \$1.58 |
| 20b | 100654 | M5 x 40 DIN7985 PH Pan HD M/S ZC <i>Used with Waterfall Arm Pads</i> | 2 | \$1.58 |
| 21 | 000087 | Height Adjustable T-Arm Receiver 1" CAT | 1 | \$15.25 |
| 22 | 000088 | Height Adj T Armrest Bracket | 1 | \$27.42 |
| 23a | 102452 | Waterfall Arm Pad Full | 1 | \$32.80 |
| 23b | 102521 | Waterfall Arm Pad Desk | 1 | \$32.80 |
| 24 | 000065 | M6 x 23.68 Threaded Barrel | 2 | \$4.13 |
| 25 | 100866 | M6 x 20 CLS12.9 DIN7991 FSHCS Blk Zc w/Patch | 2 | \$2.63 |

| Item Number | Part Number | Description | Quantity | Retail |
|---------------------------|-------------|--|----------|----------|
| 4, 10, 11, 13 | 505507 | Height Adj T-Arm Release Lever Assembly | 1 | \$8.49 |
| 16, 17, 18a | 505508 | Height Adj T-Arm Long Side Shield Assembly | 1 | \$39.18 |
| 16, 17, 18b | 506082 | Height Adj T-Arm Short Shield Assembly | 1 | \$16.44 |
| 9, 21, 22, 24, 25 | 501730 | Height Adj T-Arm Receiver Assm 1" CAT | 1 | \$55.66 |
| 1-7, 10-14, 16, 17, 18b | 004160 | Height Adjustable T-Arm Lower Assembly Short <i>Does not include t-arm upper, arm pad or arm pad screws</i> | 1 | \$132.94 |
| 1-7, 10-14, 16, 17, 18a | 004161 | Height Adjustable T-Arm Lower Assembly Long <i>Does not include t-arm upper, arm pad or arm pad screws</i> | 1 | \$132.94 |
| 1-7, 8a, 10-18a | 003465 | Height Adjustable T-Arm Long Desk <i>Does not include arm pad or arm pad screws</i> | 1 | \$207.98 |
| 1-7, 8b, 10-18a | 003466 | Height Adjustable T-Arm Long Full <i>Does not include arm pad or arm pad screws</i> | 1 | \$207.98 |
| 1-7, 8a, 10-18b | 003463 | Height Adjustable T-Arm Short Desk <i>Does not include arm pad or arm pad screws</i> | 1 | \$207.98 |
| 1-7, 8b, 10-18b | 003464 | Height Adjustable T-Arm Short Full <i>Does not include arm pad or arm pad screws</i> | 1 | \$207.98 |
| 1-7, 8a, 10-18a, 19a, 20a | 505491 | Height Adj T-Arm Assm w/ Long Side Shield and Standard Pad Full | 1 | \$234.78 |
| 1-7, 8a, 10-18a, 19a, 20a | 505492 | Height Adj T-Arm Assm w/ Long Side Shield and Standard Pad Full, PAIR | 1 | \$469.57 |
| 1-7, 8a, 10-18b, 19a, 20a | 505644 | Height Adj T-Arm Assm w/ Short Side Shield and Standard Pad Full | 1 | \$234.78 |
| 1-7, 8a, 10-18b, 19a, 20a | 505645 | Height Adj T-Arm Assm w/ Short Side Shield and Standard Pad Full, PAIR | 1 | \$469.57 |
| 1-7, 8a, 10-18a, 19b, 20a | 505493 | Height Adj T-Arm Assm w/ Long Side Shield and Standard Pad Desk | 1 | \$234.78 |
| 1-7, 8a, 10-18a, 19b, 20a | 505494 | Height Adj T-Arm Assm w/ Long Side Shield and Standard Pad Desk, PAIR | 1 | \$469.57 |
| 1-7, 8a, 10-18b, 19b, 20a | 505646 | Height Adj T-Arm Assm w/ Short Side Shield and Standard Pad Desk | 1 | \$234.78 |
| 1-7, 8a, 10-18b, 19b, 20a | 505647 | Height Adj T-Arm Assm w/ Short Side Shield and Standard Pad Desk, PAIR | 1 | \$469.57 |
| 1-7, 8a, 10-18a, 20b, 23a | 505495 | Height Adj T-Arm Assm w/ Long Side Shield and Waterfall Pad Full | 1 | \$243.94 |
| 1-7, 8a, 10-18a, 20b, 23a | 505496 | Height Adj T-Arm Assm w/ Long Side Shield and Waterfall Pad Full, PAIR | 1 | \$487.89 |
| 1-7, 8a, 10-18b, 20b, 23a | 505648 | Height Adj T-Arm Assm w/ Short Side Shield and Waterfall Pad Full | 1 | \$243.94 |
| 1-7, 8a, 10-18b, 20b, 23a | 505649 | Height Adj T-Arm Assm w/ Short Side Shield and Waterfall Pad Full, PAIR | 1 | \$487.89 |
| 1-7, 8a, 10-18a, 20b, 23b | 505497 | Height Adj T-Arm Assm w/ Long Side Shield and Waterfall Pad Desk | 1 | \$243.94 |
| 1-7, 8a, 10-18a, 20b, 23b | 505498 | Height Adj T-Arm Assm w/ Long Side Shield and Waterfall Pad Desk, PAIR | 1 | \$487.89 |

| Item Number | Part Number | Description | Quantity | Retail |
|--|-------------|--|----------|----------|
| 1-7, 8a, 10-18b, 20b, 23b | 505650 | Height Adj T-Arm Assm w/ Short Side Shield and Waterfall Pad Desk | 1 | \$243.94 |
| 1-7, 8a, 10-18b, 20b, 23b | 505651 | Height Adj T-Arm Assm w/ Short Side Shield and Waterfall Pad Desk, PAIR | 1 | \$487.89 |
| 1-7, 8a, 9-18a, 19a, 20a, 21, 22, 24, 25 | 505499 | Height Adj T-Arm Assm w/ Long Side Shield and Standard Pad Full, 1" Receiver | 1 | \$270.39 |
| 1-7, 8a, 9-18a, 19a, 20a, 21, 22, 24, 25 | 505500 | Height Adj T-Arm Assm w/ Long Side Shield and Standard Pad Full, 1" Receiver, PAIR CAT | 1 | \$540.78 |
| 1-7, 8a, 9-18b, 19a, 20a, 21, 22, 24, 25 | 505652 | Height Adj T-Arm Assm w/ Short Side Shield and Standard Pad Full, 1" Receiver | 1 | \$270.39 |
| 1-7, 8a, 9-18b, 19a, 20a, 21, 22, 24, 25 | 505653 | Height Adj T-Arm Assm w/ Short Side Shield and Standard Pad Full, 1" Receiver, PAIR CAT | 1 | \$540.78 |
| 1-7, 8a, 9-18a, 19b, 20a, 21, 22, 24, 25 | 505501 | Height Adj T-Arm Assm w/ Long Side Shield and Standard Pad Desk, 1" Receiver CAT | 1 | \$270.39 |
| 1-7, 8a, 9-18a, 19b, 20a, 21, 22, 24, 25 | 505502 | Height Adj T-Arm Assm w/ Long Side Shield and Standard Pad Desk, 1" Receiver, PAIR CAT | 1 | \$540.78 |
| 1-7, 8a, 9-18b, 19b, 20a, 21, 22, 24, 25 | 505654 | Height Adj T-Arm Assm w/ Short Side Shield and Standard Pad Desk, 1" Receiver CAT | 1 | \$270.39 |
| 1-7, 8a, 9-18b, 19b, 20a, 21, 22, 24, 25 | 505655 | Height Adj T-Arm Assm w/ Short Side Shield and Standard Pad Desk, 1" Receiver, PAIR CAT | 1 | \$540.78 |
| 1-7, 8a, 9-18a, 20b, 21, 22, 23a, 24, 25 | 505503 | Height Adj T-Arm Assm w/ Long Side Shield and Waterfall Pad Full, 1" Receiver CAT | 1 | \$280.02 |
| 1-7, 8a, 9-18a, 20b, 21, 22, 23a, 24, 25 | 505504 | Height Adj T-Arm Assm w/ Long Side Shield and Waterfall Pad Full, 1" Receiver, PAIR CAT | 1 | \$560.04 |
| 1-7, 8a, 9-18b, 20b, 21, 22, 23a, 24, 25 | 505656 | Height Adj T-Arm Assm w/ Short Side Shield and Waterfall Pad Full, 1" Receiver CAT | 1 | \$280.02 |
| 1-7, 8a, 9-18b, 20b, 21, 22, 23a, 24, 25 | 505657 | Height Adj T-Arm Assm w/ Short Side Shield and Waterfall Pad Full, 1" Receiver, PAIR CAT | 1 | \$560.04 |
| 1-7, 8a, 9-18a, 20b, 21, 22, 23b, 24, 25 | 505505 | Height Adj T-Arm Assm w/ Long Side Shield and Waterfall Pad Desk, 1" Receiver CAT | 1 | \$280.02 |
| 1-7, 8a, 9-18a, 20b, 21, 22, 23b, 24, 25 | 505506 | Height Adj T-Arm Assm w/ Long Side Shield and Waterfall Pad Desk, 1" Receiver, PAIR CAT | 1 | \$560.04 |
| 1-7, 8a, 9-18b, 20b, 21, 22, 23b, 24, 25 | 505658 | Height Adj T-Arm Assm w/ Short Side Shield and Waterfall Pad Desk, 1" Receiver CAT | 1 | \$280.02 |
| 1-7, 8a, 9-18b, 20b, 21, 22, 23b, 24, 25 | 505659 | Height Adj T-Arm Assm w/ Short Side Shield and Waterfall Pad Desk, 1" Receiver, PAIR CAT | 1 | \$560.04 |