
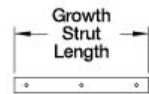


CR45 Rotary Frame - Growth

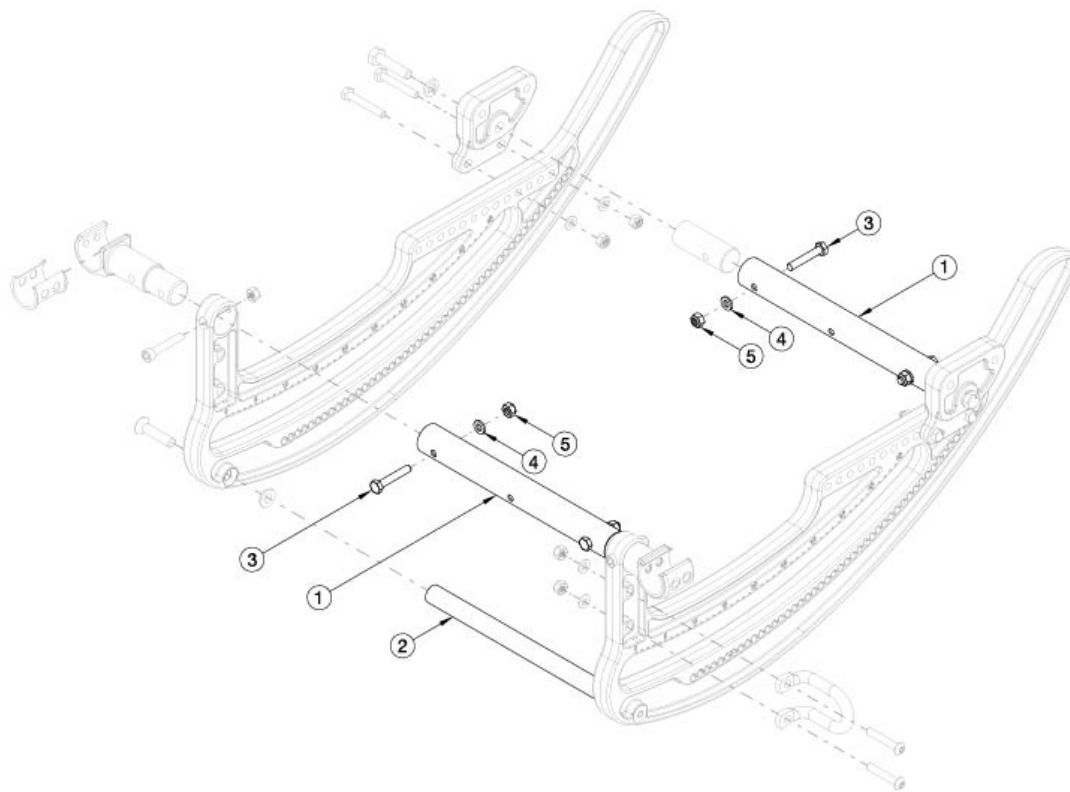

Chair Width	Strut Length
14"	8.46"
15"	9.46"
16"	10.46"
17"	11.46"
18"	12.46"
19"	13.46"
20"	14.46"
21"	15.46"
22"	16.46"

Has your width changed? Will your current Rotary Strut work? If not, please select accordingly.

Item Number	Part Number	Description	Quantity
1a	112450	Frame Strut Tube 14W	2
1b	112451	Frame Strut Tube 15W	2
1c	112452	Frame Strut Tube 16W	2
1d	112453	Frame Strut Tube 17W	2
1e	112454	Frame Strut Tube 18W	2
1f	112455	Frame Strut Tube 19W	2
1g	112456	Frame Strut Tube 20W	2
1h	112457	Frame Strut Tube 21W	2
1i	112458	Frame Strut Tube 22W	2
2a	112460	Rotary Strut 14W	1
2b	112461	Rotary Strut 15W	1

Item Number	Part Number	Description	Quantity
2c	112462	Rotary Strut 16W	1
2d	112463	Rotary Strut 17W	1
2e	112464	Rotary Strut 18W	1
2f	112465	Rotary Strut 19W	1
2h	112466	Rotary Strut 20W	1
2h	112467	Rotary Strut 21W	1
2i	112468	Rotary Strut 22W	1
3	102386	M6 x 35mm HHCS Blk Zc	4
4	100746	M6 Flat Washer Blk Zn <i>Use 12 with transit</i>	8
5	100658	M6 Nylock Nut Blk Zn <i>Use 14 with transit</i>	10