


Liberty Transit Forks and Stems

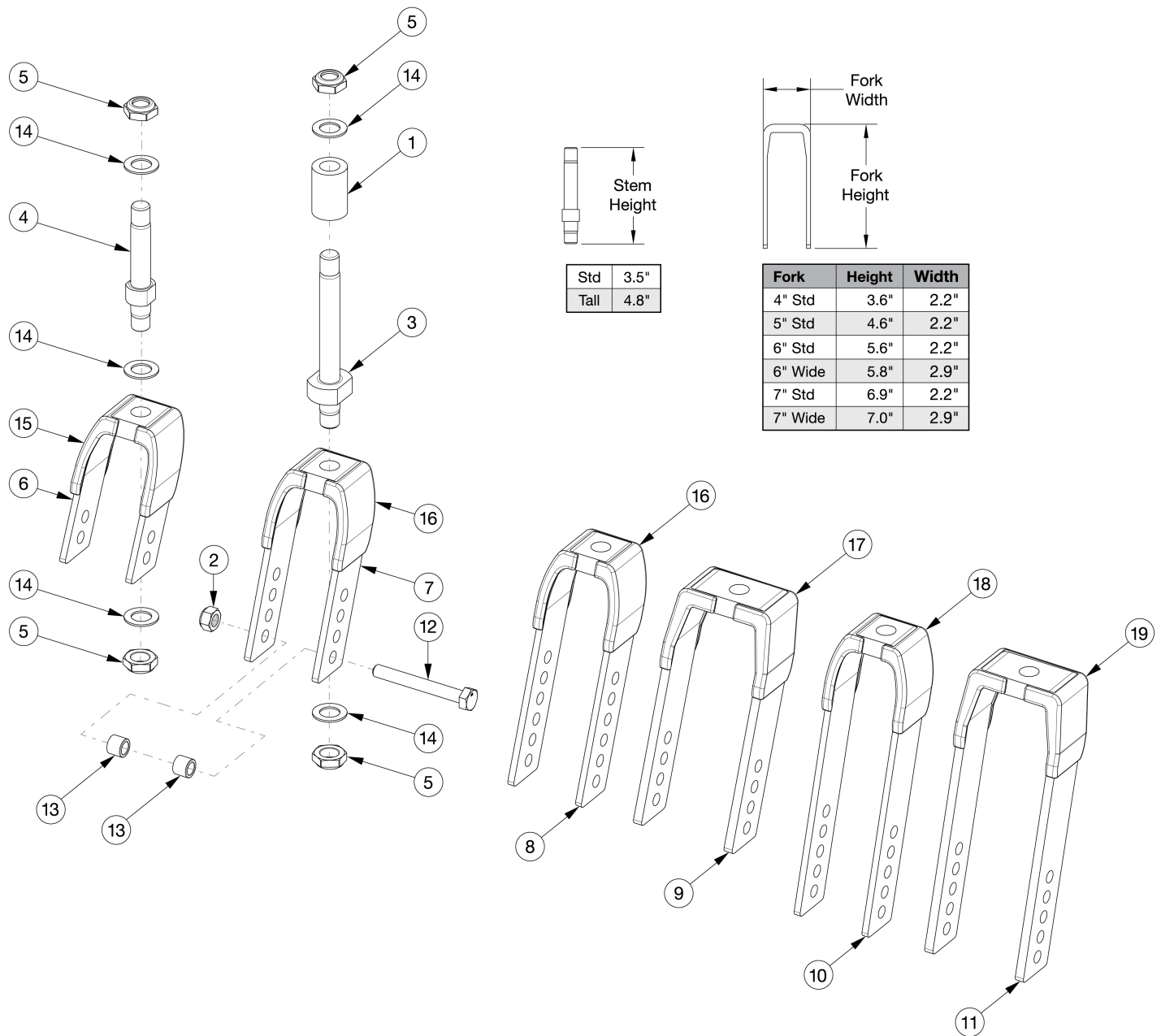

Item Number	Part Number	Description	Quantity
4, 5, 6, 14, 15	123024	Transit Fork Assembly, Black, 4" Std Stem, Tilt	1
4, 5, 7, 14, 16	123109	Transit Fork Assembly 5" Std Stem, Tilt	1
4, 5, 8, 14, 16	123107	Transit Fork Assembly 6" Std Stem, Tilt	1
4, 5, 9, 14, 17	123110	Transit Fork Assembly 6" Wide Std Stem, Tilt <i>Only used with 2" wide casters</i>	1
4, 5, 10, 14, 18	123108	Transit Fork Assembly 7" Std Stem, Tilt	1

Item Number	Part Number	Description	Quantity
4, 5, 11, 14, 19	123111	Transit Fork Assembly 7" Wide Std Stem, Tilt <i>Only used with 2" wide casters</i>	1
1, 3, 5, 6, 14, 15	123120	Transit Fork Assembly 4" Tall Stem, Tilt	1
1, 3, 5, 7, 14, 16	123121	Transit Fork Assembly 5" Tall Stem, Tilt	1
1, 3, 5, 8, 14, 16	123122	Transit Fork Assembly 6" Tall Stem, Tilt	1
1, 3, 5, 9, 14, 17	123123	Transit Fork Assembly 7" Tall Stem, Tilt <i>Only used with 2" wide casters</i>	1
1, 3, 5, 10, 14, 18	123124	Transit Fork Assembly 6" Wide Tall Stem, Tilt	1
1, 3, 5, 11, 14, 19	123125	Transit Fork Assembly 7" Wide Tall Stem, Tilt <i>Only used with 2" wide casters</i>	1
1	002476	Spacer, 3/4" OD x 1/2" ID x 1.23 <i>Only used with Tall Stems</i>	1
2	100659	M8 Nylock Nut Blk Zn	1
3, 5	506490	Caster Stem Tall, Liberty	1
4, 5	506491	Caster Stem Standard, Liberty	1
5	100529	M12 x 1.25mm Lock Nut Black	2
5, 6, 15	123132	Transit Fork 4", Tilt	1
5, 7, 16	123133	Transit Fork 5", Liberty	1
5, 8, 16	123134	Transit Fork 6", Liberty	1
5, 9, 17	123135	Transit Fork 6", Wide Liberty <i>Only used with 2" wide casters</i>	1
5, 10, 18	123136	Transit Fork 7", Liberty	1
5, 11, 19	123137	Transit Fork 7", Wide Liberty <i>Only used with 2" wide casters</i>	1
12a	101829	M8 x 65 HHCS Blk Zn	1
12b	101831	M8 x 80 HHCS Blk Zn <i>Only used with 2" wide casters</i>	1
13a	100792	Caster Spacer D8 x D12 x 10mm / Close Mount Handrim Spacer <i>Used with all casters except 6x2 and 8x2</i>	2
13b	001246	Caster Spacer Black 6 x 2 Poly .472ODX.319IDX.370LG <i>Only used with 6" x 2" casters</i>	2
14	100682	Washer 1/2" .518 x .875 x .047 F/W Black Zinc	3

Item Number	Part Number	Description	Quantity
2, 12a, 13a	111379	Caster Attaching Hardware Kit <i>Used with all casters except 6x2 and 8x2</i>	1
2, 12b, 13b	111380	Caster Attaching Hardware Kit for Wide Forks <i>Only used with 2" wide casters</i>	1