


Discontinued Catalyst 4 Height Adjustable Flip Back Armrest

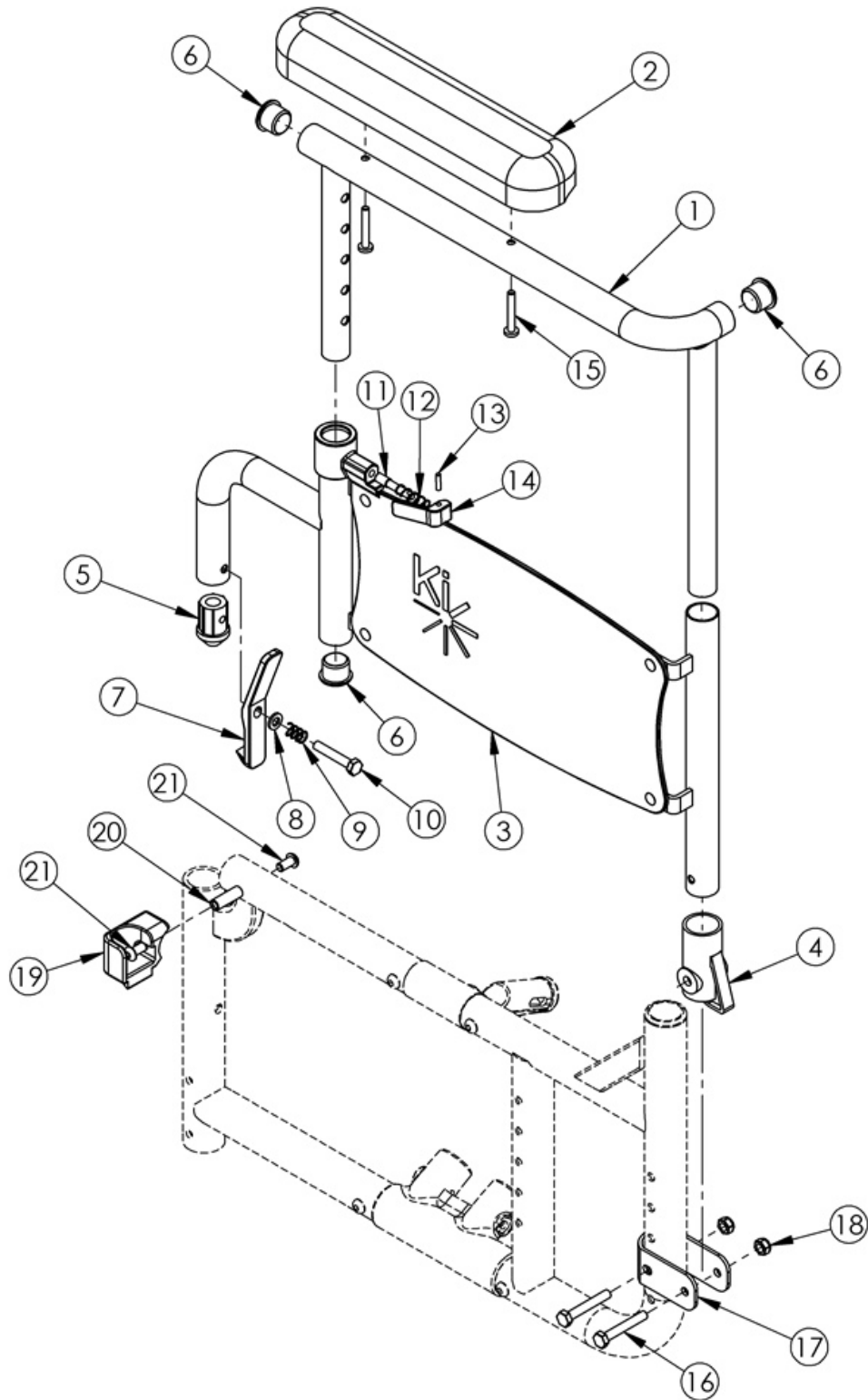

Item Number	Part Number	Description	Quantity	Retail
1b, 2b, 6, 15	503152	Armrest Assembly Height Adj Flip Back Upper Desk Right	1	\$126.64
1a, 2b, 6, 15	503153	Armrest Assembly Height Adj Flip Back Upper Desk Left	1	\$126.64
1c, 2a, 6, 15	503155	Armrest Assembly Height Adj Flip Back Upper Full Left	1	\$143.89
1b, 2b, 3b, 4-15	100192	Armrest Assembly Height Adj Flip Back Complete Desk Right	1	\$214.68
1c, 2a, 3a, 4-15	100191	Armrest Assembly Height Adj Flip Back Complete Full Left	1	\$220.48
3a, 4 - 14	503150	Armrest Assembly Height Adj Flip Back Lower Left	1	\$98.50
3b, 4 - 14	503151	Armrest Assembly Height Adj Flip Back Lower Right	1	\$98.50
16 -18	502263	Armrest Height Adjustable Flip Back Bracket Kit CAT4	1	\$12.68
19a, 20, 21	502265	Armrest Height Adjustable Flip Back Receiver Kit Left CAT	1	\$16.12
19b, 20, 21	502266	Armrest Height Adjustable Flip Back Receiver Kit Right CAT	1	\$16.12
11-14	500711	Arm Height Adj Flip Back Lever Assembly	1	\$25.85
1a	000382L	Armrest Flip Back Height Adj Frame Upper Desk Left	1	\$98.63
1c	000383L	Armrest Flip Back Height Adj Frame Upper Full Left	1	\$115.88
2a	100500	Arm Pad Full	1	\$23.64
2b	100501	Arm Pad Desk	1	\$23.64
3a, 11 - 14	000431-L	Armrest Flip Back Height Adj Frame Lower Left	1	\$85.24
3b, 11 - 14	000431-R	Armrest Flip Back Height Adj Frame Lower Right	1	\$85.24
4	100691	Armrest Flip Back Rear Pivot Bracket	1	\$3.36
5	100690	Armrest Flip Back Front Tube Plug	1	\$1.79
6	100803	Tube End Plug 7/8"	3	\$1.58
7	100689	Armrest Flip Back Release Lever	1	\$3.91
8	102072	Washer Nylon .234 ID x .406 OD x .062 Thick	1	\$0.53
9	100693	Compression Spring 0.34" OD x 0.70 LG	1	\$1.61
10	100860	M6 x 35 HHCS P/THD ZC	1	\$1.58
11	000099	Height Adj Armrest Lever Pin	1	\$19.13
12	000134	Height Adj Armrest Lever Spring	1	\$1.11
13	200257	Dowel Pin 1/8 x 1/2	1	\$2.63

Item Number	Part Number	Description	Quantity	Retail
14	000098	Height Adj Armrest Lever	1	\$3.58
15	100696	M5 x 30mm PPH Capscrew	2	\$1.58
16	100681	M6 x 40 CLS8.8 DIN931 HHCS ZC	2	\$1.58
17	000080	Armrest Flip Back Bracket CAT 4	1	\$8.22
18	100658	M6 Nylock Nut Blk Zn	2	\$1.05
19a	000081	Armrest Flip Back Receiver Left CAT	1	\$9.58
19b	000082	Armrest Flip Back Receiver Right CAT	1	\$9.58
20	000065	M6 x 23.68 Threaded Barrel	1	\$4.13
21	100667	M6 x 12 BTN SHCS BLKZC w/Patch	2	\$1.58