


Liberty 70 Degree Front Mount Hanger

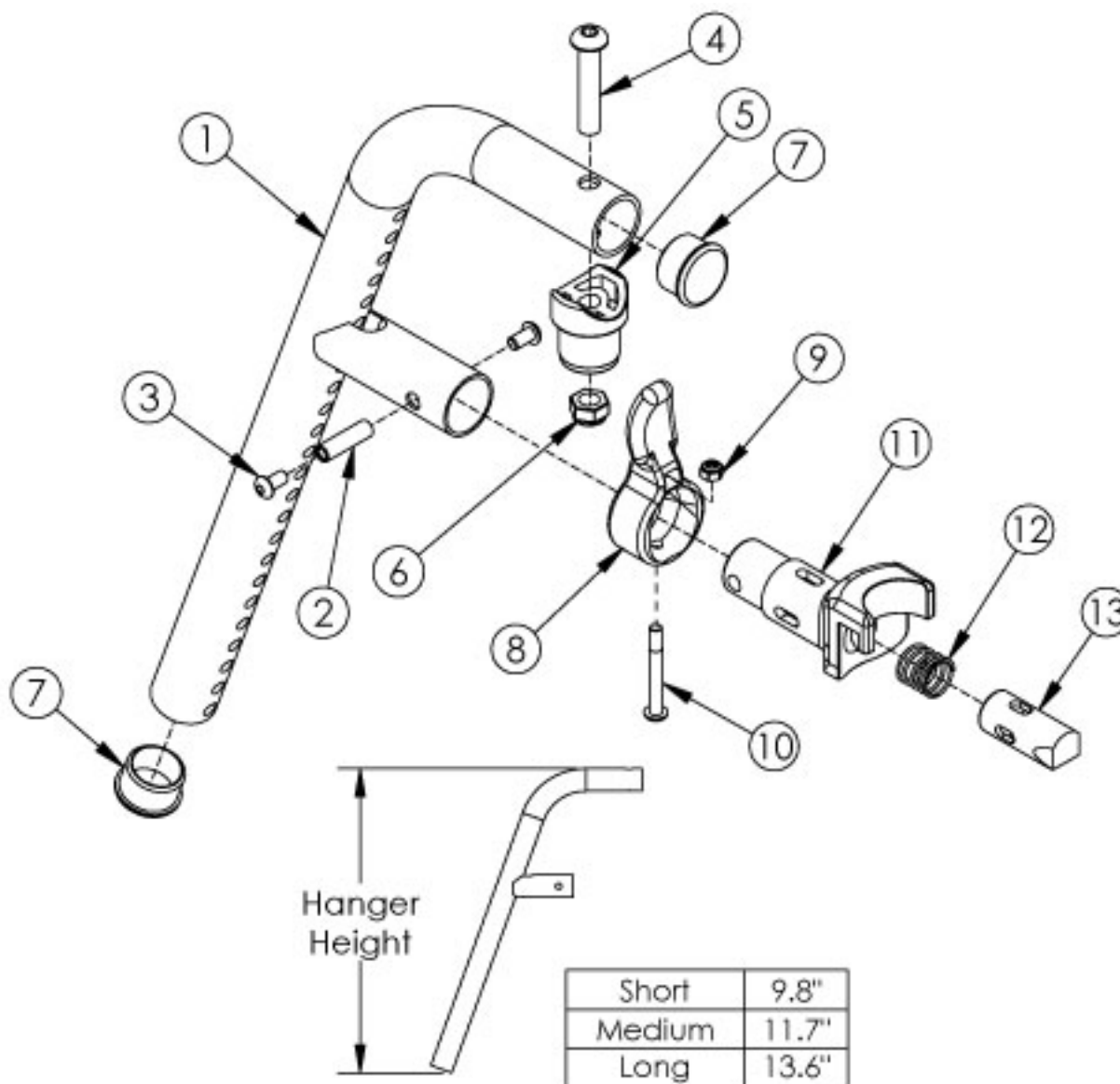

Please Note: If chair was built with hangers omitted, then latch blocks need to be ordered with hangers. Please reference the "Seat Frame" parts manual page.

Item Number	Part Number	Description	Quantity
1a, 7	501232_ _ _	Hanger Tube Assm 70 Front Mount Long _ _ _	1
1b, 7	501357_ _ _	Hanger Tube Assm 70 Front Mount Medium _ _ _	1
1c, 7	501358_ _ _	Hanger Tube Assm 70 Front Mount Short _ _ _	1
1a-13	504643_ _ _	Hanger Assm 70 Front Mount Long 4-Way - In-Line	1
1b-13	504641_ _ _	Hanger Assm 70 Front Mount Medium 4-Way - In-Line	1

Item Number	Part Number	Description	Quantity
1c-13	504639_ _ _	Hanger Assm 70 Front Mount Short 4-Way - In-Line	1
1a-13	504665_ _ _	Hanger Assm 70 Front Mount Long 4-Way - Out Left	1
1a-13	504666_ _ _	Hanger Assm 70 Front Mount Long 4-Way - Out Right	1
1b-13	504663_ _ _	Hanger Assm 70 Front Mount Medium 4-Way - Out Left	1
1b-13	504664_ _ _	Hanger Assm 70 Front Mount Medium 4-Way - Out Right	1
1c-13	504661_ _ _	Hanger Assm 70 Front Mount Short 4-Way - Out Left	1
1c-13	504662_ _ _	Hanger Assm 70 Front Mount Short 4-Way - Out Right	1
8-13	003225	Assembled 4-Way In Line Hanger Release	1
8-13	003225INST	Assembled 4-Way In Line Hanger Release with Instruction	1
8-13	003643	Assembled 4-Way Out Hanger Release, Left	1
8-13	003643INST	Assembled 4-Way Out Hanger Release, Left with Instruction	1
8-13	003642	Assembled 4-Way Out Hanger Release, Right	1
8-13	003642INST	Assembled 4-Way Out Hanger Release, Right with Instruction	1
2	000073	M5 x 24 Threaded Barrel	1
3	100669	M5 x 10 BTN SHCS BLKZC w/Patch	2
4	100673	M8 x 45 BTN SHCS BLKZC	1
5	000032	Footrest Hanger Pivot Saddle	1
6	100659	M8 Nylock Nut Blk Zn	1
7	100804	Tube End Plug 1"	2
8	003228	Hanger Latch, 4-Way, Lever	1
8	003228INST	Hanger Latch, 4-Way, Lever with Instruction	1
9	100657	M5 Nylock Nut Blk Zn	1
10	001374	M5 x 40 BHCS P/THD BLKZC	1
11	003226	Hanger Latch Bracket	1
11	003226INST	Hanger Latch Bracket with 4-way Release Instruction	1
12	000119	Hanger Spring	1
13	003227	Hanger Latch Pin	1

Item Number	Part Number	Description	Quantity
13	003227INST	Hanger Latch Pin with 4-Way Release Instruction	1

Matte Finish	
Code	Color Name
CGM	Charcoal Gray
EGM	Emerald Green
GGM	Graphite Gray*
IBM	Iridescent Burgundy
INM	Iridescent Navy
AGM	Military Green**
RRM	Retro Red
RBM	Royal Blue
SBM	Shadow Black
SWM	Smooth White

* Discontinued Color

** Formerly called Army Green

Gloss Finish	
Code	Color Name
BLK	Black
BLU	Blue
SKY	Blue Skies
BRO	Bright Orange*
PNK	Bubblegum Pink
BUR	Burgundy*
CBL	Candy Blue
PUR	Candy Purple
RED	Candy Red
COP	Copper
DPR	Deep Purple*
EBL	Electric Blue*
GRN	Forest Green*
FUS	Fuchsia*
GSM	Granny Smith
MTF	Metallic Fuchsia*
MOR	Metallic Orange
ORG	Orange*
SAP	Sapphire*
SIL	Silver
TEA	Teal
YLW	Yellow
FLM	Graphic Image Transfer