


# Ethos Wheel Lock Clamps

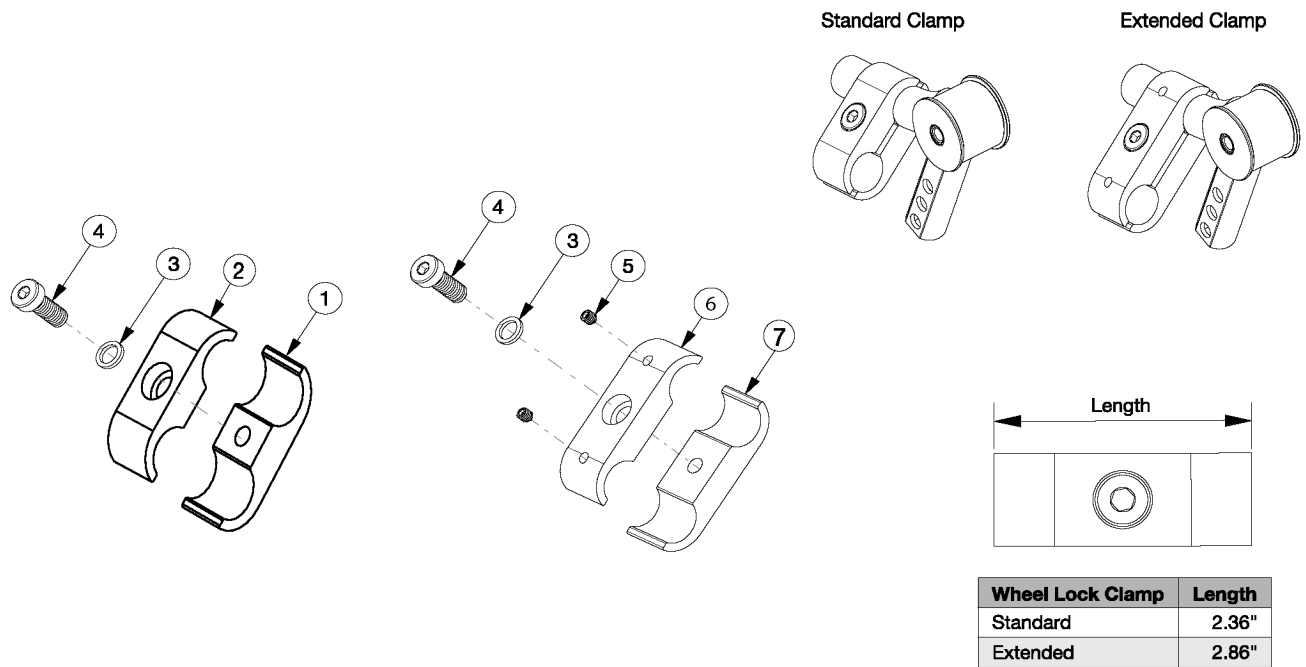

Item Number	Part Number	Description	Quantity
1 - 4	003973	Wheel Lock Clamp Assembly, Rigid <i>Used with 1.25" or less rear wheel spacing</i>	1
3 - 7	112430	Wheel Lock Clamp Assembly with Set Screws, Extended, Rigid <i>Replaced part number 004162. Used with 1.5" or greater rear wheel spacing or if Height Adjustable T-Arms are present</i>	1
1	003974	Wheel Lock Clamp Threaded	1
2	003975	Wheel Lock Clamp C'Bored	1
3	101701	Washer 3.315/.350 x .425/.525 x .059/.067 18-8 F/W	1
4	103327	M8 x 20 LHCS Blk Zc	1
5	105015	M5 X 6 DIN914 SSS CONE PT PLN	2
7	004163	Wheel Lock Clamp Threaded Long	1