


# Catalyst E Vertical Axle Plate

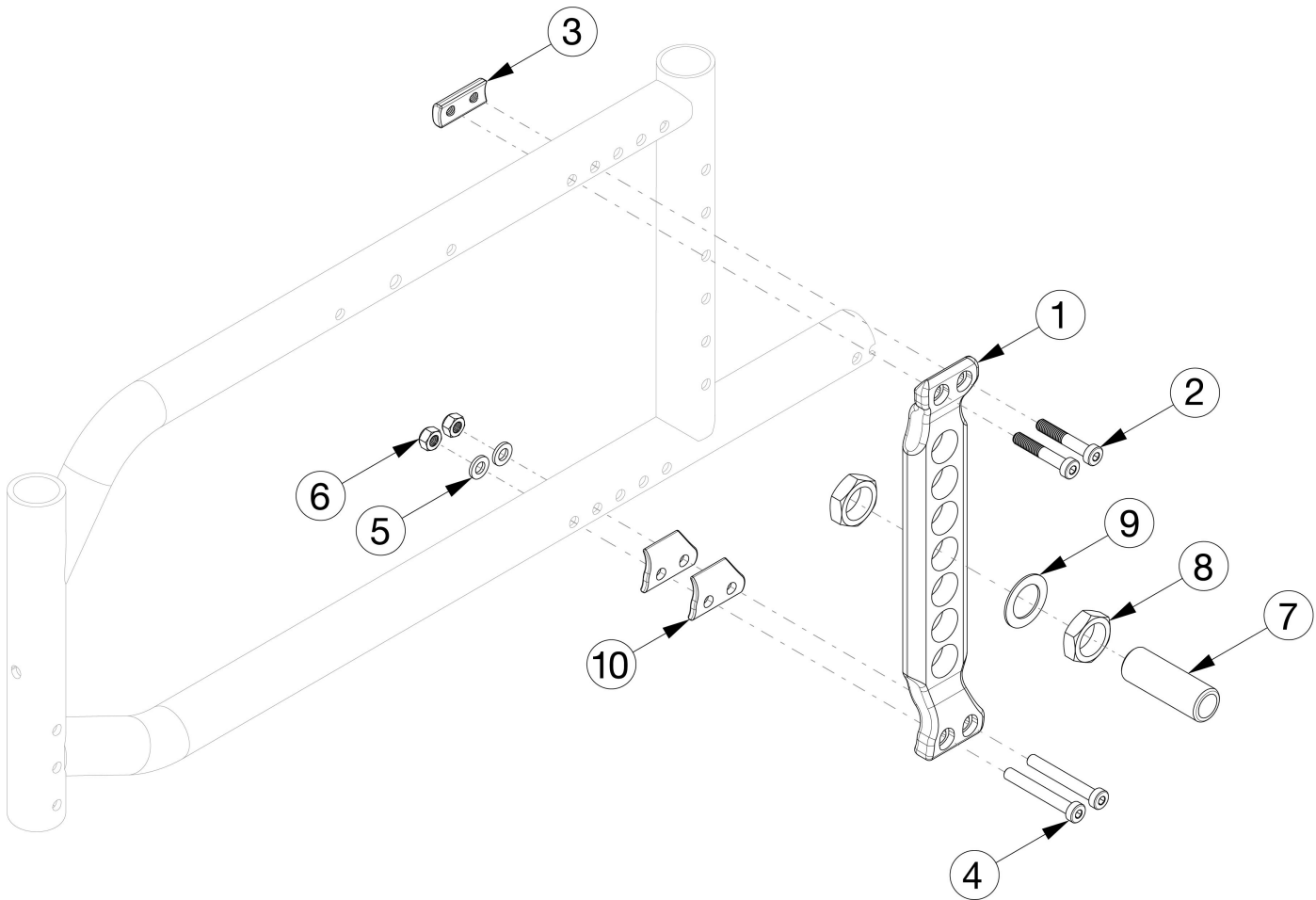

Item Number	Part Number	Description	Quantity	Retail
1, 2, 4-6, 7	506127	Vertical Axle Plate Assembly, Black, 0° Camber <i>This kit is only for one side</i>	1	\$95.27
1, 7-9	506132	Vertical Axle Plate, Black, with Receiver Assembly 1/2" <i>This kit is only for one side</i>	1	\$89.49
7-9	114172	Axle Receiver Hardware <i>This kit is only for one side</i>	1	\$22.90
1	003519INST	VX Axle Plate Black with Instruction	1	\$66.58
2	103370	M6 x 35 Low SHCS BLK ZC w/Patch	2	\$4.41
3	003520	Threaded Axle Nut Plate	1	\$11.04
4	101123	M6 x 45 Low SHCS BLK ZC	2	\$1.66
5	100746	M6 Flat Washer Blk Zn	2	\$1.66
6	100658	M6 Nylock Nut Blk Zn	2	\$1.10

<b>Item Number</b>	<b>Part Number</b>	<b>Description</b>	<b>Quantity</b>	<b>Retail</b>
7	100756	Axle Receiver 1/2"	1	\$18.56
8	100514	Axle Receiver Nut M18 x D12 x 8mm	2	\$1.62
9	100583	Axle Plate Washer 18mm	1	\$1.10