


# Focus CR / CR45 O2 Holder

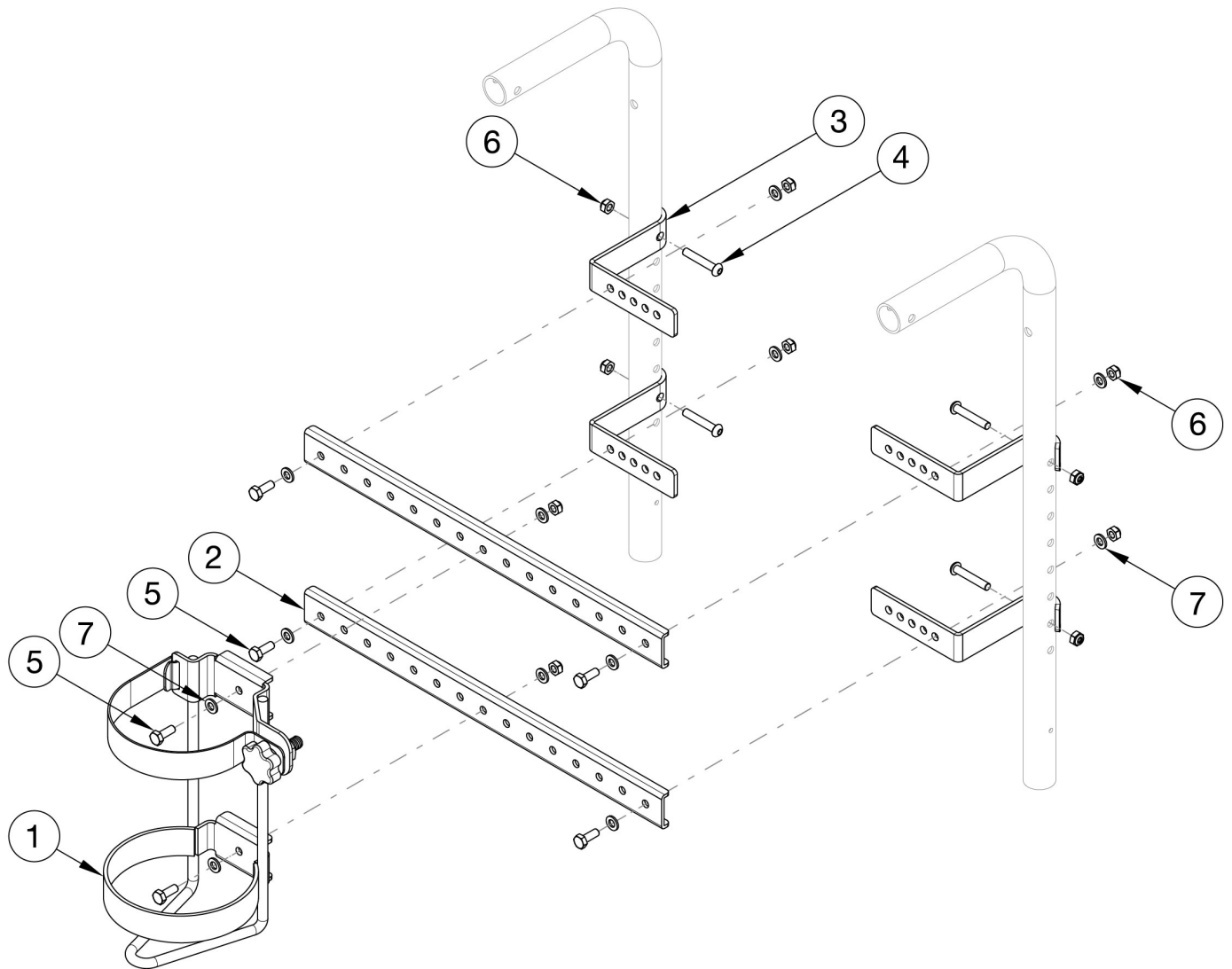

Item Number	Part Number	Description	Quantity
1, 2a, 3 - 7	501441	O2 Holder Assembly, 14-16 Width, Tilt	1
1, 2b, 3 - 7	501442	O2 Holder Assembly, 16-18 Width, Tilt	1
1, 2c, 3 - 7	501443	O2 Holder Assembly, 18-20 Width, Tilt	1
1, 2d, 3 - 7	501444	O2 Holder Assembly, 20-24 Width, Tilt	1
1	001416	O2 Holder	1
2a	001385	Strut, Channel, 14-16 Width, Tilt	2
2b	001386	Strut, Channel, 16-18 Width, Tilt	2

<b>Item Number</b>	<b>Part Number</b>	<b>Description</b>	<b>Quantity</b>
2c	001387	Strut, Channel, 18-20 Width, Tilt	2
2d	001388	Strut, Channel, 20-24 Width, Tilt	2
3	001280	Mount Bracket, 1" Tube Tilt	4
4	101811	M6 x 35 BHCS Blk Zn	4
5	102327	M6 x 16 HHCS Blk Zn	6
6	100658	M6 Nylock Nut Blk Zn	10
7	100746	M6 Flat Washer Blk Zn	12