

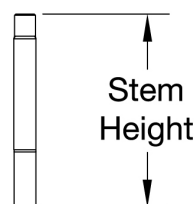


Catalyst Transit Forks and Stems

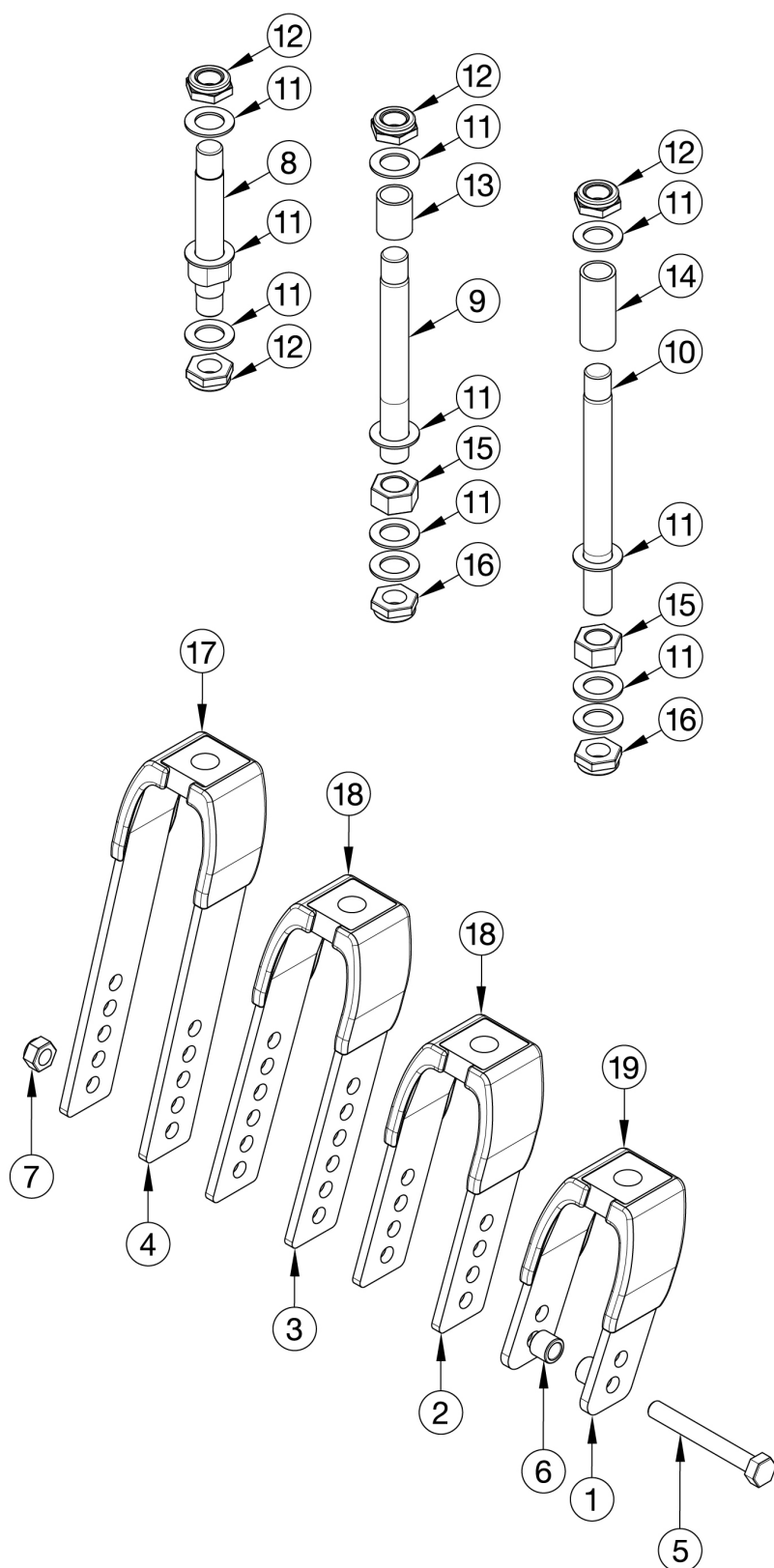

Std	3.5"
+ 3/4"	4.4"
+ 1 1/2"	5.1"

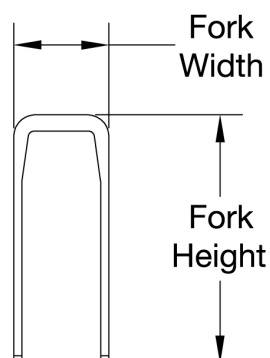

Fork	Height	Width
4" Std	3.6"	2.2"
5" Std	4.6"	2.2"
6" Std	5.6"	2.2"
6" Wide	5.8"	2.9"
7" Std	6.9"	2.2"
7" Wide	7.0"	2.9"

Item Number	Part Number	Description	Quantity
5a, 6a, 7	502653	Caster Attachment Hardware Kit CAT/FCS	1
5b, 6b, 7	502654	Caster Attachment Hardware Kit for 6" and 7" Wide Fork, CAT/FCS	1
8, 11, 12	500866	Std Caster Stem Assembly CAT	1
9, 11, 12, 13, 15, 16	500001	+3/4 Caster Stem Assembly CAT	1
10, 11, 12, 14, 15, 16	500002	+1 1/2 Caster Stem Assembly CAT	1
1a, 8, 11, 12, 19	123075	Transit Fork Assembly, Black, 4" Std Stem CAT	1
1b, 8, 11, 12, 19	123075_ _ _	Color Anodized/Color Accent Transit Fork Assembly 4" Std Stem CAT	1
1a, 9, 11, 12, 13, 15, 16, 19	123076	Transit Fork Assembly, Black, 4" +3/4" Stem CAT	1
1b, 9, 11, 12, 13, 15, 16, 19	123076_ _ _	Color Anodized/Color Accent Transit Fork Assembly 4" +3/4" Stem CAT	1
1a, 10, 11, 12, 14, 15, 16, 19	123071	Transit Fork Assembly, Black, 4" +1 1/2" Stem CAT	1
1b, 10, 11, 12, 14, 15, 16, 19	123071_ _ _	Color Anodized/Color Accent Transit Fork Assembly 4" +1 1/2" Stem CAT	1
2a, 8, 11, 12, 18a	123077	Transit Fork Assembly, Black, 5" Std Stem CAT	1
2b, 8, 11, 12, 18a	123077_ _ _	Color Anodized/Color Accent Transit Fork Assembly 5" Std Stem CAT	1
2a, 9, 11, 12, 13, 15, 16, 18a	123072	Transit Fork Assembly, Black, 5" +3/4" Stem CAT	1
2b, 9, 11, 12, 13, 15, 16, 18a	123072_ _ _	Color Anodized/Color Accent Transit Fork Assembly 5" +3/4" Stem CAT	1
2a, 10, 11, 12, 14, 15, 16, 18a	123078	Transit Fork Assembly, Black, 5" +1 1/2" Stem CAT	1
2b, 10, 11, 12, 14, 15, 16, 18a	123078_ _ _	Color Anodized/Color Accent Transit Fork Assembly 5" +1 1/2" Stem CAT	1
3a, 8, 11, 12, 18a	123073	Transit Fork Assembly, Black, 6" Std Stem CAT	1
3b, 8, 11, 12, 18a	123073_ _ _	Color Anodized/Color Accent Transit Fork Assembly 6" Std Stem CAT	1
3a, 9, 11, 12, 13, 15, 16, 18a	123079	Transit Fork Assembly, Black, 6" +3/4" Stem CAT	1
3b, 9, 11, 12, 13, 15, 16, 18a	123079_ _ _	Color Anodized/Color Accent Transit Fork Assembly 6" +3/4" Stem CAT	1
3a, 10, 11, 12, 14, 15, 16, 18a	123080	Transit Fork Assembly, Black, 6" +1 1/2" Stem CAT	1
3b, 10, 11, 12, 14, 15, 16, 18a	123080_ _ _	Color Anodized/Color Accent Transit Fork Assembly 6" +1 1/2" Stem CAT	1
4a, 8, 11, 12, 17a	123074	Transit Fork Assembly, Black, 7" Std Stem CAT	1
4b, 8, 11, 12, 17a	123074_ _ _	Color Anodized/Color Accent Transit Fork Assembly 7" Std Stem CAT	1
4a, 9, 11, 12, 13, 15, 16, 17a	123081	Transit Fork Assembly, Black, 7" +3/4" Stem CAT	1
4b, 9, 11, 12, 13, 15, 16, 17a	123081_ _ _	Color Anodized/Color Accent Transit Fork Assembly 7" +3/4" Stem CAT	1

Item Number	Part Number	Description	Quantity
4a, 10, 11, 12, 14, 15, 16, 17a	123082	Transit Fork Assembly, Black, 7" +1 1/2" Stem CAT	1
4b, 10, 11, 12, 14, 15, 16, 17a	123082_ _ _	Color Anodized/Color Accent Transit Fork Assembly 7" +1 1/2" Stem CAT	1
3c, 8, 11, 12, 18b	123088	Transit Fork Assembly 6" Wide Std Stem CAT <i>Only used with 2" wide casters</i>	1
3c, 9, 11, 12, 13, 15, 16, 18b	123090	Transit Fork Assembly 6" Wide +3/4" Stem CAT <i>Only used with 2" wide casters</i>	1
3c, 10, 11, 12, 14, 15, 16, 18b	123091	Transit Fork Assembly 6" Wide +1 1/2" Stem CAT <i>Only used with 2" wide casters</i>	1
4c, 8, 11, 12, 17b	123089	Transit Fork Assembly 7" Wide Std Stem CAT <i>Only used with 2" wide casters</i>	1
4c, 9, 11, 12, 13, 15, 16, 17b	123092	Transit Fork Assembly 7" Wide +3/4" Stem CAT <i>Only used with 2" wide casters</i>	1
4c, 10, 11, 12, 14, 15, 16, 17b	123093	Transit Fork Assembly 7" Wide +1 1/2" Stem CAT <i>Only used with 2" wide casters</i>	1
1a, 12, 16, 19	123101	Transit Fork, Black, 4" Catalyst	1
1b, 12, 16, 19	123101_ _ _	Color Anodized/Color Accent Transit Fork 4" Catalyst	1
2a, 12, 16, 18a	123102	Transit Fork, Black, 5" Catalyst	1
2b, 12, 16, 18a	123102_ _ _	Color Anodized/Color Accent Transit Fork 5" Catalyst	1
3a, 12, 16, 18a	123103	Transit Fork, Black, 6" Catalyst	1
3b, 12, 16, 18a	123103_ _ _	Color Anodized/Color Accent Transit Fork 6" Catalyst	1
3c, 12, 16, 18b	123104	Transit Fork 6" Wide Catalyst <i>Only used with 2" wide casters</i>	1
4a, 12, 16, 17a	123105	Transit Fork, Black, 7" Catalyst	1
4b, 12, 16, 17a	123105_ _ _	Color Anodized/Color Accent Transit Fork 7" Catalyst	1
4c, 12, 16, 17b	123106	Transit Fork 7" Wide Catalyst <i>Only used with 2" wide casters</i>	1
5a	101829	M8 x 65 HHCS Blk Zn	1
5b	101831	M8 x 80 HHCS Blk Zn <i>Only used with 2" wide casters</i>	1
6a	100792	Caster Spacer D8 x D12 x 10mm / Close Mount Handrim Spacer <i>Used with all casters except 6x2 and 8x2</i>	2
6b	001246	Caster Spacer Black 6 x 2 Poly .472ODX.319IDX.370LG	2

Item Number	Part Number	Description	Quantity
		<i>Only used with 6" x 2" casters</i>	
7	100659	M8 Nylock Nut Blk Zn	1
8, 12	503813	Fork Stem Standard Catalyst	1
9, 16	503812	Fork Stem 3/4" Catalyst	1
10, 16	503811	Fork Stem 1 1/2" Catalyst	1
11	100682	Washer 1/2" .518 x .875 x .047 F/W Black Zinc <i>Use quantity 3 with standard stem</i>	4
12	100529	M12 x 1.25mm Lock Nut Black <i>Use quantity 1 with 3/4" and 1 1/2" stems</i>	2
13	000115	Fork Stem Spacer 3/4"	1
14	000114	Fork Stem Spacer 1 1/2" CAT	1
15	101869	1/2-20 G5 Hex Nut Hxfree Zc	1
16	101459	1/2-20 NTE JAM NYL INS L/N BLK ZC	1