


Focus Footplates - Aluminum Locking Multi-Angle Adjustable Footplate Front Mount

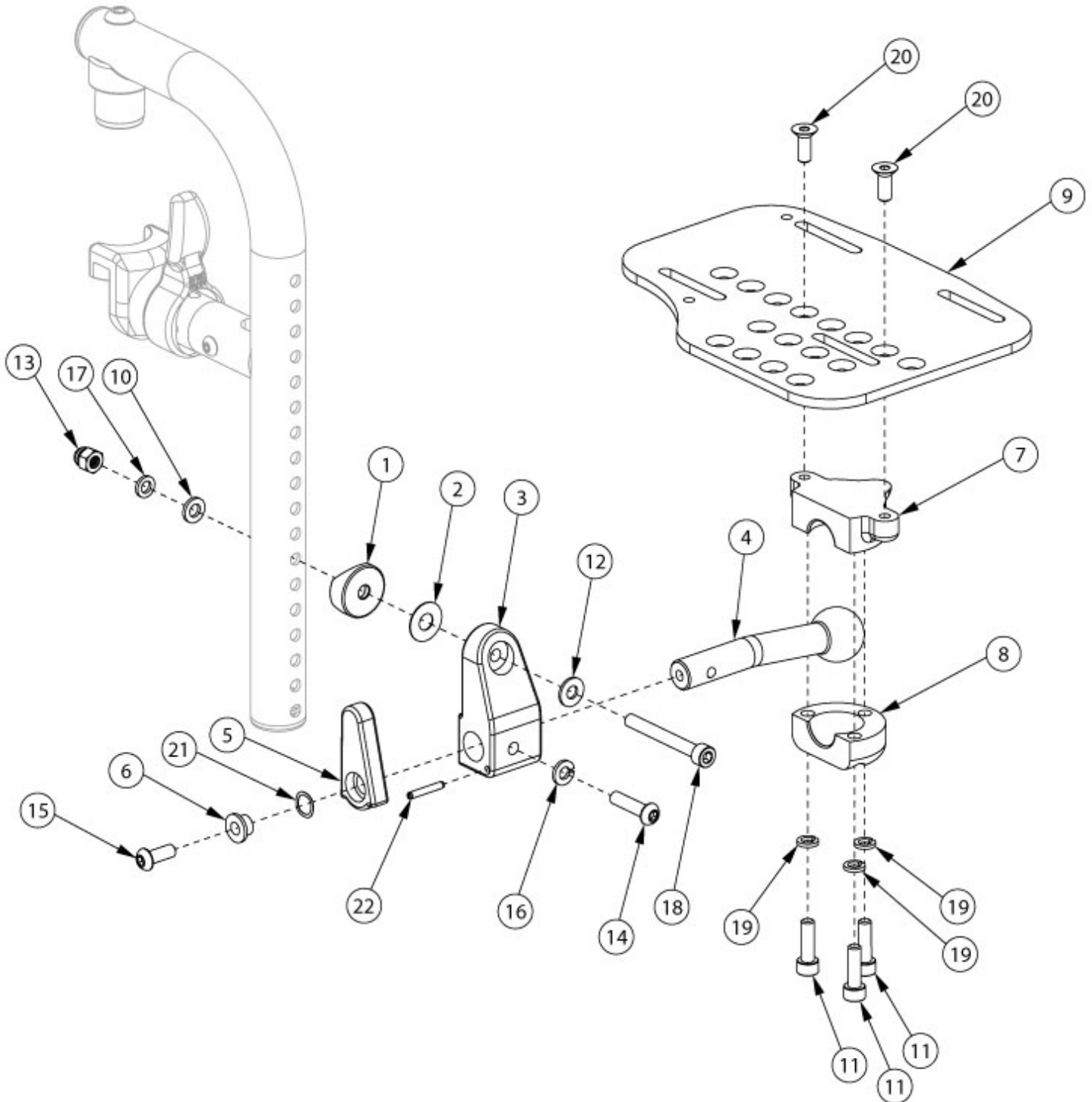

Sizes for footplates / assemblies are based on seat width with straight frame. If Focus chair has Offset Frame, add 2" to seat width.
 Only compatible with Front Mount Hangers.

Item Number	Part Number	Description	Quantity
1, 2, 3a, 4-8, 10-22	107618	Footplate Hardware Kit Aluminum Locking Multi-Angle Adjustable, Right	1
1, 2, 3b, 4-8, 10-22	107619	Footplate Hardware Kit Aluminum Locking Multi-Angle Adjustable, Left	1
1, 2, 3a-3b, 4-8, 10-22	107620	Footplate Hardware Kit Aluminum Locking Multi-Angle Adjustable, PAIR	1
1, 2, 3a, 4-8, 9a, 10-22	107605	Footplate Kit Aluminum Locking Multi-Angle Adjustable, 14-15" Wide, Right	1
1, 2, 3a, 4-8, 9c, 10-22	107606	Footplate Kit Aluminum Locking Multi-Angle Adjustable, 16-17" Wide, Right	1
1, 2, 3a, 4-8, 9e, 10-22	107607	Footplate Kit Aluminum Locking Multi-Angle Adjustable, 18-24" Wide, Right	1
1, 2, 3b, 4-8, 9b, 10-22	107610	Footplate Kit Aluminum Locking Multi-Angle Adjustable, 14-15" Wide, Left	1
1, 2, 3b, 4-8, 9d, 10-22	107611	Footplate Kit Aluminum Locking Multi-Angle Adjustable, 16-17" Wide, Left	1
1, 2, 3b, 4-8, 9f, 10-22	107612	Footplate Kit Aluminum Locking Multi-Angle Adjustable, 18-24" Wide, Left	1
1, 2, 3a-3b, 4-8, 9a-9b, 10-22	107615	Footplate Kit Aluminum Locking Multi-Angle Adjustable, 14-15" Wide, PAIR	1
1, 2, 3a-3b, 4-8, 9c-9d, 10-22	107616	Footplate Kit Aluminum Locking Multi-Angle Adjustable, 16-17" Wide, PAIR	1
1, 2, 3a-3b, 4-8, 9e-9f, 10-22	107617	Footplate Kit Aluminum Locking Multi-Angle Adjustable, 18-24" Wide, PAIR	1
1	000055	Saddle 1"	1
2	000428	Wear Washer	1
3a, 22	109294	Multi-Angle Footrest Mount 1", with Spring Pin, Right	1
3b, 22	109295	Multi-Angle Footrest Mount 1", with Spring Pin, Left	1
4	003820	Ball Shaft 1" Multi-Angle	1
5	003821	Latch, Footplate, Multi-Angle	1
6	003822	Bushing, Latch, Multi-Angle	1
7	003823	Clamp, Upper, Footplate, Multi-Angle	1
8	003824	Clamp, Lower, Footplate, Multi-Angle	1
9a	003829	Footplate, Multi-Angle, 14-15W, Right	1
9b	003830	Footplate, Multi-Angle, 14-15W, Left	1
9c	003831	Footplate, Multi-Angle, 16-17W, Right	1
9d	003832	Footplate, Multi-Angle, 16-17W, Left	1
9e	003833	Footplate, Multi-Angle, 18-24W, Right	1
9f	003834	Footplate, Multi-Angle, 18-24W, Left	1

Item Number	Part Number	Description	Quantity
10	100746	M6 Flat Washer Blk Zn	1
11	101180	M6 x 20 SHCS BLK ZC	3
12	101455	M6 Belleville Washer (.252 x .551 x .050)	1
13	101456	M6 Ovl Top L/N ZC	1
14	101593	M6 x 25 BTN SHCS BLKZC	1
15	101721	M6 X 16 Btn Shcs W/ Patch	1
16	101835	M6 Lock Washer Blk Zn	1
17	102072	Washer Nylon .234 ID x .406 OD x .062 Thick	1
18	102473	M6 X 50 CLS12.9 SHCS BLK ZC	1
19	102810	M6 Lockwasher, Hi-Collar, ZC	3
20	102936	M6 X 16 FHCS, Blk Zn	2
21	103539	Wave Spring, 3/8IDx1/2ODx.009T, SS	1