


(Discontinued) Rogue One Arm Drive Hex Style

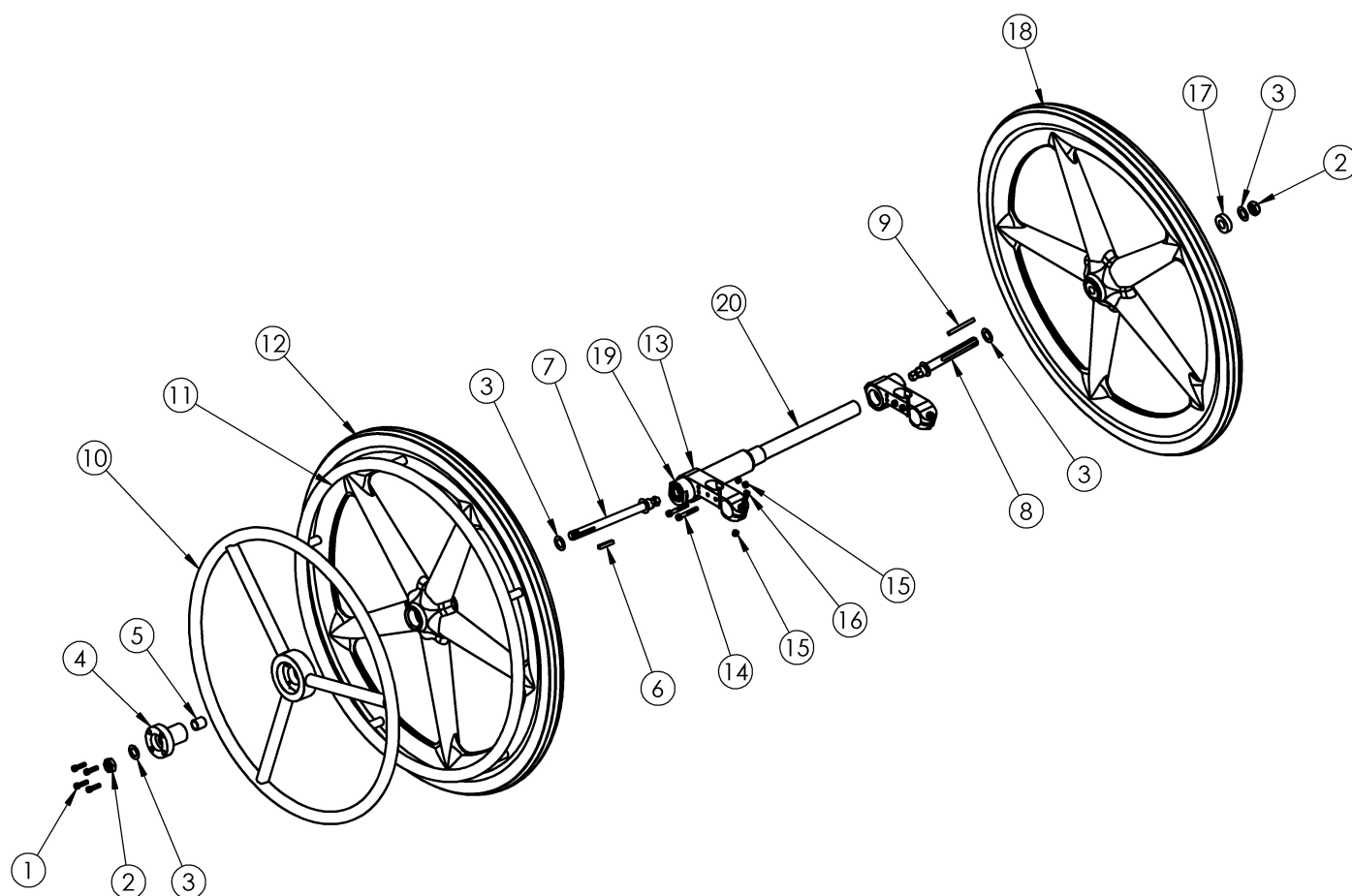

**One Arm Drive Kits do not include rear tires*

Item Number	Part Number	Description	Quantity
10a	003198	One Arm Drive Handrim Aluminum 24"	1
13a	003209	One Arm Drive Rogue/Clik Axle Mount 2 Degree Camber, PAIR	1
13b	003210	One Arm Drive Rogue/Clik Axle Mount 4 Degree Camber Each	1
14	101734	M5 x 35 SHCS BLK ZC	4
15	100658	M6 Nylock Nut Blk Zn	4
16	101717	M5 x 25 SHCS BLK ZC	4

Rogue One Arm Drive - Drive Shaft Utilization Chart

Chair Width	Camber	Armrest	Drive Shaft Measurement	Drive Shaft Part Number
11"	2°	Swing Away / Flip Up / No Armrests	8.63"	102750
		T-Arm Armrests	9.13"	102637
	4°	Any Armrest	9.13"	102637
12"	2°	Swing Away / Flip Up / No Armrests	9.13"	102637
		T-Arm Armrests	9.75"	102638
	4°	Any Armrest	9.75"	102638
13"	2°	Swing Away / Flip Up / No Armrests	9.75"	102638
		T-Arm Armrests	10.75"	102639
	4°	Any Armrest	10.75"	102639
14"	2°	Swing Away / Flip Up / No Armrests	10.75"	102639
		T-Arm Armrests	11.75"	102640
	4°	Any Armrest	11.75"	102640
15"	2°	Swing Away / Flip Up / No Armrests	11.75"	102640
		T-Arm Armrests	12.75"	102641
	4°	Any Armrest	12.75"	102641
16"	2°	Swing Away / Flip Up / No Armrests	12.75"	102641
		T-Arm Armrests	13.84"	102642
	4°	Any Armrest	13.84"	102642
17"	2°	Swing Away / Flip Up / No Armrests	13.84"	102642
		T-Arm Armrests	14.69"	102643
	4°	Any Armrest	14.69"	102643
18"	2°	Swing Away / Flip Up / No Armrests	14.69"	102643
		T-Arm Armrests	15.69"	102644
	4°	Any Armrest	15.69"	102644
19"	2°	Swing Away / Flip Up / No Armrests	15.69"	102644
		T-Arm Armrests	16.69"	102645
	4°	Any Armrest	16.69"	102645
20"	2°	Swing Away / Flip Up / No Armrests	16.69"	102645
		T-Arm Armrests	17.69"	102646
	4°	Any Armrest	17.69"	102646

Revised 5/10/2016