




PARTS MANUAL

JULY 2014

Front Cover	1
QUICKIE 2 LITE (Chairs w/ S/N Prefix Q2N & Q2L)	4
BASE FRAME	4
FRAMES	4
SWING-AWAY FRONT SIDE FRAME prior to Q2L-152965	4
SWING-AWAY FRONT SIDE FRAME After Q2L-152965	5
FIXED FRONT SIDE FRAME	6
REAR SIDE FRAME	8
CROSS TUBE ASSEMBLY QUICKIE 2 LITE	9
TRANSIT KIT QUICKIE 2 LITE prior to Q2L-152965	12
WC19TRANSIT KIT QUICKIE 2 LITE After Q2L-152965	13
AXLE PLATES & AXLES	13
QUICK RELEASE AXLES	14
AXLE PLATE ASSEMBLY	15
FRONT CASTERS & FORK	15
FRONT CASTER & FORK	16
FROG LEGS SUSPENSION FORK	20
CASTER PIN LOCKS	22
MICRO CASTERS	23
3 SPOKE CASTERS	24
SOFT ROLL CASTERS	26
ALUMINUM CASTERS	27
ALUMINUM SOFT ROLL CASTERS	28
ANTI-TIPS	28
ANTI-TIP	29
WHEEL LOCKS	29
PUSH TO LOCK WHEEL LOCK	30
PULL TO LOCK WHEEL LOCK	31
ERGO SCISSOR WHEEL LOCK	32
PUSH & PULL WITH GRADE AID ASSM	33
COMPACT WHEEL LOCK	34
WHEEL LOCK EXTENSION HANDLE	35
WHEELS & TIRES	35
WHEELS	35
MAG	36
LITE SPOKE	38
NEW ULTRA LITE SPOKE	39
SPINERGY SPOX (18 SPOKE)	40
SPINERGY LX (12 SPOKE)	41
MOUNTAIN	43
TIRES	43
PNEUMATIC & PNEU. AIRLESS TIRE	44
FULL POLY TIRE	45
HIGH PRESSURE CLINCHER TIRE	46
KNOBBY TIRE	47
SCHWALBE MARATHON PLUS TIRE	48
KIK MAKO TIRE	49
URBAN ASSAULT TIRE	50
MOUNTAIN TIRE	51
TUBES	51
TUBES AND AIRLESS INSERTS	52
SEATING AND POSITIONING	53
SEAT	53
SEAT SLING	54
SEAT CUSHION	56
BACK	56
STANDARD BACK FRAME	57
ANGLE ADJUSTABLE BACK FRAME	59
BACK UPHOLSTERY	60
FOLDING PUSH HANDLE ADJUSTABLE BACKREST UPHOLSTERY	61
ADJUSTABLE BACK UPHOLSTERY	62
INTEGRAL PUSH HANDLE BACKREST UPHOLSTERY	63
HANGERS & FOOTPLATES Prior to Q2L-152965	63
FIXED FRONT FRAME FOOTPLATES	63
TUBULAR SIDE LATCH FOOTREST ASSM	64
AUTOFOLD FOOTREST ASSM	66
PLATFORM FLIP-UP FOOTPLATE	68
COMPOSITE FOOTPLATE	70
COMPOSITE ANGLE ADJUSTABLE FOOT PLATE	74
ADULT ANGLE-ADJ FOOTPLATE EXT MOUNT	78


S/A FRAME HANGERS	82
HANGERS.....	82
70 DEG HANGER SWING IN-OUT	82
70 DEG HANGER V SWING IN-OUT	83
80 DEG HANGER SWING IN-OUT.....	84
FOOTPLATES.....	84
COMPOSITE FOOTPLATE.....	85
COMPOSITE ANGLE ADJUSTABLE FOOT PLATE	89
ADULT ANGLE-ADJ FOOTPLATE EXT MOUNT	93
PLATFORM FLIP-UP FOOTPLATE.....	96
MULTI-POSITION LATCH STUD SUPPORT	98
HANGERS AND FOOTPLATES After Q2L-152965.....	98
EXTENSION MOUNT HANGERS.....	98
60 DEG SWING IN/OUT HANGER	99
70 DEG SWING IN/OUT HANGER	100
80 DEG SWING IN/OUT HANGER	101
FRONT MOUNT HANGERS	101
70 DEG FRONT MOUNT SWING IN/OUT HANGER	102
80 DEG FRONT MOUNT SWING IN/OUT HANGER	103
90 DEG FRONT MOUNT SWING IN/OUT HANGER	105
ADULT ANGLE-ADJ FOOTPLATE FRONT MOUNT.....	107
FRONT MOUNT HANGER EXTENSIONS.....	109
ELR-ALR	109
ALR	110
EXTENSION MOUNT ELR.....	112
FRONT MOUNT ELR.....	114
ELR/ALR ALUMINUM FOOTPLATE	116
EXTENSION MOUNT HEMI HANGERS.....	117
60 DEG SWING IN/OUT HEMI HANGER	118
70 DEG SWING IN/OUT HEMI HANGER	119
FOOTPLATES.....	119
COMPOSITE FOOTPLATE.....	120
COMPOSITE ANGLE ADJUSTABLE FOOT PLATE	124
ADULT ANGLE-ADJ FOOTPLATE EXT MOUNT	127
ARMRESTS & RECEIVERS	129
ARMRESTS.....	129
FLIP BACK HEIGHT ADJUSTABLE LITE ARMREST	130
SINGLE POST HEIGHT ADJUSTABLE ARMREST	131
PADDED SWING-AWAY ARMREST	133
RECEIVERS.....	135
SINGLE POST ARMREST RECEIVER - STD	135
FLIP BACK ARMREST RECEIVER (LITE STYLE ARMREST)	137
PADDED SWING AWAY ARMREST RECEIVER.....	139
ACCESSORIES	139
BACKPACKS AND SEAT POUCHES.....	139
BACKPACKS AND SEAT POUCHES.....	140
CASE LOGIC BAGS.....	141
CELL PHONE PACK	142
SOFT FEEL LEG STRAP	143
ADJUSTABLE LEG STRAP	144
SPOKE GUARDS.....	145
SIDE GUARDS.....	147
CRUTCH HOLDER	148
QUICKIE CADDY	149
AUTO BUCKLE POSITIONING BELTS	150
AIRCRAFT BUCKLE POSITIONING BELT.....	151
TOOL KIT	152
TIRE PUMP	153
IMPACT GUARD NEOPRENE & GEL VELCRO PADS.....	154
Back Cover.....	155

NOTE: ORDER PART 12 (TUBE CAP QTY 2) WHEN SWING-AWAY FOOTRESTS ARE NOT INSTALLED.

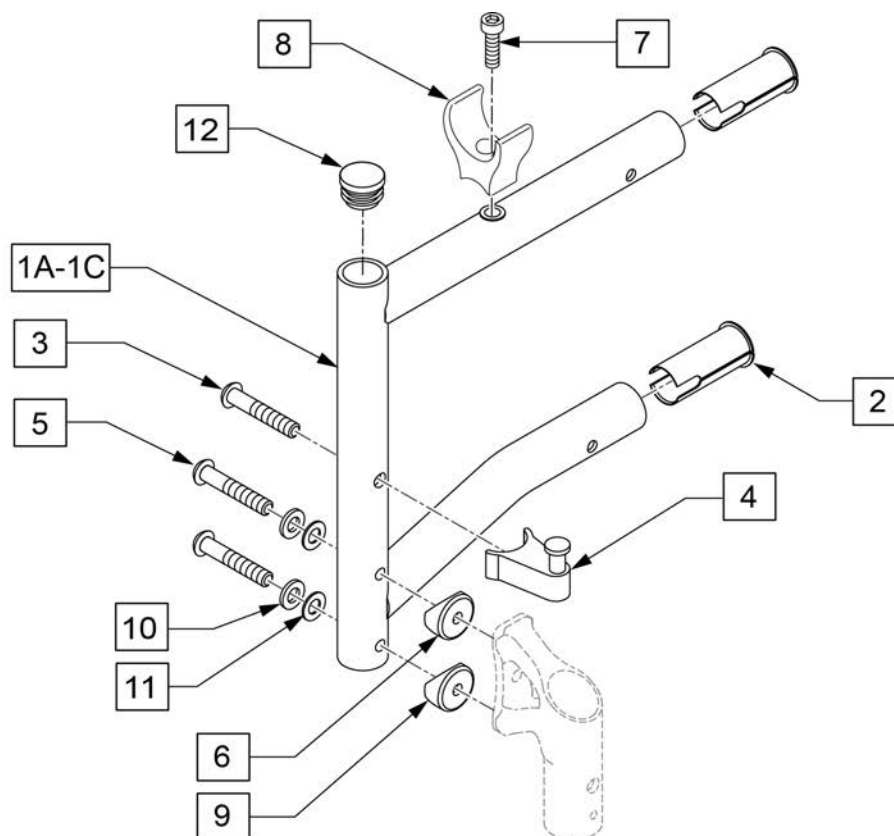


Pos.	Item Number	Description	Remarks	USD
2	107487	FRAME BUSHING KIT	Set of 4	30.00
3,4	107491	LATCH STUD KIT		35.00
5	108623	BHCS 1/4-28X1-1/2 SHLD PL ROHS		3.00
1A-4,7,8	107488S	S/A FRAME REG R	14"-16" Seat Depth,REG-Right	150.00
1A-4,7,8	107582S	S/A FRAME REG L	14"-16" Seat Depth,REG-Left	150.00
1B-4,7,8	107489S	S/A FRAME LONG R	17"-18" Seat Depth,LONG-Right	150.00
1B-4,7,8	107583S	S/A FRAME LONG L	17"-18" Seat Depth,LONG-Left	150.00
1C-4,7,8	107490S	S/A FRAME XLONG R	19"-20" Seat Depth,X-LONG-Right	150.00
1C-4,7,8	107584S	S/A FRAME XLONG L	19"-20" Seat Depth,X-LONG-Left	150.00
7-8	107486	SEAT & RAIL SADDLE KIT		45.00
5,6,9-11	107588	S/A FRM CASTR ATTACH KIT		50.00
12	560735	7/8" ROUND TUBE PLUG	Use in place of SA Hanger	2.00

NOTE: ORDER PART 14 (TUBE CAP QTY 2) WHEN SWING-AWAY FOOTRESTS ARE NOT INSTALLED.

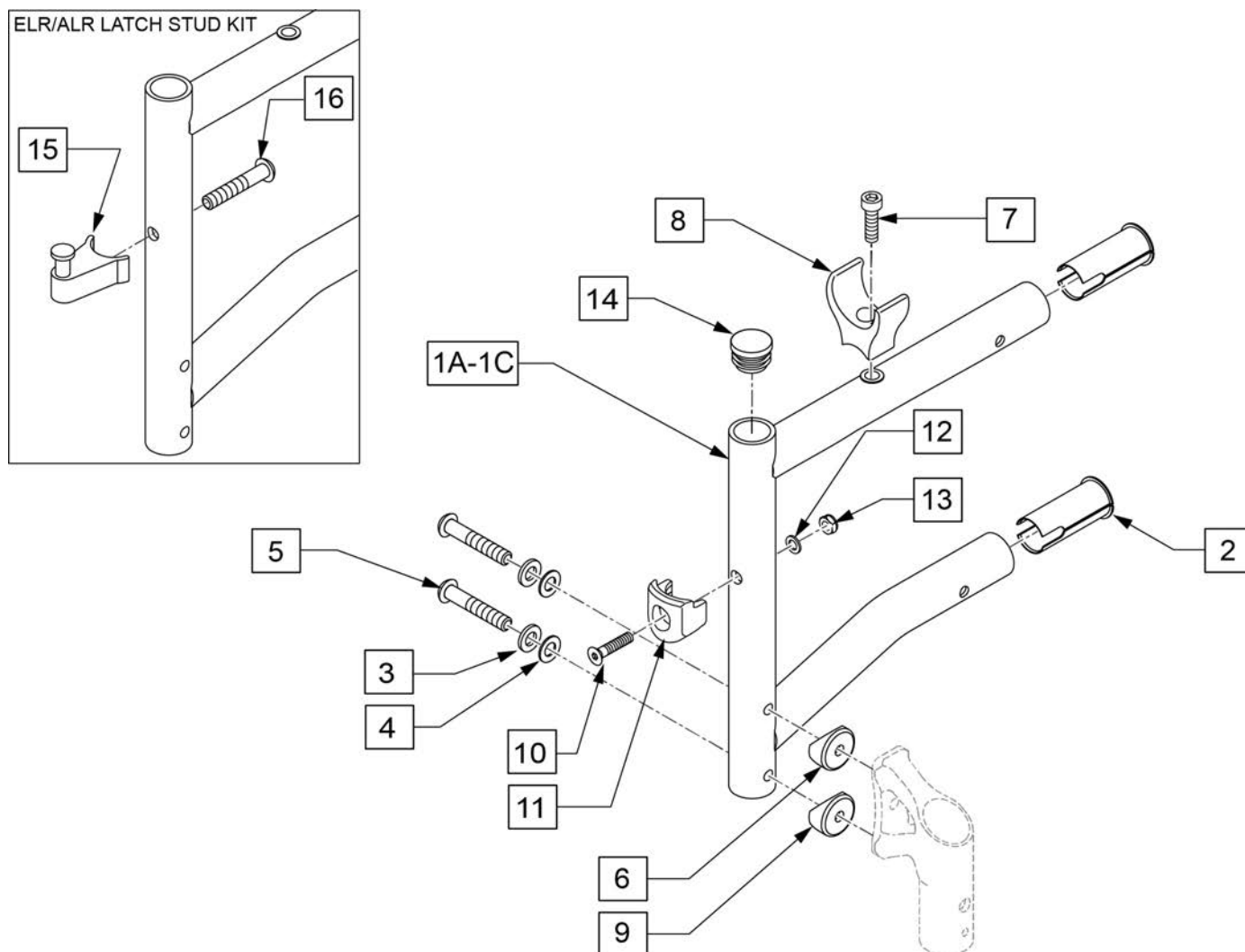


Pos.	Item Number	Description	Remarks	USD
2	107487	FRAME BUSHING KIT	Set of 4	30.00
5	108623	BHCS 1/4-28X1-1/2 SHLD PL ROHS		3.00
1A-8	129706	Q2 LITE S/A FRAME KIT REG	14"-16" Seat Depth,REG	
1B-8	129707	Q2 LITE S/A FRAME KIT LONG	17"-18" Seat Depth, LONG	
1C-8	129708	Q2 LITE S/A FRAME KIT XLONG	19"-20" Seat Depth,X-LONG	
3,4	107486	SEAT & RAIL SADDLE KIT		45.00
5-8	119514	LATCH RECIEVER KIT	For Swing in/out hangers	45.00
10-13	107588	S/A FRM CASTR ATTACH KIT		50.00
14	560735	7/8" ROUND TUBE PLUG	Use in place of SA Hanger	2.00
15,16	107491	LATCH STUD KIT	For ELR hangers	35.00

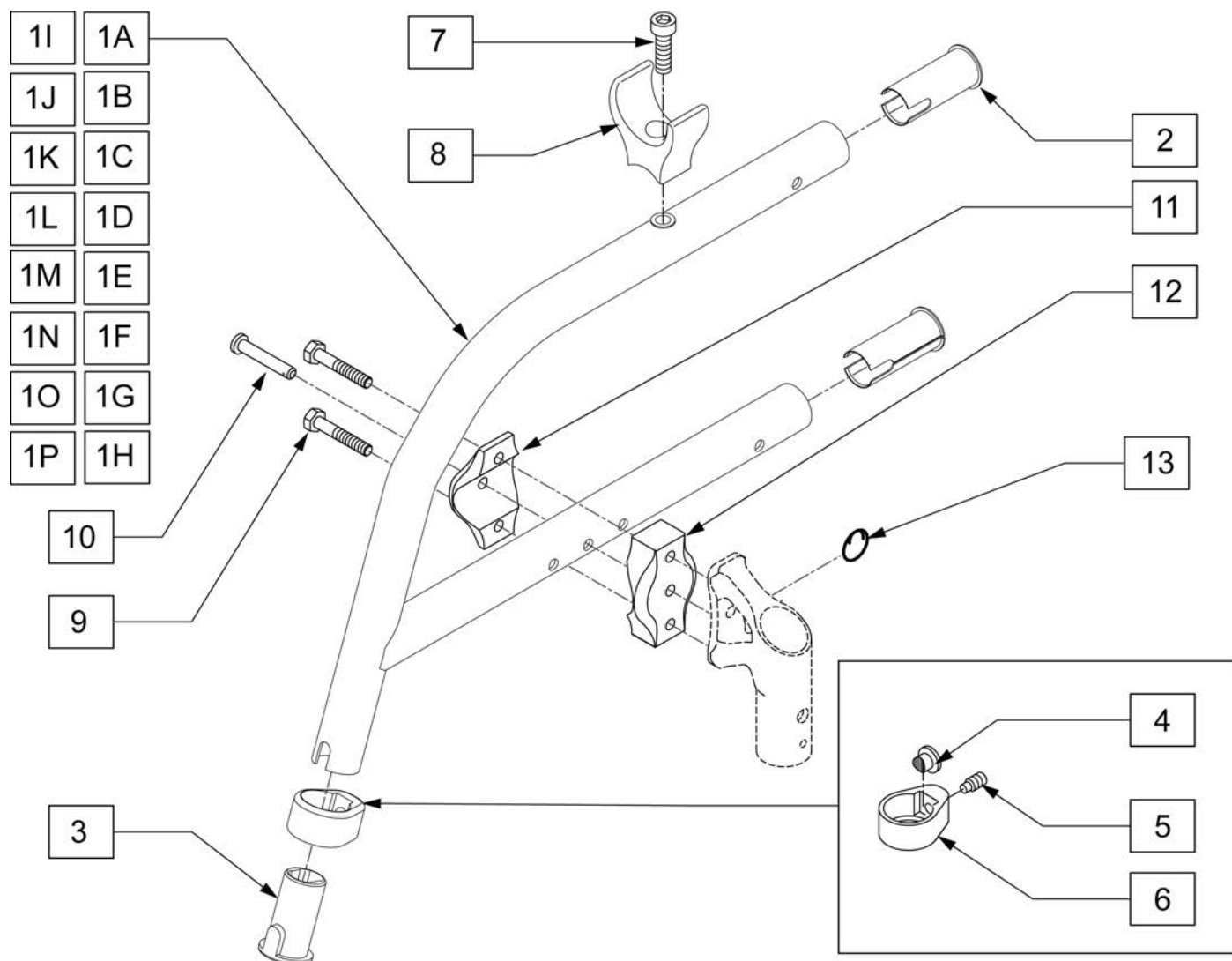


Pos.	Item Number	Description	Remarks	USD
2	107487	FRAME BUSHING KIT (4)	Set of 4	30.00
3	104160	BUSHING FOOTREST		5.00
13	349034	SAFETY COTTER		2.00
3-6	107485	FOOTREST CLAMP ASSM		30.00
1A-8	108250S	FX FRM STRT 70 DEG SML	14" Seat Depth,Short	100.00
1B-8	107467S	FIX FRAME STRAIGHT 70 DEG REG	15"-16" Seat Depth,REG	100.00
1C-8	107468S	FIX FRAME STRAIGHT 70 DEG LONG	17"-18" Seat Depth,LONG	100.00
1D-8	107469S	FIX FRAME STRAIGHT 70 DEG X-LONG	19"-20" Seat Depth,X-LONG	100.00
1E-8	108251S	FX FRM STRT 80 DEG SML	14" Seat Depth,Short	100.00
1F-8	107470S	FIX FRAME STRAIGHT 80 DEG REG	15"-16" Seat Depth,REG	100.00
1G-8	107471S	FIX FRAME STRAIGHT 80 DEG LONG	17"-18" Seat Depth,LONG	100.00
1H-8	107472S	FIX FRAME STRAIGHT 80 DEG X-LONG	19"-20" Seat Depth,X-LONG	100.00
1I-8	108252S	FX FRM NSET 70 DEG SML R	14" Seat Depth,Short,Right	100.00
1I-8	108253S	FX FRM NSET 70 DEG SML L	14" Seat Depth,Short,Left	100.00
1J-8	107473S	FIX FRAME INSET 70 DEG REG R	15"-16" Seat Depth,REG,RIGHT	100.00
1J-8	107474S	FIX FRAME INSET 70 DEG REG L	15"-16" Seat Depth,REG,LEFT	100.00
1K-8	107475S	FIX FRAME INSET 70 DEG LONG R	17"-18" Seat Depth,LONG,RIGHT	100.00
1K-8	107476S	FIX FRAME INSET 70 DEG LONG L	17"-18" Seat Depth,LONG,LEFT	100.00

Pos.	Item Number	Description	Remarks	USD
1L-8	107477S	FIX FRAME INSET 70 DEG X-LONG R	19"-20" Seat Depth,X-LONG,RIGHT	100.00
1L-8	107478S	FIX FRAME INSET 70 DEG X-LONG L	19"-20" Seat Depth,X-LONG,LEFT	100.00
1M-8	108254S	FX FRM NSET 80 DEG SML R	14" Seat Depth,Short,Right	100.00
1M-8	108255S	FX FRM NSET 80 DEG SML L	14" Seat Depth,Short,Left	100.00
1N-9	107479S	FIX FRAME INSET 80 DEG REG R	15"-16" Seat Depth,REG,RIGHT	100.00
1N-9	107480S	FIX FRAME INSET 80 DEG REG L	15"-16" Seat Depth,REG,LEFT	100.00
1O-8	107481S	FIX FRAME INSET 80 DEG LONG R	17"-18" Seat Depth,LONG,RIGHT	100.00
1O-8	107482S	FIX FRAME INSET 80 DEG LONG L	17"-18" Seat Depth,LONG,LEFT	100.00
1P-8	107483S	FIX FRAME INSET 80 DEG X-LONG R	19"-20" Seat Depth,X-LONG,RIGHT	100.00
1P-8	107484S	FIX FRAME INSET 80 DEG X-LONG L	19"-20" Seat Depth,X-LONG,LEFT	100.00
7-8	107486	SEAT RAIL SADDLE KIT		45.00
9-13	107589	FIX FRAME CASTER ATTACH KIT		30.00

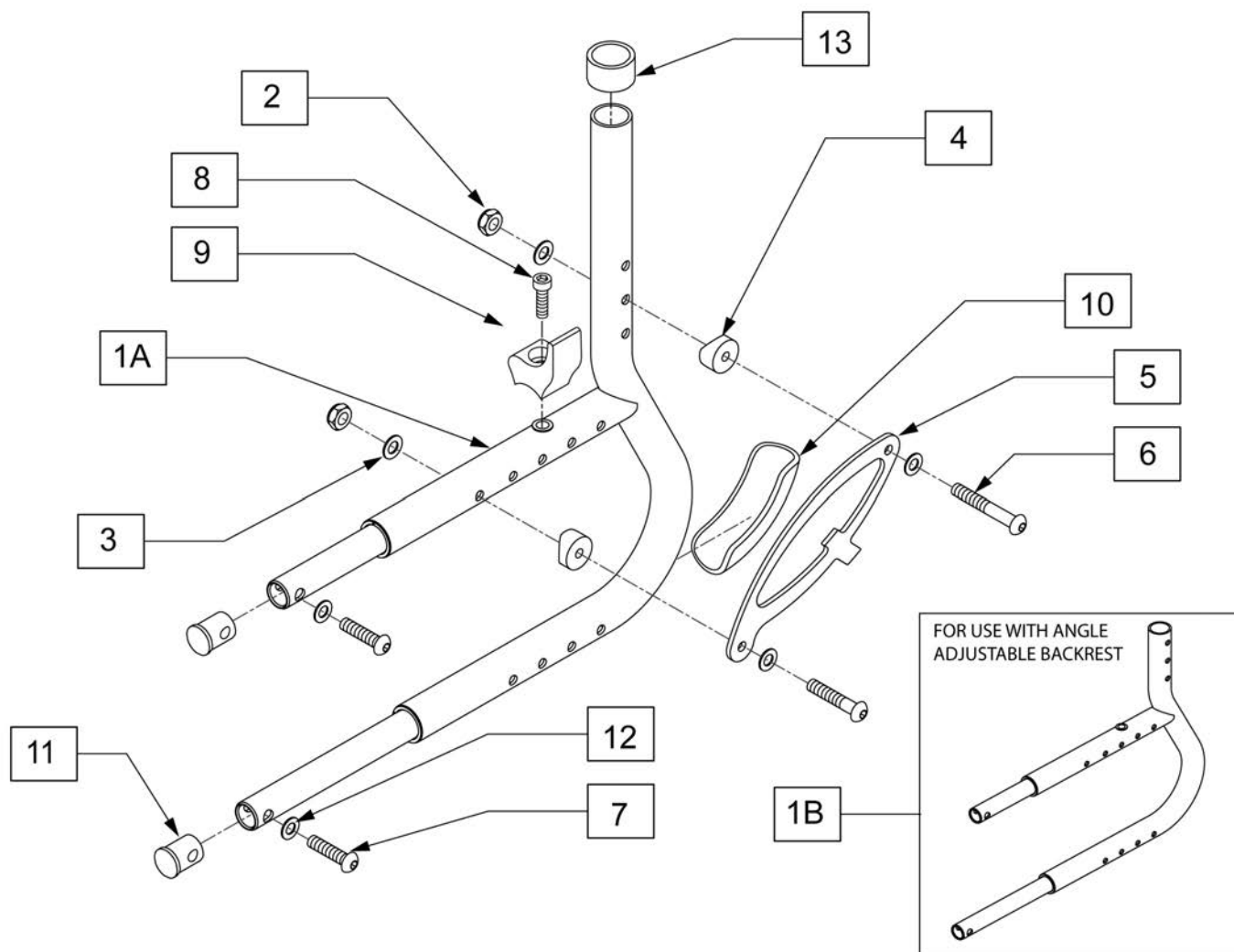

Pos.	Item Number	Description	Remarks	USD
2	106935	NUT NYLOCK THIN 1/4-28 ROHS		5.00
3	106944	WASHER FLAT THIN AN 1/4 ROHS		1.00
4	561087	TUBE SADDLE		3.59
5	104037	BACKREST GUSSET Q		10.00
6	106962	BHCS 1/4-28 X 1-3/4 ROHS		5.00
7	107353	1/4-20 X 1 BH C/S		5.00
8,9	107486	SEAT RAIL SADDLE KIT		45.00
10	104666	WEAR PAD FRAME ASSY		10.00
11	107494	RETAINING PLUG KIT (4)	Set of 4 plugs only	25.00
12	104414	WAVE WASHER 6 MM		5.00
13	561018	1" TUBE END BUSHING THICK		2.00
1A-13	107492S	REAR SDE FRM ASSM STD BCK R	STANDARD BACK-Right	125.00
1A-13	107585S	REAR SDE FRM ASSM STD BCK L	STANDARD BACK-Left	125.00
1B-13	107493S	REAR SDE FRM ASSM ANG BCK R	ANGLE ADJUSTABLE BACK-Right	125.00
1B-13	107586S	REAR SDE FRM ASSM ANG BCK L	ANGLE ADJUSTABLE BACK-Left	125.00

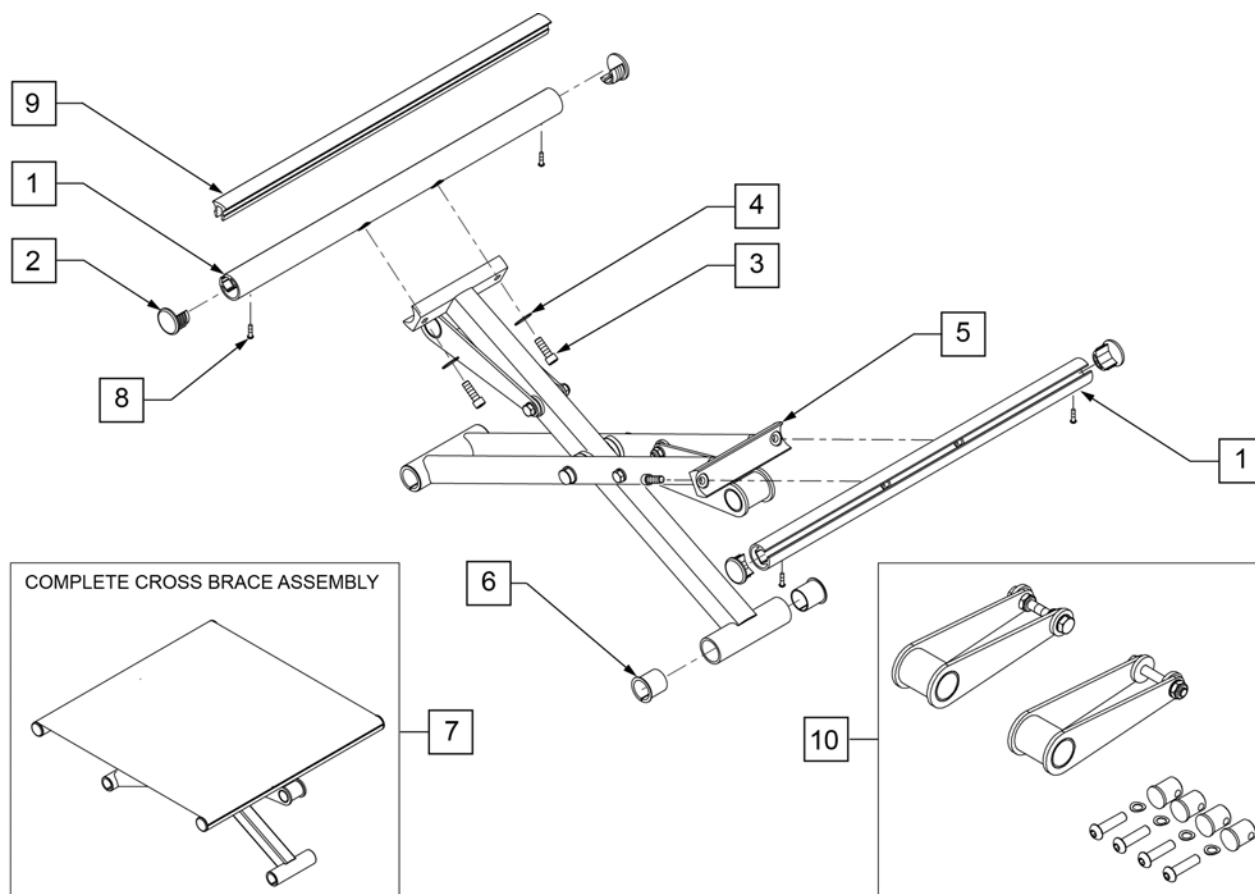


Pos.	Item Number	Description	Remarks	USD
3	106974	SHCS 1/4-28 X 3/4 PL ROHS		5.00
4	106944	WASHER FLAT THIN AN 1/4 ROHS		1.00
6	108116	3/4" NY LINER (4)	SET OF 4	50.00
7	107502	X BRACE ASSM 14X14		475.00
7	107503	X BRACE ASSM 14X15		475.00
7	107504	X BRACE ASSM 14X16		475.00
7	107505	X BRACE ASSM 14X17		475.00
7	107506	X BRACE ASSM 14X18		475.00
7	107507	X BRACE ASSM 14X19		475.00
7	107508	X BRACE ASSM 14X20		475.00
7	107509	X BRACE ASSM 15X14		475.00
7	107510	X BRACE ASSM 15X15		425.00
7	107511	X BRACE ASSM 15X16		475.00
7	107512	X BRACE ASSM 15X17		475.00
7	107514	X BRACE ASSM 15X18		475.00
7	107515	X BRACE ASSM 15X19		475.00
7	107516	X BRACE ASSM 15X20		475.00
7	107517	X BRACE ASSM 16X14		475.00
7	107518	X BRACE ASSM 16X15		475.00
7	107519	X BRACE ASSM 16X16		475.00
7	107520	X BRACE ASSM 16X17		475.00
7	107521	X BRACE ASSM 16X18		475.00
7	107522	X BRACE ASSM 16X19		475.00
7	107523	X BRACE ASSM 16X20		475.00

Pos.	Item Number	Description	Remarks	USD
7	107524	X BRACE ASSM 17X14		475.00
7	107525	X BRACE ASSM 17X15		475.00
7	107526	X BRACE ASSM 17X16		425.00
7	107527	X BRACE ASSM 17X17		475.00
7	107528	X BRACE ASSM 17X18		475.00
7	107529	X BRACE ASSM 17X19		475.00
7	107530	X BRACE ASSM 17X20		475.00
7	107531	X BRACE ASSM 18X14		475.00
7	107532	X BRACE ASSM 18X15		475.00
7	107533	X BRACE ASSM 18X16		475.00
7	107534	X BRACE ASSM 18X17		475.00
7	107535	X BRACE ASSM 18X18		475.00
7	107536	X BRACE ASSM 18X19		475.00
7	107537	X BRACE ASSM 18X20		475.00
7	107538	X BRACE ASSM 19X14		475.00
7	107539	X BRACE ASSM 19X15		475.00
7	107540	X BRACE ASSM 19X16		475.00
7	107541	X BRACE ASSM 19X17		475.00
7	107542	X BRACE ASSM 19X18		475.00
7	107543	X BRACE ASSM 19X19		475.00
7	107544	X BRACE ASSM 19X20		475.00
7	107545	X BRACE ASSM 20X14		475.00
7	107546	X BRACE ASSM 20X15		475.00
7	107547	X BRACE ASSM 20X16		475.00
7	107548	X BRACE ASSM 20X17		475.00
7	107549	X BRACE ASSM 20X18		475.00
7	107550	X BRACE ASSM 20X19		475.00
7	107551	X BRACE ASSM 20X20		475.00
8	109819	SCREW #6 x 1/4 SLF TP PHL PN HD		2.00
9	580024	SEAT RAIL INSERT STRIP 14"		5.10
9	580025	SEAT RAIL INSERT STRIP 15"		5.10
9	580026	SEAT RAIL INSERT STRIP 16"		10.00
9	580027	SEAT RAIL INSERT STRIP 17"		10.00
9	580028	SEAT RAIL INSERT STRIP 18"		10.00
9	580029	SEAT RAIL INSERT STRIP 19"		10.00
9	580030	SEAT RAIL INSERT STRIP 20"		10.00
10	115899	Q2 LT X-HINGE REPLACEMENT KIT 14"		65.00
10	115900	Q2 LT X-HINGE REPLACEMENT KIT 15"		65.00
10	115901	Q2 LT X-HINGE REPLACEMENT KIT 16"		65.00
10	115902	Q2 LT X-HINGE REPLACEMENT KIT 17"		65.00
10	115903	Q2 LT X-HINGE REPLACEMENT KIT 18"		65.00
10	115904	Q2 LT X-HINGE REPLACEMENT KIT 19"		65.00
10	115905	Q2 LT X-HINGE REPLACEMENT KIT 20"		65.00
2,8	108117	END PLUG KIT (4)	SET OF 4	30.00
5,6	107615	XTUBE W/ SERIAL LBL 14W	14W	375.00
5,6	107616	XTUBE W/ SERIAL LBL 15W	15W	375.00
5,6	107617	XTUBE W/ SERIAL LBL 16W	16W	375.00
5,6	107618	XTUBE W/ SERIAL LBL 17W	17W	375.00
5,6	107619	XTUBE W/ SERIAL LBL 18W	18W	375.00
5,6	107620	XTUBE W/ SERIAL LBL 19W	19W	375.00
5,6	107621	XTUBE W/ SERIAL LBL 20W	20W	375.00
1-4,8	107495	SEAT RAIL KIT 14D	14 DEEP-Each	45.00
1-4,8	107496	SEAT RAIL KIT 15D	15 DEEP-Each	45.00
1-4,8	107498	SEAT RAIL KIT 17D	17 DEEP-Each	45.00
1-4,8	107497	SEAT RAIL KIT 16D	16 DEEP-Each	45.00
1-4,8	107499	SEAT RAIL KIT 18D	18 DEEP-Each	45.00


Pos.	Item Number	Description	Remarks	USD
1-4,8	107500	SEAT RAIL KIT 19D	19 DEEP-Each	45.00
1-4,8	107501	SEAT RAIL KIT 20D	20 DEEP-Each	45.00

NOTE: In order to properly mount a JAY DUO CUSHION; Qty (2) of the appropriate size SEAT RAIL INSERT (Item 9) must be ordered.






NOTE: Kit Includes all items for adding the Transit Option to an existing chair.

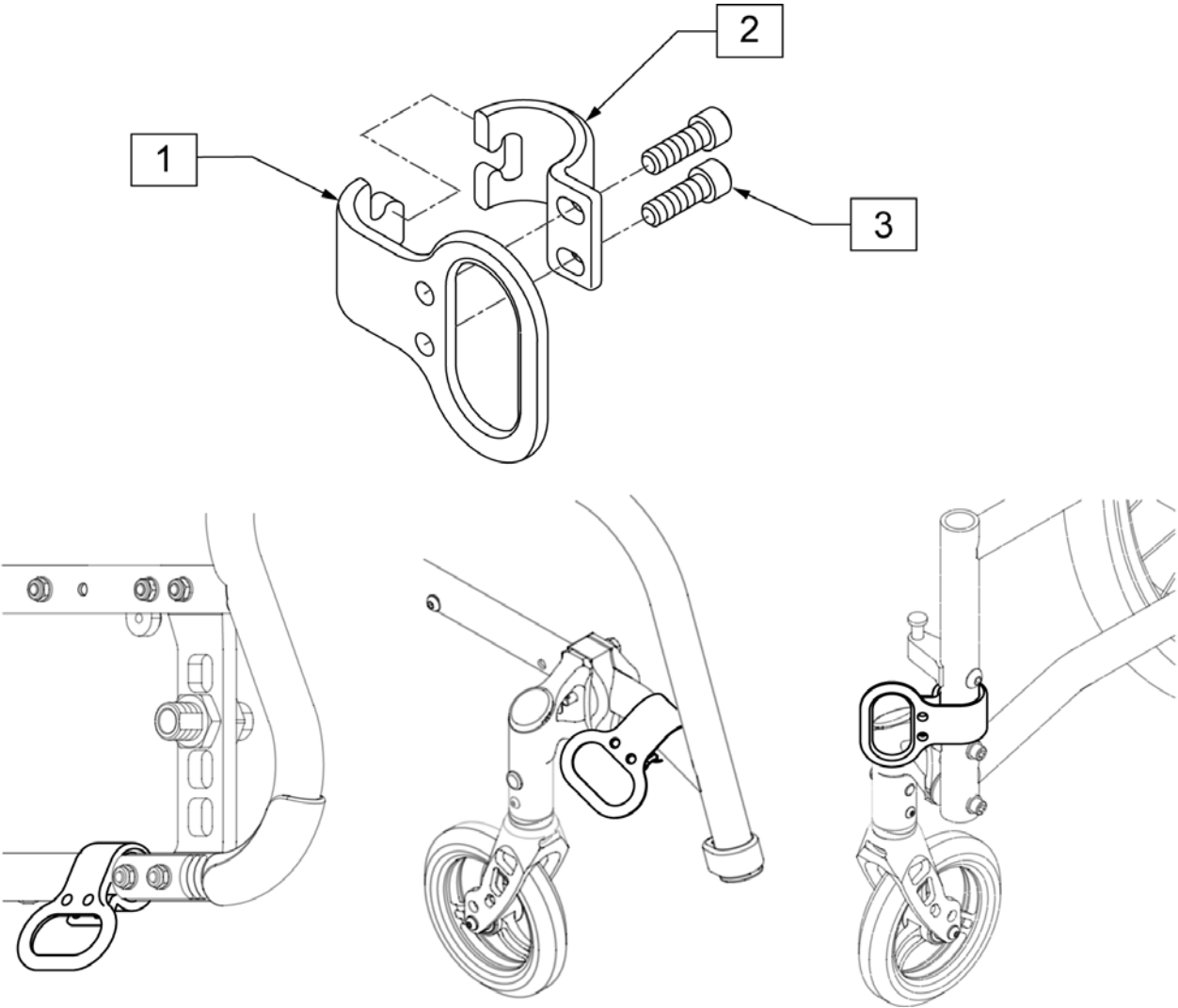


Pos.	Item Number	Description	USD
3	107350	SHCS 1/4-28 X 1/2 PL ROHS	
1-3	109277	TRANSIT KIT Q2 LITE	350.00

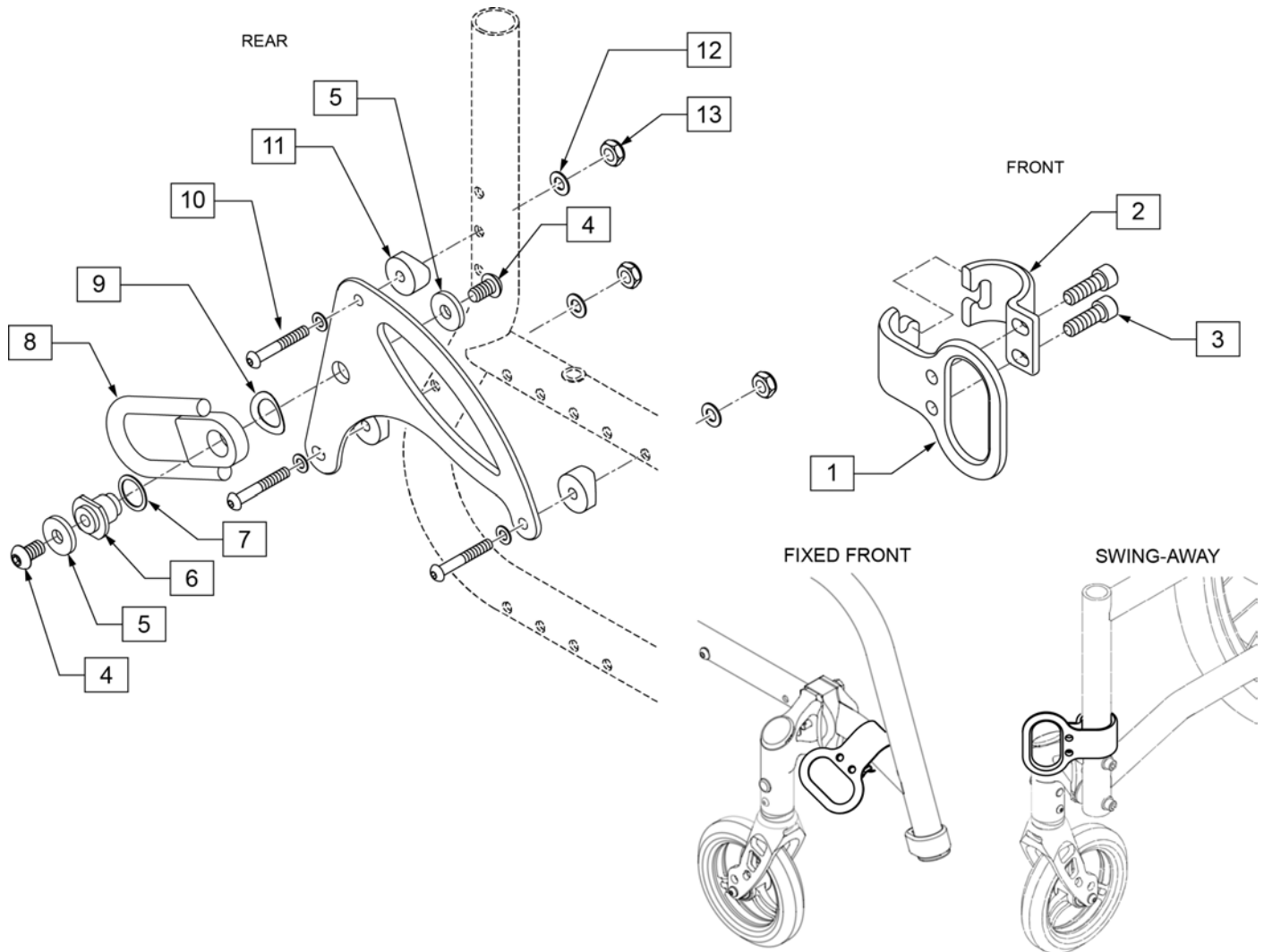


Pos.	Item Number	Description	USD
3	107350	SHCS 1/4-28 X 1/2 PL ROHS	
4	230120	BHCS M8 X 1.25 X 14	2.00
5	263113	5/16 WASHER THICK	2.00
10	106962	BHCS 1/4-28 X 1-3/4	5.00
11	561087	BACK RELEASE SADDLE-RIGID	3.59
12	106943	WASHER FLAT AN 1/4	1.00
13	106934	NUT NYLOCK 1/4-28	2.00
1-12	129709	TRANSIT ASSY LITE WC19	

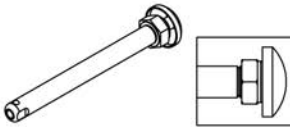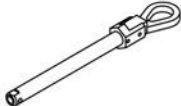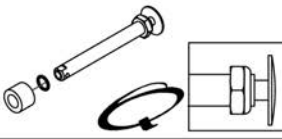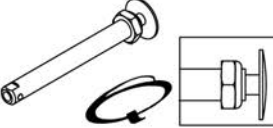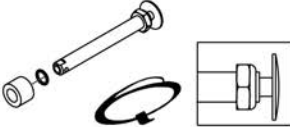Wheel Type	Axle Type		Axle P/N	Spacer P/N
Mag Wheels	Quick Release	1A 	102063 A=4 1/8	N/A
	Quad Quick Release	1B 	920171 A=4 1/8	N/A
Spinergy Wheels & Mountain Wheels	Quick Release	1C 	109385 A=4 1/2	N/A
Lite Spoke Wheels	Quick Release	1D 	107268 A=4 1/4	N/A
Ultra Lite Wheels	Quick Release	1E 	100793 Order Spacer A = 4 1/2"	2C 103093 2D 100768-001

Quick Release Measuring Guide

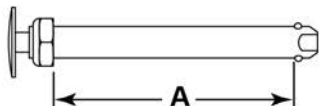

Pos.	Item Number	Description	USD
1A	102063	QR AXLE MAG W/M-12 NUT	50.00
1B	920171	QR AXLE MAG W/QUAD NUT	88.26
1C	102064	QR AXLE M12 NUT CLR BIG BTTN	50.00
1D	107268	QR AXLE AND NUT SHORT BIG BTTN	50.00
1E	100793	AXLE QR SHORT ULTRALITE	38.25
2C	103093	O-RING 1/2" ID x 11/16" OD BUNA-N	2.50
2D	100768-001	SPACER AXLE 22 OD x 12.7 ID x 7mm	7.25

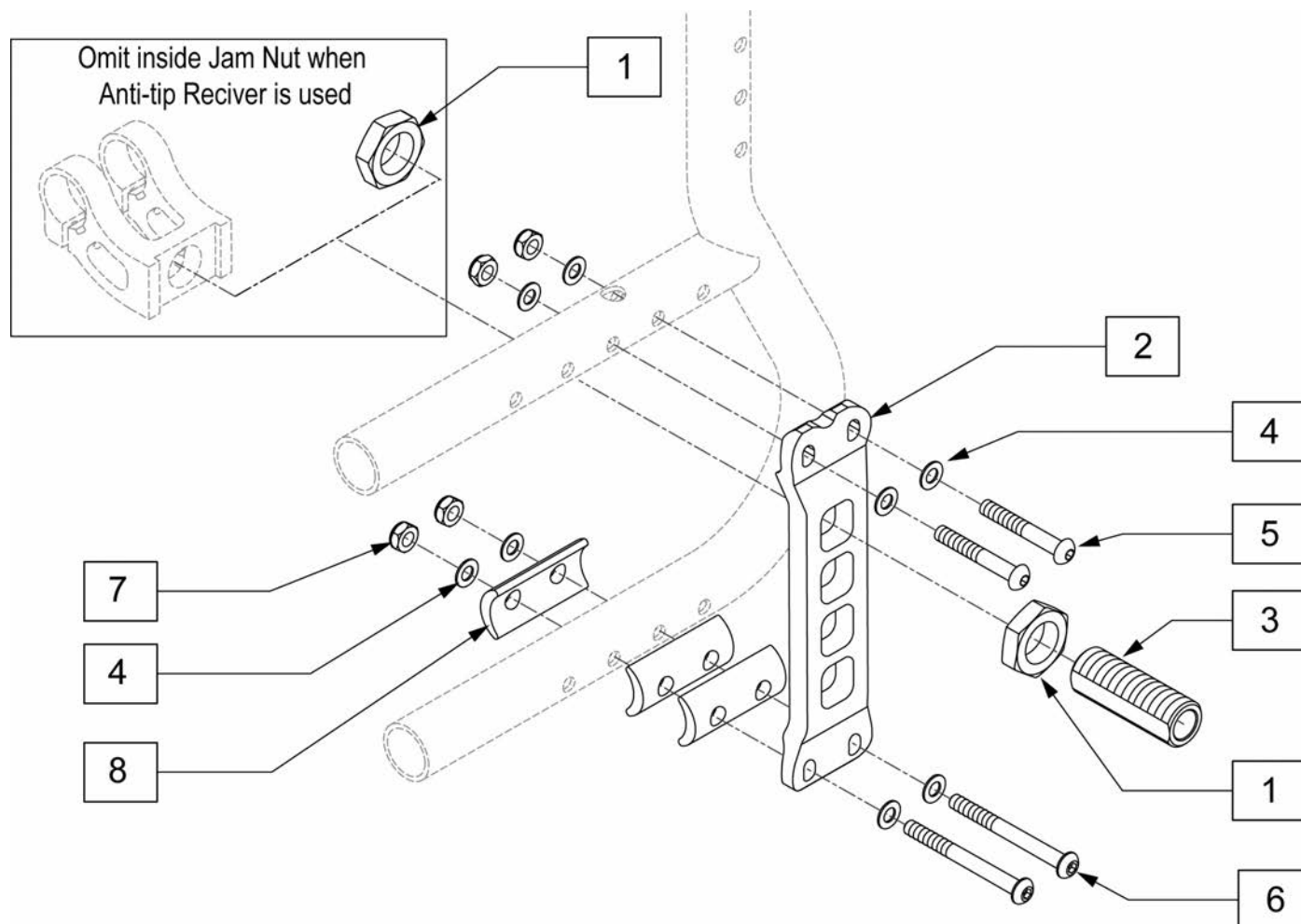


Pos.	Item Number	Description	Remarks	USD
1	104195	NUT JAM ALUMINUM 20MM X 1.5		5.00
2	104265	AXLE PLATE LEFT	Left	30.00
2	104266	AXLE PLATE RIGHT	Right	30.00
3	104196	AXLE SLEEVE 20MM NONADJ		10.00
4	106943	FLAT AN 1/4 ROHS		1.00
5	107252	BHCS 1/4-28 X 1-5/8 ROHS		5.00
6	106966	BHCS 1/4-28 X 2-1/4 SHLDR ROHS		5.00
7	106935	NYLOCK NUT THIN 1/4-28 ROHS		5.00
8	107554	CAMBER SPACER KIT	(Set of 6) 1 Spacer provides 2 deg of camber.	50.00
1-8	107552	AXLE PLATE KIT LEFT	Left	60.00
1-8	107553	AXLE PLATE KIT RIGHT	Right	60.00

Note: Item 26 is used for additional barrel adjustment. Washers can be added or removed in order to ensure caster barrel is perpendicular to the ground.

Note: As of S/N Q2L-149887 Stem assembly has changed. If replacing stem components on chairs prior to S/N Q2L-149887, replace complete Stem Assembly in pairs.

Part Changes, please see notes/remarks for important information

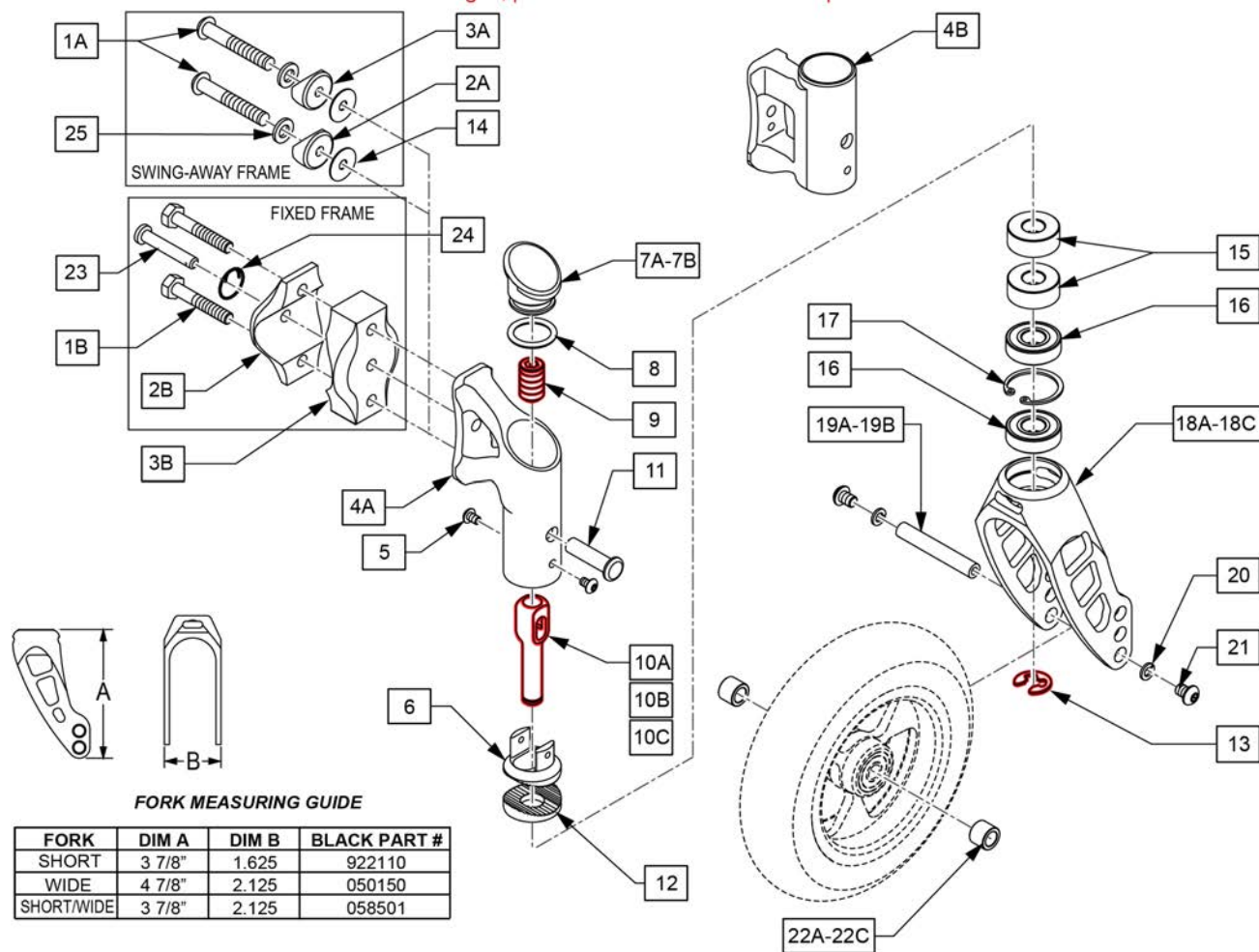

Pos.	Item Number	Description	Remarks	USD
1A	108623	BHCS 1/4-28X1-1/2 SHLD PL ROHS		3.00
1B	106972	HHCS 1/4-28 X 1-1/2 ROHS		5.00
7A,8	104096	CASTER CAP WITH O RING	Standard Caster Cap	5.00
7B,8	104120	CASTER CAP HEMI WITH O RING	Hemi Caster Cap	5.00
9	106684	SH SET SCREW 1/2-20X3/4 FLT PT		
10A	122989	CASTER STEM ADJUSTABLE SHORT		
10B	122993	CASTER STEM ADJUSTABLE MED		

Pos.	Item Number	Description	Remarks	USD
10C	122994	CASTER STEM ADJUSTABLE TALL		
12	104182	SPLINE CSTR ANGLE ADJ LOWER		10.00
13	114479	E-STYLE BOWED CLIP		5.00
14	113268	WASHER NYLON 1" OD x .312 ID		3.00
				8.95
15	483275	SPACER, CSTR, ANGLER ADJ .50	Not R'qd with Std stem bolt, Qty 1 R'qd with .5" stem bolt, Qty 2 R'qd with 1" Stem bolt	
16	349104	BRNG 28mm OD 12mm ID 8mm THK		15.00
				9.00
19A	050042	AXLE CASTER 52MM	2" - Used w/ wide fork & short wide fork	
				8.31
19B	481483	AXLE CASTER	1 1/2" - Used w/ short fork	
20	262132	WASHER LOCK M5		2.00
21	102259-001	BHCS M5 X .8 X 12 W/PL INORG		2.00
				2.14
22A	481353	SPACER, BEARING	Used w/ short fork Only	
				3.00
22B	050043-001	SPCR BRNG CSTR 9.65MM	Used w/ wide fork & short wide fork - not used w/ Soft Roll Casters	
				3.00
22C	050043-002	SPCR BRNG CSTR 2.20MM	Used w/ wide fork & short wide fork - Soft Roll Casters	
25	106944	WASHER FLAT THIN AN 1/4 ROHS		1.00
16-18A	922111	ASSM INTGRL BRNG FORK SHRT BLK	Short fork	90.00
16-18B	050150	ASSM FORK INT BRG WIDE BLK	Wide fork	87.10

Pos.	Item Number	Description	Remarks	USD
16-18C	108771	FORK SHORT WIDE ASSM	Short wide fork	80.00
4A,5-13,16-18A,19B,20-22A	107555	CSTR ASM SHRT STD	Std caster housing,short fork,std stem	195.00
4A,5-13,15-18A,19B,20-22A	107556	CSTR ASM SHRT +.5	Std caster housing,short fork,+.5 stem	195.00
4A,5-13,15-18A,19B,20-22A	107557	CSTR ASM SHRT +1	Std caster housing,short fork,+1stem	195.00
4A,5-13,16-18C,19B,20-22A	107558	CSTR ASM WD S STD	Std caster housing,short wide fork,std stem	195.00
4A,5-13,15-18C,19A,20-22A	107559	CSTR ASM WD S +.5	Std caster housing,short wide fork,+.5 stem	195.00
4A,5-13,15-18C,19A,20-22A	107560	CSTR ASM WD S +1	Std caster housing,short wide fork,+1stem	195.00
4A,5-13,16-18B,19B,20-22B	107562	CSTR ASM WIDE STD	Std caster housing,wide fork,std stem	195.00
4A,5-13,15-18B,19B,20-22B	107563	CSTR ASM WIDE +.5	Std caster housing,wide fork,+.5 stem	195.00
4A,5-13,15-18B,19B,20-22B	107564	CSTR ASM WIDE +1	Std caster housing,wide fork,+1stem	195.00
4A,5-13,16-18B,19B,20-22C	107565	CSTR ASM WD SR STD	Std caster housing,wide fork for soft roll caster,std stem	195.00
4A,5-13,15-18B,19B,20-22C	107566	CSTR ASM WD SR +.5	Std caster housing,wide fork for soft roll caster,+.5 stem	195.00
4A,5-13,15-18B,19B,20-22C	107567	CSTR ASM WD SR +1	Std caster housing,wide fork for soft roll caster,+1stem	195.00
4B,5-13,16-18A,19B,20-22A	107568	HCSTR ASM SHRT STD	Hemi caster housing,short fork,std stem	195.00


Pos.	Item Number	Description	Remarks	USD
4B,5-13,15-18A,19B,20-22A	107569	HCSTR ASM SHRT +.5	Hemi caster housing,short fork,+.5 stem	195.00
4B,5-13,15-18A,19B,20-22A	107570	HCSTR ASM SHRT +1	Hemi caster housing,short fork,+1stem	195.00
4B,5-13,16-18C,19A,20-22A	107571	HCSTR ASM WD S STD	Hemi caster housing,short wide fork,std stem	195.00
4B,5-13,15-18C,19A,20-22A	107572	HCSTR ASM WD S +.5	Hemi caster housing,short wide fork,+.5 stem	195.00
4B,5-13,15-18C,19A,20-22A	107573	HCSTR ASM WD S +1	Hemi caster housing,short wide fork,+1stem	195.00
4B,5-13,16-18B,19B,20-22B	107574	HCSTR ASM WIDE STD	Hemi caster housing, wide fork,std stem	195.00
4B,5-13,15-18B,19B,20-22B	107575	HCSTR ASM WIDE +.5	Hemi caster housing, wide fork,+.5 stem	195.00
4B,5-13,15-18B,19B,20-22B	107576	HCSTR ASM WIDE +1	Hemi caster housing, wide fork,+1stem	195.00
4B,5-13,16-18B,19B,20-22C	107577	HCSTR ASM WD SR STD	Hemi caster housing, wide fork for soft roll caster,std stem	195.00
4B,5-13,15-18B,19B,20-22C	107578	HCSTR ASM WD SR +.5	Hemi caster housing, wide fork for soft roll caster,+.5 stem	195.00
4B,5-13,15-18B,19B,20-22C	107579	HCSTR ASM WD SR +1	Hemi caster housing, wide fork for soft roll caster,+1stem	195.00
1A,2A,3A,14,25	107588	S/A FRM ATTACH KIT		50.00
1B,2B,3B,23,24	107589	FIX FRM ATTACH KIT		30.00
4A-10A,11,12	126079	STD CASTR HOUSE ASSM STD STEM		75.00
4A-10B,11,12	126080	STD CASTR HOUSE ASSM +.5 STEM		75.00
4A-10C,11,12	126081	STD CASTR HOUSE ASSM +1 STEM		75.00
4B-10A,11,12	126082	HEMI CASTR HOUSE ASSM STD STEM		75.00
4B-10B,11,12	126083	HEMI CAST HOUSE ASSM +.5 STEM		75.00
4B-10C,11,12	126084	HEMI CASTR HOUSE ASSM +1 STEM		75.00

Note: Item 10 Bearing Spacer not Required with Standard stem Bolt, 1 Required with .5" Stem bolt, 2 Required with 1" Stem bolt.

Note: Item 16 is used for additional barrel adjustment. Washers can be added or removed in order to ensure caster barrel is perpendicular to the ground.

Note: Items 25 and 26 are used to aid in aligning the upper and lower fork assembly during Frog Leg assembly and elastomer replacement. See instruction sheet for details.


Note: Frog Legs will ship unassembled to facilitate replacement of the Caster Stem.

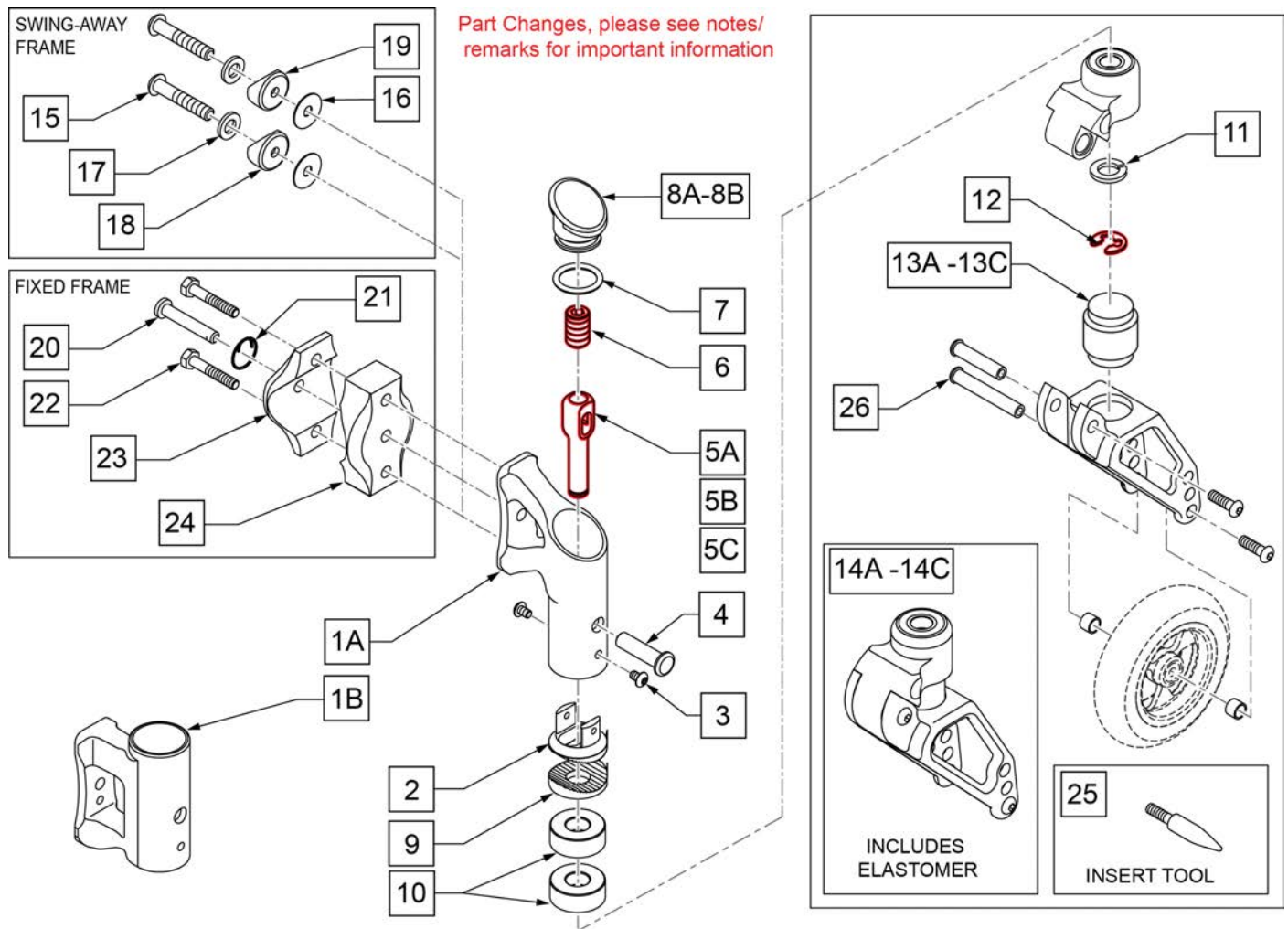


Pos.	Item Number	Description	Remarks	USD
5A	122989	CASTER STEM ADJUSTABLE SHORT		
5B	122993	CASTER STEM ADJUSTABLE MED		

Pos.	Item Number	Description	Remarks	USD
5C	122994	CASTER STEM ADJUSTABLE TALL		
6	106684	SH SET SCREW 1/2-20X3/4 FLT PT		
7,8A	104096	CASTER CAP WITH O RING		5.00
7,8B	104120	CASTER CAP HEMI WITH O RING		5.00
9	502051	SPLINE CSTR ANGLE ADJ LWR		15.00
10	483275	SPACER, CSTR, ANGLER ADJ .50	See Note	8.95
11	107359	WASHER LOCK M12		5.00
12	114479	E-STYLE BOWED CLIP		5.00
13A	050156-001	ELASTOMER FROGLEGS SOFT	0-100 lbs	6.45
13B	050156-002	ELASTOMER FROGLEGS MEDIUM	101-190 lbs	6.45
13C	050156-003	ELASTOMER FROGLEGS FIRM	191-265 lbs	6.45
14A	050225-001	FORK FROGLEGS INT BRNG SOFT	0-100 lbs	245.00
14B	050225-002	FORK FROGLEGS INT BRNG MED	101-190 lbs	245.00
14C	050225-003	FORK FROGLEGS INT BRNG FIRM	191-265 lbs	245.00
15	108623	BHCS 1/4-28X1-1/2 SHLD PL ROHS		3.00
16	113268	WASHER NYLON 1IN OD X .312 ID		3.00
17	106944	WASHER FLAT THIN AN 1/4 ROHS		1.00
18	561086	1" HALF SADDLE 1/4" HOLE		2.00
19	561004	1" SADDLE 1/4" HOLE		2.00
20	349027	1/4 X 2 CLEVIS PIN		2.27
21	349034	SAFETY COTTER		2.00
22	106972	HHCS 1/4-28 X 1-1/2 ROHS		5.00
23	500306	INNER CLAMP HALF		18.87
24	500307	OUTER CLAMP HALF		18.87
25	116351	FROG LEG FORK ASSEMBLY TOOL		40.00
15-19	107588	S/A FRM ATTACH KIT		50.00
20-24	107589	FIX FRM ATTACH KIT		30.00
1A-6	104092	CASTER HOUSING STD ASSM		65.00
1B-6	104107	CASTER HOUSING HEMI ASSM		65.00
1A,2-7,8A,9,11,14A	108062	FRG LG CSTR ASM SOFT STD	0-100 lbs, Std Caster Housing, Std Stem	375.00
1A,2-7,8A,9-11,14A	108063	FRG LG CSTR ASM SOFT +.5	0-100 lbs, Std Caster Housing, +.5"Stem	375.00
1A,2-7,8A,9-11,14A	108064	FRG LG CSTR ASM SOFT +1	0-100 lbs, Std Caster Housing, +1"Stem	375.00
1A,2-7,8A,9,11,14B	108072	FRG LG CSTR ASM MED STD	101-190 lbs, Std Caster Housing, Std Stem	375.00
1A,2-7,8A,9-11,14B	108073	FRG LG CSTR ASM MED +.5	101-190 lbs, Std Caster Housing, +.5"Stem	375.00
1A,2-7,8A,9-11,14B	108074	FRG LG CSTR ASM MED +1	101-190 lbs, Std Caster Housing, +1"Stem	375.00
1A,2-7,8A,9,11,14C	108075	FRG LG CSTR ASM FIRM STD	191-265 lbs, Std Caster Housing, Std Stem	375.00
1A,2-7,8A,9-11,14C	108076	FRG LG CSTR ASM FIRM +.5	191-265 lbs, Std Caster Housing, +.5"Stem	375.00
1A,2-7,8A,9-11,14C	108077	FRG LG CSTR ASM FIRM +1	191-265 lbs, Std Caster Housing, +1"Stem	375.00
1B,2-7,8A,9,11,14A	108078	FRG LG HCSTR ASM SOFT STD	0-100 lbs, Hemi Caster Housing, Std Stem	375.00
1B,2-7,8A,9-11,14A	108079	FRG LG HCSTR ASM SOFT +.5	0-100 lbs, Hemi Caster Housing, +.5"Stem	375.00
1B,2-7,8A,9-11,14A	108080	FRG LG HCSTR ASM SOFT +1	0-100 lbs, Hemi Caster Housing, +1"Stem	375.00
1B,2-7,8A,9,11,14B	108081	FRG LG HCSTR ASM MED STD	101-190 lbs, Hemi Caster Housing, Std Stem	375.00
1B,2-7,8A,9-11,14B	108082	FRG LG HCSTR ASM MED +.5	101-190 lbs, Hemi Caster Housing, +.5"Stem	375.00
1B,2-7,8A,9-11,14B	108083	FRG LG HCSTR ASM MED +1	101-190 lbs, Hemi Caster Housing, +1"Stem	375.00
1B,2-7,8A,9,11,14C	108084	FRG LG HCSTR ASM FIRM STD	191-265 lbs, Hemi Caster Housing, Std Stem	375.00
1B,2-7,8A,9-11,14C	108085	FRG LG HCSTR ASM FIRM +.5	191-265 lbs, Hemi Caster Housing, +.5"Stem	375.00
1B,2-7,8A,9-11,14C	108086	FRG LG HCSTR ASM FIRM +1	191-265 lbs, Hemi Caster Housing, +1"Stem	375.00

Note: Caster Pin Locks will not adapt to Short Wide Forks.

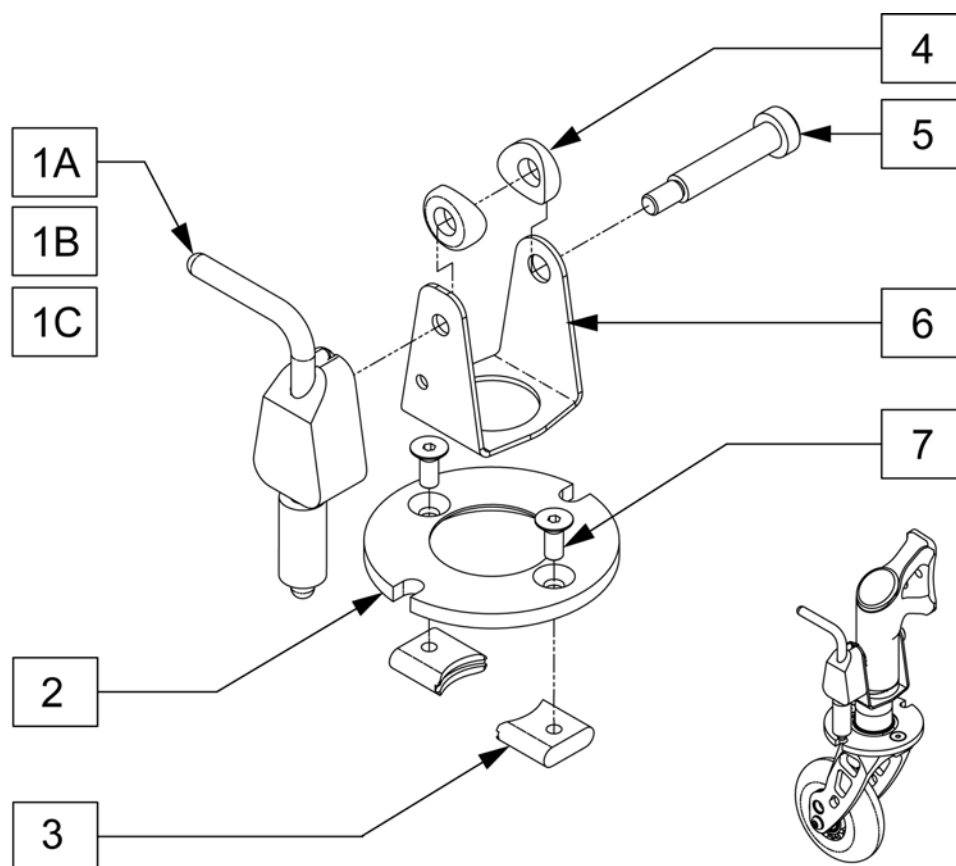

Pos.	Item Number	Description	Remarks	USD
1A	052310-001	BLOCK ASSY CSTR PIN LCK R STD	Right - Used w/ Std Length Caster Stem	140.95
1A	052311-001	BLOCK ASSY CSTR PIN LCK L STD	Left - Used w/ Std Length Caster Stem	140.95
1B	052310-002	BLOCK ASSY CSTR PIN LCK R+.5	Right - Used w/ +.5" Length Caster Stem	141.95
1B	052311-002	BLOCK ASSY CSTR PIN LCK L +.5	Left - Used w/ +.5" Length Caster Stem	141.95
1C	052310-003	BLOCK ASSY CSTR PIN LOCK R+1	Right - Used w/ +1" Length Caster Stem	143.95
1C	052311-003	BLOCK ASSY CSTR PIN LCK +1	Left - Used w/ +1" Length Caster Stem	143.95
2	050047	PLATE CASTER LOCK ANG ADJ		46.95
3	050050	KEY CASTER LOCK ANG ADJ		19.95
4	050053	SADDLE 17.8MM OD 8MM HOLE		2.95
5	050133	SHOULDER SCREW 8MMOD X 35MM LG		4.95
6	107204	YOKE CASTER PIN LOCK		30.00
7	102261	FHCS M5 X 12 INORGANIC		6.95
1A,2-7	107622	CSTR PIN LOCK R STD ASSM	Right - Used w/ Std Length Caster Stem	225.00
1B,2-7	107623	CSTR PIN LOCK R +.5 ASSM	Right - Used w/ +.5" Length Caster Stem	225.00
1C,2-7	107624	CSTR PIN LOCK R +1 ASSM	Right - Used w/ +1" Length Caster Stem	225.00
1A,2-7	107625	CSTR PIN LOCK L STD ASSM	Left - Used w/ Std Length Caster Stem	225.00
1B,2-7	107626	CSTR PIN LOCK L +.5 ASSM	Left - Used w/ +.5" Length Caster Stem	225.00
1C,2-7	107627	CSTR PIN LOCK L +1 ASSM	Left - Used w/ +1" Length Caster Stem	225.00

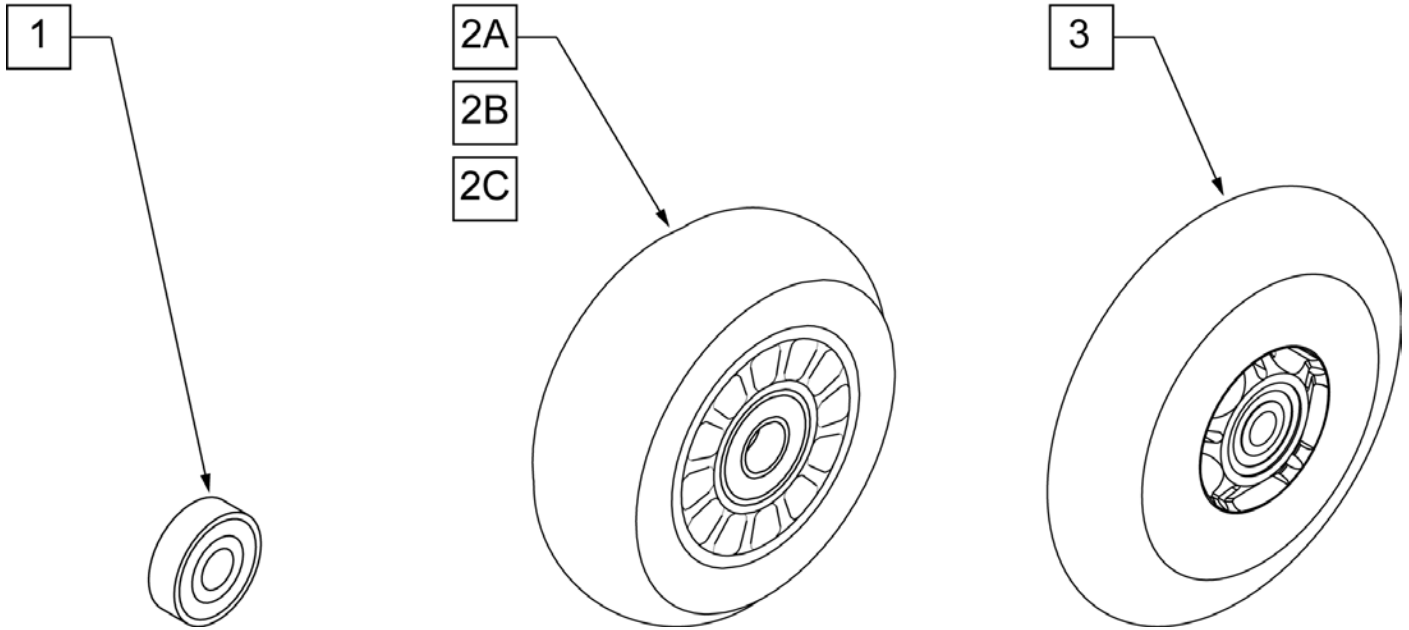


Pos.	Item Number	Description	HCPC	Remarks	USD
1	349502	7/8"x 5/16" BEARING			8.11
2A	400108	4" MICRO CASTER LIGHTED	K0077	4" MICRO LIGHTED	55.00
2B	920025	3" MICRO CASTER W/ BEARING -PAIR	K0077	PAIR 3" MICRO	55.00
2C	400107	3" MICRO CASTER LIGHTED	K0077	3" MICRO LIGHTED	55.00
3	052129	5" MICRO CASTER LIGHTED	K0077	5" MICRO LIGHTED	55.00


NOTE: 7" and 8" Casters are not available on the Q2 or the GT.

Note: If using 6" casters on Q7, Anti Flutter should be installed. See Standard Multi Position Fork page.


Note: Casters wider than 1" will not fit on Quickie Ti Titanium forks.

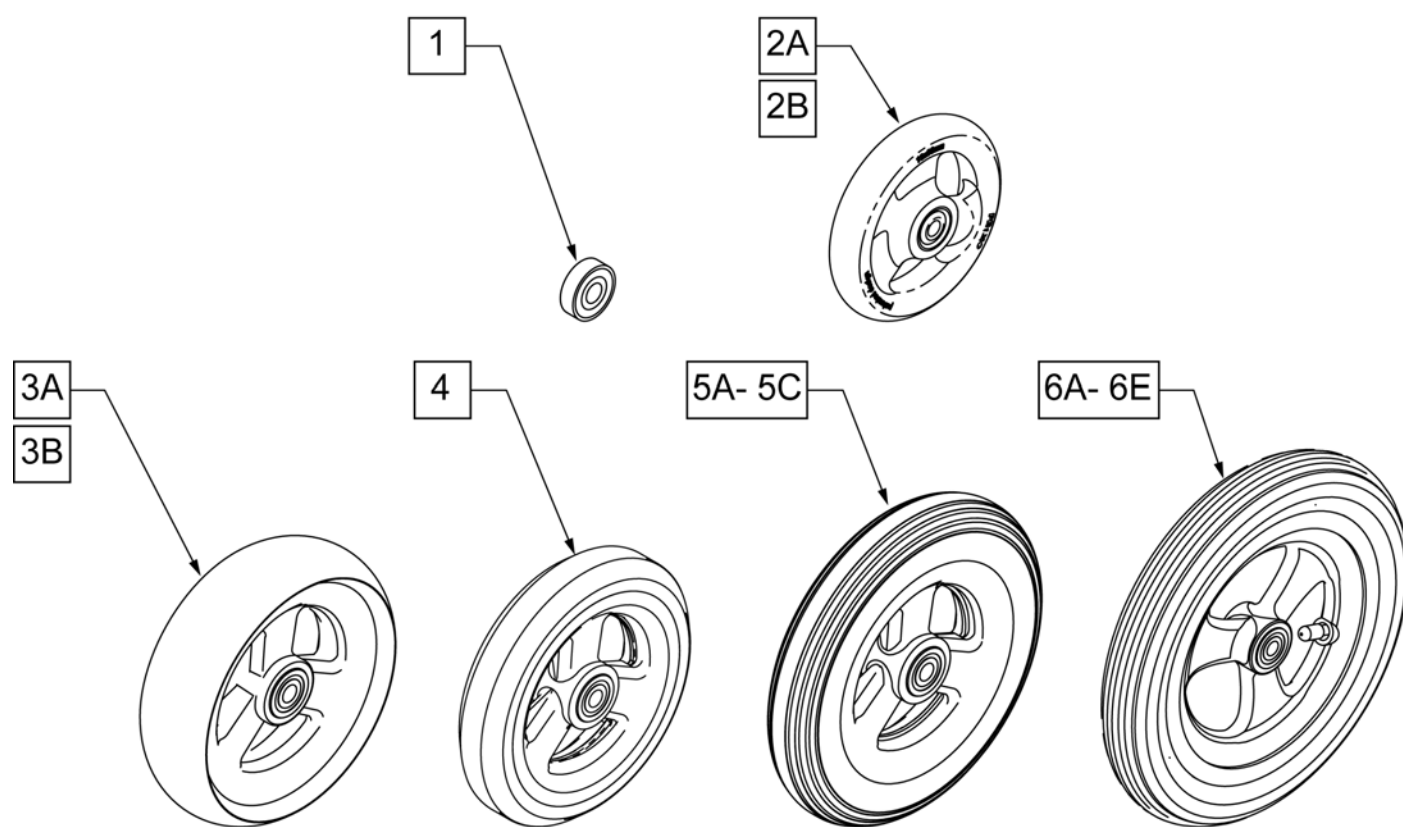

Pos.	Item Number	Description	Remarks	USD
1	349502	7/8"x 5/16" BEARING		8.11
2A	101957	4"X1-1/4 POLY 3 SPOKE CSTR	4" X 1.25 SEMI-PNEUMATIC	50.00
2B	101098	CASTER 4" PLASTIC/POLY W/BRG	4" LOW PROFILE POLY	45.00
3A	101259	5"X1-1/2" POLY 3 SPOKE CSTR	5" x 1.5" SEMI-PNEUMATIC	50.00
3B	101255	6"X1-1/2" POLY 3 SPOKE CSTR	6" x 1.5" SEMI-PNEUMATIC	55.00
4	101254	5" LOW POLY 3 SPOKE CASTER	5" LOW PROFILE POLY	52.00
5A	101256	6" SEMI PNEU 3 SPOKE CASTER	6" POLY	52.00
5B	101257	6" PNEUMATIC 3 SPOKE CASTER	6" PNEUMATIC	65.00
5C	383245	6" PNEUMATIC TIRE & TUBE		31.62
6A	400212	CASTER 8" PNEU 3 SPK ASSM	8" PNEUMATIC	65.00
6B	400213	CASTER 8" POLY 3 SPK ASSM	8" POLY	52.00
6C	400214	CASTER 8" LOW POLY 3 SPK ASSM	7" x LOW PROFILE POLY	55.00
6D	383255	8" PNEUMATIC TIRE & TUBE		24.28
6E	400215	CASTER 8" WIDE POLY 3 SPK ASSM	8" x 1.5" LOW POLY	58.00

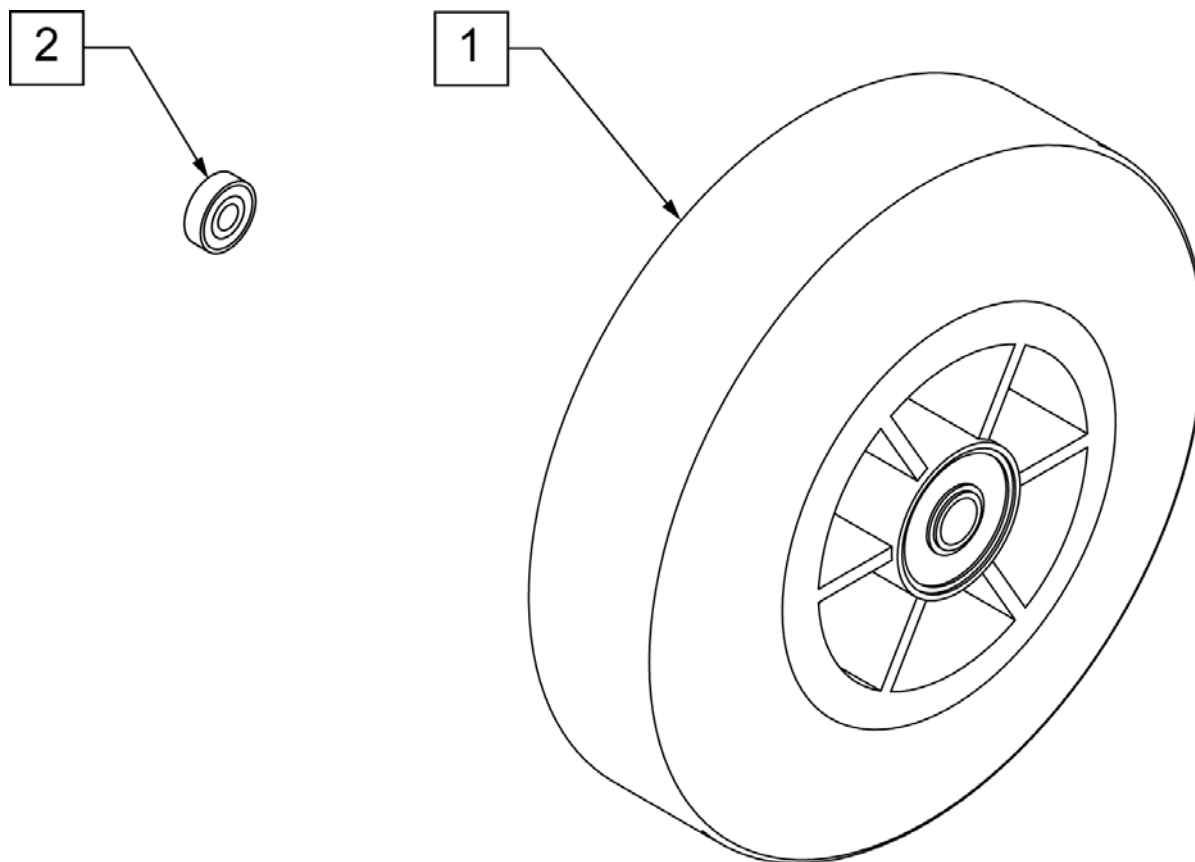


Pos.	Item Number	Description	HCPC	Remarks	USD
1	400103	CASTER 5" SOFT ROLL W/BRNGS	K0077	5" SOFT ROLL SEMI PNEU (GREY)	66.30
2	349502	7/8"x 5/16" BEARING			8.11

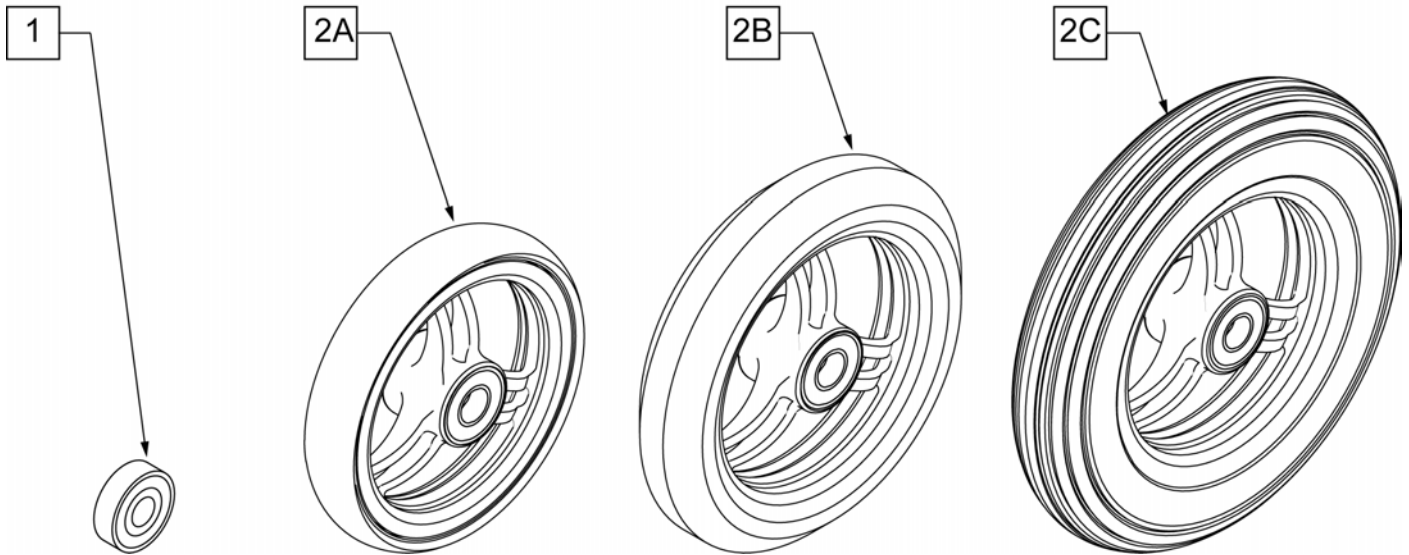

Pos.	Item Number	Description	HCPC	Remarks	USD
1	349502	7/8"x 5/16" BEARING			8.11
2A	101946-001	CASTR 4X1 ALUM/POLY W/BRG CLR	K0077	4" x 1" ALUMINUM POLY	85.00
2B	101946-002	CASTR 5X1 ALUM/POLY W/BRG CLR	K0077	5" x 1" ALUMINUM POLY	85.00
2C	101946-003	CASTR 6X1 ALUM/POLY W/BRG CLR	K0077	6" x 1" ALUMINUM POLY	85.00




SIDE VIEW

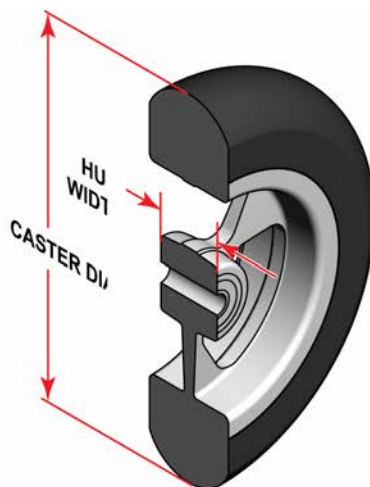

CASTER MEASURING GUIDE


TIRE DIAMETER	HUB WIDTH
3	1
4 1/8	1
5	1
6	1

Pos.	Item Number	Description	HCPC	Remarks	USD
>>>>		3 INCH DIAMETER CASTERS WITH SILVER HUBS			
1	113394	WHEEL CASTER 3" ULTRALITE SFTRLL	K0077	3" x 1.5" Aluminum Soft Roll - Black tire - Silver hub	65.00
>>>>		4 INCH DIAMETER CASTERS WITH SILVER HUBS			
1	113395	WHEEL CASTER 4" ULTRALITE SFTRLL	K0077	4" x 1.5" Aluminum Soft Roll - Black tire - Silver hub	65.00
>>>>		5 INCH DIAMETER CASTERS WITH SILVER HUBS			
1	113396	WHEEL CASTER 5" ULTRALITE SFTRLL	K0077	5" x 1.5" Aluminum Soft Roll - Black tire - Silver hub	65.00
>>>>		6 INCH DIAMETER CASTERS WITH SILVER HUBS			
1	113397	WHEEL CASTER 6" ULTRALITE SFTRLL	K0077	6" x 1.5" Aluminum Soft Roll - Black tire - Silver hub	65.00

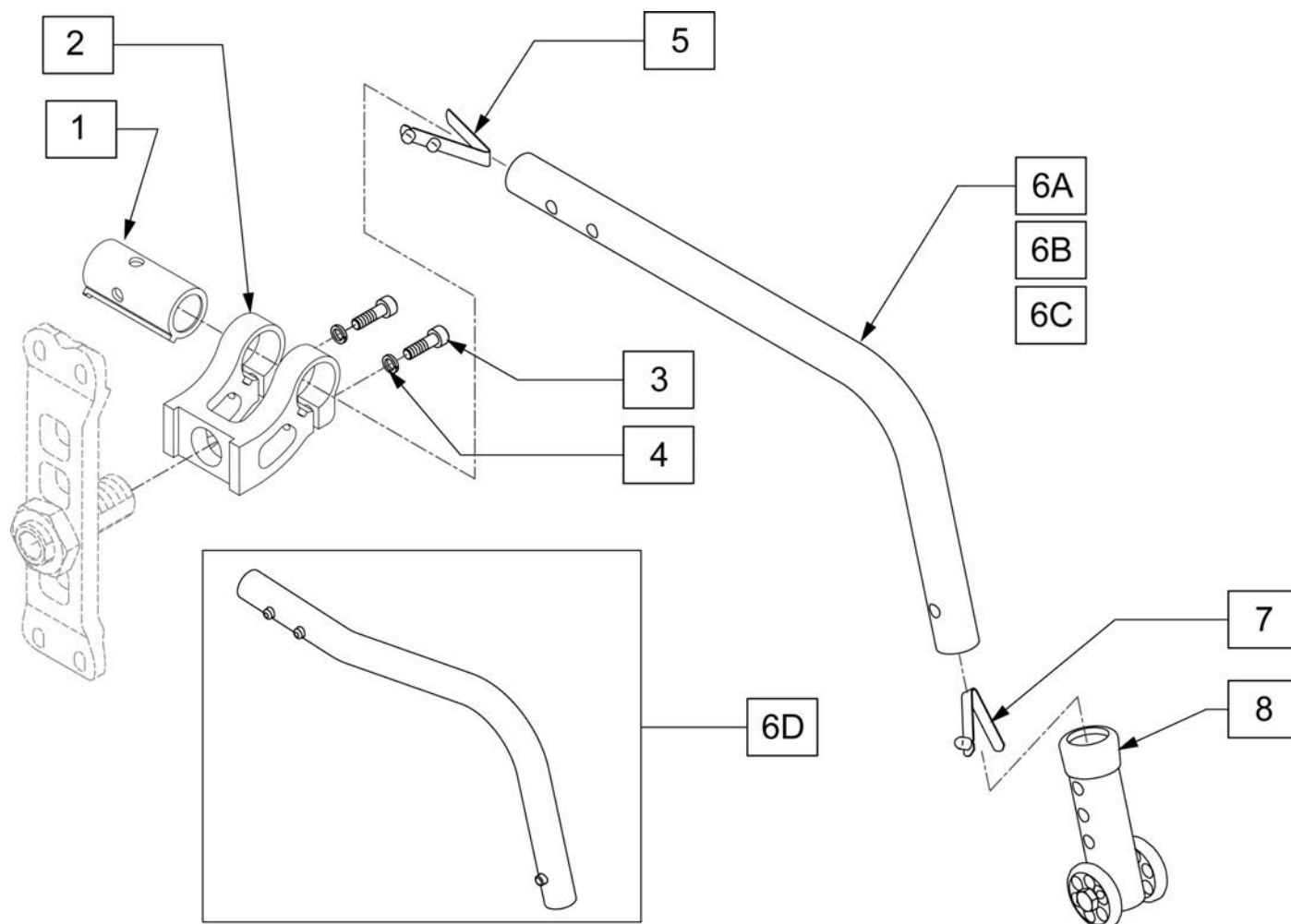


Pos.	Item Number	Description	Remarks	USD
1	104349-001	SLEEVE RECEIVER ANTI-TIP STD		10.00
2	104197	RECEIVER NON ADJ ANTI TIP		45.00
3	106974	SHCS 1/4-28 X 3/4 PL ROHS		5.00
4	107367	1/4 WASHER ROHS		5.00
8	104640	ANTI TIP ARM ADJ. LONG		45.00
1-4	107636	ANTI TIP RECEIVER ASSM		45.00
5,6A,7,8	107628	ANTI TIP ARM SML L ASM	Left - 22" wheels	65.00
5,6A,7,8	107629	ANTI TIP ARM SML R ASM	Right - 22" wheels	65.00
5,6B,7,8	107630	ANTI TIP ARM MED L ASM	Left - 24" wheels	65.00
5,6B,7,8	107631	ANTI TIP ARM MED R ASM	Right - 24" wheels	65.00
5,6C,7,8	107632	ANTI TIP ARM LRG L ASM	Left - 25" & 26" wheels	65.00
5,6C,7,8	107633	ANTI TIP ARM LRG R ASM	Right - 25" & 26" wheels	65.00
5,6D,7,8	107634	ANTI TIP ARM BND L ASM	Left - 20" wheels	65.00
5,6D,7,8	107635	ANTI TIP ARM BND R ASM	Right - 20" wheels	65.00

NOTE: ATTACHMENT CLAMP mounts to 1" diameter tubing.

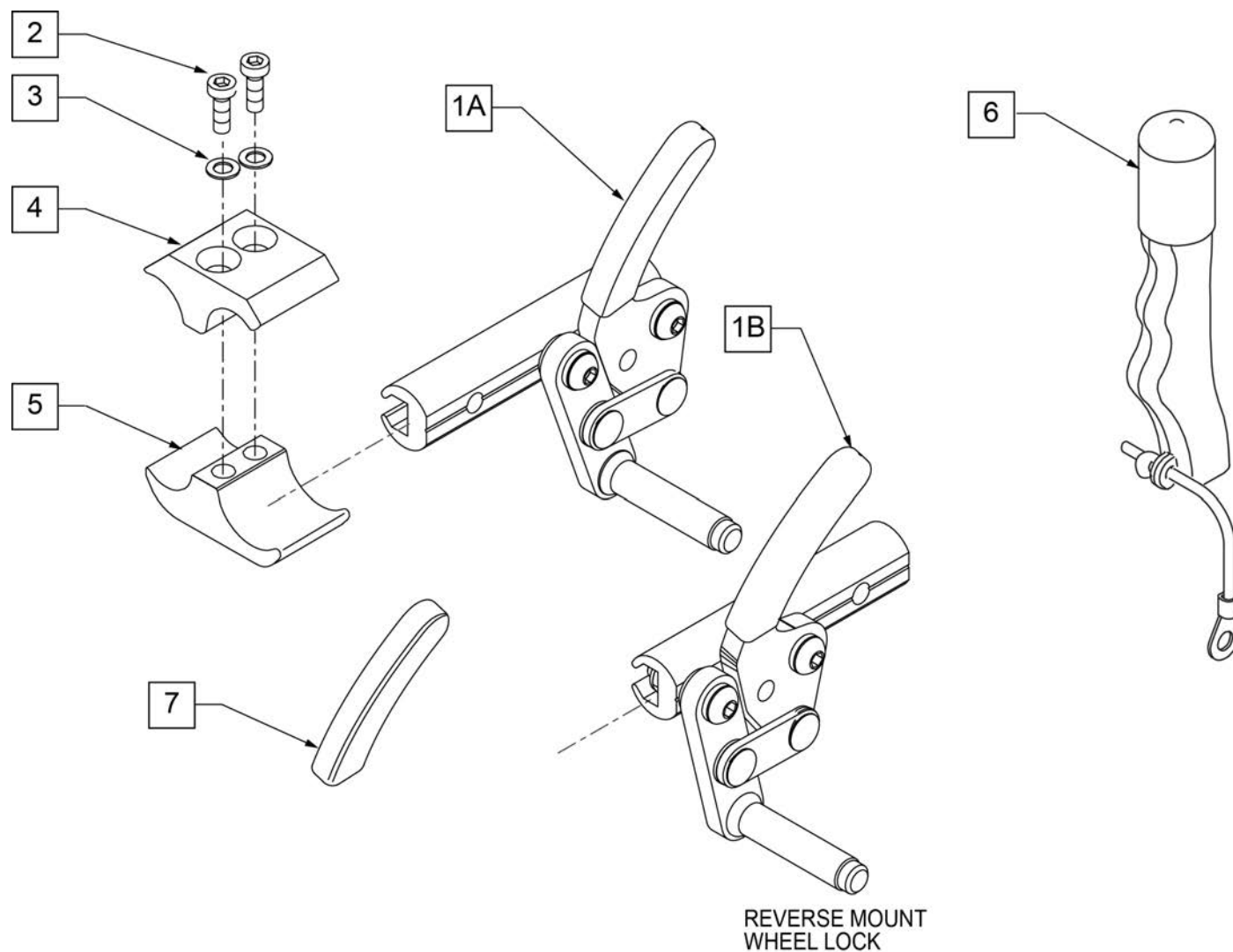


Pos.	Item Number	Description	Remarks	USD
1A	104216-001	WHEEL LOCK ASSEMBLY PUSH RIGHT	RIGHT	40.00
1A	104216-002	WHEEL LOCK ASSEMBLY PUSH LEFT	LEFT	40.00
1B	109643	WHEEL LOCK HI PUSH REV R	RIGHT	40.00
1B	109644	WHEEL LOCK HI PUSH REV LEFT	LEFT	40.00
2	107032	SHCS 1/4-28 X 1 ROHS		5.00
3	107367	1/4 WASHER HI COLLAR LOCK BLK		5.00
6	118068	WHEEL LOCK EXTENSION ASSY	each	30.00
7	104141	COVER CURVED WHEEL LOCK VINYL		3.00
1A,6	107637	WHEEL LOCK ASSM (PUSH) R W/ EXT	RIGHT	85.00
1A,6	107638	WHEEL LOCK ASSM (PUSH) L W/ EXT	LEFT	85.00
2-5	107639	WHL LCK CLMP ASM		30.00
1B,6	112623	WHEEL LOCK ASSM (PUSH REV) R W/ EXT	RIGHT	95.00
1B,6	112624	WHEEL LOCK ASSM (PUSH REV) L W/ EXT	LEFT	95.00

NOTE: ATTACHMENT CLAMP mounts to 1" diameter tubing.

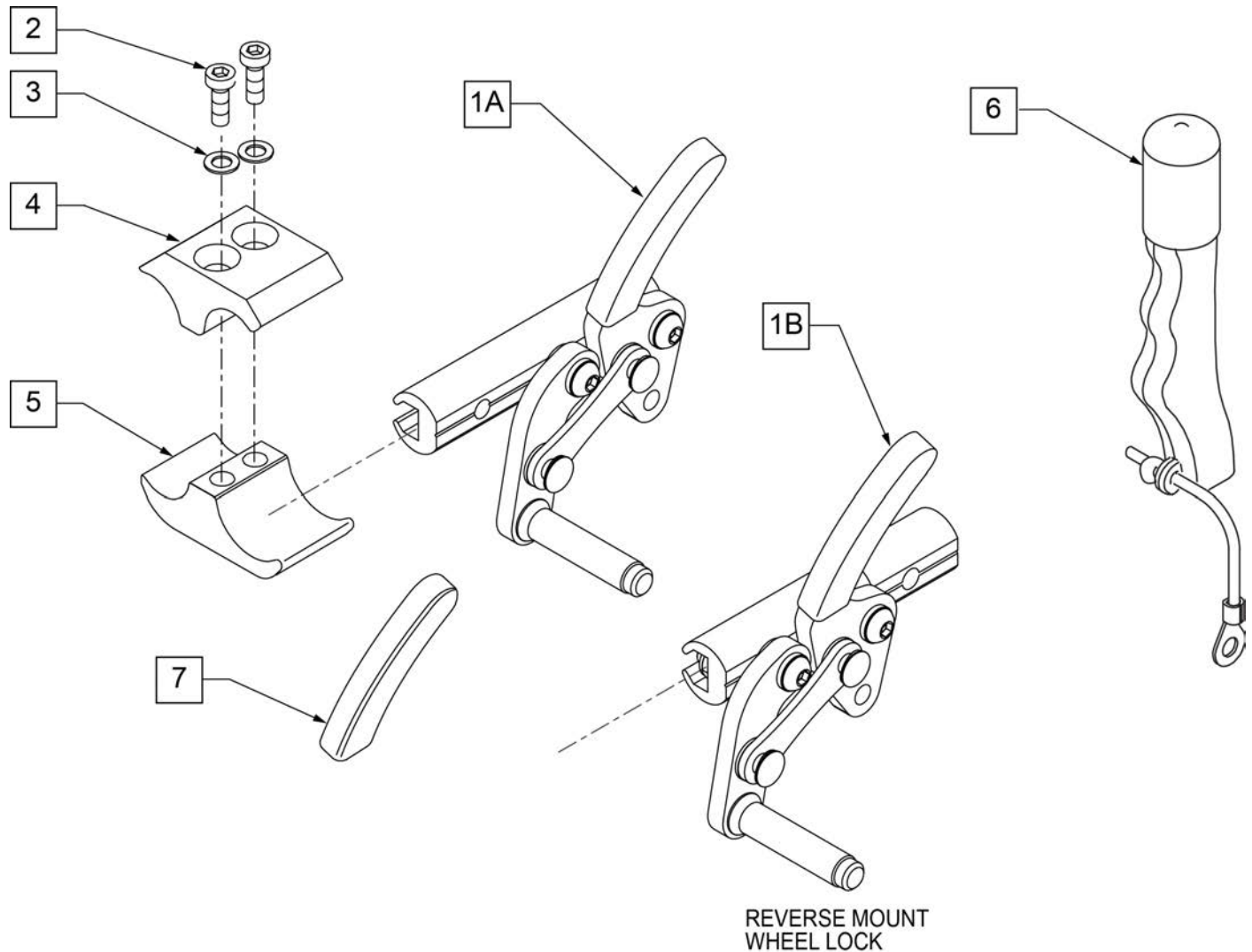


Pos.	Item Number	Description	Remarks	USD
1A	104227-001	WHEEL LOCK ASSEMBLY (PULL) RIGHT	RIGHT	40.00
1A	104227-002	WHEEL LOCK ASSM (PULL) LEFT	LEFT	40.00
1B	109647	WHEEL LOCK HI PULL REV R	RIGHT	40.00
1B	109648	WHEEL LOCK HI PULL REV L	LEFT	40.00
2	107032	SHCS 1/4-28 X 1 ROHS		5.00
3	107367	1/4 WASHER HI COLLAR LOCK BLK		5.00
6	118068	WHEEL LOCK EXTENSION ASSY	each	30.00
7	104141	COVER CURVED WHEEL LOCK VINYL		3.00
2-5	107639	WHL LCK CLMP ASM		30.00
1A,6	107640	WHEEL LOCK ASSM (PULL) R W/ EXT	RIGHT	85.00
1A,6	107641	WHEEL LOCK ASSM (PULL) L W/ EXT	LEFT	85.00
1B,6	112625	WHEEL LOCK HI PULL REV R W/ EXT	RIGHT	95.00
1B,6	112626	WHEEL LOCK HI PULL REV L W/ EXT	LEFT	95.00

NOTE: ATTACHMENT CLAMP mounts to 1" diameter tubing.

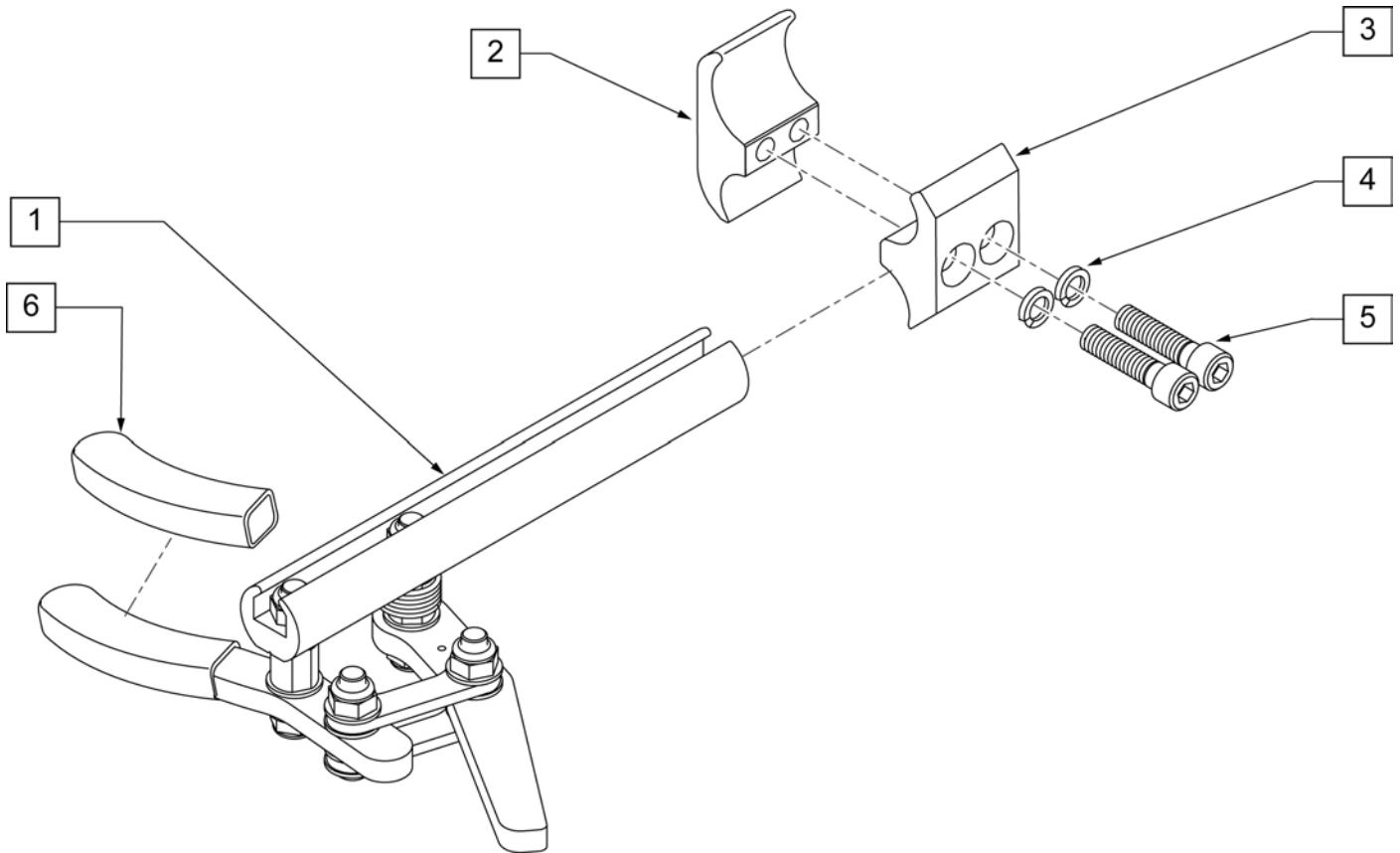

Pos.	Item Number	Description	Remarks	USD
1	104726-001	ASSY WHL LOCK SCISSOR ERGO	RIGHT	65.00
1	104726-002	ASSY WHL LOCK SCISSOR ERGO	LEFT	65.00
4	107367	1/4 WASHER HI COLLAR LOCK BLK		5.00
5	107032	SHCS 1/4-28 X 1 ROHS		5.00
6	104141	COVER CURVED WHEELLOCK VIN		3.00
2-6	107639	WHL LCK CLMP ASM		30.00


NOTE: ATTACHMENT CLAMP mounts to 1" diameter tubing.

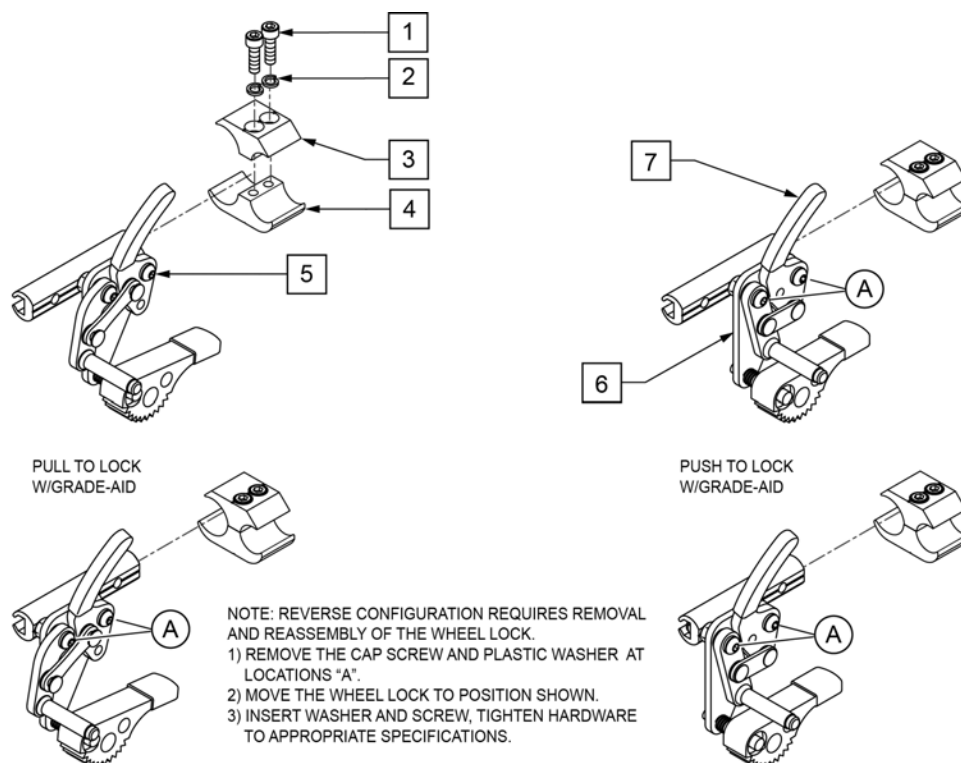

Pos.	Item Number	Description	Remarks	USD
1	107032	SHCS 1/4-28 X 1 ROHS		5.00
2	107367	1/4 WASHER HI COLLAR LOCK BLK		5.00
1-4	107639	WHL LCK CLMP ASM		30.00
5	107667	WHEEL LOCK PULL GRADE AID L	LEFT (Pull)	75.00
5	107668	WHEEL LOCK PULL GRADE AID R	RIGHT (Pull)	75.00
6	107665	WHEEL LOCK PUSH GRADE AID L	LEFT (Push)	75.00
6	107666	WHEEL LOCK PUSH GRADE AID	RIGHT (Push)	75.00
7	104141	COVER CURVED WHEEL LOCK VIN		3.00

NOTE: Refer to "WHEEL LOCK AVAILABILITY" Page for possible mount locations.


NOTE: ATTACHMENT CLAMP mounts to 1" diameter tubing.

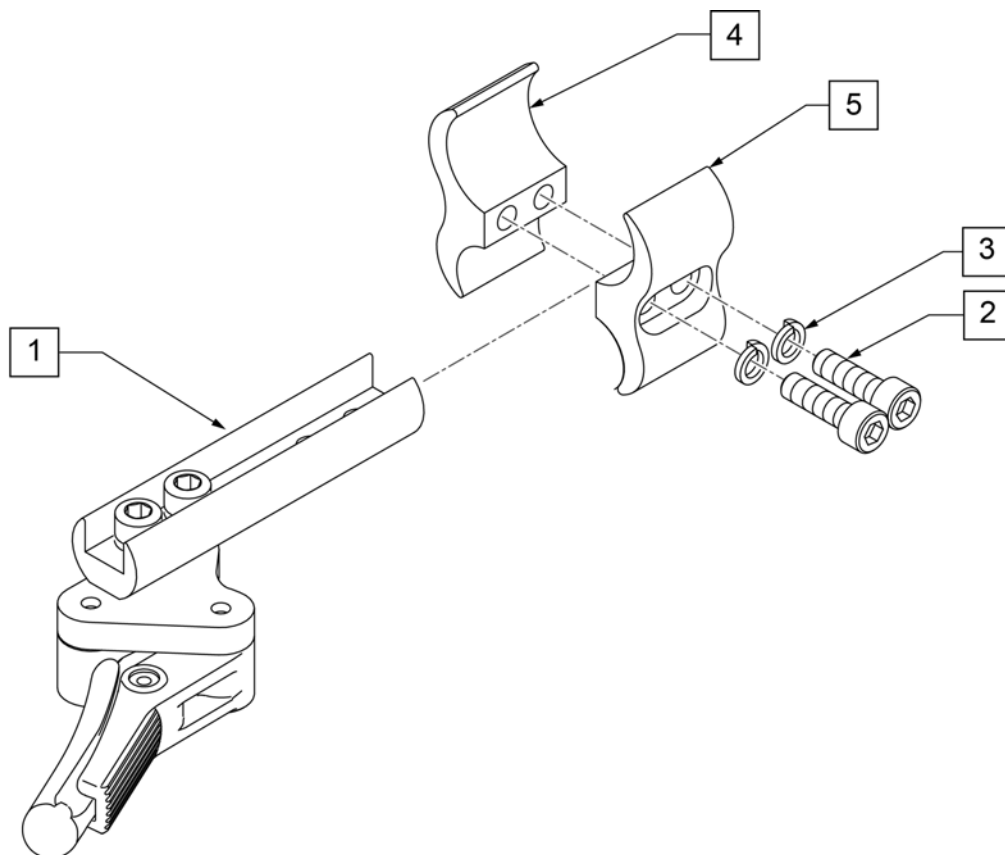

Pos.	Item Number	Description	USD
1	066608-001	CMPCT WHLLCK STD ASSY LEFT	75.00
1	066608-002	CMPCT WHLLCK STD ASSY RIGHT	75.00
2	107032	SHCS 1/4-28 X 1 ROHS	5.00
3	107367	1/4 WASHER HI COLLAR LOCK BLK	5.00
2-5	107639	WHL LCK CLMP ASM	30.00

WHEEL LOCK EXTENSION HANDLE

[089/2011]

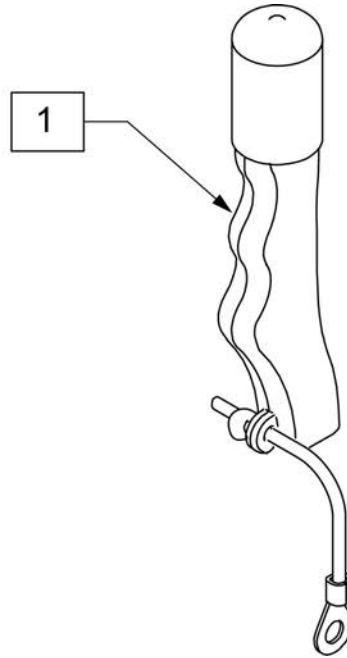

Pos.	Item Number	Description	Remarks	USD
1	119563	WHEEL LOCK EXT ASSY (PAIR)	Pair	60.00



Pos.	Item Number	Description	Remarks	USD
1-3	401094	20" MAG W/BRNG 6 SPOKE	20"	124.00
1-3	401093	22" MAG W/BRNG 6 SPOKE	22"(501mm)	131.58
1-3	401092	24" MAG W/BRNG 6 SPOKE	24"(540mm)	131.58
2	349500	1 1/8 X 1/2 BEARING		10.30
3	521564	MAG BEARING SPACER		2.48
4	101932	SPCR .194ID X .375 OD X .750	Used when previously ordered with Omit handrims or Natural Fit Handrims	5.00

(10/2010)

NOTE: THIS WHEEL UTILIZES SCREW MOUNT HANDRIMS - THE HANDRIM HAS A THREADED HOLE THAT MOUNTS THE HANDRIM TO THE WHEEL THROUGH A SPACER AND FASTNER.

NOTE: IF MAG WHEEL ASSEMBLY WAS ORDER WITH OUT HANDRIM AND A HANDRIM IS ADDED TO THE WHEEL QTY 6 OF 561037 AND 281175 MUST BE ORDERED TO PROPERLY MOUNT THE HANDRIM.

NOTE: THIS WHEEL UTILIZES SCREW MOUNT HANDRIMS - THE HANDRIM HAS A THREADED HOLE THAT MOUNTS THE HANDRIM TO THE WHEEL THROUGH A SPACER AND FASTNER.



Pos.	Item Number	Description	Remarks	USD
1A	116178	WHEEL ASSY MBL 20"	20" (451)	100.00
1B	116179	WHEEL ASSY MBL 22"	WHL RIM/SPOKE MBL 22 (501)	100.00
1C	116180	WHEEL ASSY MBL 24"	WHL RIM/SPOKE MBL 24 (540)	100.00
1D	116181	WHEEL ASSY MBL 25"	WHL RIM/SPOKE MBL 25 (559)	100.00
1E	116182	WHEEL ASSY MBL 26"	WHL RIM/SPOKE MBL 26 (590)	100.00
2	114097	BEARING, REPLACEMENT, MBL	12.7mm i.d. - 28mm o.d. - 8mm wide	10.30
3A	118100	MBL LT 20IN SPOKE RPLMNT KIT	Kit contains 6 inner and 6 outer spokes	25.00
3B	118101	MBL LT 22IN SPOKE RPLMNT KIT	Kit contains 6 inner and 6 outer spokes	25.00
3C	118102	MBL LT 24IN SPOKE RPLMNT KIT	Kit contains 6 inner and 6 outer spokes	25.00
3D	118103	MBL LT 25IN SPOKE RPLMNT KIT	Kit contains 6 inner and 6 outer spokes	25.00
3E	118104	MBL LT 26IN SPOKE RPLMNT KIT	Kit contains 6 inner and 6 outer spokes	25.00

NOTE: THIS WHEEL UTILIZES TAB MOUNT HANDRIMS - THE HANDRIM HAS A STRAIGHT TAB THAT MOUNTS TO THE WHEEL THROUGH THE RIM.



Pos.	Item Number	Description	Remarks	USD
1A	112857	ULTRA LIGHTWEIGHT WHEEL 20"	20"	160.95
1B	112858	ULTRA LIGHT WHEEL 22"	22"(501mm)	160.95
1C	112859	ULTRA LIGHTWEIGHT WHEEL 24"	24"(540mm)	165.95
1D	112860	ULTRA LIGHTWEIGHT WHEEL 25"	25"(559mm)	165.95
2	349500	1 1/8 X 1/2 BEARING		10.30
3A	118105	ULTRA LT 20IN SPOKE RPLMNT KIT	Kit contains 12 spokes	45.00
3B	118106	ULTRA LT 22IN SPOKE RPLMNT KIT	Kit contains 12 spokes	45.00
3C	118107	ULTRA LT 24IN SPOKE RPLMNT KIT	Kit contains 12 spokes	45.00
3D	118108	ULTRA LT 25IN SPOKE RPLMNT KIT	Kit contains 12 spokes	45.00

NOTE: THIS WHEEL UTILIZES TAB MOUNT HANDRIMS - THE HANDRIM HAS A STRAIGHT TAB THAT MOUNTS TO THE WHEEL THROUGH THE RIM.



Pos.	Item Number	Description	Remarks	USD
1	762438	SPINERGY STD WHEEL 540 BLK SPK	24"(540mm)	500.00
1	762441	SPINERGY STD WHL 559 BLK SPK	25"(559mm)	350.00
1	762437	SPINERGY STD WHEEL 590 BLK SPK	26"(590mm)	500.00
1	762442	SPINERGY STD WHEEL 540 BLUE	24"(540mm)	347.50
1	762444	SPINERGY STD WHEEL 559 BLU	25"(559mm)	347.50
1	762446	SPINERGY STD WHEEL 590 BLUE	26"(590mm)	347.50
1	762443	SPINERGY STD WHEEL 540 RED	24"(540mm)	347.50
1	762445	SPINERGY STD WHEEL 559 RED	25"(559mm)	347.50
1	762447	SPINERGY STD WHEEL 590 RED	26"(590mm)	347.50
1	762456	SPINERGY STD WHEEL 540 WHT	24"(540mm)	347.50
1	762463	SPINERGY STD WHL 559 WHT SPK	25"(559mm)	
1	762467	SPINERGY STD WHEEL 590 WHT	26"(590mm)	
1	762433	SPINERGY STD WHEEL 540 YELLOW	24"(540mm)	347.50
1	762462	SPINERGY STD WHL 559 YLW SPK	25"(559mm)	347.50
1	762431	SPINERGY STD WHEEL 590 YELLOW	26"(590mm)	491.00
2	349500	1 1/8 X 1/2 BEARING		10.30


NOTE: THIS WHEEL UTILIZES TAB MOUNT HANDRIMS - THE HANDRIM HAS A STRAIGHT TAB THAT MOUNTS TO THE WHEEL THROUGH THE RIM.



Pos.	Item Number	Description	Remarks	USD
1	101988-001	SPINRGY SPT LT EX WHL 540 BLK	24"(540mm)	
1	101989-001	SPINERGY LT EX WHEEL 540 BLK	24"(540mm)	495.00
1	101990-001	SPINRGY SPT LT EX WHL 559 BLK	25"(559mm)	
1	101991-001	SPINERGY LT EX WHEEL 559 BLK	25"(559mm)	495.00
1	101992-001	SPINRGY SPT LT EX WHL 590 BLK	26"(590mm)	
1	101993-001	SPINERGY LT EX WHEEL 590 BLK	26"(590mm)	495.00
1	101988-002	SPINRGY SPT LT EX WHL 540 BLU	24"(540mm)	
1	101989-002	SPINERGY LT EX WHEEL 540 BLU	24"(540mm)	447.50
1	101990-002	SPINRGY SPT LT EX WHL 559 BLU	25"(559mm)	
1	101991-002	SPINERGY LT EX WHEEL 559 BLU	25"(559mm)	447.50
1	101992-002	SPINRGY SPT LT EX WHL 590 BLU	26"(590mm)	
1	101993-002	SPINERGY LT EX WHEEL 590 BLU	26"(590mm)	
1	101988-003	SPINRGY SPT LT EX WHL 540 RED	24"(540mm)	347.50
1	101989-003	SPINERGY LT EX WHEEL 540 RED	24"(540mm)	447.50
1	101990-003	SPINRGY SPT LT EX WHL 559 RED	25"(559mm)	
1	101991-003	SPINERGY LT EX WHEEL 559 RED	25"(559mm)	347.50
1	101992-003	SPINRGY SPT LT EX WHL 590 RED	26"(590mm)	595.00
1	101993-003	SPINERGY LT EX WHEEL 590 RED	26"(590mm)	
1	101988-005	SPINRGY SPT LT EX WHL 540 WHT	24"(540mm)	495.00
1	101989-005	SPINERGY LT EX WHEEL 540 WHT	24"(540mm)	495.00
1	101990-005	SPINRGY SPT LT EX WHL 559 WHT	25"(559mm)	

Pos.	Item Number	Description	Remarks	USD
1	101991-005	SPINERGY LT EX WHEEL 559 WHT	25" (559mm)	
1	101992-005	SPINERGY SPT LT EX WHL 590 WHT	26" (590mm)	
1	101993-005	SPINERGY LT EX WHEEL 590 WHT	26" (590mm)	
1	101988-004	SPINERGY SPT LT EX WHL 540 YLW	24" (540mm)	447.50
1	101989-004	SPINERGY LT EX WHEEL 540 YLW	24" (540mm)	447.50
1	101990-004	SPINERGY SPT LT EX WHL 559 YLW	25" (559mm)	
1	101991-004	SPINERGY LT EX WHEEL 559 YLW	25" (559mm)	479.00
1	101992-004	SPINERGY SPT LT EX WHL 590 YLW	26" (590mm)	447.50
1	101993-004	SPINERGY LT EX WHEEL 590 YLW	26" (590mm)	447.50
2	349500	1 1/8 X 1/2 BEARING		10.30

NOTE: THIS WHEEL UTILIZES SCREW MOUNT HANDRIMS - THE HANDRIM HAS A THREADED HOLE THAT MOUNTS THE HANDRIM TO THE WHEEL THROUGH A SPACER AND FASTNER.

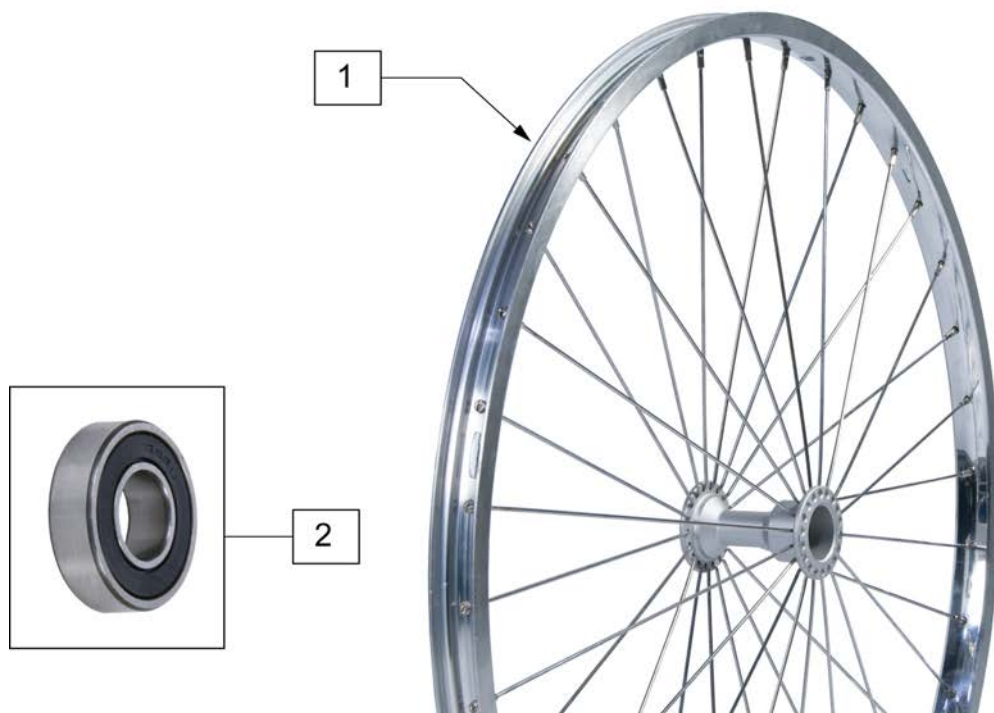

Pos.	Item Number	Description	Remarks	USD
1	401075	24 RIM ASSM X 1.5 SUN PSH BTTN	22" (507MM)	135.00
1	401076	24" HT RIM ASSM SUN PUSH BTTN	25" (559MM)	94.86
2	349500	1 1/8 X 1/2 BEARING		10.30




Pos.	Item Number	Description	Remarks	USD
1	384706	TIRE PNEU 12-1/2" X 2-1/4"	12-1/2" X 2-1/4"	32.64
1	385237	16" X 1.75" PNEU TIRE		34.68
1	385225	TIRE PNEUMATIC 18"	18" x 1 3/8	19.50
1	385250	20" PNEUMATIC TIRE	20" x 1 3/8	14.99
1	385275	22" PNEUMATIC TIRE	22" x 1 3/8 (501mm)	14.99
1	385300	24" PNEUMATIC TIRE	24" x 1 3/8 (540mm)	16.12
1	385325	26" PNEUMATIC TIRE	26" x 1 3/8 (590mm)	16.12
1	384614	24" x 2.125 PNEUMATIC TIRE	24"x2 1/8 (540mm) used w/ M6	25.00



Pos.	Item Number	Description	Remarks	USD
1	386232	20" POLY TIRE STD	20"	31.11
1	386242	22" POLY TIRE STD	22" (501mm)	31.11
1	386252	24" POLY TIRE STD	24" (540mm)	31.11




Pos.	Item Number	Description	Remarks	USD
1	385280	22" x 1" HI PRES CLINCHER	22" (501mm)	22.44
1	389020	24" x 1" HI PRES CLNCHR TIRE	24" (540mm)	22.44
1	389012	QP 24" x 1" HT TURB BLK WR/BD 559	25" (559mm)	46.92
1	389030	26" x 1" HI PRES CLNCHR TIRE	26" (590mm)	22.44
1	105919	22" x 1" HI PRESS CLNCHR ULTRALIGHT	22" (489mm)	50.00

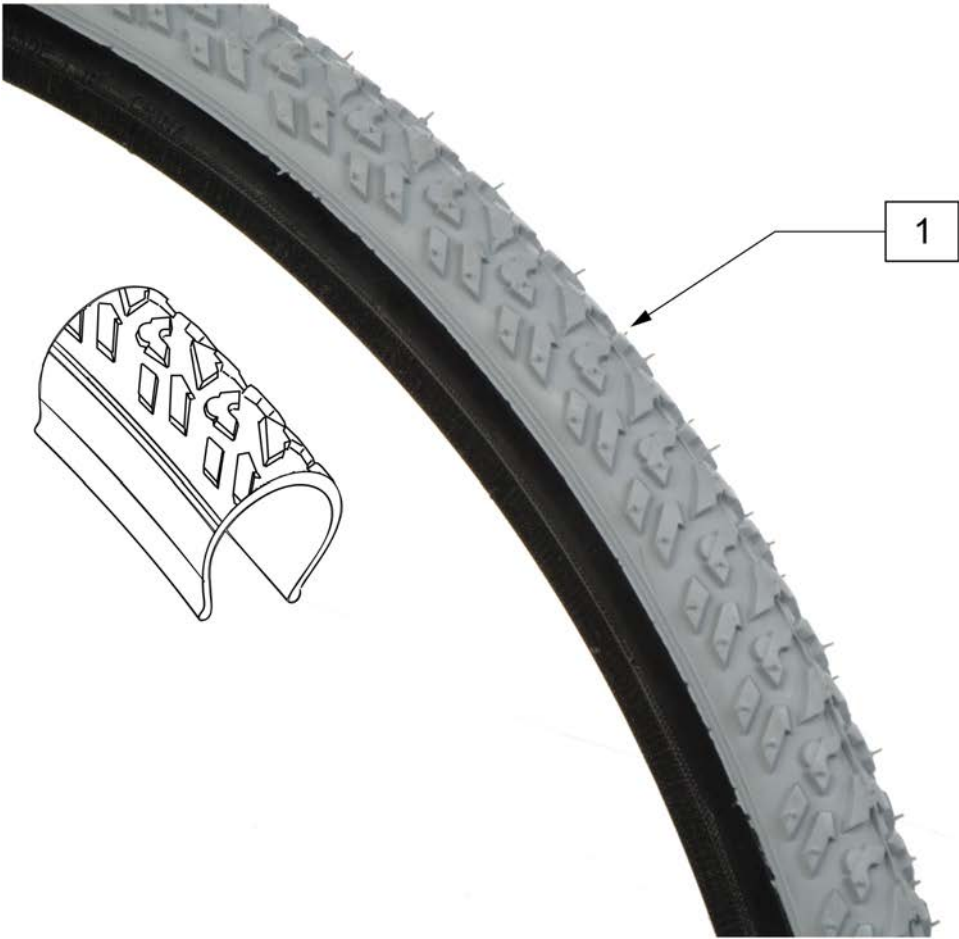

Pos.	Item Number	Description	Remarks	USD
1	385354	24" X 1-3/8" KNOBBIE TIRE	24"(540mm)	20.40



Pos.	Item Number	Description	Remarks	USD
1	102066-004	TIRE SCHWALBE MP 489 BLK	22" (489mm)	65.00
1	102066-001	TIRE SCHWBL MPE 540 BLK	24" (540mm)	65.00
1	102066-002	TIRE SCHWBL MPE 559 BLK	25" (559mm)	65.00
1	102066-003	TIRE SCHWBL MPE 590 BLK	26" (590mm)	65.00

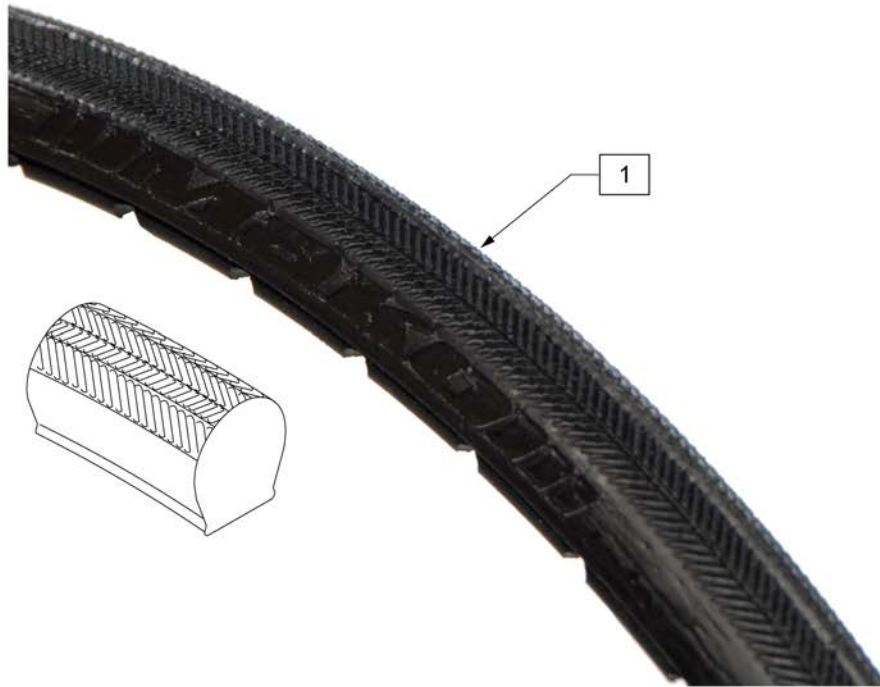


Pos.	Item Number	Description	Remarks	USD
1	102060-001	TIRE KIK MAKO MAG SUN 540 BLK	24"(540mm) Black	65.95
1	102060-002	TIRE KIK MAKO MAG SUN 540 YLW	24"(540mm) Yellow	
1	102060-003	TIRE KIK MAKO MAG SUN 540 RED	24"(540mm) Red	
1	102060-004	TIRE KIK MAKO MAG SUN 540 BLU	24"(540mm) Blue	
1	102060-005	TIRE KIK MAKO MAG SUN 540 DKG	24"(540mm) Dark Grey	
1	102061-001	TIRE KIK MAKO SPIN 559 BLK	25"(559mm) Black	65.95
1	102061-002	TIRE KIK MAKO SPIN 559 YLW	25"(559mm) Yellow	
1	102061-003	TIRE KIK MAKO SPIN 559 RED	25"(559mm) Red	
1	102061-004	TIRE KIK MAKO SPIN 559 BLU	25"(559mm) Blue	
1	102061-005	TIRE KIK MAKO SPIN 559 DKGRPH	25"(559mm) Dark Grey	



Pos.	Item Number	Description	Remarks	USD
1	109213	TIRE 24" URBAN ASSAULT	22"(507mm)	25.00
1	109214	TIRE 26" URBAN ASSAULT	26"(590mm)	25.00



Pos.	Item Number	Description	Remarks	USD
1	384604	TIRE CS 24 X 2.1 SMOKER 507MM	22"(501mm)	45.00
1	384606	TIRE CS 26 X 2.1 SMOKER 559MM	25"(559mm)	31.62

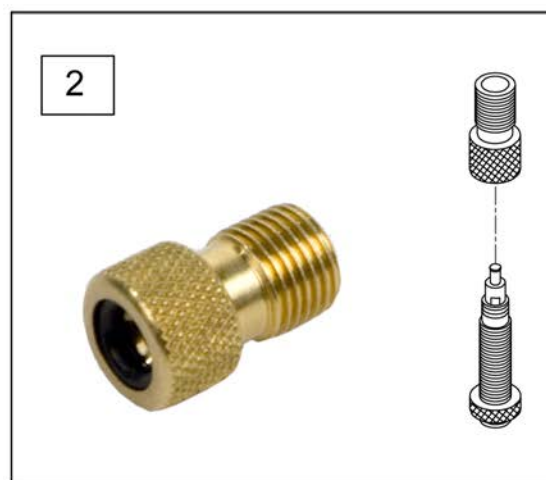


Pos.	Item Number	Description	Remarks	USD
1A	382305	22" X 1" TUBE PRESTA VALVE		7.50
1A	382330	24" TUBE PRESTA VALVE (540MM)		7.55
1A	384545	QP 24" HT INNER TUBE (559MM)		7.65
1A	382355	26" TUBE PRESTA VALVE (590MM)		7.55
1A	384550	QP 26" HT INNER TUBE (700C)		7.65
1B	384736	INNER TUBE 12-1/2" X 2-1/4"		9.79
1B	384738	16" X 1.75" INNER TUBE		9.49
1B	382255	TUBE 18" TIRE	18" X 1-3/4" TUBE	16.00
1B	382275	20" TUBE	20" 1-3/8" TUBE	7.55
1B	382300	22" TUBE	22" X 1-3/8" TUBE	7.55
1B	382336	TUBE PRIMO 24" 25-540 SV	24" X 1" TUBE	5.10
1B	382325	24 X 1.3/8IN TUBE REG		7.55
1B	382334	TUBE INLINE 24 X 1.75		5.10
1B	384551	26" X 1" TUBE SCHRADER		8.16
1B	382350	26" TUBE	26" X 1-3/8" TUBE	7.55
1B	382361	TUBE 26 X 1.5/1.95/ S/V		6.12
1B	382365	26 X 1.25/1.5 TUBE SHRADER		6.12
1B	382375	24" x 2.125 TUBE		8.16
1C	102972-001	12 1/2 IN AIRLESS INSERT	12-1/2" X 2-1/4"	50.00
1C	102973-001	16IN AIRLESS INSERT	16" X 1-3/4"	35.00
1C	102973-002	18IN AIRLESS INSERT	18" X 1-3/4"	34.95

Pos.	Item Number	Description	Remarks	USD
1C	102973-003	20IN AIRLESS INSERT	20" X 1-3/8"	34.95
1C	102972-002	AIRLESS INSERT 20 X 2-1/8 P3		60.00
1C	102973-004	22IN AIRLESS INSERT	22" X 1-3/8"	34.95
1C	102973-005	24IN AIRLESS INSERT	24" X 1-3/8"	34.95
1C	102972-003	AIRLESS INSERT 24 X 2-1/8		85.00
1C	108259	26" AIRLESS INSERT	26" X 1-3/8"	30.00
2	381230	VALVE ADAPTER PRESTA		5.10


Note: When replacing the Seat Sling it is recommended to order the Seat Rail End Plug Kit.

Note: On Quickie 2 chairs with S/N prefix (Q2V) & Quickie 2 Lite chairs the diameter of the Seat Rail Rod is 8mm (.31") which is larger than the Q2 chairs with S/N prefix (Q2 & Q2H) which is .25" in diameter.

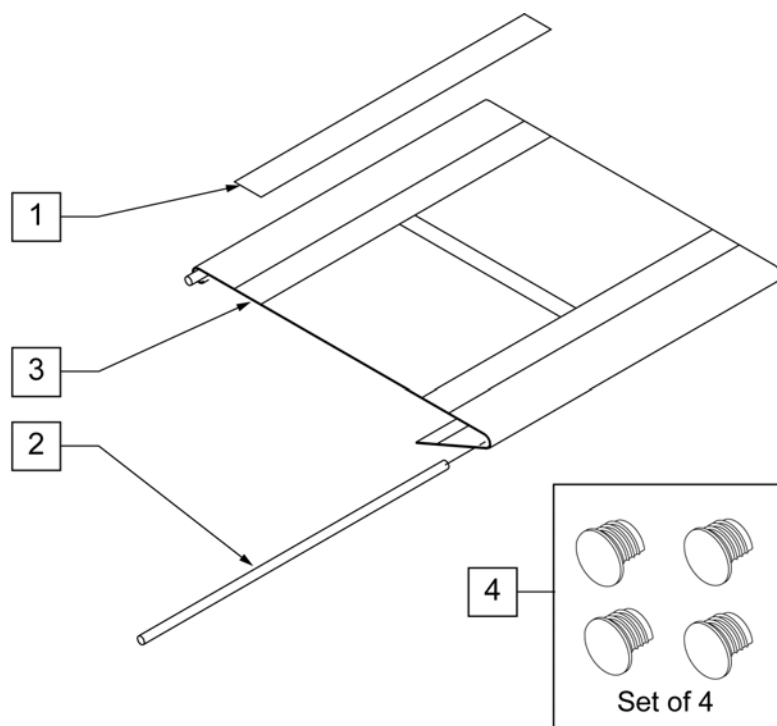


MEASURING GUIDE

CHAIR WIDTH	DIM A	DIM B
12"	16.5"	13.5"
13"	17.5"	14.5"
14"	18.5"	15.5"
15"	19.5"	16.5"
16"	20.5"	17.5"
17"	21.5"	18.5"
18"	22.5"	19.5"
19"	23.5"	20.5"
20"	24.5"	21.5"

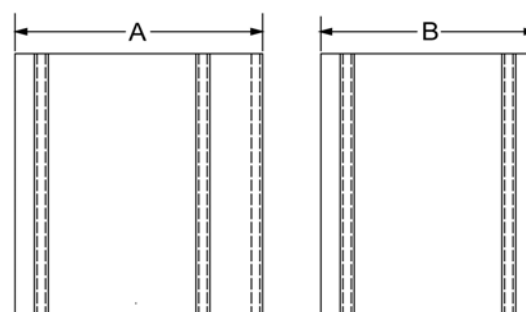


Pos.	Item Number	Description	Remarks	USD	Pos.	Item Number	Description	Remarks	USD
1	901169	SEAT SLING VELCRO STRIP 2"		3.06	3	722603	SEAT SLING 13" X 13" CHNL		55.00
2	106584	SEAT RAIL ROD 12"		7.50	3	722604	SEAT SLING 13" X 14" CHNL		55.00
2	106585	SEAT RAIL ROD 13"		7.50	3	722605	SEAT SLING 13" X 15" CHNL		55.00
2	106586	SEAT RAIL ROD 14"		7.50	3	722606	SEAT SLING 13" X 16" CHNL		55.00
2	106587	SEAT RAIL ROD 15"		7.50	3	722607	SEAT SLING 13" X 17" CHNL		55.00
2	106588	SEAT RAIL ROD 16"		7.50	3	722608	SEAT SLING 13" X 18" CHNL		55.00
2	106589	SEAT RAIL ROD 17"		7.50			14" Seat Slings		
2	106590	SEAT RAIL ROD 18"		7.50	3	722616	SEAT SLING 14" X 10" CHNL		55.00
2	106591	SEAT RAIL ROD 19"		7.50	3	722617	SEAT SLING 14" X 11" CHNL		55.00
2	106592	SEAT RAIL ROD 20"		7.50	3	722618	SEAT SLING 14" X 12" CHNL		55.00
		12" Wide Seat Slings			3	722619	SEAT SLING 14" X 13" CHNL		55.00
3	722590	SEAT SLING 12" X 10" CHNL		55.00	3	722620	SEAT SLING 14" X 14" CHNL		55.00
3	722591	SEAT SLING 12" X 11" CHNL		55.00	3	722621	SEAT SLING 14" X 15" CHNL		55.00
3	722592	SEAT SLING 12" X 12" CHNL		55.00	3	722622	SEAT SLING 14" X 16" CHNL		55.00
3	722593	SEAT SLING 12" X 13" CHNL		55.00	3	722623	SEAT SLING 14" X 17" CHNL		55.00
3	722594	SEAT SLING 12" X 14" CHNL		55.00	3	722624	SEAT SLING 14" X 18" CHNL		55.00
3	722595	SEAT SLING 12" X 15" CHNL		55.00	3	722625	SEAT SLING 14" X 19" CHNL		55.00
3	722596	SEAT SLING 12" X 16" CHNL		55.00	3	722626	SEAT SLING 14" X 20" CHNL		55.00
3	722597	SEAT SLING 12" X 17" CHNL		55.00			15" Seat Slings		
		13" Wide Seat Slings			3	722630	SEAT SLING 15" X 14" CHNL		55.00
3	722600	SEAT SLING 13" X 10" CHNL		55.00	3	722631	SEAT SLING 15" X 15" CHNL		55.00
3	722601	SEAT SLING 13" X 11" CHNL		55.00	3	722632	SEAT SLING 15" X 16" CHNL		55.00
3	722602	SEAT SLING 13" X 12" CHNL		55.00	3	722633	SEAT SLING 15" X 17" CHNL		55.00

Pos.	Item Number	Description	Remarks	USD	Pos.	Item Number	Description	Remarks	USD
3	722634	SEAT SLING 15" X 18" CHNL		55.00	3	722662	SEAT SLING 18" X 16" CHNL		55.00
3	722635	SEAT SLING 15" X 19" CHNL		55.00	3	722663	SEAT SLING 18" X 17" CHNL		55.00
3	722636	SEAT SLING 15" X 20" CHNL		55.00	3	722664	SEAT SLING 18" X 18" CHNL		55.00
		16" Seat Slings			3	722665	SEAT SLING 18" X 19" CHNL		55.00
3	722640	SEAT SLING 16" X 14" CHNL		55.00	3	722666	SEAT SLING 18" X 20" CHNL		55.00
3	722641	SEAT SLING 16" X 15" CHNL		55.00			19" Seat Slings		
3	722642	SEAT SLING 16" X 16" CHNL		55.00	3	722670	SEAT SLING 19" X 14" CHNL		55.00
3	722643	SEAT SLING 16" X 17" CHNL		55.00	3	722671	SEAT SLING 19" X 15" CHNL		55.00
3	722644	SEAT SLING 16" X 18" CHNL		55.00	3	722672	SEAT SLING 19" X 16" CHNL		55.00
3	722645	SEAT SLING 16" X 19" CHNL		55.00	3	722673	SEAT SLING 19" X 17" CHNL		55.00
3	722646	SEAT SLING 16" X 20" CHNL		55.00	3	722674	SEAT SLING 19" X 18" CHNL		55.00
		17" Seat Slings			3	722675	SEAT SLING 19" X 19" CHNL		55.00
3	722650	SEAT SLING 17" X 14" CHNL		55.00	3	722676	SEAT SLING 19" X 20" CHNL		55.00
3	722651	SEAT SLING 17" X 15" CHNL		55.00			20" Seat Slings		
3	722652	SEAT SLING 17" X 16" CHNL		55.00	3	722680	SEAT SLING 20" X 14" CHNL		55.00
3	722653	SEAT SLING 17" X 17" CHNL		55.00	3	722681	SEAT SLING 20" X 15" CHNL		55.00
3	722654	SEAT SLING 17" X 18" CHNL		55.00	3	722682	SEAT SLING 20" X 16" CHNL		55.00
3	722655	SEAT SLING 17" X 19" CHNL		55.00	3	722683	SEAT SLING 20" X 17" CHNL		55.00
3	722656	SEAT SLING 17" X 20" CHNL		55.00	3	722684	SEAT SLING 20" X 18" CHNL		55.00
		18" Seat Slings			3	722685	SEAT SLING 20" X 19" CHNL		55.00
3	722660	SEAT SLING 18" X 14" CHNL		55.00	3	722686	SEAT SLING 20" X 20" CHNL		55.00
3	722661	SEAT SLING 18" X 15" CHNL		55.00	4	108117	SEAT RAIL END PLUG KIT	Set of 4	30.00

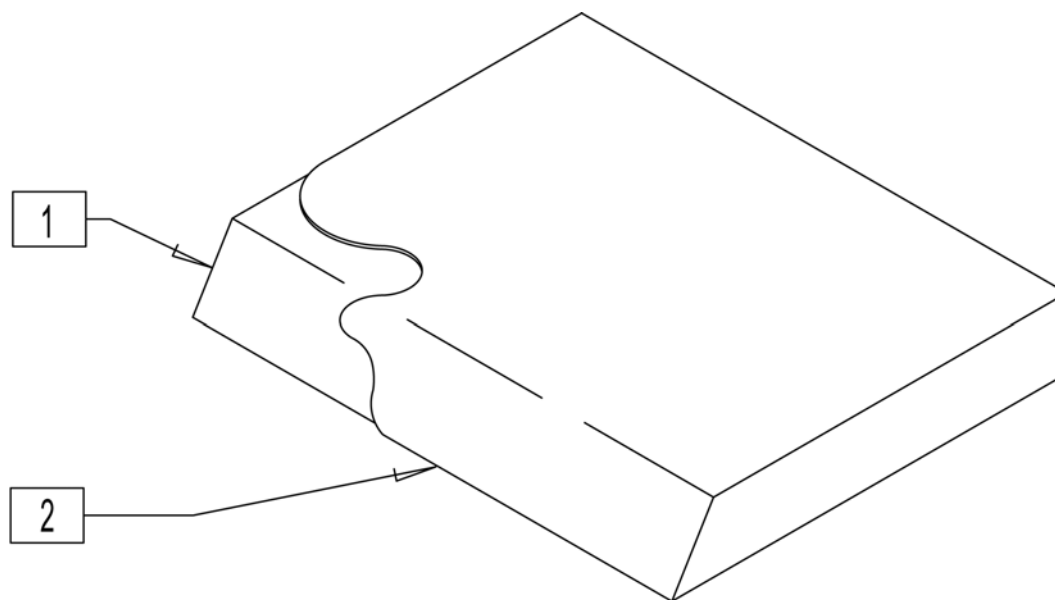


Pos.	Item Number	Description	USD	Pos.	Item Number	Description	USD
1-2	700322	SEAT CUSHION 14" X 14" X 2"	100.98	1-2	700642	SEAT CUSHION 17" X 18" X 2"	100.98
1-2	700332	SEAT CUSHION 14" X 15" X 2"	100.98	1-2	700652	SEAT CUSHION 17" X 19" X 2"	100.98
1-2	700342	SEAT CUSHION 14" X 16" X 2"	100.98	1-2	700662	SEAT CUSHION 17" X 20" X 2"	100.98
1-2	700352	SEAT CUSHION 14" X 17" X 2"	100.98	1-2	700702	SEAT CUSHION 18" X 14" X 2"	100.98
1-2	700362	SEAT CUSHION 14" X 18" X 2"	100.98	1-2	700712	SEAT CUSHION 18" X 15" X 2"	100.98
1-2	700372	SEAT CUSHION 14" X 19" X 2"	99.00	1-2	700722	SEAT CUSHION 18" X 16" X 2"	100.98
1-2	700382	SEAT CUSHION 14" X 20" X 2"	99.00	1-2	700732	SEAT CUSHION 18" X 17" X 2"	100.98
1-2	700422	SEAT CUSHION 15" X 14" X 2"	100.98	1-2	700742	SEAT CUSHION 18" X 18" X 2"	100.98
1-2	700432	SEAT CUSHION 15" X 15" X 2"	100.98	1-2	700752	SEAT CUSHION 18" X 19" X 2"	100.98
1-2	700442	SEAT CUSHION 15" X 16" X 2"	100.98	1-2	700762	SEAT CUSHION 18" X 20" X 2"	100.98
1-2	700452	SEAT CUSHION 15" X 17" X 2"	100.98	1-2	700802	SEAT CUSHION 19" X 14" X 2"	99.00
1-2	700462	SEAT CUSHION 15" X 18" X 2"	100.98	1-2	700812	SEAT CUSHION 19" X 15" X 2"	100.98
1-2	700472	SEAT CUSHION 15" X 19" X 2"	100.98	1-2	700822	SEAT CUSHION 19" X 16" X 2"	100.98
1-2	700482	SEAT CUSHION 15" X 20" X 2"	100.98	1-2	700832	SEAT CUSHION 19" X 17" X 2"	100.98
1-2	700522	SEAT CUSHION 16" X 14" X 2"	100.98	1-2	700842	SEAT CUSHION 19" X 18" X 2"	100.98
1-2	700532	SEAT CUSHION 16" X 15" X 2"	100.98	1-2	700852	SEAT CUSHION 19" X 19" X 2"	100.98
1-2	700542	SEAT CUSHION 16" X 16" X 2"	100.98	1-2	700862	SEAT CUSHION 19" X 20" X 2"	100.98
1-2	700552	SEAT CUSHION 16" X 17" X 2"	100.98	1-2	700902	SEAT CUSHION 20" X 14" X 2"	99.00
1-2	700562	SEAT CUSHION 16" X 18" X 2"	100.98	1-2	700912	SEAT CUSHION 20" X 15" X 2"	100.98
1-2	700572	SEAT CUSHION 16" X 19" X 2"	100.98	1-2	700922	SEAT CUSHION 20" X 16" X 2"	100.98
1-2	700582	SEAT CUSHION 16" X 20" X 2"	99.00	1-2	700932	SEAT CUSHION 20" X 17" X 2"	100.98
1-2	700602	SEAT CUSHION 17" X 14" X 2"	100.98	1-2	700942	SEAT CUSHION 20" X 18" X 2"	100.98
1-2	700612	SEAT CUSHION 17" X 15" X 2"	100.98	1-2	700952	SEAT CUSHION 20" X 19" X 2"	100.98
1-2	700622	SEAT CUSHION 17" X 16" X 2"	100.98	1-2	700962	SEAT CUSHION 20" X 20" X 2"	100.98
1-2	700632	SEAT CUSHION 17" X 17" X 2"	100.98				

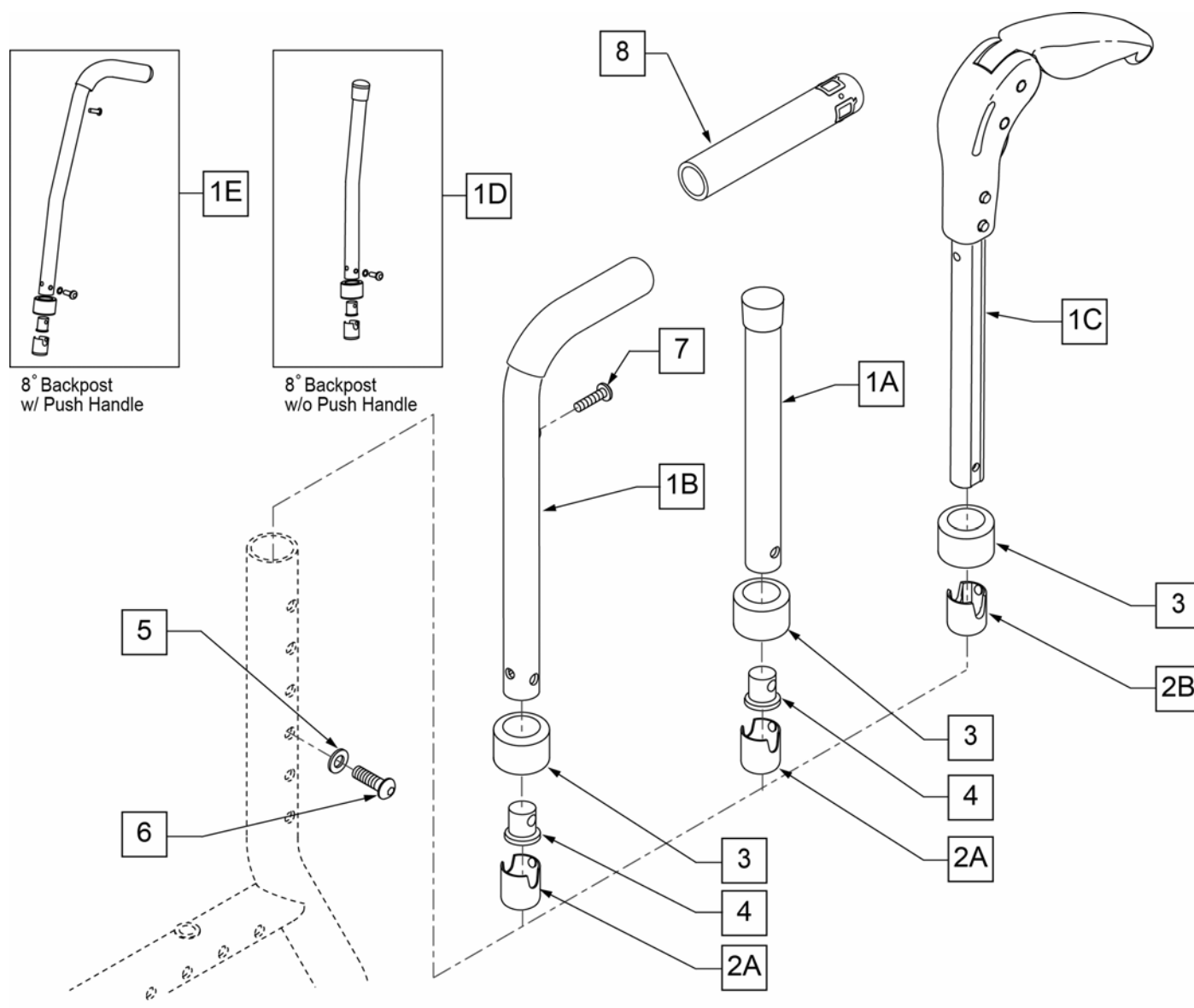


Pos.	Item Number	Description	Remarks	USD
2A	104187	BUSHING BCKRST NONFLDNG	Straight & Integral Push Handle	5.00
2B	104369	BUSHING BCKRST FOLDING HANDLE	Folding Push Handle Only	5.00
3	561018	1" TUBE END BUSHING THICK		2.00
4	483001	Q2 FRAME/FOOT RETAINING PLUG		4.63
5	104414	WAVE WASHER 6 MM		5.00
6	108107	BHCS 1/4-20 X 3/4 ROHS		5.00
7	107384	10/32 X 5/8 PAN HD PHIL		5.00
8	920026	QUICKIE GRIP 3/4" W/INSTR SH.	Replacement grips include Q-logo	2.95
1A,2A,3-6	107642	STRT BACKPOST LOW ASM	Low, 9-12" Back Ht	40.00
1A,2A,3-6	107643	STRT BACKPOST MED ASM	Med, 13-16" Back Height	40.00
1A,2A,3-6	107644	STRT BACKPOST TAL ASM	Tall, 17-20" Back Height	40.00
1B,2A-8	107645	INTGRL P/H LOW ASM	Low, 9-12" Back Ht	50.00
1B,2A-8	107646	INTGRL P/H MED ASM	Med, 13-16" Back Height	50.00
1B,2A-8	107647	INTGRL P/H TAL ASM	Tall, 17-20" Back Height	50.00
1C,2B,3,5,6	107648	FLD P/H FXD MED ASSM	Med, 12-15" Back Height	75.00

Pos.	Item Number	Description	Remarks	USD
1C,2B,3,5,6	107649	FLD P/H FXD TAL ASSM	Tall, 16-19" Back Height	75.00
1D,2A,3-6	112392	8 DEG BACKPOST MED ASM W/O PUSH HANDLE		40.00
1D,2A,3-6	112393	8 DEG BACKPOST TALL ASSM W/O PUSH HANDLE		40.00
1E,2A-8	112394	8 DEG INTGRL PUSH HANDLE MED ASSM		50.00
1E,2A-8	112395	8 DEG INTGRL PUSH HANDLE TALL ASSM		50.00

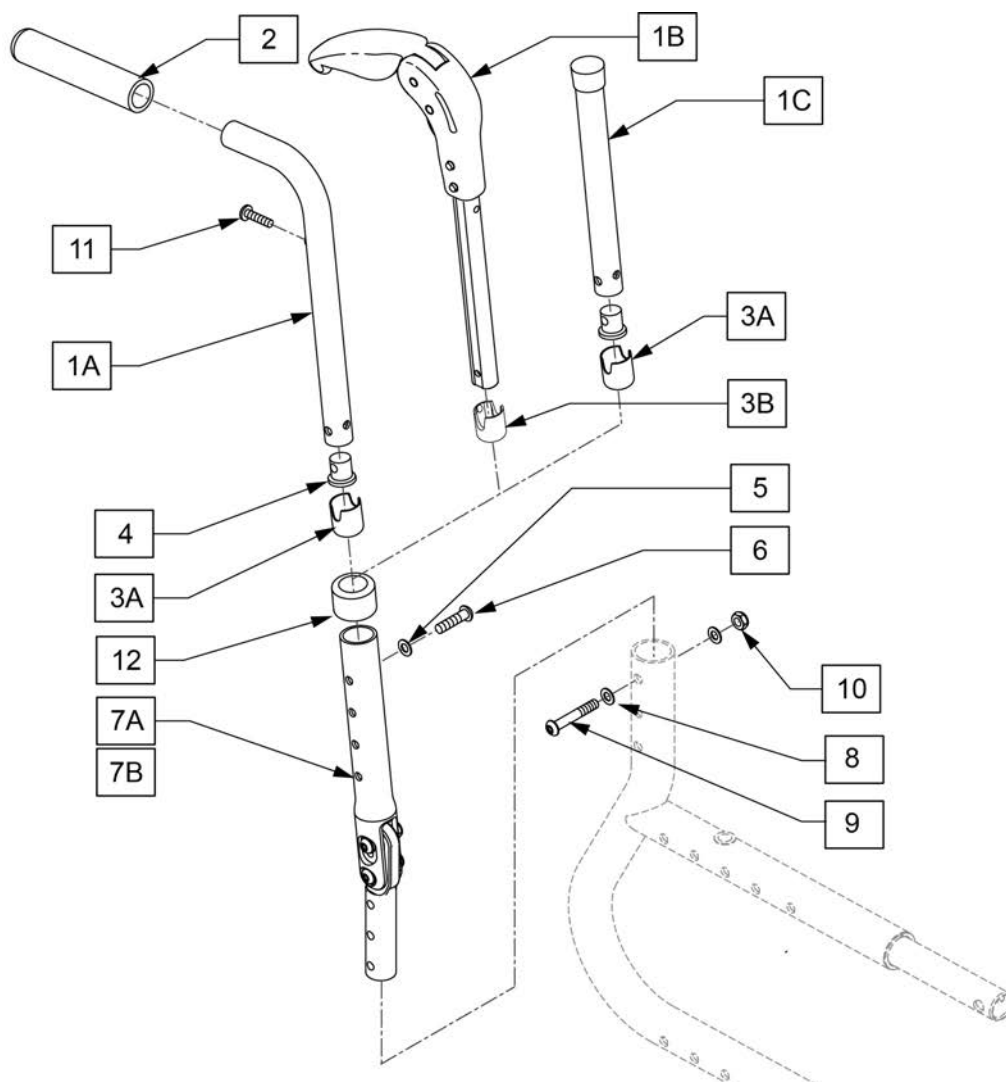


Pos.	Item Number	Description	Remarks	USD
1A,2	104093-001	NON-FOLDING PH ASSY SHORT	Low-See Assm for Back Ht	25.00
1A,2	104093-002	NON-FOLD PH ASSY MEDIUM	Med-See Assm for Back Ht	25.00
1B	882606	FLDG PSH HNDL W/INSRT MED	Med-See Assm for Back Ht	53.04
1C	104236-001	TUBE BACKREST STRAIGHT SHT.	Low-See Assm for Back Ht	20.00
1C	104236-002	TUBE BACKREST STRAIGHT MED.	Med-See Assm for Back Ht	20.00
3A	104187	BUSHING BCKRST NONFLDNG	Straight & Integral Push Handle	5.00
3B	104369	BUSHING BCKRST FOLDING HANDLE	Folding Push Handle Only	5.00
4	483001	Q2 FRAME/FOOT RETAINING PLUG		4.63
5	104414	WAVE WASHER 6 MM		5.00
6	108107	BHCS 1/4-20 X 3/4 ROHS		5.00
7A	104228-001	ANGLE ADJUST BACKREST SHT	3" PIVOT	35.00
7B	104228-003	ANGLE ADJUST BACKREST TLL	4" PIVOT	35.00
8	106943	WASHER FLAT AN 1/4 ROHS		1.00
9	106972	HHCS 1/4-28 X 1-1/2 ROHS		5.00
10	106935	NUT NYLOCK THIN 1/4-28 ROHS		5.00
11	107384	PAN HD PHIL 10/32 X 5/8 ROHS		5.00
12	561018	1" TUBE END BUSHING THICK		2.00
1C,3A-6,7A	107650	STRT 3" ANG ADJ LOW ASM	3" PIVOT - Low, 13-16" Back Height	65.00

Pos.	Item Number	Description	Remarks	USD
1C,3A-6,7A	107651	STRT 3" ANG ADJ MED ASM	3" PIVOT - Med, 17-20" Back Height	65.00
1A-6,7A,11	107652	INTGRL P/H 3" ANG ADJ LOW ASM	3" PIVOT - Low, 13-16" Back Height	75.00
1A-6,7A,11	107653	INTGRL P/H 3" ANG ADJ MED ASM	3" PIVOT - Med, 17-20" Back Height	75.00
1B,3B-6,7A	107654	FLD P/H 3" ANG ADJ MED ASSM	3" PIVOT - Med, 16-19" Back Height	100.00
1C,3A-6,7B	107655	STRT 4" ANG ADJ LOW ASM	4" PIVOT - Low, 14-17" Back Ht	65.00
1C,3A-6,7B	107656	STRT 4" ANG ADJ MED ASM	4" PIVOT - Med, 18-21" Back Ht	65.00
1A-6,7B,11	107657	INTGRL P/H 4" ANG ADJ LOW ASM	4" PIVOT - Low, 14-17" Back Ht	75.00
1A-6,7B,11	107658	INTGRL P/H 4 " ANG ADJ MED ASM	4" PIVOT - Med, 18-21" Back Ht	75.00
1B,3B-6,7B	107659	FLD P/H 4" ANG ADJ MED ASSM	4" PIVOT - Med, 17-20" Back Ht	100.00

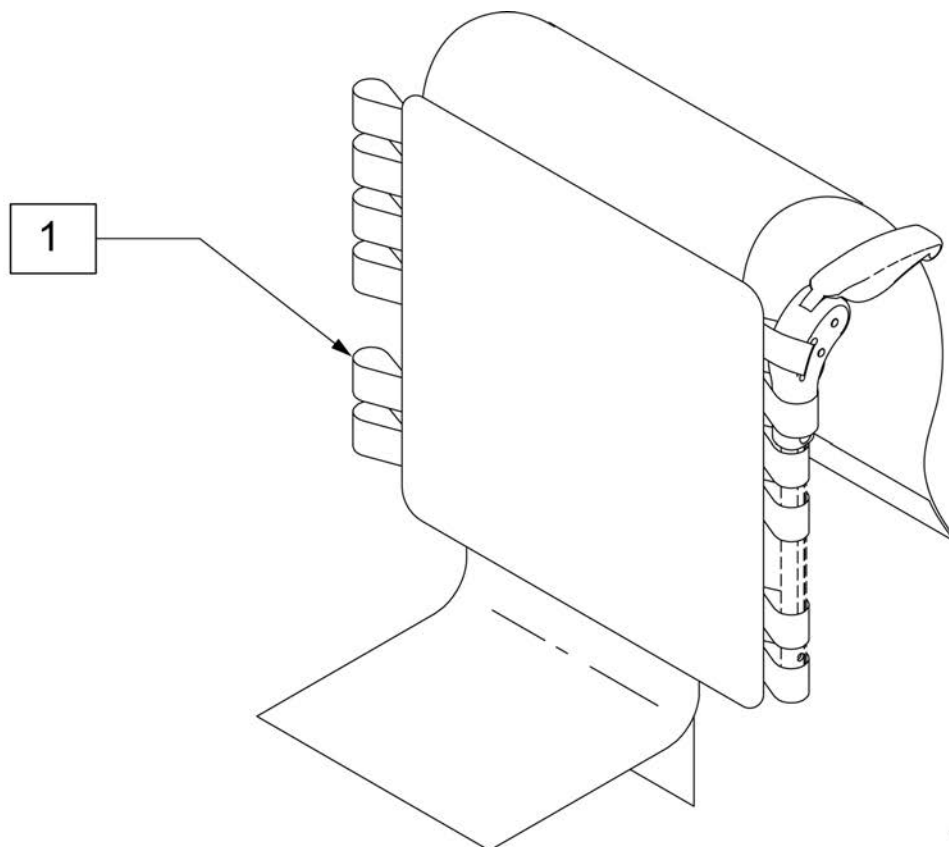

Pos.	Item Number	Description	Remarks	USD
1	687468	Upholstery, FPH, Adj, CE, 14W X 12H	Med Std Back Frame	175.00
1	687471	Upholstery, FPH, Adj, CE, 14W X 16H	Tall Std Back Frame/Med Ang Adj Back Frame	175.00
1	687474	Upholstery, FPH, Adj, CE, 15W X 12H	Med Std Back Frame	220.00
1	687477	Upholstery, FPH, Adj, CE, 15W X 16H	Tall Std Back Frame/Med Ang Adj Back Frame	175.00
1	687480	Upholstery, FPH, Adj, CE, 16W X 12H	Med Std Back Frame	260.00
1	687483	Upholstery, FPH, Adj, CE, 16W X 16H	Tall Std Back Frame/Med Ang Adj Back Frame	175.00
1	687486	Upholstery, FPH, Adj, CE, 17W X 12H	Med Std Back Frame	175.00
1	687489	Upholstery, FPH, Adj, CE, 17W X 16H	Tall Std Back Frame/Med Ang Adj Back Frame	175.00
1	687492	Upholstery, FPH, Adj, CE, 18W X 12H	Med Std Back Frame	175.00
1	687495	Upholstery, FPH, Adj, CE, 18W X 16H	Tall Std Back Frame/Med Ang Adj Back Frame	220.00
1	687541	Upholstery, FPH, Adj, CE, 19W X 12H	Med Std Back Frame	175.00
1	687543	Upholstery, FPH, Adj, CE, 19W X 16H	Tall Std Back Frame/Med Ang Adj Back Frame	220.00
1	687545	Upholstery, FPH, Adj, CE, 20W X 12H	Med Std Back Frame	175.00
1	687547	Upholstery, FPH, Adj, CE, 20W X 16H	Tall Std Back Frame/Med Ang Adj Back Frame	175.00

Note: For complete back upholstery order both Item 1 Back Upholstery & Item 2 Back Strap Cover.

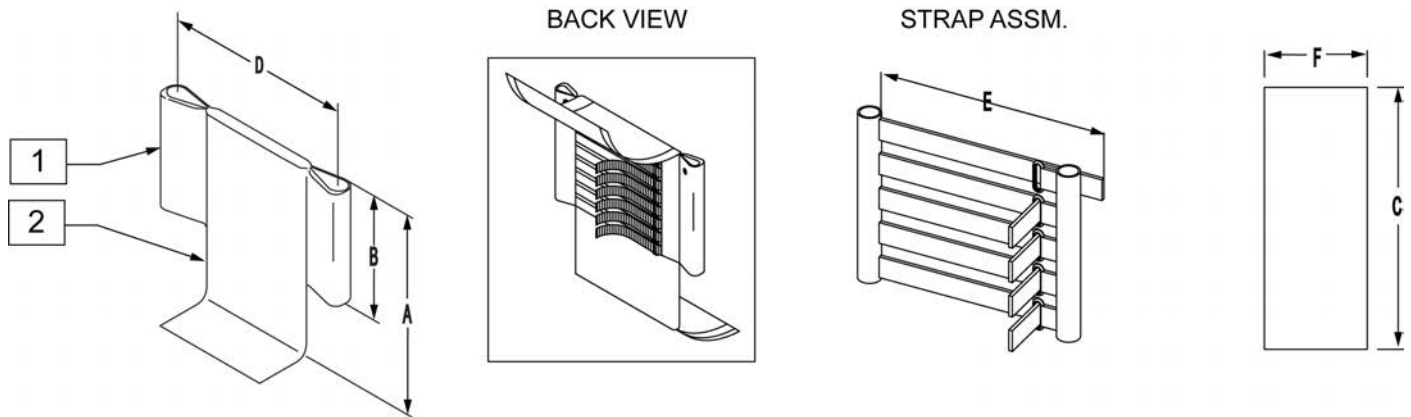


ADJUSTABLE BACK UPHOLSTERY MEASURING GUIDE


HEIGHT TABLE		STRAP ASSM.		PAD ASSM.
	BACK HEIGHT "A"	NO. OF STRAPS	HEIGHT "B"	LENGTH "C"
SHORT/ LOW	11-14"	5	6 1/2"	25"
MED	12-16"	6	8 1/2"	29"
TALL	14-18"	8	10 1/2"	33"
X-TALL	16-20"	9	12 1/2"	37"

HEIGHT TABLE		WIDTH "D"					
		SMALL 10-12"	MED 12-14"	LARGE 14-16"	X-LARGE 16-18"	X-LARGE 18-20"	X-LARGE 20-22"
STRAP ASSM	STRAP LENGTH "E"	9 1/2"	13 1/2"	17 1/2"	17 1/2"	21 1/2"	21 1/2"
PAD ASSM	WIDTH "F"	8"	10"	12"	14"	16"	18"

Pos.	Item Number	Description	Remarks	USD
1	680236	GROWING ADJ BACK UPHOLSTERY 14-16 SHORT ZGS	14-15 wide, Low Std Back	60.00
1	680238	GROWING ADJ BACK UPHOLSTERY 14-16 TALL ZGS	14-15 wide, Med Std Back/Low Ang Adj Back	60.00
1	680222	GROWING ADJ BACK UPHOLSTERY 14-16 X-TALL ZGS	14-15 wide, Tall Std Back/Med Ang Adj Back	125.00
1	680239	GROWING ADJ BACK UPHOLSTERY 16-18 SHORT ZGS	16-17 wide, Low Std Back	60.00
1	680241	GROWING ADJ BACK UPHOLSTERY 16-18 TALL ZGS	16-17 wide, Med Std Back/Low Ang Adj Back	60.00
1	680242	GROWING ADJ BACK UPHOLSTERY 16-18 X-TALL ZGS	16-17 wide, Tall Std Back/Med Ang Adj Back	60.00
1	680243	GROWING ADJ BACK UPHOLSTERY 18-22 SHORT ZGS	18-20 wide, Low Std Back	60.00
1	680245	GROWING ADJ BACK UPHOLSTERY 18-22 TALL ZGS	18-20 wide, Med Std Back/Low Ang Adj Back	60.00
1	680246	GROWING ADJ BACK UPHOLSTERY 18-22 X-TALL ZGS	18-20 wide, Tall Std Back/Med Ang Adj Back	60.00
2	684486	ADJ BCK STRP CVR 14-16 SHT ZGS	14-15 wide, Low Std Back	65.00
2	684488	ADJUSTABLE BACK STRAP COVER 14-16 TALL ZGS	14-15 wide, Med Std Back/Low Ang Adj Back	65.00
2	684499	ADJ BCK STRP CVR 14-16 XTL ZGS	14-15 wide, Tall Std Back/Med Ang Adj Back	65.00
2	684489	ADJUSTABLE BACK STRAP COVER 16-18 SHORT ZGS	16-17 wide, Low Std Back	65.00
2	684491	ADJUSTABLE BACK STRAP COVER 16-18 TALL ZGS	16-17 wide, Med Std Back/Low Ang Adj Back	65.00
2	684492	ADJUSTABLE BACK STRAP COVER 16-18 X-TALL ZGS	16-17 wide, Tall Std Back/Med Ang Adj Back	65.00
2	684493	ADJUSTABLE BACK STRAP COVER 18-20 SHORT ZGS	18-20 wide, Low Std Back	65.00
2	684495	ADJUSTABLE BACK STRAP COVER 18-20 TALL ZGS	18-20 wide, Med Std Back/Low Ang Adj Back	65.00
2	684496	ADJUSTABLE BACK STRAP COVER 18-20 X-TALL ZGS	18-20 wide, Tall Std Back/Med Ang Adj Back	65.00

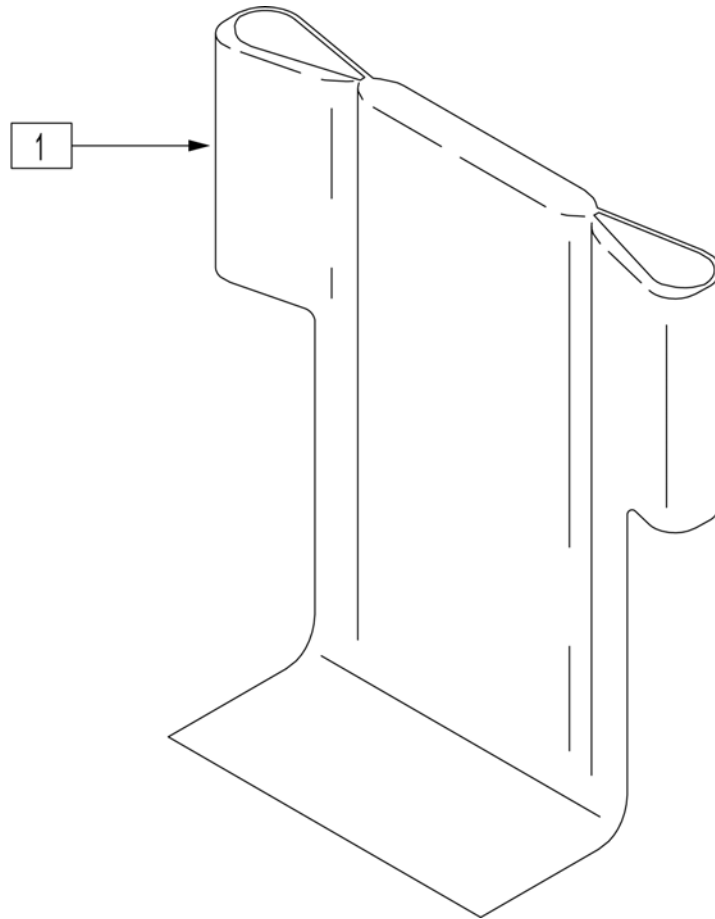


Pos.	Item Number	Description	Remarks	USD
1	680454	BACK UPH 14" 11-14/12-16 RDG	14" Wide - Low Back Height	110.00
1	680455	BACK UPH 15" 11-14/12-16 RDG	15" Wide - Low Back Height	112.20
1	680456	BACK UPH 16" 11-14/12-16 RDG	16" Wide - Low Back Height	112.20
1	680457	BACK UPH 17" 11-14/12-16 RDG	17" Wide - Low Back Height	112.20
1	680458	BACK UPH 18" 11-14/12-16 RDG	18" Wide - Low Back Height	112.20
1	680459	BACK UPH 19" 11-14/12-16 RDG	19" Wide - Low Back Height	112.20
1	680460	BACK UPH 20" 11-14/12-16 RDG	20" Wide - Low Back Height	112.20
1	680464	BACK UPH 14" 14-18 RDG	14" Wide - Med STD Back Ht/ Low Angle Adg Back	112.20
1	680465	BACK UPH 15" 14-18 RDG	15" Wide - Med STD Back Ht/ Low Angle Adg Back	112.20
1	680466	BACK UPH 16" 14-18 RDG	16" Wide - Med STD Back Ht/ Low Angle Adg Back	112.20
1	680467	BACK UPH 17" 14-18 RDG	17" Wide - Med STD Back Ht/ Low Angle Adg Back	112.20
1	680468	BACK UPH 18" 14-18 RDG	18" Wide - Med STD Back Ht/ Low Angle Adg Back	112.20
1	680469	BACK UPH 19" 14-18 RDG	19" Wide - Med STD Back Ht/ Low Angle Adg Back	110.00
1	680470	BACK UPH 20" 14-18 RDG	20" Wide - Med STD Back Ht/ Low Angle Adg Back	112.20
1	680474	BACK UPH 14" 16-20 RDG	14" Wide - Tall STD Back Ht/ Med Angle Adg Back	110.00
1	680475	BACK UPH 15" 16-20 RDG	15" Wide - Tall STD Back Ht/ Med Angle Adg Back	112.20
1	680476	BACK UPH 16" 16-20 RDG	16" Wide - Tall STD Back Ht/ Med Angle Adg Back	112.20
1	680477	BACK UPH 17" 16-20 RDG	17" Wide - Tall STD Back Ht/ Med Angle Adg Back	112.20
1	680478	BACK UPH 18" 16-20 RDG	18" Wide - Tall STD Back Ht/ Med Angle Adg Back	112.20
1	680479	BACK UPH 19" 16-20 RDG	19" Wide - Tall STD Back Ht/ Med Angle Adg Back	112.20
1	680480	BACK UPH 20" 16-20 RDG	20" Wide - Tall STD Back Ht/ Med Angle Adg Back	112.20

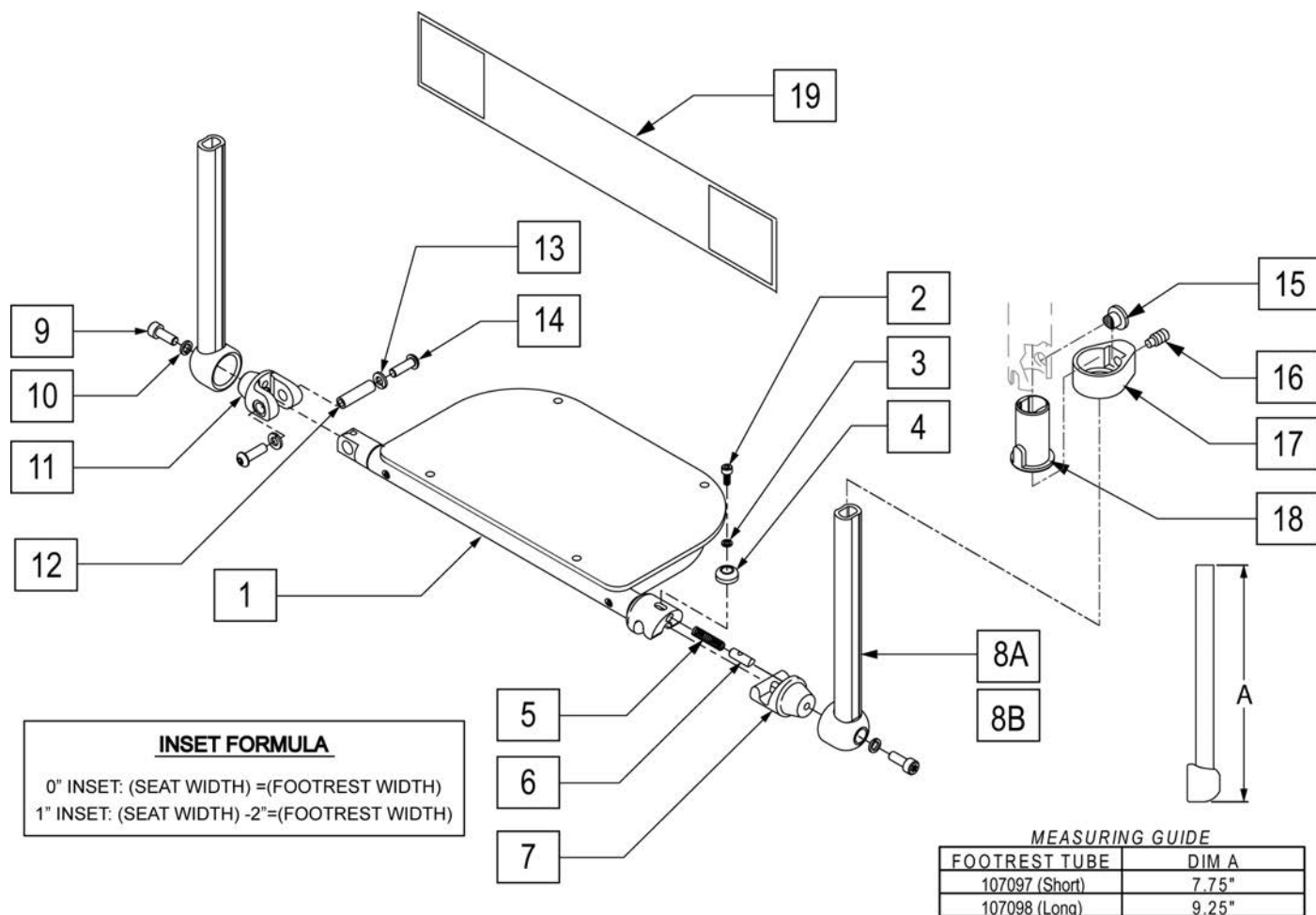


Pos.	Item Number	Description	Remarks	USD
2	230132	SHCS M4 X .7 X 10		10.00
3	050129-002	LOCKWASHER HIGH COLLAR M4		2.00
4	052289	BUTTON LEVER LATCH		7.95
5	100837	SPRING COMP .25OD X 1.0LG		10.00
6	052103	LATCH PLUNGER FOOTREST		16.95
7	100842	ASSY LATCH PIVOT FOOTREST		75.95
8A	107097	WELDMENT TUBE FOOTREST STD	Short	35.00
8B	107098	WELDMENT TUBE FOOTREST LNG	Long	35.00
9	102263-001	SHCS M6X1X16 FULL NP		2.00
10	107362	LOCKWASHER HIGH COLLAR M6		5.00
11	052022	CLEVIS PIVOT TUBE		24.95
12	052282	PIN PIVOT FOOTREST		24.95
13	262133	WASHER LOCK M6		2.00
14	013147	BHCS M6X1X12MM BLK		2.00
18	104160	BUSHING FOOTREST		5.00
19	740130	LEG STRAP 10"-14" BLK		30.60
19	740131	LEG STRAP 15"-17" BLK		30.60
19	740132	LEG STRAP 18"-20" BLK		30.60
1-6	100830-010	ASSY TUBE FRST TBLR W-CVR 12W	12 wide	300.95
1-6	100830-011	ASSY TUBE FRST TBLR W-CVR 13W	13 wide	300.95
1-6	100830-012	ASSY TUBE FRST TBLR W-CVR 14W	14 wide	306.95

Pos.	Item Number	Description	Remarks	USD
1-6	100830-013	ASSY TUBE FRST TBLR W-CVR 15W	15 wide	306.95
1-6	100830-014	ASSY TUBE FRST TBLR W-CVR 16W	16 wide	306.95
1-6	100830-015	ASSY TUBE FRST TBLR W-CVR 17W	17 wide	306.95
1-6	100830-016	ASSY TUBE FRST TBLR W-CVR 18W	18 wide	315.95
1-6	100830-017	ASSY TUBE FRST TBLR W-CVR 19W	19 wide	315.95
1-6	100830-018	ASSY TUBE FRST TBLR W-CVR 20W	20 wide	315.95
1-8A,9-14,19	107686	ASSM 12W SHORT	12 wide (Short)	350.00
1-8A,9-14,19	107687	ASSM 13W SHORT	13 wide (Short)	350.00
1-8A,9-14,19	107688	ASSM 14W SHORT	14 wide (Short)	350.00
1-8A,9-14,19	107689	ASSM 15W SHORT	15 wide (Short)	350.00
1-8A,9-14,19	107690	ASSM 16W SHORT	16 wide (Short)	350.00
1-8A,9-14,19	107691	ASSM 17W SHORT	17 wide (Short)	350.00
1-8A,9-14,19	107692	ASSM 18W SHORT	18 wide (Short)	350.00
1-8A,9-14,19	107693	ASSM 19W SHORT	19 wide (Short)	350.00
1-8A,9-14,19	107694	ASSM 20W SHORT	20 wide (Short)	350.00
1-8B,9-14,19	107695	ASSM 12W LONG	12 wide (Long)	350.00
1-8B,9-14,19	107696	ASSM 13W LONG	13 wide (Long)	350.00
1-8B,9-14,19	107697	ASSM 14W LONG	14 wide (Long)	350.00
1-8B,9-14,19	107698	ASSM 15W LONG	15 wide (Long)	350.00
1-8B,9-14,19	107699	ASSM 16W LONG	16 wide (Long)	350.00
1-8B,9-14,19	107700	ASSM 17W LONG	17 wide (Long)	350.00
1-8B,9-14,19	107701	ASSM 18W LONG	18 wide (Long)	350.00
1-8B,9-14,19	107702	ASSM 19W LONG	19 wide (Long)	350.00
1-8B,9-14,19	107703	ASSM 20W SHORT	20 wide (Long)	350.00
15-18	107485	FOOTREST CLAMP ASSM		30.00

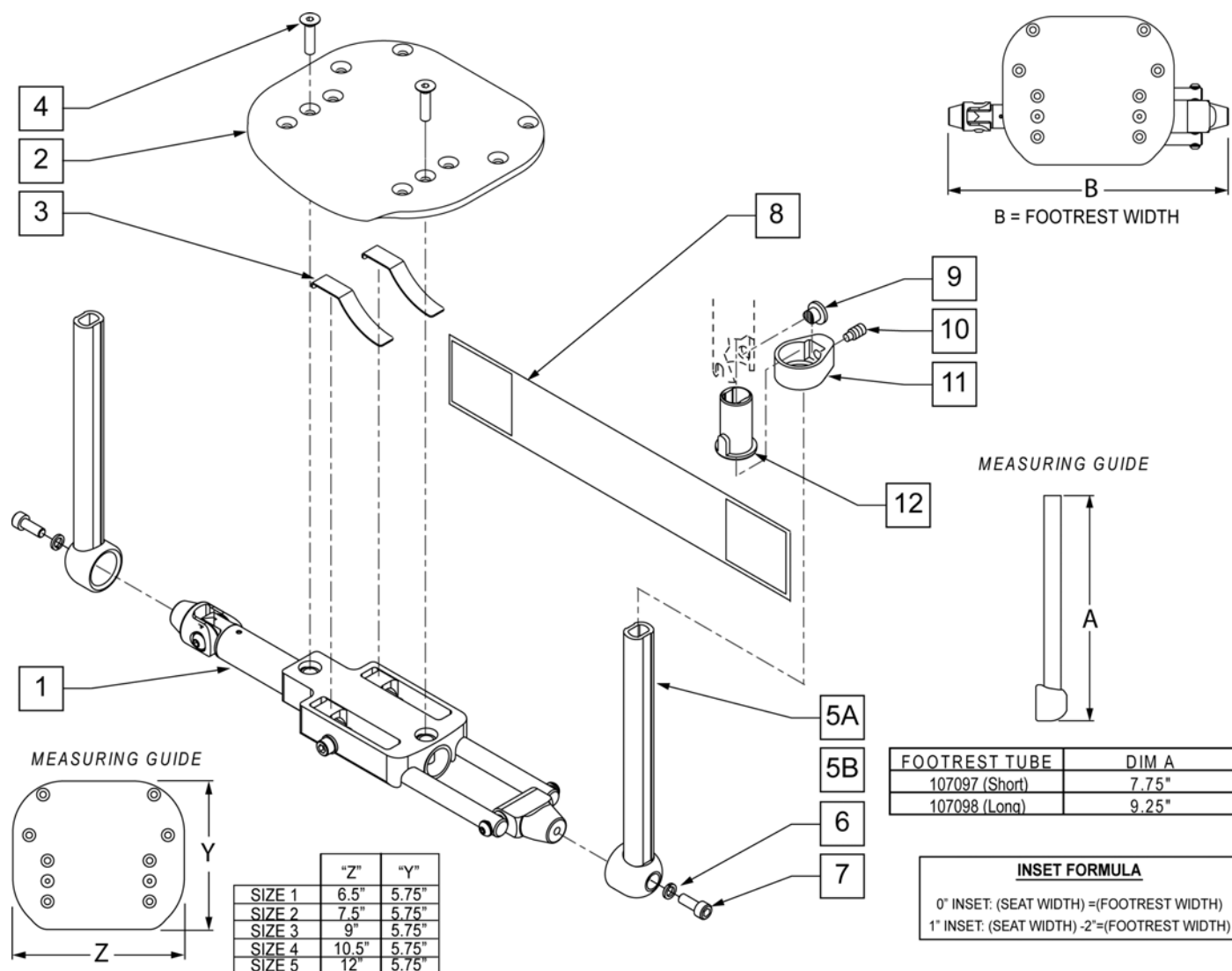

Pos.	Item Number	Description	Remarks	USD
2	054853-001	FOOTPLATE SIZE 1	12" wide	40.00
2	054853-002	FOOTPLATE SIZE 2	13-14" wide	40.00
2	054853-003	FOOTPLATE SIZE 3	15-16" wide	40.00
2	054853-004	FOOTPLATE SIZE 4	17-18" wide	40.00
2	054853-005	FOOTPLATE SIZE 5	19-20" wide	40.00
3	054852	SPRING LEAF FOOTREST		
4	102265-003	FHCS M6X1X25 FULL NP		2.50
5A	107097	WELDMENT TUBE FOOTREST STD	Short	35.00
5B	107098	WELDMENT TUBE FOOTREST LNG	Long	35.00
6	107362	LOCKWASHER HIGH COLLAR M6		5.00
7	102263-001	SHCS M6X1X16 FULL NP		2.00
8	740130	LEG STRAP 10"-14" BLK	12-14" wide	30.60
8	740131	LEG STRAP 15"-17" BLK	15-17" wide	30.60
8	740132	LEG STRAP 18"-20" BLK	18-20" wide	30.60
12	104160	BUSHING FOOTREST		5.00
1-4	102830-001	ASSY FOOTPLATE AUTOFOLD 12W	12 wide	425.00
1-4	102830-002	ASSY FOOTPLATE AUTOFOLD 13W	13 wide	425.00

Pos.	Item Number	Description	Remarks	USD
1-4	102830-003	ASSY FOOTPLATE AUTOFOLD 14W	14 wide	425.00
1-4	102830-004	ASSY FOOTPLATE AUTOFOLD 15W	15 wide	425.00
1-4	102830-005	ASSY FOOTPLATE AUTOFOLD 16W	16 wide	425.00
1-4	102830-006	ASSY FOOTPLATE AUTOFOLD 17W	17 wide	425.00
1-4	102830-007	ASSY FOOTPLATE AUTOFOLD 18W	18 wide	425.00
1-4	102830-008	ASSY FOOTPLATE AUTOFOLD 19W	19 wide	425.00
1-4	102830-009	ASSY FOOTPLATE AUTOFOLD 20W	20 wide	425.00
1-5A,6-8	107597	ASSM 12W SHORT	12 wide (Short)	500.00
1-5A,6-8	107598	ASSM 13W SHORT	13 wide (Short)	500.00
1-5A,6-8	107599	ASSM 14W SHORT	14 wide (Short)	500.00
1-5A,6-8	107600	ASSM 15W SHORT	15 wide (Short)	500.00
1-5A,6-8	107601	ASSM 16W SHORT	16 wide (Short)	500.00
1-5A,6-8	107602	ASSM 17W SHORT	17 wide (Short)	500.00
1-5A,6-8	107603	ASSM 18W SHORT	18 wide (Short)	500.00
1-5A,6-8	107604	ASSM 19W SHORT	19 wide (Short)	500.00
1-5A,6-8	107605	ASSM 20W SHORT	20 wide (Short)	500.00
1-5B,6-8	107606	ASSM 12W LONG	12 wide (Long)	500.00
1-5B,6-8	107607	ASSM 13W LONG	13 wide (Long)	500.00
1-5B,6-8	107608	ASSM 14W LONG	14 wide (Long)	500.00
1-5B,6-8	107609	ASSM 15W LONG	15 wide (Long)	500.00
1-5B,6-8	107610	ASSM 16W LONG	16 wide (Long)	500.00
1-5B,6-8	107611	ASSM 17W LONG	17 wide (Long)	500.00
1-5B,6-8	107612	ASSM 18W LONG	18 wide (Long)	500.00
1-5B,6-8	107613	ASSM 19W LONG	19 wide (Long)	500.00
1-5B,6-8	107614	ASSM 20W SHORT	20 wide (Long)	500.00
9-12	107485	FOOTREST CLAMP ASSM		30.00

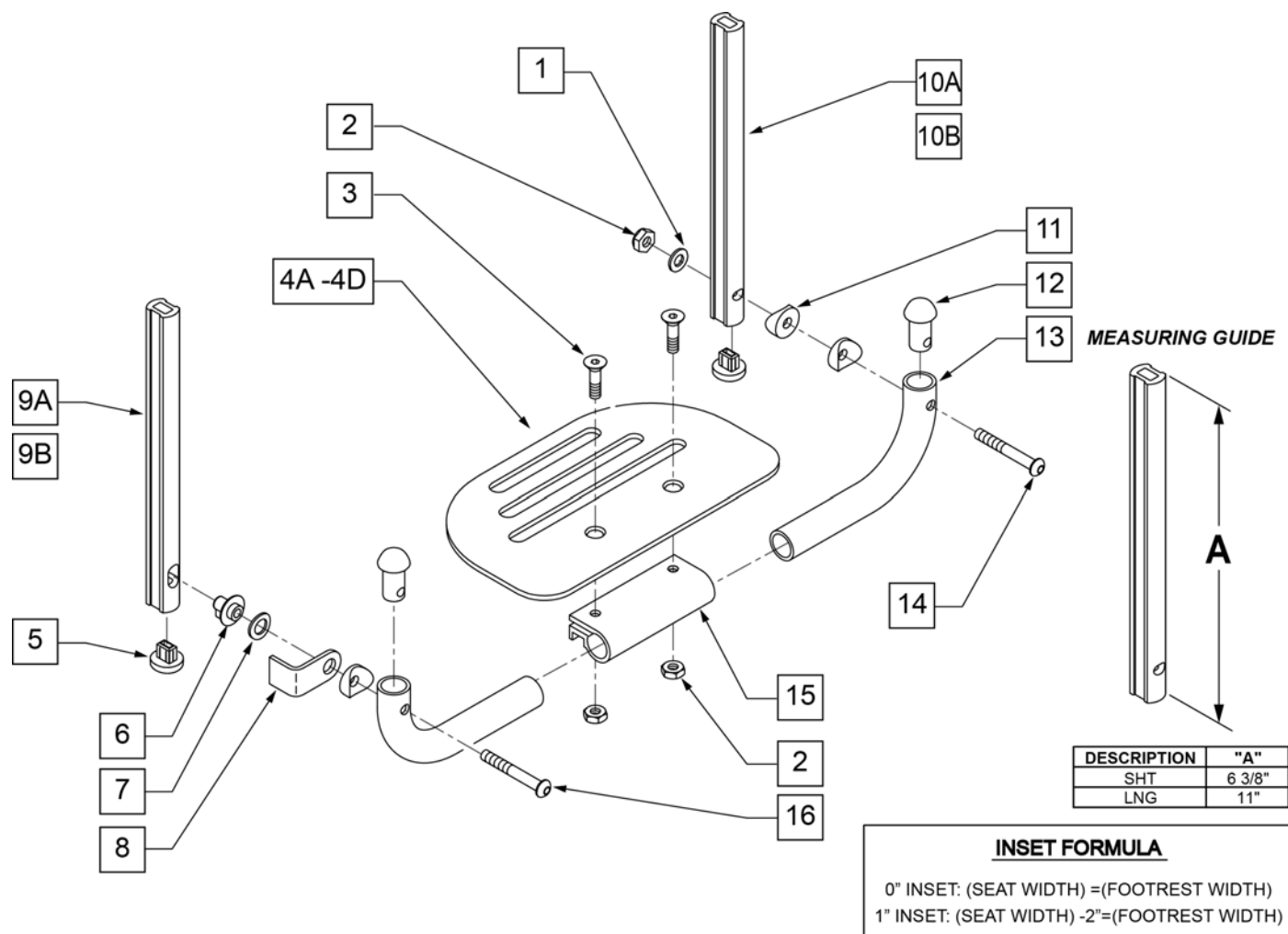

Pos.	Item Number	Description	Remarks	USD
1	106944	WASHER FLAT THIN AN 1/4 ROHS		1.00
2	106941	NUT AUTOMATION 1/4-28 ROHS		5.00
3	106978	FHCS 1/4-28 X 7/8 ROHS		5.00
4A	500651	PLATFORM F/PLT 8 x 5 1/2"	12 -14" Footrest Width	56.81
4B	500652	PLATFORM F/PLT 10" x 5 1/2"	15 -17" Footrest Width	56.81
4C	500653	PLATFORM F/PLT 12" x 5 1/2"	18 -20" Footrest Width	56.81
4D	500654	PLATFORM F/PLT 14" x 5 1/2"	21 -22" Footrest Width	62.73
5	113221	PLUG DOUBLE D EXT TUBE		4.00
6	540410	F/UP FOOTREST KEY STUD D2		6.40
7	107385	WASHER CURVED 3/8 ROHS		5.00
8	505078	FOOTREST RETAINING BRACKET		4.19
9A	106875	EXT TBE F/UP KEY SHT	Short -Latch Side	7.50
9B	106876	EXT TBE F/UP KEY LNG	Long -Latch Side	7.50
10A	106873	EXT TBE F/UP SLT SHT	Short -Pivot Side	7.50
10B	106874	EXT TBE F/UP SLT LNG	Long -Pivot Side	7.50
11	561003	3/4" SADDLE THIN		2.00
12	561034	SHORT DOME PLUG		4.00
14	106964	BHCS 1/4-28 X 2 ROHS		5.00
15	550351	PLATFORM FOOTPLATE CLAMP		25.38

Pos.	Item Number	Description	Remarks	USD
16	107027	BHCS 1/4-28 X 1-1/4 SHLDR ROHS		5.00
17A	740130	LEG STRAP 10"-14" BLK	12-14" Footrest Width	30.60
17B	740131	LEG STRAP 15"-17" BLK	15-17" Footrest Width	30.60
17C	740132	LEG STRAP 18"-20" BLK	18-20" Footrest Width	30.60
1-4A,6-8,11-16	900306	PLTFM F/P PLT BLK 12" D2 W/ INSTR	12" Footrest Width	95.00
1-4A,6-8,11-16	900307	PLTFM F/P PLT BLK 13" D2 W/ INSTR	13" Footrest Width	95.00
1-4A,6-8,11-16	900308	PLTFM F/P PLT BLK 14" D2 W /INSTR	14" Footrest Width	95.00
1-4B,6-8,11-16	900309	PLTFM F/P PLT BLK 15" D2 W /INSTR	15" Footrest Width	95.00
1-4B,6-8,11-16	900310	PLTFM F/P PLT BLK 16" D2 W /INSTR	16" Footrest Width	95.00
1-4B,6-8,11-16	900311	PLTFM F/P PLT BLK 17" D2 W /INSTR	17" Footrest Width	95.00
1-4C,6-8,11-16	900312	PLTFM F/P PLT BLK 18" D2 W /INSTR	18" Footrest Width	95.00
1-4C,6-8,11-16	900313	PLTFM F/P PLT BLK 19" D2 W /INSTR	19" Footrest Width	95.00
1-4C,6-8,11-16	900314	PLTFM F/P PLT BLK 20" D2 W/ INSTR	20" Footrest Width	95.00
1-4D,6-8,11-16	900315	PLTFM F/P PLT BLK 21" D2 W/ INSTR	21" Footrest Width	95.00
1-4D,6-8,11-16	900316	PLTFM F/P PLT BLK 22" D2 W/ INSTR	22" Footrest Width	95.00
		SHORT EXT ASSEMBLIES		
1-3,4A,5-8,9A,10B,11-16	108043	PLTFRM FLIP FTRST SHT 12W	12" wide (Short)	100.00
1-3,4A,5-8,9A,10B,11-16	108044	PLTFRM FLIP FTRST SHT 13W	13" wide (Short)	100.00
1-3,4A,5-8,9A,10B,11-16	108045	PLTFRM FLIP FTRST SHT 14W	14" wide (Short)	100.00
1-3,4B,5-8,9A,10B,11-16	108046	PLTFRM FLIP FTRST SHT 15W	15" wide (Short)	100.00
1-3,4B,5-8,9A,10B,11-16	108047	PLTFRM FLIP FTRST SHT 16W	16" wide (Short)	100.00
1-3,4B,5-8,9A,10B,11-16	108048	PLTFRM FLIP FTRST SHT 17W	17" wide (Short)	100.00
1-3,4C,5-8,9A,10B,11-16	108049	PLTFRM FLIP FTRST SHT 18W	18" wide (Short)	100.00
1-3,4C,5-8,9A,10B,11-16	108051	PLTFRM FLIP FTRST SHT 19W	19" wide (Short)	100.00
1-3,4C,5-8,9A,10B,11-16	108052	PLTFRM FLIP FTRST SHT 20W	20" wide (Short)	100.00
1-3,4D,5-8,9A,10B,11-16	111554	PLTFRM FLIP FTRST SHT 21W	21" wide (Short)	100.00
1-3,4D,5-8,9A,10B,11-16	111555	PLTFRM FLIP FTRST SHT 22W	22" wide (Short)	100.00
		LONG EXT ASSEMBLIES		
1-3,4A,5-8,9B,10A,11-16	108053	PLTFRM FLIP FTRST LONG 12W	12" wide (Long)	100.00
1-3,4A,5-8,9B,10A,11-16	108054	PLTFRM FLIP FTRST LONG 13W	13" wide (Long)	100.00
1-3,4A,5-8,9B,10A,11-16	108055	PLTFRM FLIP FTRST LONG 14W	14" wide (Long)	100.00
1-3,4B,5-8,9B,10A,11-16	108056	PLTFRM FLIP FTRST LONG 15W	15" wide (Long)	100.00
1-3,4B,5-8,9B,10A,11-16	108057	PLTFRM FLIP FTRST LONG 16W	16" wide (Long)	100.00
1-3,4B,5-8,9B,10A,11-16	108058	PLTFRM FLIP FTRST LONG 17W	17" wide (Long)	100.00
1-3,4C,5-8,9B,10A,11-16	108059	PLTFRM FLIP FTRST LONG 18W	18" wide (Long)	100.00
1-3,4C,5-8,9B,10A,11-16	108060	PLTFRM FLIP FTRST LONG 19W	19" wide (Long)	100.00
1-3,4C,5-8,9B,10A,11-16	108061	PLTFRM FLIP FTRST LONG 20W	20" wide (Long)	100.00
1-3,4D,5-8,9B,10A,11-16	111556	PLTFRM FLIP FTRST LONG 21W	21" wide (Long)	100.00
1-3,4D,5-8,9B,10A,11-16	111557	PLTFRM FLIP FTRST LONG 22W	22" wide (Long)	100.00

NOTE: FOOTPLATE ASSEMBLIES ARE SET TO FOOTREST WIDTH. PLEASE SEE "FOOTPLATE FORMULA" TABLE FOR SEAT WIDTH CONVERSION.

INSET FORMULA
 0" Inset (Seat Width) = Footrest Width
 1" Inset (Seat Width) -2 = Footrest Width
 "V" Hanger (Seat Width) -4 = Footrest Width

Composite Footplates w/ Ext & HL for Swing I/O Hangers
 (Order 1 each)

Width	60°, 70° & 70° Hemi Style Hangers (Bent Extension)			80° Style hanger ELR (Straight Extension)			70° V Style Hangers	
	Short	Medium	Long	Short	Medium	Long	Medium	Long
13" Right	107746	107760	107774	107704	107718	107732	107788	107802
13" Left	107753	107767	107781	107711	107725	107739	107795	107809
14" Right	107747	107761	107775	107705	107719	107733	107789	107803
14" Left	107754	107768	107782	107712	107726	107740	107796	107810
15" Right	107748	107762	107776	107706	107720	107734	107790	107804
15" Left	107755	107769	107783	107713	107727	107741	107797	107811
16" Right	107749	107763	107777	107707	107721	107735	107791	107805
16" Left	107756	107770	107784	107714	107728	107742	107798	107812
17" Right	107750	107764	107778	107708	107722	107736	107792	107806
17" Left	107757	107771	107785	107715	107729	107743	107799	107813
18" Right	107751	107765	107779	107709	107723	107737	107793	107807
18" Left	107758	107772	107786	107716	107730	107744	107800	107814
19"-20" Rt	107752	107766	107780	107710	107724	107738	V STYLE IS NOT AVAILABLE IN THESE SEAT WIDTHS	
19"-20" Lt	107759	107773	107787	107717	107731	107745		
21"-22" Rt	111528	111530	111532	111522	111524	111526		
21"-22" Lt	111529	111531	111533	111523	111525	111527		

FOOTPLATE MEASURING GUIDE

PART NO	DESCRIPTION	DIM -A-	DIM -B-
063216-006	FOOTPLATE COMPOSITE 157 R	6 1/8"	6"
063216-012	FOOTPLATE COMPOSITE 157 L	6 1/8"	6"
063216-015	FOOTPLATE COMPOSITE 170 R	6 5/8"	6"
063216-018	FOOTPLATE COMPOSITE 170 L	6 5/8"	6"
063216-005	FOOTPLATE COMPOSITE 182 R	7 1/8"	6"
063216-011	FOOTPLATE COMPOSITE 182 L	7 1/8"	6"
063216-014	FOOTPLATE COMPOSITE 195 R	7 5/8"	6"
063216-017	FOOTPLATE COMPOSITE 195 L	7 5/8"	6"
063216-004	FOOTPLATE COMPOSITE 207 R	8 1/8"	6"
063216-010	FOOTPLATE COMPOSITE 207 L	8 1/8"	6"
063216-013	FOOTPLATE COMPOSITE 220 R	8 5/8"	6 1/4"
063216-016	FOOTPLATE COMPOSITE 220 L	8 5/8"	6 1/4"
063216-003	FOOTPLATE COMPOSITE 232 R	9 1/8"	6 1/4"
063216-009	FOOTPLATE COMPOSITE 232 L	9 1/8"	6 1/4"
063216-002	FOOTPLATE COMPOSITE 257 R	10 1/8"	6 1/4"
063216-008	FOOTPLATE COMPOSITE 257 L	10 1/8"	6 1/4"
063216-001	FOOTPLATE COMPOSITE 282 R	11 1/8"	6 1/4"
063216-007	FOOTPLATE COMPOSITE 282 L	11 1/8"	6 1/4"

Pos.	Item Number	Description	Remarks	USD
1A-5	120660	SHT STRT EXT COMP HDWR	80 deg hangers + ELR only	20.00
1B-5	120661	MED STRT EXT COMP HDWR	80 deg hangers + ELR only	20.00
1C-5	120662	LNG STRT EXT COMP HDWR	80 deg hangers + ELR only	20.00
1D-5	120663	SHT BNT EXT COMP HDWR	60,70,70 Hemi Hangers	20.00
1E-5	120664	MED BNT EXT COMP HDWR	60,70,70 Hemi Hangers	20.00
1F-5	120665	LNG BNT EXT COMP HDWR	60,70,70 Hemi Hangers	20.00
1G-5	120666	SHT V EXT R COMP HDWR	70 deg V hangers only-LEFT	20.00
1G-5	120667	SHT V EXT L COMP HDWR	70 deg V hangers only-RIGHT	20.00
1H-5	120668	LNG V EXT R COMP HDWR	70 deg V hangers only-LEFT	20.00
1H-5	120669	LNG V EXT L COMP HDWR	70 deg V hangers only-RIGHT	20.00
2A	063216-006	COMPOSITE FOOTPLATE RIGHT	RIGHT 13" Footrest Width	20.00
2A	063216-012	COMPOSITE FOOTPLATE LEFT	LEFT 13" Footrest Width	20.00
2B	063216-015	FOOTPLATE COMPOSITE RIGHT	RIGHT 14" Footrest Width	20.00

Pos.	Item Number	Description	Remarks	USD
2B	063216-018	FOOTPLATE COMPOSITE LEFT	LEFT 14" Footrest Width	20.00
2C	063216-005	COMPOSITE FOOTPLATE RIGHT	RIGHT 15" Footrest Width	20.00
2C	063216-011	COMPOSITE FOOTPLATE LEFT	LEFT 15" Footrest Width	20.00
2D	063216-014	FOOTPLATE COMPOSITE RIGHT	RIGHT 16" Footrest Width	20.00
2D	063216-017	FOOTPLATE COMPOSITE LEFT	LEFT 16" Footrest Width	20.00
2E	063216-004	COMPOSITE FOOTPLATE RIGHT	RIGHT 17" Footrest Width	20.00
2E	063216-010	COMPOSITE FOOTPLATE LEFT	LEFT 17" Footrest Width	20.00
2F	063216-013	FOOTPLATE COMPOSITE RIGHT	RIGHT 18" Footrest Width	20.00
2F	063216-016	FOOTPLATE COMPOSITE LEFT	LEFT 18" Footrest Width	20.00
2G	063216-003	COMPOSITE FOOTPLATE RIGHT	RIGHT 19" & 20" Footrest Width	20.00
2G	063216-009	COMPOSITE FOOTPLATE LEFT	LEFT 19" & 20" Footrest Width	20.00
2H	063216-002	FOOTPLATE COMPOSITE RIGHT	RIGHT 21" & 22" Footrest Width	20.00
2H	063216-008	FOOTPLATE COMPOSITE LEFT	LEFT 21" & 22" Footrest Width	20.00
3	104377	3/4" SADDLE THIN		5.00
4	103175	SHCS M6 X 1 X 40 INORG		5.00
5	104381	NUT HEX NYLOCK M6X1.0	Inserted in bottom of footplate.	5.00
6	104328	HEEL LOOP PEG		5.00
7A	104380-001	ADJUSTABLE FOOT REST STRAP	14" wide	7.50
7B	104380-002	ADJUSTABLE FOOT REST STRAP	15-16" wide	7.50
7C	104380-003	ADJUSTABLE FOOT REST STRAP	17-18" wide	7.50
7D	104380-004	ADJUSTABLE FOOT REST STRAP	19-22" wide	7.50
8	102268-001	HHCS M6 X 1 X 25		10.00
12	104160	BUSHING FOOTREST		5.00
9-12	107485	FOOTREST CLAMP ASSM		30.00
>>>>		60°,70°,70° HEMI HANGERS - SHORT EXT - RIGHT		
1D,2A,3-6,7A,8	107746	COMP BNT SHT EXT ASM 13" W R	Right- 70D- 13" W- SHORT	65.00
1D,2B,3-6,7A,8	107747	COMP BNT SHT EXT ASM 14" W R	Right- 70D- 14" W- SHORT	65.00
1D,2C,3-6,7B,8	107748	COMP BNT SHT EXT ASM 15" W R	Right- 70D- 15" W- SHORT	65.00
1D,2D,3-6,7B,8	107749	COMP BNT SHT EXT ASM 16" W R	Right- 70D- 16" W- SHORT	65.00
1D,2E,3-6,7C,8	107750	COMP BNT SHT EXT ASM 17" W R	Right- 70D- 17" W- SHORT	65.00
1D,2F,3-6,7C,8	107751	COMP BNT SHT EXT ASM 18" W R	Right- 70D- 18" W- SHORT	65.00
1D,2G,3-6,7D,8	107752	COMP BNT SHT EXT ASM 19 & 20"W R	Right- 70D- 19 & 20" W- SHORT	65.00
1D,2H,3-6,7D,8	111528	COMP BNT SHT EXT ASM 21 & 22"W R	Right- 70D- 21 & 22" W- SHORT	65.00
>>>>		60°,70°,70° HEMI HANGERS - SHORT EXT - LEFT		
1D,2A,3-6,7A,8	107753	COMP BNT SHT EXT ASM 13" W L	Left- 70D- 13 W- SHORT	65.00
1D,2B,3-6,7A,8	107754	COMP BNT SHT EXT ASM 14" W L	Left- 70D- 14 W- SHORT	65.00
1D,2C,3-6,7B,8	107755	COMP BNT SHT EXT ASM 15" W L	Left- 70D- 15 W- SHORT	65.00
1D,2D,3-6,7B,8	107756	COMP BNT SHT EXT ASM 16" W L	Left- 70D- 16 W- SHORT	65.00
1D,2E,3-6,7C,8	107757	COMP BNT SHT EXT ASM 17" W L	Left- 70D- 17 W- SHORT	65.00
1D,2F,3-6,7C,8	107758	COMP BNT SHT EXT ASM 18" W L	Left- 70D- 18 W- SHORT	65.00
1D,2G,3-6,7D,8	107759	COMP BNT SHT EXT ASM 19 & 20"W L	Left- 70D- 19 & 20" W- SHORT	65.00
1D,2H,3-6,7D,8	111529	COMP BNT SHT EXT ASM 21 & 22"W L	Left- 70D- 21 & 22" W- SHORT	65.00
>>>>		60°,70°,70° HEMI HANGERS - MED EXT - RIGHT		
1E,2A,3-6,7A,8	107760	COMP BNT MED EXT ASM 13" W R	Right- 70D- 13" W- MED	65.00
1E,2B,3-6,7A,8	107761	COMP BNT MED EXT ASM 14" W R	Right- 70D- 14" W- MED	65.00
1E,2C,3-6,7B,8	107762	COMP BNT MED EXT ASM 15" W R	Right- 70D- 15" W- MED	65.00
1E,2D,3-6,7B,8	107763	COMP BNT MED EXT ASM 16" W R	Right- 70D- 16" W- MED	65.00
1E,2E,3-6,7C,8	107764	COMP BNT MED EXT ASM 17" W R	Right- 70D- 17" W- MED	65.00
1E,2F,3-6,7C,8	107765	COMP BNT MED EXT ASM 18" W R	Right- 70D- 18" W- MED	65.00
1E,2G,3-6,7D,8	107766	COMP BNT MED EXT ASM 19 & 20"W R	Right- 70D- 19 & 20" W- MED	65.00
1E,2H,3-6,7D,8	111530	COMP BNT MED EXT ASM 21 & 22"W R	Right- 70D- 21 & 22" W- MED	65.00
>>>>		60°,70°,70° HEMI HANGERS - MED EXT - LEFT		
1E,2A,3-6,7A,8	107767	COMP BNT MED EXT ASM 13" W L	Left- 70D- 13 W- MED	65.00
1E,2B,3-6,7A,8	107768	COMP BNT MED EXT ASM 14" W L	Left- 70D- 14 W- MED	65.00
1E,2C,3-6,7B,8	107769	COMP BNT MED EXT ASM 15" W L	Left- 70D- 15 W- MED	65.00
1E,2D,3-6,7B,8	107770	COMP BNT MED EXT ASM 16" W L	Left- 70D- 16 W- MED	65.00

Pos.	Item Number	Description	Remarks	USD
1E,2E,3-6,7C,8	107771	COMP BNT MED EXT ASM 17" W L	Left- 70D- 17 W- MED	65.00
1E,2F,3-6,7C,8	107772	COMP BNT MED EXT ASM 18" W L	Left- 70D- 18 W- MED	65.00
1E,2G,3-6,7D,8	107773	COMP BNT MED EXT ASM 19 & 20"W L	Left- 70D- 19 & 20" W- MED	65.00
1E,2H,3-6,7D,8	111531	COMP BNT MED EXT ASM 21 & 22"W L	Left- 70D- 21 & 22" W- MED	65.00
>>>>		60°,70°,70° HEMI HANGERS - LONG EXT - RIGHT		
1F,2A,3-6,7A,8	107774	COMP BNT LONG EXT ASM 13" W R	Right- 70D- 13" W- LONG	65.00
1F,2B,3-6,7A,8	107775	COMP BNT LONG EXT ASM 14" W R	Right- 70D- 14" W- LONG	65.00
1F,2C,3-6,7B,8	107776	COMP BNT LONG EXT ASM 15" W R	Right- 70D- 15" W- LONG	65.00
1F,2D,3-6,7B,8	107777	COMP BNT LONG EXT ASM 16" W R	Right- 70D- 16" W- LONG	65.00
1F,2E,3-6,7C,8	107778	COMP BNT LONG EXT ASM 17" W R	Right- 70D- 17" W- LONG	65.00
1F,2F,3-6,7C,8	107779	COMP BNT LONG EXT ASM 18" W R	Right- 70D- 18" W- LONG	65.00
1F,2G,3-6,7D,8	107780	COMP BNT LONG EXT ASM 19 & 20"W R	Right- 70D- 19 & 20" W- LONG	65.00
1F,2H,3-6,7D,8	111532	COMP BNT LONG EXT ASM 21 & 22"W R	Right- 70D- 21 & 22" W- LONG	65.00
>>>>		60°,70°,70° HEMI HANGERS - LONG EXT - LEFT		
1F,2A,3-6,7A,8	107781	COMP BNT LONG EXT ASM 13" W L	Left- 70D- 13 W- LONG	65.00
1F,2B,3-6,7A,8	107782	COMP BNT LONG EXT ASM 14" W L	Left- 70D- 14 W- LONG	65.00
1F,2C,3-6,7B,8	107783	COMP BNT LONG EXT ASM 15" W L	Left- 70D- 15 W- LONG	65.00
1F,2D,3-6,7B,8	107784	COMP BNT LONG EXT ASM 16" W L	Left- 70D- 16 W- LONG	65.00
1F,2E,3-6,7C,8	107785	COMP BNT LONG EXT ASM 17" W L	Left- 70D- 17 W- LONG	65.00
1F,2F,3-6,7C,8	107786	COMP BNT LONG EXT ASM 18" W L	Left- 70D- 18 W- LONG	65.00
1F,2G,3-6,7D,8	107787	COMP BNT LONG EXT ASM 19 & 20"W L	Left- 70D- 19 & 20" W- LONG	65.00
1F,2H,3-6,7D,8	111533	COMP BNT LONG EXT ASM 21 & 22"W L	Left- 70D- 21 & 22" W- LONG	65.00
>>>>		70 DEGREE "V" HANGERS - MED EXT - RIGHT		
1G,2A,3-8	107788	COMP V MED EXT ASM 13W R	Right- 70D V hangers only 13 W- MED	65.00
1G,2B,3-8	107789	COMP V MED EXT ASM 14W R	Right- 70D V hangers only 14 W- MED	65.00
1G,2C,3-8	107790	COMP V MED EXT ASM 15W R	Right- 70D V hangers only 15 W- MED	65.00
1G,2D,3-8	107791	COMP V MED EXT ASM 16W R	Right- 70D V hangers only 16 W- MED	65.00
1G,2E,3-8	107792	COMP V MED EXT ASM 17W R	Right- 70D V hangers only 17 W- MED	65.00
1G,2F,3-8	107793	COMP V MED EXT ASM 18W R	Right- 70D V hangers only 18 W- MED	65.00
>>>>		70 DEGREE "V" HANGERS - MED EXT - LEFT		
1G,2A,3-8	107795	COMP V MED EXT ASM 13W L	Left- 70D V hangers only 13 W- MED	65.00
1G,2B,3-8	107796	COMP V MED EXT ASM 14W L	Left- 70D V hangers only 14 W- MED	65.00
1G,2C,3-8	107797	COMP V MED EXT ASM 15W L	Left- 70D V hangers only 15 W- MED	65.00
1G,2D,3-8	107798	COMP V MED EXT ASM 16W L	Left- 70D V hangers only 16 W- MED	65.00
1G,2E,3-8	107799	COMP V MED EXT ASM 17W L	Left- 70D V hangers only 17 W- MED	65.00
1G,2F,3-8	107800	COMP V MED EXT ASM 18W L	Left- 70D V hangers only 18 W- MED	65.00
>>>>		70 DEGREE "V" HANGERS - LONG EXT - RIGHT		
1H,2A,3-8	107802	COMP V LNG EXT ASM 13W R	Right- 70D V hangers only 13 W- LONG	65.00
1H,2B,3-8	107803	COMP V LNG EXT ASM 14W R	Right- 70D V hangers only 14 W- LONG	65.00
1H,2C,3-8	107804	COMP V LNG EXT ASM 15W R	Right- 70D V hangers only 15 W- LONG	65.00
1H,2D,3-8	107805	COMP V LNG EXT ASM 16W R	Right- 70D V hangers only 16 W- LONG	65.00
1H,2E,3-8	107806	COMP V LNG EXT ASM 17W R	Right- 70D V hangers only 17 W- LONG	65.00
1H,2F,3-8	107807	COMP V LNG EXT ASM 18W R	Right- 70D V hangers only 18 W- LONG	65.00
>>>>		70 DEGREE "V" HANGERS - LONG EXT - LEFT		
1H,2A,3-8	107809	COMP V LNG EXT ASM 13W L	Left- 70D V hangers only 13 W- LONG	65.00
1H,2B,3-8	107810	COMP V LNG EXT ASM 14W L	Left- 70D V hangers only 14 W- LONG	65.00
1H,2C,3-8	107811	COMP V LNG EXT ASM 15W L	Left- 70D V hangers only 15 W- LONG	65.00
1H,2D,3-8	107812	COMP V LNG EXT ASM 16W L	Left- 70D V hangers only 16 W- LONG	65.00
1H,2E,3-8	107813	COMP V LNG EXT ASM 17W L	Left- 70D V hangers only 17 W- LONG	65.00
1H,2F,3-8	107814	COMP V LNG EXT ASM 18W L	Left- 70D V hangers only 18 W- LONG	65.00
>>>>		80 DEGREE HANGERS - SHORT EXT - RIGHT		
1A,2A,3-6,7A,8	107704	COMP STRT SHT EXT ASM 13"W R	Right- 80D- 13" W- SHORT	65.00
1A,2B,3-6,7A,8	107705	COMP STRT SHT EXT ASM 14"W R	Right- 80D- 14" W- SHORT	65.00
1A,2C,3-6,7B,8	107706	COMP STRT SHT EXT ASM 15"W R	Right- 80D- 15" W- SHORT	65.00
1A,2D,3-6,7B,8	107707	COMP STRT SHT EXT ASM 16"W R	Right- 80D- 16" W- SHORT	65.00
1A,2E,3-6,7C,8	107708	COMP STRT SHT EXT ASM 17"W R	Right- 80D- 17" W- SHORT	65.00


Pos.	Item Number	Description	Remarks	USD
1A,2F,3-6,7C,8	107709	COMP STRT SHT EXT ASM 18"W R	Right- 80D- 18" W- SHORT	65.00
1A,2G,3-6,7D,8	107710	COMP STRT SHT EXT ASM 19 & 20"W R	Right- 80D- 19 & 20" W- SHORT	65.00
1A,2H,3-6,7D,8	111522	COMP STRT SHT EXT ASM 21 & 22"W R	Right- 80D- 21 & 22" W- SHORT	65.00
>>>>		80 DEGREE HANGERS - SHORT EXT - LEFT		
1A,2A,3-6,7A,8	107711	COMP STRT SHT EXT ASM 13"W L	LEFT- 80D- 13" W- SHORT	65.00
1A,2B,3-6,7A,8	107712	COMP STRT SHT EXT ASM 14"W L	LEFT- 80D- 14" W- SHORT	65.00
1A,2C,3-6,7B,8	107713	COMP STRT SHT EXT ASM 15"W L	LEFT- 80D- 15" W- SHORT	65.00
1A,2D,3-6,7B,8	107714	COMP STRT SHT EXT ASM 16"W L	LEFT- 80D- 16" W- SHORT	65.00
1A,2E,3-6,7C,8	107715	COMP STRT SHT EXT ASM 17"W L	LEFT- 80D- 17" W- SHORT	65.00
1A,2F,3-6,7C,8	107716	COMP STRT SHT EXT ASM 18"W L	LEFT- 80D- 18" W- SHORT	65.00
1A,2G,3-6,7D,8	107717	COMP STRT SHT EXT ASM 19 & 20"W L	LEFT- 80D- 19 & 20" W- SHORT	65.00
1A,2H,3-6,7D,8	111523	COMP STRT SHT EXT ASM 21 & 22"W L	LEFT- 80D- 21 & 22" W- SHORT	65.00
>>>>		80 DEGREE HANGERS - MED EXT - RIGHT		
1B,2A,3-6,7A,8	107718	COMP STRT MED EXT ASM 13"W R	Right- 80D- 13" W- MED	65.00
1B,2B,3-6,7A,8	107719	COMP STRT MED EXT ASM 14"W R	Right- 80D- 14" W- MED	65.00
1B,2C,3-6,7B,8	107720	COMP STRT MED EXT ASM 15"W R	Right- 80D- 15" W- MED	65.00
1B,2D,3-6,7B,8	107721	COMP STRT MED EXT ASM 16"W R	Right- 80D- 16" W- MED	65.00
1B,2E,3-6,7C,8	107722	COMP STRT MED EXT ASM 17"W R	Right- 80D- 17" W- MED	65.00
1B,2F,3-6,7C,8	107723	COMP STRT MED EXT ASM 18"W R	Right- 80D- 18" W- MED	65.00
1B,2G,3-6,7D,8	107724	COMP STRT MED EXT ASM 19 & 20"W R	Right- 80D- 19 & 20" W- MED	65.00
1B,2H,3-6,7D,8	111524	COMP STRT MED EXT ASM 21 & 22"W R	Right- 80D- 21 & 22" W- MED	65.00
>>>>		80 DEGREE HANGERS - MED EXT - LEFT		
1B,2A,3-6,7A,8	107725	COMP STRT MED EXT ASM 13"W L	LEFT- 80D- 13" W- MED	65.00
1B,2B,3-6,7A,8	107726	COMP STRT MED EXT ASM 14"W L	LEFT- 80D- 14" W- MED	65.00
1B,2C,3-6,7B,8	107727	COMP STRT MED EXT ASM 15"W L	LEFT- 80D- 15" W- MED	65.00
1B,2D,3-6,7B,8	107728	COMP STRT MED EXT ASM 16"W L	LEFT- 80D- 16" W- MED	65.00
1B,2E,3-6,7C,8	107729	COMP STRT MED EXT ASM 17"W L	LEFT- 80D- 17" W- MED	65.00
1B,2F,3-6,7C,8	107730	COMP STRT MED EXT ASM 18"W L	LEFT- 80D- 18" W- MED	65.00
1B,2G,3-6,7D,8	107731	COMP STRT MED EXT ASM 19 & 20"W L	LEFT- 80D- 19 & 20" W- MED	65.00
1B,2H,3-6,7D,8	111525	COMP STRT MED EXT ASM 21 & 22"W L	LEFT- 80D- 21 & 22" W- MED	65.00
>>>>		80 DEGREE HANGERS - LONG EXT - RIGHT		
1C,2A,3-6,7A,8	107732	COMP STRT LONG EXT ASM 13"W R	Right- 80D- 13" W- LONG	65.00
1C,2B,3-6,7A,8	107733	COMP STRT LONG EXT ASM 14"W R	Right- 80D- 14" W- LONG	65.00
1C,2C,3-6,7B,8	107734	COMP STRT LONG EXT ASM 15"W R	Right- 80D- 15" W- LONG	65.00
1C,2D,3-6,7B,8	107735	COMP STRT LONG EXT ASM 16"W R	Right- 80D- 16" W- LONG	65.00
1C,2E,3-6,7C,8	107736	COMP STRT LONG EXT ASM 17"W R	Right- 80D- 17" W- LONG	65.00
1C,2F,3-6,7C,8	107737	COMP STRT LONG EXT ASM 18"W R	Right- 80D- 18" W- LONG	65.00
1C,2G,3-6,7D,8	107738	COMP STRT LONG EXT ASM 19 & 20"W R	Right- 80D- 19 & 20" W- LONG	65.00
1C,2H,3-6,7D,8	111526	COMP STRT LONG EXT ASM 21 & 22"W R	Right- 80D- 21 & 22" W- LONG	65.00
>>>>		80 DEGREE HANGERS - LONG EXT - LEFT		
1C,2A,3-6,7A,8	107739	COMP STRT LONG EXT ASM 13"W L	LEFT- 80D- 13" W- LONG	65.00
1C,2B,3-6,7A,8	107740	COMP STRT LONG EXT ASM 14"W L	LEFT- 80D- 14" W- LONG	65.00
1C,2C,3-6,7B,8	107741	COMP STRT LONG EXT ASM 15"W L	LEFT- 80D- 15" W- LONG	65.00
1C,2D,3-6,7B,8	107742	COMP STRT LONG EXT ASM 16"W L	LEFT- 80D- 16" W- LONG	65.00
1C,2E,3-6,7C,8	107743	COMP STRT LONG EXT ASM 17"W L	LEFT- 80D- 17" W- LONG	65.00
1C,2F,3-6,7C,8	107744	COMP STRT LONG EXT ASM 18"W L	LEFT- 80D- 18" W- LONG	65.00
1C,2G,3-6,7D,8	107745	COMP STRT LONG EXT ASM 19 & 20"W L	LEFT- 80D- 19 & 20" W- LONG	65.00
1C,2H,3-6,7D,8	111527	COMP STRT LONG EXT ASM 21 & 22"W L	LEFT- 80D- 21 & 22" W- LONG	65.00

NOTE: FOOTPLATE ASSEMBLIES ARE SET TO FOOTREST WIDTH. PLEASE SEE "FOOTPLATE FORMULA" TABLE FOR SEAT WIDTH CONVERSION.


INSET FORMULA
 0° Inset (Seat Width) = Footrest Width
 1° Inset (Seat Width) -2 = Footrest Width
 "V" Hanger (Seat Width) -4 = Footrest Width

Composite Angle Adjustable "CAA" Footplate w/ Ext & HL for Swing I/O Hangers (Order 1 each)								
Width	Ext. Tube	60°, 70° & 70° Hemi Style Hangers (Bent Extension)			80° Style hanger (Straight Extension)			70° V Style Hangers
		Short	Medium	Long	Short	Medium	Long	Medium
15" Right	107858	107872	107886	107816	107830	107844	107900	107914
15" Left	107865	107879	107893	107823	107837	107851	107907	107921
16" Right	107859	107873	107887	107817	107831	107845	107901	107915
16" Left	107866	107880	107894	107824	107838	107852	107908	107922
17" Right	107860	107874	107888	107818	107832	107846	107902	107916
17" Left	107867	107881	107895	107825	107839	107853	107909	107923
18" Right	107861	107875	107889	107819	107833	107847	107903	107917
18" Left	107868	107882	107896	107826	107840	107854	107910	107924
19" Right	107862	107876	107890	107820	107834	107848	V STYLE IS NOT AVAILABLE IN THESE SEAT WIDTHS	
19" Left	107869	107883	107897	107827	107841	107855		
20" Right	107863	107877	107891	107821	107835	107849		
20" Left	107870	107884	107898	107828	107842	107856		
21"-22" Rt	107864	107878	107892	107822	107836	107850		
21"-22" Lt	107871	107885	107899	107829	107843	107857		

FOOTPLATE MEASURING GUIDE

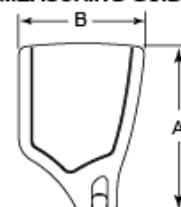

PART NO.	DESCRIPTION	DIM -A-	DIM -B-
063216-006	FOOTPLATE COMPOSITE 157 R	6 1/8"	8"
063216-012	FOOTPLATE COMPOSITE 157 L	6 1/8"	8"
063216-015	FOOTPLATE COMPOSITE 170 R	6 5/8"	8"
063216-018	FOOTPLATE COMPOSITE 170 L	6 5/8"	8"
063216-005	FOOTPLATE COMPOSITE 182 R	7 1/8"	8"
063216-011	FOOTPLATE COMPOSITE 182 L	7 1/8"	8"
063216-014	FOOTPLATE COMPOSITE 195 R	7 5/8"	8"
063216-017	FOOTPLATE COMPOSITE 195 L	7 5/8"	8"
063216-004	FOOTPLATE COMPOSITE 207 R	8 1/8"	8"
063216-010	FOOTPLATE COMPOSITE 207 L	8 1/8"	8"
063216-013	FOOTPLATE COMPOSITE 220 R	8 5/8"	6 1/4"
063216-016	FOOTPLATE COMPOSITE 220 L	8 5/8"	6 1/4"
063216-003	FOOTPLATE COMPOSITE 232 R	9 1/8"	6 1/4"
063216-009	FOOTPLATE COMPOSITE 232 L	9 1/8"	6 1/4"
063216-002	FOOTPLATE COMPOSITE 257 R	10 1/8"	6 1/4"
063216-008	FOOTPLATE COMPOSITE 257 L	10 1/8"	6 1/4"
063216-001	FOOTPLATE COMPOSITE 282 R	11 1/8"	6 1/4"
063216-007	FOOTPLATE COMPOSITE 282 L	11 1/8"	6 1/4"

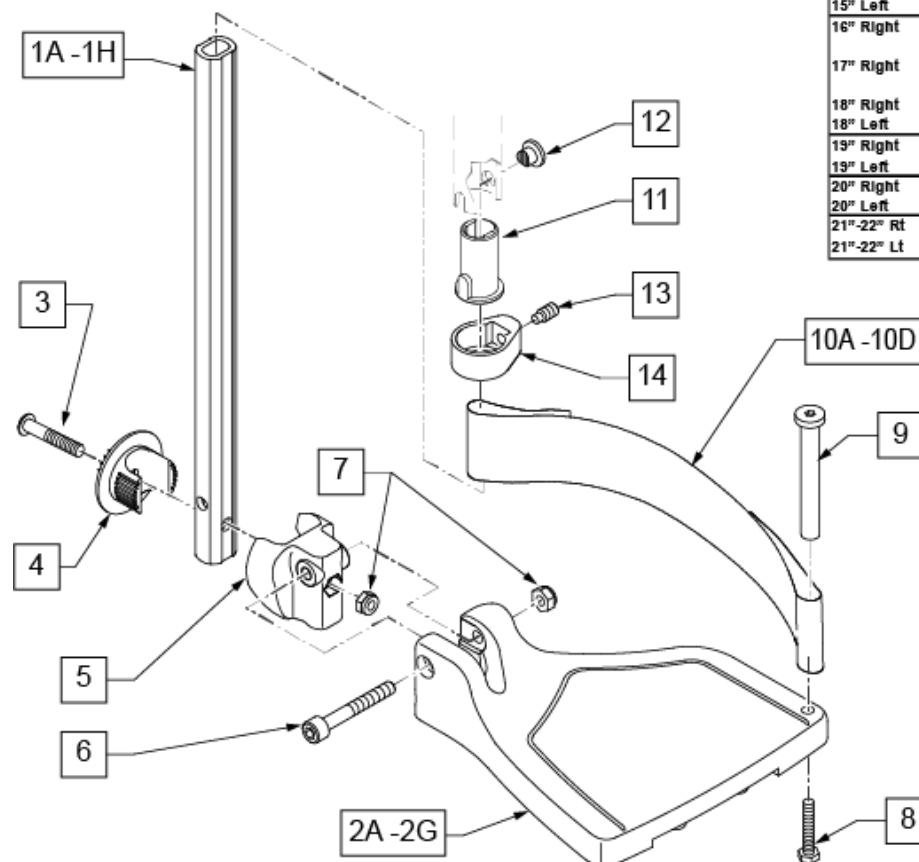

Pos.	Item Number	Description	Remarks	USD
1A,6,7	120660	SHT STRT EXT COMP HDWR	80 deg hangers only	20.00
1B,6,7	120661	MED STRT EXT COMP HDWR	80 deg hangers only	20.00
1C,6,7	120662	LNG STRT EXT COMP HDWR	80 deg hangers only	20.00
1D,6,7	120663	SHT BNT EXT COMP HDWR	60,70,70 Hemi Hangers	20.00
1E,6,7	120664	MED BNT EXT COMP HDWR	60,70,70 Hemi Hangers	20.00
1F,6,7	120665	LNG BNT EXT COMP HDWR	60,70,70 Hemi Hangers	20.00
1G,6,7	120666	SHT V EXT R COMP HDWR	70 deg V hangers only-LEFT	20.00
1G,6,7	120667	SHT V EXT L COMP HDWR	70 deg V hangers only-RIGHT	20.00
1H,6,7	120668	LNG V EXT R COMP HDWR	70 deg V hangers only-LEFT	20.00
1H,6,7	120669	LNG V EXT L COMP HDWR	70 deg V hangers only-RIGHT	20.00
2A	063216-006	COMPOSITE FOOTPLATE RIGHT	RIGHT 15" Footrest Width	20.00
2A	063216-012	COMPOSITE FOOTPLATE LEFT	LEFT 15" Footrest Width	20.00
2B	063216-015	COMPOSITE FOOTPLATE RIGHT	RIGHT 16" Footrest Width	20.00

Pos.	Item Number	Description	Remarks	USD
2B	063216-018	COMPOSITE FOOTPLATE LEFT	LEFT 16" Footrest Width	20.00
2C	063216-005	COMPOSITE FOOTPLATE RIGHT	RIGHT 17" Footrest Width	20.00
2C	063216-011	COMPOSITE FOOTPLATE LEFT	LEFT 17" Footrest Width	20.00
2D	063216-014	COMPOSITE FOOTPLATE RIGHT	RIGHT 18" Footrest Width	20.00
2D	063216-017	COMPOSITE FOOTPLATE LEFT	LEFT 18" Footrest Width	20.00
2E	063216-004	COMPOSITE FOOTPLATE RIGHT	RIGHT 19" Footrest Width	20.00
2E	063216-010	COMPOSITE FOOTPLATE LEFT	LEFT 19" Footrest Width	20.00
2F	063216-013	COMPOSITE FOOTPLATE RIGHT	RIGHT 20" Footrest Width	20.00
2F	063216-016	COMPOSITE FOOTPLATE LEFT	LEFT 20" Footrest Width	20.00
2G	063216-003	COMPOSITE FOOTPLATE RIGHT	RIGHT 21" & 22" Footrest Width	20.00
2G	063216-009	COMPOSITE FOOTPLATE LEFT	LEFT 21" & 22" Footrest Width	20.00
3	108233	BHCS M6 X 1 X 40		10.00
4	054242	ANGLE COVER		5.00
5	054241	ANGLE SEGMENT		5.00
6	103175	SHCS M6 X 1 X 40 INORG		5.00
7	104381	NUT HEX NYLOCK M6X1.0		5.00
8	102268-001	HHCS M6X1X25	Inserted in bottom of footplate.	10.00
9	104328	HEEL LOOP PEG		5.00
10A	104380-001	ADJUSTABLE FOOT REST STRAP	15" Footrest Width	7.50
10B	104380-002	ADJUSTABLE FOOT REST STRAP	16" Footrest Width	7.50
10C	104380-003	ADJUSTABLE FOOT REST STRAP	17-18" Footrest Width	7.50
10D	104380-004	ADJUSTABLE FOOT REST STRAP	19-22" Footrest Width	7.50
11	104160	BUSHING FOOTREST		5.00
11-14	107485	FOOTREST CLAMP ASSM		30.00
>>>>		60°,70°,70° HEMI HANGERS - SHORT EXT - RIGHT		
1D,2A,3-10A	107858	CAA BNT SHT EXT ASM 15W R	Right- 60° & 60° & 70° 15 W- SHORT	70.00
1D,2B,3-10B	107859	CAA BNT SHT EXT ASM 16W R	Right- 60° & 70° 16 W- SHORT	70.00
1D,2C,3-10B	107860	CAA BNT SHT EXT ASM 17W R	Right- 60° & 70° 17 W- SHORT	70.00
1D,2D,3-10C	107861	CAA BNT SHT EXT ASM 18W R	Right- 60° & 70° 18 W- SHORT	70.00
1D,2E,3-10C	107862	CAA BNT SHT EXT ASM 19W R	Right- 60° & 70° 19 W- SHORT	70.00
1D,2F,3-10D	107863	CAA BNT SHT EXT ASM 20W R	Right- 60° & 70° 20 W- SHORT	70.00
1D,2G,3-10D	107864	CAA BNT SHT EXT ASM 21&22W R	Right- 60° & 70° 21&22 W- SHORT	70.00
>>>>		60°,70°,70° HEMI HANGERS - SHORT EXT - LEFT		
1D,2A,3-10A	107865	CAA BNT SHT EXT ASM 15W L	Left- 60° & 70° 15 W- SHORT	70.00
1D,2B,3-10B	107866	CAA BNT SHT EXT ASM 16W L	Left- 60° & 70° 16 W- SHORT	70.00
1D,2C,3-10B	107867	CAA BNT SHT EXT ASM 17W L	Left- 60° & 70° 17 W- SHORT	70.00
1D,2D,3-10C	107868	CAA BNT SHT EXT ASM 18W L	Left- 60° & 70° 18 W- SHORT	70.00
1D,2E,3-10C	107869	CAA BNT SHT EXT ASM 19W L	Left- 60° & 70° 19 W- SHORT	70.00
1D,2F,3-10D	107870	CAA BNT SHT EXT ASM 20W L	Left- 60° & 70° 20 W- SHORT	70.00
1D,2G,3-10D	107871	CAA BNT SHT EXT ASM 21&22W L	Left- 60° & 70° 21&22 W- SHORT	70.00
>>>>		60°,70°,70° HEMI HANGERS - MED EXT - RIGHT		
1E,2A,3-10A	107872	CAA BNT MED EXT ASM 15W R	Right- 60° & 70° 15 W- MED	70.00
1E,2B,3-10B	107873	CAA BNT MED EXT ASM 16W R	Right- 60° & 70° 16 W- MED	70.00
1E,2C,3-10B	107874	CAA BNT MED EXT ASM 17W R	Right- 60° & 70° 17 W- MED	70.00
1E,2D,3-10C	107875	CAA BNT MED EXT ASM 18W R	Right- 60° & 70° 18 W- MED	70.00
1E,2E,3-10C	107876	CAA BNT MED EXT ASM 19W R	Right- 60° & 70° 19 W- MED	70.00
1E,2F,3-10D	107877	CAA BNT MED EXT ASM 20W R	Right- 60° & 70° 20 W- MED	70.00
1E,2G,3-10D	107878	CAA BNT MED EXT ASM 21&22W R	Right- 60° & 70° 21&22 W- MED	70.00
>>>>		60°,70°,70° HEMI HANGERS - MED EXT - LEFT		
1E,2A,3-10A	107879	CAA BNT MED EXT ASM 15W L	Left- 60° & 70° 15 W- MED	70.00
1E,2B,3-10B	107880	CAA BNT MED EXT ASM 16W L	Left- 60° & 70° 16 W- MED	70.00
1E,2C,3-10B	107881	CAA BNT MED EXT ASM 17W L	Left- 60° & 70° 17 W- MED	70.00
1E,2D,3-10C	107882	CAA BNT MED EXT ASM 18W L	Left- 60° & 70° 18 W- MED	70.00
1E,2E,3-10C	107883	CAA BNT MED EXT ASM 19W L	Left- 60° & 70° 19 W- MED	70.00
1E,2F,3-10D	107884	CAA BNT MED EXT ASM 20W L	Left- 60° & 70° 20 W- MED	70.00
1E,2G,3-10D	107885	CAA BNT MED EXT ASM 21&22W L	Left- 60° & 70° 21&22 W- MED	70.00

Pos.	Item Number	Description	Remarks	USD
>>>>		60°,70°,70° HEMI HANGERS - LONG EXT - RIGHT		
1F,2A,3-10A	107886	CAA BNT LNG EXT ASM 15W R	Right- 60° & 70° 15 W- LONG	70.00
1F,2B,3-10B	107887	CAA BNT LNG EXT ASM 16W R	Right- 60° & 70° 16 W- LONG	70.00
1F,2C,3-10B	107888	CAA BNT LNG EXT ASM 17W R	Right- 60° & 70° 17 W- LONG	70.00
1F,2D,3-10C	107889	CAA BNT LNG EXT ASM 18W R	Right- 60° & 70° 18 W- LONG	70.00
1F,2E,3-10C	107890	CAA BNT LNG EXT ASM 19W R	Right- 60° & 70° 19 W- LONG	70.00
1F,2F,3-10D	107891	CAA BNT LNG EXT ASM 20W R	Right- 60° & 70° 20 W- LONG	70.00
1F,2G,3-10D	107892	CAA BNT LNG EXT ASM 21&22W R	Right- 60° & 70° 21&22 W- LONG	70.00
>>>>		60°,70°,70° HEMI HANGERS - LONG EXT - LEFT		
1F,2A,3-10A	107893	CAA BNT LNG EXT ASM 15W L	Left- 60° & 70° 15 W- LONG	70.00
1F,2B,3-10B	107894	CAA BNT LNG EXT ASM 16W L	Left- 60° & 70° 16 W- LONG	70.00
1F,2C,3-10B	107895	CAA BNT LNG EXT ASM 17W L	Left- 60° & 70° 17 W- LONG	70.00
1F,2D,3-10C	107896	CAA BNT LNG EXT ASM 18W L	Left- 60° & 70° 18 W- LONG	70.00
1F,2E,3-10C	107897	CAA BNT LNG EXT ASM 19W L	Left- 60° & 70° 19 W- LONG	70.00
1F,2F,3-10D	107898	CAA BNT LNG EXT ASM 20W L	Left- 60° & 70° 20 W- LONG	70.00
1F,2G,3-10D	107899	CAA BNT LNG EXT ASM 21&22W L	Left- 60° & 70° 21&22 W- LONG	70.00
>>>>		70° "V" HANGERS - MED EXT - RIGHT		
1G,2A,3-10A	107900	CAA V MED EXT ASM 15W R	Right- 70° V hangers only 15 W- MED	70.00
1G,2B,3-10B	107901	CAA V MED EXT ASM 16W R	Right- 70° V hangers only 16 W- MED	70.00
1G,2C,3-10B	107902	CAA V MED EXT ASM 17W R	Right- 70° V hangers only 17 W- MED	70.00
1G,2D,3-10C	107903	CAA V MED EXT ASM 18W R	Right- 70° V hangers only 18 W- MED	70.00
>>>>		70° "V" HANGERS - MED EXT - LEFT		
1G,2A,3-10A	107907	CAA V MED EXT ASM 15W L	Left- 70° V hangers only 15 W- MED	70.00
1G,2B,3-10B	107908	CAA V MED EXT ASM 16W L	Left- 70° V hangers only 16 W- MED	70.00
1G,2C,3-10B	107909	CAA V MED EXT ASM 17W L	Left- 70° V hangers only 17 W- MED	70.00
1G,2D,3-10C	107910	CAA V MED EXT ASM 18W L	Left- 70° V hangers only 18 W- MED	70.00
>>>>		70° "V" HANGERS - LONG EXT - RIGHT		
1H,2A,3-10A	107914	CAA V LNG EXT ASM 15W R	Right- 70° V hangers only 15 W- LONG	70.00
1H,2B,3-10B	107915	CAA V LNG EXT ASM 16W R	Right- 70° V hangers only 16 W- LONG	70.00
1H,2C,3-10B	107916	CAA V LNG EXT ASM 17W R	Right- 70° V hangers only 17 W- LONG	70.00
1H,2D,3-10C	107917	CAA V LNG EXT ASM 18W R	Right- 70° V hangers only 18 W- LONG	70.00
>>>>		70° "V" HANGERS - LONG EXT - LEFT		
1H,2A,3-10A	107921	CAA V LNG EXT ASM 15W L	Left- 70° V hangers only 15 W- LONG	70.00
1H,2B,3-10B	107922	CAA V LNG EXT ASM 16W L	Left- 70° V hangers only 16 W- LONG	70.00
1H,2C,3-10B	107923	CAA V LNG EXT ASM 17W L	Left- 70° V hangers only 17 W- LONG	70.00
1H,2D,3-10C	107924	CAA V LNG EXT ASM 18W L	Left- 70° V hangers only 18 W- LONG	70.00
>>>>		80° HANGERS - SHORT EXT - RIGHT		
1A,2A,3-10A	107816	CAA STRT SHT EXT ASM 15W R	Right- 80° 15 W- SHORT	70.00
1A,2B,3-10B	107817	CAA STRT SHT EXT ASM 16W R	Right- 80° 16 W- SHORT	70.00
1A,2C,3-10B	107818	CAA STRT SHT EXT ASM 17W R	Right- 80° 17 W- SHORT	70.00
1A,2D,3-10C	107819	CAA STRT SHT EXT ASM 18W R	Right- 80° 18 W- SHORT	70.00
1A,2E,3-10C	107820	CAA STRT SHT EXT ASM 19W R	Right- 80° 19 W- SHORT	70.00
1A,2F,3-10D	107821	CAA STRT SHT EXT ASM 20W R	Right- 80° 20 W- SHORT	70.00
1A,2G,3-10D	107822	CAA STRT SHT EXT ASM 21&22W R	Right- 80° 21&22 W- SHORT	70.00
>>>>		80° HANGERS - SHORT EXT - LEFT		
1A,2A,3-10A	107823	CAA STRT SHT EXT ASM 15W L	Left- 80° 15 W- SHORT	70.00
1A,2B,3-10B	107824	CAA STRT SHT EXT ASM 16W L	Left- 80° 16 W- SHORT	70.00
1A,2C,3-10B	107825	CAA STRT SHT EXT ASM 17W L	Left- 80° 17 W- SHORT	70.00
1A,2D,3-10C	107826	CAA STRT SHT EXT ASM 18W L	Left- 80° 18 W- SHORT	70.00
1A,2E,3-10C	107827	CAA STRT SHT EXT ASM 19W L	Left- 80° 19 W- SHORT	70.00
1A,2F,3-10D	107828	CAA STRT SHT EXT ASM 20W L	Left- 80° 20 W- SHORT	70.00
1A,2G,3-10D	107829	CAA STRT SHT EXT ASM 21&22W L	Left- 80° 21&22 W- SHORT	70.00
>>>>		80° HANGERS - MED EXT - RIGHT		
1B,2A,3-10A	107830	CAA STRT MED EXT ASM 15W R	Right- 80° 15 W- MED	70.00
1B,2B,3-10B	107831	CAA STRT MED EXT ASM 16W R	Right- 80° 16 W- MED	70.00
1B,2C,3-10B	107832	CAA STRT MED EXT ASM 17W R	Right- 80° 17 W- MED	70.00


Pos.	Item Number	Description	Remarks	USD
1B,2D,3-10C	107833	CAA STRT MED EXT ASM 18W R	Right- 80° 18 W- MED	70.00
1B,2E,3-10C	107834	CAA STRT MED EXT ASM 19W R	Right- 80° 19 W- MED	70.00
1B,2F,3-10D	107835	CAA STRT MED EXT ASM 20W R	Right- 80° 20 W- MED	70.00
1B,2G,3-10D	107836	CAA STRT MED EXT ASM 21&22W R	Right- 80° 21&22 W- MED	70.00
>>>>		80° HANGERS - MED EXT - LEFT		
1B,2A,3-10A	107837	CAA STRT MED EXT ASM 15W L	Left- 80° 15 W- MED	70.00
1B,2B,3-10B	107838	CAA STRT MED EXT ASM 16W L	Left- 80° 16 W- MED	70.00
1B,2C,3-10B	107839	CAA STRT MED EXT ASM 17W L	Left- 80° 17 W- MED	70.00
1B,2D,3-10C	107840	CAA STRT MED EXT ASM 18W L	Left- 80° 18 W- MED	70.00
1B,2E,3-10C	107841	CAA STRT MED EXT ASM 19W L	Left- 80° 19 W- MED	70.00
1B,2F,3-10D	107842	CAA STRT MED EXT ASM 20W L	Left- 80° 20 W- MED	70.00
1B,2G,3-10D	107843	CAA STRT MED EXT ASM 21&22W L	Left- 80° 21&22 W- MED	70.00
>>>>		80° HANGERS - LONG EXT - RIGHT		
1C,2A,3-10A	107844	CAA STRT LNG EXT ASM 15W R	Right- 80° 15 W- LONG	70.00
1C,2B,3-10B	107845	CAA STRT LNG EXT ASM 16W R	Right- 80° 16 W- LONG	70.00
1C,2C,3-10B	107846	CAA STRT LNG EXT ASM 17W R	Right- 80° 17 W- LONG	70.00
1C,2D,3-10C	107847	CAA STRT LNG EXT ASM 18W R	Right- 80° 18 W- LONG	70.00
1C,2E,3-10C	107848	CAA STRT LNG EXT ASM 19W R	Right- 80° 19 W- LONG	70.00
1C,2F,3-10D	107849	CAA STRT LNG EXT ASM 20W R	Right- 80° 20 W- LONG	70.00
1C,2G,3-10D	107850	CAA STRT LNG EXT ASM 21&22W R	Right- 80° 21&22 W- LONG	70.00
>>>>		80° HANGERS - LONG EXT - LEFT		
1C,2A,3-10A	107851	CAA STRT LNG EXT ASM 15W L	Left- 80° 15 W- LONG	70.00
1C,2B,3-10B	107852	CAA STRT LNG EXT ASM 16W L	Left- 80° 16 W- LONG	70.00
1C,2C,3-10B	107853	CAA STRT LNG EXT ASM 17W L	Left- 80° 17 W- LONG	70.00
1C,2D,3-10C	107854	CAA STRT LNG EXT ASM 18W L	Left- 80° 18 W- LONG	70.00
1C,2E,3-10C	107855	CAA STRT LNG EXT ASM 19W L	Left- 80° 19 W- LONG	70.00
1C,2F,3-10D	107856	CAA STRT LNG EXT ASM 20W L	Left- 80° 20 W- LONG	70.00
1C,2G,3-10D	107857	CAA STRT LNG EXT ASM 21&22W L	Left- 80° 21&22 W- LONG	70.00

NOTE: FOOTPLATE ASSEMBLIES ARE SET TO FOOTREST WIDTH. PLEASE SEE "FOOTPLATE FORMULA" TABLE FOR SEAT WIDTH CONVERSION.


Adult Angle Adjustable "AAA" Footplates w/ Ext & HL for Swing I/O Hangers								
(Order 1 each)								
Ext. Tube	60°, 70° & 70° Hemi Style Hangers (Bent Extension)			80° Style hanger (Straight Extension)			70° V Style Hangers	
Width	Short	Med	Long	Short	Med	Long	Med	Long
10"-11" Rt	NOT AVAILABLE IN THIS WIDTH			NOT AVAILABLE IN THIS WIDTH			111546	111550
10"-11" Lt	NOT AVAILABLE IN THIS WIDTH			NOT AVAILABLE IN THIS WIDTH			111547	111551
12-13" Rt	111540	111542	111544	111534	111536	111538	111548	111552
12-13" Lt	111541	111543	111545	111535	111537	111539	111549	111553
14"-15" Rt	107973	107987	108001	107928	107942	107956	108015	108029
14"-15" Lt	107980	107994	108008	107935	107949	107963	108022	108036
16"-17" Rt	107975	107989	108003	107930	107944	107958	108017	108031
16"-17" Lt	107982	107996	108010	107937	107951	107965	108024	108038
18"-22" Rt	107977	107991	108005	107932	107946	107960	108019	108033
18"-22" Lt	107984	107998	108012	107939	107953	107968	108026	108040

INSET FORMULA
 0" Inset (Seat Width) = Footrest Width
 1" Inset (Seat Width) -2 = Footrest Width
 "V" Hanger (Seat Width) -4 = Footrest Width

MEASURING GUIDE

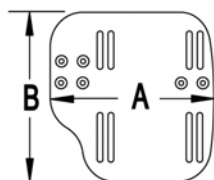


CHAIR WIDTH	"A"	"B"
10" - 11"	3 1/2"	7 1/4"
12" - 13"	4 1/2"	7 3/4"
14" - 15"	5 1/2"	7 3/4"
16" - 17"	6 1/2"	7 3/4"
18" - 20"	7 1/2"	7 3/4"

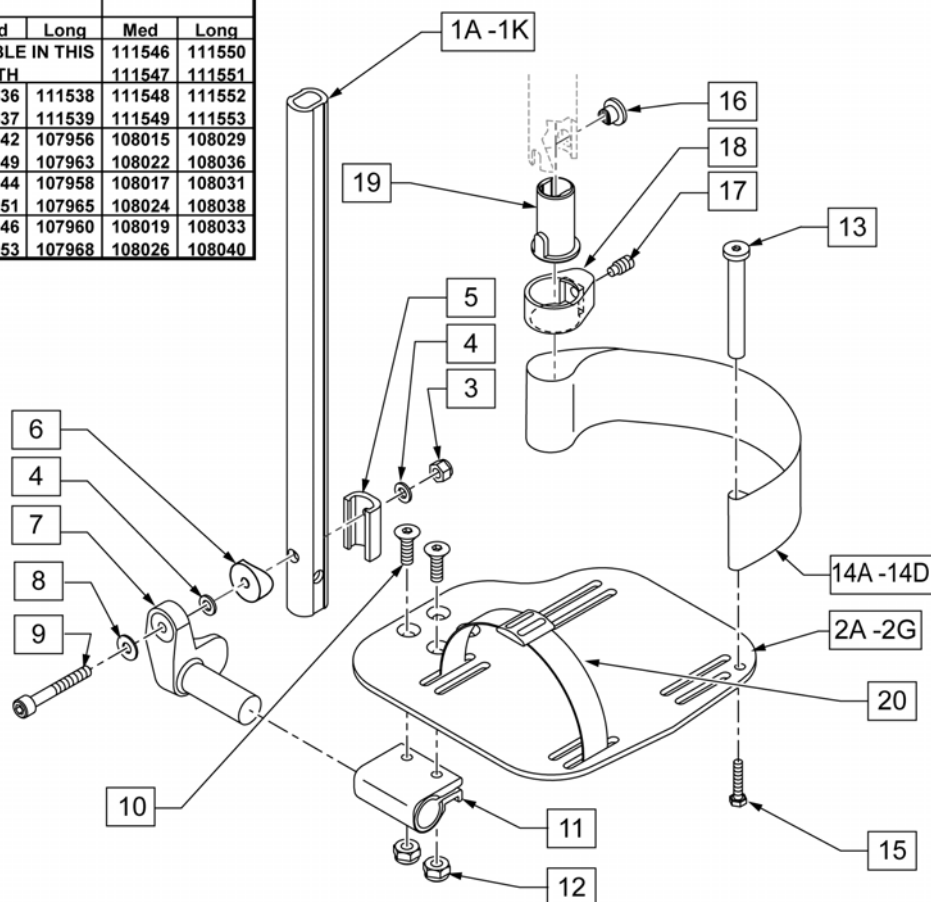


Pos.	Item Number	Description	Remarks	USD
1A,3-5,8,9	120670	SHT STRT EXT AAA HDWR	80 deg Hangers Only	30.00
1B,3-5,8,9	120671	MED STRT EXT AAA HDWR	80 deg Hangers Only	30.00
1C,3-5,8,9	120672	LNG STRT EXT AAA HDWR	80 deg Hangers Only	30.00
1D,3-5,8,9	120673	SHT BNT EXT AAA HDWR	60,70,70 Hemi Hangers	30.00
1E,3-5,8,9	120674	MED BNT EXT AAA HDWR	60,70,70 Hemi Hangers	30.00
1F,3-5,8,9	120675	LNG BNT EXT AAA HDWR	60,70,70 Hemi Hangers	30.00
1G,3-5,8,9	120676	SHT V EXT R AAA HDWR	Left - 70 deg V Hangers Only	30.00
1H,3-5,8,9	120677	SHT V EXT L AAA HDWR	Right - 70 deg V Hangers Only	30.00
1I,3-5,8,9	120678	LNG V EXT R AAA HDWR	Left - 70 deg V Hangers Only	30.00
1J,3-5,8,9	120679	LNG V EXT L AAA HDWR	Right - 70 deg V Hangers Only	30.00
2A	500656	ADULT ANG ADJ F/PLT 10/11" R	RIGHT 10-11" Footrest Width	58.14
2A	500657	ADULT ANG ADJ F/PLT 10/11" L	LEFT 10-11" Footrest Width	58.14
2B	500658	ADULT ANG ADJ F/PLT 12/13" R	RIGHT 12-13" Footrest Width	60.68

Pos.	Item Number	Description	Remarks	USD
2B	500659	ADULT ANG ADJ F/PLT 12/13" L	LEFT 12-13" Footrest Width	60.68
2C	500660	ADLT ANGL ADJ F/PLT 14/15	RIGHT 14-15" Footrest Width	60.68
2C	500661	ADLT ANGL ADJ F/PLT 14/15	LEFT 14-15" Footrest Width	60.68
2D	500662	ADLT ANGL ADJ F/PLT 16/17	RIGHT 16-17" Footrest Width	60.68
2D	500663	ADLT ANGL ADJ F/PLT 16/17	LEFT 16-17" Footrest Width	60.68
2E	500664	ADLT ANGL ADJ F/PLT 18-20	RIGHT 18-22" Footrest Width	60.68
2E	500665	ADLT ANGL ADJ F/PLT 18-20	LEFT 18-22" Footrest Width	60.68
3	104381	NUT HEX NYLOCK M6X1.0		5.00
4	106944	WASHER FLAT THIN AN 1/4 ROHS		1.00
5	107199	HANGER EXTENSION SADDLE		5.00
6	104377	3/4" SADDLE THIN		5.00
7	541002	FTRST MOUNTING BRKT-3/8 RAD R	RIGHT	18.21
7	541003	FTRST MOUNTING BRKT-3/8 RAD L	LEFT	18.21
8	106945	WASHER FLAT SAE 1/4 ROHS		1.00
9	104399	SHCS M6X1X45 12 MM PART THD		5.00
10	106978	FHCS 1/4-28 X 7/8 ROHS		5.00
11	550350	3/4" CLMP ADJ FT PLT		11.04
12	106935	NUT NYLOCK THIN 1/4-28 ROHS		5.00
13	104328	POST HEEL LOOP		5.00
14A	104380-001	ADJUSTABLE FOOT REST STRAP	12 - 14" WIDE	7.50
14B	104380-002	ADJUSTABLE FOOT REST STRAP	15 - 16" WIDE	7.50
14C	104380-003	ADJUSTABLE FOOT REST STRAP	17 - 18" WIDE	7.50
14D	104380-004	ADJUSTABLE FOOT REST STRAP	19 - 20" WIDE	7.50
15	102268-001	HHCS M6 X 1 X 25		10.00
3-6,8,9	120659	EXT MNT HARDWARE 1 SIDE		20.00
16-19	107485	FOOTREST CLAMP ASSM		30.00
20	740260	TOE-ANKLE STRAP ADJ ADULT FOOTPLATE		20.40
>>>>		80° HANGERS - SHORT EXT - RIGHT		
1A,2B,3-14A,15	111534	AAA STRT SHT EXT ASM 12-13" W R	Right- 80D- 12-13" W- SHORT	100.00
1A,2C,3-14B,15	107928	AAA STRT SHT EXT ASM 14-15" W R	Right- 80D- 14-15" W- SHORT	100.00
1A,2D,3-14C,15	107930	AAA STRT SHT EXT ASM 16-17" W R	Right- 80D- 16-17" W- SHORT	100.00
1A,2E,3-14D,15	107932	AAA STRT SHT EXT ASM 18-22" W R	Right- 80D- 18-22" W- SHORT	100.00
>>>>		80° HANGERS - SHORT EXT - LEFT		
1A,2B,3-14A,15	111535	AAA STRT SHT EXT ASM 12-13" W L	Left- 80D- 12-13" W- SHORT	100.00
1A,2C,3-14B,15	107935	AAA STRT SHT EXT ASM 14" W L	Left- 80D- 14-15" W- SHORT	100.00
1A,2D,3-14C,15	107937	AAA STRT SHT EXT ASM 16" W L	Left- 80D- 16-17" W- SHORT	100.00
1A,2E,3-14D,15	107939	AAA STRT SHT EXT ASM 18" W L	Left- 80D- 18-22" W- SHORT	100.00
>>>>		80° HANGERS - MED EXT - RIGHT		
1B,2B,3-14A,15	111536	AAA STRT MED EXT ASM 12-13" W R	Right- 80D- 12-13" W- MED	100.00
1B,2C,3-14B,15	107942	AAA STRT MED EXT ASM 14-15" W R	Right- 80D- 14-15" W- MED	100.00
1B,2D,3-14C,15	107944	AAA STRT MED EXT ASM 16-17" W R	Right- 80D- 16-17" W- MED	100.00
1B,2E,3-14D,15	107946	AAA STRT MED EXT ASM 18-22" W R	Right- 80D- 18-22" W- MED	100.00
>>>>		80° HANGERS - MED EXT - LEFT		
1B,2B,3-14A,15	111537	AAA STRT MED EXT ASM 12-13" W L	Left- 80D- 12-13" W- MED	100.00
1B,2C,3-14B,15	107949	AAA STRT MED EXT ASM 14" W L	Left- 80D- 14-15" W- MED	100.00
1B,2D,3-14C,15	107951	AAA STRT MED EXT ASM 16" W L	Left- 80D- 16-17" W- MED	100.00
1B,2E,3-14D,15	107953	AAA STRT MED EXT ASM 18" W L	Left- 80D- 18-22" W- MED	100.00
>>>>		80° HANGERS - LONG EXT - RIGHT		
1C,2B,3-14A,15	111538	AAA STRT LNG EXT ASM 12-13" W R	Right- 80D- 12-13" W- LONG	100.00
1C,2C,3-14B,15	107956	AAA STRT LNG EXT ASM 14 -15" W R	Right- 80D- 14-15" W- LONG	100.00
1C,2D,3-14C,15	107958	AAA STRT LNG EXT ASM 16 -17" W R	Right- 80D- 16-17" W- LONG	100.00
1C,2E,3-14D,15	107960	AAA STRT LNG EXT ASM 18 -22" W R	Right- 80D- 18-22" W- LONG	100.00
>>>>		80° HANGERS - LONG EXT - LEFT		
1C,2B,3-14A,15	111539	AAA STRT LNG EXT ASM 12-13" W L	Left- 80D- 12-13" W- LONG	100.00
1C,2C,3-14B,15	107963	AAA STRT LNG EXT ASM 14W L	Left- 80D- 14-15" W- LONG	100.00
1C,2D,3-14C,15	107965	AAA STRT LNG EXT ASM 16W L	Left- 80D- 16-17" W- LONG	100.00


Pos.	Item Number	Description	Remarks	USD
1C,2E,3-14D,15	107968	AAA STRT LNG EXT ASM 18W L	Left- 80D- 18-22" W- LONG	100.00
>>>>		70° HANGERS - SHORT EXT - RIGHT		
1D,2B,3-14A,15	111540	AAA BNT SHT EXT ASM 12-13" W R	Right- 60D,70D,70D HEMI - 12-13" W- SHORT	100.00
1D,2C,3-14B,15	107973	AAA BNT SHT EXT ASM 14 -15" W R	Right- 60D,70D,70D HEMI - 14-15" W- SHORT	100.00
1D,2D,3-14C,15	107975	AAA BNT SHT EXT ASM 16 -17" W R	Right- 60D,70D,70D HEMI - 16-17" W- SHORT	100.00
1D,2E,3-14D,15	107977	AAA BNT SHT EXT ASM 18 -22" W R	Right- 60D,70D,70D HEMI - 18-22" W- SHORT	100.00
>>>>		70° HANGERS - SHORT EXT - LEFT		
1D,2B,3-14A,15	111541	AAA BNT SHT EXT ASM 12-13" W L	Left- 60D,70D,70D HEMI - 12-13" W- SHORT	100.00
1D,2C,3-14B,15	107980	AAA BNT SHT EXT ASM 14 -15" W L	Left- 60D,70D,70D HEMI - 14-15" W- SHORT	100.00
1D,2D,3-14C,15	107982	AAA BNT SHT EXT ASM 16 -17" W L	Left- 60D,70D,70D HEMI - 16-17" W- SHORT	100.00
1D,2E,3-14D,15	107984	AAA BNT SHT EXT ASM 18 -22" W L	Left- 60D,70D,70D HEMI - 18-22" W- SHORT	100.00
>>>>		70° HANGERS - MED EXT - RIGHT		
1E,2B,3-14A,15	111542	AAA BNT MED EXT ASM 12-13" W R	Right- 60D,70D,70D HEMI - 12-13" W- MED	100.00
1E,2C,3-14B,15	107987	AAA BNT MED EXT ASM 14 -15" W R	Right- 60D,70D,70D HEMI - 14-15" W- MED	100.00
1E,2D,3-14C,15	107989	AAA BNT MED EXT ASM 16 -17" W R	Right- 60D,70D,70D HEMI - 16-17" W- MED	100.00
1E,2E,3-14D,15	107991	AAA BNT MED EXT ASM 18 -22" W R	Right- 60D,70D,70D HEMI - 18-22" W- MED	100.00
>>>>		70° HANGERS - MED EXT - LEFT		
1E,2B,3-14A,15	111543	AAA BNT MED EXT ASM 12-13" W L	Left- 60D,70D,70D HEMI - 12-13" W- MED	100.00
1E,2C,3-14B,15	107994	AAA BNT MED EXT ASM 14 -15" W L	Left- 60D,70D,70D HEMI - 14-15" W- MED	100.00
1E,2D,3-14C,15	107996	AAA BNT MED EXT ASM 16 -17" W L	Left- 60D,70D,70D HEMI - 16-17" W- MED	100.00
1E,2E,3-14D,15	107998	AAA BNT MED EXT ASM 18 -22" W L	Left- 60D,70D,70D HEMI - 18-22" W- MED	100.00
>>>>		70° HANGERS - LONG EXT - RIGHT		
1F,2B,3-14A,15	111544	AAA BNT LONG EXT ASM 12-13" W R	Right- 60D,70D,70D HEMI - 12-13" W- LONG	100.00
1F,2C,3-14B,15	108001	AAA BNT LONG EXT ASM 14 -15" W R	Right- 60D,70D,70D HEMI - 14-15" W- LONG	100.00
1F,2D,3-14C,15	108003	AAA BNT LONG EXT ASM 16 -17" W R	Right- 60D,70D,70D HEMI - 16-17" W- LONG	100.00
1F,2E,3-14D,15	108005	AAA BNT LONG EXT ASM 18 -22" W R	Right- 60D,70D,70D HEMI - 18-22" W- LONG	100.00
>>>>		70° HANGERS - LONG EXT - LEFT		
1F,2B,3-14A,15	111545	AAA BNT LONG EXT ASM 12-13" W L	Left- 60D,70D,70D HEMI - 12-13" W- LONG	100.00
1F,2C,3-14B,15	108008	AAA BNT LONG EXT ASM 14 -15" W L	Left- 60D,70D,70D HEMI - 14-15" W- LONG	100.00
1F,2D,3-14C,15	108010	AAA BNT LONG EXT ASM 16 -17" W L	Left- 60D,70D,70D HEMI - 16-17" W- LONG	100.00
1F,2E,3-14D,15	108012	AAA BNT LONG EXT ASM 18 -22" W L	Left- 60D,70D,70D HEMI - 18-22" W- LONG	100.00
>>>>		70° "V" HANGERS - MED EXT - RIGHT		
1G,2A,3-15	111546	AAA V MED EXT ASM 10-11" W R	Right- 70D V hangers only 10-11" W- MED	100.00
1G,2B,3-15	111548	AAA V MED EXT ASM 12-13" W R	Right- 70D V hangers only 12-13" W- MED	100.00
1G,2C,3-15	108015	AAA V MED EXT ASM 14 -15" W R	Right- 70D V hangers only 14-15" W- MED	100.00
1G,2D,3-15	108017	AAA V MED EXT ASM 16 -17" W R	Right- 70D V hangers only 16-18" W- MED	100.00
1G,2E,3-15	108019	AAA V MED EXT ASM 18 -18" W R	Right- 70D V hangers only 18-22" W- MED	100.00
>>>>		70° "V" HANGERS - MED EXT - LEFT		
1G,2A,3-15	111547	AAA V MED EXT ASM 10-11" W L	Left- 70D V hangers only 10-11" W- MED	100.00
1G,2B,3-15	111549	AAA V MED EXT ASM 12-13" W L	Left- 70D V hangers only 12-13" W- MED	100.00
1G,2C,3-15	108022	AAA V MED EXT ASM 14 -15" W L	Left- 70D V hangers only 14-15" W- MED	100.00
1G,2D,3-15	108024	AAA V MED EXT ASM 16 -17" W L	Left- 70D V hangers only 16-18" W- MED	100.00
1G,2E,3-15	108026	AAA V MED EXT ASM 18 -22" W L	Left- 70D V hangers only 18-22" W- MED	100.00
>>>>		70° "V" HANGERS - LONG EXT - RIGHT		
1H,2A,3-15	111550	AAA V LONG EXT ASM 10-11" W R	Right- 70D V hangers only 10-11" W- LONG	100.00
1H,2B,3-15	111552	AAA V LONG EXT ASM 12-13" W R	Right- 70D V hangers only 12-13" W- LONG	100.00
1H,2C,3-15	108029	AAA V LONG EXT ASM 14 -15" W R	Right- 70D V hangers only 14-15" W- LONG	100.00
1H,2D,3-15	108031	AAA V LONG EXT ASM 16 -17" W R	Right- 70D V hangers only 16-18" W- LONG	100.00
1H,2E,3-15	108033	AAA V LONG EXT ASM 18 -22" W R	Right- 70D V hangers only 18-22" W- LONG	100.00
>>>>		70° "V" HANGERS - LONG EXT - LEFT		
1H,2A,3-15	111551	AAA V LONG EXT ASM 10-11" W L	Left- 70D V hangers only 10-11" W- LONG	100.00
1H,2B,3-15	111553	AAA V LONG EXT ASM 12-13" W L	Left- 70D V hangers only 12-13" W- LONG	100.00
1H,2C,3-15	108036	AAA V LONG EXT ASM 14 -15" W L	Left- 70D V hangers only 14-15" W- LONG	100.00
1H,2D,3-15	108038	AAA V LONG EXT ASM 16 -17" W L	Left- 70D V hangers only 16-18" W- LONG	100.00
1H,2E,3-15	108040	AAA V LONG EXT ASM 18 -22" W L	Left- 70D V hangers only 18-22" W- LONG	100.00

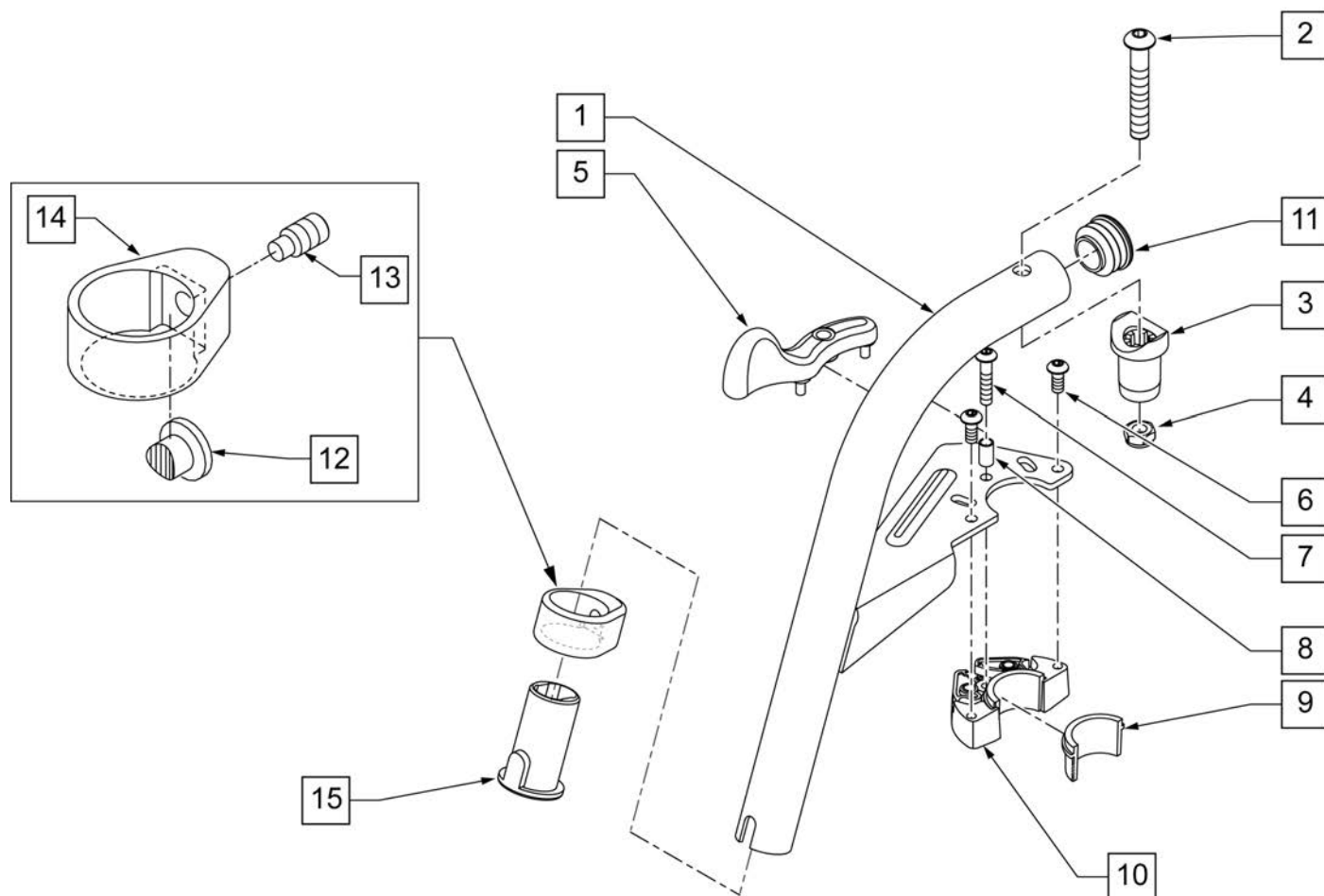

Pos.	Item Number	Description	Remarks	USD
2	106983	5/16-18 X 2 BH C/S SHT THREAD		5.00
3,4	561197	PIVOT SADDLE PLASTIC		2.00
5	052296	SWING I/O LATCH LEVER RIGHT	Right	11.25
5	052297	SWING I/O LATCH LEVER LEFT	Left	11.25
6	102259-001	BHCS M5 X .8 X 12 W/PL INORG		2.00
7	230494	M5-30 BTTN HD SOC C/S BLK		2.57
8	052054	BUSHING PIVOT HANGER SWING I/O		2.00
11	561062	1" INSERT PLUG .065 RIBS		2.00
15	104160	BUSHING FOOTREST		5.00
1-15	107671S	70 DEG SWING IN/OUT HANGER ASSEMBLY LEFT	SPECIFY COLOR,LEFT	105.00
1-15	107670S	70 DEG SWING IN/OUT HANGER ASSEMBLY RIGHT	SPECIFY COLOR,RIGHT	105.00
12-15	107485	FOOTREST CLAMP ASSM		30.00
6,9,10	120976	ASSY LATCH BLOCK ANTI LIFT		45.00

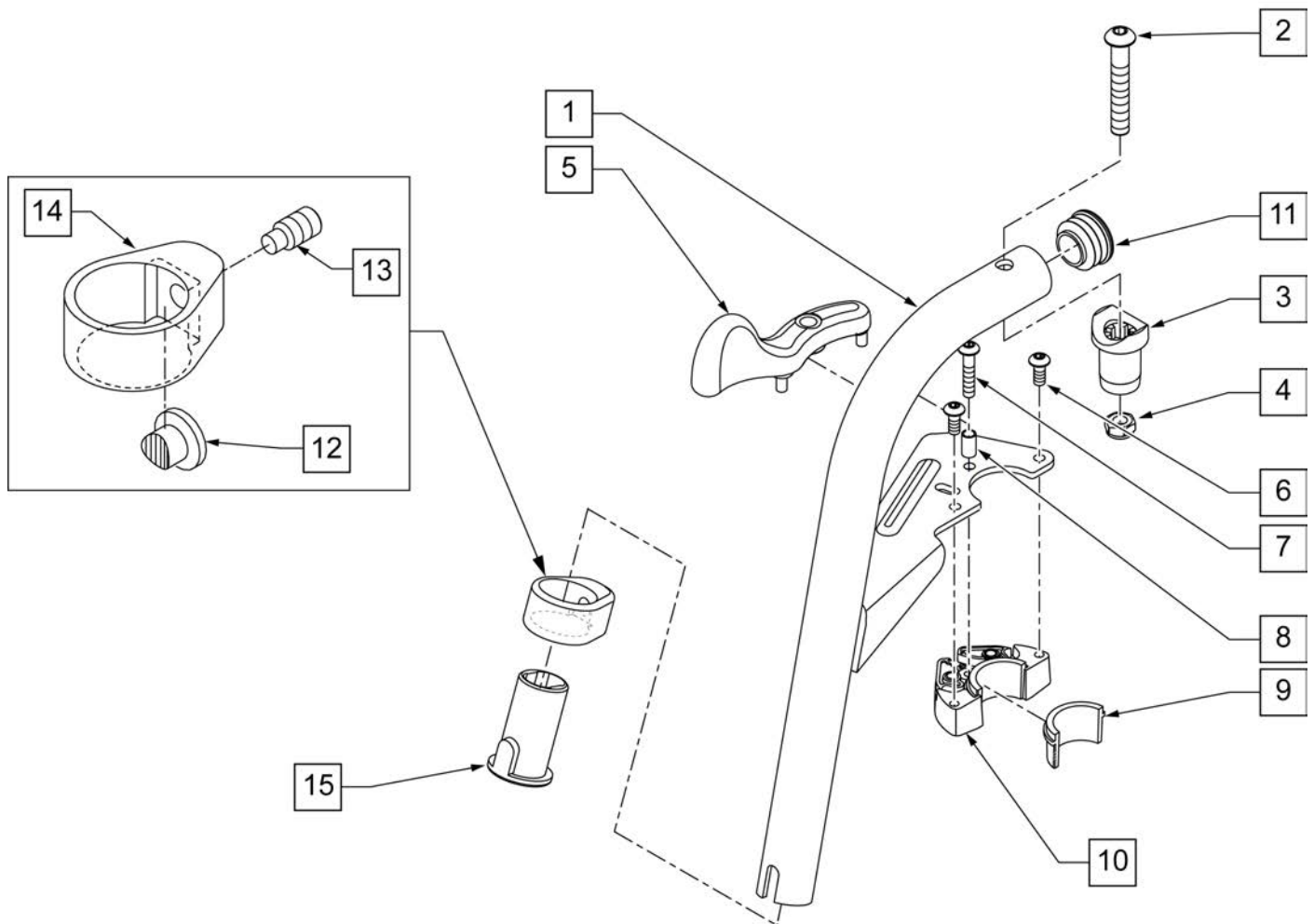

Pos.	Item Number	Description	Remarks	USD
2	106983	5/16-18 X 2 BH C/S SHT THREAD		5.00
3	561197	PIVOT SADDLE PLASTIC	Includes nylock nut	2.00
5	052296	SWING I/O LATCH LEVER RIGHT	Right	11.25
5	052297	SWING I/O LATCH LEVER LEFT	Left	11.25
7	230494	M5-30 BTTN HD SOC C/S BLK		2.57
8	052054	BUSHING PIVOT HANGER SWING I/O		2.00
11	561062	1" INSERT PLUG .065 RIBS		2.00
15	104160	BUSHING FOOTREST		5.00
1-15	107675S	ASSEMBLY LEFT	SPECIFY COLOR,LEFT	105.00
1-15	107674S	ASSEMBLY RIGHT	SPECIFY COLOR,RIGHT	105.00
12-15	107485	FOOTREST CLAMP ASSM		30.00
6,9,10	120976	ASSY LATCH BLOCK ANTI LIFT		45.00

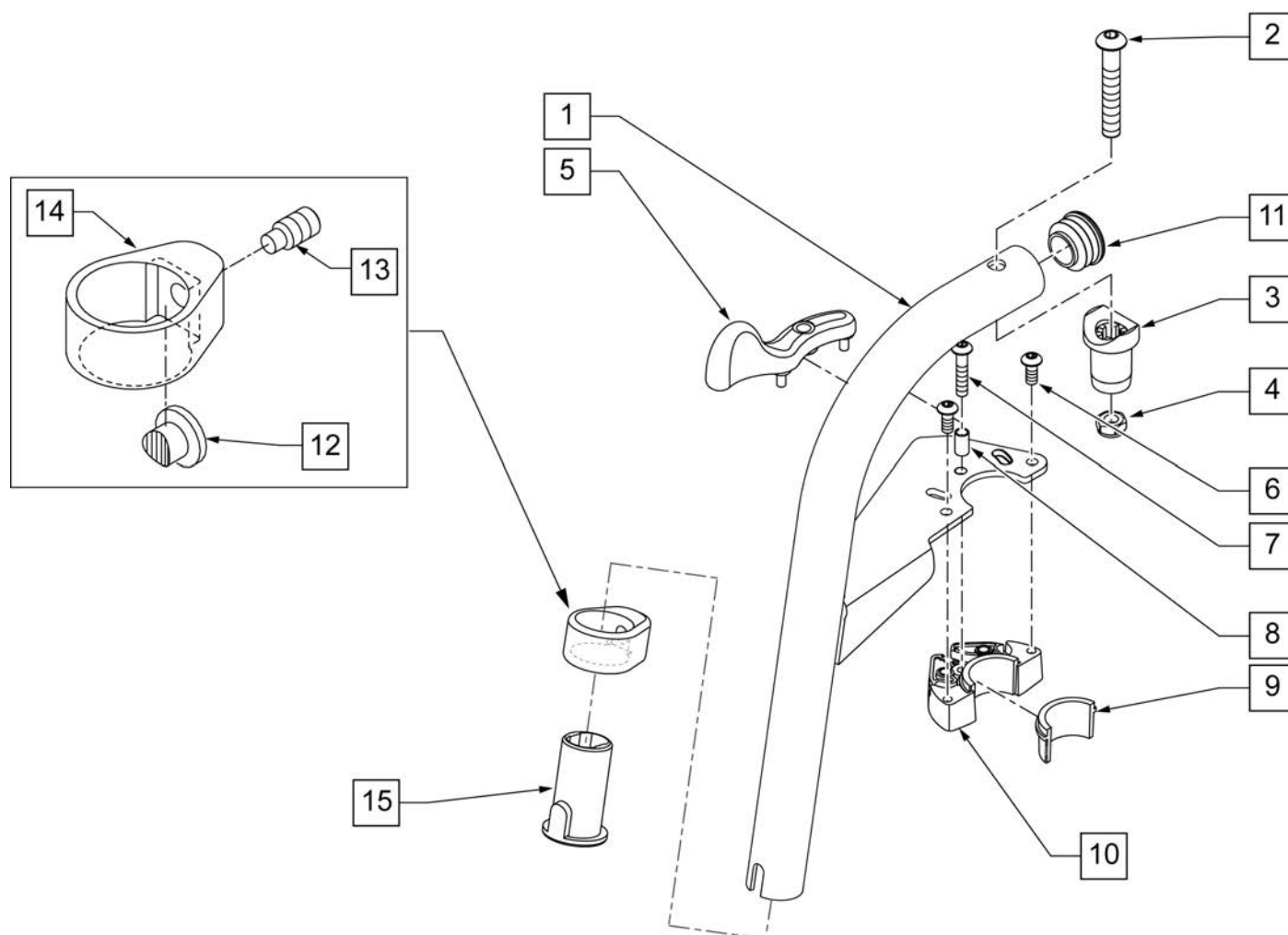

Pos.	Item Number	Description	Remarks	USD
2	106983	5/16-18 X 2 BH C/S SHT THREAD		5.00
3,4	561197	PIVOT SADDLE PLASTIC		2.00
5	052296	SWING I/O LATCH LEVER RIGHT	Right	11.25
5	052297	SWING I/O LATCH LEVER LEFT	Left	11.25
6	102259-001	BHCS M5 X .8 X 12 W/PL INORG		2.00
7	230494	M5-30 BTTN HD SOC C/S BLK		2.57
8	052054	BUSHING PIVOT HANGER SWING I/O		2.00
11	561062	1" INSERT PLUG .065 RIBS		2.00
15	104160	BUSHING FOOTREST		5.00
1-15	107673S	ASSEMBLY LEFT	SPECIFY COLOR,LEFT	105.00
1-15	107672S	ASSEMBLY RIGHT	SPECIFY COLOR,RIGHT	105.00
12-15	107485	FOOTREST CLAMP ASSM		30.00
6,9,10	120976	ASSY LATCH BLOCK ANTI LIFT		45.00

COMPOSITE FOOTPLATE

[09/2011]

NOTE: FOOTPLATE ASSEMBLIES ARE SET TO FOOTREST WIDTH. PLEASE SEE "FOOTPLATE FORMULA" TABLE FOR SEAT WIDTH CONVERSION.

INSET FORMULA
0" Inset (Seat Width) = Footrest Width
1" Inset (Seat Width) -2 = Footrest Width
"V" Hanger (Seat Width) -4 = Footrest Width

Composite Footplates w/ Ext & HL for Swing I/O Hangers
(Order 1 each)

Width	60°, 70° & 70° Hemi Style Hangers (Bent Extension)			80° Style hanger ELR (Straight Extension)			70° V Style Hangers	
	Short	Medium	Long	Short	Medium	Long	Medium	Long
13" Right	107746	107760	107774	107704	107718	107732	107788	107802
13" Left	107753	107767	107781	107711	107725	107739	107795	107809
14" Right	107747	107761	107775	107705	107719	107733	107789	107803
14" Left	107754	107768	107782	107712	107726	107740	107796	107810
15" Right	107748	107762	107776	107706	107720	107734	107790	107804
15" Left	107755	107769	107783	107713	107727	107741	107797	107811
16" Right	107749	107763	107777	107707	107721	107735	107791	107805
16" Left	107756	107770	107784	107714	107728	107742	107798	107812
17" Right	107750	107764	107778	107708	107722	107736	107792	107806
17" Left	107757	107771	107785	107715	107729	107743	107799	107813
18" Right	107751	107765	107779	107709	107723	107737	107793	107807
18" Left	107758	107772	107786	107716	107730	107744	107800	107814
19"-20" Rt	107752	107766	107780	107710	107724	107738	V STYLE IS NOT AVAILABLE IN THESE SEAT WIDTHS	
19"-20" Lt	107759	107773	107787	107717	107731	107745		
21"-22" Rt	111528	111530	111532	111522	111524	111526		
21"-22" Lt	111529	111531	111533	111523	111525	111527		

FOOTPLATE MEASURING GUIDE

PART NO	DESCRIPTION	DIM - A-	DIM - B-
063216-006	FOOTPLATE COMPOSITE 157 R	6 1/8"	6"
063216-012	FOOTPLATE COMPOSITE 157 L	6 1/8"	6"
063216-015	FOOTPLATE COMPOSITE 170 R	6 5/8"	6"
063216-018	FOOTPLATE COMPOSITE 170 L	6 5/8"	6"
063216-005	FOOTPLATE COMPOSITE 182 R	7 1/8"	6"
063216-011	FOOTPLATE COMPOSITE 182 L	7 1/8"	6"
063216-014	FOOTPLATE COMPOSITE 195 R	7 5/8"	6"
063216-017	FOOTPLATE COMPOSITE 195 L	7 5/8"	6"
063216-004	FOOTPLATE COMPOSITE 207 R	8 1/8"	6"
063216-010	FOOTPLATE COMPOSITE 207 L	8 1/8"	6"
063216-013	FOOTPLATE COMPOSITE 220 R	8 5/8"	6 1/4"
063216-016	FOOTPLATE COMPOSITE 220 L	8 5/8"	6 1/4"
063216-003	FOOTPLATE COMPOSITE 232 R	9 1/8"	6 1/4"
063216-009	FOOTPLATE COMPOSITE 232 L	9 1/8"	6 1/4"
063216-002	FOOTPLATE COMPOSITE 257 R	10 1/8"	6 1/4"
063216-008	FOOTPLATE COMPOSITE 257 L	10 1/8"	6 1/4"
063216-001	FOOTPLATE COMPOSITE 282 R	11 1/8"	6 1/4"
063216-007	FOOTPLATE COMPOSITE 282 L	11 1/8"	6 1/4"

Pos.	Item Number	Description	Remarks	USD
1A-5	120660	SHT STRT EXT COMP HDWR	80 deg hangers + ELR only	20.00
1B-5	120661	MED STRT EXT COMP HDWR	80 deg hangers + ELR only	20.00
1C-5	120662	LNG STRT EXT COMP HDWR	80 deg hangers + ELR only	20.00
1D-5	120663	SHT BNT EXT COMP HDWR	60,70,70 Hemi Hangers	20.00
1E-5	120664	MED BNT EXT COMP HDWR	60,70,70 Hemi Hangers	20.00
1F-5	120665	LNG BNT EXT COMP HDWR	60,70,70 Hemi Hangers	20.00
1G-5	120666	SHT V EXT R COMP HDWR	70 deg V hangers only-LEFT	20.00
1G-5	120667	SHT V EXT L COMP HDWR	70 deg V hangers only-RIGHT	20.00
1H-5	120668	LNG V EXT R COMP HDWR	70 deg V hangers only-LEFT	20.00
1H-5	120669	LNG V EXT L COMP HDWR	70 deg V hangers only-RIGHT	20.00
2A	063216-006	COMPOSITE FOOTPLATE RIGHT	RIGHT 13" Footrest Width	20.00
2A	063216-012	COMPOSITE FOOTPLATE LEFT	LEFT 13" Footrest Width	20.00
2B	063216-015	FOOTPLATE COMPOSITE RIGHT	RIGHT 14" Footrest Width	20.00

Pos.	Item Number	Description	Remarks	USD
2B	063216-018	FOOTPLATE COMPOSITE LEFT	LEFT 14" Footrest Width	20.00
2C	063216-005	COMPOSITE FOOTPLATE RIGHT	RIGHT 15" Footrest Width	20.00
2C	063216-011	COMPOSITE FOOTPLATE LEFT	LEFT 15" Footrest Width	20.00
2D	063216-014	FOOTPLATE COMPOSITE RIGHT	RIGHT 16" Footrest Width	20.00
2D	063216-017	FOOTPLATE COMPOSITE LEFT	LEFT 16" Footrest Width	20.00
2E	063216-004	COMPOSITE FOOTPLATE RIGHT	RIGHT 17" Footrest Width	20.00
2E	063216-010	COMPOSITE FOOTPLATE LEFT	LEFT 17" Footrest Width	20.00
2F	063216-013	FOOTPLATE COMPOSITE RIGHT	RIGHT 18" Footrest Width	20.00
2F	063216-016	FOOTPLATE COMPOSITE LEFT	LEFT 18" Footrest Width	20.00
2G	063216-003	COMPOSITE FOOTPLATE RIGHT	RIGHT 19" & 20" Footrest Width	20.00
2G	063216-009	COMPOSITE FOOTPLATE LEFT	LEFT 19" & 20" Footrest Width	20.00
2H	063216-002	FOOTPLATE COMPOSITE RIGHT	RIGHT 21" & 22" Footrest Width	20.00
2H	063216-008	FOOTPLATE COMPOSITE LEFT	LEFT 21" & 22" Footrest Width	20.00
3	104377	3/4" SADDLE THIN		5.00
4	103175	SHCS M6 X 1 X 40 INORG		5.00
5	104381	NUT HEX NYLOCK M6X1.0	Inserted in bottom of footplate.	5.00
6	104328	HEEL LOOP PEG		5.00
7A	104380-001	ADJUSTABLE FOOT REST STRAP	14" wide	7.50
7B	104380-002	ADJUSTABLE FOOT REST STRAP	15-16" wide	7.50
7C	104380-003	ADJUSTABLE FOOT REST STRAP	17-18" wide	7.50
7D	104380-004	ADJUSTABLE FOOT REST STRAP	19-22" wide	7.50
8	102268-001	HHCS M6 X 1 X 25		10.00
12	104160	BUSHING FOOTREST		5.00
9-12	107485	FOOTREST CLAMP ASSM		30.00
>>>>		60°,70°,70° HEMI HANGERS - SHORT EXT - RIGHT		
1D,2A,3-6,7A,8	107746	COMP BNT SHT EXT ASM 13" W R	Right- 70D- 13" W- SHORT	65.00
1D,2B,3-6,7A,8	107747	COMP BNT SHT EXT ASM 14" W R	Right- 70D- 14" W- SHORT	65.00
1D,2C,3-6,7B,8	107748	COMP BNT SHT EXT ASM 15" W R	Right- 70D- 15" W- SHORT	65.00
1D,2D,3-6,7B,8	107749	COMP BNT SHT EXT ASM 16" W R	Right- 70D- 16" W- SHORT	65.00
1D,2E,3-6,7C,8	107750	COMP BNT SHT EXT ASM 17" W R	Right- 70D- 17" W- SHORT	65.00
1D,2F,3-6,7C,8	107751	COMP BNT SHT EXT ASM 18" W R	Right- 70D- 18" W- SHORT	65.00
1D,2G,3-6,7D,8	107752	COMP BNT SHT EXT ASM 19 & 20"W R	Right- 70D- 19 & 20" W- SHORT	65.00
1D,2H,3-6,7D,8	111528	COMP BNT SHT EXT ASM 21 & 22"W R	Right- 70D- 21 & 22" W- SHORT	65.00
>>>>		60°,70°,70° HEMI HANGERS - SHORT EXT - LEFT		
1D,2A,3-6,7A,8	107753	COMP BNT SHT EXT ASM 13" W L	Left- 70D- 13 W- SHORT	65.00
1D,2B,3-6,7A,8	107754	COMP BNT SHT EXT ASM 14" W L	Left- 70D- 14 W- SHORT	65.00
1D,2C,3-6,7B,8	107755	COMP BNT SHT EXT ASM 15" W L	Left- 70D- 15 W- SHORT	65.00
1D,2D,3-6,7B,8	107756	COMP BNT SHT EXT ASM 16" W L	Left- 70D- 16 W- SHORT	65.00
1D,2E,3-6,7C,8	107757	COMP BNT SHT EXT ASM 17" W L	Left- 70D- 17 W- SHORT	65.00
1D,2F,3-6,7C,8	107758	COMP BNT SHT EXT ASM 18" W L	Left- 70D- 18 W- SHORT	65.00
1D,2G,3-6,7D,8	107759	COMP BNT SHT EXT ASM 19 & 20"W L	Left- 70D- 19 & 20" W- SHORT	65.00
1D,2H,3-6,7D,8	111529	COMP BNT SHT EXT ASM 21 & 22"W L	Left- 70D- 21 & 22" W- SHORT	65.00
>>>>		60°,70°,70° HEMI HANGERS - MED EXT - RIGHT		
1E,2A,3-6,7A,8	107760	COMP BNT MED EXT ASM 13" W R	Right- 70D- 13" W- MED	65.00
1E,2B,3-6,7A,8	107761	COMP BNT MED EXT ASM 14" W R	Right- 70D- 14" W- MED	65.00
1E,2C,3-6,7B,8	107762	COMP BNT MED EXT ASM 15" W R	Right- 70D- 15" W- MED	65.00
1E,2D,3-6,7B,8	107763	COMP BNT MED EXT ASM 16" W R	Right- 70D- 16" W- MED	65.00
1E,2E,3-6,7C,8	107764	COMP BNT MED EXT ASM 17" W R	Right- 70D- 17" W- MED	65.00
1E,2F,3-6,7C,8	107765	COMP BNT MED EXT ASM 18" W R	Right- 70D- 18" W- MED	65.00
1E,2G,3-6,7D,8	107766	COMP BNT MED EXT ASM 19 & 20"W R	Right- 70D- 19 & 20" W- MED	65.00
1E,2H,3-6,7D,8	111530	COMP BNT MED EXT ASM 21 & 22"W R	Right- 70D- 21 & 22" W- MED	65.00
>>>>		60°,70°,70° HEMI HANGERS - MED EXT - LEFT		
1E,2A,3-6,7A,8	107767	COMP BNT MED EXT ASM 13" W L	Left- 70D- 13 W- MED	65.00
1E,2B,3-6,7A,8	107768	COMP BNT MED EXT ASM 14" W L	Left- 70D- 14 W- MED	65.00
1E,2C,3-6,7B,8	107769	COMP BNT MED EXT ASM 15" W L	Left- 70D- 15 W- MED	65.00
1E,2D,3-6,7B,8	107770	COMP BNT MED EXT ASM 16" W L	Left- 70D- 16 W- MED	65.00

Pos.	Item Number	Description	Remarks	USD
1E,2E,3-6,7C,8	107771	COMP BNT MED EXT ASM 17" W L	Left- 70D- 17 W- MED	65.00
1E,2F,3-6,7C,8	107772	COMP BNT MED EXT ASM 18" W L	Left- 70D- 18 W- MED	65.00
1E,2G,3-6,7D,8	107773	COMP BNT MED EXT ASM 19 & 20"W L	Left- 70D- 19 & 20" W- MED	65.00
1E,2H,3-6,7D,8	111531	COMP BNT MED EXT ASM 21 & 22"W L	Left- 70D- 21 & 22" W- MED	65.00
>>>>		60°,70°,70° HEMI HANGERS - LONG EXT - RIGHT		
1F,2A,3-6,7A,8	107774	COMP BNT LONG EXT ASM 13" W R	Right- 70D- 13" W- LONG	65.00
1F,2B,3-6,7A,8	107775	COMP BNT LONG EXT ASM 14" W R	Right- 70D- 14" W- LONG	65.00
1F,2C,3-6,7B,8	107776	COMP BNT LONG EXT ASM 15" W R	Right- 70D- 15" W- LONG	65.00
1F,2D,3-6,7B,8	107777	COMP BNT LONG EXT ASM 16" W R	Right- 70D- 16" W- LONG	65.00
1F,2E,3-6,7C,8	107778	COMP BNT LONG EXT ASM 17" W R	Right- 70D- 17" W- LONG	65.00
1F,2F,3-6,7C,8	107779	COMP BNT LONG EXT ASM 18" W R	Right- 70D- 18" W- LONG	65.00
1F,2G,3-6,7D,8	107780	COMP BNT LONG EXT ASM 19 & 20"W R	Right- 70D- 19 & 20" W- LONG	65.00
1F,2H,3-6,7D,8	111532	COMP BNT LONG EXT ASM 21 & 22"W R	Right- 70D- 21 & 22" W- LONG	65.00
>>>>		60°,70°,70° HEMI HANGERS - LONG EXT - LEFT		
1F,2A,3-6,7A,8	107781	COMP BNT LONG EXT ASM 13" W L	Left- 70D- 13 W- LONG	65.00
1F,2B,3-6,7A,8	107782	COMP BNT LONG EXT ASM 14" W L	Left- 70D- 14 W- LONG	65.00
1F,2C,3-6,7B,8	107783	COMP BNT LONG EXT ASM 15" W L	Left- 70D- 15 W- LONG	65.00
1F,2D,3-6,7B,8	107784	COMP BNT LONG EXT ASM 16" W L	Left- 70D- 16 W- LONG	65.00
1F,2E,3-6,7C,8	107785	COMP BNT LONG EXT ASM 17" W L	Left- 70D- 17 W- LONG	65.00
1F,2F,3-6,7C,8	107786	COMP BNT LONG EXT ASM 18" W L	Left- 70D- 18 W- LONG	65.00
1F,2G,3-6,7D,8	107787	COMP BNT LONG EXT ASM 19 & 20"W L	Left- 70D- 19 & 20" W- LONG	65.00
1F,2H,3-6,7D,8	111533	COMP BNT LONG EXT ASM 21 & 22"W L	Left- 70D- 21 & 22" W- LONG	65.00
>>>>		70 DEGREE "V" HANGERS - MED EXT - RIGHT		
1G,2A,3-8	107788	COMP V MED EXT ASM 13W R	Right- 70D V hangers only 13 W- MED	65.00
1G,2B,3-8	107789	COMP V MED EXT ASM 14W R	Right- 70D V hangers only 14 W- MED	65.00
1G,2C,3-8	107790	COMP V MED EXT ASM 15W R	Right- 70D V hangers only 15 W- MED	65.00
1G,2D,3-8	107791	COMP V MED EXT ASM 16W R	Right- 70D V hangers only 16 W- MED	65.00
1G,2E,3-8	107792	COMP V MED EXT ASM 17W R	Right- 70D V hangers only 17 W- MED	65.00
1G,2F,3-8	107793	COMP V MED EXT ASM 18W R	Right- 70D V hangers only 18 W- MED	65.00
>>>>		70 DEGREE "V" HANGERS - MED EXT - LEFT		
1G,2A,3-8	107795	COMP V MED EXT ASM 13W L	Left- 70D V hangers only 13 W- MED	65.00
1G,2B,3-8	107796	COMP V MED EXT ASM 14W L	Left- 70D V hangers only 14 W- MED	65.00
1G,2C,3-8	107797	COMP V MED EXT ASM 15W L	Left- 70D V hangers only 15 W- MED	65.00
1G,2D,3-8	107798	COMP V MED EXT ASM 16W L	Left- 70D V hangers only 16 W- MED	65.00
1G,2E,3-8	107799	COMP V MED EXT ASM 17W L	Left- 70D V hangers only 17 W- MED	65.00
1G,2F,3-8	107800	COMP V MED EXT ASM 18W L	Left- 70D V hangers only 18 W- MED	65.00
>>>>		70 DEGREE "V" HANGERS - LONG EXT - RIGHT		
1H,2A,3-8	107802	COMP V LNG EXT ASM 13W R	Right- 70D V hangers only 13 W- LONG	65.00
1H,2B,3-8	107803	COMP V LNG EXT ASM 14W R	Right- 70D V hangers only 14 W- LONG	65.00
1H,2C,3-8	107804	COMP V LNG EXT ASM 15W R	Right- 70D V hangers only 15 W- LONG	65.00
1H,2D,3-8	107805	COMP V LNG EXT ASM 16W R	Right- 70D V hangers only 16 W- LONG	65.00
1H,2E,3-8	107806	COMP V LNG EXT ASM 17W R	Right- 70D V hangers only 17 W- LONG	65.00
1H,2F,3-8	107807	COMP V LNG EXT ASM 18W R	Right- 70D V hangers only 18 W- LONG	65.00
>>>>		70 DEGREE "V" HANGERS - LONG EXT - LEFT		
1H,2A,3-8	107809	COMP V LNG EXT ASM 13W L	Left- 70D V hangers only 13 W- LONG	65.00
1H,2B,3-8	107810	COMP V LNG EXT ASM 14W L	Left- 70D V hangers only 14 W- LONG	65.00
1H,2C,3-8	107811	COMP V LNG EXT ASM 15W L	Left- 70D V hangers only 15 W- LONG	65.00
1H,2D,3-8	107812	COMP V LNG EXT ASM 16W L	Left- 70D V hangers only 16 W- LONG	65.00
1H,2E,3-8	107813	COMP V LNG EXT ASM 17W L	Left- 70D V hangers only 17 W- LONG	65.00
1H,2F,3-8	107814	COMP V LNG EXT ASM 18W L	Left- 70D V hangers only 18 W- LONG	65.00
>>>>		80 DEGREE HANGERS - SHORT EXT - RIGHT		
1A,2A,3-6,7A,8	107704	COMP STRT SHT EXT ASM 13"W R	Right- 80D- 13" W- SHORT	65.00
1A,2B,3-6,7A,8	107705	COMP STRT SHT EXT ASM 14"W R	Right- 80D- 14" W- SHORT	65.00
1A,2C,3-6,7B,8	107706	COMP STRT SHT EXT ASM 15"W R	Right- 80D- 15" W- SHORT	65.00
1A,2D,3-6,7B,8	107707	COMP STRT SHT EXT ASM 16"W R	Right- 80D- 16" W- SHORT	65.00
1A,2E,3-6,7C,8	107708	COMP STRT SHT EXT ASM 17"W R	Right- 80D- 17" W- SHORT	65.00


Pos.	Item Number	Description	Remarks	USD
1A,2F,3-6,7C,8	107709	COMP STRT SHT EXT ASM 18"W R	Right- 80D- 18" W- SHORT	65.00
1A,2G,3-6,7D,8	107710	COMP STRT SHT EXT ASM 19 & 20"W R	Right- 80D- 19 & 20" W- SHORT	65.00
1A,2H,3-6,7D,8	111522	COMP STRT SHT EXT ASM 21 & 22"W R	Right- 80D- 21 & 22" W- SHORT	65.00
>>>>		80 DEGREE HANGERS - SHORT EXT - LEFT		
1A,2A,3-6,7A,8	107711	COMP STRT SHT EXT ASM 13"W L	LEFT- 80D- 13" W- SHORT	65.00
1A,2B,3-6,7A,8	107712	COMP STRT SHT EXT ASM 14"W L	LEFT- 80D- 14" W- SHORT	65.00
1A,2C,3-6,7B,8	107713	COMP STRT SHT EXT ASM 15"W L	LEFT- 80D- 15" W- SHORT	65.00
1A,2D,3-6,7B,8	107714	COMP STRT SHT EXT ASM 16"W L	LEFT- 80D- 16" W- SHORT	65.00
1A,2E,3-6,7C,8	107715	COMP STRT SHT EXT ASM 17"W L	LEFT- 80D- 17" W- SHORT	65.00
1A,2F,3-6,7C,8	107716	COMP STRT SHT EXT ASM 18"W L	LEFT- 80D- 18" W- SHORT	65.00
1A,2G,3-6,7D,8	107717	COMP STRT SHT EXT ASM 19 & 20"W L	LEFT- 80D- 19 & 20" W- SHORT	65.00
1A,2H,3-6,7D,8	111523	COMP STRT SHT EXT ASM 21 & 22"W L	LEFT- 80D- 21 & 22" W- SHORT	65.00
>>>>		80 DEGREE HANGERS - MED EXT - RIGHT		
1B,2A,3-6,7A,8	107718	COMP STRT MED EXT ASM 13"W R	Right- 80D- 13" W- MED	65.00
1B,2B,3-6,7A,8	107719	COMP STRT MED EXT ASM 14"W R	Right- 80D- 14" W- MED	65.00
1B,2C,3-6,7B,8	107720	COMP STRT MED EXT ASM 15"W R	Right- 80D- 15" W- MED	65.00
1B,2D,3-6,7B,8	107721	COMP STRT MED EXT ASM 16"W R	Right- 80D- 16" W- MED	65.00
1B,2E,3-6,7C,8	107722	COMP STRT MED EXT ASM 17"W R	Right- 80D- 17" W- MED	65.00
1B,2F,3-6,7C,8	107723	COMP STRT MED EXT ASM 18"W R	Right- 80D- 18" W- MED	65.00
1B,2G,3-6,7D,8	107724	COMP STRT MED EXT ASM 19 & 20"W R	Right- 80D- 19 & 20" W- MED	65.00
1B,2H,3-6,7D,8	111524	COMP STRT MED EXT ASM 21 & 22"W R	Right- 80D- 21 & 22" W- MED	65.00
>>>>		80 DEGREE HANGERS - MED EXT - LEFT		
1B,2A,3-6,7A,8	107725	COMP STRT MED EXT ASM 13"W L	LEFT- 80D- 13" W- MED	65.00
1B,2B,3-6,7A,8	107726	COMP STRT MED EXT ASM 14"W L	LEFT- 80D- 14" W- MED	65.00
1B,2C,3-6,7B,8	107727	COMP STRT MED EXT ASM 15"W L	LEFT- 80D- 15" W- MED	65.00
1B,2D,3-6,7B,8	107728	COMP STRT MED EXT ASM 16"W L	LEFT- 80D- 16" W- MED	65.00
1B,2E,3-6,7C,8	107729	COMP STRT MED EXT ASM 17"W L	LEFT- 80D- 17" W- MED	65.00
1B,2F,3-6,7C,8	107730	COMP STRT MED EXT ASM 18"W L	LEFT- 80D- 18" W- MED	65.00
1B,2G,3-6,7D,8	107731	COMP STRT MED EXT ASM 19 & 20"W L	LEFT- 80D- 19 & 20" W- MED	65.00
1B,2H,3-6,7D,8	111525	COMP STRT MED EXT ASM 21 & 22"W L	LEFT- 80D- 21 & 22" W- MED	65.00
>>>>		80 DEGREE HANGERS - LONG EXT - RIGHT		
1C,2A,3-6,7A,8	107732	COMP STRT LONG EXT ASM 13"W R	Right- 80D- 13" W- LONG	65.00
1C,2B,3-6,7A,8	107733	COMP STRT LONG EXT ASM 14"W R	Right- 80D- 14" W- LONG	65.00
1C,2C,3-6,7B,8	107734	COMP STRT LONG EXT ASM 15"W R	Right- 80D- 15" W- LONG	65.00
1C,2D,3-6,7B,8	107735	COMP STRT LONG EXT ASM 16"W R	Right- 80D- 16" W- LONG	65.00
1C,2E,3-6,7C,8	107736	COMP STRT LONG EXT ASM 17"W R	Right- 80D- 17" W- LONG	65.00
1C,2F,3-6,7C,8	107737	COMP STRT LONG EXT ASM 18"W R	Right- 80D- 18" W- LONG	65.00
1C,2G,3-6,7D,8	107738	COMP STRT LONG EXT ASM 19 & 20"W R	Right- 80D- 19 & 20" W- LONG	65.00
1C,2H,3-6,7D,8	111526	COMP STRT LONG EXT ASM 21 & 22"W R	Right- 80D- 21 & 22" W- LONG	65.00
>>>>		80 DEGREE HANGERS - LONG EXT - LEFT		
1C,2A,3-6,7A,8	107739	COMP STRT LONG EXT ASM 13"W L	LEFT- 80D- 13" W- LONG	65.00
1C,2B,3-6,7A,8	107740	COMP STRT LONG EXT ASM 14"W L	LEFT- 80D- 14" W- LONG	65.00
1C,2C,3-6,7B,8	107741	COMP STRT LONG EXT ASM 15"W L	LEFT- 80D- 15" W- LONG	65.00
1C,2D,3-6,7B,8	107742	COMP STRT LONG EXT ASM 16"W L	LEFT- 80D- 16" W- LONG	65.00
1C,2E,3-6,7C,8	107743	COMP STRT LONG EXT ASM 17"W L	LEFT- 80D- 17" W- LONG	65.00
1C,2F,3-6,7C,8	107744	COMP STRT LONG EXT ASM 18"W L	LEFT- 80D- 18" W- LONG	65.00
1C,2G,3-6,7D,8	107745	COMP STRT LONG EXT ASM 19 & 20"W L	LEFT- 80D- 19 & 20" W- LONG	65.00
1C,2H,3-6,7D,8	111527	COMP STRT LONG EXT ASM 21 & 22"W L	LEFT- 80D- 21 & 22" W- LONG	65.00

NOTE: FOOTPLATE ASSEMBLIES ARE SET TO FOOTREST WIDTH. PLEASE SEE "FOOTPLATE FORMULA" TABLE FOR SEAT WIDTH CONVERSION.


INSET FORMULA
 0" Inset (Seat Width) = Footrest Width
 1" Inset (Seat Width) -2 = Footrest Width
 "V" Hanger (Seat Width) -4 = Footrest Width

Composite Angle Adjustable "CAA" Footplate w/ Ext & HL for Swing I/O Hangers (Order 1 each)								
Width	60°, 70° & 70° Hemi Style Hangers (Bent Extension)			80° Style hanger (Straight Extension)			70° V Style Hangers	
	Short	Medium	Long	Short	Medium	Long	Medium	Long
15" Right	107858	107872	107886	107816	107830	107844	107900	107914
15" Left	107865	107879	107893	107823	107837	107851	107907	107921
16" Right	107859	107873	107887	107817	107831	107845	107901	107915
16" Left	107866	107880	107894	107824	107838	107852	107908	107922
17" Right	107860	107874	107888	107818	107832	107846	107902	107916
17" Left	107867	107881	107895	107825	107839	107853	107909	107923
18" Right	107861	107875	107889	107819	107833	107847	107903	107917
18" Left	107868	107882	107896	107826	107840	107854	107910	107924
19" Right	107862	107876	107890	107820	107834	107848	V STYLE IS NOT AVAILABLE IN THESE SEAT WIDTHS	
19" Left	107869	107883	107897	107827	107841	107855		
20" Right	107863	107877	107891	107821	107835	107849		
20" Left	107870	107884	107898	107828	107842	107856		
21"-22" Rt	107864	107878	107892	107822	107836	107850		
21"-22" Lt	107871	107885	107899	107829	107843	107857		

FOOTPLATE MEASURING GUIDE

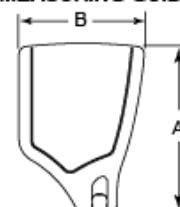

PART NO	DESCRIPTION	DIM -A-	DIM -B-
063216-006	FOOTPLATE COMPOSITE 157 R	6 1/8"	6"
063216-012	FOOTPLATE COMPOSITE 157 L	6 1/8"	6"
063216-015	FOOTPLATE COMPOSITE 170 R	6 5/8"	6"
063216-018	FOOTPLATE COMPOSITE 170 L	6 5/8"	6"
063216-005	FOOTPLATE COMPOSITE 182 R	7 1/8"	6"
063216-011	FOOTPLATE COMPOSITE 182 L	7 1/8"	6"
063216-014	FOOTPLATE COMPOSITE 195 R	7 5/8"	6"
063216-017	FOOTPLATE COMPOSITE 195 L	7 5/8"	6"
063216-004	FOOTPLATE COMPOSITE 207 R	8 1/8"	6"
063216-010	FOOTPLATE COMPOSITE 207 L	8 1/8"	6"
063216-013	FOOTPLATE COMPOSITE 220 R	8 5/8"	6 1/4"
063216-016	FOOTPLATE COMPOSITE 220 L	8 5/8"	6 1/4"
063216-003	FOOTPLATE COMPOSITE 232 R	9 1/8"	6 1/4"
063216-009	FOOTPLATE COMPOSITE 232 L	9 1/8"	6 1/4"
063216-002	FOOTPLATE COMPOSITE 257 R	10 1/8"	6 1/4"
063216-008	FOOTPLATE COMPOSITE 257 L	10 1/8"	6 1/4"
063216-001	FOOTPLATE COMPOSITE 282 R	11 1/8"	6 1/4"
063216-007	FOOTPLATE COMPOSITE 282 L	11 1/8"	6 1/4"

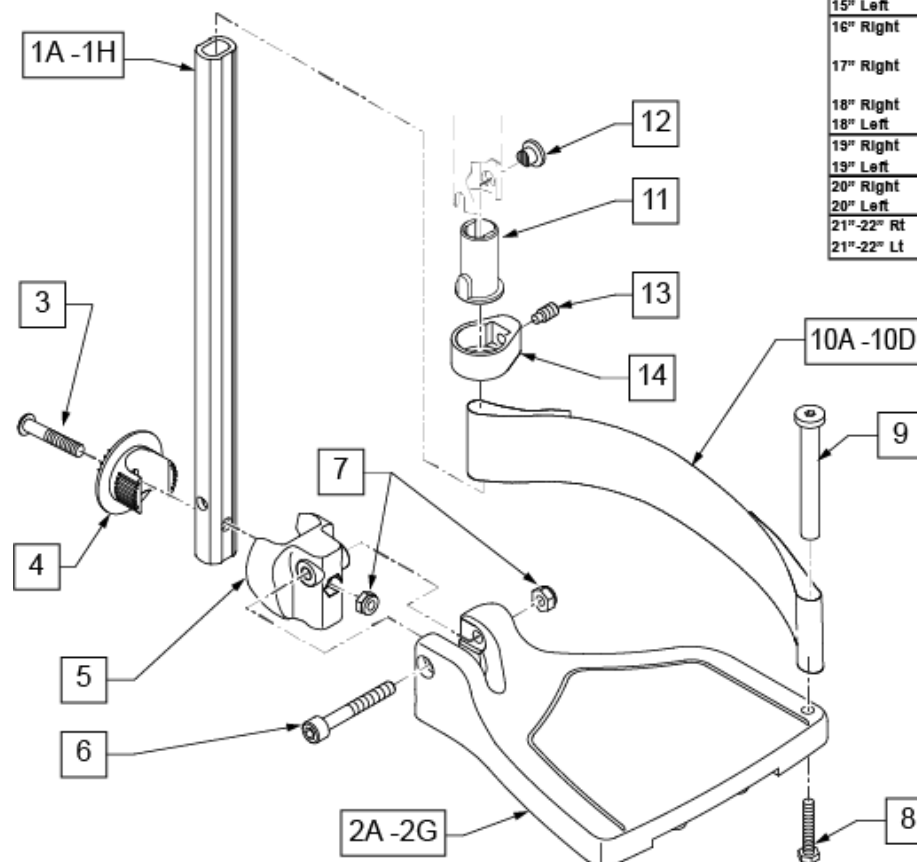

Pos.	Item Number	Description	Remarks	USD
1A,6,7	120660	SHT STRT EXT COMP HDWR	80 deg hangers only	20.00
1B,6,7	120661	MED STRT EXT COMP HDWR	80 deg hangers only	20.00
1C,6,7	120662	LNG STRT EXT COMP HDWR	80 deg hangers only	20.00
1D,6,7	120663	SHT BNT EXT COMP HDWR	60,70,70 Hemi Hangers	20.00
1E,6,7	120664	MED BNT EXT COMP HDWR	60,70,70 Hemi Hangers	20.00
1F,6,7	120665	LNG BNT EXT COMP HDWR	60,70,70 Hemi Hangers	20.00
1G,6,7	120666	SHT V EXT R COMP HDWR	70 deg V hangers only-LEFT	20.00
1G,6,7	120667	SHT V EXT L COMP HDWR	70 deg V hangers only-RIGHT	20.00
1H,6,7	120668	LNG V EXT R COMP HDWR	70 deg V hangers only-LEFT	20.00
1H,6,7	120669	LNG V EXT L COMP HDWR	70 deg V hangers only-RIGHT	20.00
2A	063216-006	COMPOSITE FOOTPLATE RIGHT	RIGHT 15" Footrest Width	20.00
2A	063216-012	COMPOSITE FOOTPLATE LEFT	LEFT 15" Footrest Width	20.00
2B	063216-015	COMPOSITE FOOTPLATE RIGHT	RIGHT 16" Footrest Width	20.00

Pos.	Item Number	Description	Remarks	USD
2B	063216-018	COMPOSITE FOOTPLATE LEFT	LEFT 16" Footrest Width	20.00
2C	063216-005	COMPOSITE FOOTPLATE RIGHT	RIGHT 17" Footrest Width	20.00
2C	063216-011	COMPOSITE FOOTPLATE LEFT	LEFT 17" Footrest Width	20.00
2D	063216-014	COMPOSITE FOOTPLATE RIGHT	RIGHT 18" Footrest Width	20.00
2D	063216-017	COMPOSITE FOOTPLATE LEFT	LEFT 18" Footrest Width	20.00
2E	063216-004	COMPOSITE FOOTPLATE RIGHT	RIGHT 19" Footrest Width	20.00
2E	063216-010	COMPOSITE FOOTPLATE LEFT	LEFT 19" Footrest Width	20.00
2F	063216-013	COMPOSITE FOOTPLATE RIGHT	RIGHT 20" Footrest Width	20.00
2F	063216-016	COMPOSITE FOOTPLATE LEFT	LEFT 20" Footrest Width	20.00
2G	063216-003	COMPOSITE FOOTPLATE RIGHT	RIGHT 21" & 22" Footrest Width	20.00
2G	063216-009	COMPOSITE FOOTPLATE LEFT	LEFT 21" & 22" Footrest Width	20.00
3	108233	BHCS M6 X 1 X 40		10.00
4	054242	ANGLE COVER		5.00
5	054241	ANGLE SEGMENT		5.00
6	103175	SHCS M6 X 1 X 40 INORG		5.00
7	104381	NUT HEX NYLOCK M6X1.0		5.00
8	102268-001	HHCS M6X1X25	Inserted in bottom of footplate.	10.00
9	104328	HEEL LOOP PEG		5.00
10A	104380-001	ADJUSTABLE FOOT REST STRAP	15" Footrest Width	7.50
10B	104380-002	ADJUSTABLE FOOT REST STRAP	16" Footrest Width	7.50
10C	104380-003	ADJUSTABLE FOOT REST STRAP	17-18" Footrest Width	7.50
10D	104380-004	ADJUSTABLE FOOT REST STRAP	19-22" Footrest Width	7.50
11	104160	BUSHING FOOTREST		5.00
11-14	107485	FOOTREST CLAMP ASSM		30.00
>>>>		60°,70°,70° HEMI HANGERS - SHORT EXT - RIGHT		
1D,2A,3-10A	107858	CAA BNT SHT EXT ASM 15W R	Right- 60° & 60° & 70° 15 W- SHORT	70.00
1D,2B,3-10B	107859	CAA BNT SHT EXT ASM 16W R	Right- 60° & 70° 16 W- SHORT	70.00
1D,2C,3-10B	107860	CAA BNT SHT EXT ASM 17W R	Right- 60° & 70° 17 W- SHORT	70.00
1D,2D,3-10C	107861	CAA BNT SHT EXT ASM 18W R	Right- 60° & 70° 18 W- SHORT	70.00
1D,2E,3-10C	107862	CAA BNT SHT EXT ASM 19W R	Right- 60° & 70° 19 W- SHORT	70.00
1D,2F,3-10D	107863	CAA BNT SHT EXT ASM 20W R	Right- 60° & 70° 20 W- SHORT	70.00
1D,2G,3-10D	107864	CAA BNT SHT EXT ASM 21&22W R	Right- 60° & 70° 21&22 W- SHORT	70.00
>>>>		60°,70°,70° HEMI HANGERS - SHORT EXT - LEFT		
1D,2A,3-10A	107865	CAA BNT SHT EXT ASM 15W L	Left- 60° & 70° 15 W- SHORT	70.00
1D,2B,3-10B	107866	CAA BNT SHT EXT ASM 16W L	Left- 60° & 70° 16 W- SHORT	70.00
1D,2C,3-10B	107867	CAA BNT SHT EXT ASM 17W L	Left- 60° & 70° 17 W- SHORT	70.00
1D,2D,3-10C	107868	CAA BNT SHT EXT ASM 18W L	Left- 60° & 70° 18 W- SHORT	70.00
1D,2E,3-10C	107869	CAA BNT SHT EXT ASM 19W L	Left- 60° & 70° 19 W- SHORT	70.00
1D,2F,3-10D	107870	CAA BNT SHT EXT ASM 20W L	Left- 60° & 70° 20 W- SHORT	70.00
1D,2G,3-10D	107871	CAA BNT SHT EXT ASM 21&22W L	Left- 60° & 70° 21&22 W- SHORT	70.00
>>>>		60°,70°,70° HEMI HANGERS - MED EXT - RIGHT		
1E,2A,3-10A	107872	CAA BNT MED EXT ASM 15W R	Right- 60° & 70° 15 W- MED	70.00
1E,2B,3-10B	107873	CAA BNT MED EXT ASM 16W R	Right- 60° & 70° 16 W- MED	70.00
1E,2C,3-10B	107874	CAA BNT MED EXT ASM 17W R	Right- 60° & 70° 17 W- MED	70.00
1E,2D,3-10C	107875	CAA BNT MED EXT ASM 18W R	Right- 60° & 70° 18 W- MED	70.00
1E,2E,3-10C	107876	CAA BNT MED EXT ASM 19W R	Right- 60° & 70° 19 W- MED	70.00
1E,2F,3-10D	107877	CAA BNT MED EXT ASM 20W R	Right- 60° & 70° 20 W- MED	70.00
1E,2G,3-10D	107878	CAA BNT MED EXT ASM 21&22W R	Right- 60° & 70° 21&22 W- MED	70.00
>>>>		60°,70°,70° HEMI HANGERS - MED EXT - LEFT		
1E,2A,3-10A	107879	CAA BNT MED EXT ASM 15W L	Left- 60° & 70° 15 W- MED	70.00
1E,2B,3-10B	107880	CAA BNT MED EXT ASM 16W L	Left- 60° & 70° 16 W- MED	70.00
1E,2C,3-10B	107881	CAA BNT MED EXT ASM 17W L	Left- 60° & 70° 17 W- MED	70.00
1E,2D,3-10C	107882	CAA BNT MED EXT ASM 18W L	Left- 60° & 70° 18 W- MED	70.00
1E,2E,3-10C	107883	CAA BNT MED EXT ASM 19W L	Left- 60° & 70° 19 W- MED	70.00
1E,2F,3-10D	107884	CAA BNT MED EXT ASM 20W L	Left- 60° & 70° 20 W- MED	70.00
1E,2G,3-10D	107885	CAA BNT MED EXT ASM 21&22W L	Left- 60° & 70° 21&22 W- MED	70.00

Pos.	Item Number	Description	Remarks	USD
>>>>		60°,70°,70° HEMI HANGERS - LONG EXT - RIGHT		
1F,2A,3-10A	107886	CAA BNT LNG EXT ASM 15W R	Right- 60° & 70° 15 W- LONG	70.00
1F,2B,3-10B	107887	CAA BNT LNG EXT ASM 16W R	Right- 60° & 70° 16 W- LONG	70.00
1F,2C,3-10B	107888	CAA BNT LNG EXT ASM 17W R	Right- 60° & 70° 17 W- LONG	70.00
1F,2D,3-10C	107889	CAA BNT LNG EXT ASM 18W R	Right- 60° & 70° 18 W- LONG	70.00
1F,2E,3-10C	107890	CAA BNT LNG EXT ASM 19W R	Right- 60° & 70° 19 W- LONG	70.00
1F,2F,3-10D	107891	CAA BNT LNG EXT ASM 20W R	Right- 60° & 70° 20 W- LONG	70.00
1F,2G,3-10D	107892	CAA BNT LNG EXT ASM 21&22W R	Right- 60° & 70° 21&22 W- LONG	70.00
>>>>		60°,70°,70° HEMI HANGERS - LONG EXT - LEFT		
1F,2A,3-10A	107893	CAA BNT LNG EXT ASM 15W L	Left- 60° & 70° 15 W- LONG	70.00
1F,2B,3-10B	107894	CAA BNT LNG EXT ASM 16W L	Left- 60° & 70° 16 W- LONG	70.00
1F,2C,3-10B	107895	CAA BNT LNG EXT ASM 17W L	Left- 60° & 70° 17 W- LONG	70.00
1F,2D,3-10C	107896	CAA BNT LNG EXT ASM 18W L	Left- 60° & 70° 18 W- LONG	70.00
1F,2E,3-10C	107897	CAA BNT LNG EXT ASM 19W L	Left- 60° & 70° 19 W- LONG	70.00
1F,2F,3-10D	107898	CAA BNT LNG EXT ASM 20W L	Left- 60° & 70° 20 W- LONG	70.00
1F,2G,3-10D	107899	CAA BNT LNG EXT ASM 21&22W L	Left- 60° & 70° 21&22 W- LONG	70.00
>>>>		70° "V" HANGERS - MED EXT - RIGHT		
1G,2A,3-10A	107900	CAA V MED EXT ASM 15W R	Right- 70° V hangers only 15 W- MED	70.00
1G,2B,3-10B	107901	CAA V MED EXT ASM 16W R	Right- 70° V hangers only 16 W- MED	70.00
1G,2C,3-10B	107902	CAA V MED EXT ASM 17W R	Right- 70° V hangers only 17 W- MED	70.00
1G,2D,3-10C	107903	CAA V MED EXT ASM 18W R	Right- 70° V hangers only 18 W- MED	70.00
>>>>		70° "V" HANGERS - MED EXT - LEFT		
1G,2A,3-10A	107907	CAA V MED EXT ASM 15W L	Left- 70° V hangers only 15 W- MED	70.00
1G,2B,3-10B	107908	CAA V MED EXT ASM 16W L	Left- 70° V hangers only 16 W- MED	70.00
1G,2C,3-10B	107909	CAA V MED EXT ASM 17W L	Left- 70° V hangers only 17 W- MED	70.00
1G,2D,3-10C	107910	CAA V MED EXT ASM 18W L	Left- 70° V hangers only 18 W- MED	70.00
>>>>		70° "V" HANGERS - LONG EXT - RIGHT		
1H,2A,3-10A	107914	CAA V LNG EXT ASM 15W R	Right- 70° V hangers only 15 W- LONG	70.00
1H,2B,3-10B	107915	CAA V LNG EXT ASM 16W R	Right- 70° V hangers only 16 W- LONG	70.00
1H,2C,3-10B	107916	CAA V LNG EXT ASM 17W R	Right- 70° V hangers only 17 W- LONG	70.00
1H,2D,3-10C	107917	CAA V LNG EXT ASM 18W R	Right- 70° V hangers only 18 W- LONG	70.00
>>>>		70° "V" HANGERS - LONG EXT - LEFT		
1H,2A,3-10A	107921	CAA V LNG EXT ASM 15W L	Left- 70° V hangers only 15 W- LONG	70.00
1H,2B,3-10B	107922	CAA V LNG EXT ASM 16W L	Left- 70° V hangers only 16 W- LONG	70.00
1H,2C,3-10B	107923	CAA V LNG EXT ASM 17W L	Left- 70° V hangers only 17 W- LONG	70.00
1H,2D,3-10C	107924	CAA V LNG EXT ASM 18W L	Left- 70° V hangers only 18 W- LONG	70.00
>>>>		80° HANGERS - SHORT EXT - RIGHT		
1A,2A,3-10A	107816	CAA STRT SHT EXT ASM 15W R	Right- 80° 15 W- SHORT	70.00
1A,2B,3-10B	107817	CAA STRT SHT EXT ASM 16W R	Right- 80° 16 W- SHORT	70.00
1A,2C,3-10B	107818	CAA STRT SHT EXT ASM 17W R	Right- 80° 17 W- SHORT	70.00
1A,2D,3-10C	107819	CAA STRT SHT EXT ASM 18W R	Right- 80° 18 W- SHORT	70.00
1A,2E,3-10C	107820	CAA STRT SHT EXT ASM 19W R	Right- 80° 19 W- SHORT	70.00
1A,2F,3-10D	107821	CAA STRT SHT EXT ASM 20W R	Right- 80° 20 W- SHORT	70.00
1A,2G,3-10D	107822	CAA STRT SHT EXT ASM 21&22W R	Right- 80° 21&22 W- SHORT	70.00
>>>>		80° HANGERS - SHORT EXT - LEFT		
1A,2A,3-10A	107823	CAA STRT SHT EXT ASM 15W L	Left- 80° 15 W- SHORT	70.00
1A,2B,3-10B	107824	CAA STRT SHT EXT ASM 16W L	Left- 80° 16 W- SHORT	70.00
1A,2C,3-10B	107825	CAA STRT SHT EXT ASM 17W L	Left- 80° 17 W- SHORT	70.00
1A,2D,3-10C	107826	CAA STRT SHT EXT ASM 18W L	Left- 80° 18 W- SHORT	70.00
1A,2E,3-10C	107827	CAA STRT SHT EXT ASM 19W L	Left- 80° 19 W- SHORT	70.00
1A,2F,3-10D	107828	CAA STRT SHT EXT ASM 20W L	Left- 80° 20 W- SHORT	70.00
1A,2G,3-10D	107829	CAA STRT SHT EXT ASM 21&22W L	Left- 80° 21&22 W- SHORT	70.00
>>>>		80° HANGERS - MED EXT - RIGHT		
1B,2A,3-10A	107830	CAA STRT MED EXT ASM 15W R	Right- 80° 15 W- MED	70.00
1B,2B,3-10B	107831	CAA STRT MED EXT ASM 16W R	Right- 80° 16 W- MED	70.00
1B,2C,3-10B	107832	CAA STRT MED EXT ASM 17W R	Right- 80° 17 W- MED	70.00


Pos.	Item Number	Description	Remarks	USD
1B,2D,3-10C	107833	CAA STRT MED EXT ASM 18W R	Right- 80° 18 W- MED	70.00
1B,2E,3-10C	107834	CAA STRT MED EXT ASM 19W R	Right- 80° 19 W- MED	70.00
1B,2F,3-10D	107835	CAA STRT MED EXT ASM 20W R	Right- 80° 20 W- MED	70.00
1B,2G,3-10D	107836	CAA STRT MED EXT ASM 21&22W R	Right- 80° 21&22 W- MED	70.00
>>>>		80° HANGERS - MED EXT - LEFT		
1B,2A,3-10A	107837	CAA STRT MED EXT ASM 15W L	Left- 80° 15 W- MED	70.00
1B,2B,3-10B	107838	CAA STRT MED EXT ASM 16W L	Left- 80° 16 W- MED	70.00
1B,2C,3-10B	107839	CAA STRT MED EXT ASM 17W L	Left- 80° 17 W- MED	70.00
1B,2D,3-10C	107840	CAA STRT MED EXT ASM 18W L	Left- 80° 18 W- MED	70.00
1B,2E,3-10C	107841	CAA STRT MED EXT ASM 19W L	Left- 80° 19 W- MED	70.00
1B,2F,3-10D	107842	CAA STRT MED EXT ASM 20W L	Left- 80° 20 W- MED	70.00
1B,2G,3-10D	107843	CAA STRT MED EXT ASM 21&22W L	Left- 80° 21&22 W- MED	70.00
>>>>		80° HANGERS - LONG EXT - RIGHT		
1C,2A,3-10A	107844	CAA STRT LNG EXT ASM 15W R	Right- 80° 15 W- LONG	70.00
1C,2B,3-10B	107845	CAA STRT LNG EXT ASM 16W R	Right- 80° 16 W- LONG	70.00
1C,2C,3-10B	107846	CAA STRT LNG EXT ASM 17W R	Right- 80° 17 W- LONG	70.00
1C,2D,3-10C	107847	CAA STRT LNG EXT ASM 18W R	Right- 80° 18 W- LONG	70.00
1C,2E,3-10C	107848	CAA STRT LNG EXT ASM 19W R	Right- 80° 19 W- LONG	70.00
1C,2F,3-10D	107849	CAA STRT LNG EXT ASM 20W R	Right- 80° 20 W- LONG	70.00
1C,2G,3-10D	107850	CAA STRT LNG EXT ASM 21&22W R	Right- 80° 21&22 W- LONG	70.00
>>>>		80° HANGERS - LONG EXT - LEFT		
1C,2A,3-10A	107851	CAA STRT LNG EXT ASM 15W L	Left- 80° 15 W- LONG	70.00
1C,2B,3-10B	107852	CAA STRT LNG EXT ASM 16W L	Left- 80° 16 W- LONG	70.00
1C,2C,3-10B	107853	CAA STRT LNG EXT ASM 17W L	Left- 80° 17 W- LONG	70.00
1C,2D,3-10C	107854	CAA STRT LNG EXT ASM 18W L	Left- 80° 18 W- LONG	70.00
1C,2E,3-10C	107855	CAA STRT LNG EXT ASM 19W L	Left- 80° 19 W- LONG	70.00
1C,2F,3-10D	107856	CAA STRT LNG EXT ASM 20W L	Left- 80° 20 W- LONG	70.00
1C,2G,3-10D	107857	CAA STRT LNG EXT ASM 21&22W L	Left- 80° 21&22 W- LONG	70.00

NOTE: FOOTPLATE ASSEMBLIES ARE SET TO FOOTREST WIDTH. PLEASE SEE "FOOTPLATE FORMULA" TABLE FOR SEAT WIDTH CONVERSION.


Adult Angle Adjustable "AAA" Footplates w/ Ext & HL for Swing I/O Hangers								
(Order 1 each)								
Ext. Tube	60°, 70° & 70° Hemi Style Hangers (Bent Extension)			80° Style hanger (Straight Extension)			70° V Style Hangers	
Width	Short	Med	Long	Short	Med	Long	Med	Long
10"-11" Rt	NOT AVAILABLE IN THIS WIDTH			NOT AVAILABLE IN THIS WIDTH			111546	111550
10"-11" Lt	NOT AVAILABLE IN THIS WIDTH			NOT AVAILABLE IN THIS WIDTH			111547	111551
12"-13" Rt	111540	111542	111544	111534	111536	111538	111548	111552
12"-13" Lt	111541	111543	111545	111535	111537	111539	111549	111553
14"-15" Rt	107973	107987	108001	107928	107942	107956	108015	108029
14"-15" Lt	107980	107994	108008	107935	107949	107963	108022	108036
16"-17" Rt	107975	107989	108003	107930	107944	107958	108017	108031
16"-17" Lt	107982	107996	108010	107937	107951	107965	108024	108038
18"-22" Rt	107977	107991	108005	107932	107946	107960	108019	108033
18"-22" Lt	107984	107998	108012	107939	107953	107968	108026	108040

INSET FORMULA
 0" Inset (Seat Width) = Footrest Width
 1" Inset (Seat Width) -2 = Footrest Width
 "V" Hanger (Seat Width) -4 = Footrest Width

MEASURING GUIDE

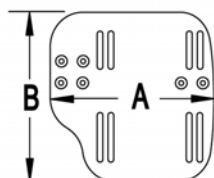


CHAIR WIDTH	"A"	"B"
10" - 11"	3 1/2"	7 1/4"
12" - 13"	4 1/2"	7 3/4"
14" - 15"	5 1/2"	7 3/4"
16" - 17"	6 1/2"	7 3/4"
18" - 20"	7 1/2"	7 3/4"

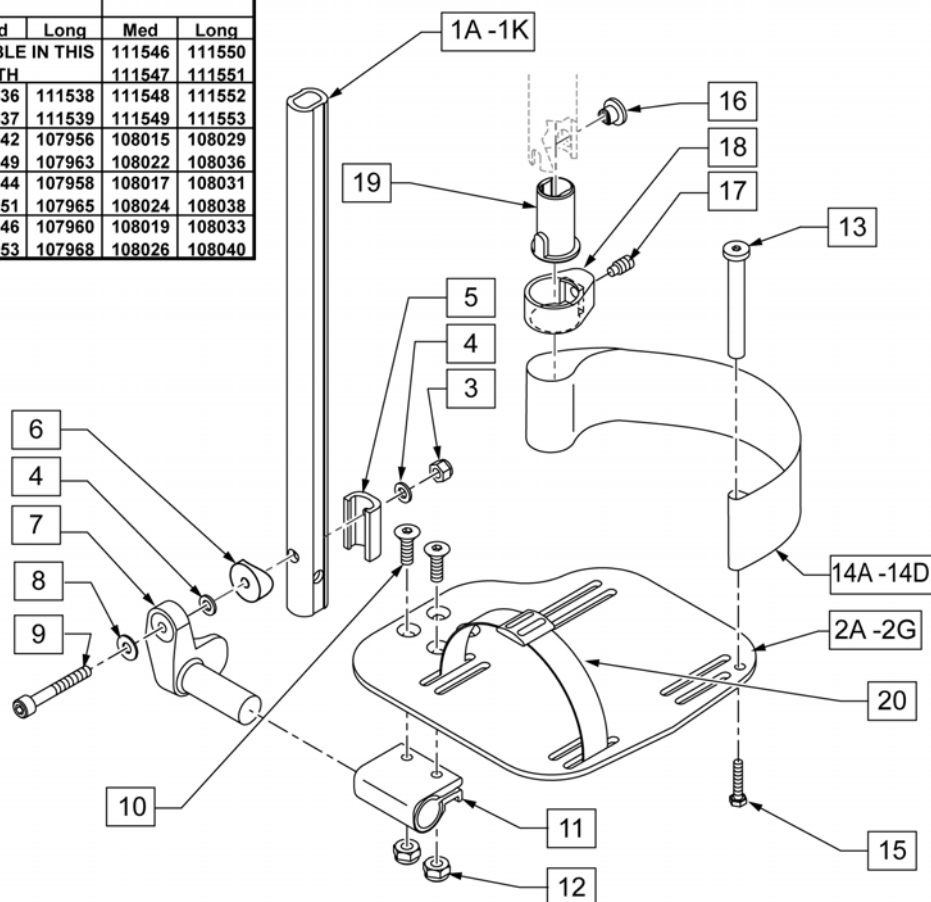

Pos.	Item Number	Description	Remarks	USD
1A,3-5,8,9	120670	SHT STRT EXT AAA HDWR	80 deg Hangers Only	30.00
1B,3-5,8,9	120671	MED STRT EXT AAA HDWR	80 deg Hangers Only	30.00
1C,3-5,8,9	120672	LNG STRT EXT AAA HDWR	80 deg Hangers Only	30.00
1D,3-5,8,9	120673	SHT BNT EXT AAA HDWR	60,70,70 Hemi Hangers	30.00
1E,3-5,8,9	120674	MED BNT EXT AAA HDWR	60,70,70 Hemi Hangers	30.00
1F,3-5,8,9	120675	LNG BNT EXT AAA HDWR	60,70,70 Hemi Hangers	30.00
1G,3-5,8,9	120676	SHT V EXT R AAA HDWR	Left - 70 deg V Hangers Only	30.00
1H,3-5,8,9	120677	SHT V EXT L AAA HDWR	Right - 70 deg V Hangers Only	30.00
1I,3-5,8,9	120678	LNG V EXT R AAA HDWR	Left - 70 deg V Hangers Only	30.00
1J,3-5,8,9	120679	LNG V EXT L AAA HDWR	Right - 70 deg V Hangers Only	30.00
2A	500656	ADULT ANG ADJ F/PLT 10/11" R	RIGHT 10-11" Footrest Width	58.14
2A	500657	ADULT ANG ADJ F/PLT 10/11" L	LEFT 10-11" Footrest Width	58.14
2B	500658	ADULT ANG ADJ F/PLT 12/13" R	RIGHT 12-13" Footrest Width	60.68



Pos.	Item Number	Description	Remarks	USD
2B	500659	ADULT ANG ADJ F/PLT 12/13" L	LEFT 12-13" Footrest Width	60.68
2C	500660	ADLT ANGL ADJ F/PLT 14/15	RIGHT 14-15" Footrest Width	60.68
2C	500661	ADLT ANGL ADJ F/PLT 14/15	LEFT 14-15" Footrest Width	60.68
2D	500662	ADLT ANGL ADJ F/PLT 16/17	RIGHT 16-17" Footrest Width	60.68
2D	500663	ADLT ANGL ADJ F/PLT 16/17	LEFT 16-17" Footrest Width	60.68
2E	500664	ADLT ANGL ADJ F/PLT 18-20	RIGHT 18-22" Footrest Width	60.68
2E	500665	ADLT ANGL ADJ F/PLT 18-20	LEFT 18-22" Footrest Width	60.68
3	104381	NUT HEX NYLOCK M6X1.0		5.00
4	106944	WASHER FLAT THIN AN 1/4 ROHS		1.00
5	107199	HANGER EXTENSION SADDLE		5.00
6	104377	3/4" SADDLE THIN		5.00
7	541002	FTRST MOUNTING BRKT-3/8 RAD R	RIGHT	18.21
7	541003	FTRST MOUNTING BRKT-3/8 RAD L	LEFT	18.21
8	106945	WASHER FLAT SAE 1/4 ROHS		1.00
9	104399	SHCS M6X1X45 12 MM PART THD		5.00
10	106978	FHCS 1/4-28 X 7/8 ROHS		5.00
11	550350	3/4" CLMP ADJ FT PLT		11.04
12	106935	NUT NYLOCK THIN 1/4-28 ROHS		5.00
13	104328	POST HEEL LOOP		5.00
14A	104380-001	ADJUSTABLE FOOT REST STRAP	12 - 14" WIDE	7.50
14B	104380-002	ADJUSTABLE FOOT REST STRAP	15 - 16" WIDE	7.50
14C	104380-003	ADJUSTABLE FOOT REST STRAP	17 - 18" WIDE	7.50
14D	104380-004	ADJUSTABLE FOOT REST STRAP	19 - 20" WIDE	7.50
15	102268-001	HHCS M6 X 1 X 25		10.00
3-6,8,9	120659	EXT MNT HARDWARE 1 SIDE		20.00
16-19	107485	FOOTREST CLAMP ASSM		30.00
20	740260	TOE-ANKLE STRAP ADJ ADULT FOOTPLATE		20.40
>>>>		80° HANGERS - SHORT EXT - RIGHT		
1A,2B,3-14A,15	111534	AAA STRT SHT EXT ASM 12-13" W R	Right- 80D- 12-13" W- SHORT	100.00
1A,2C,3-14B,15	107928	AAA STRT SHT EXT ASM 14-15" W R	Right- 80D- 14-15" W- SHORT	100.00
1A,2D,3-14C,15	107930	AAA STRT SHT EXT ASM 16-17" W R	Right- 80D- 16-17" W- SHORT	100.00
1A,2E,3-14D,15	107932	AAA STRT SHT EXT ASM 18-22" W R	Right- 80D- 18-22" W- SHORT	100.00
>>>>		80° HANGERS - SHORT EXT - LEFT		
1A,2B,3-14A,15	111535	AAA STRT SHT EXT ASM 12-13" W L	Left- 80D- 12-13" W- SHORT	100.00
1A,2C,3-14B,15	107935	AAA STRT SHT EXT ASM 14" W L	Left- 80D- 14-15" W- SHORT	100.00
1A,2D,3-14C,15	107937	AAA STRT SHT EXT ASM 16" W L	Left- 80D- 16-17" W- SHORT	100.00
1A,2E,3-14D,15	107939	AAA STRT SHT EXT ASM 18" W L	Left- 80D- 18-22" W- SHORT	100.00
>>>>		80° HANGERS - MED EXT - RIGHT		
1B,2B,3-14A,15	111536	AAA STRT MED EXT ASM 12-13" W R	Right- 80D- 12-13" W- MED	100.00
1B,2C,3-14B,15	107942	AAA STRT MED EXT ASM 14-15" W R	Right- 80D- 14-15" W- MED	100.00
1B,2D,3-14C,15	107944	AAA STRT MED EXT ASM 16-17" W R	Right- 80D- 16-17" W- MED	100.00
1B,2E,3-14D,15	107946	AAA STRT MED EXT ASM 18-22" W R	Right- 80D- 18-22" W- MED	100.00
>>>>		80° HANGERS - MED EXT - LEFT		
1B,2B,3-14A,15	111537	AAA STRT MED EXT ASM 12-13" W L	Left- 80D- 12-13" W- MED	100.00
1B,2C,3-14B,15	107949	AAA STRT MED EXT ASM 14" W L	Left- 80D- 14-15" W- MED	100.00
1B,2D,3-14C,15	107951	AAA STRT MED EXT ASM 16" W L	Left- 80D- 16-17" W- MED	100.00
1B,2E,3-14D,15	107953	AAA STRT MED EXT ASM 18" W L	Left- 80D- 18-22" W- MED	100.00
>>>>		80° HANGERS - LONG EXT - RIGHT		
1C,2B,3-14A,15	111538	AAA STRT LNG EXT ASM 12-13" W R	Right- 80D- 12-13" W- LONG	100.00
1C,2C,3-14B,15	107956	AAA STRT LNG EXT ASM 14 -15" W R	Right- 80D- 14-15" W- LONG	100.00
1C,2D,3-14C,15	107958	AAA STRT LNG EXT ASM 16 -17" W R	Right- 80D- 16-17" W- LONG	100.00
1C,2E,3-14D,15	107960	AAA STRT LNG EXT ASM 18 -22" W R	Right- 80D- 18-22" W- LONG	100.00
>>>>		80° HANGERS - LONG EXT - LEFT		
1C,2B,3-14A,15	111539	AAA STRT LNG EXT ASM 12-13" W L	Left- 80D- 12-13" W- LONG	100.00
1C,2C,3-14B,15	107963	AAA STRT LNG EXT ASM 14W L	Left- 80D- 14-15" W- LONG	100.00
1C,2D,3-14C,15	107965	AAA STRT LNG EXT ASM 16W L	Left- 80D- 16-17" W- LONG	100.00

Pos.	Item Number	Description	Remarks	USD
1C,2E,3-14D,15	107968	AAA STRT LNG EXT ASM 18W L	Left- 80D- 18-22" W- LONG	100.00
>>>>		70° HANGERS - SHORT EXT - RIGHT		
1D,2B,3-14A,15	111540	AAA BNT SHT EXT ASM 12-13" W R	Right- 60D,70D,70D HEMI - 12-13" W- SHORT	100.00
1D,2C,3-14B,15	107973	AAA BNT SHT EXT ASM 14 -15" W R	Right- 60D,70D,70D HEMI - 14-15" W- SHORT	100.00
1D,2D,3-14C,15	107975	AAA BNT SHT EXT ASM 16 -17" W R	Right- 60D,70D,70D HEMI - 16-17" W- SHORT	100.00
1D,2E,3-14D,15	107977	AAA BNT SHT EXT ASM 18 -22" W R	Right- 60D,70D,70D HEMI - 18-22" W- SHORT	100.00
>>>>		70° HANGERS - SHORT EXT - LEFT		
1D,2B,3-14A,15	111541	AAA BNT SHT EXT ASM 12-13" W L	Left- 60D,70D,70D HEMI - 12-13" W- SHORT	100.00
1D,2C,3-14B,15	107980	AAA BNT SHT EXT ASM 14 -15" W L	Left- 60D,70D,70D HEMI - 14-15" W- SHORT	100.00
1D,2D,3-14C,15	107982	AAA BNT SHT EXT ASM 16 -17" W L	Left- 60D,70D,70D HEMI - 16-17" W- SHORT	100.00
1D,2E,3-14D,15	107984	AAA BNT SHT EXT ASM 18 -22" W L	Left- 60D,70D,70D HEMI - 18-22" W- SHORT	100.00
>>>>		70° HANGERS - MED EXT - RIGHT		
1E,2B,3-14A,15	111542	AAA BNT MED EXT ASM 12-13" W R	Right- 60D,70D,70D HEMI - 12-13" W- MED	100.00
1E,2C,3-14B,15	107987	AAA BNT MED EXT ASM 14 -15" W R	Right- 60D,70D,70D HEMI - 14-15" W- MED	100.00
1E,2D,3-14C,15	107989	AAA BNT MED EXT ASM 16 -17" W R	Right- 60D,70D,70D HEMI - 16-17" W- MED	100.00
1E,2E,3-14D,15	107991	AAA BNT MED EXT ASM 18 -22" W R	Right- 60D,70D,70D HEMI - 18-22" W- MED	100.00
>>>>		70° HANGERS - MED EXT - LEFT		
1E,2B,3-14A,15	111543	AAA BNT MED EXT ASM 12-13" W L	Left- 60D,70D,70D HEMI - 12-13" W- MED	100.00
1E,2C,3-14B,15	107994	AAA BNT MED EXT ASM 14 -15" W L	Left- 60D,70D,70D HEMI - 14-15" W- MED	100.00
1E,2D,3-14C,15	107996	AAA BNT MED EXT ASM 16 -17" W L	Left- 60D,70D,70D HEMI - 16-17" W- MED	100.00
1E,2E,3-14D,15	107998	AAA BNT MED EXT ASM 18 -22" W L	Left- 60D,70D,70D HEMI - 18-22" W- MED	100.00
>>>>		70° HANGERS - LONG EXT - RIGHT		
1F,2B,3-14A,15	111544	AAA BNT LONG EXT ASM 12-13" W R	Right- 60D,70D,70D HEMI - 12-13" W- LONG	100.00
1F,2C,3-14B,15	108001	AAA BNT LONG EXT ASM 14 -15" W R	Right- 60D,70D,70D HEMI - 14-15" W- LONG	100.00
1F,2D,3-14C,15	108003	AAA BNT LONG EXT ASM 16 -17" W R	Right- 60D,70D,70D HEMI - 16-17" W- LONG	100.00
1F,2E,3-14D,15	108005	AAA BNT LONG EXT ASM 18 -22" W R	Right- 60D,70D,70D HEMI - 18-22" W- LONG	100.00
>>>>		70° HANGERS - LONG EXT - LEFT		
1F,2B,3-14A,15	111545	AAA BNT LONG EXT ASM 12-13" W L	Left- 60D,70D,70D HEMI - 12-13" W- LONG	100.00
1F,2C,3-14B,15	108008	AAA BNT LONG EXT ASM 14 -15" W L	Left- 60D,70D,70D HEMI - 14-15" W- LONG	100.00
1F,2D,3-14C,15	108010	AAA BNT LONG EXT ASM 16 -17" W L	Left- 60D,70D,70D HEMI - 16-17" W- LONG	100.00
1F,2E,3-14D,15	108012	AAA BNT LONG EXT ASM 18 -22" W L	Left- 60D,70D,70D HEMI - 18-22" W- LONG	100.00
>>>>		70° "V" HANGERS - MED EXT - RIGHT		
1G,2A,3-15	111546	AAA V MED EXT ASM 10-11" W R	Right- 70D V hangers only 10-11" W- MED	100.00
1G,2B,3-15	111548	AAA V MED EXT ASM 12-13" W R	Right- 70D V hangers only 12-13" W- MED	100.00
1G,2C,3-15	108015	AAA V MED EXT ASM 14 -15" W R	Right- 70D V hangers only 14-15" W- MED	100.00
1G,2D,3-15	108017	AAA V MED EXT ASM 16 -17" W R	Right- 70D V hangers only 16-18" W- MED	100.00
1G,2E,3-15	108019	AAA V MED EXT ASM 18 -18" W R	Right- 70D V hangers only 18-22" W- MED	100.00
>>>>		70° "V" HANGERS - MED EXT - LEFT		
1G,2A,3-15	111547	AAA V MED EXT ASM 10-11" W L	Left- 70D V hangers only 10-11" W- MED	100.00
1G,2B,3-15	111549	AAA V MED EXT ASM 12-13" W L	Left- 70D V hangers only 12-13" W- MED	100.00
1G,2C,3-15	108022	AAA V MED EXT ASM 14 -15" W L	Left- 70D V hangers only 14-15" W- MED	100.00
1G,2D,3-15	108024	AAA V MED EXT ASM 16 -17" W L	Left- 70D V hangers only 16-18" W- MED	100.00
1G,2E,3-15	108026	AAA V MED EXT ASM 18 -22" W L	Left- 70D V hangers only 18-22" W- MED	100.00
>>>>		70° "V" HANGERS - LONG EXT - RIGHT		
1H,2A,3-15	111550	AAA V LONG EXT ASM 10-11" W R	Right- 70D V hangers only 10-11" W- LONG	100.00
1H,2B,3-15	111552	AAA V LONG EXT ASM 12-13" W R	Right- 70D V hangers only 12-13" W- LONG	100.00
1H,2C,3-15	108029	AAA V LONG EXT ASM 14 -15" W R	Right- 70D V hangers only 14-15" W- LONG	100.00
1H,2D,3-15	108031	AAA V LONG EXT ASM 16 -17" W R	Right- 70D V hangers only 16-18" W- LONG	100.00
1H,2E,3-15	108033	AAA V LONG EXT ASM 18 -22" W R	Right- 70D V hangers only 18-22" W- LONG	100.00
>>>>		70° "V" HANGERS - LONG EXT - LEFT		
1H,2A,3-15	111551	AAA V LONG EXT ASM 10-11" W L	Left- 70D V hangers only 10-11" W- LONG	100.00
1H,2B,3-15	111553	AAA V LONG EXT ASM 12-13" W L	Left- 70D V hangers only 12-13" W- LONG	100.00
1H,2C,3-15	108036	AAA V LONG EXT ASM 14 -15" W L	Left- 70D V hangers only 14-15" W- LONG	100.00
1H,2D,3-15	108038	AAA V LONG EXT ASM 16 -17" W L	Left- 70D V hangers only 16-18" W- LONG	100.00
1H,2E,3-15	108040	AAA V LONG EXT ASM 18 -22" W L	Left- 70D V hangers only 18-22" W- LONG	100.00

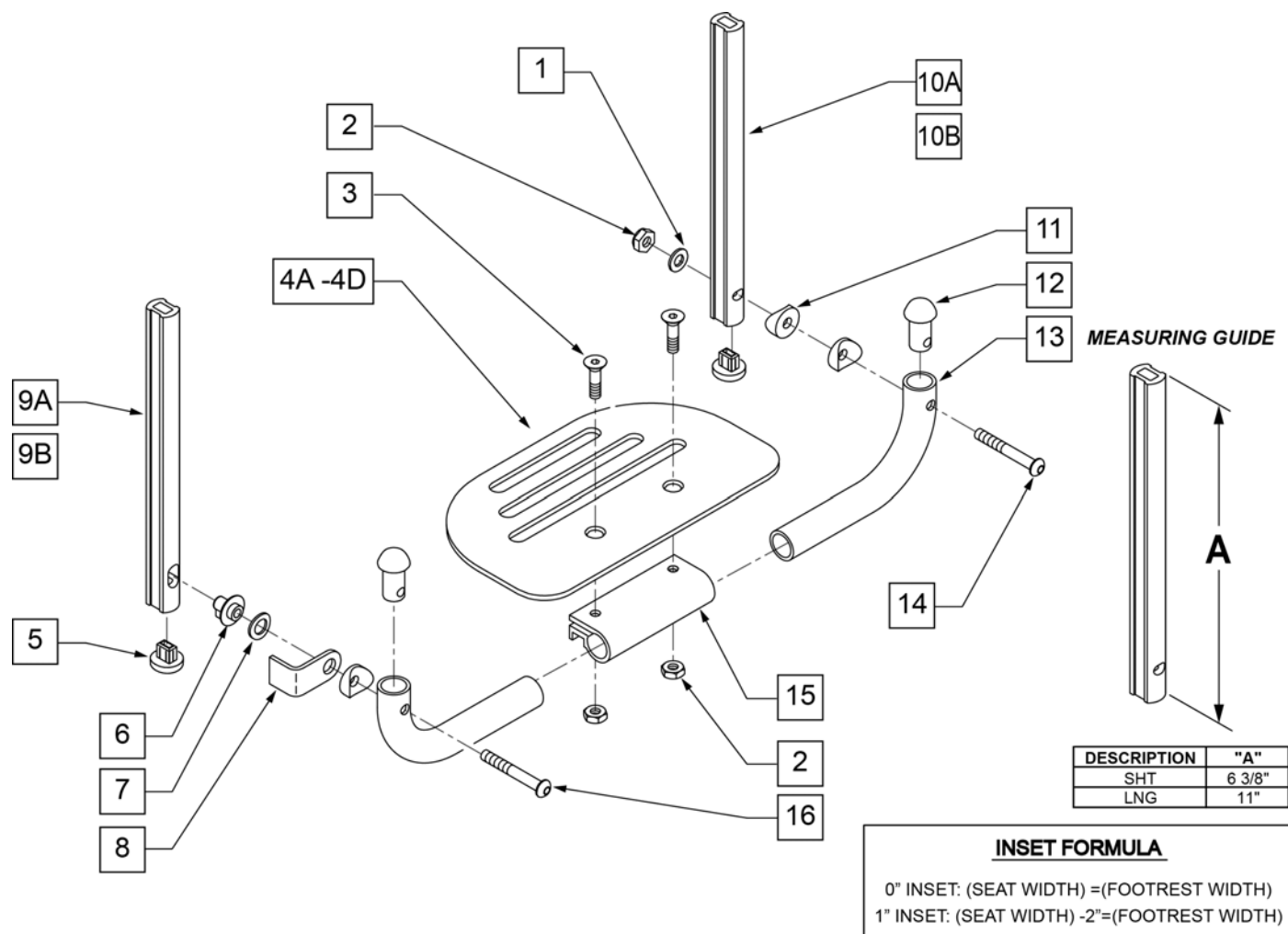


Pos.	Item Number	Description	Remarks	USD
1	106944	WASHER FLAT THIN AN 1/4 ROHS		1.00
2	106941	NUT AUTOMATION 1/4-28 ROHS		5.00
3	106978	FHCS 1/4-28 X 7/8 ROHS		5.00
4A	500651	PLATFORM F/PLT 8 x 5 1/2"	12 -14" Footrest Width	56.81
4B	500652	PLATFORM F/PLT 10" x 5 1/2"	15 -17" Footrest Width	56.81
4C	500653	PLATFORM F/PLT 12" x 5 1/2"	18 -20" Footrest Width	56.81
4D	500654	PLATFORM F/PLT 14" x 5 1/2"	21 -22" Footrest Width	62.73
5	113221	PLUG DOUBLE D EXT TUBE		4.00
6	540410	F/UP FOOTREST KEY STUD D2		6.40
7	107385	WASHER CURVED 3/8 ROHS		5.00
8	505078	FOOTREST RETAINING BRACKET		4.19
9A	106875	EXT TBE F/UP KEY SHT	Short -Latch Side	7.50
9B	106876	EXT TBE F/UP KEY LNG	Long -Latch Side	7.50
10A	106873	EXT TBE F/UP SLT SHT	Short -Pivot Side	7.50
10B	106874	EXT TBE F/UP SLT LNG	Long -Pivot Side	7.50
11	561003	3/4" SADDLE THIN		2.00
12	561034	SHORT DOME PLUG		4.00
14	106964	BHCS 1/4-28 X 2 ROHS		5.00
15	550351	PLATFORM FOOTPLATE CLAMP		25.38


Pos.	Item Number	Description	Remarks	USD
16	107027	BHCS 1/4-28 X 1-1/4 SHLDR ROHS		5.00
17A	740130	LEG STRAP 10"-14" BLK	12-14" Footrest Width	30.60
17B	740131	LEG STRAP 15"-17" BLK	15-17" Footrest Width	30.60
17C	740132	LEG STRAP 18"-20" BLK	18-20" Footrest Width	30.60
1-4A,6-8,11-16	900306	PLTFM F/P PLT BLK 12" D2 W/ INSTR	12" Footrest Width	95.00
1-4A,6-8,11-16	900307	PLTFM F/P PLT BLK 13" D2 W/ INSTR	13" Footrest Width	95.00
1-4A,6-8,11-16	900308	PLTFM F/P PLT BLK 14" D2 W /INSTR	14" Footrest Width	95.00
1-4B,6-8,11-16	900309	PLTFM F/P PLT BLK 15" D2 W /INSTR	15" Footrest Width	95.00
1-4B,6-8,11-16	900310	PLTFM F/P PLT BLK 16" D2 W /INSTR	16" Footrest Width	95.00
1-4B,6-8,11-16	900311	PLTFM F/P PLT BLK 17" D2 W /INSTR	17" Footrest Width	95.00
1-4C,6-8,11-16	900312	PLTFM F/P PLT BLK 18" D2 W /INSTR	18" Footrest Width	95.00
1-4C,6-8,11-16	900313	PLTFM F/P PLT BLK 19" D2 W /INSTR	19" Footrest Width	95.00
1-4C,6-8,11-16	900314	PLTFM F/P PLT BLK 20" D2 W/ INSTR	20" Footrest Width	95.00
1-4D,6-8,11-16	900315	PLTFM F/P PLT BLK 21" D2 W/ INSTR	21" Footrest Width	95.00
1-4D,6-8,11-16	900316	PLTFM F/P PLT BLK 22" D2 W/ INSTR	22" Footrest Width	95.00
		SHORT EXT ASSEMBLIES		
1-3,4A,5-8,9A,10B,11-16	108043	PLTFRM FLIP FTRST SHT 12W	12" wide (Short)	100.00
1-3,4A,5-8,9A,10B,11-16	108044	PLTFRM FLIP FTRST SHT 13W	13" wide (Short)	100.00
1-3,4A,5-8,9A,10B,11-16	108045	PLTFRM FLIP FTRST SHT 14W	14" wide (Short)	100.00
1-3,4B,5-8,9A,10B,11-16	108046	PLTFRM FLIP FTRST SHT 15W	15" wide (Short)	100.00
1-3,4B,5-8,9A,10B,11-16	108047	PLTFRM FLIP FTRST SHT 16W	16" wide (Short)	100.00
1-3,4B,5-8,9A,10B,11-16	108048	PLTFRM FLIP FTRST SHT 17W	17" wide (Short)	100.00
1-3,4C,5-8,9A,10B,11-16	108049	PLTFRM FLIP FTRST SHT 18W	18" wide (Short)	100.00
1-3,4C,5-8,9A,10B,11-16	108051	PLTFRM FLIP FTRST SHT 19W	19" wide (Short)	100.00
1-3,4C,5-8,9A,10B,11-16	108052	PLTFRM FLIP FTRST SHT 20W	20" wide (Short)	100.00
1-3,4D,5-8,9A,10B,11-16	111554	PLTFRM FLIP FTRST SHT 21W	21" wide (Short)	100.00
1-3,4D,5-8,9A,10B,11-16	111555	PLTFRM FLIP FTRST SHT 22W	22" wide (Short)	100.00
		LONG EXT ASSEMBLIES		
1-3,4A,5-8,9B,10A,11-16	108053	PLTFRM FLIP FTRST LONG 12W	12" wide (Long)	100.00
1-3,4A,5-8,9B,10A,11-16	108054	PLTFRM FLIP FTRST LONG 13W	13" wide (Long)	100.00
1-3,4A,5-8,9B,10A,11-16	108055	PLTFRM FLIP FTRST LONG 14W	14" wide (Long)	100.00
1-3,4B,5-8,9B,10A,11-16	108056	PLTFRM FLIP FTRST LONG 15W	15" wide (Long)	100.00
1-3,4B,5-8,9B,10A,11-16	108057	PLTFRM FLIP FTRST LONG 16W	16" wide (Long)	100.00
1-3,4B,5-8,9B,10A,11-16	108058	PLTFRM FLIP FTRST LONG 17W	17" wide (Long)	100.00
1-3,4C,5-8,9B,10A,11-16	108059	PLTFRM FLIP FTRST LONG 18W	18" wide (Long)	100.00
1-3,4C,5-8,9B,10A,11-16	108060	PLTFRM FLIP FTRST LONG 19W	19" wide (Long)	100.00
1-3,4C,5-8,9B,10A,11-16	108061	PLTFRM FLIP FTRST LONG 20W	20" wide (Long)	100.00
1-3,4D,5-8,9B,10A,11-16	111556	PLTFRM FLIP FTRST LONG 21W	21" wide (Long)	100.00
1-3,4D,5-8,9B,10A,11-16	111557	PLTFRM FLIP FTRST LONG 22W	22" wide (Long)	100.00

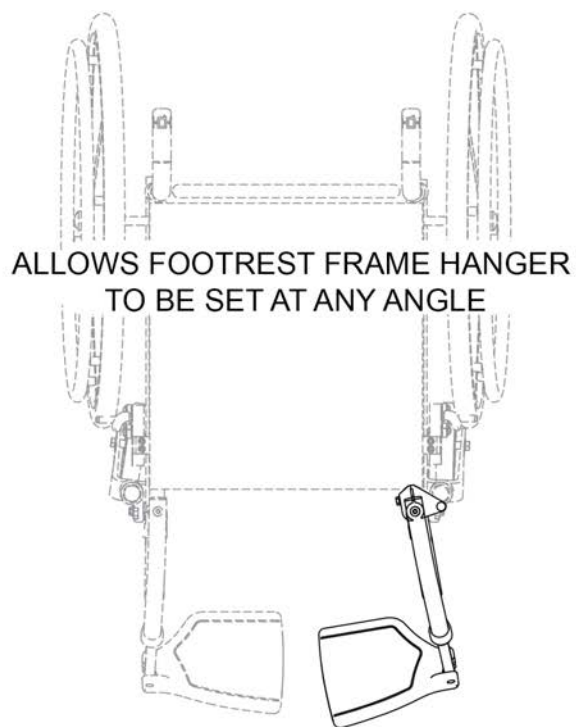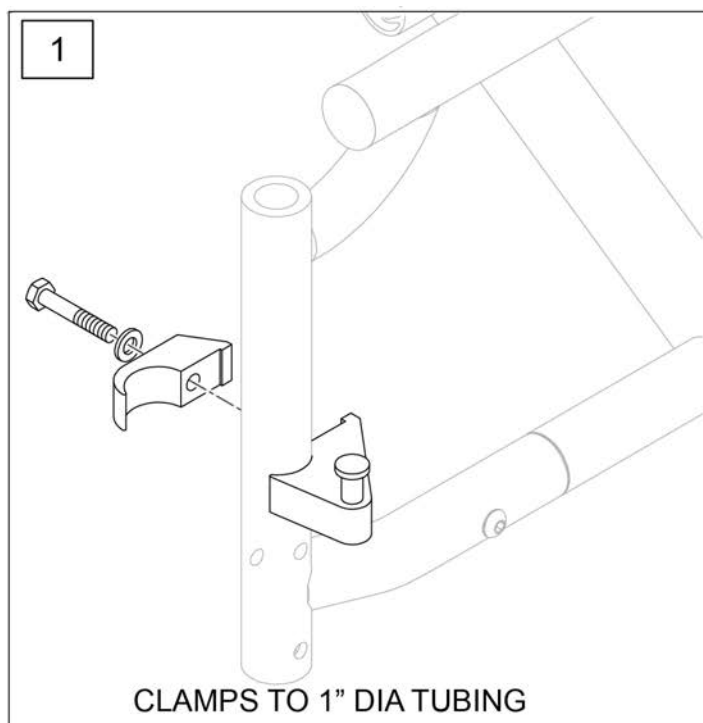


Pos.	Item Number	Description	Remarks
1	112807	LATCH STUD SUPPORT R MULTIPOS	Single
1	112808	LATCH STUD SUPPORT L MULTIPOS	Single
1	112809	LATCH STUD SUPPORT PR MULTIPOS	Pair

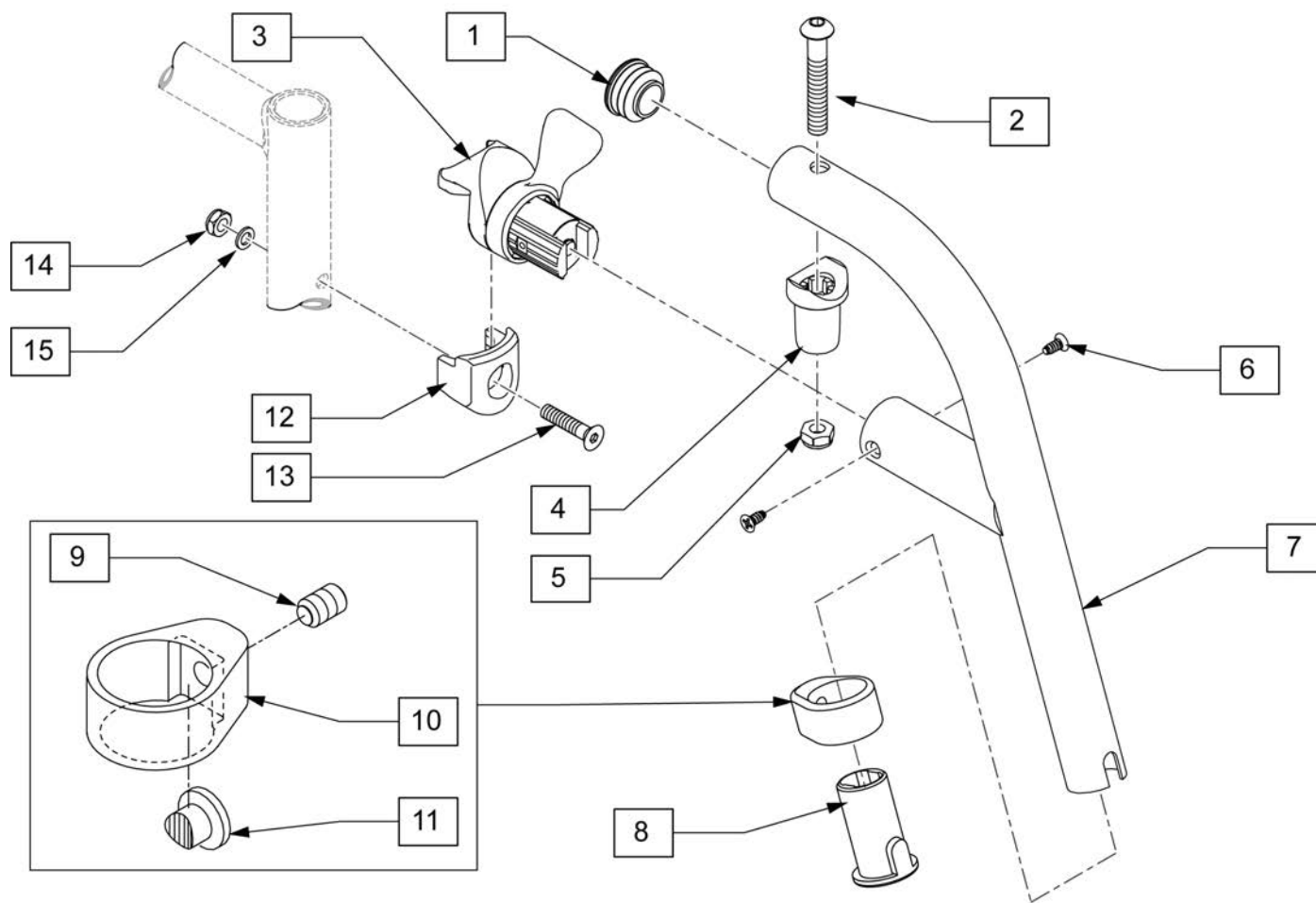


Pos.	Item Number	Description	Remarks	USD
1	561062	1" INSERT PLUG .065 RIBS		2.00
8	104160	BUSHING FOOTREST		5.00
13	103500-010	FHCSM6X1X35FULLNP		2.00
14	104381	NUT HEX NYLOCK M6X1.0		5.00
15	104414	WAVE WASHER 6 MM		5.00
1,2,4,5	119518	PIVOT SADDLE KIT		25.00
3,6	119509	HANGER LATCH ASSM W/HDWR L	Left	60.00
3,6	119510	HANGER LATCH ASSM W/HDWR R	Right	60.00
1-11	122699S	60 D SWING I/O HNGR PNTD L		105.00
1-11	122700S	60 D SWING I/O HNGR PNTD R		105.00
8-11	107485	FOOTREST CLAMP ASSM		30.00
12-15	119514	LATCH RECIEVER KIT		45.00

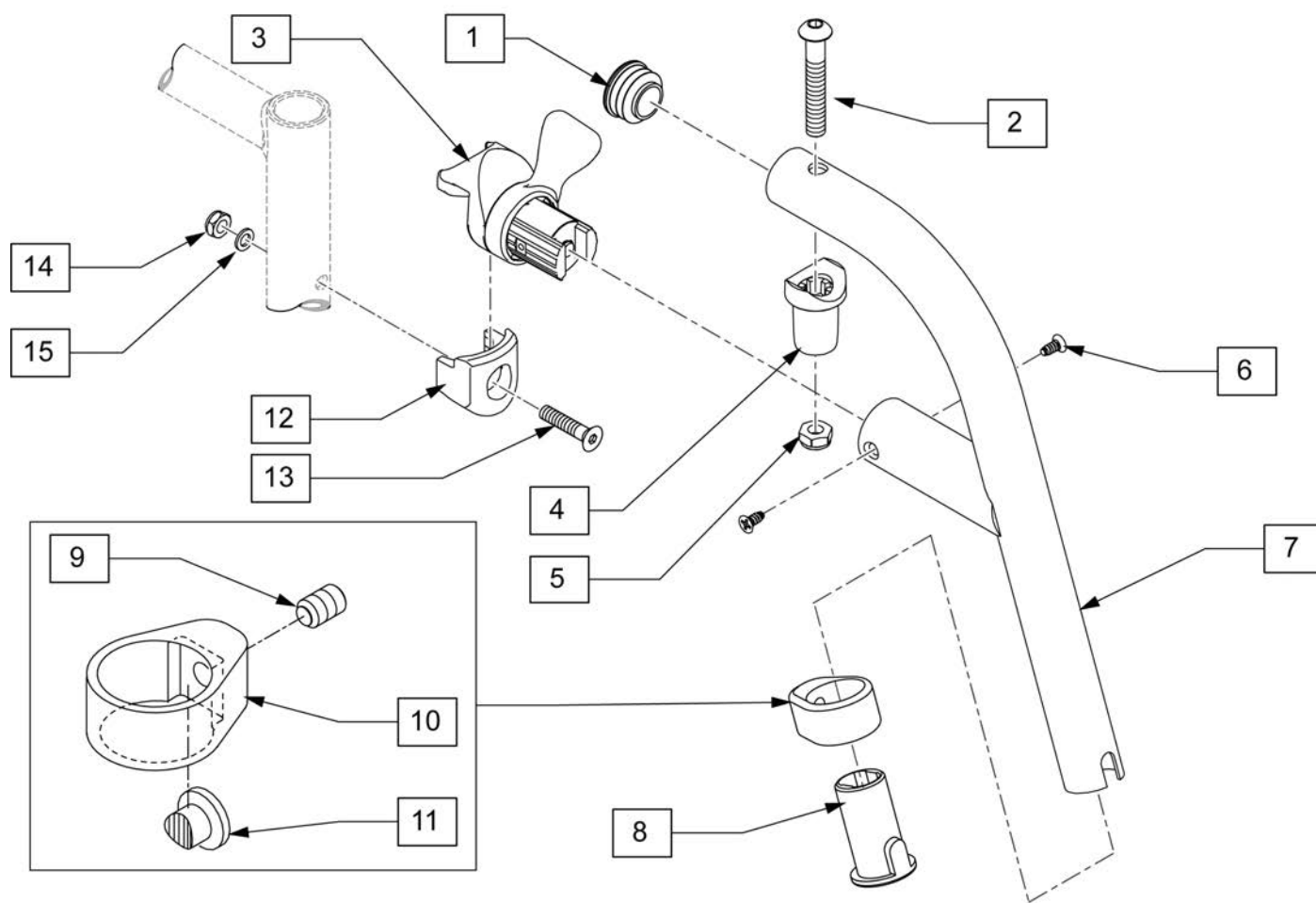

Pos.	Item Number	Description	Remarks	USD
1	561062	1" INSERT PLUG .065 RIBS		2.00
8	104160	BUSHING FOOTREST		5.00
13	103500-010	FHCSM6X1X35FULLNP		2.00
14	104381	NUT HEX NYLOCK M6X1.0		5.00
15	104414	WAVE WASHER 6 MM		5.00
1,2,4,5	119518	PIVOT SADDLE KIT		25.00
3,6	119509	HANGER LATCH ASSM W/HDWR L	Left	60.00
3,6	119510	HANGER LATCH ASSM W/HDWR R	Right	60.00
1-11	119458S	70 D SWING I/O HNGR PNTD L	Left	105.00
1-11	119459S	70 D SWING I/O HNGR PNTD R	Right	105.00
8-11	107485	FOOTREST CLAMP ASSM		30.00
12-15	119514	LATCH RECIEVER KIT		45.00

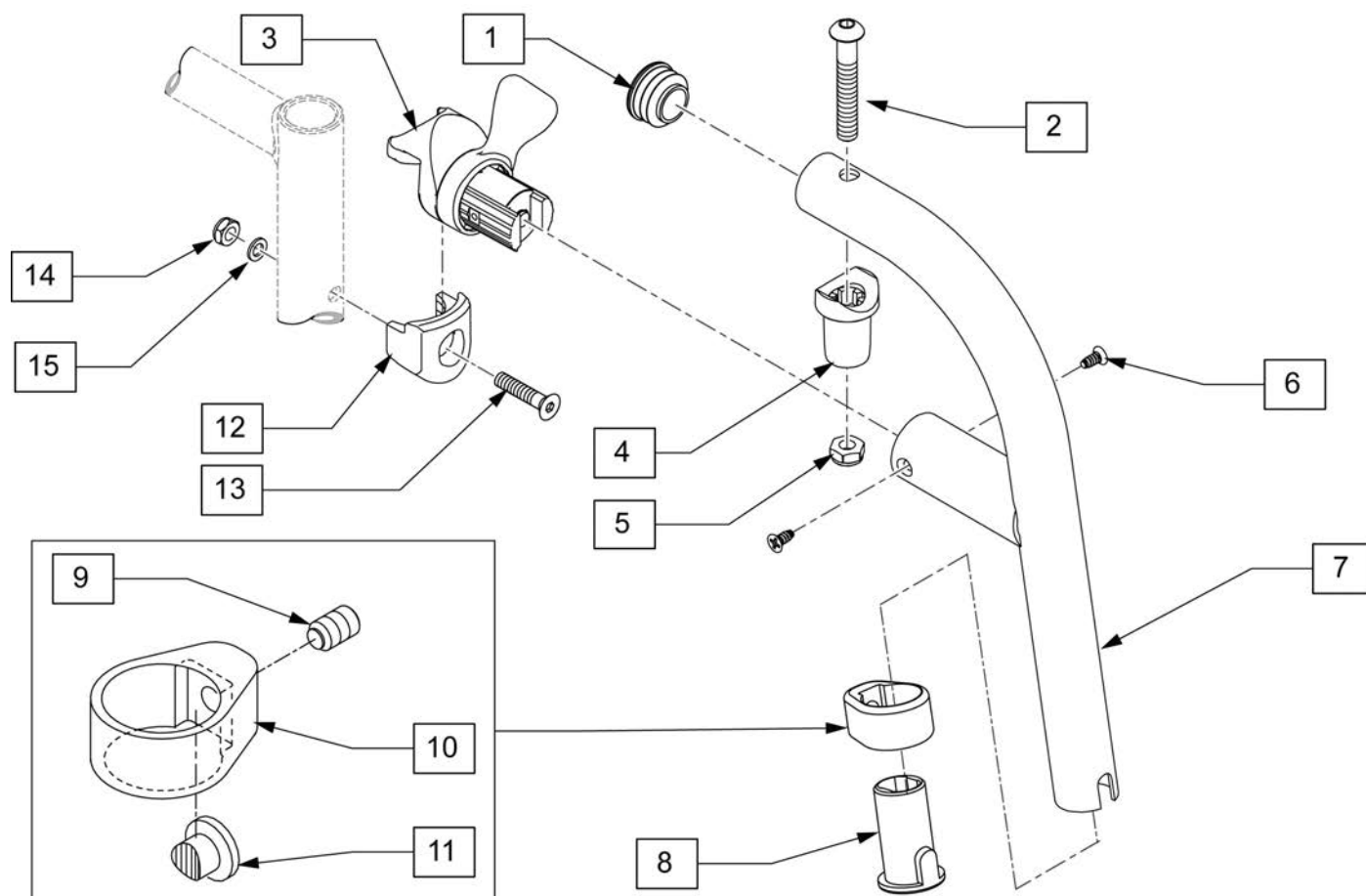


Pos.	Item Number	Description	Remarks	USD
1	561062	1" INSERT PLUG .065 RIBS		2.00
8	104160	BUSHING FOOTREST		5.00
13	103500-010	FHCSM6X1X35FULLNP		2.00
14	104381	NUT HEX NYLOCK M6X1.0		5.00
15	104414	WAVE WASHER 6 MM		5.00
1,2,4,5	119518	PIVOT SADDLE KIT		25.00
3,6	119509	HANGER LATCH ASSM W/HDWR L	Left	60.00
3,6	119510	HANGER LATCH ASSM W/HDWR R	Right	60.00
1-11	119466S	80 D SWING I/O HNGR PNTD L	Left	105.00
1-11	119467S	80 D SWING I/O HNGR PNTD R	Right	105.00
8-11	107485	FOOTREST CLAMP ASSM		30.00
12-15	119514	LATCH RECIEVER KIT		45.00

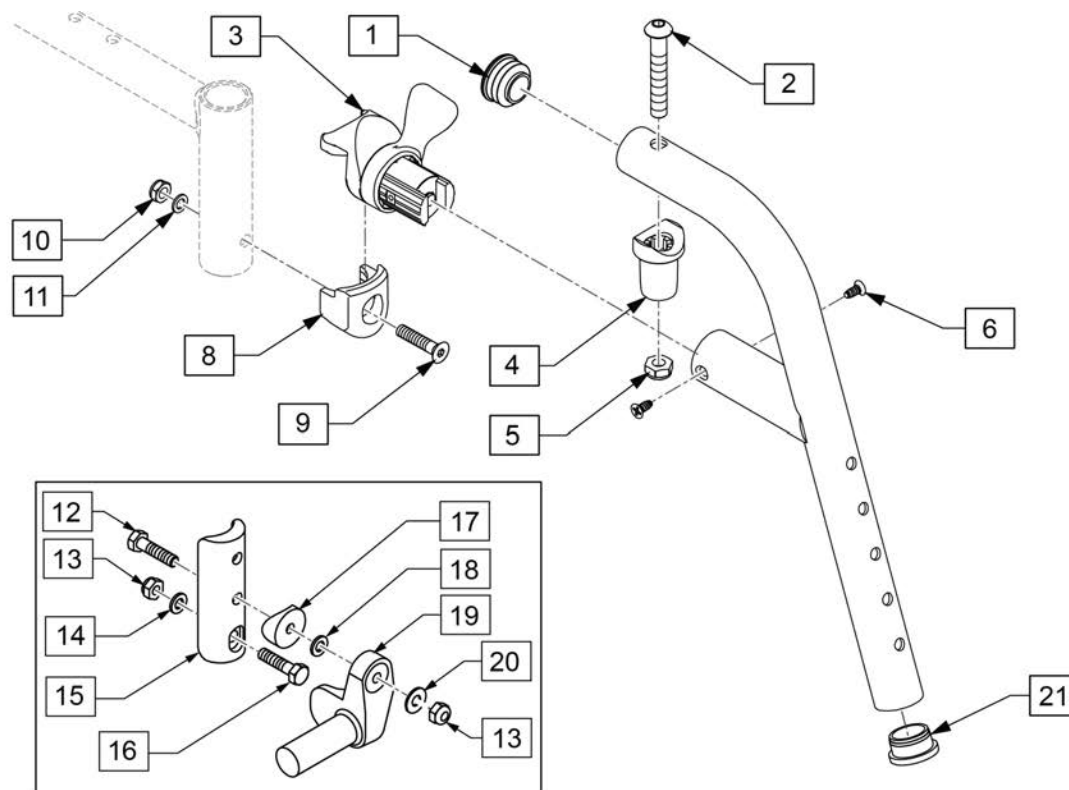

*FRONT MOUNT ADAPTER KIT FOR ADULT ANGLE-ADJUSTABLE FOOTPLATE.


Pos.	Item Number	Description	Remarks	USD
1	561062	1" INSERT PLUG .065 RIBS		2.00
2	226217	5/16-18 X 2" BH C/S BLK		2.00
4	561197	PLASTIC PIVOT SADDLE		2.00
5	202102	5/16-18 NYLOCK NUT THIN		
6	118695	#8 LOW FLT HD TAP SCREW		
9	103500-010	FHCSM6X1X35FULLNP		2.00
10	104381	NUT HEX NYLOCK M6X1.0		5.00
11	104414	WAVE WASHER 6 MM		5.00
12	102268-001	HHCS M6 X 1 X 25		10.00
13	104381	NUT HEX NYLOCK M6 X 1		5.00
14	107383	WASHER FLAT M6 ROHS		2.00
15	119240	FOOTPLATE ADAPTER FM		
16	118198	HHCS M6X1X40 PART NP		3.00
17	561004	1" SADDLE 1/4" HOLE		2.00
18	106943	WASHER FLAT AN 1/4 ROHS		1.00
19	541000	FOOTREST MOUNTING BRKT-1" RAD	Right	18.21
19	541001	FOOTREST MOUNTING BRKT-1" RAD	Left	18.21
20	106945	WASHER FLAT SAE 1/4 ROHS		1.00
1,2,4,5	119518	PIVOT SADDLE KIT		25.00
1-7	119456S	70 D FRNT MNT SWING IN/OUT HANGER L	Left	105.00
1-7	119457S	70 D FRNT MNT SWING IN/OUT HANGER R	Right	105.00
8-11	119514	LATCH RECIEVER KIT		45.00
3,6	119509	HANGER LATCH ASSM W/HDWR L	Left	60.00
3,6	119510	HANGER LATCH ASSM W/HDWR R	Right	60.00
12-20	129764	AAA FRNT MNT ADAPTER KIT PR	Pair	
21	561002	1" INSERT PLUG .083		2.00

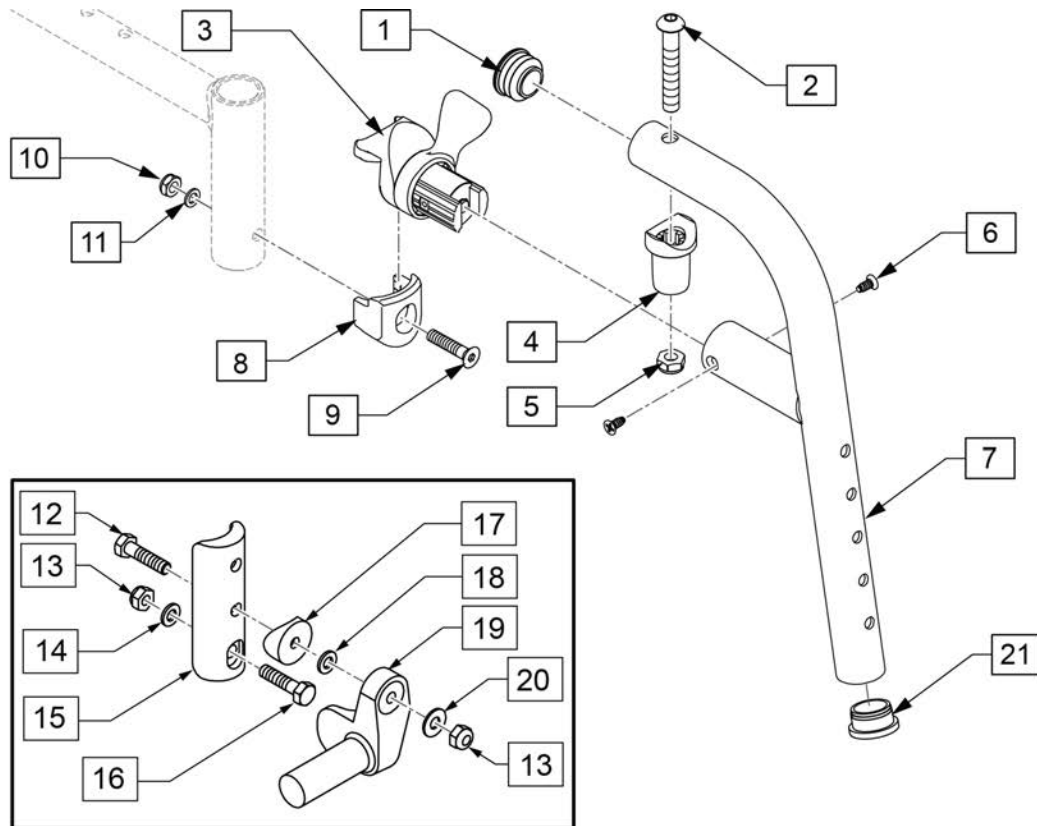

*FRONT MOUNT ADAPTER KIT FOR ADULT ANGLE-ADJUSTABLE FOOTPLATE.

Pos.	Item Number	Description	Remarks	USD
1	561062	1" INSERT PLUG .065 RIBS		2.00
2	226217	5/16-18 X 2" BH C/S BLK		2.00
4	561197	PLASTIC PIVOT SADDLE		2.00
5	202102	5/16-18 NYLOCK NUT THIN		
6	118695	#8 LOW FLT HD TAP SCREW		
9	103500-010	FHCSM6X1X35FULLNP		2.00
10	104381	NUT HEX NYLOCK M6X1.0		5.00
11	104414	WAVE WASHER 6 MM		5.00
12	102268-001	HHCS M6 X 1 X 25		10.00
13	104381	NUT HEX NYLOCK M6 X 1		5.00
14	107383	WASHER FLAT M6 ROHS		2.00
15	119240	FOOTPLATE ADAPTER FM		
16	118198	HHCS M6X1X40 PART NP		3.00
17	561004	1" SADDLE 1/4" HOLE		2.00
18	106943	WASHER FLAT AN 1/4 ROHS		1.00
19	541000	FOOTREST MOUNTING BRKT-1" RAD	Right	18.21
19	541001	FOOTREST MOUNTING BRKT-1" RAD	Left	18.21
20	106945	WASHER FLAT SAE 1/4 ROHS		1.00
1,2,4,5	119518	PIVOT SADDLE KIT		25.00
1-7	119464S	80 D FRNT MNT SWING IN/OUT HANGER L	Left	105.00
1-7	119465S	80 D FRNT MNT SWING IN/OUT HANGER R	Right	105.00
8-11	119514	LATCH RECIEVER KIT		45.00
3,6	119509	HANGER LATCH ASSM W/HDWR L	Left	60.00
3,6	119510	HANGER LATCH ASSM W/HDWR R	Right	60.00
21	561002	1" INSERT PLUG .083		2.00


Pos.	Item Number	Description	Remarks	USD
12-20	129764	AAA FRNT MNT ADAPTER KIT PR	Pair	
21	561002	1" INSERT PLUG .083		2.00

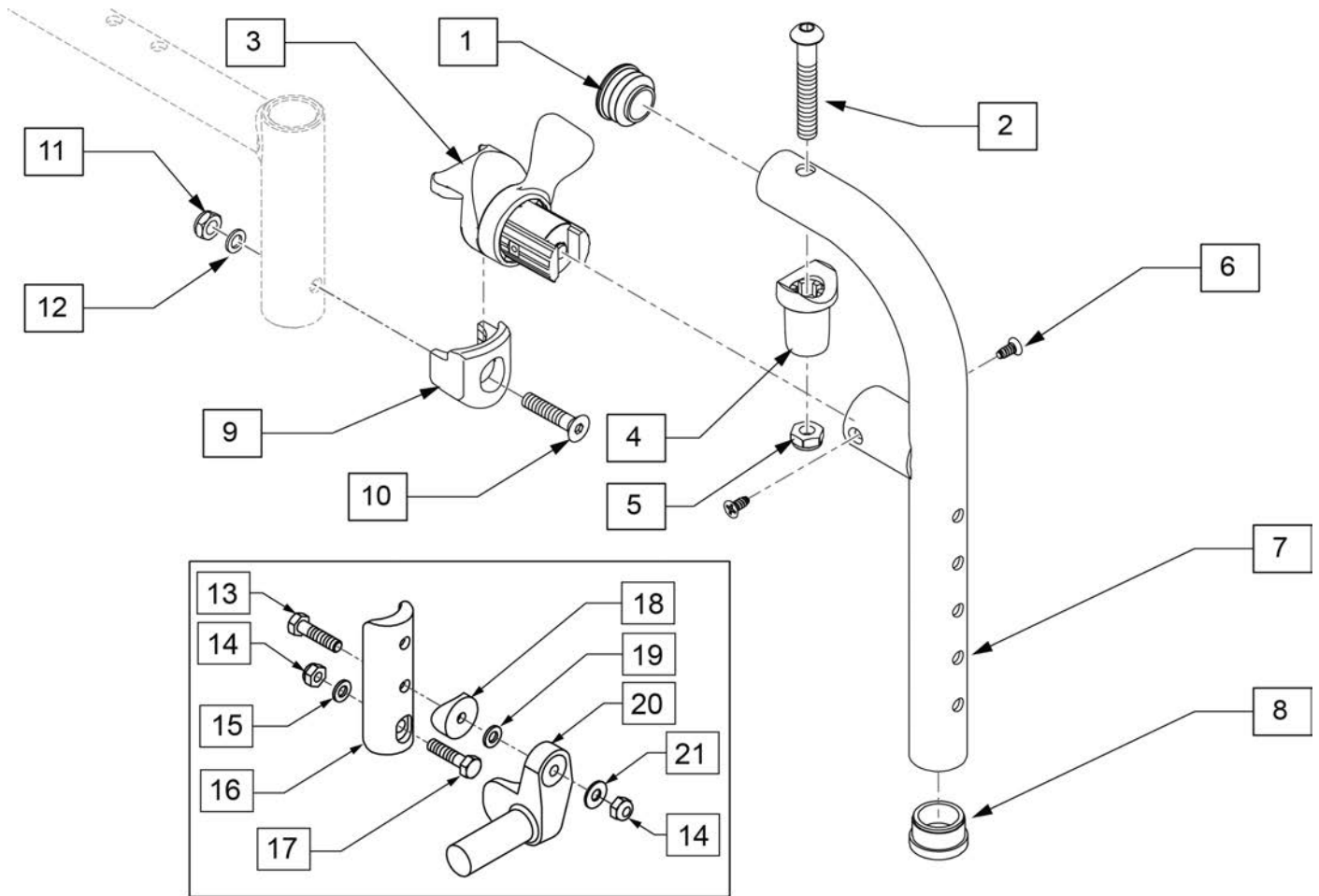


*FRONT MOUNT ADAPTER KIT FOR ADULT ANGLE-ADJUSTABLE FOOTPLATE.

Pos.	Item Number	Description	Remarks	USD
1	561062	1" INSERT PLUG .065 RIBS		2.00
2	226217	5/16-18 X 2" BH C/S BLK		2.00
4	561197	PLASTIC PIVOT SADDLE		2.00
5	202102	5/16-18 NYLOCK NUT THIN		
6	118695	#8 LOW FLT HD TAP SCREW		
8	561002	1" INSERT PLUG .083		2.00
10	103500-010	FHCSM6X1X35FULLNP		2.00
11	104381	NUT HEX NYLOCK M6X1.0		5.00
12	104414	WAVE WASHER 6 MM		5.00
13	102268-001	HHCS M6 X 1 X 25		10.00
14	104381	NUT HEX NYLOCK M6 X 1		5.00
15	107383	WASHER FLAT M6 ROHS		2.00
16	119240	FOOTPLATE ADAPTER FM		
17	118198	HHCS M6X1X40 PART NP		3.00
18	561004	1" SADDLE 1/4" HOLE		2.00
19	106943	WASHER FLAT AN 1/4 ROHS		1.00
20	541000	FOOTREST MOUNTING BRKT-1" RAD	Right	18.21
20	541001	FOOTREST MOUNTING BRKT-1" RAD	Left	18.21
21	106945	WASHER FLAT SAE 1/4 ROHS		1.00
1,2,4,5	119518	PIVOT SADDLE KIT		25.00
1-7	129644S	90 D FRNT MNT SWING IN/OUT HANGER R	Right	


Pos.	Item Number	Description	Remarks	USD
1-7	129645S	90 D FRNT MNT SWING IN/OUT HANGER L	Left	
9-12	119514	LATCH RECIEVER KIT		45.00
3,6	119509	HANGER LATCH ASSM W/HDWR L	Left	60.00
3,6	119510	HANGER LATCH ASSM W/HDWR R	Right	60.00
13-21	129764	AAA FRNT MNT ADAPTER KIT PR	Pair	

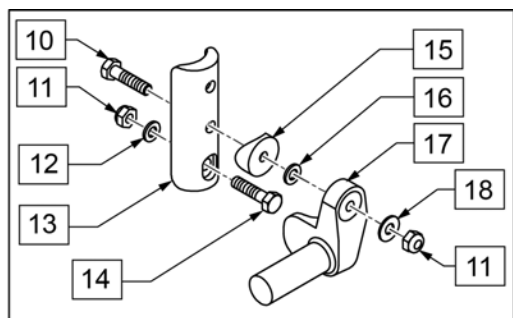

MEASURING GUIDE

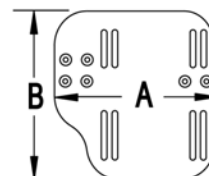


CHAIR WIDTH	"A"	"B"
10" - 11"	3 1/2"	7 1/4"
12" - 13"	4 1/2"	7 3/4"
14" - 15"	5 1/2"	7 3/4"
16" - 17"	6 1/2"	7 3/4"
18" - 20"	7 1/2"	7 3/4"

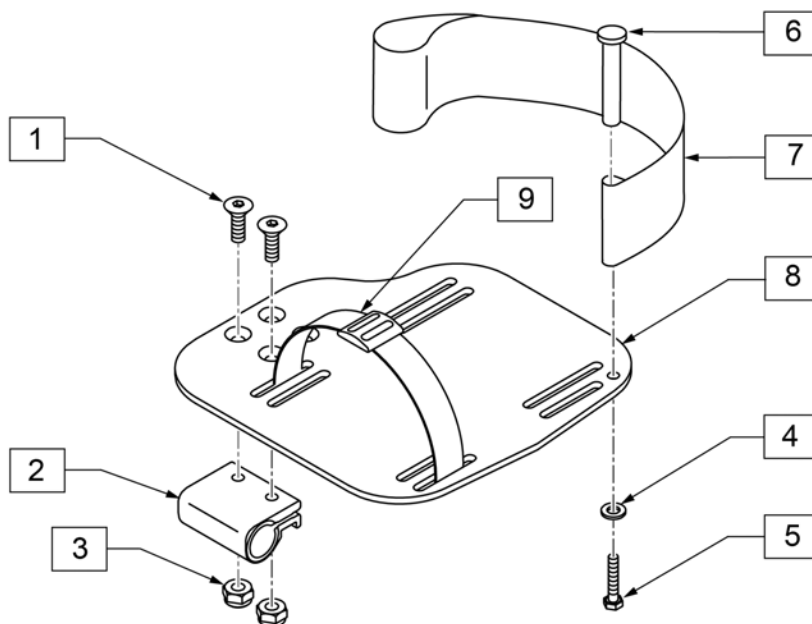


Pos.	Item Number	Description	Remarks	USD
1	106978	FHCS 1/4-28 X 7/8 ROHS		5.00
2	550350	3/4" CLAMP ADJ FOOTPLATE		11.04
3	106935	NUT NYLOCK THIN 1/4-28 ROHS		5.00
4	107383	WASHER FLAT M6 ROHS		2.00
5	102268-001	HHCS M6 X 1 X 25		10.00
6	561040	HEEL LOOP PEG		2.43
7	104380-001	ADJ FOOTREST STRAP 14W		7.50
7	104380-002	ADJ FOOTREST STRAP 15-16W		7.50
7	104380-003	ADJ FOOTREST STRAP 17-18W		7.50
7	104380-004	ADJ FOOTREST STRAP 19-20W		7.50
8	500656	ADULT ANGL ADJ F/PLT 10/11	RIGHT	58.14
8	500657	ADULT ANGL ADJ F/PLT 10/11	LEFT	58.14
8	500658	ADULT ANGL ADJ F/PLT 12/13	RIGHT	60.68
8	500659	ADULT ANGL ADJ F/PLT 12/13	LEFT	60.68
8	500660	ADULT ANGL ADJ F/PLT 14/15	RIGHT	60.68
8	500661	ADULT ANGL ADJ F/PLT 14/15	LEFT	60.68
8	500662	ADULT ANGL ADJ F/PLT 16/17	RIGHT	60.68
8	500663	ADULT ANGL ADJ F/PLT 16/17	LEFT	60.68
8	500664	ADULT ANGL ADJ F/PLT 18-20	RIGHT	60.68
8	500665	ADULT ANGL ADJ F/PLT 18-20	LEFT	60.68
9	740260	TOE-ANKLE STRAP ADJ ADULT FOOTPLATE		20.40

Pos.	Item Number	Description	Remarks	USD
10	102268-001	HHCS M6 X 1 X 25		10.00
11	104381	NUT HEX NYLOCK M6 X 1		5.00
12	107383	WASHER FLAT M6 ROHS		2.00
14	118198	HHCS M6X1X40 PART NP		3.00
15	561004	1" SADDLE 1/4" HOLE		2.00
16	106943	WASHER FLAT AN 1/4 ROHS		1.00
18	106945	WASHER FLAT SAE 1/4 ROHS		1.00
10-18	129764	AAA FRNT MNT ADAPTER KIT PR	Pair	
1-8	129765	AAA F/PLT KIT 10-11 RT	RIGHT	
1-8	129766	AAA F/PLT KIT 10-11 LFT	LEFT	
1-8	129767	AAA F/PLT KIT 12-13 RT	RIGHT	
1-8	129768	AAA F/PLT KIT 12-13 LFT	LEFT	
1-8	129769	AAA F/PLT KIT 14-15 RT	RIGHT	
1-8	129770	AAA F/PLT KIT 14-15 LFT	LEFT	
1-8	129771	AAA F/PLT KIT 17-18 RT	RIGHT	
1-8	129772	AAA F/PLT KIT 17-18 LFT	LEFT	
1-8	129773	AAA F/PLT KIT 18-20 RT	RIGHT	

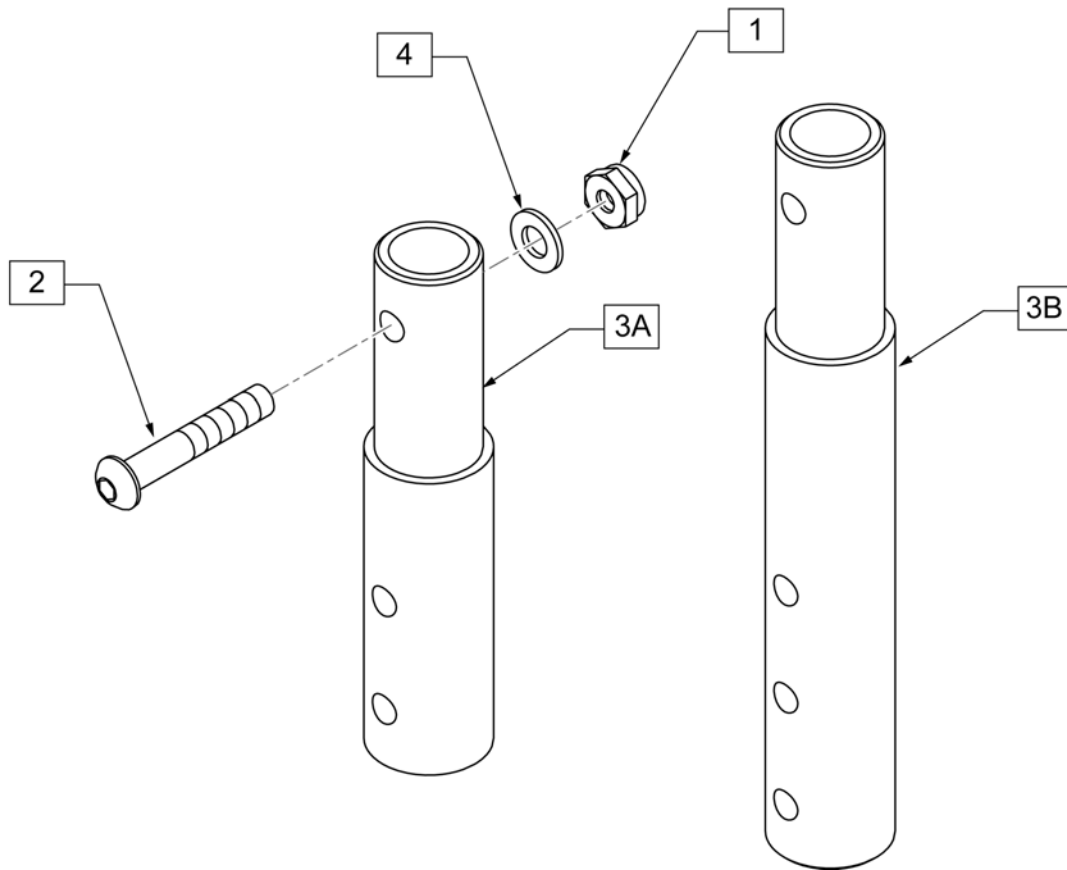


Pos.	Item Number	Description	USD
1	106935	NUT NYLOCK THIN 1/4 -28 ROHS	5.00
2	106961	BHCS 1/4 -28 x 1 1/4 ROHS	5.00
4	107383	WASHER FLAT M6 ROHS	2.00
1-3A,4	119507	FRONT MOUNT HANGER EXT 2"	35.00
1-3B,4	119508	FRONT MOUNT HANGER EXT 4"	35.00

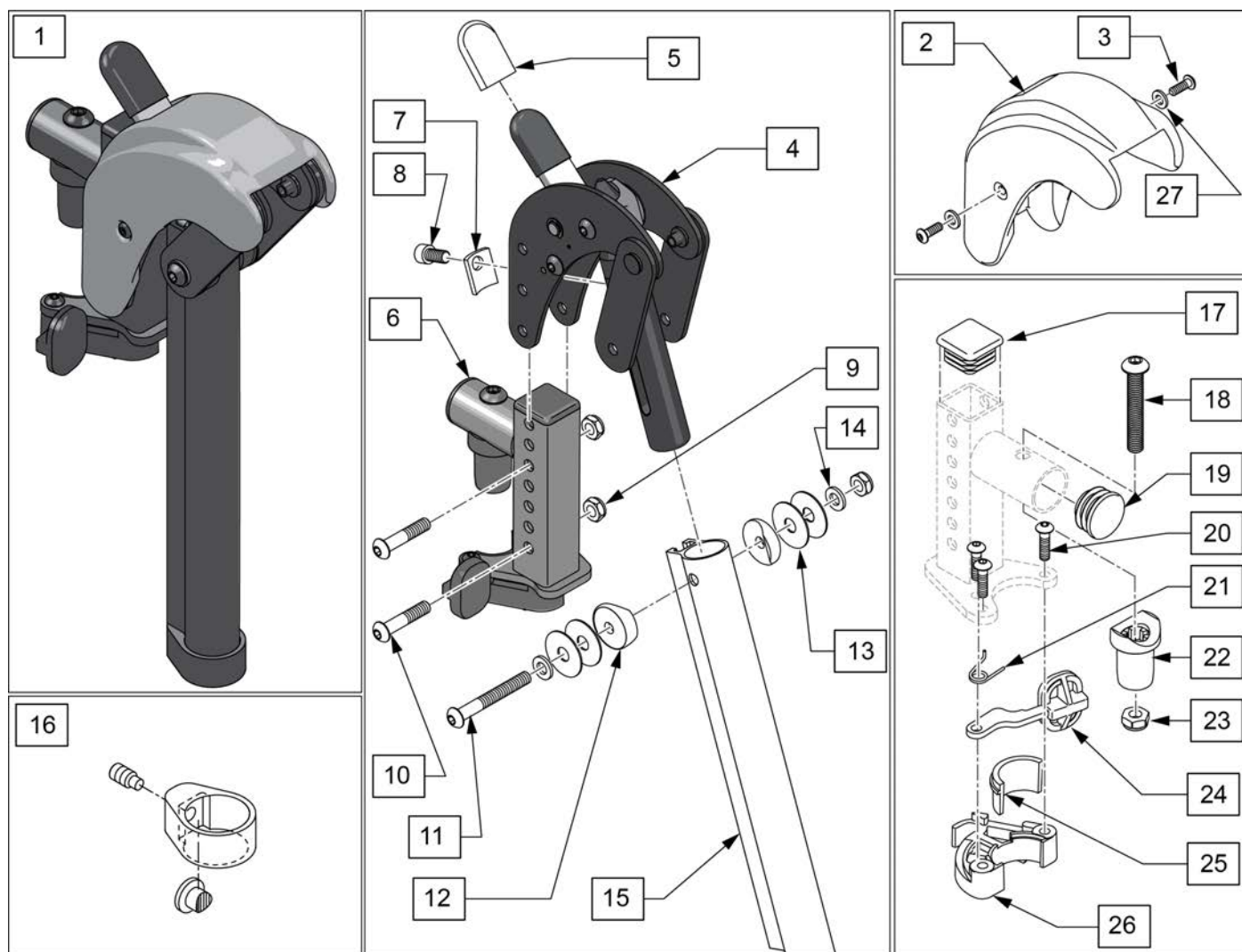


Pos.	Item Number	Description	Remarks	USD
1	129631S	ALR KIT R		
1	129632S	ALR KIT L		
1	129630S	ALR PAIR KIT		
2,3,27	129633	ALR SHROUD KIT	1 side	
3	127135	BHCS 8-32 X 1/2 BLK		
4	127153	HEMI ALR SUB ASSM		
5	560501	ALR REL LEVER COVER		2.00
6	127114	HEMI RECIVER ASSM ELR ALR R	Right	
6	127115	HEMI RECIVER ASSM ELR ALR L	Left	
7	127129	ANGLE STOP ELR		
8	107350	SHCS 1/4-28 x 1/2		
9	106935	1/4-28 NYLOCK NUT THIN BLK ZN		5.00
10	108623	BHCS 1/4-28 x 1-1/2		3.00
11	106964	BHCS 1/4-28 X 2		5.00
12	561086	1" HALF SADDLE 1/4" HOLE		2.00
13	113268	WASHER NYLON 1IN OD X .312 ID		3.00
14	106943	WASHER FLAT AN 1/4 ROHS		1.00
15	127116	TUBE HANGER ELR ALR		

Pos.	Item Number	Description	Remarks	USD
16	107485	FOOTREST CLAMP ASSM		30.00
17	010615	SQUARE END CAP CNTR LEG		2.00
18	226217	5/16-18 X 2" BH C/S BLK		2.00
19	561062	1" INSERT PLUG .065 RIBS		2.00
20	226057	BHCS 10-24 X .725 PL		2.00
22	561197	PLASTIC PIVOT SADDLE		2.00
23	202102	5/16-18 NYLOCK NUT THIN		
25,26	881500	LATCH BLOCK ASSM R	Right	7.65
25,26	881501	LATCH BLOCK ASSM L	Left	7.65
27	113282	WASHER FLAT AN #10 ROHS		
20,21,24-26	881521	S/A LATCH KIT CASTING RIGHT	Right	17.85
20,21,24-26	881522	S/A LATCH KIT CASTING LEFT	Left	17.85

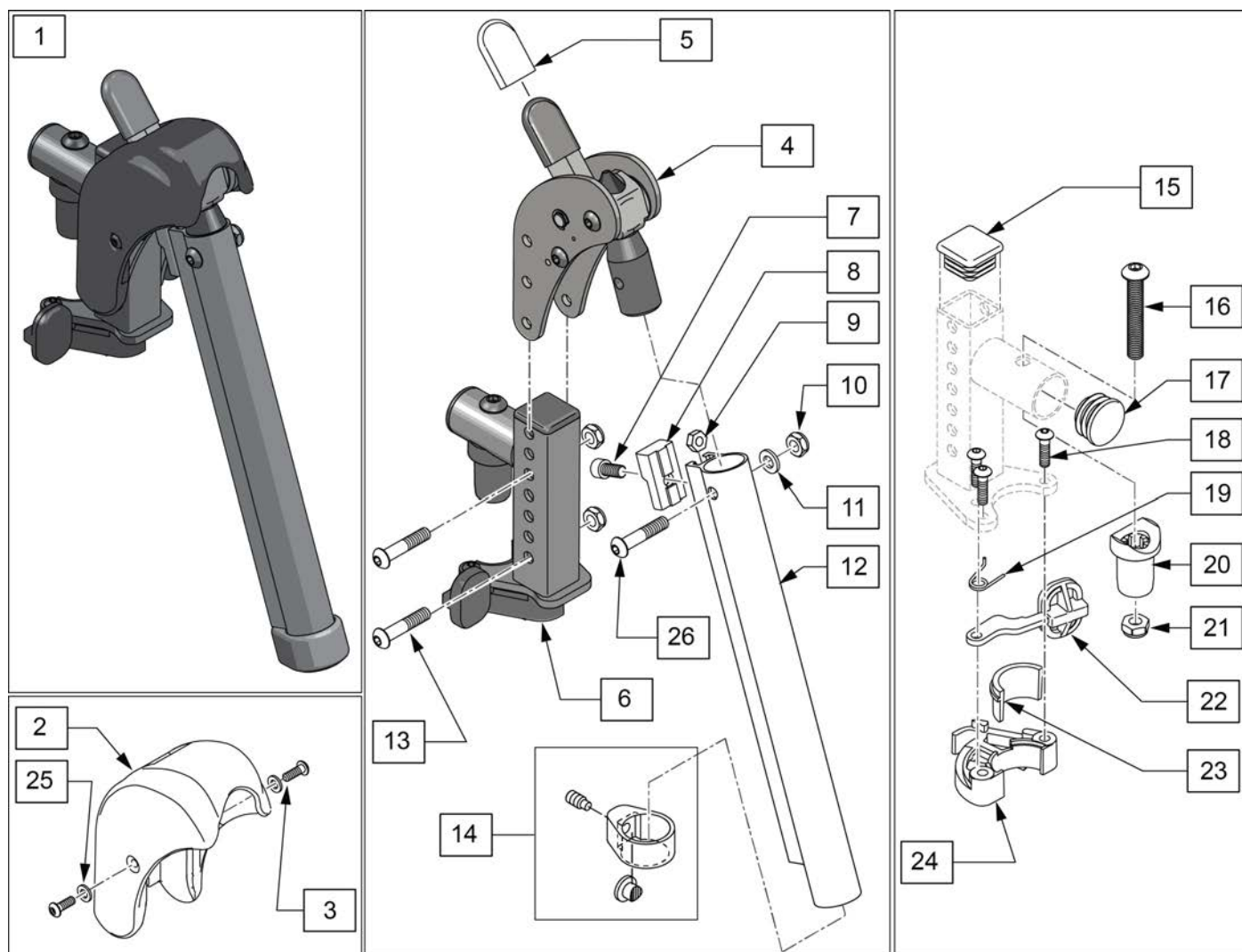


Pos.	Item Number	Description	Remarks	USD
1	129638S	EXTENSION MOUNT ELR KIT R	Right	
1	129639S	EXTENSION MOUNT ELR KIT L	Left	
1	129637S	EXTENSION MOUNT ELR PAIR KIT	Pair	
2,3,25	129640	ELR SHROUD KIT	Kit w/ 2 screws	
3	127135	BHCS 8-32 X 1/2 BLK		
4	126992	HEMI ELR SUB ASSM		
5	560501	ALR REL LEVER COVER		2.00
6	127114	HEMI RECIVER ASSM ELR ALR R	Right	
6	127115	HEMI RECIVER ASSM ELR ALR L	Left	
7	107350	SHCS 1/4-28 x 1/2		
8	127129	ANGLE STOP ELR		
9	107249	NUT HEX THIN 1/4-28		2.00
10	106935	1/4-28 NYLOCK NUT THIN BLK ZN		5.00
11	104414	WAVE WASHER 6 MM		5.00
12	127116	TUBE HANGER ELR ALR		
13	108623	BHCS 1/4-28 x 1-1/2		3.00
14	107485	FOOTREST CLAMP ASSM		30.00
15	010615	SQUARE END CAP CNTR LEG		2.00

Pos.	Item Number	Description	Remarks	USD
16	226217	5/16-18 X 2" BH C/S BLK		2.00
17	561062	1" INSERT PLUG .065 RIBS		2.00
18	226057	BHCS 10-24 X .725 PL		2.00
20	561197	PLASTIC PIVOT SADDLE		2.00
21	202102	5/16-18 NYLOCK NUT THIN		
23,24	881500	LATCH BLOCK ASSM R	Right	7.65
23,24	881501	LATCH BLOCK ASSM L	Left	7.65
25	113282	WASHER FLAT AN #10 ROHS		
26	106961	BHCS 1/4 -28 x 1 1/4 ROHS		5.00
18,19,22-24	881521	S/A LATCH KIT CASTING RIGHT	Right	17.85
18,19,22-24	881522	S/A LATCH KIT CASTING LEFT	Left	17.85

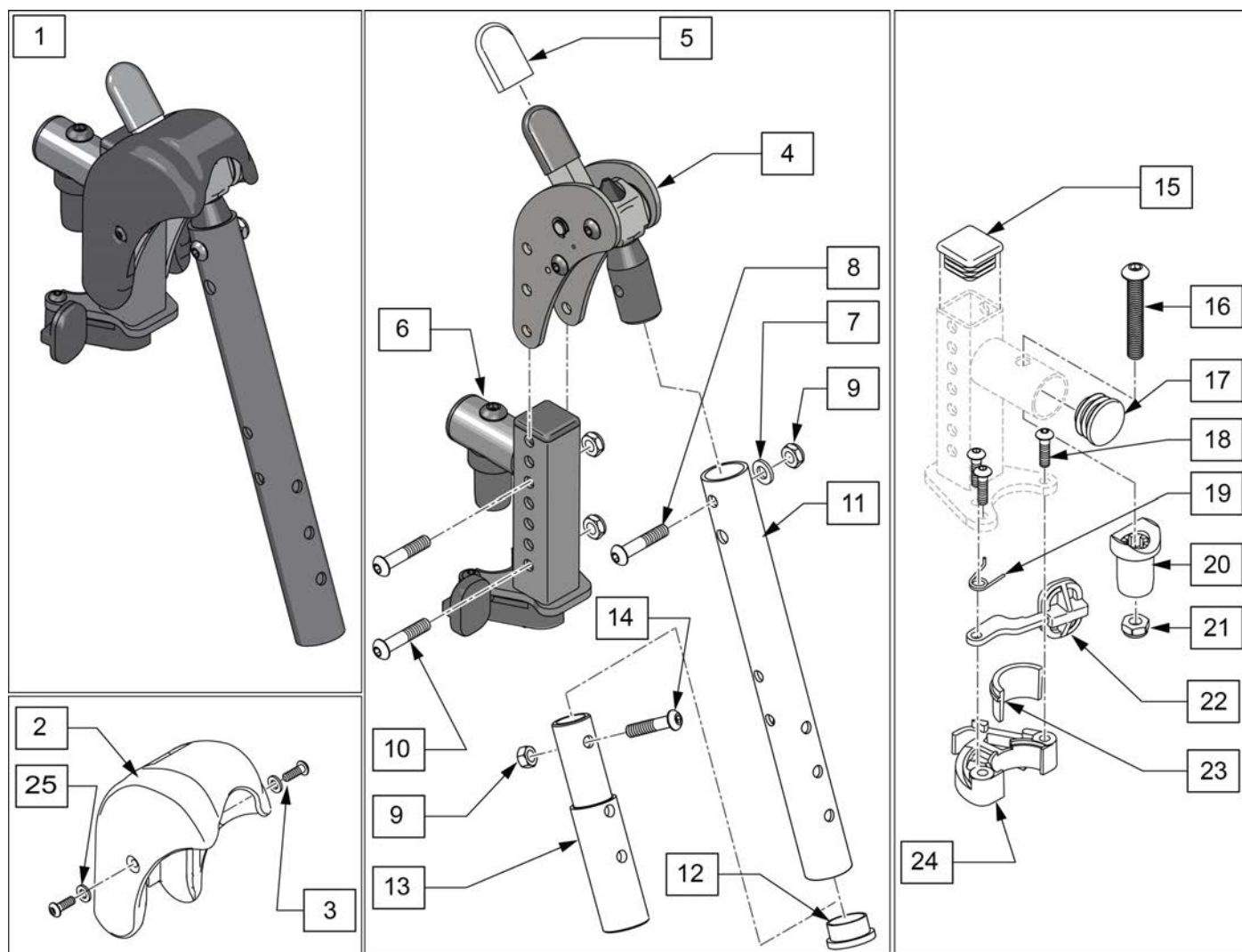


Pos.	Item Number	Description	Remarks	USD
1	129635S	DIRECT MOUNT ELR KIT R	Right	
1	129636S	DIRECT MOUNT ELR KIT L	Left	
1	129634S	DIRECT MOUNT ELR PAIR KIT	Pair	
2,3,25	129640	ELR SHROUD KIT	Kit w/ 2 screws	
3	127135	BHCS 8-32 X 1/2 BLK		
4	126992	HEMI ELR SUB ASSM		
5	560501	ALR REL LEVER COVER		2.00
6	127114	HEMI RECIVER ASSM ELR ALR R	Right	
6	127115	HEMI RECIVER ASSM ELR ALR L	Left	
7	104414	WAVE WASHER 6 MM		5.00
8	106961	BHCS 1/4 -28 x 1 1/4 ROHS		5.00
9	106935	1/4-28 NYLOCK NUT THIN BLK ZN		5.00
10	108623	BHCS 1/4-28 x 1-1/2		3.00
11	505097	ELR KIDS TUBE		26.48
12	561002	1" INSERT PLUG .083		2.00
13	500150	2" FOOTREST EXT TUBE		23.46
14	106961	1/4-28 x 1 1/4 BH C/S		5.00
15	10615	SQUARE END CAP CNTR LEG		

Pos.	Item Number	Description	Remarks	USD
17	561062	1" INSERT PLUG .065 RIBS		2.00
18	226057	BHCS 10-24 X .725 PL		2.00
16,17,20,21	119518	PIVOT SADDLE KIT		25.00
23,24	881500	LATCH BLOCK ASSM R	Right	7.65
23,24	881501	LATCH BLOCK ASSM L	Left	7.65
25	113282	WASHER FLAT AN #10 ROHS		
18,19,22-24	881521	S/A LATCH KIT CASTING RIGHT	Right	17.85
18,19,22-24	881522	S/A LATCH KIT CASTING LEFT	Left	17.85

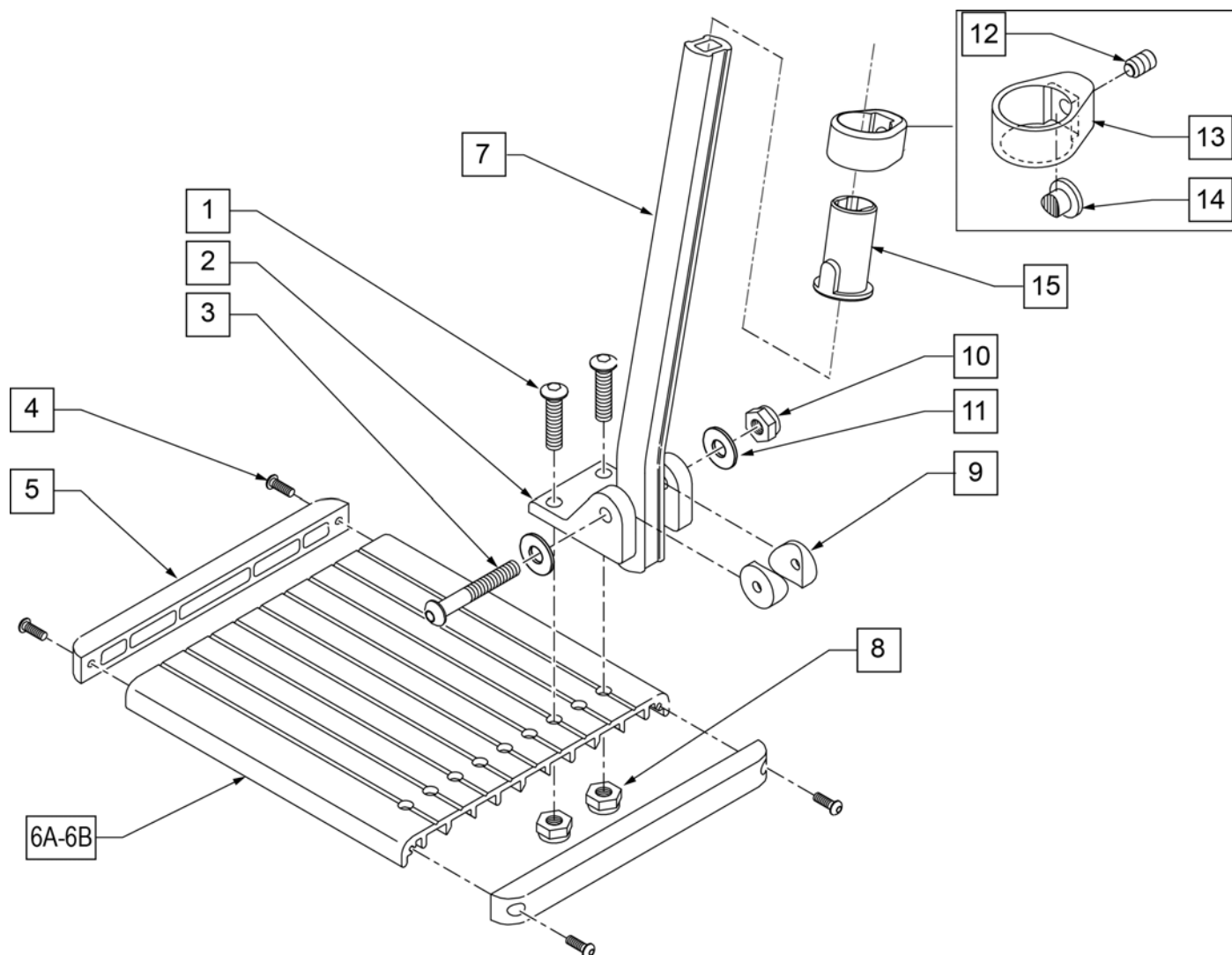


Pos.	Item Number	Description	USD
1	226090	1/4-28 x 5/8 BH C/S	
2	118093	BRACKET FOOTPLATE ALUM	5.00
3	102262-008	BHCS M6 X 1 X 50 INORGANIC	7.95
4	281043	#4 x 5/8 PAN PHIL S/TAP	2.00
5	560101	END CAPS FOOTREST	2.75
7	107126	SHORT BENT	10.00
7	107127	MED BENT	10.00
7	107128	LONG BENT	10.00
8	106935	1/4-28 NYLOCK NUT THIN	5.00
9	561003	3/4" SADDLE THIN	2.00
10	104381	NUT HEX NYLOCK M6X1.0	5.00
11	107383	WASHER FLAT M6 ROHS	2.00
4,5,6A	882337	ELR FOOTPLATE ASSEMBLY 14 -15"	31.62
4,5,6B	882338	ELR FOOTPLATE ASSEMBLY 16 -17"	31.62
4,5,6C	882339	ELR FOOTPLATE ASSEMBLY 18 -20"	31.62
12-15	107485	FOOTREST CLAMP ASSM	30.00
1,3,8-11	129775	ALU FOOTPLATE HDWARE KIT	

NOTE: Left Side Shown

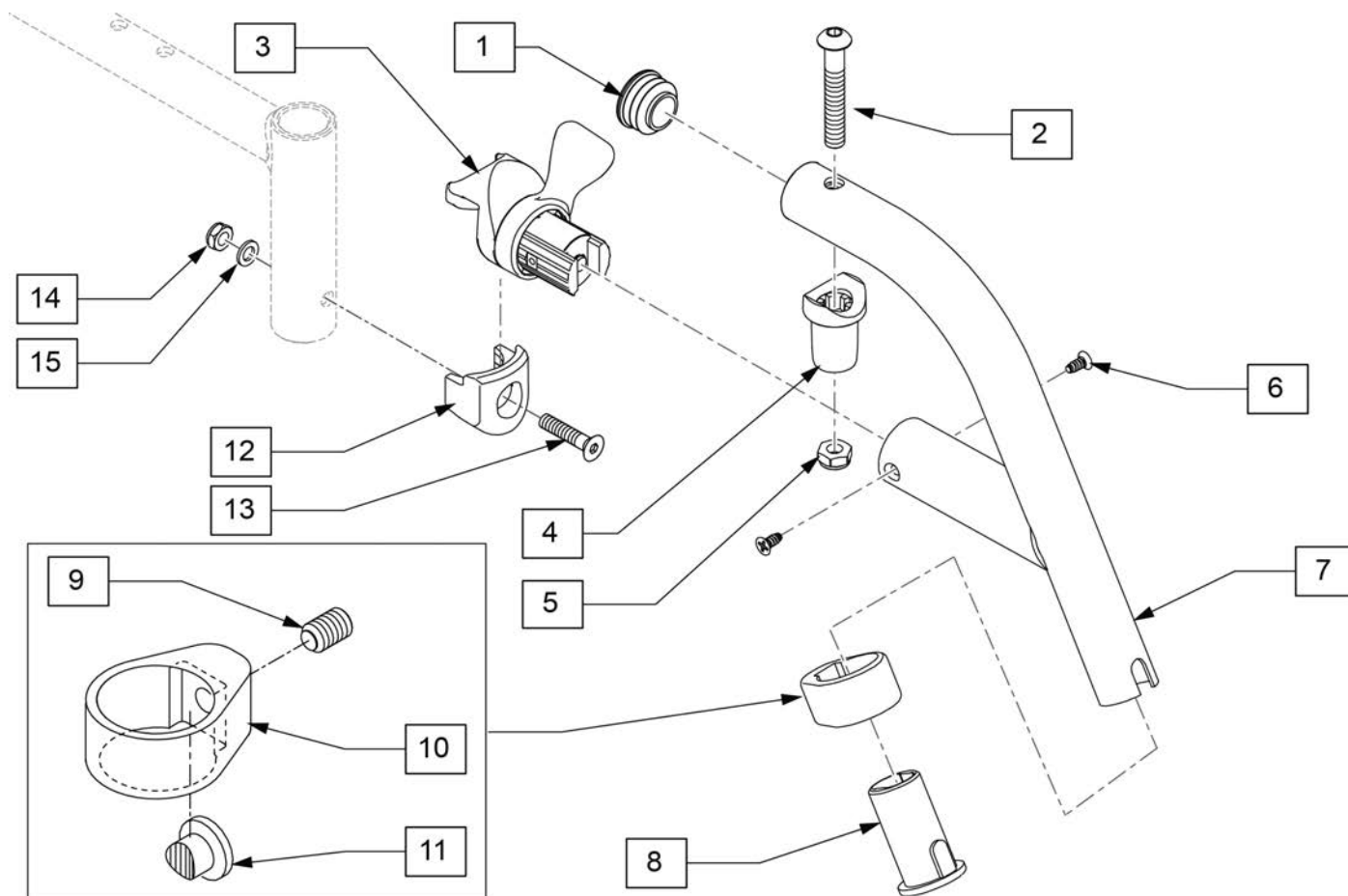


Pos.	Item Number	Description	Remarks	USD
1	561062	1" INSERT PLUG .065 RIBS		2.00
2	226217	5/16-18 X 2" BH C/S BLK		2.00
4	561197	PLASTIC PIVOT SADDLE		2.00
5	202102	5/16-18 NYLOCK NUT THIN		
6	118695	#8 LOW FLT HD TAP SCREW		
8	104160	BUSHING FOOTREST		5.00
13	103500-010	FHCSM6X1X35FULLNP		2.00
14	104381	NUT HEX NYLOCK M6X1.0		5.00
15	104414	WAVE WASHER 6 MM		5.00
1,2,4,5	119518	PIVOT SADDLE KIT		25.00
3,6	119510	HANGER LATCH ASSM W/HDWR R	Right	60.00
3,6	119509	HANGER LATCH ASSM W/HDWR L	Left	60.00
1-11	129702S	60 D SWING HANGER SHORT R	Right	
1-11	129703S	60 D SWING HANGER SHORT L	Left	
8-11	107485	FOOTREST CLAMP ASSM		30.00
12-15	119514	LATCH RECIEVER KIT		45.00

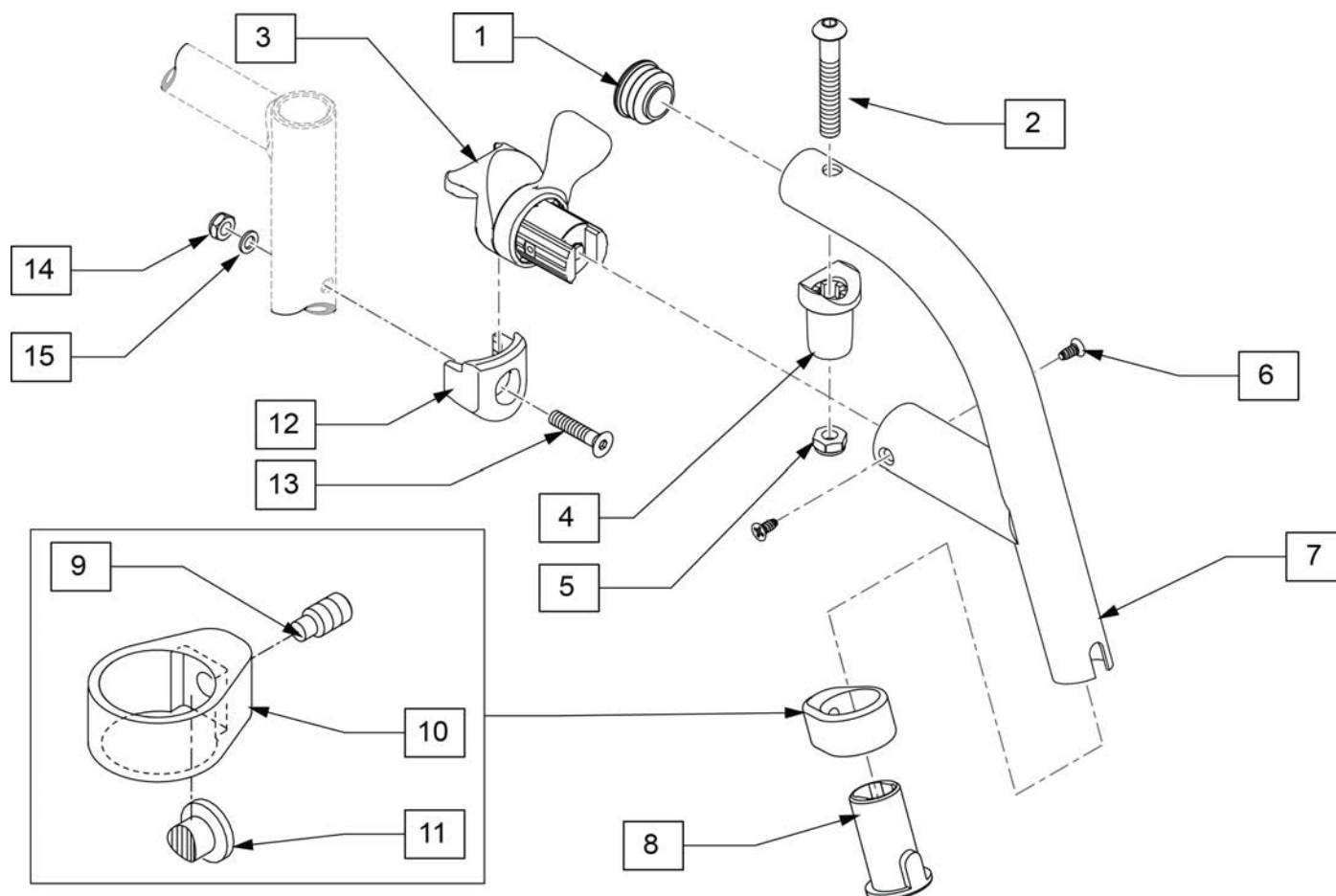

Pos.	Item Number	Description	Remarks	USD
1	561062	1" INSERT PLUG .065 RIBS		2.00
2	226217	5/16-18 X 2" BH C/S BLK		2.00
4	561197	PLASTIC PIVOT SADDLE		2.00
5	202102	5/16-18 NYLOCK NUT THIN		
6	118695	#8 LOW FLT HD TAP SCREW		
8	104160	BUSHING FOOTREST		5.00
13	103500-010	FHCSM6X1X35FULLNP		2.00
14	104381	NUT HEX NYLOCK M6X1.0		5.00
15	104414	WAVE WASHER 6 MM		5.00
1,2,4,5	119518	PIVOT SADDLE KIT		25.00
3,6	119510	HANGER LATCH ASSM W/HDWR R	Right	60.00
3,6	119509	HANGER LATCH ASSM W/HDWR L	Left	60.00
1-11	129704S	70 D SWING HANGER SHORT R	Right	
1-11	129705S	70 D SWING HANGER SHORT L	Left	
8-11	107485	FOOTREST CLAMP ASSM		30.00
12-15	119514	LATCH RECIEVER KIT		45.00


NOTE: Footplate Assemblies are set to Footrest width. Please see "Footplate ordering Guides" in the information panel for proper conversion.

Note: If using Short Ext Tube & Hemi Hanger with Composite Footplate use 1G (bent) .

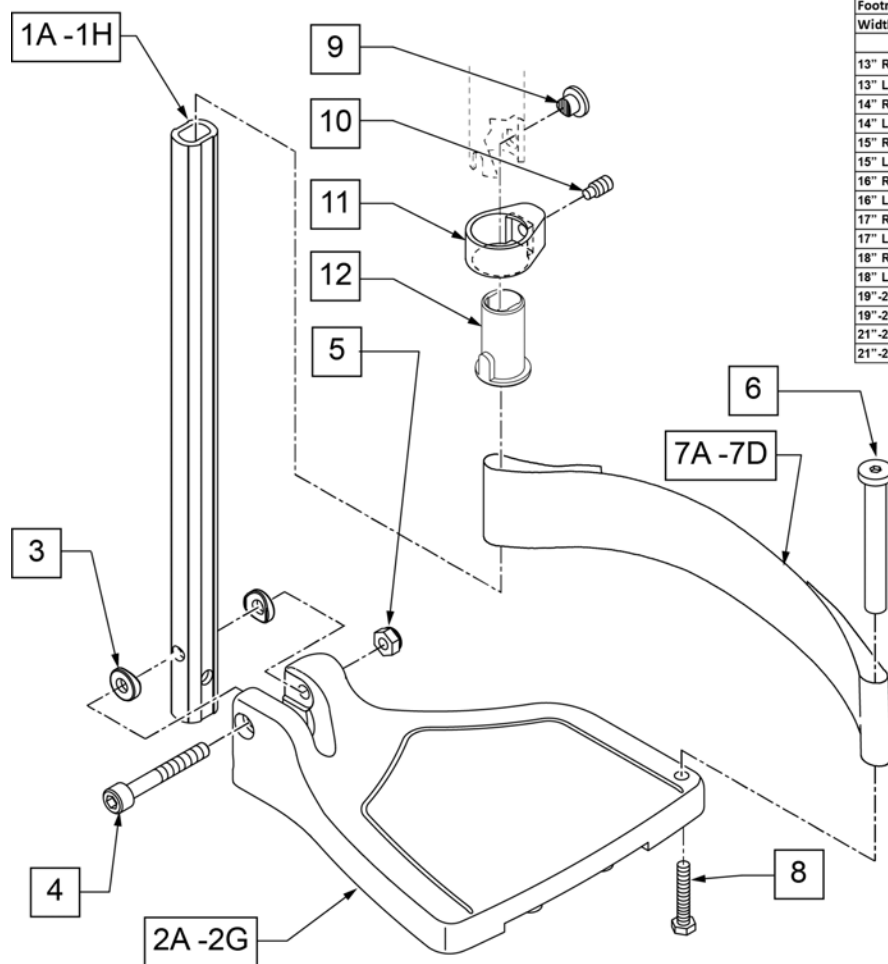

Composite Footplates w/ Ext & HL for Swing I/O Hangers							
(Order 1 each)							
Ext. Tube	Hemi Short Hanger	60°, 70° & 70° Hemi Style, ELR & ALR Hangers (Bent Extension)				80° Style hanger (Straight Extension)	
Footrest Width		Bent	Short	Medium	Long	Short	Medium
13" Right	129724	107746	107760	107774	107704	107718	107732
13" Left	129725	107753	107767	107781	107711	107725	107739
14" Right	129726	107747	107761	107775	107705	107719	107733
14" Left	129727	107754	107768	107782	107712	107726	107740
15" Right	129728	107748	107762	107776	107706	107720	107734
15" Left	129729	107755	107769	107783	107713	107727	107741
16" Right	129730	107749	107763	107777	107707	107721	107735
16" Left	129731	107756	107770	107784	107714	107728	107742
17" Right	129732	107750	107764	107778	107708	107722	107736
17" Left	129733	107757	107771	107785	107715	107729	107743
18" Right	129734	107751	107765	107779	107709	107723	107737
18" Left	129735	107758	107772	107786	107716	107730	107744
19"-20" Rt	129736	107752	107766	107780	107710	107724	107738
19"-20" Lt	129737	107759	107773	107787	107717	107731	107745
21"-22" Rt	129738	111528	111530	111532	111522	111524	111526
21"-22" Lt	129739	111529	111531	111533	111523	111525	111527

FOOTPLATE MEASURING GUIDE

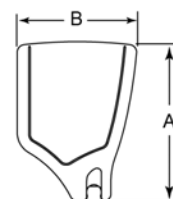

PART NO	DESCRIPTION	DIM -A-	DIM -B-
063216-006	FOOTPLATE COMPOSITE 157 R	6 1/8"	6"
063216-012	FOOTPLATE COMPOSITE 157 L	6 1/8"	6"
063216-015	FOOTPLATE COMPOSITE 170 R	6 5/8"	6"
063216-018	FOOTPLATE COMPOSITE 170 L	6 5/8"	6"
063216-005	FOOTPLATE COMPOSITE 182 R	7 1/8"	6"
063216-011	FOOTPLATE COMPOSITE 182 L	7 1/8"	6"
063216-014	FOOTPLATE COMPOSITE 195 R	7 5/8"	6"
063216-017	FOOTPLATE COMPOSITE 195 L	7 5/8"	6"
063216-004	FOOTPLATE COMPOSITE 207 R	8 1/8"	6"
063216-010	FOOTPLATE COMPOSITE 207 L	8 1/8"	6"
063216-013	FOOTPLATE COMPOSITE 220 R	8 5/8"	6 1/4"
063216-016	FOOTPLATE COMPOSITE 220 L	8 5/8"	6 1/4"
063216-003	FOOTPLATE COMPOSITE 232 R	9 1/8"	6 1/4"
063216-009	FOOTPLATE COMPOSITE 232 L	9 1/8"	6 1/4"
063216-002	FOOTPLATE COMPOSITE 257 R	10 1/8"	6 1/4"
063216-008	FOOTPLATE COMPOSITE 257 L	10 1/8"	6 1/4"
063216-001	FOOTPLATE COMPOSITE 282 R	11 1/8"	6 1/4"
063216-007	FOOTPLATE COMPOSITE 282 L	11 1/8"	6 1/4"

Pos.	Item Number	Description	Remarks	USD
1A,3-5	120660	SHT STRT EXT COMP HDWR	80 deg hangers + ELR only	20.00
1B,3-5	120661	MED STRT EXT COMP HDWR	80 deg hangers + ELR only	20.00
1C,3-5	120662	LNG STRT EXT COMP HDWR	80 deg hangers + ELR only	20.00
1D,3-5	120663	SHT BNT EXT COMP HDWR	60,70,70 Hemi Hangers	20.00
1E,3-5	120664	MED BNT EXT COMP HDWR	60,70,70 Hemi Hangers	20.00
1F,3-5	120665	LNG BNT EXT COMP HDWR	60,70,70 Hemi Hangers	20.00
1G	128078	TUBE FOOTREST EXT SHT HEMI	use for short hangers-bent	
2A	063216-006	COMPOSITE FOOTPLATE RIGHT	RIGHT 13" Footrest Width	20.00
2A	063216-012	COMPOSITE FOOTPLATE LEFT	LEFT 13" Footrest Width	20.00
2B	063216-015	FOOTPLATE COMPOSITE RIGHT	RIGHT 14" Footrest Width	20.00


Pos.	Item Number	Description	Remarks	USD
2B	063216-018	FOOTPLATE COMPOSITE LEFT	LEFT 14" Footrest Width	20.00
2C	063216-005	COMPOSITE FOOTPLATE RIGHT	RIGHT 15" Footrest Width	20.00
2C	063216-011	COMPOSITE FOOTPLATE LEFT	LEFT 15" Footrest Width	20.00
2D	063216-014	FOOTPLATE COMPOSITE RIGHT	RIGHT 16" Footrest Width	20.00
2D	063216-017	FOOTPLATE COMPOSITE LEFT	LEFT 16" Footrest Width	20.00
2E	063216-004	COMPOSITE FOOTPLATE RIGHT	RIGHT 17" Footrest Width	20.00
2E	063216-010	COMPOSITE FOOTPLATE LEFT	LEFT 17" Footrest Width	20.00
2F	063216-013	FOOTPLATE COMPOSITE RIGHT	RIGHT 18" Footrest Width	20.00
2F	063216-016	FOOTPLATE COMPOSITE LEFT	LEFT 18" Footrest Width	20.00
2G	063216-003	COMPOSITE FOOTPLATE RIGHT	RIGHT 19" & 20" Footrest Width	20.00
2G	063216-009	COMPOSITE FOOTPLATE LEFT	LEFT 19" & 20" Footrest Width	20.00
2H	063216-002	FOOTPLATE COMPOSITE RIGHT	RIGHT 21" & 22" Footrest Width	20.00
2H	063216-008	FOOTPLATE COMPOSITE LEFT	LEFT 21" & 22" Footrest Width	20.00
3	104377	3/4" SADDLE THIN		5.00
4	103175	SHCS M6 X 1 X 40 INORG		5.00
5	104381	NUT HEX NYLOCK M6X1.0	Inserted in bottom of footplate.	5.00
6	104328	HEEL LOOP PEG		5.00
7A	104380-001	ADJUSTABLE FOOT REST STRAP	14" wide	7.50
7B	104380-002	ADJUSTABLE FOOT REST STRAP	15-16" wide	7.50
7C	104380-003	ADJUSTABLE FOOT REST STRAP	17-18" wide	7.50
7D	104380-004	ADJUSTABLE FOOT REST STRAP	19-22" wide	7.50
8	102268-001	HHCS M6 X 1 X 25		10.00
12	104160	BUSHING FOOTREST		5.00
9-12	107485	FOOTREST CLAMP ASSM		30.00
>>>>		80 DEGREE HANGERS - SHORT EXT - RIGHT		
1A,2A,3-6,7A,8	107704	COMP STRT SHT EXT ASM 13"W R	Right- 80D- 13" W- SHORT	65.00
1A,2B,3-6,7A,8	107705	COMP STRT SHT EXT ASM 14"W R	Right- 80D- 14" W- SHORT	65.00
1A,2C,3-6,7B,8	107706	COMP STRT SHT EXT ASM 15"W R	Right- 80D- 15" W- SHORT	65.00
1A,2D,3-6,7B,8	107707	COMP STRT SHT EXT ASM 16"W R	Right- 80D- 16" W- SHORT	65.00
1A,2E,3-6,7C,8	107708	COMP STRT SHT EXT ASM 17"W R	Right- 80D- 17" W- SHORT	65.00
1A,2F,3-6,7C,8	107709	COMP STRT SHT EXT ASM 18"W R	Right- 80D- 18" W- SHORT	65.00
1A,2G,3-6,7D,8	107710	COMP STRT SHT EXT ASM 19 & 20"W R	Right- 80D- 19 & 20" W- SHORT	65.00
1A,2H,3-6,7D,8	111522	COMP STRT SHT EXT ASM 21 & 22"W R	Right- 80D- 21 & 22" W- SHORT	65.00
>>>>		80 DEGREE HANGERS - SHORT EXT - LEFT		
1A,2A,3-6,7A,8	107711	COMP STRT SHT EXT ASM 13"W L	LEFT- 80D- 13" W- SHORT	65.00
1A,2B,3-6,7A,8	107712	COMP STRT SHT EXT ASM 14"W L	LEFT- 80D- 14" W- SHORT	65.00
1A,2C,3-6,7B,8	107713	COMP STRT SHT EXT ASM 15"W L	LEFT- 80D- 15" W- SHORT	65.00
1A,2D,3-6,7B,8	107714	COMP STRT SHT EXT ASM 16"W L	LEFT- 80D- 16" W- SHORT	65.00
1A,2E,3-6,7C,8	107715	COMP STRT SHT EXT ASM 17"W L	LEFT- 80D- 17" W- SHORT	65.00
1A,2F,3-6,7C,8	107716	COMP STRT SHT EXT ASM 18"W L	LEFT- 80D- 18" W- SHORT	65.00
1A,2G,3-6,7D,8	107717	COMP STRT SHT EXT ASM 19 & 20"W L	LEFT- 80D- 19 & 20" W- SHORT	65.00
1A,2H,3-6,7D,8	111523	COMP STRT SHT EXT ASM 21 & 22"W L	LEFT- 80D- 21 & 22" W- SHORT	65.00
>>>>		80 DEGREE HANGERS - MED EXT - RIGHT		
1B,2A,3-6,7A,8	107718	COMP STRT MED EXT ASM 13"W R	Right- 80D- 13" W- MED	65.00
1B,2B,3-6,7A,8	107719	COMP STRT MED EXT ASM 14"W R	Right- 80D- 14" W- MED	65.00
1B,2C,3-6,7B,8	107720	COMP STRT MED EXT ASM 15"W R	Right- 80D- 15" W- MED	65.00
1B,2D,3-6,7B,8	107721	COMP STRT MED EXT ASM 16"W R	Right- 80D- 16" W- MED	65.00
1B,2E,3-6,7C,8	107722	COMP STRT MED EXT ASM 17"W R	Right- 80D- 17" W- MED	65.00
1B,2F,3-6,7C,8	107723	COMP STRT MED EXT ASM 18"W R	Right- 80D- 18" W- MED	65.00
1B,2G,3-6,7D,8	107724	COMP STRT MED EXT ASM 19 & 20"W R	Right- 80D- 19 & 20" W- MED	65.00
1B,2H,3-6,7D,8	111524	COMP STRT MED EXT ASM 21 & 22"W R	Right- 80D- 21 & 22" W- MED	65.00
>>>>		80 DEGREE HANGERS - MED EXT - LEFT		
1B,2A,3-6,7A,8	107725	COMP STRT MED EXT ASM 13"W L	LEFT- 80D- 13" W- MED	65.00
1B,2B,3-6,7A,8	107726	COMP STRT MED EXT ASM 14"W L	LEFT- 80D- 14" W- MED	65.00
1B,2C,3-6,7B,8	107727	COMP STRT MED EXT ASM 15"W L	LEFT- 80D- 15" W- MED	65.00
1B,2D,3-6,7B,8	107728	COMP STRT MED EXT ASM 16"W L	LEFT- 80D- 16" W- MED	65.00

Pos.	Item Number	Description	Remarks	USD
1B,2E,3-6,7C,8	107729	COMP STRT MED EXT ASM 17"W L	LEFT- 80D- 17" W- MED	65.00
1B,2F,3-6,7C,8	107730	COMP STRT MED EXT ASM 18"W L	LEFT- 80D- 18" W- MED	65.00
1B,2G,3-6,7D,8	107731	COMP STRT MED EXT ASM 19 & 20"W L	LEFT- 80D- 19 & 20" W- MED	65.00
1B,2H,3-6,7D,8	111525	COMP STRT MED EXT ASM 21 & 22"W L	LEFT- 80D- 21 & 22" W- MED	65.00
>>>>		80 DEGREE HANGERS - LONG EXT - RIGHT		
1C,2A,3-6,7A,8	107732	COMP STRT LONG EXT ASM 13"W R	Right- 80D- 13" W- LONG	65.00
1C,2B,3-6,7A,8	107733	COMP STRT LONG EXT ASM 14"W R	Right- 80D- 14" W- LONG	65.00
1C,2C,3-6,7B,8	107734	COMP STRT LONG EXT ASM 15"W R	Right- 80D- 15" W- LONG	65.00
1C,2D,3-6,7B,8	107735	COMP STRT LONG EXT ASM 16"W R	Right- 80D- 16" W- LONG	65.00
1C,2E,3-6,7C,8	107736	COMP STRT LONG EXT ASM 17"W R	Right- 80D- 17" W- LONG	65.00
1C,2F,3-6,7C,8	107737	COMP STRT LONG EXT ASM 18"W R	Right- 80D- 18" W- LONG	65.00
1C,2G,3-6,7D,8	107738	COMP STRT LONG EXT ASM 19 & 20"W R	Right- 80D- 19 & 20" W- LONG	65.00
1C,2H,3-6,7D,8	111526	COMP STRT LONG EXT ASM 21 & 22"W R	Right- 80D- 21 & 22" W- LONG	65.00
>>>>		80 DEGREE HANGERS - LONG EXT - LEFT		
1C,2A,3-6,7A,8	107739	COMP STRT LONG EXT ASM 13"W L	LEFT- 80D- 13" W- LONG	65.00
1C,2B,3-6,7A,8	107740	COMP STRT LONG EXT ASM 14"W L	LEFT- 80D- 14" W- LONG	65.00
1C,2C,3-6,7B,8	107741	COMP STRT LONG EXT ASM 15"W L	LEFT- 80D- 15" W- LONG	65.00
1C,2D,3-6,7B,8	107742	COMP STRT LONG EXT ASM 16"W L	LEFT- 80D- 16" W- LONG	65.00
1C,2E,3-6,7C,8	107743	COMP STRT LONG EXT ASM 17"W L	LEFT- 80D- 17" W- LONG	65.00
1C,2F,3-6,7C,8	107744	COMP STRT LONG EXT ASM 18"W L	LEFT- 80D- 18" W- LONG	65.00
1C,2G,3-6,7D,8	107745	COMP STRT LONG EXT ASM 19 & 20"W L	LEFT- 80D- 19 & 20" W- LONG	65.00
1C,2H,3-6,7D,8	111527	COMP STRT LONG EXT ASM 21 & 22"W L	LEFT- 80D- 21 & 22" W- LONG	65.00
>>>>		60°,70° SHORT EXT - RIGHT		
1D,2A,3-6,7A,8	107746	COMP BNT SHT EXT ASM 13" W R	Right- 70D- 13" W- SHORT	65.00
1D,2B,3-6,7A,8	107747	COMP BNT SHT EXT ASM 14" W R	Right- 70D- 14" W- SHORT	65.00
1D,2C,3-6,7B,8	107748	COMP BNT SHT EXT ASM 15" W R	Right- 70D- 15" W- SHORT	65.00
1D,2D,3-6,7B,8	107749	COMP BNT SHT EXT ASM 16" W R	Right- 70D- 16" W- SHORT	65.00
1D,2E,3-6,7C,8	107750	COMP BNT SHT EXT ASM 17" W R	Right- 70D- 17" W- SHORT	65.00
1D,2F,3-6,7C,8	107751	COMP BNT SHT EXT ASM 18" W R	Right- 70D- 18" W- SHORT	65.00
1D,2G,3-6,7D,8	107752	COMP BNT SHT EXT ASM 19 & 20"W R	Right- 70D- 19 & 20" W- SHORT	65.00
1D,2H,3-6,7D,8	111528	COMP BNT SHT EXT ASM 21 & 22"W R	Right- 70D- 21 & 22" W- SHORT	65.00
>>>>		60°,70° - SHORT EXT - LEFT		
1D,2A,3-6,7A,8	107753	COMP BNT SHT EXT ASM 13" W L	Left- 70D- 13 W- SHORT	65.00
1D,2B,3-6,7A,8	107754	COMP BNT SHT EXT ASM 14" W L	Left- 70D- 14 W- SHORT	65.00
1D,2C,3-6,7B,8	107755	COMP BNT SHT EXT ASM 15" W L	Left- 70D- 15 W- SHORT	65.00
1D,2D,3-6,7B,8	107756	COMP BNT SHT EXT ASM 16" W L	Left- 70D- 16 W- SHORT	65.00
1D,2E,3-6,7C,8	107757	COMP BNT SHT EXT ASM 17" W L	Left- 70D- 17 W- SHORT	65.00
1D,2F,3-6,7C,8	107758	COMP BNT SHT EXT ASM 18" W L	Left- 70D- 18 W- SHORT	65.00
1D,2G,3-6,7D,8	107759	COMP BNT SHT EXT ASM 19 & 20"W L	Left- 70D- 19 & 20" W- SHORT	65.00
1D,2H,3-6,7D,8	111529	COMP BNT SHT EXT ASM 21 & 22"W L	Left- 70D- 21 & 22" W- SHORT	65.00
>>>>		60°,70°,60°&70° HEMI HANGERS - MED EXT - RIGHT		
1E,2A,3-6,7A,8	107760	COMP BNT MED EXT ASM 13" W R	Right- 70D- 13" W- MED	65.00
1E,2B,3-6,7A,8	107761	COMP BNT MED EXT ASM 14" W R	Right- 70D- 14" W- MED	65.00
1E,2C,3-6,7B,8	107762	COMP BNT MED EXT ASM 15" W R	Right- 70D- 15" W- MED	65.00
1E,2D,3-6,7B,8	107763	COMP BNT MED EXT ASM 16" W R	Right- 70D- 16" W- MED	65.00
1E,2E,3-6,7C,8	107764	COMP BNT MED EXT ASM 17" W R	Right- 70D- 17" W- MED	65.00
1E,2F,3-6,7C,8	107765	COMP BNT MED EXT ASM 18" W R	Right- 70D- 18" W- MED	65.00
1E,2G,3-6,7D,8	107766	COMP BNT MED EXT ASM 19 & 20"W R	Right- 70D- 19 & 20" W- MED	65.00
1E,2H,3-6,7D,8	111530	COMP BNT MED EXT ASM 21 & 22"W R	Right- 70D- 21 & 22" W- MED	65.00
>>>>		60°,70°,60°&70° HEMI HANGERS - MED EXT - LEFT		
1E,2A,3-6,7A,8	107767	COMP BNT MED EXT ASM 13" W L	Left- 70D- 13 W- MED	65.00
1E,2B,3-6,7A,8	107768	COMP BNT MED EXT ASM 14" W L	Left- 70D- 14 W- MED	65.00
1E,2C,3-6,7B,8	107769	COMP BNT MED EXT ASM 15" W L	Left- 70D- 15 W- MED	65.00
1E,2D,3-6,7B,8	107770	COMP BNT MED EXT ASM 16" W L	Left- 70D- 16 W- MED	65.00
1E,2E,3-6,7C,8	107771	COMP BNT MED EXT ASM 17" W L	Left- 70D- 17 W- MED	65.00
1E,2F,3-6,7C,8	107772	COMP BNT MED EXT ASM 18" W L	Left- 70D- 18 W- MED	65.00

Pos.	Item Number	Description	Remarks	USD
1E,2G,3-6,7D,8	107773	COMP BNT MED EXT ASM 19 & 20"W L	Left- 70D- 19 & 20" W- MED	65.00
1E,2H,3-6,7D,8	111531	COMP BNT MED EXT ASM 21 & 22"W L	Left- 70D- 21 & 22" W- MED	65.00
>>>>		60°,70°,60°&70° HEMI HANGERS - LONG EXT - RIGHT		
1F,2A,3-6,7A,8	107774	COMP BNT LONG EXT ASM 13" W R	Right- 70D- 13" W- LONG	65.00
1F,2B,3-6,7A,8	107775	COMP BNT LONG EXT ASM 14" W R	Right- 70D- 14" W- LONG	65.00
1F,2C,3-6,7B,8	107776	COMP BNT LONG EXT ASM 15" W R	Right- 70D- 15" W- LONG	65.00
1F,2D,3-6,7B,8	107777	COMP BNT LONG EXT ASM 16" W R	Right- 70D- 16" W- LONG	65.00
1F,2E,3-6,7C,8	107778	COMP BNT LONG EXT ASM 17" W R	Right- 70D- 17" W- LONG	65.00
1F,2F,3-6,7C,8	107779	COMP BNT LONG EXT ASM 18" W R	Right- 70D- 18" W- LONG	65.00
1F,2G,3-6,7D,8	107780	COMP BNT LONG EXT ASM 19 & 20"W R	Right- 70D- 19 & 20" W- LONG	65.00
1F,2H,3-6,7D,8	111532	COMP BNT LONG EXT ASM 21 & 22"W R	Right- 70D- 21 & 22" W- LONG	65.00
>>>>		60°,70°,60°&70° HEMI HANGERS - LONG EXT - LEFT		
1F,2A,3-6,7A,8	107781	COMP BNT LONG EXT ASM 13" W L	Left- 70D- 13 W- LONG	65.00
1F,2B,3-6,7A,8	107782	COMP BNT LONG EXT ASM 14" W L	Left- 70D- 14 W- LONG	65.00
1F,2C,3-6,7B,8	107783	COMP BNT LONG EXT ASM 15" W L	Left- 70D- 15 W- LONG	65.00
1F,2D,3-6,7B,8	107784	COMP BNT LONG EXT ASM 16" W L	Left- 70D- 16 W- LONG	65.00
1F,2E,3-6,7C,8	107785	COMP BNT LONG EXT ASM 17" W L	Left- 70D- 17 W- LONG	65.00
1F,2F,3-6,7C,8	107786	COMP BNT LONG EXT ASM 18" W L	Left- 70D- 18 W- LONG	65.00
1F,2G,3-6,7D,8	107787	COMP BNT LONG EXT ASM 19 & 20"W L	Left- 70D- 19 & 20" W- LONG	65.00
1F,2H,3-6,7D,8	111533	COMP BNT LONG EXT ASM 21 & 22"W L	Left- 70D- 21 & 22" W- LONG	65.00
>>>>		60°&70° HEMI HANGER SHORT EXT- RIGHT		
1G,2A,3-6,7A,8	129724	CMP FT BNT KIT SHT HNGR 13W R	Right-Hemi-13W	
1G,2B,3-6,7A,8	129726	CMP FT BNT KIT SHT HNGR 14W R	Right-Hemi-14W	
1G,2C,3-6,7B,8	129728	CMP FT BNT KIT SHT HNGR 15W R	Right-Hemi-15W	
1G,2D,3-6,7B,8	129730	CMP FT BNT KIT SHT HNGR 16W R	Right-Hemi-16W	
1G,2E,3-6,7C,8	129732	CMP FT BNT KIT SHT HNGR 17W R	Right-Hemi-17W	
1G,2F,3-6,7C,8	129734	CMP FT BNT KIT SHT HNGR 18W R	Right-Hemi-18W	
1G,2G,3-6,7D,8	129736	CMP FT BNT KIT SHT HNGR 19-20W R	Right-Hemi-19-20W	
1G,2H,3-6,7D,8	129738	CMP FT BNT KIT SHT HNGR 21-22W R	Right-Hemi-21-22W	
>>>>		60°&70° HEMI HANGER SHORT EXT- LEFT		
1G,2A,3-6,7A,8	129725	CMP FT BNT KIT SHT HNGR 13W L	Left-Hemi-13W	
1G,2B,3-6,7A,8	129727	CMP FT BNT KIT SHT HNGR 14W L	Left-Hemi-14W	
1G,2C,3-6,7B,8	129729	CMP FT BNT KIT SHT HNGR 15W L	Left-Hemi-15W	
1G,2D,3-6,7B,8	129731	CMP FT BNT KIT SHT HNGR 16W L	Left-Hemi-16W	
1G,2E,3-6,7C,8	129733	CMP FT BNT KIT SHT HNGR 17W L	Left-Hemi-17W	
1G,2F,3-6,7C,8	129735	CMP FT BNT KIT SHT HNGR 18W L	Left-Hemi-18W	
1G,2G,3-6,7D,8	129737	CMP FT BNT KIT SHT HNGR 19-20W L	Left-Hemi-19-20W	
1G,2H,3-6,7D,8	129739	CMP FT BNT KIT SHT HNGR 21-22W L	Left-Hemi-21-22W	


NOTE: Footplate Assemblies are set to Footrest width. Please see "Footplate ordering Guides" in the information panel for proper conversion.

Note: If using Short Ext Tube & Hemi Hanger with Composite Angle Adjustable Footplate use 1D (straight) .

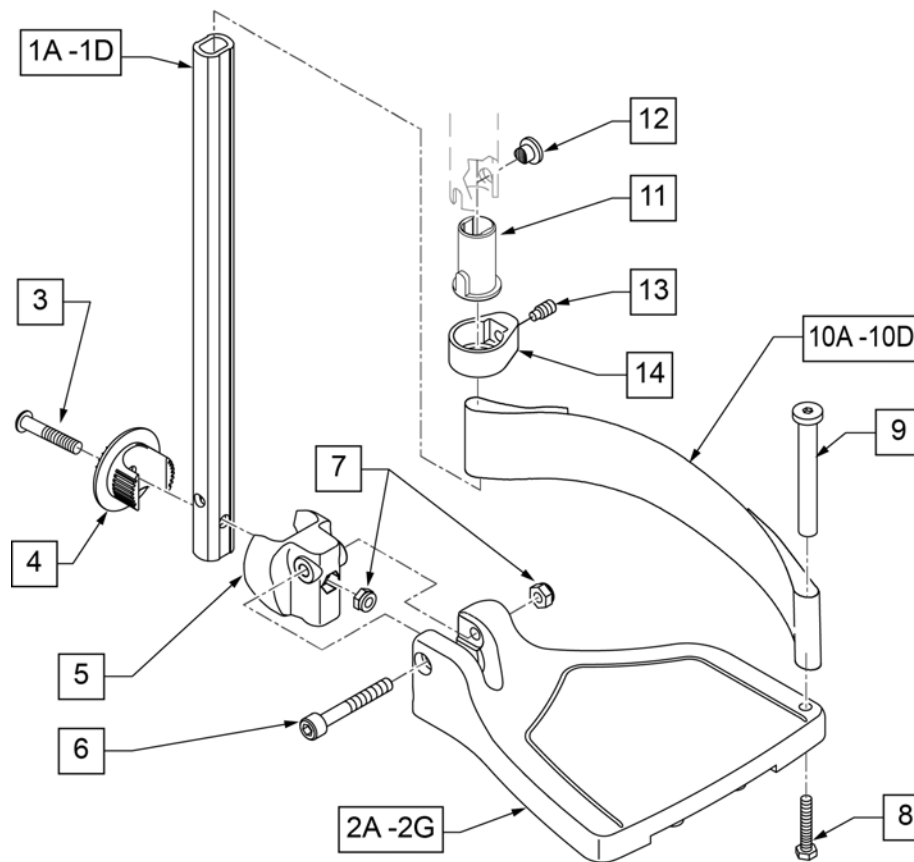

Composite Angle Adjustable "CAA" Footplate w/ Ext & HL for Swing I/O Hangers (Order 1 each)				
Ext. Tube	Short Style Hangers	Standard Style hanger (Straight Extension)		
Width	Short	Short	Medium	Long
15" Right	129710	107816	107830	107844
15" Left	129711	107823	107837	107851
16" Right	129712	107817	107831	107845
16" Left	129713	107824	107838	107852
17" Right	129714	107818	107832	107846
17" Left	129715	107825	107839	107853
18" Right	129716	107819	107833	107847
18" Left	129717	107826	107840	107854
19" Right	129718	107820	107834	107848
19" Left	129719	107827	107841	107855
20" Right	129720	107821	107835	107849
20" Left	129721	107828	107842	107856
21"-22" Rt	129722	107822	107836	107850
21"-22" Lt	129723	107829	107843	107857

FOOTPLATE MEASURING GUIDE

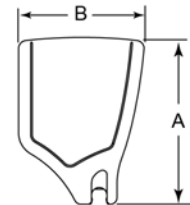

PART NO	DESCRIPTION	DIM -A-	DIM -B-
063216-006	FOOTPLATE COMPOSITE 157 R	6 1/8"	6"
063216-012	FOOTPLATE COMPOSITE 157 L	6 1/8"	6"
063216-015	FOOTPLATE COMPOSITE 170 R	6 5/8"	6"
063216-018	FOOTPLATE COMPOSITE 170 L	6 5/8"	6"
063216-005	FOOTPLATE COMPOSITE 182 R	7 1/8"	6"
063216-011	FOOTPLATE COMPOSITE 182 L	7 1/8"	6"
063216-014	FOOTPLATE COMPOSITE 195 R	7 5/8"	6"
063216-017	FOOTPLATE COMPOSITE 195 L	7 5/8"	6"
063216-004	FOOTPLATE COMPOSITE 207 R	8 1/8"	6"
063216-010	FOOTPLATE COMPOSITE 207 L	8 1/8"	6"
063216-013	FOOTPLATE COMPOSITE 220 R	8 5/8"	6 1/4"
063216-016	FOOTPLATE COMPOSITE 220 L	8 5/8"	6 1/4"
063216-003	FOOTPLATE COMPOSITE 232 R	9 1/8"	6 1/4"
063216-009	FOOTPLATE COMPOSITE 232 L	9 1/8"	6 1/4"
063216-002	FOOTPLATE COMPOSITE 257 R	10 1/8"	6 1/4"
063216-008	FOOTPLATE COMPOSITE 257 L	10 1/8"	6 1/4"
063216-001	FOOTPLATE COMPOSITE 282 R	11 1/8"	6 1/4"
063216-007	FOOTPLATE COMPOSITE 282 L	11 1/8"	6 1/4"

Pos.	Item Number	Description	Remarks	USD
1A,6,7	120660	SHT STRT EXT COMP HDWR		20.00
1B,6,7	120661	MED STRT EXT COMP HDWR		20.00
1C,6,7	120662	LNG STRT EXT COMP HDWR		20.00
1D	128079	TUBE FTRST EXT STRGHT SHT HEMI		
2A	063216-006	COMPOSITE FOOTPLATE RIGHT	RIGHT 15" Footrest Width	20.00
2A	063216-012	COMPOSITE FOOTPLATE LEFT	LEFT 15" Footrest Width	20.00
2B	063216-015	COMPOSITE FOOTPLATE RIGHT	RIGHT 16" Footrest Width	20.00
2B	063216-018	COMPOSITE FOOTPLATE LEFT	LEFT 16" Footrest Width	20.00
2C	063216-005	COMPOSITE FOOTPLATE RIGHT	RIGHT 17" Footrest Width	20.00
2C	063216-011	COMPOSITE FOOTPLATE LEFT	LEFT 17" Footrest Width	20.00

Pos.	Item Number	Description	Remarks	USD
2D	063216-014	COMPOSITE FOOTPLATE RIGHT	RIGHT 18" Footrest Width	20.00
2D	063216-017	COMPOSITE FOOTPLATE LEFT	LEFT 18" Footrest Width	20.00
2E	063216-004	COMPOSITE FOOTPLATE RIGHT	RIGHT 19" Footrest Width	20.00
2E	063216-010	COMPOSITE FOOTPLATE LEFT	LEFT 19" Footrest Width	20.00
2F	063216-013	COMPOSITE FOOTPLATE RIGHT	RIGHT 20" Footrest Width	20.00
2F	063216-016	COMPOSITE FOOTPLATE LEFT	LEFT 20" Footrest Width	20.00
2G	063216-003	COMPOSITE FOOTPLATE RIGHT	RIGHT 21" & 22" Footrest Width	20.00
2G	063216-009	COMPOSITE FOOTPLATE LEFT	LEFT 21" & 22" Footrest Width	20.00
3	108233	BHCS M6 X 1 X 40		10.00
4	054242	ANGLE COVER		5.00
5	054241	ANGLE SEGMENT		5.00
6	103175	SHCS M6 X 1 X 40 INORG		5.00
7	104381	NUT HEX NYLOCK M6X1.0		5.00
8	102268-001	HHCS M6X1X25	Inserted in bottom of footplate.	10.00
9	104328	HEEL LOOP PEG		5.00
10A	104380-001	ADJUSTABLE FOOT REST STRAP	15" Footrest Width	7.50
10B	104380-002	ADJUSTABLE FOOT REST STRAP	16" Footrest Width	7.50
10C	104380-003	ADJUSTABLE FOOT REST STRAP	17-18" Footrest Width	7.50
10D	104380-004	ADJUSTABLE FOOT REST STRAP	19-22" Footrest Width	7.50
11	104160	BUSHING FOOTREST		5.00
11-14	107485	FOOTREST CLAMP ASSM		30.00
>>>>		60,70,80 HANGERS - SHORT EXT - RIGHT		
1A,2A,3-10A	107816	CAA STRT SHT EXT ASM 15W R	Right- 15 W- SHORT	70.00
1A,2B,3-10B	107817	CAA STRT SHT EXT ASM 16W R	Right- 16 W- SHORT	70.00
1A,2C,3-10B	107818	CAA STRT SHT EXT ASM 17W R	Right- 17 W- SHORT	70.00
1A,2D,3-10C	107819	CAA STRT SHT EXT ASM 18W R	Right- 18 W- SHORT	70.00
1A,2E,3-10C	107820	CAA STRT SHT EXT ASM 19W R	Right- 19 W- SHORT	70.00
1A,2F,3-10D	107821	CAA STRT SHT EXT ASM 20W R	Right- 20 W- SHORT	70.00
1A,2G,3-10D	107822	CAA STRT SHT EXT ASM 21&22W R	Right- 21&22 W- SHORT	70.00
>>>>		60,70,80 HANGERS - SHORT EXT - LEFT		
1A,2A,3-10A	107823	CAA STRT SHT EXT ASM 15W L	Left- 15 W- SHORT	70.00
1A,2B,3-10B	107824	CAA STRT SHT EXT ASM 16W L	Left- 16 W- SHORT	70.00
1A,2C,3-10B	107825	CAA STRT SHT EXT ASM 17W L	Left- 17 W- SHORT	70.00
1A,2D,3-10C	107826	CAA STRT SHT EXT ASM 18W L	Left- 18 W- SHORT	70.00
1A,2E,3-10C	107827	CAA STRT SHT EXT ASM 19W L	Left- 19 W- SHORT	70.00
1A,2F,3-10D	107828	CAA STRT SHT EXT ASM 20W L	Left- 20 W- SHORT	70.00
1A,2G,3-10D	107829	CAA STRT SHT EXT ASM 21&22W L	Left- 21&22 W- SHORT	70.00
>>>>		60,70,80& 60,70 HEMI HANGER- RIGHT		
1B,2A,3-10A	107830	CAA STRT MED EXT ASM 15W R	Right- 15 W- MED	70.00
1B,2B,3-10B	107831	CAA STRT MED EXT ASM 16W R	Right- 16 W- MED	70.00
1B,2C,3-10B	107832	CAA STRT MED EXT ASM 17W R	Right- 17 W- MED	70.00
1B,2D,3-10C	107833	CAA STRT MED EXT ASM 18W R	Right- 18 W- MED	70.00
1B,2E,3-10C	107834	CAA STRT MED EXT ASM 19W R	Right- 19 W- MED	70.00
1B,2F,3-10D	107835	CAA STRT MED EXT ASM 20W R	Right- 20 W- MED	70.00
1B,2G,3-10D	107836	CAA STRT MED EXT ASM 21&22W R	Right- 21&22 W- MED	70.00
>>>>		60,70,80& 60,70 HEMI HANGER- LEFT		
1B,2A,3-10A	107837	CAA STRT MED EXT ASM 15W L	Left- 15 W- MED	70.00
1B,2B,3-10B	107838	CAA STRT MED EXT ASM 16W L	Left- 16 W- MED	70.00
1B,2C,3-10B	107839	CAA STRT MED EXT ASM 17W L	Left- 17 W- MED	70.00
1B,2D,3-10C	107840	CAA STRT MED EXT ASM 18W L	Left- 18 W- MED	70.00
1B,2E,3-10C	107841	CAA STRT MED EXT ASM 19W L	Left- 19 W- MED	70.00
1B,2F,3-10D	107842	CAA STRT MED EXT ASM 20W L	Left- 20 W- MED	70.00
1B,2G,3-10D	107843	CAA STRT MED EXT ASM 21&22W L	Left- 21&22 W- MED	70.00
>>>>		60,70,80& 60,70 HEMI HANGER- RIGHT		
1C,2A,3-10A	107844	CAA STRT LNG EXT ASM 15W R	Right- 15 W- LONG	70.00
1C,2B,3-10B	107845	CAA STRT LNG EXT ASM 16W R	Right- 16 W- LONG	70.00


Pos.	Item Number	Description	Remarks	USD
1C,2C,3-10B	107846	CAA STRT LNG EXT ASM 17W R	Right- 17 W- LONG	70.00
1C,2D,3-10C	107847	CAA STRT LNG EXT ASM 18W R	Right- 18 W- LONG	70.00
1C,2E,3-10C	107848	CAA STRT LNG EXT ASM 19W R	Right- 19 W- LONG	70.00
1C,2F,3-10D	107849	CAA STRT LNG EXT ASM 20W R	Right- 20 W- LONG	70.00
1C,2G,3-10D	107850	CAA STRT LNG EXT ASM 21&22W R	Right- 21&22 W- LONG	70.00
>>>>		60,70,80& 60,70 HEMI HANGER- LEFT		
1C,2A,3-10A	107851	CAA STRT LNG EXT ASM 15W L	Left- 15 W- LONG	70.00
1C,2B,3-10B	107852	CAA STRT LNG EXT ASM 16W L	Left- 16 W- LONG	70.00
1C,2C,3-10B	107853	CAA STRT LNG EXT ASM 17W L	Left- 17 W- LONG	70.00
1C,2D,3-10C	107854	CAA STRT LNG EXT ASM 18W L	Left- 18 W- LONG	70.00
1C,2E,3-10C	107855	CAA STRT LNG EXT ASM 19W L	Left- 19 W- LONG	70.00
1C,2F,3-10D	107856	CAA STRT LNG EXT ASM 20W L	Left- 20 W- LONG	70.00
1C,2G,3-10D	107857	CAA STRT LNG EXT ASM 21&22W L	Left- 21&22 W- LONG	70.00
>>>>		60&70 HEMI HANGERS- SHORT EXT- RIGHT		
1D,2A,3-10A	129710	CAA FOOT KIT SHT HNGR 15W R	Right-Hemi-15W	
1D,2B,3-10B	129712	CAA FOOT KIT SHT HNGR 16W R	Right-Hemi-16W	
1D,2C,3-10B	129714	CAA FOOT KIT SHT HNGR 17W R	Right-Hemi-17W	
1D,2D,3-10C	129716	CAA FOOT KIT SHT HNGR 18W R	Right-Hemi-18W	
1D,2E,3-10C	129718	CAA FOOT KIT SHT HNGR 19W R	Right-Hemi-19W	
1D,2F,3-10D	129720	CAA FOOT KIT SHT HNGR 20W R	Right-Hemi-20W	
1D,2G,3-10D	129722	CAA FOOT KIT SHT HNGR 22W R	Right-Hemi-21&22W	
>>>>		60&70 HEMI HANGERS- SHORT EXT - LEFT		
1D,2A,3-10A	129711	CAA FOOT KIT SHT HNGR 15W L	Left-Hemi-15W	
1D,2B,3-10B	129713	CAA FOOT KIT SHT HNGR 16W L	Left-Hemi-16W	
1D,2C,3-10B	129715	CAA FOOT KIT SHT HNGR 17W L	Left-Hemi-17W	
1D,2D,3-10C	129717	CAA FOOT KIT SHT HNGR 18W L	Left-Hemi-18W	
1D,2E,3-10C	129719	CAA FOOT KIT SHT HNGR 19W L	Left-Hemi-19W	
1D,2F,3-10D	129721	CAA FOOT KIT SHT HNGR 20W L	Left-Hemi-20W	
1D,2G,3-10D	129723	CAA FOOT KIT SHT HNGR 22W L	Left-Hemi-21&22W	

NOTE: Footplate Assemblies are set to Footrest width. Please see "Footplate ordering Guides" in the information panel for proper conversion.


Note: If using Short Ext Tube & Hemi Hanger with Adult Angle Adjustable Footplate use 1D (straight) .

Adult Angle Adjustable "AAA" Footplates w/ Ext & HL for Swing I/O Hangers (Order 1 each)						
Ext.	Short Style Hangers		Standard Style hanger			
Tube	(Short Straight Extension)		(Straight Extension)			
Width			Short	Med	Long	
12-13" Rt	129756		111534	111536	111538	
12-13" Lt	129757		111535	111537	111539	
14"-15" Rt	129758		107928	107942	107956	
14"-15" Lt	129759		107935	107949	107963	
16"-17" Rt	129760		107930	107944	107958	
16"-17" Lt	129761		107937	107951	107965	
18"-22" Rt	129762		107932	107946	107960	
18"-22" Lt	129763		107939	107953	107968	

MEASURING GUIDE

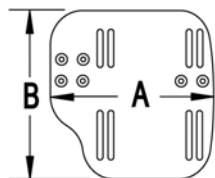



CHAIR WIDTH	"A"	"B"
10" - 11"	3 1/2"	7 1/4"
12" - 13"	4 1/2"	7 3/4"
14" - 15"	5 1/2"	7 3/4"
16" - 17"	6 1/2"	7 3/4"
18" - 20"	7 1/2"	7 3/4"

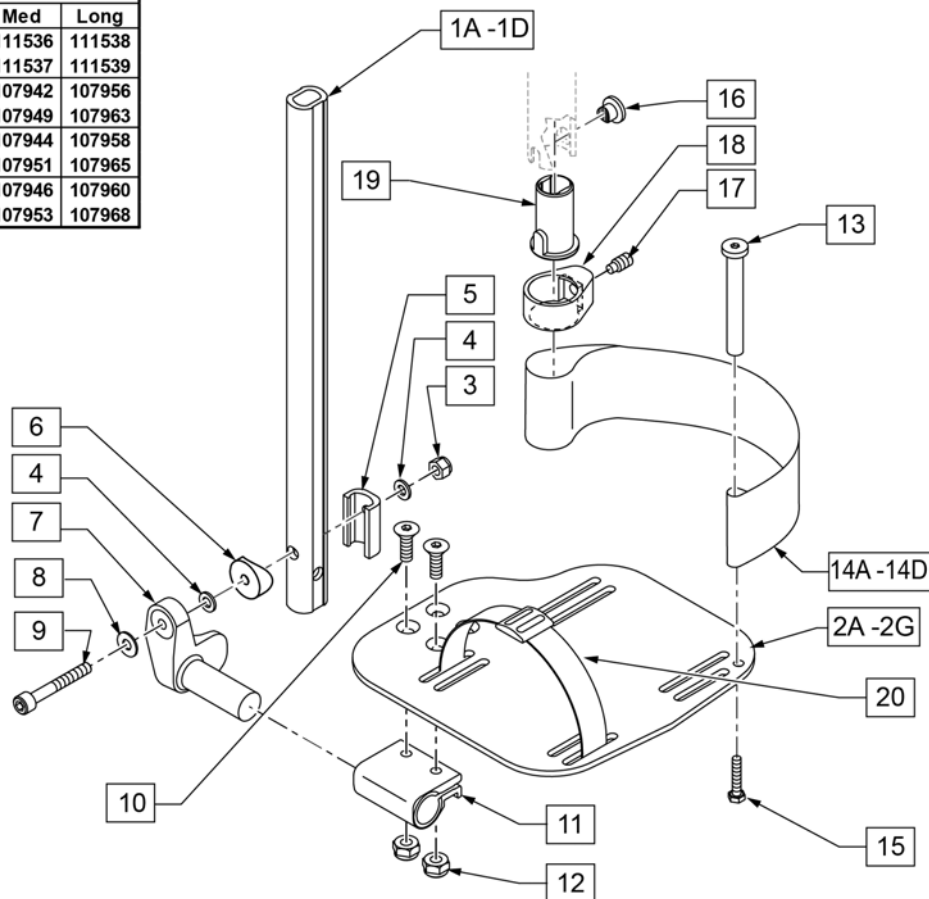


Pos.	Item Number	Description	Remarks	USD
1A,3-5,8,9	120670	SHT STRT EXT AAA HDWR		30.00
1B,3-5,8,9	120671	MED STRT EXT AAA HDWR		30.00
1C,3-5,8,9	120672	LNG STRT EXT AAA HDWR		30.00
1D	128079	TUBE FTRST EXT STRGHT SHT HEMI		
2A	500656	ADULT ANG ADJ F/PLT 10/11" R	RIGHT 10-11" Footrest Width	58.14
2A	500657	ADULT ANG ADJ F/PLT 10/11" L	LEFT 10-11" Footrest Width	58.14
2B	500658	ADULT ANG ADJ F/PLT 12/13" R	RIGHT 12-13" Footrest Width	60.68
2B	500659	ADULT ANG ADJ F/PLT 12/13" L	LEFT 12-13" Footrest Width	60.68
2C	500660	ADLT ANGL ADJ F/PLT 14/15	RIGHT 14-15" Footrest Width	60.68
2C	500661	ADLT ANGL ADJ F/PLT 14/15	LEFT 14-15" Footrest Width	60.68

Pos.	Item Number	Description	Remarks	USD
2D	500662	ADLT ANGL ADJ F/PLT 16/17	RIGHT 16-17" Footrest Width	60.68
2D	500663	ADLT ANGL ADJ F/PLT 16/17	LEFT 16-17" Footrest Width	60.68
2E	500664	ADLT ANGL ADJ F/PLT 18-20	RIGHT 18-22" Footrest Width	60.68
2E	500665	ADLT ANGL ADJ F/PLT 18-20	LEFT 18-22" Footrest Width	60.68
3	104381	NUT HEX NYLOCK M6X1.0		5.00
4	106944	WASHER FLAT THIN AN 1/4 ROHS		1.00
5	107199	HANGER EXTENSION SADDLE		5.00
7	541002	FTRST MOUNTING BRKT-3/8 RAD R	RIGHT	18.21
7	541003	FTRST MOUNTING BRKT-3/8 RAD L	LEFT	18.21
8	106945	WASHER FLAT SAE 1/4 ROHS		1.00
9	104399	SHCS M6X1X45 12 MM PART THD		5.00
10	106978	FHCS 1/4-28 X 7/8 ROHS		5.00
11	550350	3/4" CLMP ADJ FT PLT		11.04
12	106935	NUT NYLOCK THIN 1/4-28 ROHS		5.00
13	104328	POST HEEL LOOP		5.00
14A	104380-001	ADJUSTABLE FOOT REST STRAP	12 - 14" WIDE	7.50
14B	104380-002	ADJUSTABLE FOOT REST STRAP	15 - 16" WIDE	7.50
14C	104380-003	ADJUSTABLE FOOT REST STRAP	17 - 18" WIDE	7.50
14D	104380-004	ADJUSTABLE FOOT REST STRAP	19 - 20" WIDE	7.50
15	102268-001	HHCS M6 X 1 X 25		10.00
3-6,8,9	120659	EXT MNT HARDWARE 1 SIDE		20.00
16-19	107485	FOOTREST CLAMP ASSM		30.00
20	740260	TOE-ANKLE STRAP ADJ ADULT FOOTPLATE		20.40
>>>>		60,70,80 STD HANGERS - RIGHT		
1A,2B,3-14A,15	111534	AAA STRT SHT EXT ASM 12-13" W R	Right- 12-13" W- SHORT	100.00
1A,2C,3-14B,15	107928	AAA STRT SHT EXT ASM 14-15" W R	Right- 14-15" W- SHORT	100.00
1A,2D,3-14C,15	107930	AAA STRT SHT EXT ASM 16-17" W R	Right- 16-17" W- SHORT	100.00
1A,2E,3-14D,15	107932	AAA STRT SHT EXT ASM 18-22" W R	Right- 18-22" W- SHORT	100.00
>>>>		60,70,80 STD HANGERS - LEFT		
1A,2B,3-14A,15	111535	AAA STRT SHT EXT ASM 12-13" W L	Left- 12-13" W- SHORT	100.00
1A,2C,3-14B,15	107935	AAA STRT SHT EXT ASM 14" W L	Left- 14-15" W- SHORT	100.00
1A,2D,3-14C,15	107937	AAA STRT SHT EXT ASM 16" W L	Left- 16-17" W- SHORT	100.00
1A,2E,3-14D,15	107939	AAA STRT SHT EXT ASM 18" W L	Left- 18-22" W- SHORT	100.00
>>>>		60,70,80 & 60,70 HEMI HANGERS - RIGHT		
1B,2B,3-14A,15	111536	AAA STRT MED EXT ASM 12-13" W R	Right- 12-13" W- MED	100.00
1B,2C,3-14B,15	107942	AAA STRT MED EXT ASM 14-15" W R	Right- 14-15" W- MED	100.00
1B,2D,3-14C,15	107944	AAA STRT MED EXT ASM 16-17" W R	Right- 16-17" W- MED	100.00
1B,2E,3-14D,15	107946	AAA STRT MED EXT ASM 18-22" W R	Right- 18-22" W- MED	100.00
>>>>		60,70,80 & 60,70 HEMI HANGERS - LEFT		
1B,2B,3-14A,15	111537	AAA STRT MED EXT ASM 12-13" W L	Left- 12-13" W- MED	100.00
1B,2C,3-14B,15	107949	AAA STRT MED EXT ASM 14" W L	Left- 14-15" W- MED	100.00
1B,2D,3-14C,15	107951	AAA STRT MED EXT ASM 16" W L	Left- 16-17" W- MED	100.00
1B,2E,3-14D,15	107953	AAA STRT MED EXT ASM 18" W L	Left- 18-22" W- MED	100.00
>>>>		60,70,80 & 60,70 HEMI HANGERS - RIGHT		
1C,2B,3-14A,15	111538	AAA STRT LNG EXT ASM 12-13" W R	Right- 12-13" W- LONG	100.00
1C,2C,3-14B,15	107956	AAA STRT LNG EXT ASM 14 -15" W R	Right- 14-15" W- LONG	100.00
1C,2D,3-14C,15	107958	AAA STRT LNG EXT ASM 16 -17" W R	Right- 16-17" W- LONG	100.00
1C,2E,3-14D,15	107960	AAA STRT LNG EXT ASM 18 -22" W R	Right- 18-22" W- LONG	100.00
>>>>		60,70,80 & 60,70 HEMI HANGERS - LEFT		
1C,2B,3-14A,15	111539	AAA STRT LNG EXT ASM 12-13" W L	Left- 12-13" W- LONG	100.00
1C,2C,3-14B,15	107963	AAA STRT LNG EXT ASM 14W L	Left- 14-15" W- LONG	100.00
1C,2D,3-14C,15	107965	AAA STRT LNG EXT ASM 16W L	Left- 16-17" W- LONG	100.00
1C,2E,3-14D,15	107968	AAA STRT LNG EXT ASM 18W L	Left- 18-22" W- LONG	100.00
>>>>		60&70 HEMI HANGERS - SHORT EXT - RIGHT		
1D,2B,3-14A,15	129756	AAA FOOT KIT SHT HNGR 12W R	Right- 12-13" W- SHORT	
1D,2C,3-14B,15	129758	AAA FOOT KIT SHT HNGR 14W R	Right- 14-15" W- SHORT	

Pos.	Item Number	Description	Remarks	USD
1D,2D,3-14C,15	129760	AAA FOOT KIT SHT HNGR 16W R	Right- 16-17" W- SHORT	
1D,2E,3-14D,15	129762	AAA FOOT KIT SHT HNGR 18W R	Right- 18-22" W- SHORT	
>>>>		60&70 HEMI HANGERS - SHORT EXT -LEFT		
1D,2B,3-14A,15	129756	AAA FOOT KIT SHT HNGR 12W L	Left- 12-13" W-SHORT	
1D,2C,3-14B,15	129758	AAA FOOT KIT SHT HNGR 14W L	Left- 14-15" W- SHORT	
1D,2D,3-14C,15	129760	AAA FOOT KIT SHT HNGR 16W L	Left- 16-17" W- SHORT	
1D,2E,3-14D,15	129762	AAA FOOT KIT SHT HNGR 18W L	Left- 18-22" W- SHORT	

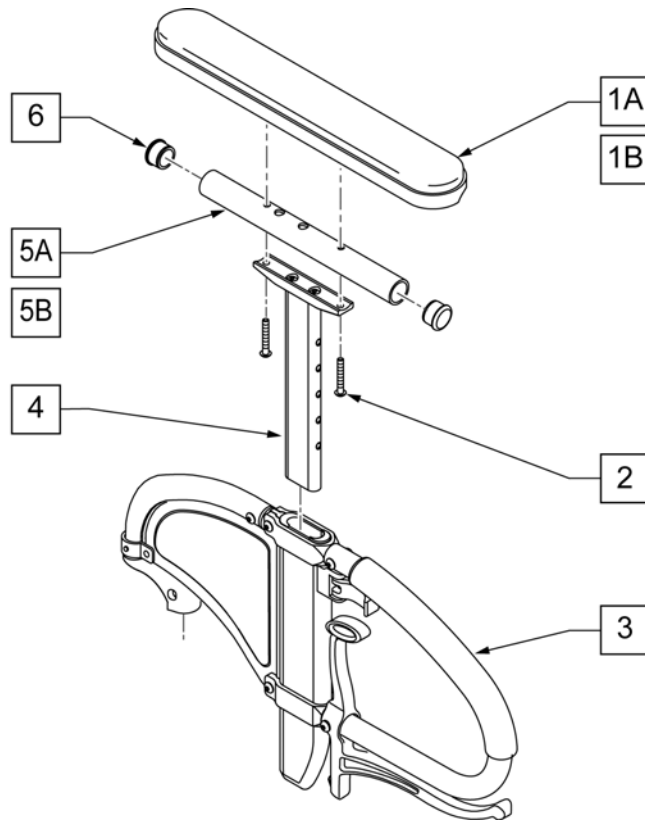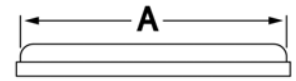
**MEASURING GUIDE**

	DESK	FULL
A	10"	14"

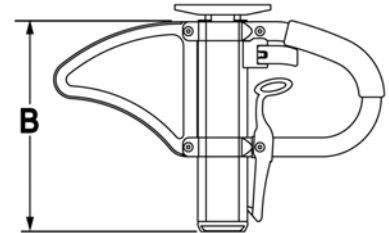

	STD
B	9.75"

Pos.	Item Number	Description	Remarks	USD
1A	561024	ARMREST PAD 10" DESK	CLASSIC DESK	22.50
1B	561031	ARMREST PAD 14" FULL	CLASSIC FULL	22.50
2	107024	10-32 X 1 1/2 BH C/S BLK		2.00
3,4	109405	SPHA ARMREST ASSM RIGHT	RIGHT	100.00
3,4	109400	SPHA ARMREST ASSM LEFT	LEFT	100.00
4	116313	SPHA ARM PAD MOUNT		25.00
5A	521812	HT ADJ PD TUBE 10IN DESK POWE		30.00
5B	521813	HT ADJ PD TUBE 14IN FULL POWE		30.00
6	560735	7/8" ROUND TUBE PLUG		2.00
1A,2,4,5A,6	115221	SPHA UPPER CLASSIC DESK PAD	Classic-Desk	25.00
1B,2,4,5B,6	115222	SPHA UPPER CLASSIC FULL PAD	Classic-Full	25.00
1A-5A,6	117834	SPHA FLP BCK LITE ARM DESK R	RIGHT	115.00
1A-5A,6	117835	SPHA FLP BCK LITE ARM DESK L	LEFT	115.00
1B-5B,6	117836	SPHA FLP BCK LITE ARM FULL R	RIGHT	130.00
1B-5B,6	117837	SPHA FLP BCK LITE ARM FULL L	LEFT	130.00

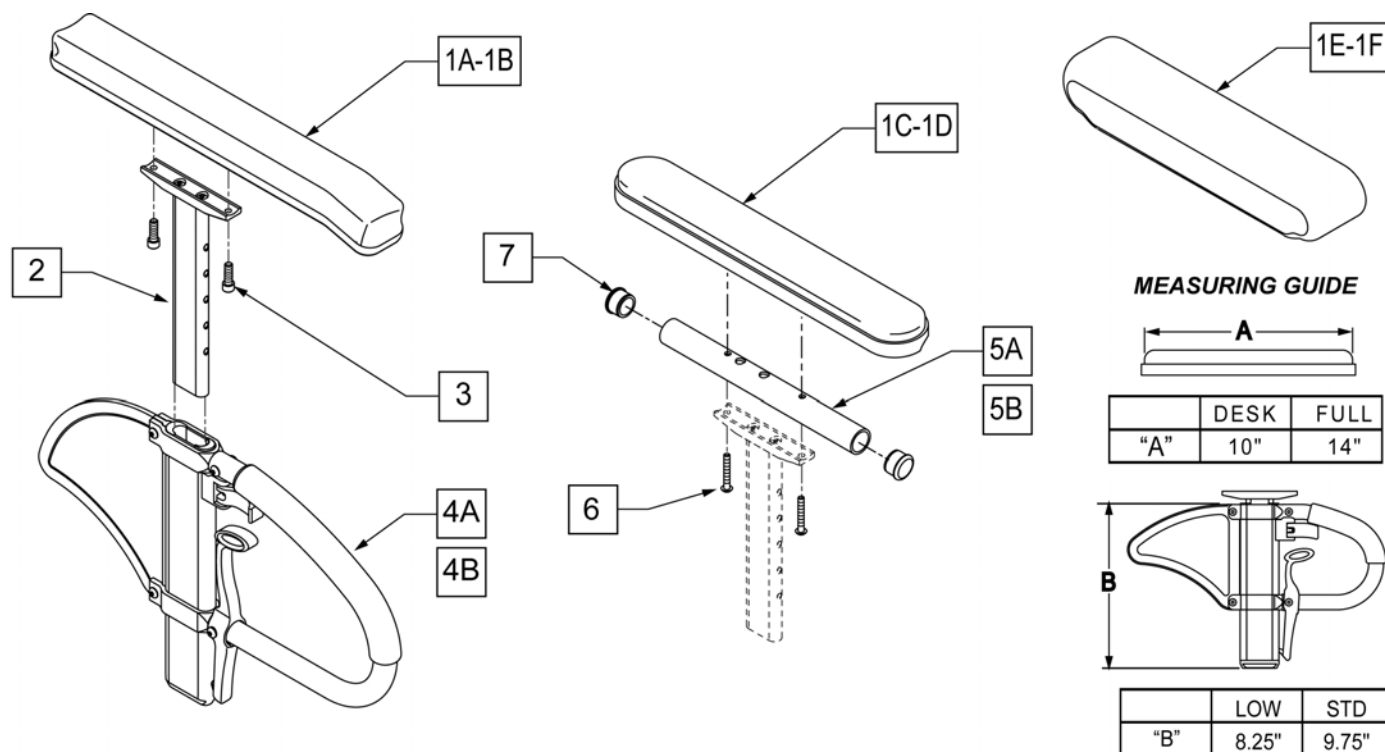

Pos.	Item Number	Description	Remarks	USD
1A	104568-001	SPHA PAD DESK	CONTOUR DESK	22.50
1B	104568-002	SPHA PAD FULL	CONTOUR FULL	22.50
1C	561024	ARMREST PAD 10" DESK	CLASSIC DESK	22.50
1D	561031	ARMREST PAD 14" FULL	CLASSIC FULL	22.50
1E	561280	ARMREST PAD 10" LEFT W/F	Leftt-Waterfall Desk	22.50
1E	561281	ARMREST PAD 10" RIGHT W/F	Rightt-Waterfall Desk	22.50
1F	561284	ARMREST PAD 14" LEFT W/F	Leftt-Waterfall Full	22.50
1F	561285	ARMREST PAD 14" RIGHT W/F	Rightt-Waterfall Full	22.50
2	116312	SNGL POST UPPR ASSM LOW	LOW/SMALL	25.00
2	116313	SNGL POST UPPR ASSM STD	STANDARD	25.00
3	106956	SHCS 10-32 X 3/4 ROHS		5.00
2,4A	104509-001	SPHA ARMREST ASSY STD	STANDARD	75.00
2,4B	104509-002	SPHA ARMREST ASSY LOW	LOW/SMALL	75.00
5A	521812	HT ADJ PD TUBE 10IN DESK POWE	Desk	30.00
5B	521813	HT ADJ PD TUBE 14IN FULL POWE	Full	30.00
6	107024	BHCS 10-32 X 1 1/2 ROHS		2.00
7	560735	7/8" ROUND TUBE PLUG		2.00
1C-4A-5A,6,7	108599	SPHA STD CLASIC DESK PAD R	Right	110.00
1C-4A-5A,6,7	108603	SPHA STD CLASIC DESK PAD L	Left	110.00
1D-4A-5B-7	108601	SPHA STD CLASIC FULL PAD R	Right	110.00
1D-4A-5B-7	108605	SPHA STD CLASIC FULL PAD L	Left	110.00
1A-4A	108595	SPHA STD CNTUR DESK PAD		110.00
1B-4A	108597	SPHA STD CNTUR FULL PAD		110.00
1C-4B-5A,6,7	108600	SPHA LOW CLASIC DESK PAD R	Right	110.00
1C-4B-5A,6,7	108604	SPHA LOW CLASIC DESK PAD L	Left	110.00
1D-4B-5B-7	108602	SPHA LOW CLASIC FULL PAD R	Right	110.00
1D-4B-5B-7	108606	SPHA LOW CLASIC FULL PAD L	Left	110.00
1A-4B	108596	SPHA LOW CNTUR DESK PAD		110.00

Pos.	Item Number	Description	Remarks	USD
1B-4B	108598	SPHA LOW CNTUR FULL PAD		110.00


NOTE: Effective on GTi chairs after S/N T4-505800 & T4A-505900.

NOTE: Effective on GT chairs after S/N R3-012700 & R3A-012762.

NOTE: Armrest heights that can be achieved are; 7, 8, & 9 with the RECEIVER IN THE DOWN position, and 10, 11, & 12 with RECEIVER IN THE UP position.

NOTE: GT,GTi & Q2 Lite chair uses Low Armrest only.

Part Changes, please see notes/remarks for important information

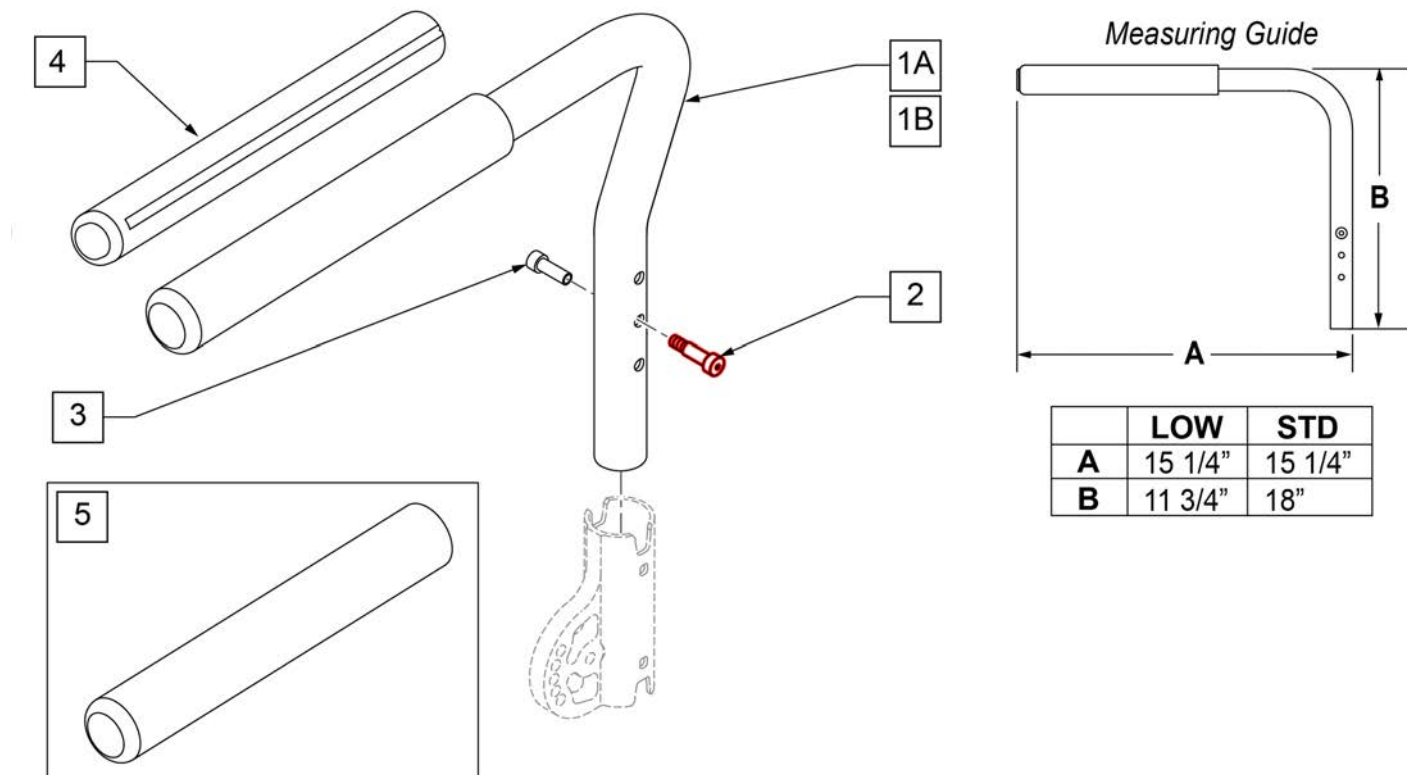

Pos.	Item Number	Description	Remarks	USD
1A-3	111153	SWING-AWAY ARMREST LOW W/ BOLT R	RIGHT-LOW	90.00
1A-3	111154	SWING-AWAY ARMREST LOW W/ BOLT L	LEFT -LOW	90.00
1B-3	111155	SWING-AWAY ARMREST STD W/ BOLT R	RIGHT-STD	90.00
1B-3	111156	SWING-AWAY ARMREST STD W/ BOLT L	LEFT -STD	90.00
2,3	108118	SEXBOLTS& ARMREST KIT		90.00
4	108315	GEL PAD S/ ARM	PAIR	105.00
5	560640	9X.93X.155 FOAM GRIP	Single replacement grip	4.00

NOTE: USED ON QUICKIE 2 CHAIRS W/ S/N PREFIX Q2V.


NOTE: USED ON QUICKIE 2 LITE CHAIRS.

NOTE: USED ON ZIPPIE TS CHAIRS W/ S/N PREFIX Z1A & Z1SE.


NOTE: USED ON ZIPPIE GS CHAIRS W/ S/N PREFIX ZGS & ZGSE.

NOTE: USED ON ZIPPIE 2 CHAIRS W/ S/N PREFIX Z2A.

NOTE: CLAMP MOUNTS TO 1" TUBING.

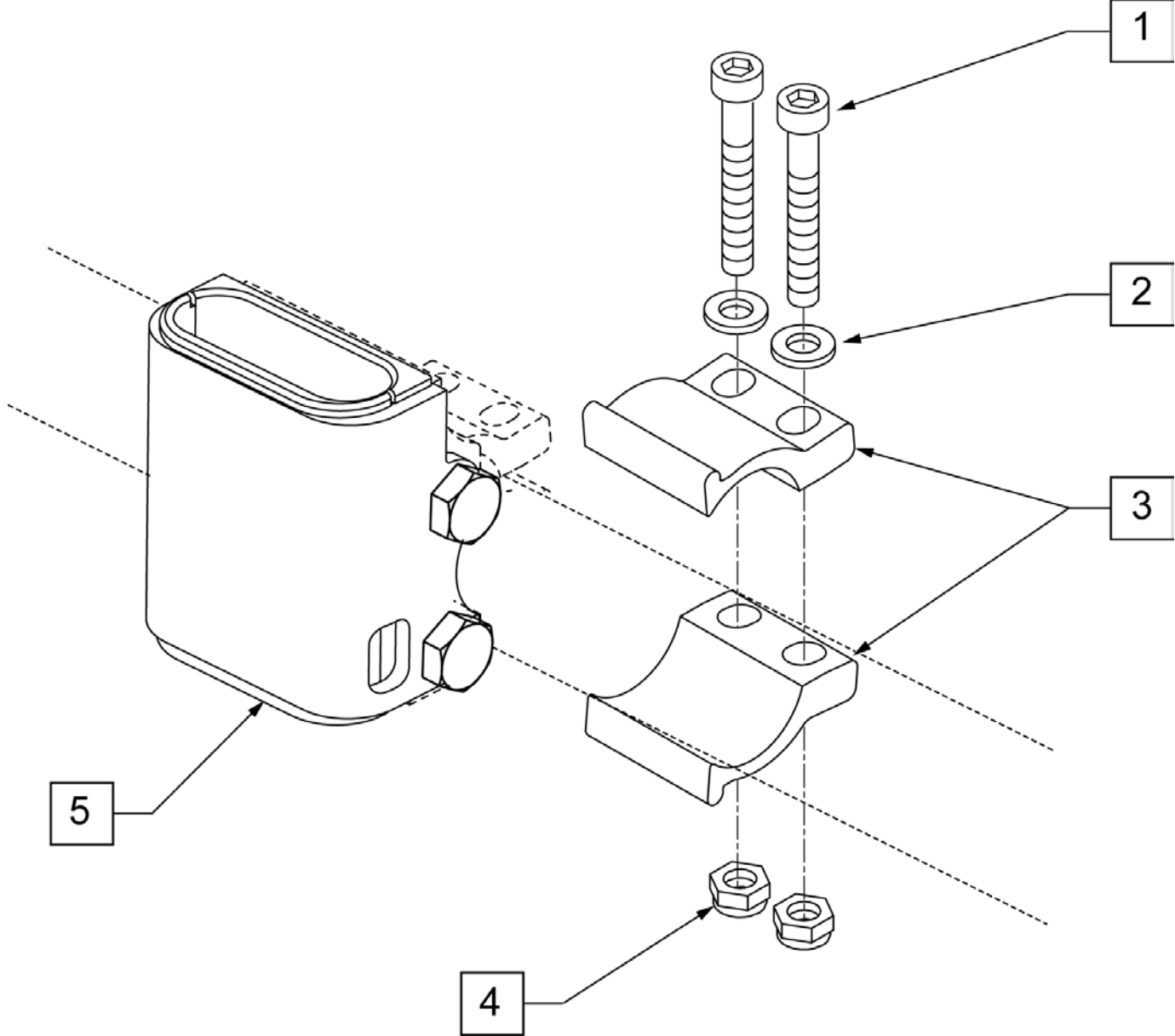

Pos.	Item Number	Description	USD
1	106975	SHCS 1/4 -28 x 1 1/4 ROHS	5.00
2	106943	WASHER FLAT AN 1/4 ROHS	1.00
3	107123	CLAMP, RECEIVER, 1" OD	10.00
4	106935	NUT NYLOCK THIN 1/4 -28 ROHS	5.00
5	107124	RECEIVER, SPHA 1" OD	30.00
1-5	108101	SPHA RECEIVER W/ CLAMP KIT	50.00

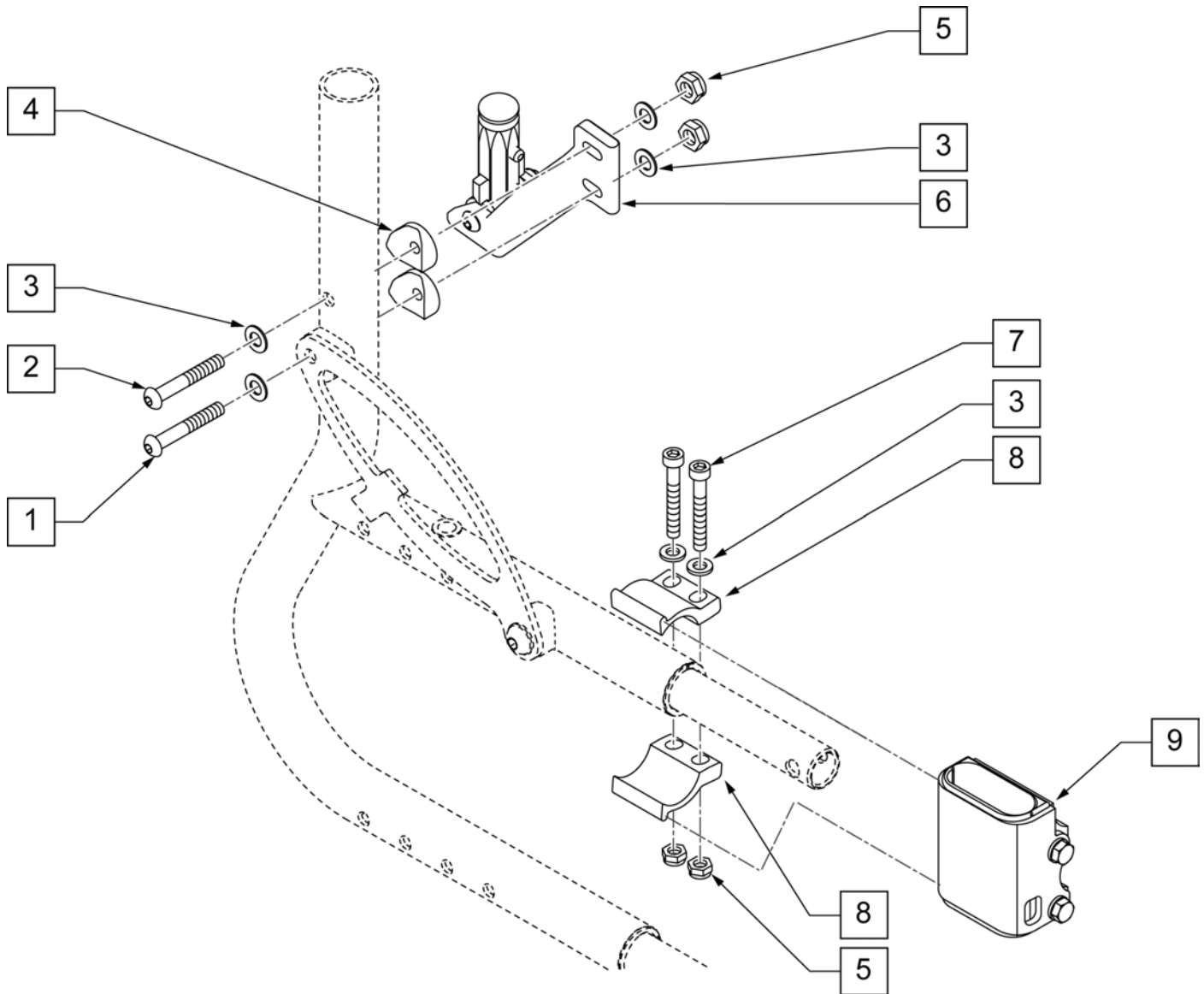


Pos.	Item Number	Description	Remarks	USD
1	106964	BHCS 1/4-28 X 2 ROHS		5.00
2	106962	BHCS 1/4 -28 x 1 3/4 ROHS		5.00
3	106943	WASHER FLAT AN 1/4 ROHS		1.00
4	561004	1" SADDLE 1/4" HOLE		2.00
5	106935	NUT NYLOCK THIN 1/4 -28 ROHS		5.00
6	109413	SPHA FB RECEIVER ASSM RH	RIGHT	50.00
6	109414	SPHA FB RECEIVER ASSM LH	LEFT	50.00
7	106975	SHCS 1/4 -28 x 1 1/4 ROHS		5.00
8	107123	CLAMP, RECEIVER, 1" OD		10.00
9	107124	SPHA RECEIVER KIT		30.00
3,5,7-9	108101	SPHA RECIEVER W/ CLAMP KIT		50.00
1-6	120657	REAR RECIVER ASSM - RIGHT	RIGHT	35.00
1-6	120658	REAR RECIVER ASSM - LEFT	LEFT	35.00

(04/2010)


NOTE: USED ON QUICKIE 2 LITE CHAIRS.

NOTE: Item 6 can be flipped for additional adjustment, when flipped Left becomes Right and Right becomes Left.

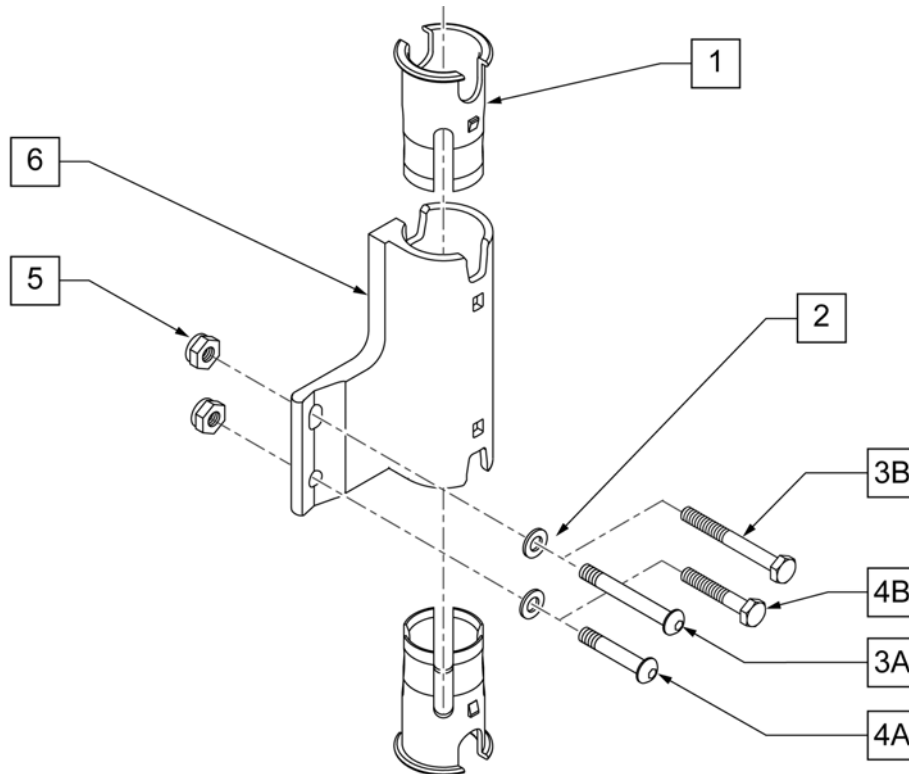


Pos.	Item Number	Description	Remarks	USD
1	106505	SLEEVE SWING AWAY ARMREST		5.00
2	106943	WASHER FLAT AN 1/4 ROHS		1.00
3A	106964	BHCS 1/4-28 X 2 ROHS	Q2 LITE ONLY	5.00
3B	107351	HHCS 1/4-28 X 2 1/4 ROHS	Q2 ONLY	2.00
4A	106962	BHCS 1/4-28 X 1-3/4 ROHS	Q2 LITE ONLY	5.00
4B	109265	HHCS 1/4-28 X 2-1/2 ROHS	Q2 ONLY	2.00
5	106935	NUT NYLOCK THIN 1/4-28 ROHS		5.00
1,6	106720	RECEIVER, SA ASSY, RH	RIGHT - SHOWN	30.00
1,6	106721	RECEIVER, SA ASSY, LH	LEFT	30.00


NOTE: KIDS BACKPACKS DO NOT HAVE ADJUSTABLE STRAPS W/ BUCKLES.

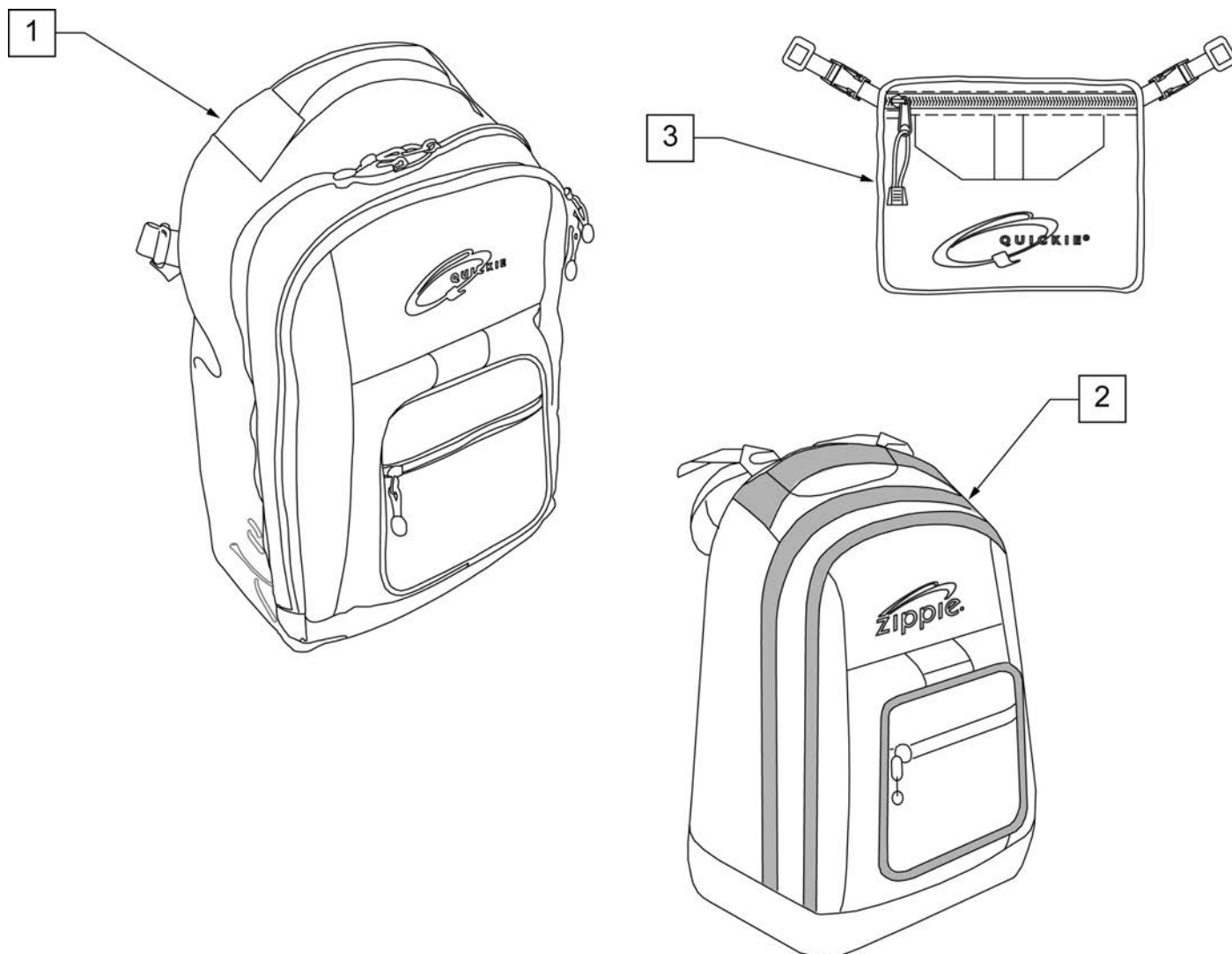


Pos.	Item Number	Description	USD
1	750115	BACK PACK QUICKIE	56.10
2	104572	BACKPACK ZIPPIE	55.00
3	750307	SEAT POUCH QUICKIE	40.80

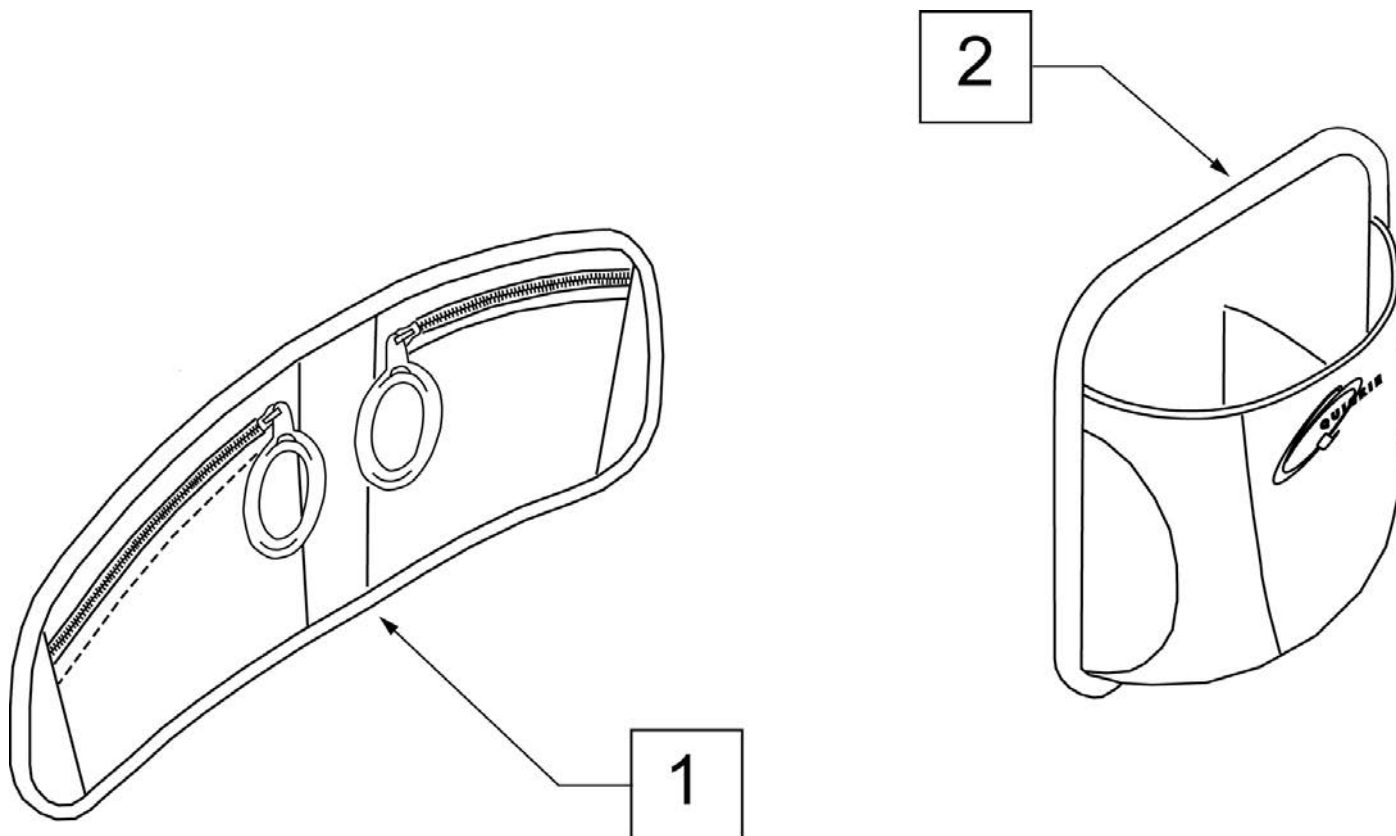


Pos.	Item Number	Description	Remarks	USD
1	013477	LAP ORGANIZER	Size: 18.25"x 7.75"x .75"	25.00
2	013475	QUICKIE SIDE POCKET	Size: 6.5"x 6.5"x 2"	9.99

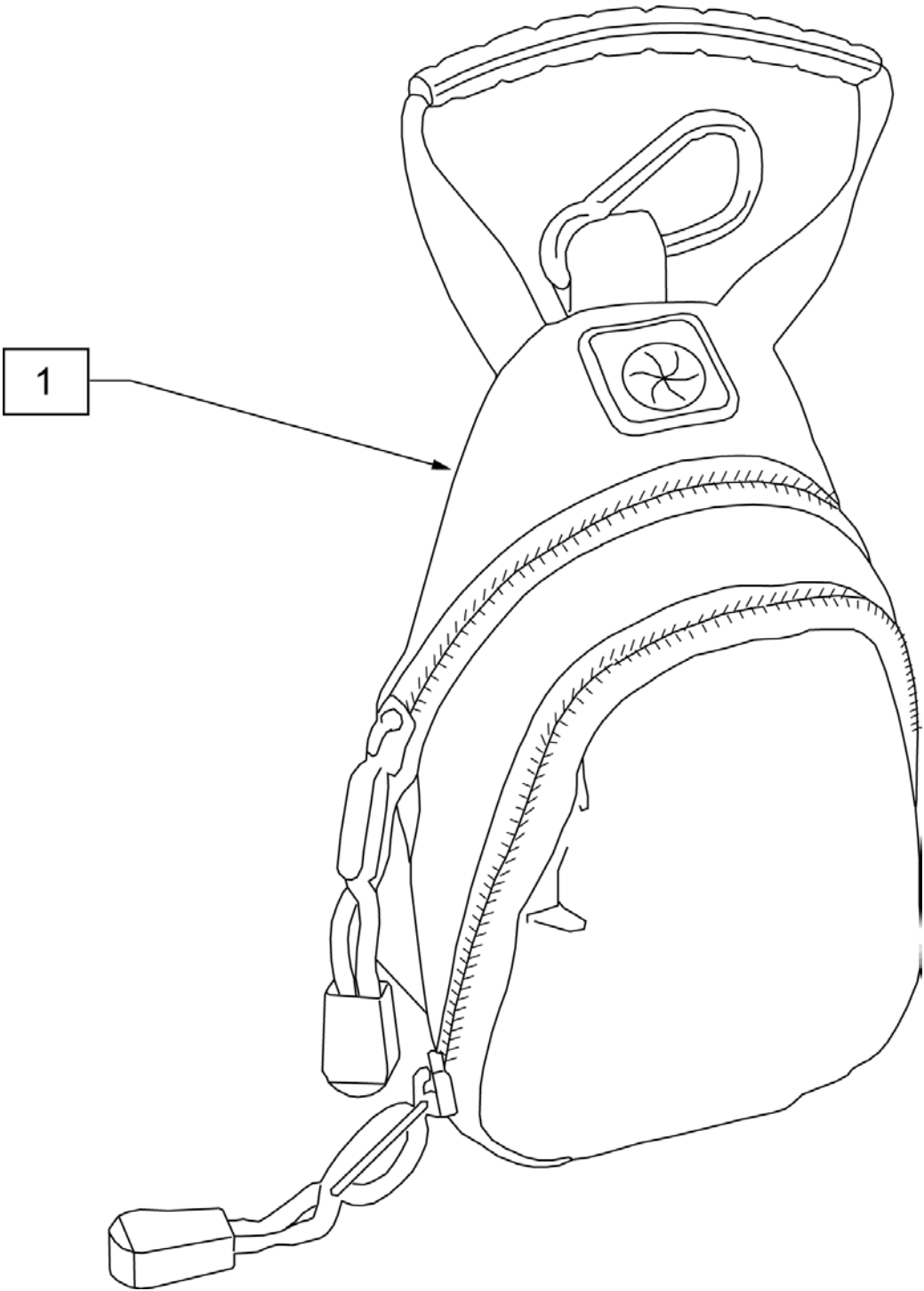


Pos.	Item Number	Description	USD
1	109229	CELL PHONE PACK (ORANGE)	25.00

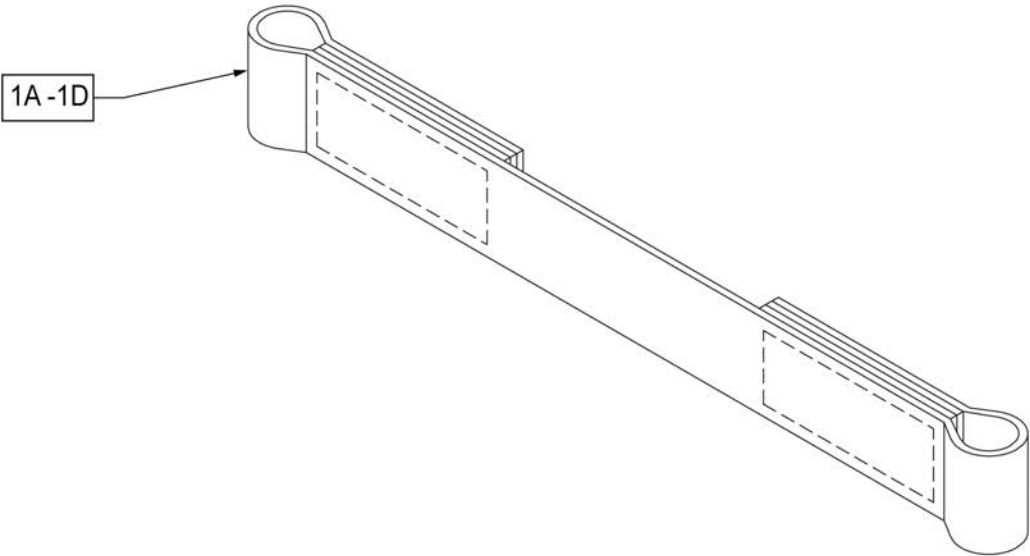

Pos.	Item Number	Description
1A	109356	LEG STRAP SOFT FEEL Q 11 -12"
1B	109357	LEG STRAP SOFT FEEL Q 13 -15"
1C	109366	LEG STRAP SOFT FEEL Q 16 -18"
1D	111487	LEG STRAP SOFT FEEL Q 19 -20"

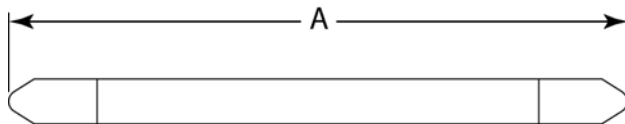MEASURING GUIDE


DESCRIPTION	DIM "A"
LEG STRAP 11"-12" BLK	27 1/2
LEG STRAP 13"-15" BLK	32 1/2
LEG STRAP 16"-18" BLK	39 1/2
LEG STRAP 19"-20" BLK	45 1/2

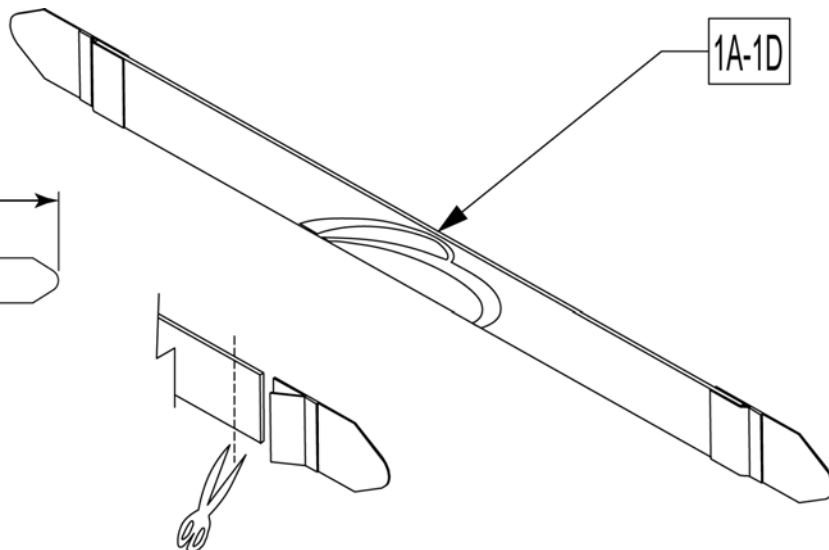



Leg Straps may be custom cut to a smaller size

Pos.	Item Number	Description
1A	113255	LEG STRAP 11"-12" BLK
1B	113256	LEG STRAP 13"-15" BLK
1C	113257	LEG STRAP 16"-18" BLK
1D	113258	LEG STRAP 19"-20" BLK

1A

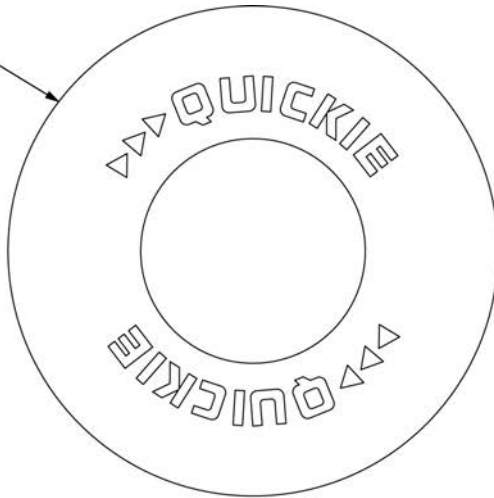


1B

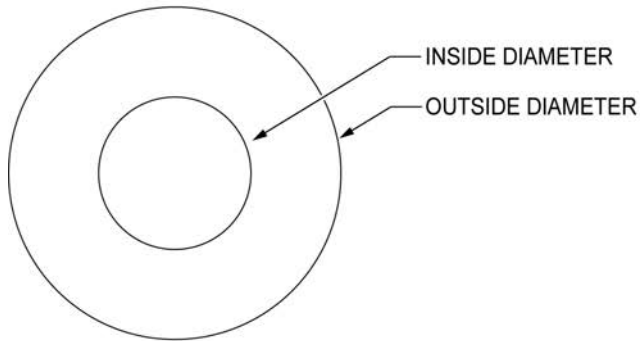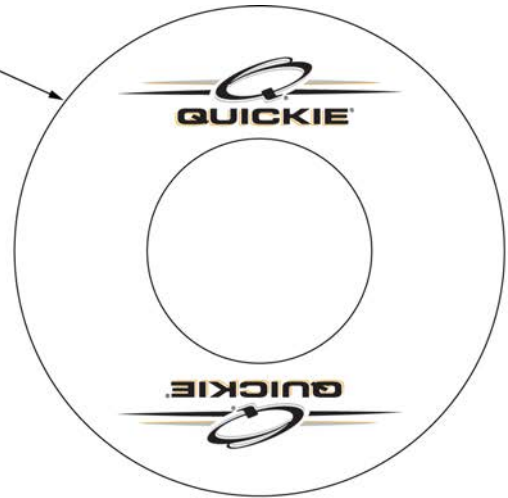
**SPOKE GUARD MEASURING GUIDE**

PART NUMBER	INSIDE DIAMETER	OUTSIDE DIAMETER
902001	8 1/4	15 7/8
902011	8 1/4	18 1/8
902021	10 1/4	19 5/8
902025	10 1/4	20 3/8
902037	10 1/4	21 7/8
902038	10 1/4	23 3/8
103660-001	8 1/4	16
103660-002	8 1/4	18
103663-001	10 1/4	20 1/2
103663-002	10 1/4	23 1/2
103734-001	10 1/4	19 1/4
103734-002	10 1/4	22
119336	10 1/4	19 1/4


Pos.	Item Number	Description	Remarks	USD
>>>>		BLACK SPOKE GUARDS		
1A	902001	SPK GRDS 20" QUICKIE BLK	Pair-BLACK-FITS 20" WHEELS	91.80
1A	902011	SPK GRDS 22" QUICKIE BLK	Pair-BLACK-FITS 22" WHEELS or 24" (540) WHEELS W/ PROJECTION HANDRIMS	91.80
1A	902021	SPK GRD 24" QUICKIE SCR MT BLK	Pair-BLACK-FITS 24" (540) MAG & SPOKE WHEELS	91.80
1A	902025	SPK GRD 24" QUICKIE TAB MT BLK	Pair-BLACK-FITS 25" (559) SPOKE WHEELS	91.80
1A	902037	SPK GRD 26" QUICKIE TAB MT BLK	Pair-BLACK-FITS 26" (590) WHEELS	91.80
1A	902038	SPK GRD 26" QUICKIE HT BLK	Pair-BLACK-FITS 27" (700) WHEELS	91.80
>>>>		CLEAR SPOKE GUARDS		
1B	103660-001	CLEAR SPOKE GUARD 20"	Pair-CLEAR-FITS 20" WHEELS	100.00
1B	103660-002	CLEAR SPOKE GUARD 22"	Pair-CLEAR-FITS 22" WHEELS or 24" (540) WHEELS W/ PROJECTION HANDRIMS	100.00
1B	103734-001	SPOKE GUARD 24" CLR	Pair-CLEAR-FITS 24" (540) SPOKE WHEELS	100.00
1B	119336	SPOKE GUARD 24" CLR	Pair-CLEAR-FITS 24" (540) MAG WHEELS	100.00
1B	103663-001	SPOKE GUARD 24" QP CLR	Pair-CLEAR-FITS 25" (559) WHEELS	100.00
1B	103734-002	SPOKE GUARD 26" CLR	Pair-CLEAR-FITS 26" (590) WHEELS	100.00
1B	103663-002	SPOKE GUARD 26" QP CLR	Pair-CLEAR-FITS 27"(700) WHEELS	100.00

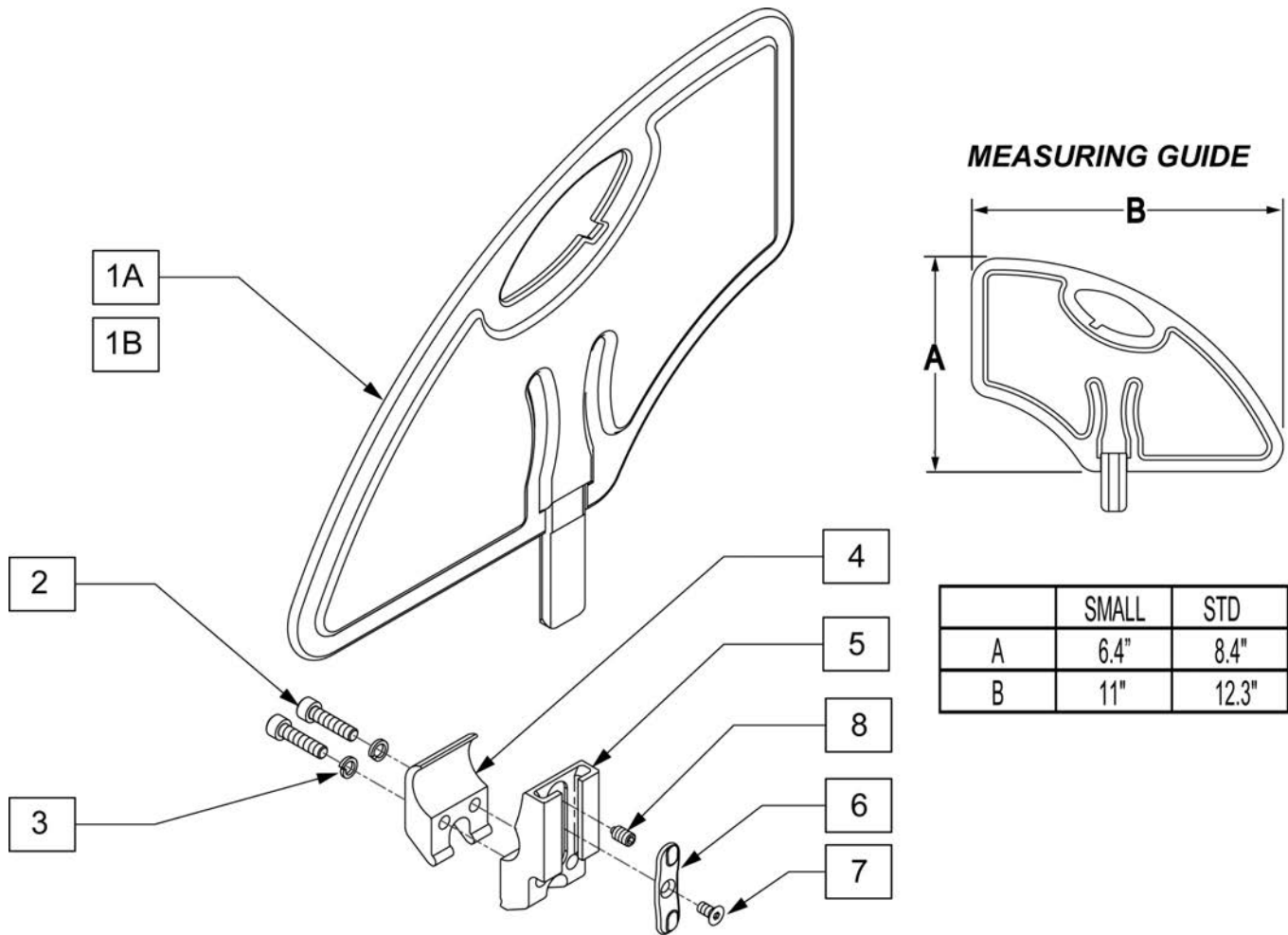


Pos.	Item Number	Description	USD
1A	104164-001	SIDE GUARD ASSY STD	45.00
1B	104164-002	SIDE GUARD ASSY SMALL	45.00
2	106961	BHCS 1/4-28 X 1-1/4 ROHS	5.00
3	107367	WASHER LOCK HI COLLAR 1/4 ROHS	5.00
4	106714	REC SIDE GUARD CLAMP THREADED	10.00
6	108780	RECEIVER SG PLASTIC INSERT	5.00
7	107135	FHCS #6-32 X 1/4 BLK	5.00
8	107136	SETSCREW 1/4-28 X 5/16 OVAL PT	5.00
5-8	108781	RECIEVER SIDE GUARD ASSY	15.00
2-8	107592	RECEIVER SIDE GUARD ASSY	35.00
1A-8	107590	SIDE GUARD ASSY W/ CLAMP STD	75.00
1B-8	107591	SIDE GUARD ASSY W/ CLAMP SMALL	75.00

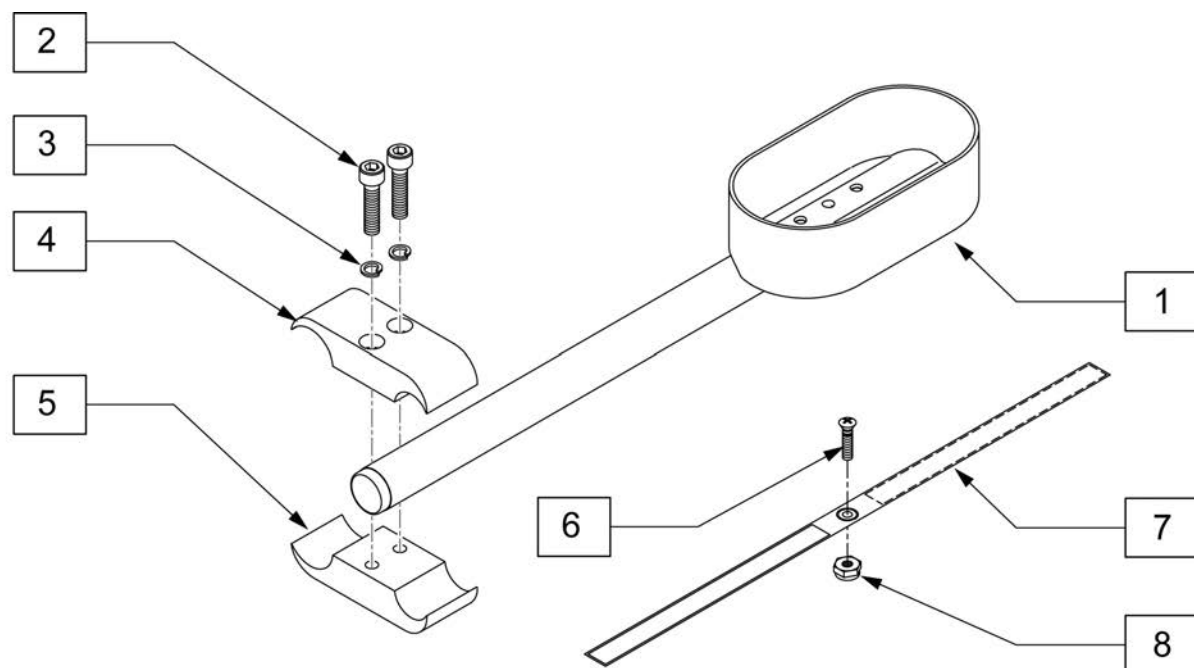


Pos.	Item Number	Description	Remarks	USD
1	119353	12" CRUTCH HOLDER ASSY RH		
1	119354	12" CRUTCH HOLDER ASSY LH		
2	107032	SHCS 1/4-28 X 1		5.00
3	455037-QU	1/4 X 1 FLAT WASHER		0.32
2-5	884101	VENT TRAY CLAMP ASSM S626		30.00
6	281110	10-32 x 1/2 PAN PHIL		2.00
7	750495	CRUTCH HOLDER STRAP F		11.00
8	200100-QU	10-32 NYLOCK NUT		2.00
1-8	119242	CSTM Q2 LITE CRUTCH HOLDER R	Right	105.00
1-8	119270	CSTM Q2 LITE CRUTCH HOLDER L	Left	105.00

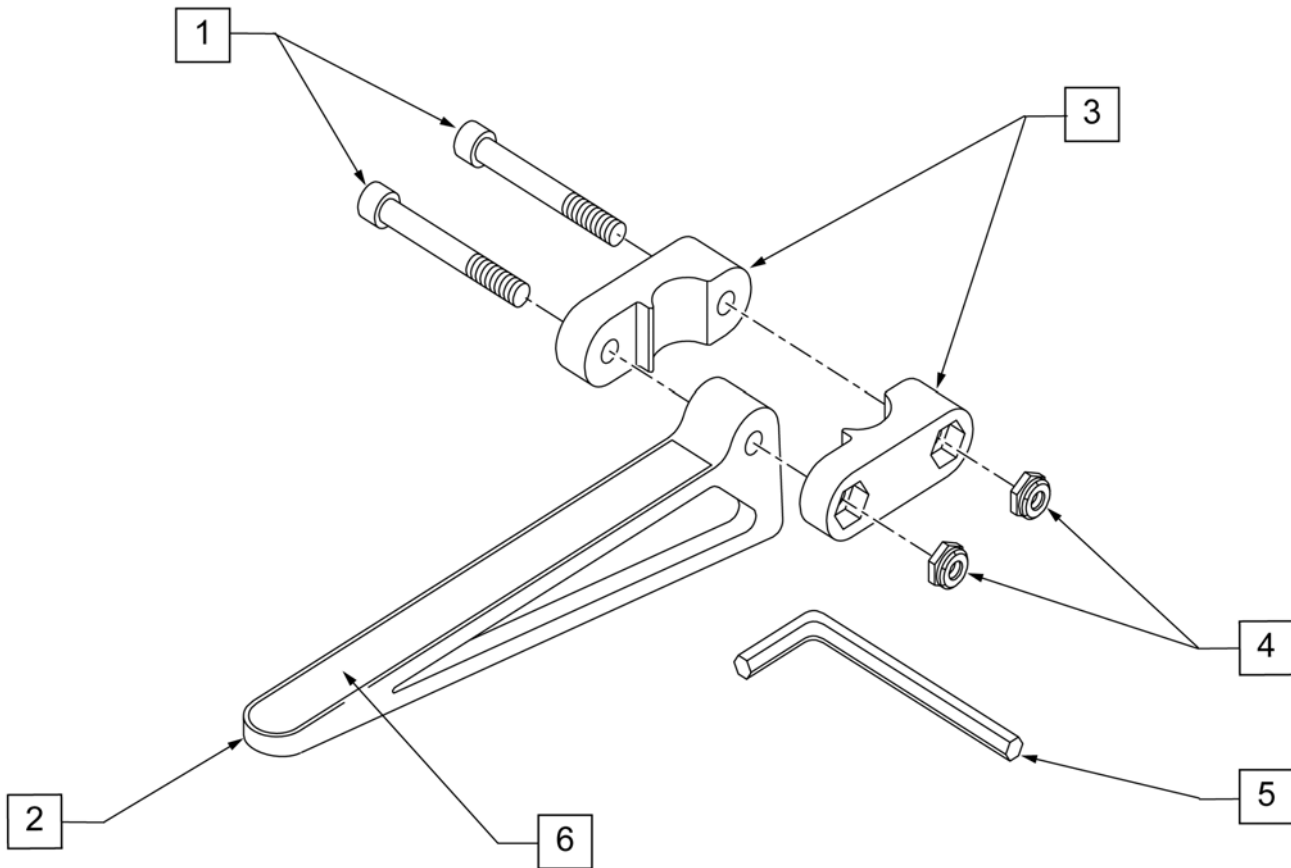


Pos.	Item Number	Description	Remarks	USD
1	226128	1/4-28 x 1 1/4 SOC C/S		
2	561105	CADDY ARM		21.90
3	561104	CADDY CLAMP		8.27
4	106935	NUT NYLOCK THIN 1/4-28 ROHS		5.00
5	349998	3/16" ALLEN WRENCH		4.63
6	580050	CADDY TAPE		2.43
1-6	902350	QUICKIE CADDY (PR)	PAIR	45.90

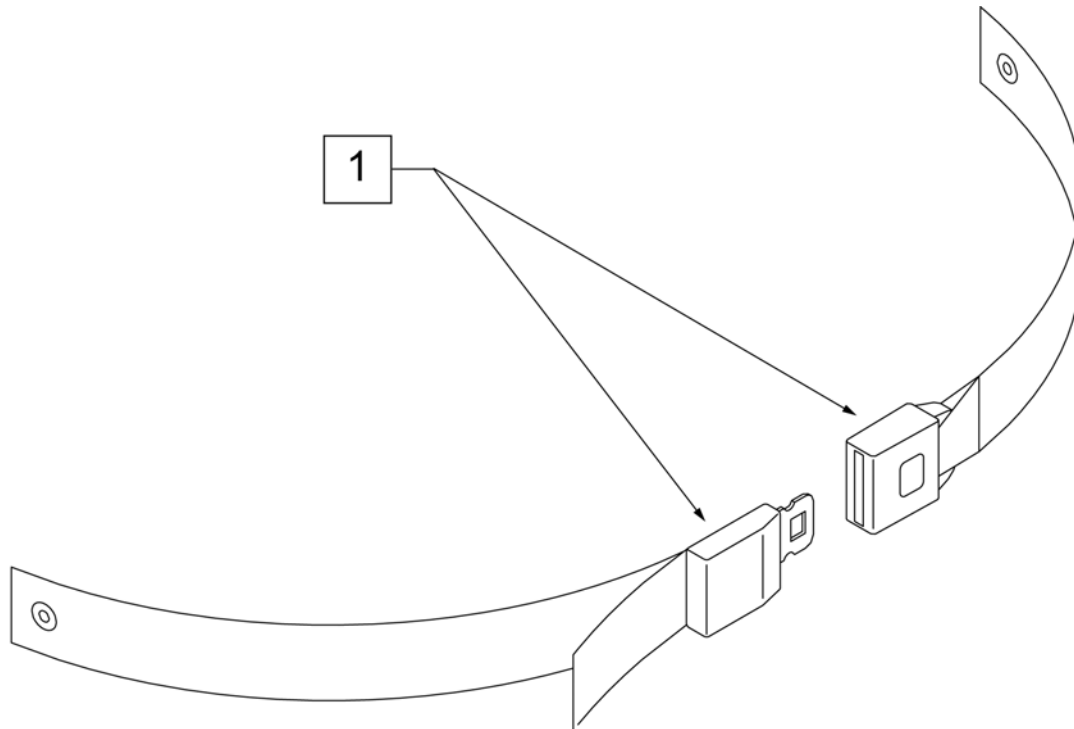


Pos.	Item Number	Description	Remarks	USD
1	750276	1 1/2" POSITION BELT	AUTOSTYLE-64" Max length of belt.	51.00

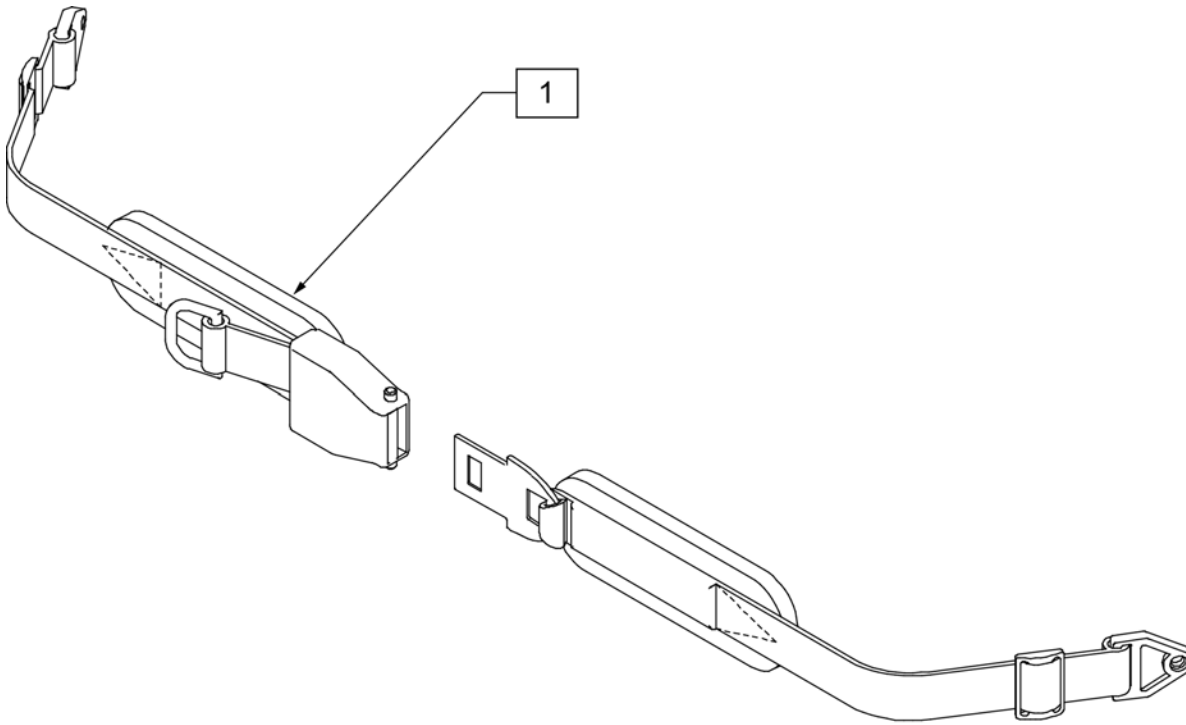


Pos.	Item Number	Description	USD
1	450418-QU	BELT 2" AIRCRAFT BUCKLE	130.00
1	450423-QU	BELT 2" AIRCRAFT BUCKLE W/PAD	180.00
1	450424	BELT 2" AIRCRAFT BKL LONG W/PAD	115.00
1	450425	BELT 2" AIRCRAFT BKL LONG	85.00
1	450419-QU	BELT 2" AIRCRAFT BKL W/QUAD LP	150.00
1	468419	BELT 2" AIRCRAFT BKL W/QD LPCZ	100.00

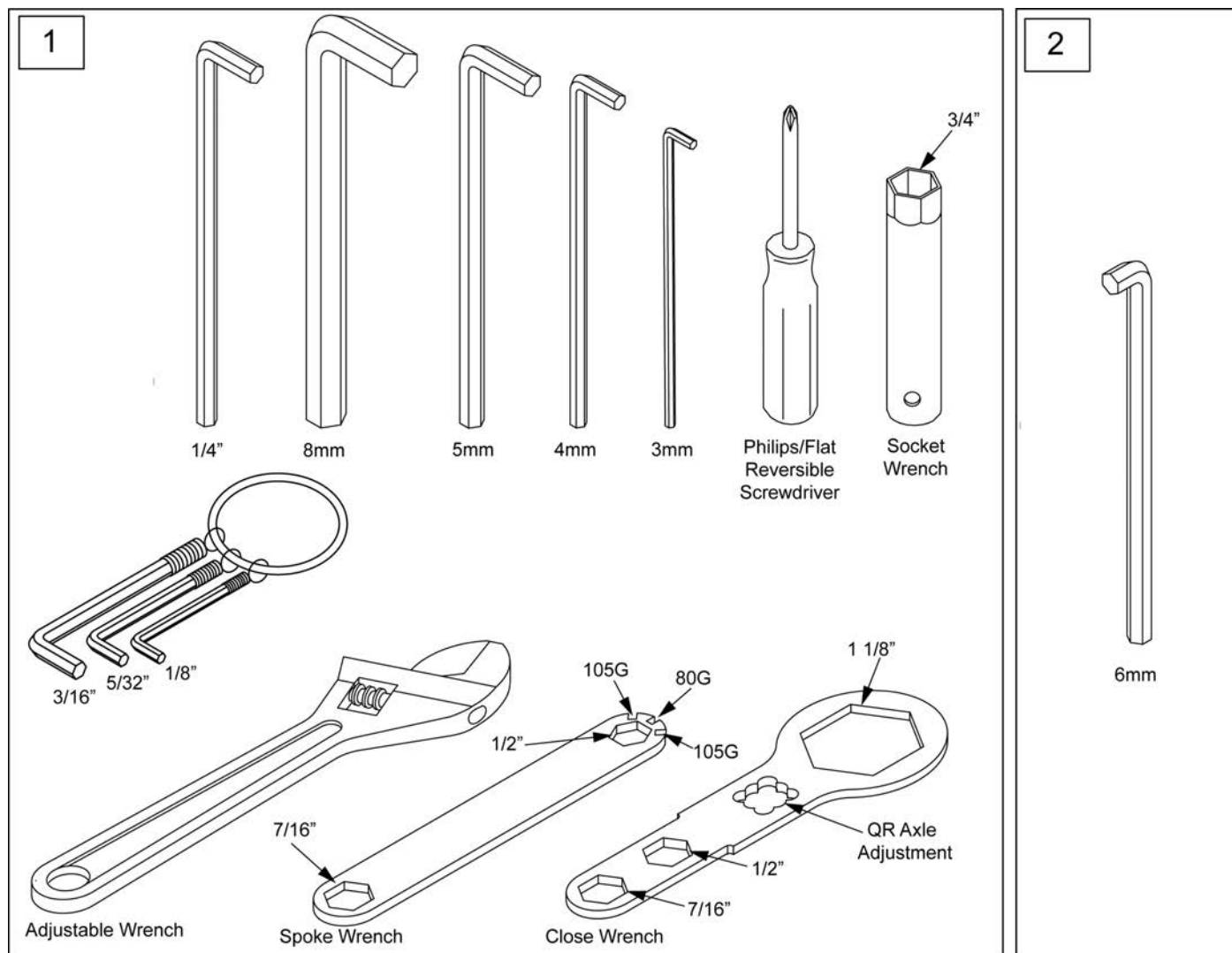

Pos.	Item Number	Description	Remarks	USD
1	108314	QUICKIE COMBO TOOL KIT	Includes pouch	40.00
2	113403	6MM ALLEN WRENCH L, 7MM SHORT	caster spline chairs only	7.50

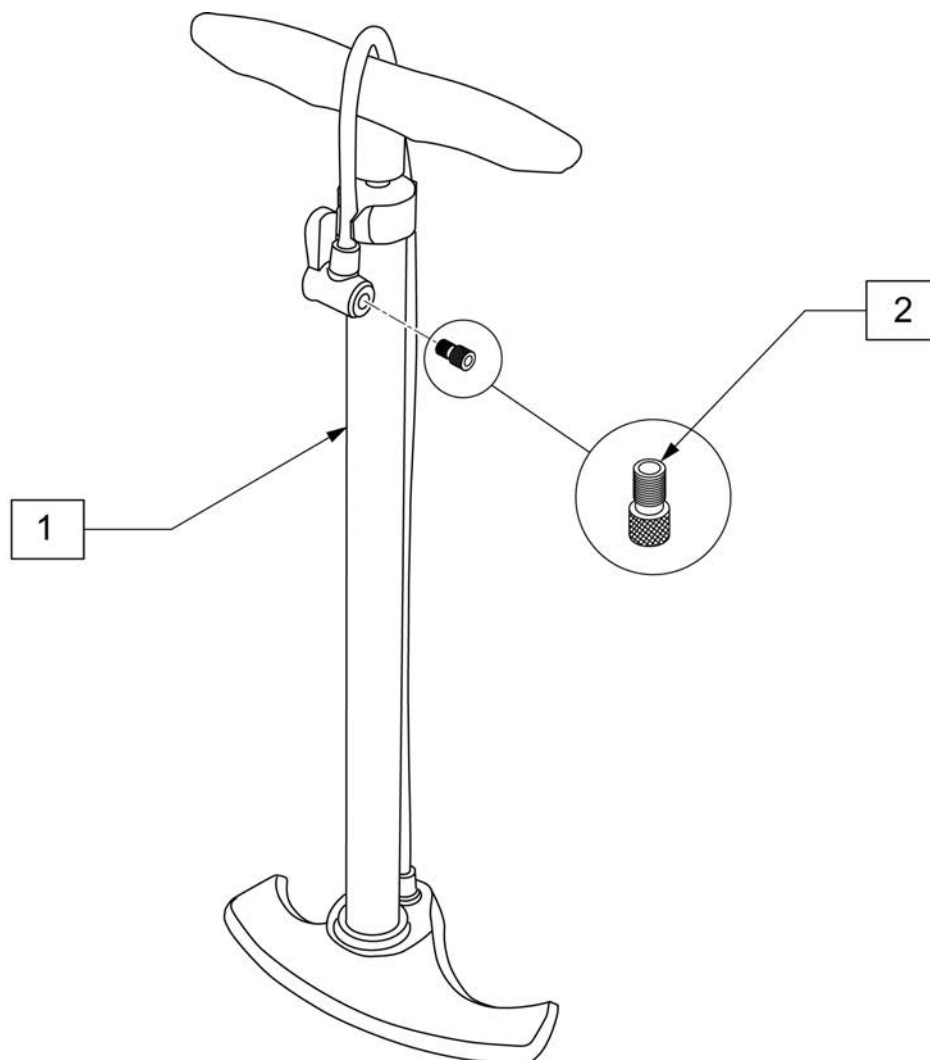

Pos.	Item Number	Description	USD
1	389050	TIRE PUMP	81.60
2	381230	VALVE ADAPTER PRESTA	5.10

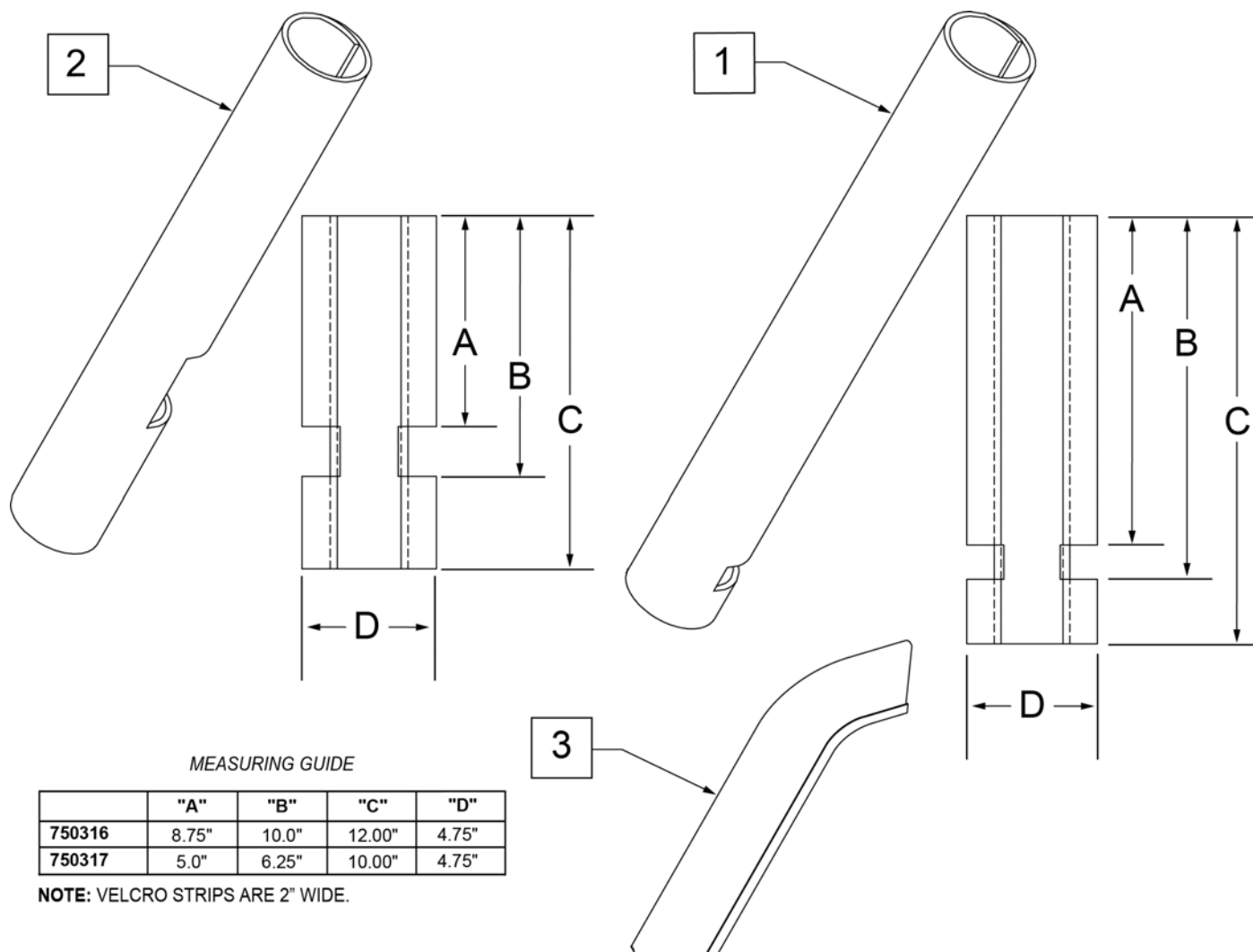

Pos.	Item Number	Description	Remarks	USD
1	750316	IMPACT GUARDS NEOPRENE FIXED (PR)	PAIR-Fixed Front Frame	85.00
2	750317	IMPACT GUARDS NEOPRENE S/A	PAIR-S/A Footrest & Q7	85.00
3	109231	GEL VELCRO PAD HANGER/FRAME	PAIR- fits S/A & Fixed Front Frame	105.00
4	108315	GEL PAD SA ARM	Single, fits Padded SA Armrest	



 **QUICKIE®** ²
Lite

PARTS MANUAL

MAY 2013