


# Discontinued Catalyst Aluminum Cross Braces

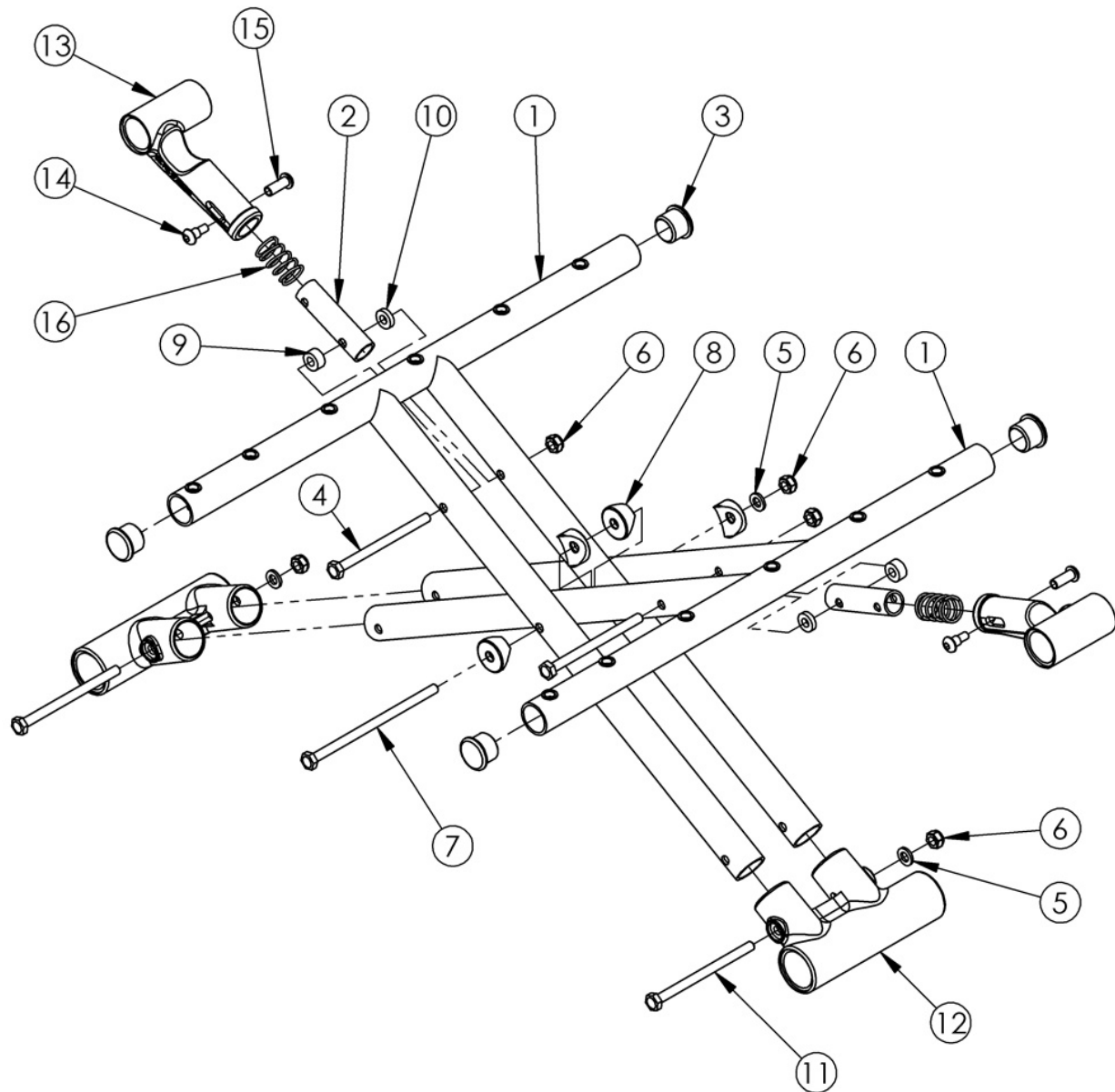

*Pictured round-style cross braces effective prior to 4/13/16 on 21" and 22" wide chairs ending with serial numbers C4017704, C4HD002592, C5030817, C5HD002748, C5TH003678. Pictured round-style cross braces effective prior to 6/15/16 on 14" - 20" wide chairs ending with serial numbers C4018743, C4HD002700, C5032543, C5037979, C5HD002834, C5TH003715.*

| Item Number    | Part Number | Description                    | Quantity |
|----------------|-------------|--------------------------------|----------|
| 2a, 13a, 14-16 | 500024      | Upper Pivot Assembly 14" Width | 2        |
| 2b, 13b, 14-16 | 500025      | Upper Pivot Assembly 15" Width | 2        |
| 2c, 13b, 14-16 | 500026      | Upper Pivot Assembly 16" Width | 2        |

| Item Number    | Part Number | Description                              | Quantity |
|----------------|-------------|--|----------|
| 2d, 13b, 14-16 | 500027      | Upper Pivot Assembly 17" Width           | 2        |
| 2e, 13b, 14-16 | 500028      | Upper Pivot Assembly 18" Width           | 2        |
| 2f, 13b, 14-16 | 500029      | Upper Pivot Assembly 19" Width           | 2        |
| 2g, 13b, 14-16 | 500030      | Upper Pivot Assembly 20" Width           | 2        |
| 2h, 13b, 14-16 | 501479      | Upper Pivot Assembly 21/22" Width        | 2        |
| 3              | 101162      | 7/8 OD End Plug BLK                      | 4        |
| 4              | 100660      | M6 x 80mm CLS8.8 DIN931 HHCS ZC          | 2        |
| 5              | 100746      | M6 Flat Washer Blk Zn                    | 3        |
| 6              | 100658      | M6 Nylock Nut Blk Zn                     | 5        |
| 7              | 100678      | M6 x 110 CLS8.8 DIN931 HHCS ZC           | 1        |
| 8              | 000054      | Saddle 7/8"                              | 4        |
| 9              | 100880      | 1/4" Plastic Spacer                      | 2        |
| 10             | 105019      | 3/16" Plastic Spacer                     | 2        |
| 11             | 100704      | M6 x 90 CLS8.8 DIN931 HHCS ZC            | 2        |
| 4-11           | 501398      | Round-Style Cross Brace Hardware Kit CAT | 1        |
| 12             | 000001      | Lower Frame Pivot CAT                    | 2        |