


Flip Stroller Handle Back Post

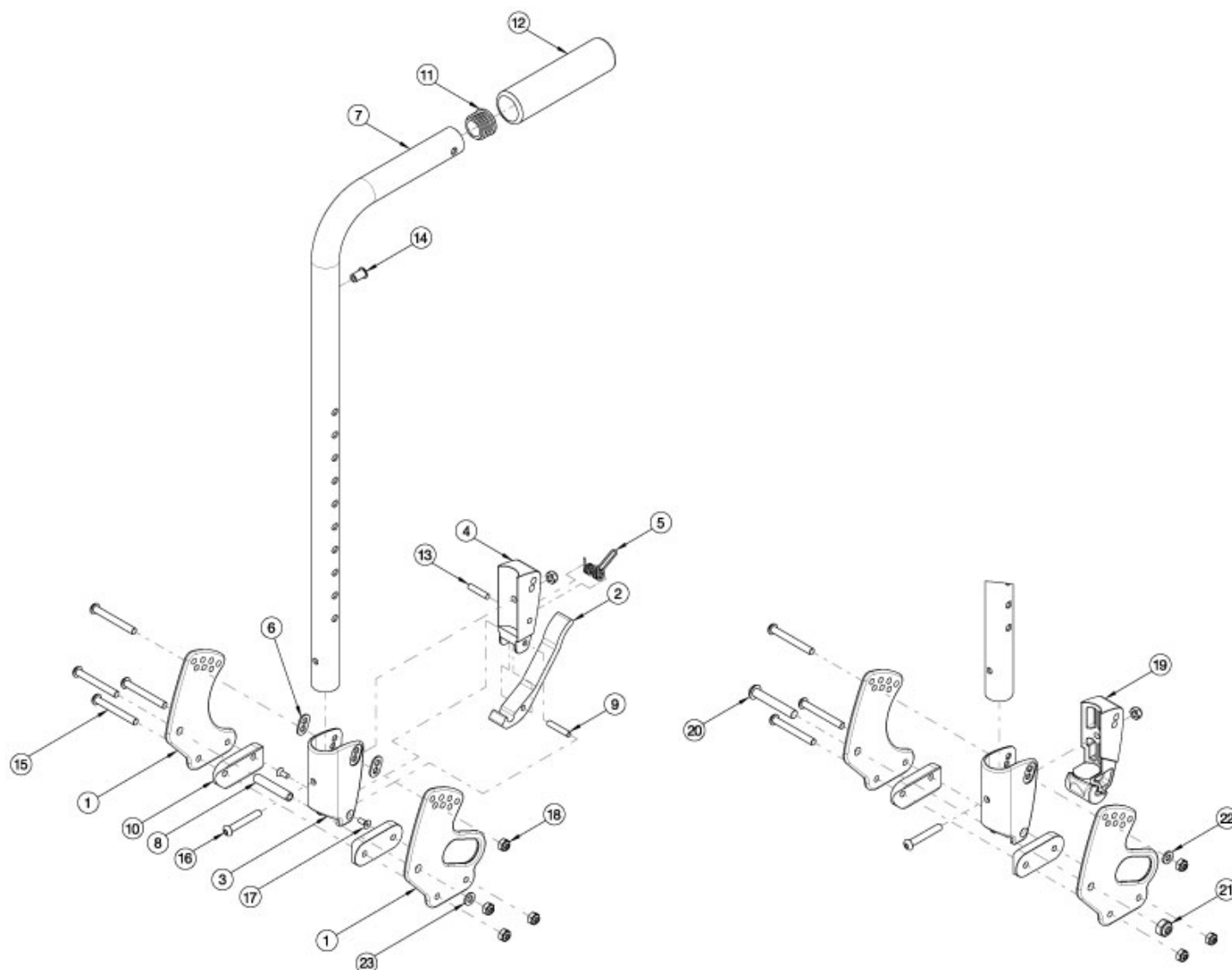

Note: Tilt Mechanism parts may be required if changing to a different backrest style. Please see the applicable tilt mechanism parts manual page for verification.

Item Number	Part Number	Description	Quantity
1a, 6, 8, 10, 15, 18, 23	502819	Backrest Hardware Kit Std FLIP	1
1a, 1b, 6, 8, 10, 15, 18, 23	502820	Backrest Hardware Kit Std Transit (one side), FLIP	1
1a, 10, 15, 16, 18, 20-22	503027	Backrest Hardware Kit, Non Folding, Flip	1
1a, 1b, 10, 15, 16, 18, 20-22	503163	Backrest Hardware Kit Non-Folding (one side), Transit, Flip	1
2-6, 7a, 9, 11, 12a, 13 - 15, 16-18, 23	502657	Back Cane Assembly w/ Latch Insert, Fixed Height, Short, Flip	1

Item Number	Part Number	Description	Quantity
2-5, 7b, 9, 11, 12a, 13 - 15, 16, 18, 23	502658	Back Cane Assembly w/ Latch Insert, Fixed Height, Tall, Flip	1
2-6, 7a, 9, 12b, 13 - 15, 16-18, 23	502659	Back Cane Assembly w/ Latch Insert, Fixed Stroller, Short, Flip <i>Used with Stroller Handle Extensions</i>	1
2-6, 7b, 9, 12b, 13 - 15, 16-18, 23	502660	Back Cane Assembly w/ Latch Insert, Fixed Stroller, Tall, Flip <i>Used with Stroller Handle Extensions</i>	1
3, 7a, 11, 12a, 14 - 16, 18, 19 - 22	502667	Back Cane Assembly w/ Latch Block, Fixed Height, Short, Non-Folding, Flip	1
3, 7b, 11, 12a, 14 - 16, 18, 19 - 22	502668	Back Cane Assembly w/ Latch Block, Fixed Height, Tall, Non-Folding, Flip	1
3, 7a, 12b, 14 - 16, 18, 19 - 22	502669	Back Cane Assembly w/ Latch Block, Fixed Stroller, Short, Non-Folding, Flip <i>Used with Stroller Handle Extensions</i>	1
3, 7b, 12b, 14 - 16, 18, 19 - 22	502670	Back Cane Assembly w/ Latch Block, Fixed Stroller, Tall, Non-Folding, Flip <i>Used with Stroller Handle Extensions</i>	1
1a	001890	Back Plate, Std	2
1b	001891	Back Plate, Transit	1
2	001893	Latch Lever, Backrest	1
3, 6	503834	Cover Plate, Latch Insert Kit	1
4	001895	Latch Insert, Folding Backrest	1
5	001896	Spring, Backrest Latch	1
6	001897	Washer, Dual, PSA	2
8	002738	Spacer .313 OD x .243 ID x 1.68 long	1
9	002513	Threaded Barrel, 5mm OD x M4	1
10	002595	Saddle, Dual, 1" OD X 1.5 SP	2
11	100804	Tube End Plug 1"	1
13	102694	Dowel Pin, M5 X 24, Blk Zn	1
15	101781	M6 x 50mm Partial Thread BHCS Blk ZC	4
16	100797	M6 x 40 BHCS BLK ZC	1
17	101818	M4 x 10 FHCS BLKZC w/Patch	2
18	100658	M6 Nylock Nut Blk Zn	5

Item Number	Part Number	Description	Quantity
19	002035	Insert, Backrest, Non-Folding	1
20	101815	M8 x 50 BHCS Blk Zn	1
21	100659	M8 Nylock Nut Blk Zn	1
22	100746	M6 Flat Washer Blk Zn	1
23	101861	Nylon Washer 5/16 x 1/2 x .03	1