


Catalyst Height Adjustable Low T-Arm

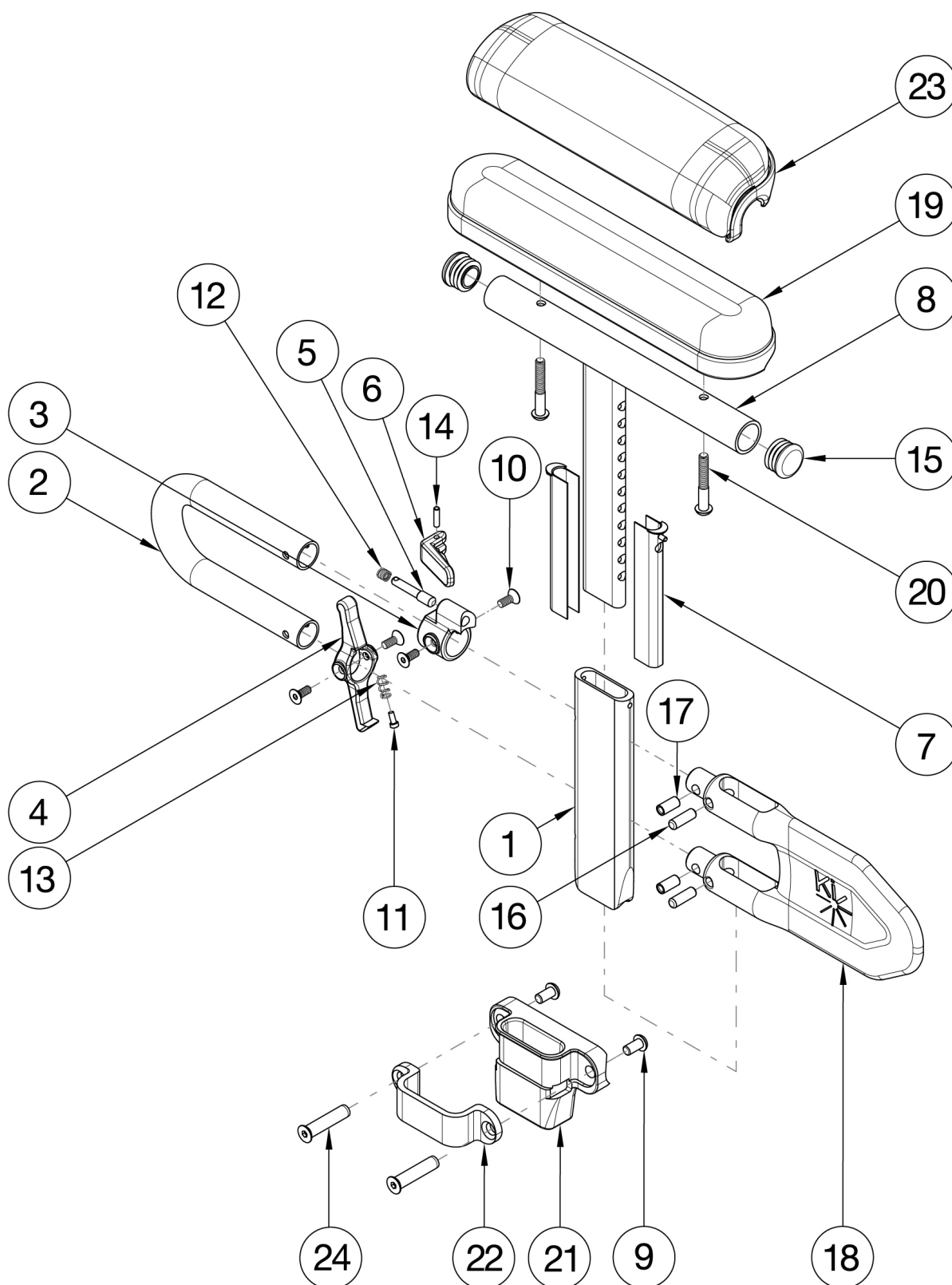

Receiver and Hardware update effective 11/25/24 If retrofitting, Receivers are needed. If retrofitting to Height Adjustable Low T-Arms and One Arm Drive is present, a new drive shaft may be required. In addition to a new drive shaft, order the corresponding hardware kit: 109438 for Blade Style One Arm Drive, 503433 for Hex Style One Arm Drive or 112898 for Quick Release Style One Arm Drive

Item Number	Part Number	Description	Quantity
1-8a, 9, 11-15, 18, 19b, 20a, 21, 22, 24	124088	Height Adj T-Arm Assm Low, Desk w/ Std Pad and Receiver PAIR Catalyst	1
1-8a, 9, 11-15, 18, 19b, 20a, 21, 22, 24	124086	Height Adj T-Arm Assm Low, Desk w/ Std Pad and Receiver Catalyst	1
1-8b, 9, 11-15, 18, 19a, 20a, 21, 22, 24	124084	Height Adj T-Arm Assm Low, Full w/ Std Pad and Receiver PAIR Catalyst	1
1-8b, 9, 11-15, 18, 19a, 20a, 21, 22, 24	124082	Height Adj T-Arm Assm Low, Full w/ Std Pad and Receiver Catalyst	1
1-8a, 9, 11-15, 18, 20a, 21, 22, 23b, 24	124089	Height Adj T-Arm Assm Low, Desk w/ Waterfall Pad and Receiver PAIR Catalyst	1
1-8a, 9, 11-15, 18, 20b, 21, 22, 23b, 24	124087	Height Adj T-Arm Assm Low, Desk w/ Waterfall Pad and Receiver Catalyst	1
1-8b, 9, 11-15, 18, 20b, 21, 22, 23a, 24	124085	Height Adj T-Arm Assm Low, Full w/ Waterfall Pad and Receiver PAIR Catalyst	1
1-8b, 9, 11-15, 18, 20b, 21, 22, 23a, 24	124083	Height Adj T-Arm Assm Low, Full w/ Waterfall Pad and Receiver Catalyst	1
1-8a, 11-15, 18, 19b, 20a	121241	Height Adjustable T-Arm Assembly Low, Desk w/ Std Pad	1
1-8b, 9, 11-15, 18, 19a, 20a	503169	Height Adjustable T-Arm Assembly Low Full w/ Std Pad	1
1-8a, 9, 11-15, 18, 20b, 23b	121243	Height Adjustable T-Arm Assembly Low, Desk w/ Waterfall Pad	1
1-8b, 11-15, 18, 20b, 23a	503185	Height Adjustable T-Arm Assembly Low Full w/ Waterfall Pad	1
1-8a, 11-15, 18	114488	Height Adjustable T-Arm, Desk, Low <i>Replaced 002681 / 002682</i>	1
1-8b, 11-15, 18	114489	Height Adjustable T-Arm, Full, Low <i>Replaced 002683 / 002684</i>	1
9, 21, 22, 24	501730	Height Adj T-Arm Receiver Assm 1" CAT	1
1	002105	Tube, Outer, T-Arm, Low	1
2	002104	Transfer Tube, T-Arm, Low	1
3	000096	Height Adjustable T Armrest Pin Lock Housing	1
4	002106	Lever, T-Arm, Low	1
5	000099	Height Adjustable Armrest Lever Pin	1

Item Number	Part Number	Description	Quantity
6	003418	Armrest Height Adjustment Lever	1
7	000090	Height Adjustable T Armrest Shim	2
8a	002648	Post Weldment, T-Arm, Low, Desk	1
8b	002649	Post Weldment, T-Arm, Low, Full	1
9	100667	M6 x 12 BTN SHCS BLKZC w/Patch	2
10	003458	M5 X 12mm FHCS BLK ZN W/ Patch	4
11	000152	Height Adjustable T Armrest Release Spring Button	1
12	000134	Height Adjustable Armrest Lever Spring	1
13	000135	Height Adjustable T Armrest Release Lever Spring	1
14	200257	Dowel Pin 1/8 x 1/2	1
15	100803	Tube End Plug 7/8"	2
16	003641	.25" x .75" Dowel Pin	2
17	003457	M5 x 14mm Barrel Nut	2
18	114487	HA T-Arm Side Shield, Low	1
19a	100500	Arm Pad Full	1
19b	100501	Arm Pad Desk	1
20a	100696	M5 x 30mm PPH Capscrew <i>Used with Standard Arm Pads</i>	2
20b	100654	M5 x 40 DIN7985 PH Pan HD M/S ZC <i>Used with Waterfall Arm Pads</i>	2
21	122221	T-Arm Receiver 1", Cat	1
22	122222	Height Adjustable T Armrest Bracket, CAT/SPARK	1
23a	102452	Waterfall Arm Pad Full	1
23b	102521	Waterfall Arm Pad Desk	1
24	122055	Binding Post, Flat Head, M6, .38"	2