


Liberty Height Adjustable Backrest

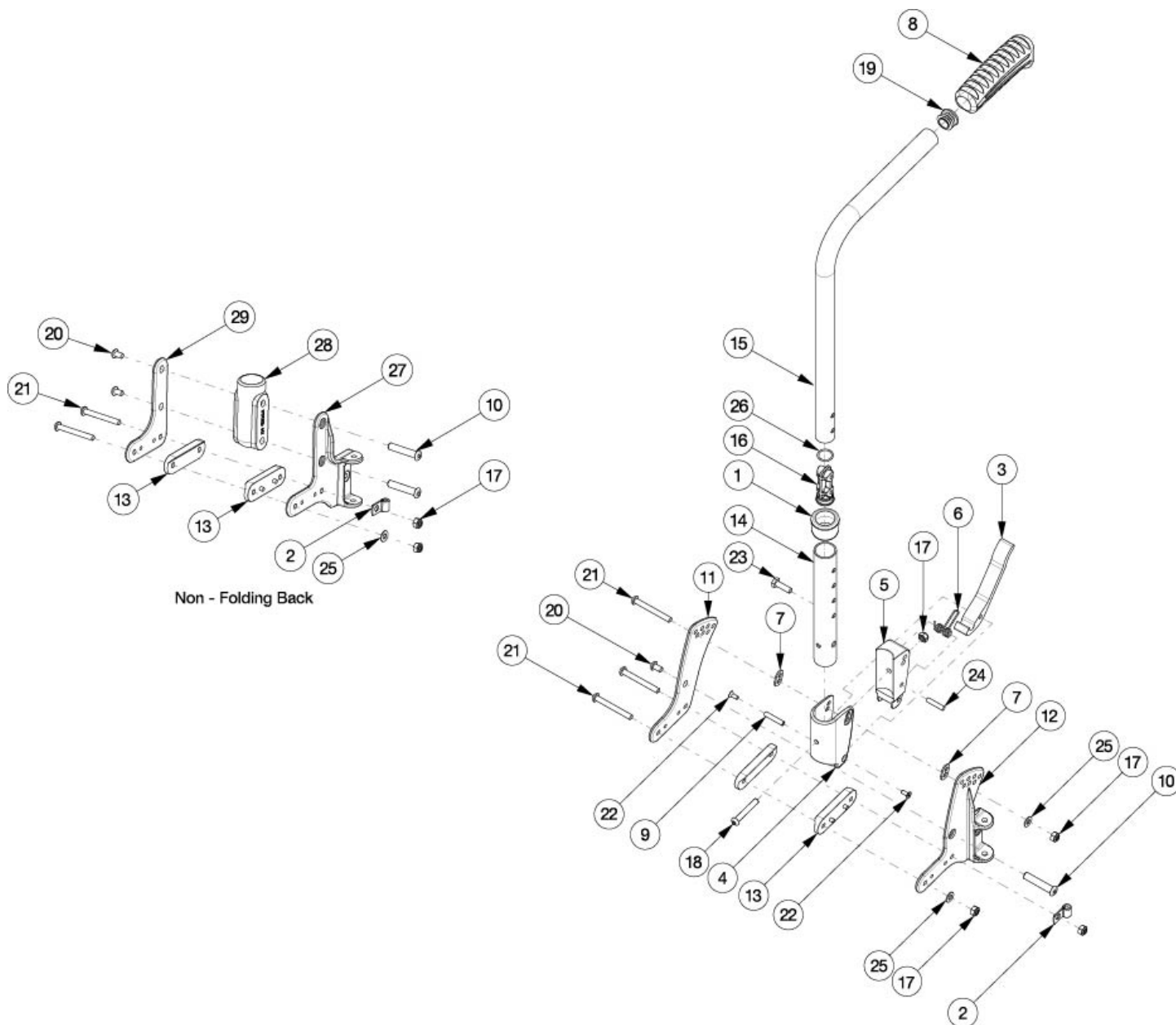

If changing to a new back post style or height, please see the Liberty Tilt Mechanism page as new tilt cables may be required. If changing from a non-folding stroller handle backrest to a non-folding height adjustable backrest, #28 (003310) is required.

Item Number	Part Number	Description	Quantity
1, 3-10, 11a, 12a, 12b, 13, 14, 15a, 16-26	120785	Short Height Adjustable Straight with Push Handle Folding Pair Kit Liberty	1
1, 3-10, 11a, 12a, 13, 14, 15a, 16-27	122869	Short Height Adjustable Straight with Push Handle Folding Right Liberty	1
1, 3-10, 11a, 12b, 13, 14, 15a, 16-28	122857	Short Height Adjustable Straight with Push Handle Folding Left Liberty	1

Item Number	Part Number	Description	Quantity
1, 3-10, 11b, 12a, 12b, 13, 14, 15a, 16-26	120791	Short Height Adjustable Straight with Push Handle Folding Pair Kit Transit Liberty	1
1, 3-10, 11b, 12a, 13, 14, 15a, 16-27	122875	Short Height Adjustable Straight with Push Handle Folding Right Transit Liberty	1
1, 3-10, 11b, 12b, 13, 14, 15a, 16-28	122863	Short Height Adjustable Straight with Push Handle Folding Left Transit Liberty	1
1, 3-10, 11a, 12a, 12b, 13, 14, 15b, 16-26	120786	Medium Height Adjustable Straight with Push Handle Folding Pair Kit Liberty	1
1, 3-10, 11a, 12a, 13, 14, 15b, 16-27	122870	Medium Height Adjustable Straight with Push Handle Folding Right Liberty	1
1, 3-10, 11a, 12b, 13, 14, 15b, 16-28	122858	Medium Height Adjustable Straight with Push Handle Folding Left Liberty	1
1, 3-10, 11b, 12a, 12b, 13, 14, 15b, 16-26	120792	Medium Height Adjustable Straight with Push Handle Folding Pair Kit Transit Liberty	1
1, 3-10, 11b, 12a, 13, 14, 15b, 16-27	122876	Medium Height Adjustable Straight with Push Handle Folding Right Transit Liberty	1
1, 3-10, 11b, 12b, 13, 14, 15b, 16-28	122864	Medium Height Adjustable Straight with Push Handle Folding Left Transit Liberty	1
1, 3-10, 11a, 12a, 12b, 13, 14, 15c, 16-26	120787	Tall Height Adjustable Straight with Push Handle Folding Pair Kit Liberty	1
1, 3-10, 11a, 12a, 13, 14, 15c, 16-27	122871	Tall Height Adjustable Straight with Push Handle Folding Right Liberty	1
1, 3-10, 11a, 12b, 13, 14, 15c, 16-28	122859	Tall Height Adjustable Straight with Push Handle Folding Left Liberty	1
1, 3-10, 11b, 12a, 12b, 13, 14, 15c, 16-26	120793	Tall Height Adjustable Straight with Push Handle Folding Pair Kit Transit Liberty	1
1, 3-10, 11b, 12a, 13, 14, 15c, 16-27	122877	Tall Height Adjustable Straight with Push Handle Folding Right Transit Liberty	1
1, 3-10, 11b, 12b, 13, 14, 15c, 16-28	122865	Tall Height Adjustable Straight with Push Handle Folding Left Transit Liberty	1
1, 8, 10, 13, 14, 15a, 16, 19-21, 23, 26, 27a, 27b, 28, 29a	120782	Short Height Adjustable Straight with Push Handle Non-Folding Pair Kit Liberty	1
1, 8, 10, 13, 14, 15a, 16, 19-21, 23, 26, 27a, 28, 29a	122872	Short Height Adjustable Straight with Push Handle Non-Folding Right Tranist Liberty	1

Item Number	Part Number	Description	Quantity
1, 8, 10, 13, 14, 15a, 16, 19-21, 23, 26, 27b, 28, 29a	122854	Short Height Adjustable Straight with Push Handle Non-Folding Left Liberty	1
1, 8, 10, 13, 14, 15a, 16, 19-21, 23, 26, 27a, 27b, 28, 29b	120788	Short Height Adjustable Straight with Push Handle Non-Folding Pair Kit Tranist Liberty	1
1, 8, 10, 13, 14, 15a, 16, 19-21, 23, 26, 27a, 28, 29b	122872	Short Height Adjustable Straight with Push Handle Non-Folding Right Tranist Liberty	1
1, 8, 10, 13, 14, 15a, 16, 19-21, 23, 26, 27b, 28, 29b	122860	Short Height Adjustable Straight with Push Handle Non-Folding Left Tranist Liberty	1
1, 8, 10, 13, 14, 15b, 16, 19-21, 23, 26, 27a, 27b, 28, 29a	120783	Medium Height Adjustable Straight with Push Handle Non-Folding Pair Kit Liberty	1
1, 8, 10, 13, 14, 15b, 16, 19-21, 23, 26, 27a, 28, 29a	122867	Medium Height Adjustable Straight with Push Handle Non-Folding Right Liberty	1
1, 8, 10, 13, 14, 15b, 16, 19-21, 23, 26, 27b, 28, 29a	122855	Medium Height Adjustable Straight with Push Handle Non-Folding Left Liberty	1
1, 8, 10, 13, 14, 15b, 16, 19-21, 23, 26, 27a, 27b, 28, 29b	120789	Medium Height Adjustable Straight with Push Handle Non-Folding Pair Kit Tranist Liberty	1
1, 8, 10, 13, 14, 15b, 16, 19-21, 23, 26, 27a, 28, 29b	122873	Medium Height Adjustable Straight with Push Handle Non-Folding Right Tranist Liberty	1
1, 8, 10, 13, 14, 15b, 16, 19-21, 23, 26, 27b, 28, 29b	122861	Medium Height Adjustable Straight with Push Handle Non-Folding Left Tranist Liberty	1
1, 8, 10, 13, 14, 15c, 16, 19-21, 23, 26, 27a, 27b, 28, 29a	120784	Tall Height Adjustable Straight with Push Handle Non-Folding Pair Kit Liberty	1
1, 8, 10, 13, 14, 15c, 16, 19-21, 23, 26, 27a, 28, 29a	122868	Tall Height Adjustable Straight with Push Handle Non-Folding Right Liberty	1
1, 8, 10, 13, 14, 15c, 16, 19-21, 23, 26, 27b, 28, 29a	122856	Tall Height Adjustable Straight with Push Handle Non-Folding Left Liberty	1

Item Number	Part Number	Description	Quantity
1, 8, 10, 13, 14, 15c, 16, 19-21, 23, 26, 27a, 27b, 28, 29b	120790	Tall Height Adjustable Straight with Push Handle Non-Folding Pair Kit Tranist Liberty	1
1, 8, 10, 13, 14, 15c, 16, 19-21, 23, 26, 27a, 28, 29b	122874	Tall Height Adjustable Straight with Push Handle Non-Folding Right Tranist Liberty	1
1, 8, 10, 13, 14, 15c, 16, 19-21, 23, 26, 27b, 28, 29b	122862	Tall Height Adjustable Straight with Push Handle Non-Folding Left Tranist Liberty	1
1, 3-9, 14, 15a, 16, 18, 19, 22-24, 26	506327	Backpost Assembly Short Height Adjustable Straight with Push Handle, Folding, Liberty	1
1, 3-9, 14, 15b, 16, 18, 19, 22-24, 26	506328	Backpost Assembly Medium Height Adjustable Straight with Push Handle, Folding, Liberty	1
1, 3-9, 14, 15c, 16, 18, 19, 22-24, 26	506329	Backpost Assembly Tall Height Adjustable Straight with Push Handle, Folding, Liberty	1
1, 8, 14, 15a, 16, 19, 23, 26, 28	506166	Backpost Assembly Short Height Adjustable Straight with Push Handle, Non-Folding, Liberty	1
1, 8, 14, 15b, 16, 19, 23, 26, 28	506167	Backpost Assembly Medium Height Adjustable Straight with Push Handle, Non-Folding, Liberty	1
1, 8, 14, 15c, 16, 19, 23, 26, 28	506168	Backpost Assembly Tall Height Adjustable Straight with Push Handle, Non-Folding, Liberty	1
1, 14, 23, 28	506317	Lower Back Tube Assembly Height Adjustable, Non-Folding, Liberty	1
1, 3-7, 9, 14, 17, 18, 22-24	506318	Lower Back Tube Assembly Height Adjustable, Folding, Liberty	1
8, 15a, 16, 19, 26	506314	Upper Back Tube Assembly Height Adjustable, Straight, Short, Liberty	1
8, 15b, 16, 19, 26	506315	Upper Back Tube Assembly Height Adjustable, Straight, Medium, Liberty	1
8, 15c, 16, 19, 26	506316	Upper Back Tube Assembly Height Adjustable, Straight, Tall, Liberty	1
3-7, 9, 10, 11a, 12a, 12b, 13, 17, 18, 20-22, 24, 25	506164	Folding Backrest Hardware PAIR Kit, Liberty <i>Kit Includes Hardware for Left and Right side.</i>	1
3-7, 9, 10, 11b, 12a, 12b 13, 17, 18, 20-22, 24, 25	506165	Folding Backrest Hardware PAIR Kit, Transit, Liberty <i>Kit Includes Hardware for Left and Right side.</i>	1
10, 13, 20, 21, 27a, 27b, 28, 29a	506163	Non-Folding Backrest Hardware PAIR Kit, Liberty <i>Kit Includes Hardware for Left and Right side.</i>	1

Item Number	Part Number	Description	Quantity
10, 13, 20, 21, 27a, 27b, 28, 29b	120794	Non-Folding Backrest Hardware PAIR Kit, Transit, Liberty <i>Kit Includes Hardware for Left and Right side.</i>	1
10, 11a, 12a, 12b 13, 17, 20, 21, 25	120795	Folding Backplates w/Hardware Liberty (Pair) <i>Kit Includes Hardware for Left and Right side.</i>	1
10, 11b, 12a, 12b 13, 17, 20, 21, 25	120796	Folding Backplates w/Hardware Transit Liberty (Pair) <i>Kit Includes Hardware for Left and Right side.</i>	1
10, 13, 20, 21, 27a, 27b, 29a	112924	Non-Folding Backplates w/Hardware Liberty (Pair) <i>Kit Includes Hardware for Left and Right side.</i>	1
10, 13, 20, 21, 27a, 27b, 29b	120797	Non-Folding Backplates w/Hardware Transit Liberty (Pair) <i>Kit Includes Hardware for Left and Right side.</i>	1
1	001083	Bushing Height Adjustable Back Tilt	1
2	001239	P-Clip 3/16"	1
3	001893	Latch Lever, Backrest	1
4, 7	503834	Cover Plate, Latch Insert Kit	1
5	001895	Latch Insert, Folding Backrest	1
6	001896	Spring, Backrest Latch	1
7	001897	Washer, Dual, PSA	2
8	109433	Ergo Push Handle Grip 7/8" Standard with Instruction	1
9	002513	Threaded Barrel, 5mm OD x M4	1
10	003284	Binding Post, 8mm x 40mm x M6	1
11a	003307	Back Plate, Outer, Folding	1
11b	506322	Transit Back Plate Folding, Liberty	1
12a	003308	Back Plate, Inner, Folding, Right	1
12b	003309	Back Plate, Inner, Folding, Left	1
13	003312	Saddle, Seat Tube	2
16, 26	112386	Threaded Insert, M6, with O-Ring, Liberty	1
17	100658	M6 Nylock Nut Blk Zn	8
18	100797	M6 x 40 BHCS BLK ZC	1
19	100803	Tube End Plug 7/8"	1

Item Number	Part Number	Description	Quantity
20	101184	M6 x 10 BTN SHCS BLKZC w/Patch	1
21	101781	M6 x 50mm Partial Thread BHCS Blk ZC	3
22	101818	M4 x 10 FHCS BLKZC w/Patch	2
23	101823	M6 X 20 Hhcs Blk Zn	1
24	102694	Dowel Pin, M5 X 24, Blk Zn	1
25	102803	Washer Flat .250 X .500 X .027 Hx Blz	2
27a	003305	Back Plate, Inner, Non-Folding, Right	1
27b	003306	Back Plate, Inner, Non-Folding, Left	1
28	003310	Receiver, Back Cane, Ht Adj	1
29a	003304	Back Plate, Outer, Non-Folding	1
29b	506564	Transit Back Plate Non-Folding, Liberty	1