# **HYGIENE AIDS**

Digi-Sert 30500, Supp-a-Sert 30600 Shur-Gryp Digi-Sert 30505, Shur-Gryp Supp-a-Sert 30605 E-Z Reach Digi-Sert 30511, E-Z Reach Supp-a-Sert 30611 E-Z Reach Digi-Supp Combo 30700

# HANDLE ADJUSTMENTS

#### Digi-Sert & Supp-a Sert:

- Determine whether the right or left hand will be used. Hold Digi-Sert with Delrin tip pointing upward and the longer part of the stainless steel handle (with the adjustment holes) facing away.
- If fitting right hand, the spanner bar attached to the stainless steel shaft should be to the left of the handle. If fitting left hand, it should be to the right.
- Loosen the bolts holding the spanner bar in place and move to desired position.
- 4. With tip still pointing upwards, place fingers between larger portion of loop and shaft; place thumb on other side of shaft.
- 5. To change handle size, remove the nut and bolt from backside of handle; align holes for larger or smaller handle loop size as required. Replace bolt and tighten cap nut. (**Note:** By aligning base of shaft with any of the various holes along the spanner bar, the angle can be changed to accommodate differing wrist functions.)



- Hold the unit with the stainless steel shaft facing user and probe or inserter facing upward.
   30505 & 30605
- 2. Place fingers through the loop formed by the hook-n-loop strap and plastic handpiece. The thumb should be in front of tube holding stainless steel shaft; plastic handpiece should fit securely in palm.
- To tighten and fasten to hand, pull the thumb loop at the end of the strap downward and press hook-n-loop strap together so it securely adheres to itself.
- If the molded plastic handpiece does not fit properly, soften plastic by heating with heat gun or hair dryer and bend into shape.
   Caution: Plastic will be hot.
- 5. The shaft angle can be rotated, shortened, or lengthened according to wrist function. To adjust the *length or rotation* of the shaft, loosen BOTH screws. To adjust the *angle* of the shaft, loosen the TOP screw and remove the BOTTOM screw, reposition shaft, and replace screw.





9450 W Laraway Rd • Frankfort, IL 60423 Customer 800-843-7234 • Fax 888-479-1515 www.Therafin.com • info@Therafin.com

### E-Z Reach Digi-Sert, E-Z Reach Supp-a-Sert & Combo:

- Release clear vinyl strap from stainless steel pin on base of handle.
   30511, 30611, & 30700
- Position unit so that gray handle fits into user's palm, with clear vinyl strap placed over back of hand.
- 3. To secure unit to hand, grasp thumb loop at bottom of clear vinyl strap; tug firmly until strap is **tight.** Insert metal tip on base of handle into hole position most appropriate for secure handling.
- 4. Units are shipped as right-handed models. To change for left-handed use, or to adjust, hold onto gray handle and pull clear vinyl hand strap first at the top and then at the bottom to twist into desired orientation.
- 5. The probe/inserter on the E-Z
  Reach series is curved. This curve
  can be repositioned to a users specific
  need by softening the bend with a heat
  gun, reshaping into desired position.
  Caution: Material will be hot.



## **USING**

30500 & 30600

#### **Digital Stimulators:**

Therafin recommends using these devices while in a lying position.

- Lie on side opposite your most maneuverable arm. Keep lower leg straight and upper leg flexed for better balance.
- Guide the digital stimulator (Digi-Sert, Shur-Gryp or E-Z Reach Digi-Sert) into rectum until the retaining washer is against buttocks. (Mirrors might prove to be helpful; available from Therafin). Agitate unit as needed, until stimulated.
- 3. By moving retaining washer into a different position on probe tip, depth of penetration can be adjusted for user preference.

#### **Suppository Inserters:**

Therafin recommends using these devices while in a lying position.

- Lie on side opposite your most maneuverable arm. Keep lower leg straight and upper leg flexed for better balance.
- Guide the loaded suppository inserter (Supp-a-Sert, Shur-Gryp Supp-a-Sert, or E-Z Reach Supp-a-Sert) gently into rectum. (Mirrors might prove to be useful; available from Therafin). The suppository will release when the retaining washer is pressed against buttocks.

# **CLEANING**

All parts are waterproof, except hook-n-loop on Shur-Gryp models. Clean and/or sterilize by washing in warm/hot water; use a mild detergent. Rinse thoroughly.