


# Rogue ALX / Little Wave XP Caster Forks and Stems (Formerly Tsunami)

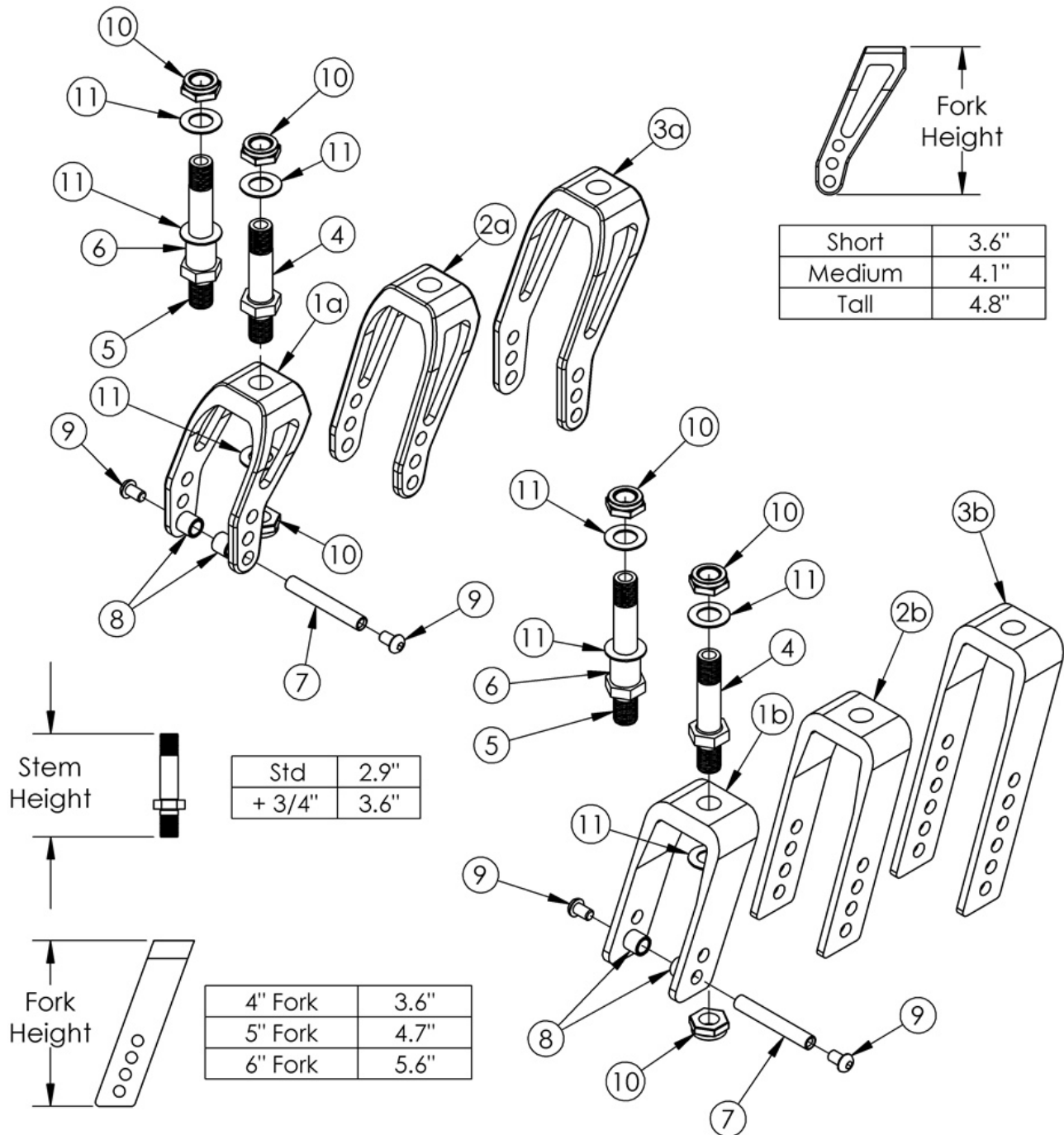

Item Number	Part Number	Description	Quantity
2a, 4, 10, 11	500788	Fork Medium Slotted Assembly Std Stem, RG ALX/LW XP	1

Item Number	Part Number	Description	Quantity
2a, 5, 6, 10, 11	500789	Fork Medium Slotted Assembly 3/4 Stem, RG ALX/LW XP	1
3a, 4, 10, 11	500790	Fork Tall Slotted Assembly Std Stem, RG ALX/LW XP	1
3a, 5, 6, 10, 11	500791	Fork Tall Slotted Assembly 3/4 Stem, RG ALX/LW XP	1
1b, 4, 10, 11	501469	Fork 4" Std Assembly Std Stem, RG ALX/LW XP	1
1b, 5, 6, 10, 11	501470	Fork 4" Std Assembly 3/4 Stem, RG ALX/LW XP	1
2b, 4, 10, 11	501471	Fork 5" Std Assembly Std Stem, RG ALX/LW XP	1
2b, 5, 6, 10, 11	501472	Fork 5" Std Assembly 3/4 Stem, RG ALX/LW XP	1
3b, 4, 10, 11	501473	Fork 6" Std Assembly Std Stem, RG ALX/LW XP	1
3b, 5, 6, 10, 11	501474	Fork 6" Std Assembly 3/4 Stem, RG ALX/LW XP	1
1b, 10	504447	Caster Fork 4", RG ALX/LW XP	1
2a, 10	504439	Fork Slotted Medium, RG ALX/LW XP	1
2b, 10	504448	Caster Fork 5", RG ALX/LW XP	1
3a, 10	504440	Fork Slotted Tall, RG ALX/LW XP	1
3b, 10	504449	Caster Fork 6", RG ALX/LW XP	1
4, 10	000424INST	Caster Stem Standard, RG ALX/LW XP	1
5, 10	000425INST	Caster Stem + 3/4", RG ALX/LW XP	1
7-9	116069	Caster Attachment Hardware Kit, Rigid	1
6	000115	Fork Stem Spacer 3/4"	1
7	000422	M6 x 56 Threaded Barrel	1
8	100792	Caster Spacer D8 x D12 x 10mm / Close Mount Handrim Spacer	2
9	100667	M6 x 12 BTN SHCS BLKZC w/Patch	2
10	101459	1/2-20 NTE JAM NYL INS L/N BLK ZC	2
11	100682	Washer 1/2" .518 x .875 x .047 F/W Black Zinc <i>Standard stem assemblies use quantity 2. 3/4" stem assemblies use quantity 3.</i>	3