

LATCH HANDLE REPLACEMENT INSTRUCTIONS ZIPPIE X'CAPE



Please read these instructions carefully before beginning the installation. Failure to understand and follow installation instructions may result in injury to installer and/or end user and may void the warranty. If you have any questions call Sunrise Medical Technical support at 800-333-4000.

Latch handle parts:

- 1. 1 ea Latch Handle
- 2. 2 ea Retaining Clip
- 3. 1 ea E-clip

Tools needed for assembly:

- 1. Flat head screwdriver
- 2. 2 ea 10mm wrench

A. PRE-INSTALLATION


1. Prep the chair:


- **a.** Separate the push to lock loop from the latch handle, Fig. 1.
- b. Note which holes the cross tubes are mounted to in the side frame. Using the 10mm wrenches, remove the 4 screws holding the cross tubes to the side frame, Fig. 2 & 5. Repeat on the opposite side. Save hardware for later.
- **c.** Remove retaining clips (1) and e-clip (2) from pins (3 & 4), Fig. 3. Discard clips, save pins.
- d. Discard latch handle.


B. INSTALLATON


1. Assembly of new latch handle:


- a. Install new handle provided using saved pins from step 1c and new clips.
- b. With the cross frame in the unfolded position, adjust the set screw (5) using a flat head screwdriver so that the spring plunger tip makes contact with the top of the cross tube. After contact is made turn the set screw clockwise an additional 1/2 rotation, Fig. 4 & 4a.
- **c.** Install the cross tubes back onto the side frame using saved hardware from step 1b. Ensure that cross tubes are mounted to same holes noted in step 1b. Repeat on opposite side. Tighten hardware to 120 in-lbs, Fig. 5.
- d. Re-attach the push to lock loop onto the new latch handle, Fig 1.

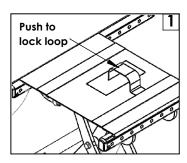


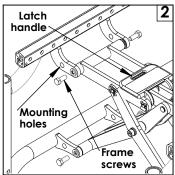

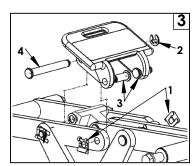

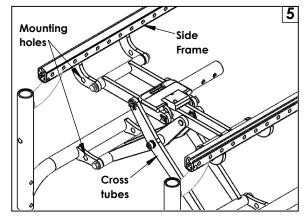

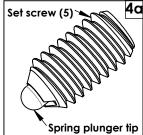

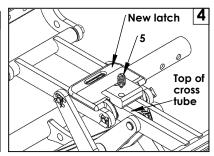

Sunrise Medical 2842 Business Park Ave Fresno, CA 93727 USA In Canada (800) 263-3390

