


Catalyst E Aluminum Cross Braces

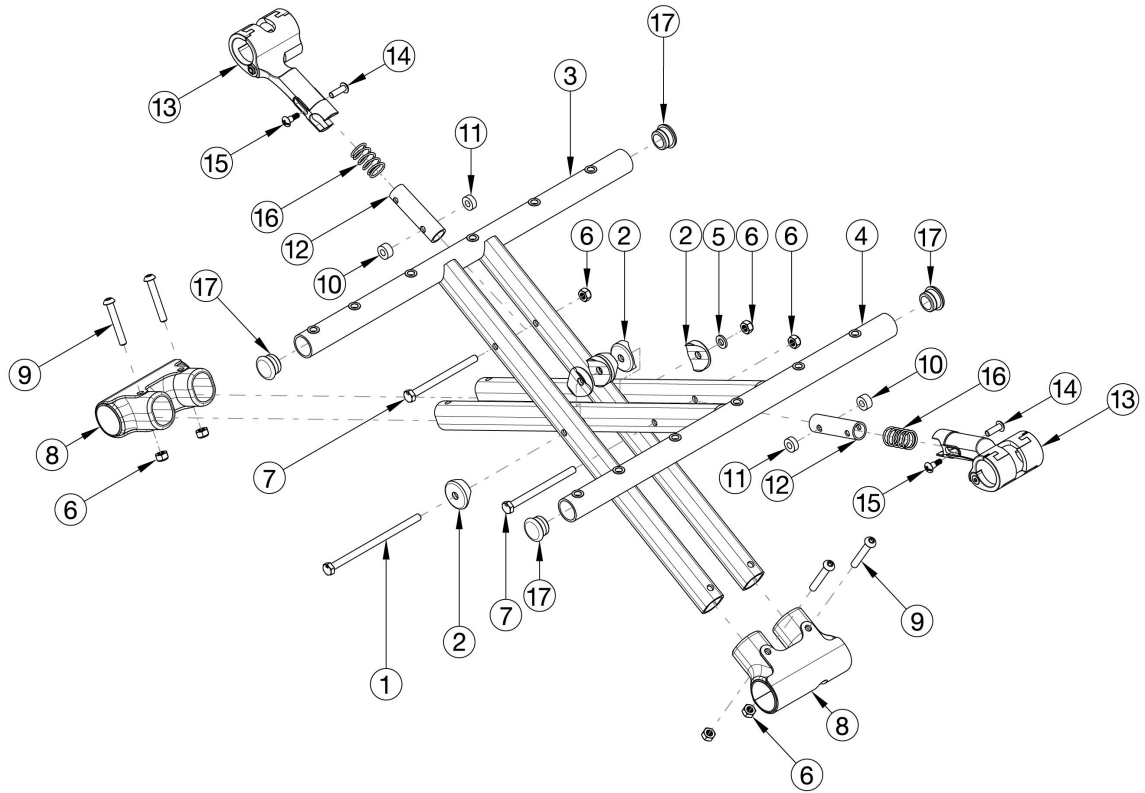

Cross Brace Overall Width	
Seat Width	Cross Brace Measurement
14"-16"	Approx. 1" less than seat width
17"	Approx. 1.25" less than seat width
18"-21"	Approx. 1.5" less than seat width
22"	Approx. 1.75" less than seat width

Cross Brace Overall Depth	
Seat Depth	Cross Brace Measurement
14"-20"	Approx. 0.375" less than seat depth

Item Number	Part Number	Description	Quantity	Retail
12a, 13 - 16	115770	Upper Pivot Assembly, 14" Wide	1	\$27.85
12b, 13-16	115771	Upper Pivot Assembly, 16" Wide	1	\$27.85
12c, 13-16	115772	Upper Pivot Assembly, 18" Wide	1	\$27.85
12d, 13-16	115773	Upper Pivot Assembly, 20" Wide	1	\$27.85

Item Number	Part Number	Description	Quantity	Retail
12e, 13-16	115774	Upper Pivot Assembly, 22" Wide	1	\$43.49
6, 8, 9	115775	Lower Pivot Assembly	1	\$11.25
1 - 2, 5 - 7, 10 - 11	115776	Cross Brace Hardware Kit	1	\$27.64
3a, 17	115791	Cross Brace, Right, 14x16	1	\$175.63
4a, 17	115777	Cross Brace, Left, 14x16	1	\$175.63
3b, 17	115792	Cross Brace, Right, 14x18	1	\$175.63
4b, 17	115778	Cross Brace, Left, 14x18	1	\$175.63
3c, 17	115793	Cross Brace, Right, 16x16	1	\$175.63
4c, 17	115779	Cross Brace, Left, 16x16	1	\$175.63
3d, 17	115794	Cross Brace, Right, 16x18	1	\$175.63
4d, 17	115780	Cross Brace, Left, 16x18	1	\$175.63
3e, 17	115795	Cross Brace, Right, 16x20	1	\$175.63
4e, 17	115781	Cross Brace, Left, 16x20	1	\$175.63
3f, 17	115796	Cross Brace, Right, 18x16	1	\$175.63
4f, 17	115782	Cross Brace, Left, 18x16	1	\$175.63
3g, 17	115797	Cross Brace, Right, 18x18	1	\$175.63
4g, 17	115783	Cross Brace, Left, 18x18	1	\$175.63
3h, 17	115798	Cross Brace, Right, 18x20	1	\$175.63
4h, 17	115784	Cross Brace, Left, 18x20	1	\$175.63
3i, 17	115799	Cross Brace, Right, 20x16	1	\$175.63
4i, 17	115785	Cross Brace, Left, 20x16	1	\$175.63
3j, 17	115800	Cross Brace, Right, 20x18	1	\$175.63
4j, 17	115786	Cross Brace, Left, 20x18	1	\$175.63
3k, 17	115801	Cross Brace, Right, 20x20	1	\$175.63
4k, 17	115787	Cross Brace, Left, 20x20	1	\$175.63
3l, 17	115802	Cross Brace, Right, 22x16	1	\$319.53
4l, 17	115788	Cross Brace, Left, 22x16	1	\$319.53
3m, 17	115803	Cross Brace, Right, 22x18	1	\$319.53

Item Number	Part Number	Description	Quantity	Retail
4m, 17	115789	Cross Brace, Left, 22x18	1	\$319.53
3n, 17	115804	Cross Brace, Right, 22x20	1	\$319.53
4n, 17	115790	Cross Brace, Left, 22x20	1	\$319.53
1	100678	M6 x 110 CLS8.8 DIN931 HHCS ZC	1	\$2.76
2	002807	Cross Tube Saddle CAT	6	\$1.66
5	100746	M6 Flat Washer Blk Zn	1	\$1.66
7	100660	M6 x 80mm CLS8.8 DIN931 HHCS ZC	2	\$1.66
10	100880	1/4" Plastic Spacer	2	\$2.76
11	105019	3/16" Plastic Spacer	2	\$0.56