


Rigid Height Adjustable T-Arm

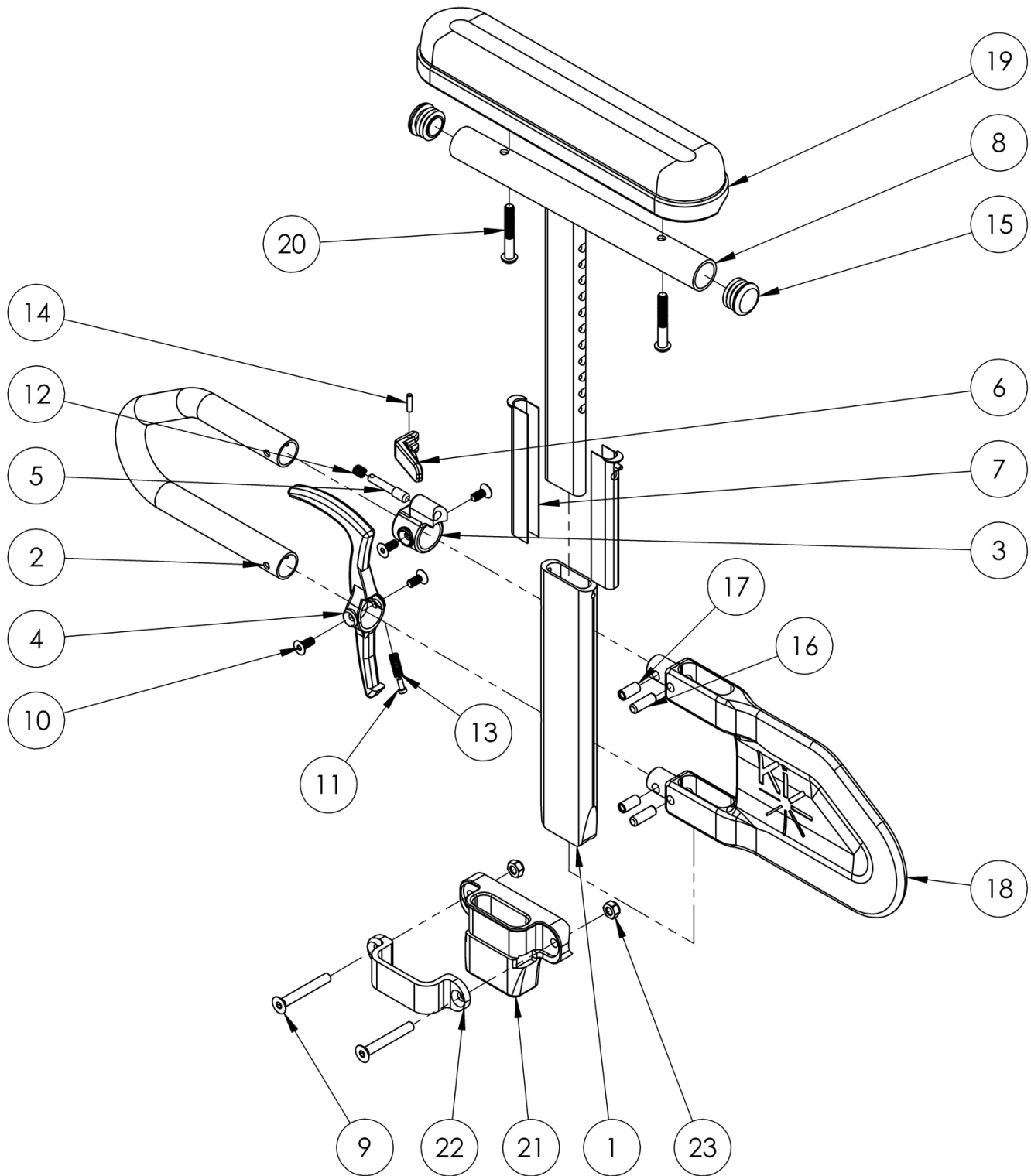

This design effective December 6, 2017 Height Adjustable T-Arms cannot be retro-fitted

Item Number	Part Number	Description	Quantity
1-7, 8a, 10-18, 19b, 20	505646	Height Adjustable T-Arm Assembly w/ Short Side Shield and Standard Pad Desk	1
1-7, 8b, 10-18, 19a, 20	505644	Height Adjustable T-Arm Assembly w/ Short Side Shield and Standard Pad Full	1
1-7, 8a, 10-18, 19b, 20	505647	Height Adjustable T-Arm Assembly w/ Short Side Shield and Standard Pad Desk, Pair	1
1-7, 8b, 10-18, 19a, 20	505645	Height Adjustable T-Arm Assembly w/ Short Side Shield and Standard Pad Full, Pair	1
1-7, 8a, 9-18, 19b, 20-23	506081	Height Adj T-Arm Assm w/ Short Side Shield and Standard Pad Desk, 1-3/8" Receiver, RG/RG ALX/LW XP	1
1-7, 8b, 9-18, 19a, 20-23	506093	Height Adj T-Arm Assm w/ Short Side Shield and Standard Pad Full, 1-3/8" Receiver, RG/RG ALX/LW XP	1
1-7, 8a, 9-18, 19b, 20-23	506091	Height Adj T-Arm Assm w/ Short Side Shield and Standard Pad Desk, 1-3/8" Receiver, RG/RG ALX/LW XP, PAIR	1
1-7, 8b, 9-18, 19a, 20-23	506092	Height Adj T-Arm Assm w/ Short Side Shield and Standard Pad Full, 1-3/8" Receiver, RG/RG ALX/LW XP, PAIR	1
3, 5, 6, 10, 12, 14	505490	Height Adjustable T-Arm Upper Latch Assembly	1
4, 10, 11, 13	505507	Height Adjustable T-Arm Release Lever Assembly	1
9, 21-23	501731	Height Adj T-Arm Receiver Assembly 1-3/8", RG ALX/LW XP	1
16-18	506082	Height Adjustable T-Arm Short Shield Assembly	1
1-7, 10-14, 16-18	004160	Height Adjustable T-Arm Lower Assembly Short <i>Does not include t-arm upper, arm pad or arm pad screws</i>	1
1-7, 8a, 10-18	003463	Height Adjustable T-Arm Short Desk <i>Does not include arm pad or arm pad screws</i>	1
1-7, 8b, 10-18	003464	Height Adjustable T-Arm Short Full <i>Does not include arm pad or arm pad screws</i>	1
1	000089	Height Adjustable T Armrest Post	1
2	000095	Height Adjustable T Armrest Transfer Tube	1
7	000090	Height Adjustable T Armrest Shim	2
8a, 15	112906	Height Adjustable T Armrest Upper Desk w/ Tube End Plug	1
8b, 15	112907	Height Adjustable T Armrest Upper Full w/ Tube End Plug	1
9	101130	M6 x 55 FSHCS P/THD BLKZC	2
10	003458	M5 X 12mm FHCS BLK ZN W/ Patch	4
11	000152	Height Adjustable T Armrest Release Spring Button	1

Item Number	Part Number	Description	Quantity
13	000135	Height Adjustable T Armrest Release Lever Spring	1
15	100803	Tube End Plug 7/8"	2
16	003641	.25" x .75" Dowel Pin	2
17	003457	M5 x 14mm Barrel Nut	2
19a	100500	Arm Pad Full	1
19b	100501	Arm Pad Desk	1
20	100696	M5 x 30mm PPH Capscrew	2
21	004578	Height Adjustable T-Arm Receiver 1.375"	1
22	000088	Height Adjustable T Armrest Bracket	1
23	100658	M6 Nylock Nut Blk Zn	2