


## CR45 Footplates - Angle Adjustable Extension Mount

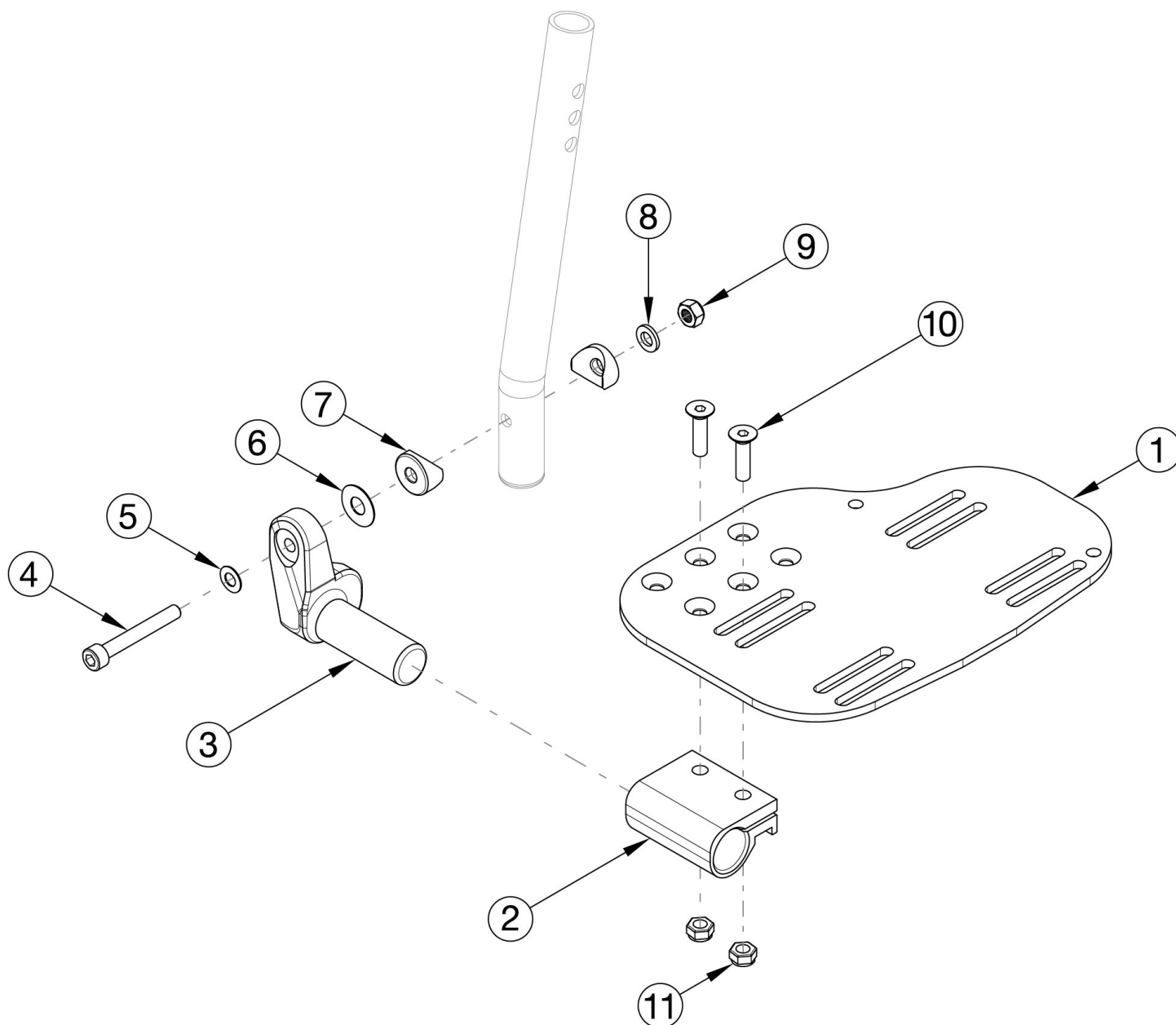

Sizes for footplates/assemblies are based on seat width. If changing footplate style, new extension tubes may be needed.

| Item Number     | Part Number | Description   | Quantity |
|-----------------|-------------|---|----------|
| 2, 3b, 4-11     | 502202      | Footplate Hardware Assembly Angle Adjustable Extension Mount/ELR Left         | 1        |
| 2, 3a, 5-12     | 502203      | Footplate Hardware Assembly Angle Adjustable Extension Mount/ELR Right        | 1        |
| 2, 3a, 3b, 4-11 | 502204      | Footplate Hardware Assembly Angle Adjustable Extension Mount/ELR Pair         | 1        |
| 1a, 2, 3b, 4-11 | 100252      | Footplate Angle Adjustable 14-15 Wide Extension Mount Left/ELR Right Assembly | 1        |
| 1b, 2, 3a, 4-11 | 100253      | Footplate Angle Adjustable 14-15 Wide Extension Mount Right/ELR Left Assembly | 1        |

| Item Number             | Part Number | Description   | Quantity |
|-------------------------|-------------|---|----------|
| 1a, 1b, 2, 3a, 3b, 4-11 | 100254      | Footplate Angle Adjustable 14-15 Wide Extension Mount/ELR PAIR Assembly   | 1        |
| 1c, 2, 3b, 4-11         | 100255      | Footplate Angle Adjustable 16-17 Wide Extension Mount Left/ELR Right Assembly   | 1        |
| 1d, 2, 3a, 4-11         | 100256      | Footplate Angle Adjustable 16-17 Wide Extension Mount Right/ELR Left Assembly   | 1        |
| 1c, 1d, 2, 3a, 3b, 4-11 | 100257      | Footplate Angle Adjustable 16-17 Wide Extension Mount/ELR PAIR Assembly   | 1        |
| 1e, 2, 3b, 4-11         | 100258      | Footplate Angle Adjustable 18-24 Wide Extension Mount Left/ELR Right Assembly   | 1        |
| 1f, 2, 3a, 4-11         | 100259      | Footplate Angle Adjustable 18-24 Wide Extension Mount Right/ELR Left Assembly   | 1        |
| 1e, 1f, 2, 3a, 3b, 4-11 | 100260      | Footplate Angle Adjustable 18-24 Wide Extension Mount/ELR PAIR Assembly   | 1        |
| 1a, 2, 3b, 4-11         | 502184      | Footplate Kit Angle Adjustable w/ Heel Loop for Ext. Mount Hanger 14-15 Wide Left<br><i>Heel Loop with attaching hardware is included but not pictured</i>  | 1        |
| 1b, 2, 3a, 4-11         | 502185      | Footplate Kit Angle Adjustable w/ Heel Loop for Ext. Mount Hanger 14-15 Wide Right<br><i>Heel Loop with attaching hardware is included but not pictured</i> | 1        |
| 1a, 1b, 2, 3a, 3b, 4-11 | 502186      | Footplate Kit Angle Adjustable w/ Heel Loop for Ext. Mount Hanger 14-15 Wide Pair<br><i>Heel Loop with attaching hardware is included but not pictured</i>  | 1        |
| 1c, 2, 3b, 4-11         | 502187      | Footplate Kit Angle Adjustable w/ Heel Loop for Ext. Mount Hanger 16-17 Wide Left<br><i>Heel Loop with attaching hardware is included but not pictured</i>  | 1        |
| 1d, 2, 3a, 4-11         | 502188      | Footplate Kit Angle Adjustable w/ Heel Loop for Ext. Mount Hanger 16-17 Wide Right<br><i>Heel Loop with attaching hardware is included but not pictured</i> | 1        |
| 1c, 1d, 2, 3a, 3b, 4-11 | 502189      | Footplate Kit Angle Adjustable w/ Heel Loop for Ext. Mount Hanger 16-17 Wide Pair<br><i>Heel Loop with attaching hardware is included but not pictured</i>  | 1        |
| 1e, 2, 3b, 4-11         | 502190      | Footplate Kit Angle Adjustable w/ Heel Loop for Ext. Mount Hanger 18-24 Wide Left<br><i>Heel Loop with attaching hardware is included but not pictured</i>  | 1        |
| 1f, 2, 3a, 4-11         | 502191      | Footplate Kit Angle Adjustable w/ Heel Loop for Ext. Mount Hanger 18-24 Wide Right<br><i>Heel Loop with attaching hardware is included but not pictured</i> | 1        |
| 1e, 1f, 2, 3a, 3b, 4-11 | 502192      | Footplate Kit Angle Adjustable w/ Heel Loop for Ext. Mount Hanger 18-24 Wide Pair<br><i>Heel Loop with attaching hardware is included but not pictured</i>  | 1        |
| 1a                      | 000105      | Footplate Angle Adjustable 14-15" Wide CAT/TILT Left (Std ELR Right)  | 1        |
| 1b                      | 000106      | Footplate Angle Adjustable 14-15" Wide CAT/TILT Right (Std ELR Left)  | 1        |
| 1c                      | 000107      | Footplate Angle Adjustable 16-17" Wide CAT/TILT Left (Std ELR Right)  | 1        |

| Item Number | Part Number | Description  | Quantity |
|-------------|-------------|--|----------|
| 1d          | 000108      | Footplate Angle Adjustable 16-17" Wide CAT/TILT Right (Std ELR Left) | 1        |
| 1e          | 000109      | Footplate Angle Adjustable 18-24" Wide CAT/TILT Left (Std ELR Right) | 1        |
| 1f          | 000110      | Footplate Angle Adjustable 18-24" Wide CAT/TILT Right (Std ELR Left) | 1        |
| 2           | 100502      | Angle Adj Footplate Clamp 2"   | 1        |
| 3a          | 000047      | Footplate Angle Adjustable Casting 3/4 Right                         | 1        |
| 3b          | 000049      | Footplate Angle Adjustable Casting 3/4" Left                         | 1        |
| 4           | 101472      | M6 x 45 SHCS BLK ZC  | 1        |
| 5           | 101455      | M6 Belleville Washer (.252 x .551 x .050)                            | 1        |
| 6           | 000428      | Wear Washer  | 1        |
| 7           | 000053      | Saddle 3/4"  | 2        |
| 8           | 100746      | M6 Flat Washer Blk Zn  | 1        |
| 9           | 101456      | M6 Ovl Top L/N ZC  | 1        |
| 10          | 100662      | M6 x 25 FSHCS BLK ZC   | 2        |
| 11          | 100658      | M6 Nylock Nut Blk Zn   | 2        |