



# Clik Caster Mount

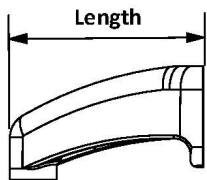

Caster Mount Arm Length	
Arm Size	Length
Short	2.8"
Medium	3.8"
Long	5"

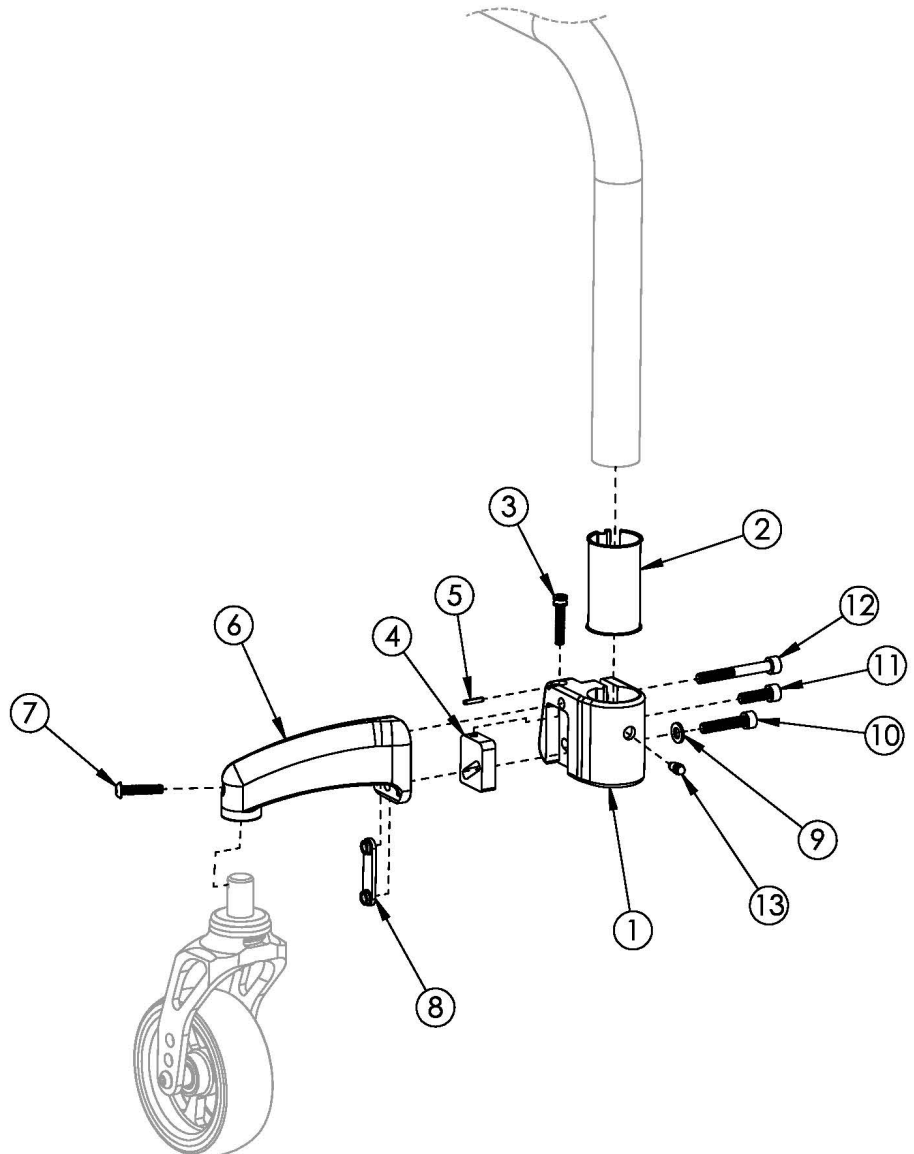

If replacing a forward facing wing, quote opposite caster arm. <p>

The caster mount is an accent color option. When a chair is originally ordered this part gets painted according to the accent color selection and not the frame color selection.

Item Number	Part Number	Description	Quantity	Retail
1a, 2, 3, 4, 5, 7, 13	501222	Caster Arm Mount Assembly Clik Right	1	\$94.92
1b, 2, 3, 4, 5, 7, 13	501223	Caster Arm Mount Assembly Clik Left	1	\$94.92
1a, 2, 3, 4, 5, 6a, 7-13	502524_ _ _	Caster Mount Assembly, Short, Right Clik	1	\$174.12
1b, 2, 3, 4, 5, 6b, 7-13	502525_ _ _	Caster Mount Assembly, Short, Left, Clik	1	\$174.12

Item Number	Part Number	Description	Quantity	Retail
1a, 1b, 2, 3, 4, 5, 6a, 6b, 7-13	502526_ _ _	Caster Mount Assembly, Short, Klik, PAIR	1	\$348.25
1a, 2, 3, 4, 5, 6c, 7-13	502527_ _ _	Caster Mount Assembly, Medium, Right, Klik	1	\$169.05 - \$174.12
1b, 2, 3, 4, 5, 6d, 7-13	502528_ _ _	Caster Mount Assembly, Medium, Left, Klik	1	\$169.05 - \$174.12
1a, 1b, 2, 3, 4, 5, 6c, 6d, 7-13	502529_ _ _	Caster Mount Assembly, Medium, Klik, PAIR	1	\$338.11 - \$348.25
1a, 2, 3, 4, 5, 6e, 7-13	502530_ _ _	Caster Mount Assembly, Long, Right, Klik	1	\$174.12
1b, 2, 3, 4, 5, 6f, 7-13	502531_ _ _	Caster Mount Assembly, Long, Left, Klik	1	\$174.12
1a, 1b, 2, 3, 4, 5, 6e, 6f, 7-13	502532_ _ _	Caster Mount Assembly, Long, Klik, PAIR	1	\$348.25
2	002079	Caster Mount Sleeve LW	1	\$1.61
6a, 7	002561_ _ _	Caster Arm Short Right Klik <i>Used with 3" casters or with 4" casters and a high mounted footplate</i>	1	\$48.25
6b, 7	002562_ _ _	Caster Arm Short Left Klik <i>Used with 3" casters or with 4" casters and a high mounted footplate</i>	1	\$48.25
6c, 7	002069_ _ _	Caster Arm Medium Right Klik <i>Used with 5" casters or with 4" casters and a frame inserted footplate or with 6" casters and a high mounted footplate</i>	1	\$46.84 - \$48.25
6d, 7	002070_ _ _	Caster Arm Medium Left Klik <i>Used with 5" casters or with 4" casters and a frame inserted footplate or with 6" casters and a high mounted footplate</i>	1	\$46.84 - \$48.25
6e, 7	002071_ _ _	Caster Arm Long Right Klik <i>Used with 6" casters and a frame inserted footplate</i>	1	\$48.25
6f, 7	002072_ _ _	Caster Arm Long Left Klik <i>Used with 6" casters and a frame inserted footplate</i>	1	\$48.25
7	102616	M5 X 25 BHCS BLK ZC	1	\$1.05
8	002462	M6 X 1.0 Dual Hole Nut	1	\$24.65
9	100746	M6 Flat Washer Blk Zn	1	\$1.58
10	102627	M6 X 30 SHCS BLK ZC	1	\$1.05
11	101844	M6 x 18 SHCS Blk Zn	1	\$1.05
12	101472	M6 x 45 SHCS BLK ZC	1	\$1.58

Matte Finish	
Code	Color Name
CGM	Charcoal Gray
EGM	Emerald Green
GGM	Graphite Gray*
IBM	Iridescent Burgundy
INM	Iridescent Navy
AGM	Military Green**
RRM	Retro Red
RBM	Royal Blue
SBM	Shadow Black
SWM	Smooth White

\* Discontinued Color

\*\* Formerly called Army Green

Gloss Finish	
Code	Color Name
BLK	Black
BLU	Blue
SKY	Blue Skies
BRO	Bright Orange*
PNK	Bubblegum Pink
BUR	Burgundy*
CBL	Candy Blue
PUR	Candy Purple
RED	Candy Red
COP	Copper
DPR	Deep Purple*
EBL	Electric Blue*
GRN	Forest Green*
FUS	Fuchsia*
GSM	Granny Smith
MTF	Metallic Fuchsia*
MOR	Metallic Orange
ORG	Orange*
SAP	Sapphire*
SIL	Silver
TEA	Teal
YLW	Yellow
FLM	Graphic Image Transfer