
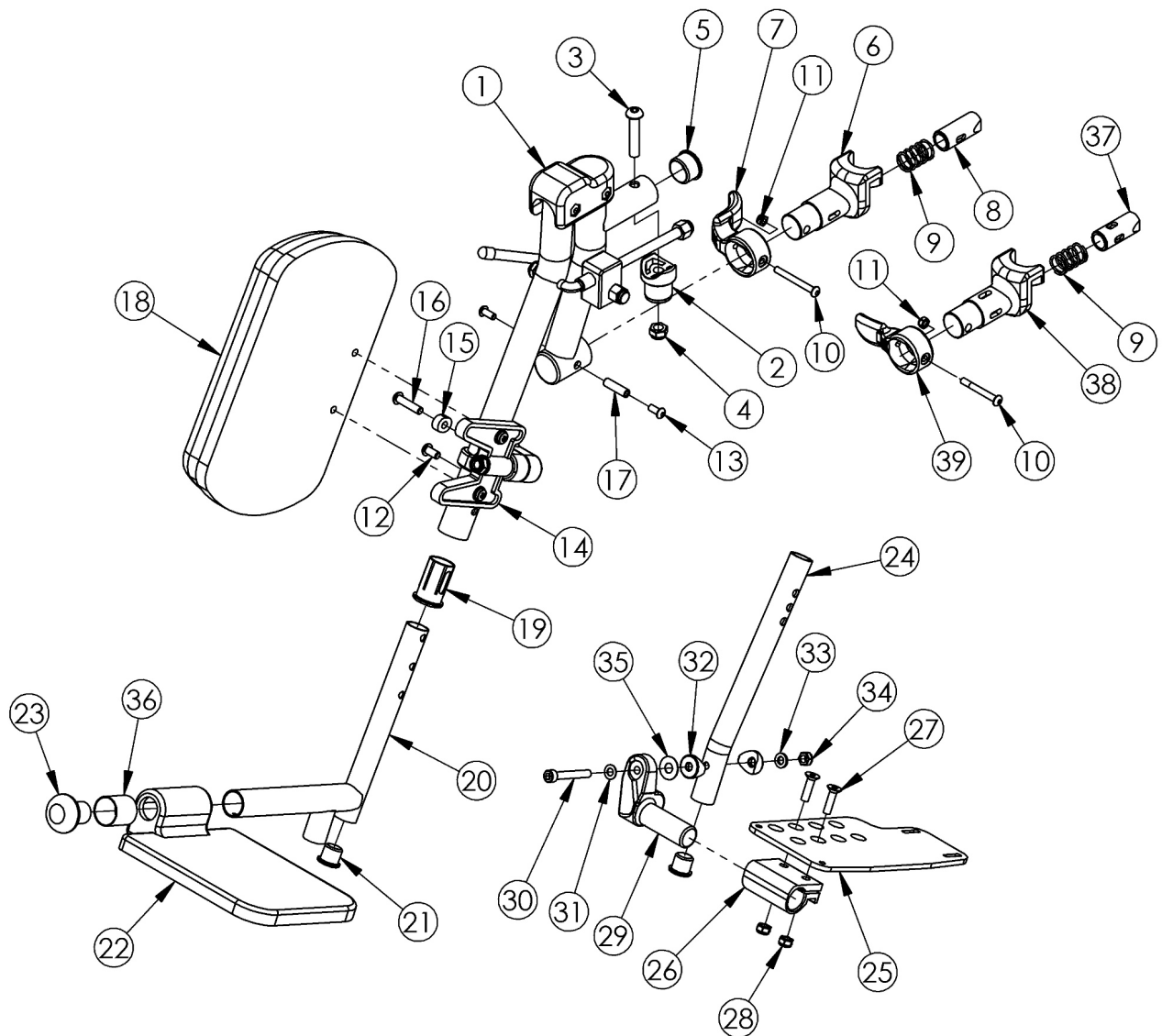


AADL Catalyst Elevating Leg Rest

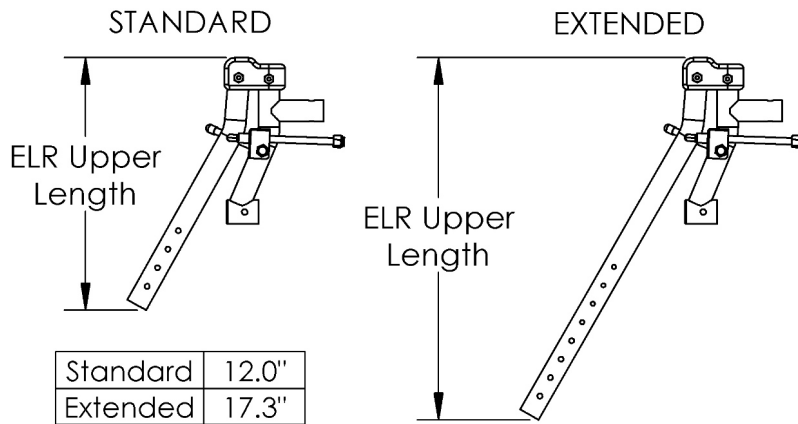

Small composite footplates are used on 14-16" seat widths. Large composite footplates are used on 17-22" seat widths Sizes for assemblies are based on seat width For a complete Elevating Leg Rest, select an upper assembly and the desired extension tube and footplate assembly

Item Number	Part Number	Description	Quantity
1a, 1b, 2-17, 18a, 19, 20b, 21, 22b, 23	500556	ELR Assembly 17-22 Large Composite Footplate Long Ext, Classic Release, PAIR	1
1a, 2-17, 18a, 19, 20b, 21, 22b, 23	500563	ELR Assembly 17-22 Large Composite Footplate Long Ext, Classic Release, Right	1
1b, 2-17, 18a, 19, 20a, 21, 22b, 23	500568	ELR Assembly 17-22 Large Composite Footplate Short Ext, Classic Release, Left	1
1b, 2-17, 18a, 19, 20b, 21, 22b, 23	500569	ELR Assembly 17-22 Large Composite Footplate Long Ext, Classic Release, Left	1
1a, 1b, 2-17, 18b, 19, 21, 24b, 25a, 25b, 26-35	501040	ELR Assembly 14 Angle Adj Footplate Long Ext, Classic Release, PAIR	1
1a, 1b, 2-17, 18b, 19, 21, 24a, 25a, 25b, 26-35	501041	ELR Assembly 14 Angle Adj Footplate Short Ext, Classic Release, PAIR	1
1a, 1b, 2-17, 18a, 19, 21, 24b, 25a, 25b, 26-35	500570	ELR Assembly 15 Angle Adj Footplate Long Ext, Classic Release, PAIR	1
1a, 1b, 2-17, 18a, 19, 21, 24a, 25a, 25b, 26-35	500571	ELR Assembly 15 Angle Adj Footplate Short Ext, Classic Release, PAIR	1
1a, 1b, 2-17, 18a, 19, 21, 24b, 25c, 25d, 26-35	500572	ELR Assembly 16-17 Angle Adj Footplate Long Ext, Classic Release, PAIR	1
1a, 1b, 2-17, 18a, 19, 21, 24a, 25c, 25d, 26-35	500573	ELR Assembly 16-17 Angle Adj Footplate Short Ext, Classic Release, PAIR	1
1a, 1b, 2-17, 18a, 19, 21, 24b, 25e, 25f, 26-35	500574	ELR Assembly 18-22 Angle Adj Footplate Long Ext, Classic Release, PAIR	1
1a, 1b, 2-17, 18a, 19, 21, 24a, 25e, 25f, 26-35	500575	ELR Assembly 18-22 Angle Adj Footplate Short Ext, Classic Release, PAIR	1
1a, 2-17, 18b, 19, 21, 24b, 25a, 26-28, 29a, 30-35	501042	ELR Assembly 14 Angle Adj Footplate Long Ext, Classic Release, Right	1
1a, 2-17, 18b, 19, 21, 24a, 25a, 26-28, 29a, 30-35	501043	ELR Assembly 14 Angle Adj Footplate Short Ext, Classic Release, Right	1

Item Number	Part Number	Description	Quantity
1a, 2-17, 18a, 19, 21, 24b, 25a, 26-28, 29a, 30-35	500576	ELR Assembly 15 Angle Adj Footplate Long Ext, Classic Release, Right	1
1a, 2-17, 18a, 19, 21, 24a, 25a, 26-28, 29a, 30-35	500577	ELR Assembly 15 Angle Adj Footplate Short Ext, Classic Release, Right	1
1a, 2-17, 18a, 19, 21, 24b, 25c, 26-28, 29a, 30-35	500578	ELR Assembly 16-17 Angle Adj Footplate Long Ext, Classic Release, Right	1
1a, 2-17, 18a, 19, 21, 24a, 25c, 26-28, 29a, 30-35	500579	ELR Assembly 16-17 Angle Adj Footplate Short Ext, Classic Release, Right	1
1a, 2-17, 18a, 19, 21, 24b, 25e, 26-28, 29a, 30-35	500580	ELR Assembly 18-22 Angle Adj Footplate Long Ext, Classic Release, Right	1
1a, 2-17, 18a, 19, 21, 24a, 25e, 26-28, 29a, 30-35	500581	ELR Assembly 18-22 Angle Adj Footplate Short Ext, Classic Release, Right	1
1b, 2-17, 18b, 19, 21, 24b, 25b, 26-28, 29b, 30-35	501044	ELR Assembly 14 Angle Adj Footplate Long Ext, Classic Release, Left	1
1b, 2-17, 18b, 19, 21, 24a, 25b, 26-28, 29b, 30-35	501045	ELR Assembly 14 Angle Adj Footplate Short Ext, Classic Release, Left	1
1b, 2-17, 18a, 19, 21, 24b, 25b, 26-28, 29b, 30-35	500582	ELR Assembly 15 Angle Adj Footplate Long Ext, Classic Release, Left	1
1b, 2-17, 18a, 19, 21, 24a, 25b, 26-28, 29b, 30-35	500583	ELR Assembly 15 Angle Adj Footplate Short Ext, Classic Release, Left	1
1b, 2-17, 18a, 19, 21, 24b, 25d, 26-28, 29b, 30-35	500584	ELR Assembly 16-17 Angle Adj Footplate Long Ext, Classic Release, Left	1
1b, 2-17, 18a, 19, 21, 24a, 25d, 26-28, 29b, 30-35	500585	ELR Assembly 16-17 Angle Adj Footplate Short Ext, Classic Release, Left	1
1b, 2-17, 18a, 19, 21, 24b, 25f, 26-28, 29b, 30-35	500586	ELR Assembly 18-22 Angle Adj Footplate Long Ext, Classic Release, Left	1
1b, 2-17, 18a, 19, 21, 24a, 25f, 26-28, 29b, 30-35	500587	ELR Assembly 18-22 Angle Adj Footplate Short Ext, Classic Release, Left	1

Item Number	Part Number	Description	Quantity
1c, 1d, 2-17, 18a, 19, 20b, 21, 22b, 23	501563	Extended Length ELR Assembly 17-22 Large Composite Footplate Long Ext, Classic Release, PAIR	1
1c, 2-17, 18a, 19, 20b, 21, 22b, 23	501570	Extended Length ELR Assembly 17-22 Large Composite Footplate Long Ext, Classic Release, Right	1
1d, 2-17, 18a, 19, 20b, 21, 22b, 23	501576	Extended Length ELR Assembly 17-22 Large Composite Footplate Long Ext, Classic Release, Left	1
1c, 1d, 2-17, 18b, 19, 21, 24b, 25a, 25b, 26-35	501577	Extended Length ELR Assembly 14 Angle Adj Footplate Long Ext, Classic Release, PAIR	1
1c, 1d, 2-17, 18b, 19, 21, 24a, 25a, 25b, 26-35	501578	Extended Length ELR Assembly 14 Angle Adj Footplate Short Ext, Classic Release, PAIR	1
1c, 1d, 2-17, 18a, 19, 21, 24b, 25a, 25b, 26-35	501579	Extended Length ELR Assembly 15 Angle Adj Footplate Long Ext, Classic Release, PAIR	1
1c, 1d, 2-17, 18a, 19, 21, 24a, 25a, 25b, 26-35	501580	Extended Length ELR Assembly 15 Angle Adj Footplate Short Ext, Classic Release, PAIR	1
1c, 1d, 2-17, 18a, 19, 21, 24b, 25c, 25d, 26-35	501581	Extended Length ELR Assembly 16-17 Angle Adj Footplate Long Ext, Classic Release, PAIR	1
1c, 1d, 2-17, 18a, 19, 21, 24a, 25c, 25d, 26-35	501582	Extended Length ELR Assembly 16-17 Angle Adj Footplate Short Ext, Classic Release, PAIR	1
1c, 1d, 2-17, 18a, 19, 21, 24b, 25e, 25f, 26-35	501583	Extended Length ELR Assembly 18-22 Angle Adj Footplate Long Ext, Classic Release, PAIR	1
1c, 1d, 2-17, 18a, 19, 21, 24a, 25e, 25f, 26-35	501584	Extended Length ELR Assembly 18-22 Angle Adj Footplate Short Ext, Classic Release, PAIR	1
1c, 2-17, 18b, 19, 21, 24b, 25a, 26-28, 29a, 30-35	501585	Extended Length ELR Assembly 14 Angle Adj Footplate Long Ext, Classic Release, Right	1
1c, 2-17, 18b, 19, 21, 24a, 25a, 26-28, 29a, 30-35	501586	Extended Length ELR Assembly 14 Angle Adj Footplate Short Ext, Classic Release, Right	1
1c, 2-17, 18a, 19, 21, 24b, 25a, 26-28, 29a, 30-35	501587	Extended Length ELR Assembly 15 Angle Adj Footplate Long Ext, Classic Release, Right	1
1c, 2-17, 18a, 19, 21, 24a, 25a, 26-28, 29a, 30-35	501588	Extended Length ELR Assembly 15 Angle Adj Footplate Short Ext, Classic Release, Right	1

Item Number	Part Number	Description	Quantity
1c, 2-17, 18a, 19, 21, 24b, 25c, 26-28, 29a, 30-35	501589	Extended Length ELR Assembly 16-17 Angle Adj Footplate Long Ext, Classic Release, Right	1
1c, 2-17, 18a, 19, 21, 24a, 25c, 26-28, 29a, 30-35	501590	Extended Length ELR Assembly 16-17 Angle Adj Footplate Short Ext, Classic Release, Right	1
1c, 2-17, 18a, 19, 21, 24b, 25e, 26-28, 29a, 30-35	501591	Extended Length ELR Assembly 18-22 Angle Adj Footplate Long Ext, Classic Release, Right	1
1c, 2-17, 18a, 19, 21, 24a, 25e, 26-28, 29a, 30-35	501592	Extended Length ELR Assembly 18-22 Angle Adj Footplate Short Ext, Classic Release, Right	1
1d, 2-17, 18b, 19, 21, 24b, 25b, 26-28, 29b, 30-35	501593	Extended Length ELR Assembly 14 Angle Adj Footplate Long Ext, Classic Release, Left	1
1d, 2-17, 18b, 19, 21, 24a, 25b, 26-28, 29b, 30-35	501594	Extended Length ELR Assembly 14 Angle Adj Footplate Short Ext, Classic Release, Left	1
1d, 2-17, 18a, 19, 21, 24b, 25b, 26-28, 29b, 30-35	501595	Extended Length ELR Assembly 15 Angle Adj Footplate Long Ext, Classic Release, Left	1
1d, 2-17, 18a, 19, 21, 24a, 25b, 26-28, 29b, 30-35	501596	Extended Length ELR Assembly 15 Angle Adj Footplate Short Ext, Classic Release, Left	1
1d, 2-17, 18a, 19, 21, 24b, 25d, 26-28, 29b, 30-35	501597	Extended Length ELR Assembly 16-17 Angle Adj Footplate Long Ext, Classic Release, Left	1
1d, 2-17, 18a, 19, 21, 24a, 25d, 26-28, 29b, 30-35	501598	Extended Length ELR Assembly 16-17 Angle Adj Footplate Short Ext, Classic Release, Left	1
1d, 2-17, 18a, 19, 21, 24b, 25f, 26-28, 29b, 30-35	501599	Extended Length ELR Assembly 18-22 Angle Adj Footplate Long Ext, Classic Release, Left	1
1d, 2-17, 18a, 19, 21, 24a, 25f, 26-28, 29b, 30-35	501600	Extended Length ELR Assembly 18-22 Angle Adj Footplate Short Ext, Classic Release, Left	1
1a	500553-R	ELR Upper Standard Length Right	1
1b	500553L	ELR Upper Standard Length Left	1
1c	500601R	ELR Upper Extended Length Right	1
1d	500601L	ELR Upper Extended Length Left	1
2	000032	Footrest Hanger Pivot Saddle	1

Item Number	Part Number	Description	Quantity
3	100673	M8 x 45 BTN SHCS BLKZC	1
4	100659	M8 Nylock Nut Blk Zn	1
5	100804	Tube End Plug 1"	1
7	001158	Lever Swing Away Short	1
9	000119	Hanger Spring	1
10	001374	M5 x 40 BHCS P/THD BLKZC	1
11	100657	M5 Nylock Nut Blk Zn	1
6-11	003417	Classic Hanger Latch Assembly, Left	1
6-11	003416	Classic Hanger Latch Assembly, Right	1
12	100667	M6 x 12 BTN SHCS BLKZC w/Patch	1
13	100669	M5 x 10 BTN SHCS BLKZC w/Patch	2
14	101193	Calf Pad Mount Assembly	1
15	000556	ELR Index Spacer	1
16	100668	M6 x 20 CLS10.9 BTN SHCS ZC w/Patch	1
17	000073	M5 x 24 Threaded Barrel	1
18a	101585	Calf Pad	1
18b	101470	Pediatric Calf Pad <i>Used on 14" wide chairs</i>	1
19	000034	Sleeve 3/4" x 1 1/2"	1
20b	110389	Footrest Extension Tube ELR Long <i>Replacement for 000539</i>	1
21	100638	Tube End Plug 3/4"	1
20b, 21, 22b, 23	505360	Long ELR Ext Tube with Large Composite Footplate 17-22 Assembly <i>This kit is required if needing to replace 101584 with long extension tubes</i>	1
22c	003583	Footplate ELR Composite for Heel Loop Small Right	1
22d	003584	Footplate ELR Composite for Heel Loop Small Left	1
22e	003585	Footplate ELR Composite for Heel Loop Large Right	1
22f	003586	Footplate ELR Composite for Heel Loop Large Left	1

Item Number	Part Number	Description	Quantity
23	101586	ELR Footrest Bumper	1
24a	110386	ELR Extension Tube Short Angle Adjustable <i>Replacement for 000622</i>	1
24b	110387	ELR Extension Tube Long Angle Adjustable <i>Replacement for 000623</i>	1
25a	000105	Footplate Angle Adjustable 14-15" Wide CAT/TILT Left (Std ELR Right)	1
25b	000106	Footplate Angle Adjustable 14-15" Wide CAT/TILT Right (Std ELR Left)	1
25c	000107	Footplate Angle Adjustable 16-17" Wide CAT/TILT Left (Std ELR Right)	1
25d	000108	Footplate Angle Adjustable 16-17" Wide CAT/TILT Right (Std ELR Left)	1
25e	000109	Footplate Angle Adjustable 18-24" Wide CAT/TILT Left (Std ELR Right)	1
25f	000110	Footplate Angle Adjustable 18-24" Wide CAT/TILT Right (Std ELR Left)	1
26	100502	Angle Adj Footplate Clamp 2"	1
27	100662	M6 x 25 FSHCS BLK ZC	2
28	100658	M6 Nylock Nut Blk Zn	2
29a	000047	Footplate Angle Adjustable Casting 3/4 Right	1
29b	000049	Footplate Angle Adjustable Casting 3/4" Left	1
30	101472	M6 x 45 SHCS BLK ZC	1
31	101455	M6 Belleville Washer (.252 x .551 x .050)	1
32	000053	Saddle 3/4"	2
33	100746	M6 Flat Washer Blk Zn	1
34	101456	M6 Ovl Top L/N ZC	1
35	000428	Wear Washer	1
36	000621	ELR Spacer Small Footplate <i>Only used with small composite footplate</i>	1
37	003227INST	Hanger Latch Pin with 4-Way Release Instruction	1
38	003226INST	Hanger Latch Bracket with 4-way Release Instruction	1
39	003228INST	Hanger Latch, 4-Way, Lever with Instruction	1
9, 10, 11, 37, 38, 39	003643INST	Assembled 4-Way Out Hanger Release, Left with Instruction	1
9, 10, 11, 37, 38, 39	003642INST	Assembled 4-Way Out Hanger Release, Right with Instruction	1

Item Number	Part Number	Description	Quantity
1b, 2-17, 18b, 19	501046	ELR Upper Assembly with Classic Hanger Latch 14 Left	1
1a, 2-17, 18b, 19	501047	ELR Upper Assembly with Classic Hanger Latch 14 Right	1
1b, 2-17, 18a, 19	500615L	ELR Upper Assembly with Classic Hanger Latch Left	1
1a, 2-17, 18a, 19	500615R	ELR Upper Assembly with Classic Hanger Latch Right	1
1d, 2-17, 18b, 19	501601	Extended Length ELR Upper Assembly with Classic Hanger Latch 14 Left	1
1c, 2-17, 18b, 19	501602	Extended Length ELR Upper Assembly with Classic Hanger Latch 14 Right	1
1d, 2-17, 18a, 19	501603	Extended Length ELR Upper Assembly with Classic Hanger Latch Left	1
1c, 2-17, 18a, 19	501604	Extended Length ELR Upper Assembly with Classic Hanger Latch Right	1
1b, 2-5, 9-17, 18b, 19, 37-39	505307	ELR Upper Assembly with 4-Way Out Hanger Latch 14 Left	1
1a, 2-5, 9-17, 18b, 19, 37-39	505308	ELR Upper Assembly with 4-Way Out Hanger Latch 14 Right	1
1b, 2-5, 9-17, 18a, 19, 37-39	505309	ELR Upper Assembly with 4-Way Out Hanger Latch Left	1
1a, 2-5, 9-17, 18a, 19, 37-39	505310	ELR Upper Assembly with 4-Way Out Hanger Latch Right	1
1d, 2-5, 9-17, 18b, 19, 37-39	505311	Extended Length ELR Upper Assembly with 4-Way Out Hanger Latch 14 Left	1
1c, 2-5, 9-17, 18b, 19, 37-39	505312	Extended Length ELR Upper Assembly with 4-Way Out Hanger Latch 14 Right	1
1d, 2-5, 9-17, 18a, 19, 37-39	505313	Extended Length ELR Upper Assembly with 4-Way Out Hanger Latch Left	1
1c, 2-5, 9-17, 18a, 19, 37-39	505314	Extended Length ELR Upper Assembly with 4-Way Out Hanger Latch Right	1
21, 24a, 25b, 26 - 28, 29b, 30 - 35	505331	Short ELR Ext Tube with Angle Adjustable Footplate 14/15 Left Assembly	1
21, 24a, 25a, 26 - 28, 29a, 30 - 35	505332	Short ELR Ext Tube with Angle Adjustable Footplate 14/15 Right Assembly	1
21, 24a, 25a, 25b, 26 - 35	505333	Short ELR Ext Tube with Angle Adjustable Footplate 14/15 Assembly PAIR	1
21, 24a, 25b, 26 - 28, 29b, 30 - 35	505334	Short ELR Ext Tube with Angle Adjustable Footplate 14/15 with Heel Loop Left Assembly	1
21, 24a, 25a, 26 - 28, 29a, 30 - 35	505335	Short ELR Ext Tube with Angle Adjustable Footplate 14/15 with Heel Loop Right Assembly	1
21, 24a, 25a, 25b, 26 - 35	505336	Short ELR Ext Tube with Angle Adjustable Footplate 14/15 with Heel Loop Assembly PAIR	1
21, 24a, 25b, 26 - 28, 29b, 30 - 35	505337	Short ELR Ext Tube with Angle Adjustable Footplate 14/15 with Adjustable Heel Loop Left Assembly	1
21, 24a, 25a, 26 - 28, 29a, 30 - 35	505338	Short ELR Ext Tube with Angle Adjustable Footplate 14/15 with Adjustable Heel Loop Right Assembly	1

Item Number	Part Number	Description	Quantity
21, 24a, 25a, 25b, 26 - 35	505339	Short ELR Ext Tube with Angle Adjustable Footplate 14/15 with Adjustable Heel Loop Assembly PAIR	1
21, 24a, 25d, 26 - 28, 29b, 30 - 35	505340	Short ELR Ext Tube with Angle Adjustable Footplate 16/17 Left Assembly	1
21, 24a, 25c, 26 - 28, 29a, 30 - 35	505341	Short ELR Ext Tube with Angle Adjustable Footplate 16/17 Right Assembly	1
21, 24a, 25c, 25d, 26 - 35	505342	Short ELR Ext Tube with Angle Adjustable Footplate 16/17 Assembly PAIR	1
21, 24a, 25d, 26 - 28, 29b, 30 - 35	505343	Short ELR Ext Tube with Angle Adjustable Footplate 16/17 with Heel Loop Left Assembly	1
21, 24a, 25c, 26 - 28, 29a, 30 - 35	505344	Short ELR Ext Tube with Angle Adjustable Footplate 16/17 with Heel Loop Right Assembly	1
21, 24a, 25c, 25d, 26 - 35	505345	Short ELR Ext Tube with Angle Adjustable Footplate 16/17 with Heel Loop Assembly PAIR	1
21, 24a, 25d, 26 - 28, 29b, 30 - 35	505346	Short ELR Ext Tube with Angle Adjustable Footplate 16/17 with Adjustable Heel Loop Left Assembly	1
21, 24a, 25c, 26 - 28, 29a, 30 - 35	505347	Short ELR Ext Tube with Angle Adjustable Footplate 16/17 with Adjustable Heel Loop Right Assembly	1
21, 24a, 25c, 25d, 26 - 35	505348	Short ELR Ext Tube with Angle Adjustable Footplate 16/17 with Adjustable Heel Loop Assembly PAIR	1
21, 24a, 25f, 26 - 28, 29b, 30 - 35	505349	Short ELR Ext Tube with Angle Adjustable Footplate 18-24 Left Assembly	1
21, 24a, 25e, 26 - 28, 29a, 30 - 35	505350	Short ELR Ext Tube with Angle Adjustable Footplate 18-24 Right Assembly	1
21, 24a, 25e, 25f, 26 - 35	505351	Short ELR Ext Tube with Angle Adjustable Footplate 18-24 Assembly PAIR	1
21, 24a, 25f, 26 - 28, 29b, 30 - 35	505352	Short ELR Ext Tube with Angle Adjustable Footplate 18-24 with Heel Loop Left Assembly	1
21, 24a, 25e, 26 - 28, 29a, 30 - 35	505353	Short ELR Ext Tube with Angle Adjustable Footplate 18-24 with Heel Loop Right Assembly	1
21, 24a, 25e, 25f, 26 - 35	505354	Short ELR Ext Tube with Angle Adjustable Footplate 18-24 with Heel Loop Assembly PAIR	1
21, 24a, 25f, 26 - 28, 29b, 30 - 35	505355	Short ELR Ext Tube with Angle Adjustable Footplate 18-24 with Adjustable Heel Loop Left Assembly	1
21, 24a, 25e, 26 - 28, 29a, 30 - 35	505356	Short ELR Ext Tube with Angle Adjustable Footplate 18-24 with Adjustable Heel Loop Right Assembly	1
21, 24a, 25e, 25f, 26 - 35	505357	Short ELR Ext Tube with Angle Adjustable Footplate 18-24 with Adjustable Heel Loop Assembly PAIR	1
20b, 21, 22b, 23	505360	Long ELR Ext Tube with Large Composite Footplate 17-22 Assembly	1

Item Number	Part Number	Description	Quantity
20b, 21, 22b, 23	505361	Long ELR Ext Tube with Large Composite Footplate 17-22 Assembly PAIR	1
20b, 21, 22d, 23, 36	505362	Long ELR Ext Tube with Small Composite Footplate 14-16 with Heel Loop Left Assembly	1
20b, 21, 22c, 23, 36	505363	Long ELR Ext Tube with Small Composite Footplate 14-16 with Heel Loop Right Assembly	1
20b, 21, 22c, 22d, 23, 36	505364	Long ELR Ext Tube with Small Composite Footplate 14-16 with Heel Loop Assembly PAIR	1
20b, 21, 22d, 23, 36	505365	Long ELR Ext Tube with Small Composite Footplate 14-16 with Adjustable Heel Loop Left Assembly	1
20b, 21, 22c, 23, 36	505366	Long ELR Ext Tube with Small Composite Footplate 14-16 with Adjustable Heel Loop Right Assembly	1
20b, 21, 22c, 22d, 23, 36	505367	Long ELR Ext Tube with Small Composite Footplate 14-16 with Adjustable Heel Loop Assembly PAIR	1
20b, 21, 22f, 23	505368	Long ELR Ext Tube with Large Composite Footplate 17-22 with Heel Loop Left Assembly	1
20b, 21, 22e, 23	505369	Long ELR Ext Tube with Large Composite Footplate 17-22 with Heel Loop Right Assembly	1
20b, 21, 22e, 22f, 23	505370	Long ELR Ext Tube with Large Composite Footplate 17-22 with Heel Loop Assembly PAIR	1
20b, 21, 22f, 23	505371	Long ELR Ext Tube with Large Composite Footplate 17-22 with Adjustable Heel Loop Left Assembly	1
20b, 21, 22e, 23	505372	Long ELR Ext Tube with Large Composite Footplate 17-22 with Adjustable Heel Loop Right Assembly	1
20b, 21, 22e, 22f, 23	505373	Long ELR Ext Tube with Large Composite Footplate 17-22 with Adjustable Heel Loop Assembly PAIR	1
21, 24b, 25b, 26 - 28, 29b, 30 - 35	505374	Long ELR Ext Tube with Angle Adjustable Footplate 14/15 Left Assembly	1
21, 24b, 25a, 26 - 28, 29a, 30 - 35	505375	Long ELR Ext Tube with Angle Adjustable Footplate 14/15 Right Assembly	1
21, 24b, 25a, 25b, 26 - 35	505376	Long ELR Ext Tube with Angle Adjustable Footplate 14/15 Assembly PAIR	1
21, 24b, 25b, 26 - 28, 29b, 30 - 35	505377	Long ELR Ext Tube with Angle Adjustable Footplate 14/15 with Heel Loop Left Assembly	1
21, 24b, 25a, 26 - 28, 29a, 30 - 35	505378	Long ELR Ext Tube with Angle Adjustable Footplate 14/15 with Heel Loop Right Assembly	1
21, 24b, 25a, 25b, 26 - 35	505379	Long ELR Ext Tube with Angle Adjustable Footplate 14/15 with Heel Loop Assembly PAIR	1
21, 24b, 25b, 26 - 28, 29b, 30 - 35	505380	Long ELR Ext Tube with Angle Adjustable Footplate 14/15 with Adjustable Heel Loop Left Assembly	1
21, 24b, 25a, 26 - 28, 29a, 30 - 35	505381	Long ELR Ext Tube with Angle Adjustable Footplate 14/15 with Adjustable Heel Loop Right Assembly	1

Item Number	Part Number	Description	Quantity
21, 24b, 25a, 25b, 26 - 35	505382	Long ELR Ext Tube with Angle Adjustable Footplate 14/15 with Adjustable Heel Loop Assembly PAIR	1
21, 24b, 25d, 26 - 28, 29b, 30 - 35	505383	Long ELR Ext Tube with Angle Adjustable Footplate 16/17 Left Assembly	1
21, 24b, 25c, 26 - 28, 29a, 30 - 35	505384	Long ELR Ext Tube with Angle Adjustable Footplate 16/17 Right Assembly	1
21, 24b, 25c, 25d, 26 - 35	505385	Long ELR Ext Tube with Angle Adjustable Footplate 16/17 Assembly PAIR	1
21, 24b, 25d, 26 - 28, 29b, 30 - 35	505386	Long ELR Ext Tube with Angle Adjustable Footplate 16/17 with Heel Loop Left Assembly	1
21, 24b, 25c, 26 - 28, 29a, 30 - 35	505387	Long ELR Ext Tube with Angle Adjustable Footplate 16/17 with Heel Loop Right Assembly	1
21, 24b, 25c, 25d, 26 - 35	505388	Long ELR Ext Tube with Angle Adjustable Footplate 16/17 with Heel Loop Assembly PAIR	1
21, 24b, 25d, 26 - 28, 29b, 30 - 35	505389	Long ELR Ext Tube with Angle Adjustable Footplate 16/17 with Adjustable Heel Loop Left Assembly	1
21, 24b, 25c, 26 - 28, 29a, 30 - 35	505390	Long ELR Ext Tube with Angle Adjustable Footplate 16/17 with Adjustable Heel Loop Right Assembly	1
21, 24b, 25c, 25d, 26 - 35	505391	Long ELR Ext Tube with Angle Adjustable Footplate 16/17 with Adjustable Heel Loop Assembly PAIR	1
21, 24b, 25f, 26 - 28, 29b, 30 - 35	505392	Long ELR Ext Tube with Angle Adjustable Footplate 18-24 Left Assembly	1
21, 24b, 25e, 26 - 28, 29a, 30 - 35	505393	Long ELR Ext Tube with Angle Adjustable Footplate 18-24 Right Assembly	1
21, 24b, 25e, 25f, 26 - 35	505394	Long ELR Ext Tube with Angle Adjustable Footplate 18-24 Assembly PAIR	1
21, 24b, 25f, 26 - 28, 29b, 30 - 35	505395	Long ELR Ext Tube with Angle Adjustable Footplate 18-24 with Heel Loop Left Assembly	1
21, 24b, 25e, 26 - 28, 29a, 30 - 35	505396	Long ELR Ext Tube with Angle Adjustable Footplate 18-24 with Heel Loop Right Assembly	1
21, 24b, 25e, 25f, 26 - 35	505397	Long ELR Ext Tube with Angle Adjustable Footplate 18-24 with Heel Loop Assembly PAIR	1
21, 24b, 25f, 26 - 28, 29b, 30 - 35	505398	Long ELR Ext Tube with Angle Adjustable Footplate 18-24 with Adjustable Heel Loop Left Assembly	1
21, 24b, 25e, 26 - 28, 29a, 30 - 35	505399	Long ELR Ext Tube with Angle Adjustable Footplate 18-24 with Adjustable Heel Loop Right Assembly	1
21, 24b, 25e, 25f, 26 - 35	505400	Long ELR Ext Tube with Angle Adjustable Footplate 18-24 with Adjustable Heel Loop Assembly PAIR	1