


Rigid Transit Wheels

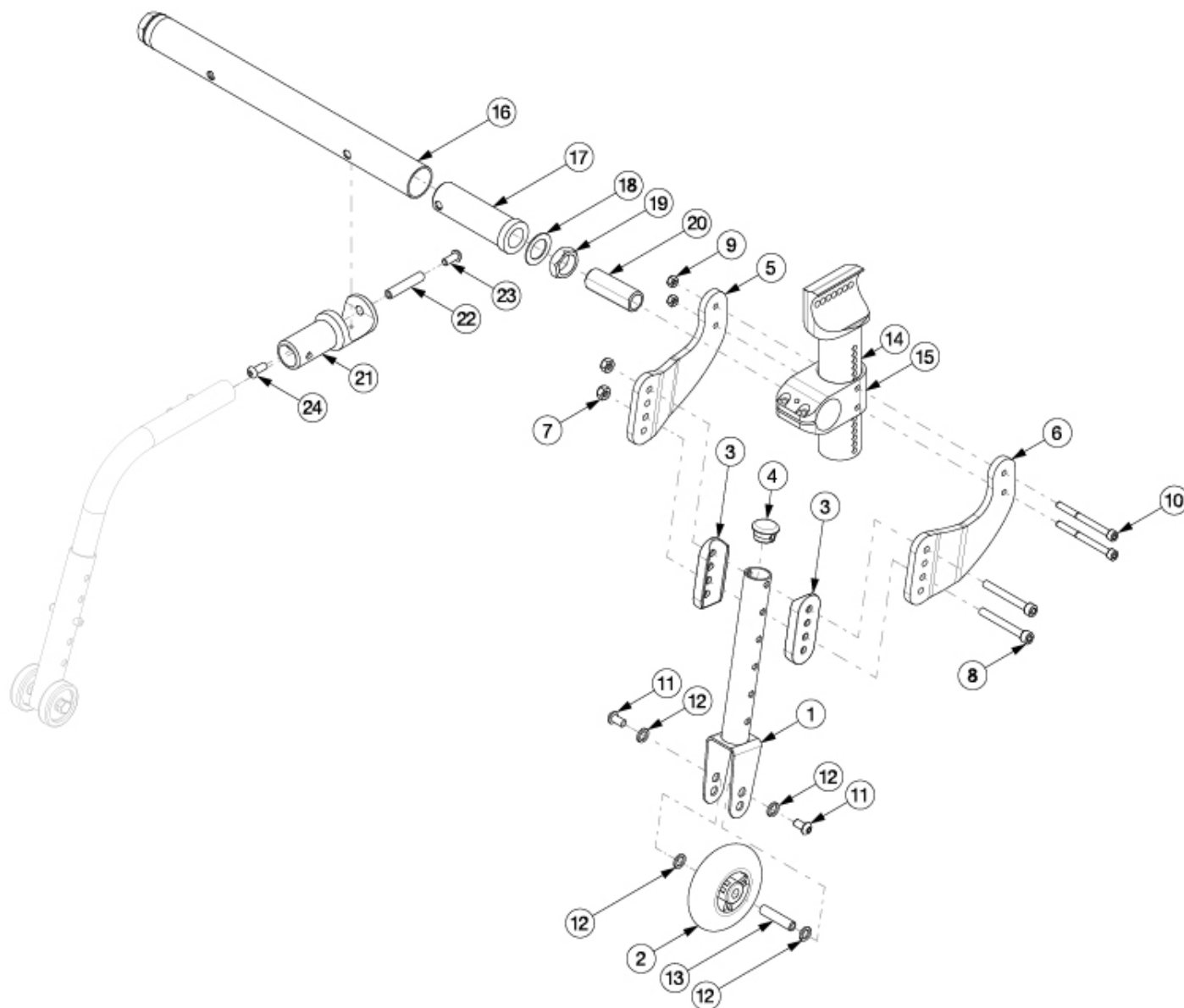

When retrofitting Transit Wheels, a new camber tube, adaptor, MX Towers, and anti-tip receiver are required. Not available with User-Activated Anti-Tips

Item Number	Part Number	Description	Quantity
1a, 2-13	122951	Rigid Transit Wheels Assembly Kit, Short (Pair)	1
1b, 2-13	122953	Rigid Transit Wheels Assembly Kit, Medium (Pair)	1
1a, 2-13	122946	Rigid Transit Wheels Assembly Kit, Short (One Side)	1
1b, 2-13	122948	Rigid Transit Wheels Assembly Kit, Medium (One Side)	1

Item Number	Part Number	Description	Quantity
1c, 2-13	122950	Rigid Transit Wheels Assembly Kit, Long (One Side)	1
1a	003701	Transit Wheel Tower Short	1
1b	003702	Transit Wheel Tower Medium	1
1c	003703	Transit Wheel Tower Long	1
2	100790	Caster 3" Rollerblade	1
3	122532	Rigid Transit Wheel Saddle	2
4	122533	7/8" OD End Plug, Black with Hole	1
7	100658	M6 Nylock Nut Blk Zn	2
8	102473	M6 X 50 CLS12.9 SHCS BLK ZC	2
9	100657	M5 Nylock Nut Blk Zn	2
11	100667	M6 x 12 BTN SHCS BLKZC w/Patch	2
12	101701	Washer 3.315/.350 x .425/.525 x .059/.067 18-8 F/W	2
13	000347	M6 x 1.0 Threaded Barrel	1
14	000493	MX Tower, RG ALX	1
15	000462	Camber Mount Offset, RG ALX/LW XP	1
16a	500722	Camber Tube Rigid 12" Kit Black	1
16b	500723	Camber Tube Rigid 13" Kit Black	1
16c	500724	Camber Tube Rigid 14" Kit Black	1
16d	500725	Camber Tube Rigid 15" Kit Black	1
16e	500726	Camber Tube Rigid 16" Kit Black	1
16f	500727	Camber Tube Rigid 17" Kit Black	1
16g	500728	Camber Tube Rigid 18" Kit Black	1
16h	500729	Camber Tube Rigid 19" Kit Black	1
16i	500730	Camber Tube Rigid 20" Kit Black	1
16j	501749	Camber Tube Rigid Blue Anodized	1
16k	501750	Camber Tube Rigid Bright Green Anodized	1
16l	501751	Camber Tube Rigid Orange Anodized	1
16m	501752	Camber Tube Rigid Pink Anodized	1

Item Number	Part Number	Description	Quantity
16n	501753	Camber Tube Rigid Pewter Anodized	1
16o	501754	Camber Tube Rigid Purple Anodized	1
16p	501755	Camber Tube Rigid Red Anodized	1
16q	501756	Camber Tube Rigid Gold Anodized	1
17a	000252	0° Camber Adapter Rigid	1
17b	000253	2° Camber Adapter Rigid	1
17c	000254	4° Camber Adapter Rigid	1
17d	000255	6° Camber Adapter Rigid	1
17e	000256	8° Camber Adapter Rigid	1
18	100583	Axle Plate Washer 18mm	1
19	100514	Axle Receiver Nut M18 x D12 x 8mm	1
20	000985	Axle Receiver with Flats M18 x 1/2" ID	1
21	000351	Anti Tip Receiver Rogue2 / Rogue ALX	1
22	000347	M6 x 1.0 Threaded Barrel	1
23	100667	M6 x 12 BTN SHCS BLKZC w/Patch	1
24	100668	M6 x 20 CLS10.9 BTN SHCS ZC w/Patch	1