


Clik Camber Tube and Adapters

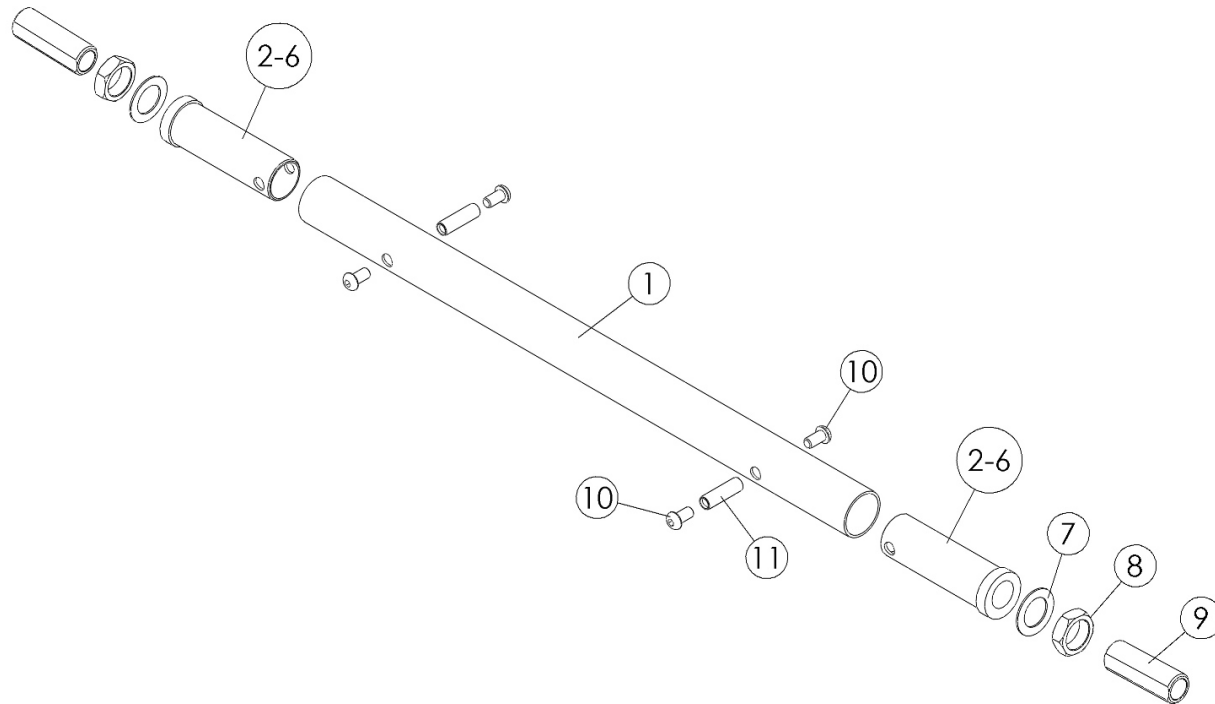

Item Number	Part Number	Description	Quantity
1a	503238	Camber Tube Rigid 8" Kit Black	1
1b	503239	Camber Tube Rigid 9" Kit Black	1
1c	500720	Camber Tube Rigid 10" Kit Black	1
1d	500721	Camber Tube Rigid 11" Kit Black	1
1e	500722	Camber Tube Rigid 12" Kit Black	1
1f	500723	Camber Tube Rigid 13" Kit Black	1
1g	500724	Camber Tube Rigid 14" Kit Black	1
1h	500725	Camber Tube Rigid 15" Kit Black	1
1i	500726	Camber Tube Rigid 16" Kit Black	1
1p	501749	Camber Tube Rigid Blue Anodized	1
1q	501750	Camber Tube Rigid Bright Green Anodized	1

Item Number	Part Number	Description	Quantity
1r	501751	Camber Tube Rigid Orange Anodized	1
1s	501752	Camber Tube Rigid Pink Anodized	1
1t	501753	Camber Tube Rigid Pewter Anodized	1
1u	501754	Camber Tube Rigid Purple Anodized	1
1v	501755	Camber Tube Rigid Red Anodized	1
1w	501756	Camber Tube Rigid Gold Anodized	1
1x	123012_ _ _	Camber Tube Rigid Color Accent	1
2a	000252	0° Camber Adapter Rigid	2
3a	000253	2° Camber Adapter Rigid	2
4a	000254	4° Camber Adapter Rigid	2
5a	000255	6° Camber Adapter Rigid	2
6a	000256	8° Camber Adapter Rigid	2
2b, 10, 11	113209	0° Camber Adapter Kit, Drum Brake <i>For complete drum brake adapter set up, see Rogue XP / Clik / Little Wave XP Drum Brake page</i>	2
3b, 10, 11	113210	2° Camber Adapter Kit, Drum Brake <i>For complete drum brake adapter set up, see Rogue XP / Clik / Little Wave XP Drum Brake page</i>	2
4b, 10, 11	113211	4° Camber Adapter Kit, Drum Brake <i>For complete drum brake adapter set up, see Rogue XP / Clik / Little Wave XP Drum Brake page</i>	2
5b, 10, 11	113212	6° Camber Adapter Kit, Drum Brake <i>For complete drum brake adapter set up, see Rogue XP / Clik / Little Wave XP Drum Brake page</i>	2
6b, 10, 11	113213	8° Camber Adapter Kit, Drum Brake <i>For complete drum brake adapter set up, see Rogue XP / Clik / Little Wave XP Drum Brake page</i>	2
7	100583	Axle Plate Washer 18mm	2
8	100514	Axle Receiver Nut M18 x D12 x 8mm	2
9	000985	Axle Receiver with Flats M18 x 1/2" ID	2
10	100667	M6 x 12 BTN SHCS BLKZC w/Patch	4
11	000066	M6 x 27 Threaded Barrel	2