


Discontinued Fender Side Guard

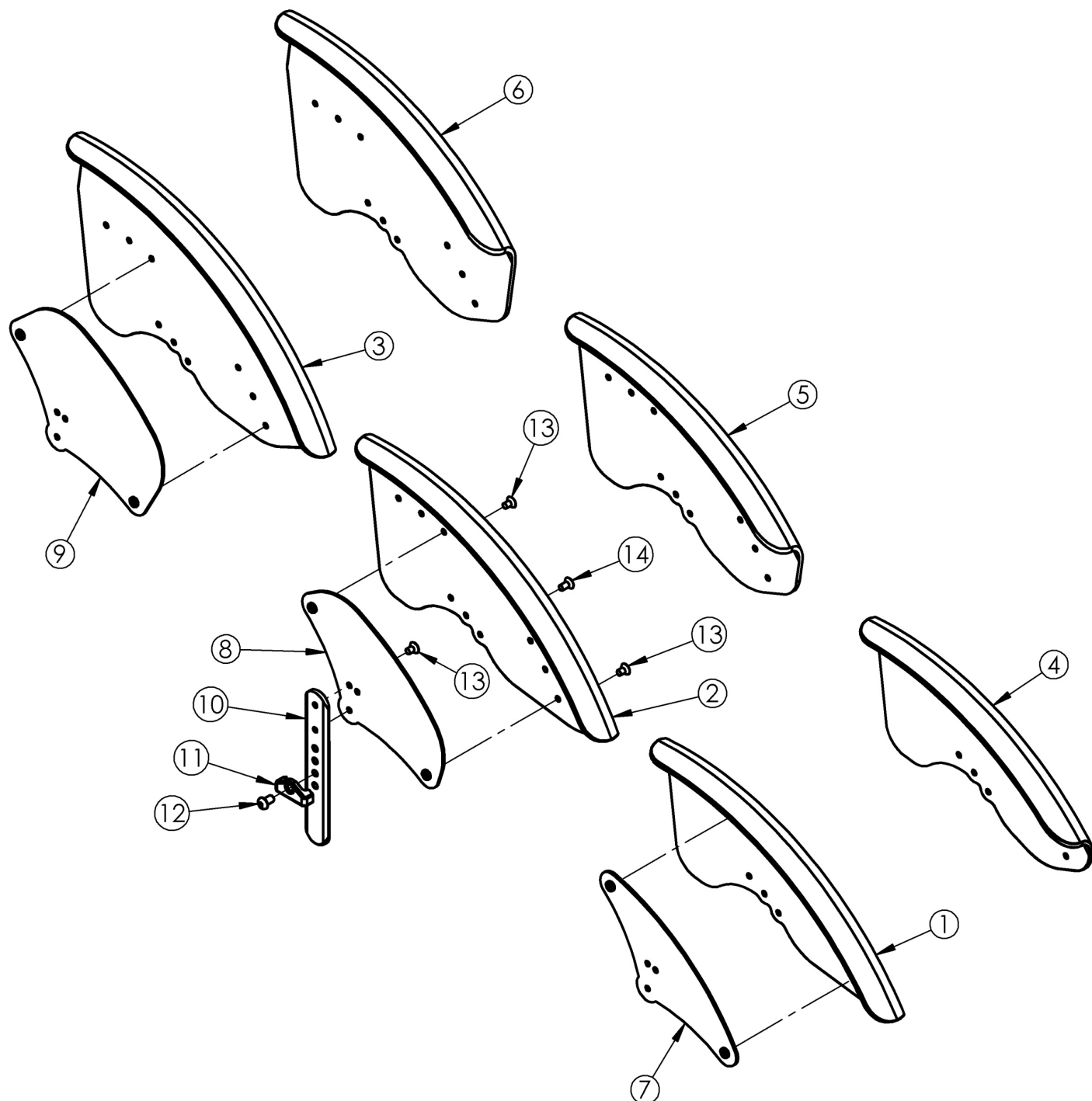

If retrofitting, side guard receivers may be required

Item Number	Part Number	Description	Quantity
3a, 9a, 10 - 14	501194	Fender Side Guard Assm Tall Left	1
3b, 9b, 10 - 14	501195	Fender Side Guard Assm Tall Right	1

Item Number	Part Number	Description	Quantity
3a, 3b, 9a, 9b, 10 - 14	501196	Fender Side Guard Assm Tall PAIR	1
2b	001335	Fender Side Guard Panel Med Right	1
3a	001336	Fender Side Guard Panel Tall Left	1
3b	001337	Fender Side Guard Panel Tall Right	1
8b	001363	Fender Support Medium Right	1
9a	001366	Fender Support Tall Left	1
9b	001367	Fender Support Tall Right	1
10	001343	Fender Side Guard Post	2
11	001342	Side Guard Post Stop	2
12	101184	M6 x 10 BTN SHCS BLKZC w/Patch	2
13	102308	M5 x 8mm FHCS Blk Zc	6
14	102309	M5 x 10mm FHCS Blk Zc	2