


Catalyst 5Vx Side Guards - Removable Adult Composite

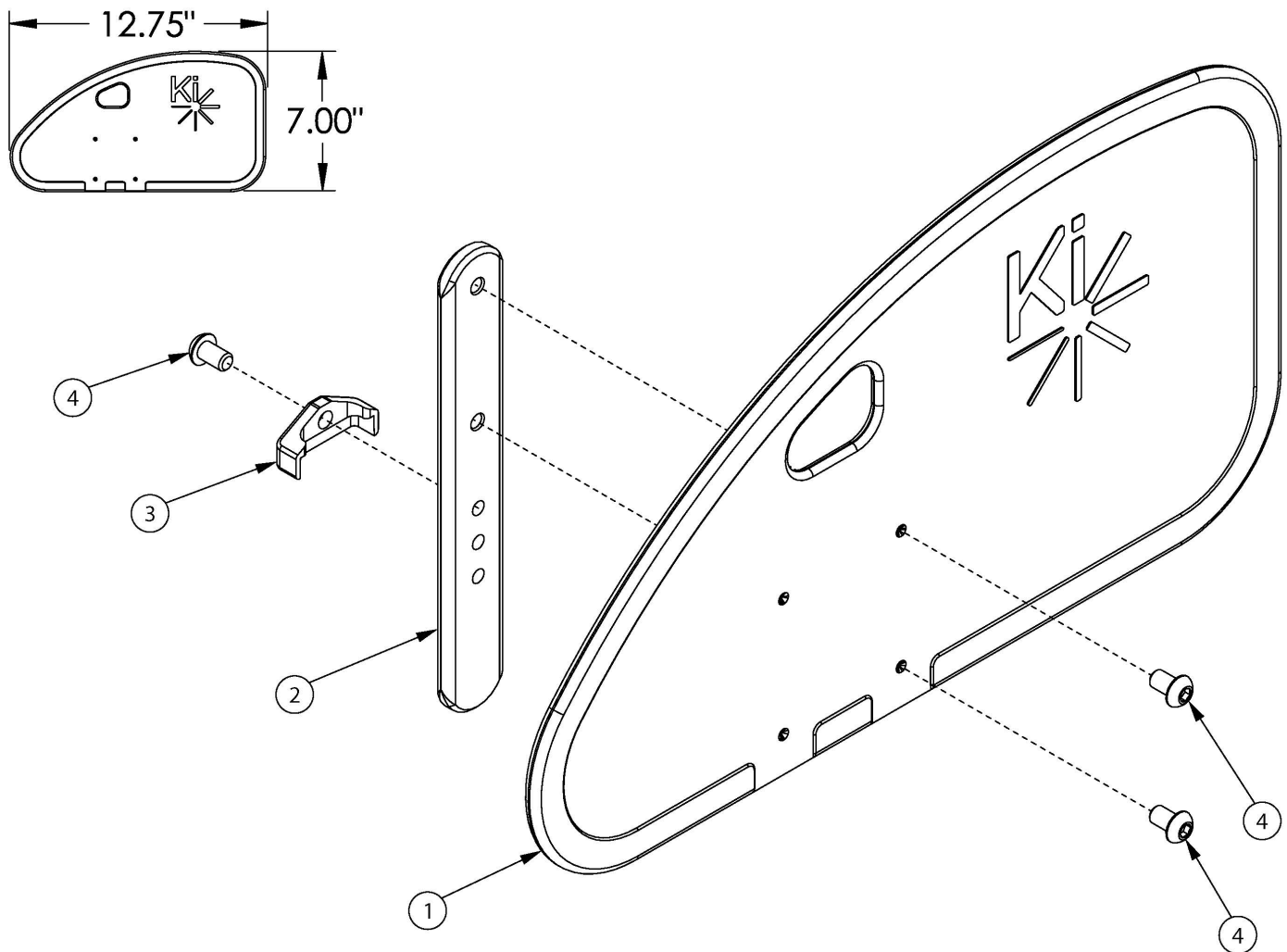

If retrofitting, side guard receivers may be required

Item Number	Part Number	Description	Quantity
1a, 2 - 4	112575	Side Guard Assembly, Front Drilled, Left <i>Used on folding chairs with 17" depth or less</i>	1
1a, 2 - 4	112576	Side Guard Assembly, Front Drilled, Right <i>Used on folding chairs with 17" depth or less</i>	1
1a, 2 - 4	112577	Side Guard Assembly, Front Drilled, PAIR <i>Used on folding chairs with 17" depth or less</i>	1
1b, 2 - 4	112578	Side Guard Assembly, Rear Drilled, Left <i>Used on folding chairs 18" deep or greater.</i>	1
1b, 2 - 4	112579	Side Guard Assembly, Rear Drilled, Right <i>Used on folding chairs 18" deep or greater.</i>	1

Item Number	Part Number	Description	Quantity
1b, 2 - 4	112580	Side Guard Assembly, Rear Drilled, PAIR <i>Used on folding chairs 18" deep or greater.</i>	1
1a	112573	Side Guard Panel Front Drilled	1
1b	112574	Side Guard Panel Rear Drilled	1
2	000140	Side Guard Post	1
3	001342	Side Guard Post Stop	1
4	101184	M6 x 10 BTN SHCS BLKZC w/Patch	3