


Rigid Camber Tube and Adapters

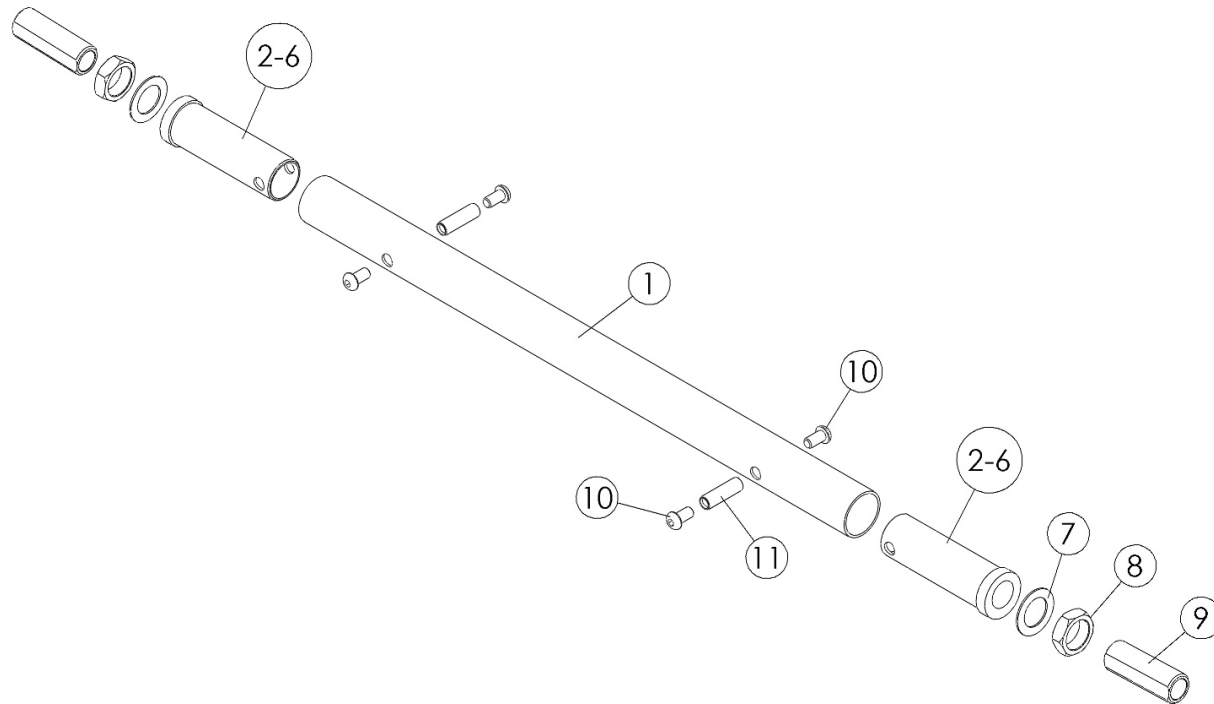

Black Camber Tube kits based on seat width. Color Anodized Camber Tube kits used on all chair widths. New camber tube required when changing the degree of camber by more than 2 degrees. If Wheelchair Drive Ready Option is present, replacement parts need to be processed through Alber USA.

Item Number	Part Number	Description	Quantity
1a	503238	Camber Tube Rigid 8" Kit Black	1
1b	503239	Camber Tube Rigid 9" Kit Black	1
1c	500720	Camber Tube Rigid 10" Kit Black	1
1d	500721	Camber Tube Rigid 11" Kit Black	1
1e	500722	Camber Tube Rigid 12" Kit Black	1
1f	500723	Camber Tube Rigid 13" Kit Black	1
1g	500724	Camber Tube Rigid 14" Kit Black	1
1h	500725	Camber Tube Rigid 15" Kit Black	1
1i	500726	Camber Tube Rigid 16" Kit Black	1

Item Number	Part Number	Description	Quantity
1j	500727	Camber Tube Rigid 17" Kit Black	1
1k	500728	Camber Tube Rigid 18" Kit Black	1
1l	500729	Camber Tube Rigid 19" Kit Black	1
1m	500730	Camber Tube Rigid 20" Kit Black	1
1n	500731	Camber Tube Rigid 21" Kit Black	1
1o	500732	Camber Tube Rigid 22" Kit Black	1
1p	501749	Camber Tube Rigid Blue Anodized	1
1q	501750	Camber Tube Rigid Bright Green Anodized	1
1r	501751	Camber Tube Rigid Orange Anodized	1
1s	501752	Camber Tube Rigid Pink Anodized	1
1t	501753	Camber Tube Rigid Pewter Anodized	1
1u	501754	Camber Tube Rigid Purple Anodized	1
1v	501755	Camber Tube Rigid Red Anodized	1
1w	501756	Camber Tube Rigid Gold Anodized	1
1x	123012_ _ _	Camber Tube Rigid Color Accent	1
2a	000252	0° Camber Adapter Rigid	2
3a	000253	2° Camber Adapter Rigid	2
4a	000254	4° Camber Adapter Rigid	2
5a	000255	6° Camber Adapter Rigid	2
6a	000256	8° Camber Adapter Rigid	2
2b, 10, 11	113209	0° Camber Adapter Kit, Drum Brake <i>For complete drum brake adapter set up, see Rogue XP / Clik / Little Wave XP Drum Brake page</i>	2
3b, 10, 11	113210	2° Camber Adapter Kit, Drum Brake <i>For complete drum brake adapter set up, see Rogue XP / Clik / Little Wave XP Drum Brake page</i>	2
4b, 10, 11	113211	4° Camber Adapter Kit, Drum Brake <i>For complete drum brake adapter set up, see Rogue XP / Clik / Little Wave XP Drum Brake page</i>	2
5b, 10, 11	113212	6° Camber Adapter Kit, Drum Brake <i>For complete drum brake adapter set up, see Rogue XP / Clik / Little Wave XP Drum Brake page</i>	2

Item Number	Part Number	Description	Quantity
6b, 10, 11	113213	8° Camber Adapter Kit, Drum Brake <i>For complete drum brake adapter set up, see Rogue XP / Clik / Little Wave XP Drum Brake page</i>	2
7	100583	Axle Plate Washer 18mm	2
8	100514	Axle Receiver Nut M18 x D12 x 8mm	2
9	000985	Axle Receiver with Flats M18 x 1/2" ID	2
10	100667	M6 x 12 BTN SHCS BLKZC w/Patch	4
11	000066	M6 x 27 Threaded Barrel	2

Color Anodizing Samples

