


# Discontinued Flip Attendant Foot Lock

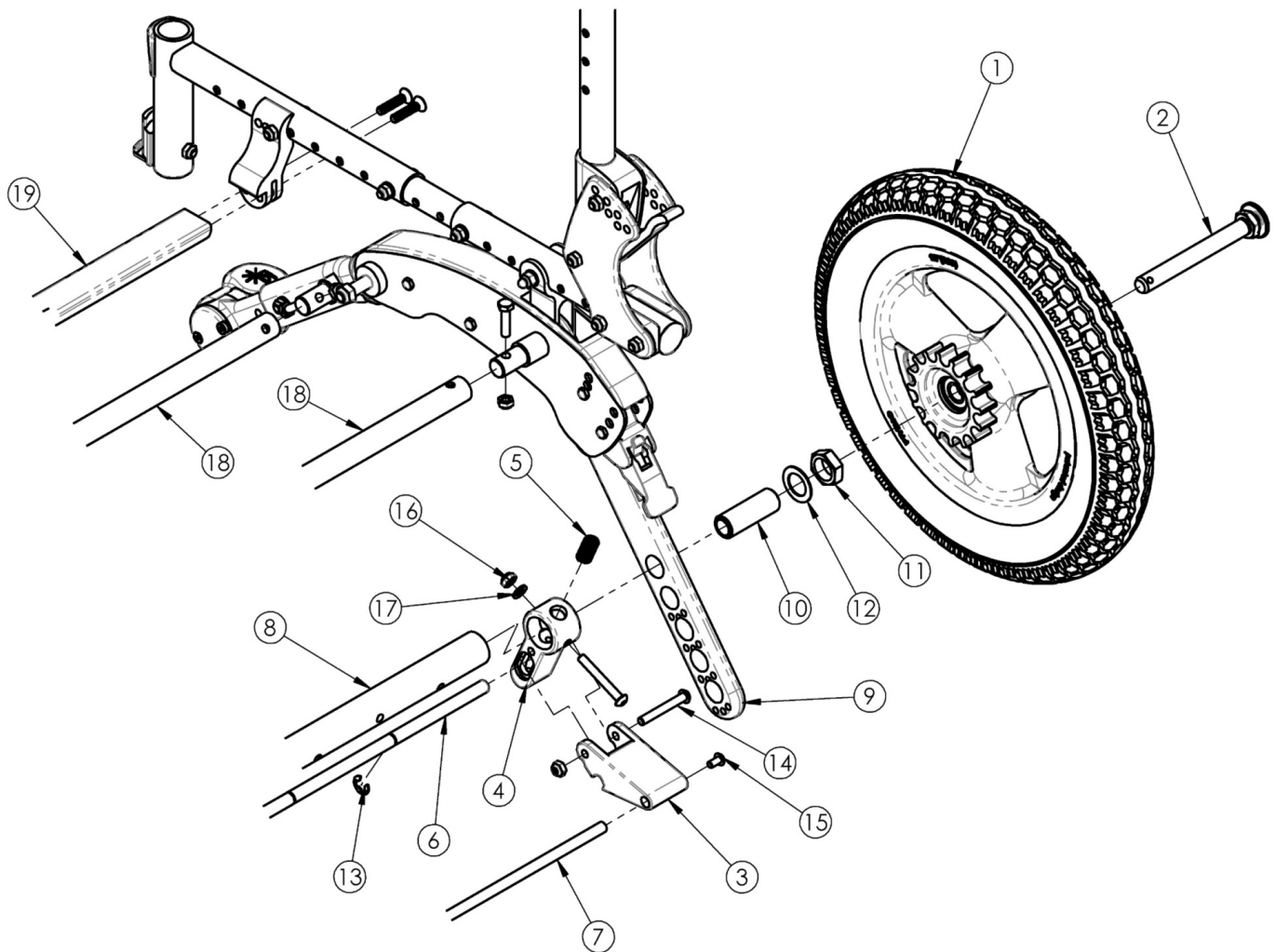

*This Flip Attendant Foot Lock is discontinued and was used on serial numbers prior to FLP007536.*

Item Number	Part Number	Description	Quantity
1a	502853	Flip Attendant Foot Lock Wheel, 12" Pneumatic EACH	1
1b	502854	Flip Attendant Foot Lock Wheel, 12" Airless EACH	1
1c	502855	Flip Attendant Foot Lock Wheel, 12" Full Poly EACH	1
1d	502856	Flip Attendant Foot Lock Wheel, 16" Pneumatic EACH	1
1e	502857	Flip Attendant Foot Lock Wheel, 16" Airless EACH	1
1f	502858	Flip Attendant Foot Lock Wheel, 16" Full Poly EACH	1
2	101108	Axle Quick Release w/ Button 1/2" x 99MM	2

Item Number	Part Number	Description	Quantity
3	001928	Lever, Hub Lock	2
5	001929	Spring, Hub Lock	2
7a	001939	Link Rod Hub Lock 10 Wide	1
7b	001940	Link Rod Hub Lock 11 Wide	1
7c	001941	Link Rod Hub Lock 12 Wide	1
7d	001942	Link Rod Hub Lock 13 Wide	1
7e	001943	Link Rod Hub Lock 14 Wide	1
7f	001944	Link Rod Hub Lock 15 Wide	1
7g	001945	Link Rod Hub Lock 16 Wide	1
7h	001946	Link Rod Hub Lock 17 Wide	1
7i	001947	Link Rod Hub Lock 18 Wide	1
8a	001840	Axle Tube, 10W	1
8b	001841	Axle Tube, 11W	1
8c	001842	Axle Tube, 12W	1
8d	001843	Axle Tube, 13W	1
8e	001844	Axle Tube, 14W	1
8f	001845	Axle Tube, 15W	1
8g	001846	Axle Tube, 16W	1
8h	001847	Axle Tube, 17W	1
8i	001848	Axle Tube, 18W	1
9a	001838	Axle Bar, Standard	2
9b	001839	Axle Bar, Self Propel	2
10	100756	Axle Receiver 1/2" Diameter	2
11	100514	Axle Receiver Nut M18 x D12 x 8mm	2
12	100583	Axle Plate Washer 18mm	2
13	102811	E-Ring, 3/8"	2
14	100670	M6 x 45 BTN SHCS BLK ZC	4
15	100669	M5 x 10 BTN SHCS BLKZC w/Patch	2

Item Number	Part Number	Description	Quantity
16	100658	M6 Nylock Nut Blk Zn	4
17	100746	M6 Flat Washer Blk Zn	2
18a	002719	Frame Growth Strut 6.5" <i>Used on 10" wide chairs</i>	1
18b	002629	Frame Growth Strut 7.5" <i>Used on 11" wide chairs</i>	1
18c	002630	Frame Growth Strut 8.5" <i>Used on 12" wide chairs</i>	1
18d	002631	Frame Growth Strut 9.5" <i>Used on 10" and 13" wide chairs</i>	1
18e	002632	Frame Growth Strut 10.5" <i>Used on 11" and 14" wide chairs</i>	1
18f	002633	Frame Growth Strut 11.5" <i>Used on 12" and 15" wide chairs</i>	1
18g	002634	Frame Growth Strut 12.5" <i>Used on 13" and 16" wide chairs</i>	1
18h	002635	Frame Growth Strut 13.5" <i>Used on 14" and 17" wide chairs</i>	1
18i	002636	Frame Growth Strut 14.5" <i>Used on 15" and 18" wide chairs</i>	1
18j	002637	Frame Growth Strut 15.5" <i>Used on 16" wide chairs</i>	1
18k	002638	Frame Growth Strut 16.5" <i>Used on 17" wide chairs</i>	1
18l	002639	Frame Growth Strut 17.5" <i>Used on 18" wide chairs</i>	1
19a	001879	Strut Tube Seat 10 Wide	1
19b	001880	Strut Tube Seat 11 Wide	1
19c	001881	Strut Tube Seat 12 Wide	1
19d	001882	Strut Tube Seat 13 Wide	1
19e	001883	Strut Tube Seat 14 Wide	1
19f	001884	Strut Tube Seat 15 Wide	1
19g	001885	Strut Tube Seat 16 Wide	1

<b>Item Number</b>	<b>Part Number</b>	<b>Description</b>	<b>Quantity</b>
19h	001886	Strut Tube Seat 17 Wide	1
19i	001887	Strut Tube Seat 18 Wide	1