



Liberty Transit Wheels - Rigid Frame

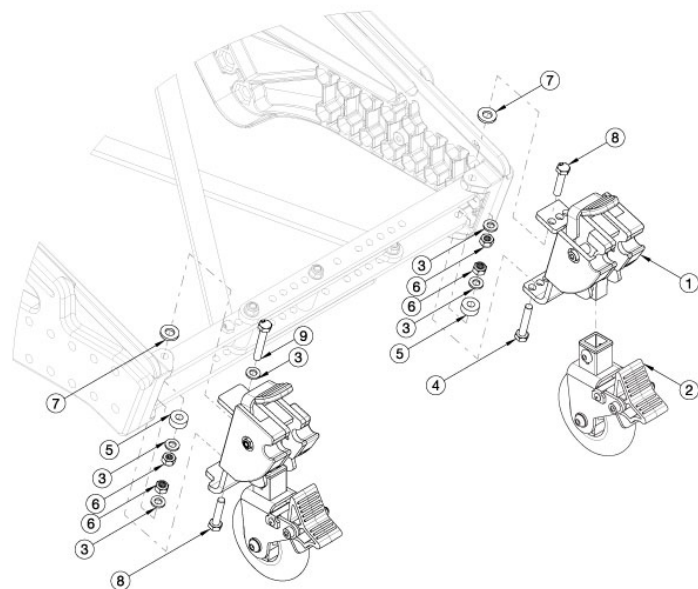

Rear Seat Height	Item 1 Left Side	Item 1 Right Side	Item 2
14"	113951	113950	111298
14.5"	113951	113950	111299
15" - 15.5"	113953	113952	111300
16" - 16.5"	113955	113954	111300
17" - 18"	113955	113954	111301

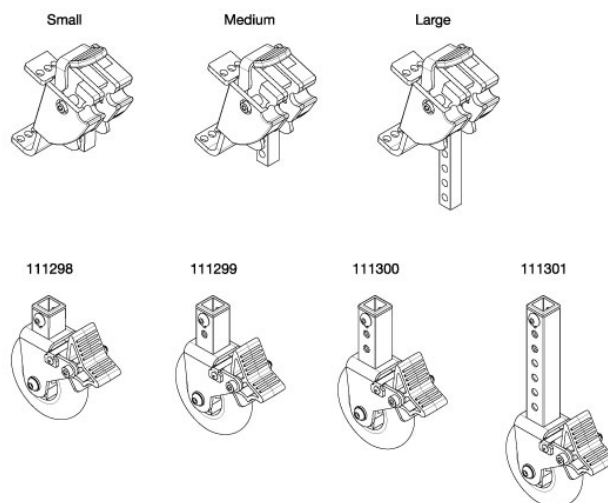

Item Number	Part Number	Description	Quantity
1a, 1b, 2a, 3-9	118076	Liberty Transit Wheels Kit, Small/Extra Small (PAIR), Short/Medium Depth Range, Rigid <i>Used with 14" rear seat height</i>	1
1a, 1b, 2b, 3-9	118077	Liberty Transit Wheels Kit, Small/Small (PAIR), Short/Medium Depth Range, Rigid <i>Used with 14.5" rear seat height</i>	1
1c, 1d, 2c, 3-9	118078	Liberty Transit Wheels Kit, Medium/Medium (PAIR), Short/Medium Depth Range, Rigid <i>Used with 15"-15.5" rear seat heights</i>	1
1e, 1f, 2c, 3-9	118079	Liberty Transit Wheels Kit, Large/Medium (PAIR), Short/Medium Depth Range, Rigid <i>Used with 16"-16.5" rear seat heights</i>	1
1e, 1f, 2d, 3-9	118080	Liberty Transit Wheels Kit, Large/Large (PAIR), Short/Medium Depth Range, Rigid <i>Used with 17"-18" rear seat heights</i>	1
1a, 2a, 3-8	118081	Liberty Transit Wheel Kit, Small/Extra Small (Right), Short/Medium Depth Range, Rigid <i>Used with 14" rear seat height</i>	1
1b, 2a, 3, 5-9	118082	Liberty Transit Wheel Kit, Small/Extra Small (Left), Short/Medium Depth Range, Rigid <i>Used with 14" rear seat height</i>	1
1a, 2b, 3-8	118083	Liberty Transit Wheel Kit, Small/Small (Right), Short/Medium Depth Range, Rigid <i>Used with 14.5" rear seat height</i>	1

Item Number	Part Number	Description	Quantity
1b, 2b, 3, 5-9	118084	Liberty Transit Wheel Kit, Small/Small (Left), Short/Medium Depth Range, Rigid <i>Used with 14.5" rear seat height</i>	1
1c, 2c, 3-8	118085	Liberty Transit Wheel Kit, Medium/Medium (Right), Short/Medium Depth Range, Rigid <i>Used with 15" - 15.5" rear seat heights</i>	1
1d, 2c, 3, 5-9	118086	Liberty Transit Wheel Kit, Medium/Medium (Left), Short/Medium Depth Range, Rigid <i>Used with 15" - 15.5" rear seat heights</i>	1
1e, 2c, 3-8	118087	Liberty Transit Wheel Kit, Large/Medium (Right), Short/Medium Depth Range, Rigid <i>Used with 16" - 16.5" rear seat heights</i>	1
1f, 2c, 3, 5-9	118088	Liberty Transit Wheel Kit, Large/Medium (Left), Short/Medium Depth Range, Rigid <i>Used with 16" - 16.5" rear seat heights</i>	1
1e, 2d, 3-8	118089	Liberty Transit Wheel Kit, Large/Large (Right), Short/Medium Depth Range, Rigid <i>Used with 17" - 18" rear seat heights</i>	1
1f, 2d, 3, 5-9	118090	Liberty Transit Wheel Kit, Large/Large (Left), Short/Medium Depth Range, Rigid <i>Used with 17" - 18" rear seat heights</i>	1
3-9	118075	Transit Wheels Rigid Hardware Kit, Liberty	1
1a	113950	Liberty Transit Wheel Upper Assembly, Small, Right <i>Used with 14" - 14.5" rear seat heights</i>	1
1b	113951	Liberty Transit Wheel Upper Assembly, Small, Left <i>Used with 14" - 14.5" rear seat heights</i>	1
1c	113952	Liberty Transit Wheel Upper Assembly, Medium, Right <i>Used with 15" - 15.5" rear seat heights</i>	1
1d	113953	Liberty Transit Wheel Upper Assembly, Medium, Left <i>Used with 15" - 15.5" rear seat heights</i>	1
1e	113954	Liberty Transit Wheel Upper Assembly, Large, Right <i>Used with 16" - 18" rear sea heights</i>	1
1f	113955	Liberty Transit Wheel Upper Assembly, Large, Left <i>Used with 16" - 18" rear sea heights</i>	1
2a	111298	Liberty Transit Wheel Bottom Assembly, Extra Small, Short/Medium Depth Range <i>Used with 14" rear seat height</i>	1
2b	111299	Liberty Transit Wheel Bottom Assembly, Small, Short/Medium Depth Range <i>Used with 14.5" rear seat height</i>	1
2c	111300	Liberty Transit Wheel Bottom Assembly, Medium, Short/Medium Depth Range <i>Used with 15" - 16.5" rear seat heights</i>	1
2d	111301	Liberty Transit Wheel Bottom Assembly, Large, Short/Medium Depth Range <i>Used with 17" - 18" rear seat heights</i>	1

Item Number	Part Number	Description	Quantity
3	100746	M6 Flat Washer Blk Zn	4
4	102386	M6 x 35mm HHCS Blk Zc	1
5	116440	Spacer, .59 OD x .251 ID x .250 L	2
6	100658	M6 Nylock Nut Blk Zn	4
7	003260	Spacer, .62ODx.316IDx.090L	2
8	116854	HHCS, M6x1, 0x30, CLS10.9, PT, BLZ, DIN931	2
9	101825	M6 x 40 HHCS Blk Zn	1