


# Little Wave XP Fixed Height Backrest with Non-Adjustable Rigidizer Bar

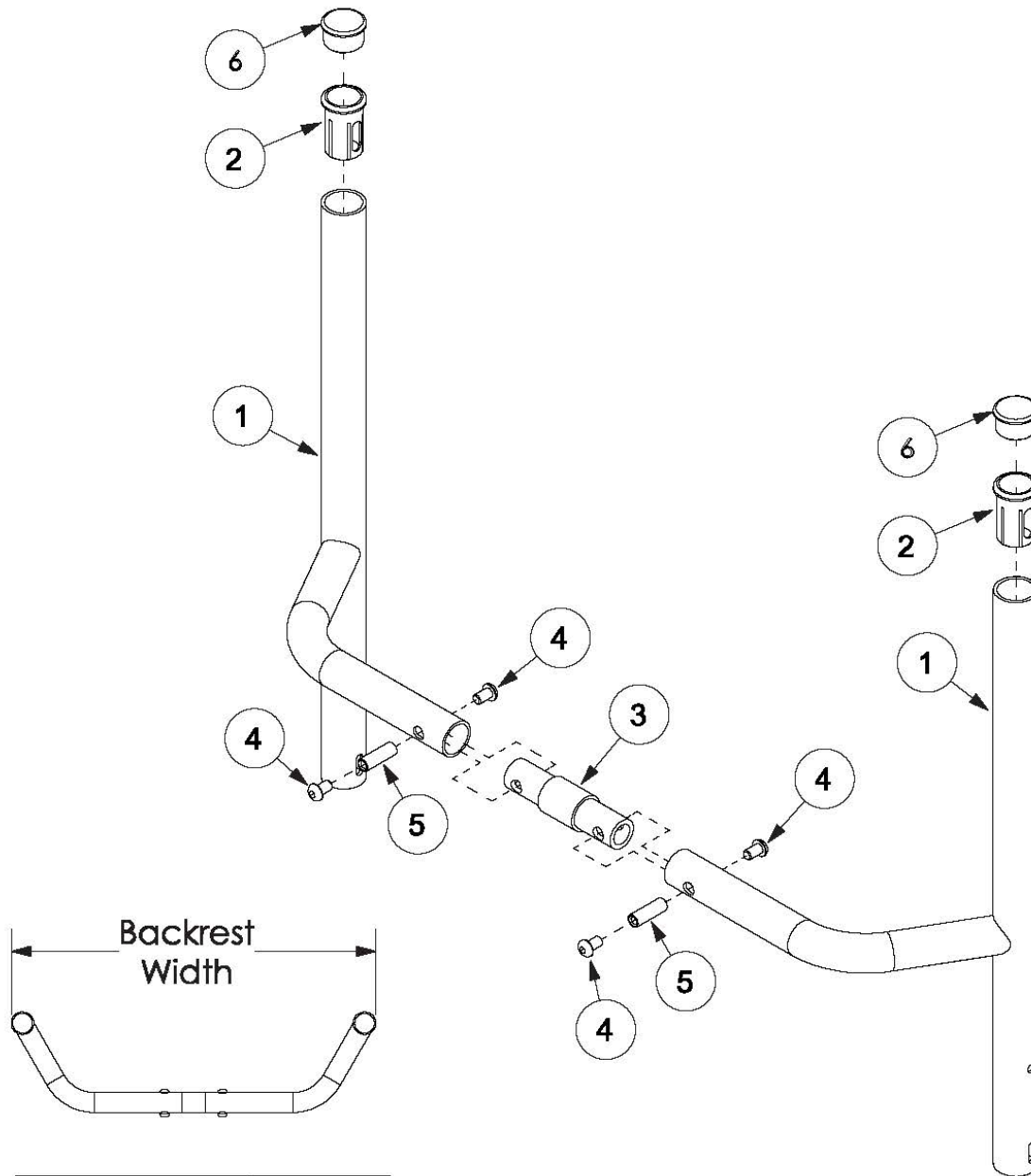

Backrest 10"	10.13"
Backrest 11"	11.13"
Backrest 12"	12.13"
Backrest 13"	13.13"
Backrest 14"	14.13"
Backrest 15"	15.13"
Backrest 16"	16.13"

Item Number	Part Number	Description	Quantity
1a, 3a, 4, 5	110699	Fixed Height Rigid Backrest 7.5", 10" Wide with 0" Connector and Hardware <i>Used with 9" back height with omit push handles, 13" back height with fold down push handles or 10" back height with standard or low profile push handles</i>	1
1a, 3b, 4, 5	110700	Fixed Height Rigid Backrest 7.5", 10" Wide with 1" Connector and Hardware <i>Used with 9" back height with omit push handles, 13" back height with fold down push handles or 10" back height with standard or low profile push handles</i>	1
1a, 3c, 4, 5	110701	Fixed Height Rigid Backrest 7.5", 10" Wide with 2" Connector and Hardware <i>Used with 9" back height with omit push handles, 13" back height with fold down push handles or 10" back height with standard or low profile push handles</i>	1
1a, 3d, 4, 5	110702	Fixed Height Rigid Backrest 7.5", 10" Wide with 3" Connector and Hardware <i>Used with 9" back height with omit push handles, 13" back height with fold down push handles or 10" back height with standard or low profile push handles</i>	1
1b, 3a, 4, 5	110703	Fixed Height Rigid Backrest 8.5", 10" Wide with 0" Connector and Hardware <i>Used with 10" back height with omit push handles, 14" back height with fold down push handles or 11" back height with standard or low profile push handles</i>	1
1b, 3b, 4, 5	110704	Fixed Height Rigid Backrest 8.5", 10" Wide with 1" Connector and Hardware <i>Used with 10" back height with omit push handles, 14" back height with fold down push handles or 11" back height with standard or low profile push handles</i>	1
1b, 3c, 4, 5	110705	Fixed Height Rigid Backrest 8.5", 10" Wide with 2" Connector and Hardware <i>Used with 10" back height with omit push handles, 14" back height with fold down push handles or 11" back height with standard or low profile push handles</i>	1
1b, 3d, 4, 5	110706	Fixed Height Rigid Backrest 8.5", 10" Wide with 3" Connector and Hardware <i>Used with 10" back height with omit push handles, 14" back height with fold down push handles or 11" back height with standard or low profile push handles</i>	1
1c, 3a, 4, 5	110707	Fixed Height Rigid Backrest 9.5", 10" Wide with 0" Connector and Hardware <i>Used with 11" back height with omit push handles, 15" back height with fold down push handles or 12" back height with standard or low profile push handles</i>	1
1c, 3b, 4, 5	110708	Fixed Height Rigid Backrest 9.5", 10" Wide with 1" Connector and Hardware <i>Used with 11" back height with omit push handles, 15" back height with fold down push handles or 12" back height with standard or low profile push handles</i>	1
1c, 3c, 4, 5	110709	Fixed Height Rigid Backrest 9.5", 10" Wide with 2" Connector and Hardware <i>Used with 11" back height with omit push handles, 15" back height with fold down push handles or 12" back height with standard or low profile push handles</i>	1
1c, 3d, 4, 5	110710	Fixed Height Rigid Backrest 9.5", 10" Wide with 3" Connector and Hardware <i>Used with 11" back height with omit push handles, 15" back height with fold down push handles or 12" back height with standard or low profile push handles</i>	1

Item Number	Part Number	Description	Quantity
1d, 3a, 4, 5	110711	Fixed Height Rigid Backrest 10.5", 10" Wide with 0" Connector and Hardware <i>Used with 12" back height with omit push handles, 16" back height with fold down push handles or 13" back height with standard or low profile push handles</i>	1
1d, 3b, 4, 5	110712	Fixed Height Rigid Backrest 10.5", 10" Wide with 1" Connector and Hardware <i>Used with 12" back height with omit push handles, 16" back height with fold down push handles or 13" back height with standard or low profile push handles</i>	1
1d, 3c, 4, 5	110713	Fixed Height Rigid Backrest 10.5", 10" Wide with 2" Connector and Hardware <i>Used with 12" back height with omit push handles, 16" back height with fold down push handles or 13" back height with standard or low profile push handles</i>	1
1d, 3d, 4, 5	110714	Fixed Height Rigid Backrest 10.5", 10" Wide with 3" Connector and Hardware <i>Used with 12" back height with omit push handles, 16" back height with fold down push handles or 13" back height with standard or low profile push handles</i>	1
1e, 3a, 4, 5	110715	Fixed Height Rigid Backrest 11.5", 10" Wide with 0" Connector and Hardware <i>Used with 13" back height with omit push handles, 17" back height with fold down push handles or 14" back height with standard or low profile push handles</i>	1
1e, 3b, 4, 5	110716	Fixed Height Rigid Backrest 11.5", 10" Wide with 1" Connector and Hardware <i>Used with 13" back height with omit push handles, 17" back height with fold down push handles or 14" back height with standard or low profile push handles</i>	1
1e, 3c, 4, 5	110717	Fixed Height Rigid Backrest 11.5", 10" Wide with 2" Connector and Hardware <i>Used with 13" back height with omit push handles, 17" back height with fold down push handles or 14" back height with standard or low profile push handles</i>	1
1e, 3d, 4, 5	110718	Fixed Height Rigid Backrest 11.5", 10" Wide with 3" Connector and Hardware <i>Used with 13" back height with omit push handles, 17" back height with fold down push handles or 14" back height with standard or low profile push handles</i>	1
1f, 3a, 4, 5	110719	Fixed Height Rigid Backrest 12.5", 10" Wide with 0" Connector and Hardware <i>Used with 14" back height with omit push handles, 18" back height with fold down push handles or 15" back height with standard or low profile push handles</i>	1
1f, 3b, 4, 5	110720	Fixed Height Rigid Backrest 12.5", 10" Wide with 1" Connector and Hardware <i>Used with 14" back height with omit push handles, 18" back height with fold down push handles or 15" back height with standard or low profile push handles</i>	1
1f, 3c, 4, 5	110721	Fixed Height Rigid Backrest 12.5", 10" Wide with 2" Connector and Hardware <i>Used with 14" back height with omit push handles, 18" back height with fold down push handles or 15" back height with standard or low profile push handles</i>	1
1f, 3d, 4, 5	110722	Fixed Height Rigid Backrest 12.5", 10" Wide with 3" Connector and Hardware <i>Used with 14" back height with omit push handles, 18" back height with fold down push handles or 15" back height with standard or low profile push handles</i>	1

Item Number	Part Number	Description	Quantity
1g, 3a, 4, 5	110723	Fixed Height Rigid Backrest 13.5", 10" Wide with 0" Connector and Hardware <i>Used with 15" back height with omit push handles or 16" back height with standard or low profile push handles</i>	1
1g, 3b, 4, 5	110724	Fixed Height Rigid Backrest 13.5", 10" Wide with 1" Connector and Hardware <i>Used with 15" back height with omit push handles or 16" back height with standard or low profile push handles</i>	1
1g, 3c, 4, 5	110725	Fixed Height Rigid Backrest 13.5", 10" Wide with 2" Connector and Hardware <i>Used with 15" back height with omit push handles or 16" back height with standard or low profile push handles</i>	1
1g, 3d, 4, 5	110726	Fixed Height Rigid Backrest 13.5", 10" Wide with 3" Connector and Hardware <i>Used with 15" back height with omit push handles or 16" back height with standard or low profile push handles</i>	1
1h, 3a, 4, 5	110727	Fixed Height Rigid Backrest 14.5", 10" Wide with 0" Connector and Hardware <i>Used with 16" back height with omit push handles or 17" back height with standard or low profile push handles</i>	1
1h, 3b, 4, 5	110728	Fixed Height Rigid Backrest 14.5", 10" Wide with 1" Connector and Hardware <i>Used with 16" back height with omit push handles or 17" back height with standard or low profile push handles</i>	1
1h, 3c, 4, 5	110729	Fixed Height Rigid Backrest 14.5", 10" Wide with 2" Connector and Hardware <i>Used with 16" back height with omit push handles or 17" back height with standard or low profile push handles</i>	1
1h, 3d, 4, 5	110730	Fixed Height Rigid Backrest 14.5", 10" Wide with 3" Connector and Hardware <i>Used with 16" back height with omit push handles or 17" back height with standard or low profile push handles</i>	1
1i, 3a, 4, 5	110731	Fixed Height Rigid Backrest 15.5", 10" Wide with 0" Connector and Hardware <i>Used with 17" back height with omit push handles or 18" back height with standard or low profile push handles</i>	1
1i, 3b, 4, 5	110732	Fixed Height Rigid Backrest 15.5", 10" Wide with 1" Connector and Hardware <i>Used with 17" back height with omit push handles or 18" back height with standard or low profile push handles</i>	1
1i, 3c, 4, 5	110733	Fixed Height Rigid Backrest 15.5", 10" Wide with 2" Connector and Hardware <i>Used with 17" back height with omit push handles or 18" back height with standard or low profile push handles</i>	1
1i, 3d, 4, 5	110734	Fixed Height Rigid Backrest 15.5", 10" Wide with 3" Connector and Hardware <i>Used with 17" back height with omit push handles or 18" back height with standard or low profile push handles</i>	1
1j, 3a, 4, 5	110735	Fixed Height Rigid Backrest 16.5", 10" Wide with 0" Connector and Hardware <i>Used with 18" back height with omit push handles</i>	1
1j, 3b, 4, 5	110736	Fixed Height Rigid Backrest 16.5", 10" Wide with 1" Connector and Hardware <i>Used with 18" back height with omit push handles</i>	1
1j, 3c, 4, 5	110737	Fixed Height Rigid Backrest 16.5", 10" Wide with 2" Connector and Hardware <i>Used with 18" back height with omit push handles</i>	1

Item Number	Part Number	Description	Quantity
1j, 3d, 4, 5	110738	Fixed Height Rigid Backrest 16.5", 10" Wide with 3" Connector and Hardware <i>Used with 18" back height with omit push handles</i>	1
1k, 3a, 4, 5	110739	Fixed Height Rigid Backrest 7.5", 11" Wide with 0" Connector and Hardware <i>Used with 9" back height with omit push handles, 13" back height with fold down push handles or 10" back height with standard or low profile push handles</i>	1
1k, 3b, 4, 5	110740	Fixed Height Rigid Backrest 7.5", 11" Wide with 1" Connector and Hardware <i>Used with 9" back height with omit push handles, 13" back height with fold down push handles or 10" back height with standard or low profile push handles</i>	1
1k, 3c, 4, 5	110741	Fixed Height Rigid Backrest 7.5", 11" Wide with 2" Connector and Hardware <i>Used with 9" back height with omit push handles, 13" back height with fold down push handles or 10" back height with standard or low profile push handles</i>	1
1k, 3d, 4, 5	110742	Fixed Height Rigid Backrest 7.5", 11" Wide with 3" Connector and Hardware <i>Used with 9" back height with omit push handles, 13" back height with fold down push handles or 10" back height with standard or low profile push handles</i>	1
1l, 3a, 4, 5	110743	Fixed Height Rigid Backrest 8.5", 11" Wide with 0" Connector and Hardware <i>Used with 10" back height with omit push handles, 14" back height with fold down push handles or 11" back height with standard or low profile push handles</i>	1
1l, 3b, 4, 5	110744	Fixed Height Rigid Backrest 8.5", 11" Wide with 1" Connector and Hardware <i>Used with 10" back height with omit push handles, 14" back height with fold down push handles or 11" back height with standard or low profile push handles</i>	1
1l, 3c, 4, 5	110745	Fixed Height Rigid Backrest 8.5", 11" Wide with 2" Connector and Hardware <i>Used with 10" back height with omit push handles, 14" back height with fold down push handles or 11" back height with standard or low profile push handles</i>	1
1l, 3d, 4, 5	110746	Fixed Height Rigid Backrest 8.5", 11" Wide with 3" Connector and Hardware <i>Used with 10" back height with omit push handles, 14" back height with fold down push handles or 11" back height with standard or low profile push handles</i>	1
1m, 3a, 4, 5	110747	Fixed Height Rigid Backrest 9.5", 11" Wide with 0" Connector and Hardware <i>Used with 11" back height with omit push handles, 15" back height with fold down push handles or 12" back height with standard or low profile push handles</i>	1
1m, 3b, 4, 5	110748	Fixed Height Rigid Backrest 9.5", 11" Wide with 1" Connector and Hardware <i>Used with 11" back height with omit push handles, 15" back height with fold down push handles or 12" back height with standard or low profile push handles</i>	1
1m, 3c, 4, 5	110749	Fixed Height Rigid Backrest 9.5", 11" Wide with 2" Connector and Hardware <i>Used with 11" back height with omit push handles, 15" back height with fold down push handles or 12" back height with standard or low profile push handles</i>	1
1m, 3d, 4, 5	110750	Fixed Height Rigid Backrest 9.5", 11" Wide with 3" Connector and Hardware	1

Item Number	Part Number	Description	Quantity
		<i>Used with 11" back height with omit push handles, 15" back height with fold down push handles or 12" back height with standard or low profile push handles</i>	
1n, 3a, 4, 5	110751	Fixed Height Rigid Backrest 10.5", 11" Wide with 0" Connector and Hardware <i>Used with 12" back height with omit push handles, 16" back height with fold down push handles or 13" back height with standard or low profile push handles</i>	1
1n, 3b, 4, 5	110752	Fixed Height Rigid Backrest 10.5", 11" Wide with 1" Connector and Hardware <i>Used with 12" back height with omit push handles, 16" back height with fold down push handles or 13" back height with standard or low profile push handles</i>	1
1n, 3c, 4, 5	110753	Fixed Height Rigid Backrest 10.5", 11" Wide with 2" Connector and Hardware <i>Used with 12" back height with omit push handles, 16" back height with fold down push handles or 13" back height with standard or low profile push handles</i>	1
1n, 3d, 4, 5	110754	Fixed Height Rigid Backrest 10.5", 11" Wide with 3" Connector and Hardware <i>Used with 12" back height with omit push handles, 16" back height with fold down push handles or 13" back height with standard or low profile push handles</i>	1
1o, 3a, 4, 5	110755	Fixed Height Rigid Backrest 11.5", 11" Wide with 0" Connector and Hardware <i>Used with 13" back height with omit push handles, 17" back height with fold down push handles or 14" back height with standard or low profile push handles</i>	1
1o, 3b, 4, 5	110756	Fixed Height Rigid Backrest 11.5", 11" Wide with 1" Connector and Hardware <i>Used with 13" back height with omit push handles, 17" back height with fold down push handles or 14" back height with standard or low profile push handles</i>	1
1o, 3c, 4, 5	110757	Fixed Height Rigid Backrest 11.5", 11" Wide with 2" Connector and Hardware <i>Used with 13" back height with omit push handles, 17" back height with fold down push handles or 14" back height with standard or low profile push handles</i>	1
1o, 3d, 4, 5	110758	Fixed Height Rigid Backrest 11.5", 11" Wide with 3" Connector and Hardware <i>Used with 13" back height with omit push handles, 17" back height with fold down push handles or 14" back height with standard or low profile push handles</i>	1
1p, 3a, 4, 5	110759	Fixed Height Rigid Backrest 12.5", 11" Wide with 0" Connector and Hardware <i>Used with 14" back height with omit push handles, 18" back height with fold down push handles or 15" back height with standard or low profile push handles</i>	1
1p, 3b, 4, 5	110760	Fixed Height Rigid Backrest 12.5", 11" Wide with 1" Connector and Hardware <i>Used with 14" back height with omit push handles, 18" back height with fold down push handles or 15" back height with standard or low profile push handles</i>	1
1p, 3c, 4, 5	110761	Fixed Height Rigid Backrest 12.5", 11" Wide with 2" Connector and Hardware <i>Used with 14" back height with omit push handles, 18" back height with fold down push handles or 15" back height with standard or low profile push handles</i>	1
1p, 3d, 4, 5	110762	Fixed Height Rigid Backrest 12.5", 11" Wide with 3" Connector and Hardware	1

Item Number	Part Number	Description	Quantity
		<i>Used with 14" back height with omit push handles, 18" back height with fold down push handles or 15" back height with standard or low profile push handles</i>	
1q, 3a, 4, 5	110763	Fixed Height Rigid Backrest 13.5", 11" Wide with 0" Connector and Hardware <i>Used with 15" back height with omit push handles or 16" back height with standard or low profile push handles</i>	1
1q, 3b, 4, 5	110764	Fixed Height Rigid Backrest 13.5", 11" Wide with 1" Connector and Hardware <i>Used with 15" back height with omit push handles or 16" back height with standard or low profile push handles</i>	1
1q, 3c, 4, 5	110765	Fixed Height Rigid Backrest 13.5", 11" Wide with 2" Connector and Hardware <i>Used with 15" back height with omit push handles or 16" back height with standard or low profile push handles</i>	1
1q, 3d, 4, 5	110766	Fixed Height Rigid Backrest 13.5", 11" Wide with 3" Connector and Hardware <i>Used with 15" back height with omit push handles or 16" back height with standard or low profile push handles</i>	1
1r, 3a, 4, 5	110767	Fixed Height Rigid Backrest 14.5", 11" Wide with 0" Connector and Hardware <i>Used with 16" back height with omit push handles or 17" back height with standard or low profile push handles</i>	1
1r, 3b, 4, 5	110768	Fixed Height Rigid Backrest 14.5", 11" Wide with 1" Connector and Hardware <i>Used with 16" back height with omit push handles or 17" back height with standard or low profile push handles</i>	1
1r, 3c, 4, 5	110769	Fixed Height Rigid Backrest 14.5", 11" Wide with 2" Connector and Hardware <i>Used with 16" back height with omit push handles or 17" back height with standard or low profile push handles</i>	1
1r, 3d, 4, 5	110770	Fixed Height Rigid Backrest 14.5", 11" Wide with 3" Connector and Hardware <i>Used with 16" back height with omit push handles or 17" back height with standard or low profile push handles</i>	1
1s, 3a, 4, 5	110771	Fixed Height Rigid Backrest 15.5", 11" Wide with 0" Connector and Hardware <i>Used with 17" back height with omit push handles or 18" back height with standard or low profile push handles</i>	1
1s, 3b, 4, 5	110772	Fixed Height Rigid Backrest 15.5", 11" Wide with 1" Connector and Hardware <i>Used with 17" back height with omit push handles or 18" back height with standard or low profile push handles</i>	1
1s, 3c, 4, 5	110773	Fixed Height Rigid Backrest 15.5", 11" Wide with 2" Connector and Hardware <i>Used with 17" back height with omit push handles or 18" back height with standard or low profile push handles</i>	1
1s, 3d, 4, 5	110774	Fixed Height Rigid Backrest 15.5", 11" Wide with 3" Connector and Hardware <i>Used with 17" back height with omit push handles or 18" back height with standard or low profile push handles</i>	1
1t, 3a, 4, 5	110775	Fixed Height Rigid Backrest 16.5", 11" Wide with 0" Connector and Hardware <i>Used with 18" back height with omit push handles</i>	1
1t, 3b, 4, 5	110776	Fixed Height Rigid Backrest 16.5", 11" Wide with 1" Connector and Hardware <i>Used with 18" back height with omit push handles</i>	1

Item Number	Part Number	Description	Quantity
1t, 3c, 4, 5	110777	Fixed Height Rigid Backrest 16.5", 11" Wide with 2" Connector and Hardware <i>Used with 18" back height with omit push handles</i>	1
1t, 3d, 4, 5	110778	Fixed Height Rigid Backrest 16.5", 11" Wide with 3" Connector and Hardware <i>Used with 18" back height with omit push handles</i>	1
1u, 3a, 4, 5	110779	Fixed Height Rigid Backrest 7.5", 12" Wide with 0" Connector and Hardware <i>Used with 9" back height with omit push handles, 13" back height with fold down push handles or 10" back height with standard or low profile push handles</i>	1
1u, 3b, 4, 5	110780	Fixed Height Rigid Backrest 7.5", 12" Wide with 1" Connector and Hardware <i>Used with 9" back height with omit push handles, 13" back height with fold down push handles or 10" back height with standard or low profile push handles</i>	1
1u, 3c, 4, 5	110781	Fixed Height Rigid Backrest 7.5", 12" Wide with 2" Connector and Hardware <i>Used with 9" back height with omit push handles, 13" back height with fold down push handles or 10" back height with standard or low profile push handles</i>	1
1u, 3d, 4, 5	110782	Fixed Height Rigid Backrest 7.5", 12" Wide with 3" Connector and Hardware <i>Used with 9" back height with omit push handles, 13" back height with fold down push handles or 10" back height with standard or low profile push handles</i>	1
1v, 3a, 4, 5	110783	Fixed Height Rigid Backrest 8.5", 12" Wide with 0" Connector and Hardware <i>Used with 10" back height with omit push handles, 14" back height with fold down push handles or 11" back height with standard or low profile push handles</i>	1
1v, 3b, 4, 5	110784	Fixed Height Rigid Backrest 8.5", 12" Wide with 1" Connector and Hardware <i>Used with 10" back height with omit push handles, 14" back height with fold down push handles or 11" back height with standard or low profile push handles</i>	1
1v, 3c, 4, 5	110785	Fixed Height Rigid Backrest 8.5", 12" Wide with 2" Connector and Hardware <i>Used with 10" back height with omit push handles, 14" back height with fold down push handles or 11" back height with standard or low profile push handles</i>	1
1v, 3d, 4, 5	110786	Fixed Height Rigid Backrest 8.5", 12" Wide with 3" Connector and Hardware <i>Used with 10" back height with omit push handles, 14" back height with fold down push handles or 11" back height with standard or low profile push handles</i>	1
1w, 3a, 4, 5	110787	Fixed Height Rigid Backrest 9.5", 12" Wide with 0" Connector and Hardware <i>Used with 11" back height with omit push handles, 15" back height with fold down push handles or 12" back height with standard or low profile push handles</i>	1
1w, 3b, 4, 5	110788	Fixed Height Rigid Backrest 9.5", 12" Wide with 1" Connector and Hardware <i>Used with 11" back height with omit push handles, 15" back height with fold down push handles or 12" back height with standard or low profile push handles</i>	1
1w, 3c, 4, 5	110789	Fixed Height Rigid Backrest 9.5", 12" Wide with 2" Connector and Hardware <i>Used with 11" back height with omit push handles, 15" back height with fold down push handles or 12" back height with standard or low profile push handles</i>	1



Item Number	Part Number	Description	Quantity
1w, 3d, 4, 5	110790	Fixed Height Rigid Backrest 9.5", 12" Wide with 3" Connector and Hardware <i>Used with 11" back height with omit push handles, 15" back height with fold down push handles or 12" back height with standard or low profile push handles</i>	1
1x, 3a, 4, 5	110791	Fixed Height Rigid Backrest 10.5", 12" Wide with 0" Connector and Hardware <i>Used with 12" back height with omit push handles, 16" back height with fold down push handles or 13" back height with standard or low profile push handles</i>	1
1x, 3b, 4, 5	110792	Fixed Height Rigid Backrest 10.5", 12" Wide with 1" Connector and Hardware <i>Used with 12" back height with omit push handles, 16" back height with fold down push handles or 13" back height with standard or low profile push handles</i>	1
1x, 3c, 4, 5	110793	Fixed Height Rigid Backrest 10.5", 12" Wide with 2" Connector and Hardware <i>Used with 12" back height with omit push handles, 16" back height with fold down push handles or 13" back height with standard or low profile push handles</i>	1
1x, 3d, 4, 5	110794	Fixed Height Rigid Backrest 10.5", 12" Wide with 3" Connector and Hardware <i>Used with 12" back height with omit push handles, 16" back height with fold down push handles or 13" back height with standard or low profile push handles</i>	1
1y, 3a, 4, 5	110795	Fixed Height Rigid Backrest 11.5", 12" Wide with 0" Connector and Hardware <i>Used with 13" back height with omit push handles, 17" back height with fold down push handles or 14" back height with standard or low profile push handles</i>	1
1y, 3b, 4, 5	110796	Fixed Height Rigid Backrest 11.5", 12" Wide with 1" Connector and Hardware <i>Used with 13" back height with omit push handles, 17" back height with fold down push handles or 14" back height with standard or low profile push handles</i>	1
1y, 3c, 4, 5	110797	Fixed Height Rigid Backrest 11.5", 12" Wide with 2" Connector and Hardware <i>Used with 13" back height with omit push handles, 17" back height with fold down push handles or 14" back height with standard or low profile push handles</i>	1
1y, 3d, 4, 5	110798	Fixed Height Rigid Backrest 11.5", 12" Wide with 3" Connector and Hardware <i>Used with 13" back height with omit push handles, 17" back height with fold down push handles or 14" back height with standard or low profile push handles</i>	1
1z, 3a, 4, 5	110799	Fixed Height Rigid Backrest 12.5", 12" Wide with 0" Connector and Hardware <i>Used with 14" back height with omit push handles, 18" back height with fold down push handles or 15" back height with standard or low profile push handles</i>	1
1z, 3b, 4, 5	110800	Fixed Height Rigid Backrest 12.5", 12" Wide with 1" Connector and Hardware <i>Used with 14" back height with omit push handles, 18" back height with fold down push handles or 15" back height with standard or low profile push handles</i>	1
1z, 3c, 4, 5	110801	Fixed Height Rigid Backrest 12.5", 12" Wide with 2" Connector and Hardware <i>Used with 14" back height with omit push handles, 18" back height with fold down push handles or 15" back height with standard or low profile push handles</i>	1

Item Number	Part Number	Description	Quantity
1z, 3d, 4, 5	110802	Fixed Height Rigid Backrest 12.5", 12" Wide with 3" Connector and Hardware <i>Used with 14" back height with omit push handles, 18" back height with fold down push handles or 15" back height with standard or low profile push handles</i>	1
1aa, 3a, 4, 5	110803	Fixed Height Rigid Backrest 13.5", 12" Wide with 0" Connector and Hardware <i>Used with 15" back height with omit push handles or 16" back height with standard or low profile push handles</i>	1
1aa, 3b, 4, 5	110804	Fixed Height Rigid Backrest 13.5", 12" Wide with 1" Connector and Hardware <i>Used with 15" back height with omit push handles or 16" back height with standard or low profile push handles</i>	1
1aa, 3c, 4, 5	110805	Fixed Height Rigid Backrest 13.5", 12" Wide with 2" Connector and Hardware <i>Used with 15" back height with omit push handles or 16" back height with standard or low profile push handles</i>	1
1aa, 3d, 4, 5	110806	Fixed Height Rigid Backrest 13.5", 12" Wide with 3" Connector and Hardware <i>Used with 15" back height with omit push handles or 16" back height with standard or low profile push handles</i>	1
1ab, 3a, 4, 5	110807	Fixed Height Rigid Backrest 14.5", 12" Wide with 0" Connector and Hardware <i>Used with 16" back height with omit push handles or 17" back height with standard or low profile push handles</i>	1
1ab, 3b, 4, 5	110808	Fixed Height Rigid Backrest 14.5", 12" Wide with 1" Connector and Hardware <i>Used with 16" back height with omit push handles or 17" back height with standard or low profile push handles</i>	1
1ab, 3c, 4, 5	110809	Fixed Height Rigid Backrest 14.5", 12" Wide with 2" Connector and Hardware <i>Used with 16" back height with omit push handles or 17" back height with standard or low profile push handles</i>	1
1ab, 3d, 4, 5	110810	Fixed Height Rigid Backrest 14.5", 12" Wide with 3" Connector and Hardware <i>Used with 16" back height with omit push handles or 17" back height with standard or low profile push handles</i>	1
1ac, 3a, 4, 5	110811	Fixed Height Rigid Backrest 15.5", 12" Wide with 0" Connector and Hardware <i>Used with 17" back height with omit push handles or 18" back height with standard or low profile push handles</i>	1
1ac, 3b, 4, 5	110812	Fixed Height Rigid Backrest 15.5", 12" Wide with 1" Connector and Hardware <i>Used with 17" back height with omit push handles or 18" back height with standard or low profile push handles</i>	1
1ac, 3c, 4, 5	110813	Fixed Height Rigid Backrest 15.5", 12" Wide with 2" Connector and Hardware <i>Used with 17" back height with omit push handles or 18" back height with standard or low profile push handles</i>	1
1ac, 3d, 4, 5	110814	Fixed Height Rigid Backrest 15.5", 12" Wide with 3" Connector and Hardware <i>Used with 17" back height with omit push handles or 18" back height with standard or low profile push handles</i>	1
1ad, 3a, 4, 5	110815	Fixed Height Rigid Backrest 16.5", 12" Wide with 0" Connector and Hardware <i>Used with 18" back height with omit push handles</i>	1
1ad, 3b, 4, 5	110816	Fixed Height Rigid Backrest 16.5", 12" Wide with 1" Connector and Hardware	1

Item Number	Part Number	Description	Quantity
		<i>Used with 18" back height with omit push handles</i>	
1ad, 3c, 4, 5	110817	Fixed Height Rigid Backrest 16.5", 12" Wide with 2" Connector and Hardware <i>Used with 18" back height with omit push handles</i>	1
1ad, 3d, 4, 5	110818	Fixed Height Rigid Backrest 16.5", 12" Wide with 3" Connector and Hardware <i>Used with 18" back height with omit push handles</i>	1
1ae, 3a, 4, 5	110819	Fixed Height Rigid Backrest 7.5", 13" Wide with 0" Connector and Hardware <i>Used with 9" back height with omit push handles, 13" back height with fold down push handles or 10" back height with standard or low profile push handles</i>	1
1ae, 3b, 4, 5	110820	Fixed Height Rigid Backrest 7.5", 13" Wide with 1" Connector and Hardware <i>Used with 9" back height with omit push handles, 13" back height with fold down push handles or 10" back height with standard or low profile push handles</i>	1
1ae, 3c, 4, 5	110821	Fixed Height Rigid Backrest 7.5", 13" Wide with 2" Connector and Hardware <i>Used with 9" back height with omit push handles, 13" back height with fold down push handles or 10" back height with standard or low profile push handles</i>	1
1ae, 3d, 4, 5	110822	Fixed Height Rigid Backrest 7.5", 13" Wide with 3" Connector and Hardware <i>Used with 9" back height with omit push handles, 13" back height with fold down push handles or 10" back height with standard or low profile push handles</i>	1
1af, 3a, 4, 5	110823	Fixed Height Rigid Backrest 8.5", 13" Wide with 0" Connector and Hardware <i>Used with 10" back height with omit push handles, 14" back height with fold down push handles or 11" back height with standard or low profile push handles</i>	1
1af, 3b, 4, 5	110824	Fixed Height Rigid Backrest 8.5", 13" Wide with 1" Connector and Hardware <i>Used with 10" back height with omit push handles, 14" back height with fold down push handles or 11" back height with standard or low profile push handles</i>	1
1af, 3c, 4, 5	110825	Fixed Height Rigid Backrest 8.5", 13" Wide with 2" Connector and Hardware <i>Used with 10" back height with omit push handles, 14" back height with fold down push handles or 11" back height with standard or low profile push handles</i>	1
1af, 3d, 4, 5	110826	Fixed Height Rigid Backrest 8.5", 13" Wide with 3" Connector and Hardware <i>Used with 10" back height with omit push handles, 14" back height with fold down push handles or 11" back height with standard or low profile push handles</i>	1
1ag, 3a, 4, 5	110827	Fixed Height Rigid Backrest 9.5", 13" Wide with 0" Connector and Hardware <i>Used with 11" back height with omit push handles, 15" back height with fold down push handles or 12" back height with standard or low profile push handles</i>	1
1ag, 3b, 4, 5	110828	Fixed Height Rigid Backrest 9.5", 13" Wide with 1" Connector and Hardware <i>Used with 11" back height with omit push handles, 15" back height with fold down push handles or 12" back height with standard or low profile push handles</i>	1
1ag, 3c, 4, 5	110829	Fixed Height Rigid Backrest 9.5", 13" Wide with 2" Connector and Hardware	1

Item Number	Part Number	Description	Quantity
		<i>Used with 11" back height with omit push handles, 15" back height with fold down push handles or 12" back height with standard or low profile push handles</i>	
1ag, 3d, 4, 5	110830	Fixed Height Rigid Backrest 9.5", 13" Wide with 3" Connector and Hardware <i>Used with 11" back height with omit push handles, 15" back height with fold down push handles or 12" back height with standard or low profile push handles</i>	1
1ah, 3a, 4, 5	110831	Fixed Height Rigid Backrest 10.5", 13" Wide with 0" Connector and Hardware <i>Used with 12" back height with omit push handles, 16" back height with fold down push handles or 13" back height with standard or low profile push handles</i>	1
1ah, 3b, 4, 5	110832	Fixed Height Rigid Backrest 10.5", 13" Wide with 1" Connector and Hardware <i>Used with 12" back height with omit push handles, 16" back height with fold down push handles or 13" back height with standard or low profile push handles</i>	1
1ah, 3c, 4, 5	110833	Fixed Height Rigid Backrest 10.5", 13" Wide with 2" Connector and Hardware <i>Used with 12" back height with omit push handles, 16" back height with fold down push handles or 13" back height with standard or low profile push handles</i>	1
1ah, 3d, 4, 5	110834	Fixed Height Rigid Backrest 10.5", 13" Wide with 3" Connector and Hardware <i>Used with 12" back height with omit push handles, 16" back height with fold down push handles or 13" back height with standard or low profile push handles</i>	1
1ai, 3a, 4, 5	110835	Fixed Height Rigid Backrest 11.5", 13" Wide with 0" Connector and Hardware <i>Used with 13" back height with omit push handles, 17" back height with fold down push handles or 14" back height with standard or low profile push handles</i>	1
1ai, 3b, 4, 5	110836	Fixed Height Rigid Backrest 11.5", 13" Wide with 1" Connector and Hardware <i>Used with 13" back height with omit push handles, 17" back height with fold down push handles or 14" back height with standard or low profile push handles</i>	1
1ai, 3c, 4, 5	110837	Fixed Height Rigid Backrest 11.5", 13" Wide with 2" Connector and Hardware <i>Used with 13" back height with omit push handles, 17" back height with fold down push handles or 14" back height with standard or low profile push handles</i>	1
1ai, 3d, 4, 5	110838	Fixed Height Rigid Backrest 11.5", 13" Wide with 3" Connector and Hardware <i>Used with 13" back height with omit push handles, 17" back height with fold down push handles or 14" back height with standard or low profile push handles</i>	1
1aj, 3a, 4, 5	110839	Fixed Height Rigid Backrest 12.5", 13" Wide with 0" Connector and Hardware <i>Used with 14" back height with omit push handles, 18" back height with fold down push handles or 15" back height with standard or low profile push handles</i>	1
1aj, 3b, 4, 5	110840	Fixed Height Rigid Backrest 12.5", 13" Wide with 1" Connector and Hardware <i>Used with 14" back height with omit push handles, 18" back height with fold down push handles or 15" back height with standard or low profile push handles</i>	1
1aj, 3c, 4, 5	110841	Fixed Height Rigid Backrest 12.5", 13" Wide with 2" Connector and Hardware	1

Item Number	Part Number	Description	Quantity
		<i>Used with 14" back height with omit push handles, 18" back height with fold down push handles or 15" back height with standard or low profile push handles</i>	
1aj, 3d, 4, 5	110842	Fixed Height Rigid Backrest 12.5", 13" Wide with 3" Connector and Hardware <i>Used with 14" back height with omit push handles, 18" back height with fold down push handles or 15" back height with standard or low profile push handles</i>	1
1ak, 3a, 4, 5	110843	Fixed Height Rigid Backrest 13.5", 13" Wide with 0" Connector and Hardware <i>Used with 15" back height with omit push handles or 16" back height with standard or low profile push handles</i>	1
1ak, 3b, 4, 5	110844	Fixed Height Rigid Backrest 13.5", 13" Wide with 1" Connector and Hardware <i>Used with 15" back height with omit push handles or 16" back height with standard or low profile push handles</i>	1
1ak, 3c, 4, 5	110845	Fixed Height Rigid Backrest 13.5", 13" Wide with 2" Connector and Hardware <i>Used with 15" back height with omit push handles or 16" back height with standard or low profile push handles</i>	1
1ak, 3d, 4, 5	110846	Fixed Height Rigid Backrest 13.5", 13" Wide with 3" Connector and Hardware <i>Used with 15" back height with omit push handles or 16" back height with standard or low profile push handles</i>	1
1al, 3a, 4, 5	110847	Fixed Height Rigid Backrest 14.5", 13" Wide with 0" Connector and Hardware <i>Used with 16" back height with omit push handles or 17" back height with standard or low profile push handles</i>	1
1al, 3b, 4, 5	110848	Fixed Height Rigid Backrest 14.5", 13" Wide with 1" Connector and Hardware <i>Used with 16" back height with omit push handles or 17" back height with standard or low profile push handles</i>	1
1al, 3c, 4, 5	110849	Fixed Height Rigid Backrest 14.5", 13" Wide with 2" Connector and Hardware <i>Used with 16" back height with omit push handles or 17" back height with standard or low profile push handles</i>	1
1al, 3d, 4, 5	110850	Fixed Height Rigid Backrest 14.5", 13" Wide with 3" Connector and Hardware <i>Used with 16" back height with omit push handles or 17" back height with standard or low profile push handles</i>	1
1am, 3a, 4, 5	110851	Fixed Height Rigid Backrest 15.5", 13" Wide with 0" Connector and Hardware <i>Used with 17" back height with omit push handles or 18" back height with standard or low profile push handles</i>	1
1am, 3b, 4, 5	110852	Fixed Height Rigid Backrest 15.5", 13" Wide with 1" Connector and Hardware <i>Used with 17" back height with omit push handles or 18" back height with standard or low profile push handles</i>	1
1am, 3c, 4, 5	110853	Fixed Height Rigid Backrest 15.5", 13" Wide with 2" Connector and Hardware <i>Used with 17" back height with omit push handles or 18" back height with standard or low profile push handles</i>	1
1am, 3d, 4, 5	110854	Fixed Height Rigid Backrest 15.5", 13" Wide with 3" Connector and Hardware <i>Used with 17" back height with omit push handles or 18" back height with standard or low profile push handles</i>	1
1an, 3a, 4, 5	110855	Fixed Height Rigid Backrest 16.5", 13" Wide with 0" Connector and Hardware <i>Used with 18" back height with omit push handles</i>	1

Item Number	Part Number	Description	Quantity
1an, 3b, 4, 5	110856	Fixed Height Rigid Backrest 16.5", 13" Wide with 1" Connector and Hardware <i>Used with 18" back height with omit push handles</i>	1
1an, 3c, 4, 5	110857	Fixed Height Rigid Backrest 16.5", 13" Wide with 2" Connector and Hardware <i>Used with 18" back height with omit push handles</i>	1
1an, 3d, 4, 5	110858	Fixed Height Rigid Backrest 16.5", 13" Wide with 3" Connector and Hardware <i>Used with 18" back height with omit push handles</i>	1
1ao, 3a, 4, 5	110859	Fixed Height Rigid Backrest 7.5", 14" Wide with 0" Connector and Hardware <i>Used with 9" back height with omit push handles, 13" back height with fold down push handles or 10" back height with standard or low profile push handles</i>	1
1ao, 3b, 4, 5	110860	Fixed Height Rigid Backrest 7.5", 14" Wide with 1" Connector and Hardware <i>Used with 9" back height with omit push handles, 13" back height with fold down push handles or 10" back height with standard or low profile push handles</i>	1
1ao, 3c, 4, 5	110861	Fixed Height Rigid Backrest 7.5", 14" Wide with 2" Connector and Hardware <i>Used with 9" back height with omit push handles, 13" back height with fold down push handles or 10" back height with standard or low profile push handles</i>	1
1ao, 3d, 4, 5	110862	Fixed Height Rigid Backrest 7.5", 14" Wide with 3" Connector and Hardware <i>Used with 9" back height with omit push handles, 13" back height with fold down push handles or 10" back height with standard or low profile push handles</i>	1
1ap, 3a, 4, 5	110863	Fixed Height Rigid Backrest 8.5", 14" Wide with 0" Connector and Hardware <i>Used with 10" back height with omit push handles, 14" back height with fold down push handles or 11" back height with standard or low profile push handles</i>	1
1ap, 3b, 4, 5	110864	Fixed Height Rigid Backrest 8.5", 14" Wide with 1" Connector and Hardware <i>Used with 10" back height with omit push handles, 14" back height with fold down push handles or 11" back height with standard or low profile push handles</i>	1
1ap, 3c, 4, 5	110865	Fixed Height Rigid Backrest 8.5", 14" Wide with 2" Connector and Hardware <i>Used with 10" back height with omit push handles, 14" back height with fold down push handles or 11" back height with standard or low profile push handles</i>	1
1ap, 3d, 4, 5	110866	Fixed Height Rigid Backrest 8.5", 14" Wide with 3" Connector and Hardware <i>Used with 10" back height with omit push handles, 14" back height with fold down push handles or 11" back height with standard or low profile push handles</i>	1
1aq, 3a, 4, 5	110867	Fixed Height Rigid Backrest 9.5", 14" Wide with 0" Connector and Hardware <i>Used with 11" back height with omit push handles, 15" back height with fold down push handles or 12" back height with standard or low profile push handles</i>	1
1aq, 3b, 4, 5	110868	Fixed Height Rigid Backrest 9.5", 14" Wide with 1" Connector and Hardware <i>Used with 11" back height with omit push handles, 15" back height with fold down push handles or 12" back height with standard or low profile push handles</i>	1

Item Number	Part Number	Description	Quantity
1aq, 3c, 4, 5	110869	Fixed Height Rigid Backrest 9.5", 14" Wide with 2" Connector and Hardware <i>Used with 11" back height with omit push handles, 15" back height with fold down push handles or 12" back height with standard or low profile push handles</i>	1
1aq, 3d, 4, 5	110870	Fixed Height Rigid Backrest 9.5", 14" Wide with 3" Connector and Hardware <i>Used with 11" back height with omit push handles, 15" back height with fold down push handles or 12" back height with standard or low profile push handles</i>	1
1ar, 3a, 4, 5	110871	Fixed Height Rigid Backrest 10.5", 14" Wide with 0" Connector and Hardware <i>Used with 12" back height with omit push handles, 16" back height with fold down push handles or 13" back height with standard or low profile push handles</i>	1
1ar, 3b, 4, 5	110872	Fixed Height Rigid Backrest 10.5", 14" Wide with 1" Connector and Hardware <i>Used with 12" back height with omit push handles, 16" back height with fold down push handles or 13" back height with standard or low profile push handles</i>	1
1ar, 3c, 4, 5	110873	Fixed Height Rigid Backrest 10.5", 14" Wide with 2" Connector and Hardware <i>Used with 12" back height with omit push handles, 16" back height with fold down push handles or 13" back height with standard or low profile push handles</i>	1
1ar, 3d, 4, 5	110874	Fixed Height Rigid Backrest 10.5", 14" Wide with 3" Connector and Hardware <i>Used with 12" back height with omit push handles, 16" back height with fold down push handles or 13" back height with standard or low profile push handles</i>	1
1as, 3a, 4, 5	110875	Fixed Height Rigid Backrest 11.5", 14" Wide with 0" Connector and Hardware <i>Used with 13" back height with omit push handles, 17" back height with fold down push handles or 14" back height with standard or low profile push handles</i>	1
1as, 3b, 4, 5	110876	Fixed Height Rigid Backrest 11.5", 14" Wide with 1" Connector and Hardware <i>Used with 13" back height with omit push handles, 17" back height with fold down push handles or 14" back height with standard or low profile push handles</i>	1
1as, 3c, 4, 5	110877	Fixed Height Rigid Backrest 11.5", 14" Wide with 2" Connector and Hardware <i>Used with 13" back height with omit push handles, 17" back height with fold down push handles or 14" back height with standard or low profile push handles</i>	1
1as, 3d, 4, 5	110878	Fixed Height Rigid Backrest 11.5", 14" Wide with 3" Connector and Hardware <i>Used with 13" back height with omit push handles, 17" back height with fold down push handles or 14" back height with standard or low profile push handles</i>	1
1at, 3a, 4, 5	110879	Fixed Height Rigid Backrest 12.5", 14" Wide with 0" Connector and Hardware <i>Used with 14" back height with omit push handles, 18" back height with fold down push handles or 15" back height with standard or low profile push handles</i>	1
1at, 3b, 4, 5	110880	Fixed Height Rigid Backrest 12.5", 14" Wide with 1" Connector and Hardware <i>Used with 14" back height with omit push handles, 18" back height with fold down push handles or 15" back height with standard or low profile push handles</i>	1

Item Number	Part Number	Description	Quantity
1at, 3c, 4, 5	110881	Fixed Height Rigid Backrest 12.5", 14" Wide with 2" Connector and Hardware <i>Used with 14" back height with omit push handles, 18" back height with fold down push handles or 15" back height with standard or low profile push handles</i>	1
1at, 3d, 4, 5	110882	Fixed Height Rigid Backrest 12.5", 14" Wide with 3" Connector and Hardware <i>Used with 14" back height with omit push handles, 18" back height with fold down push handles or 15" back height with standard or low profile push handles</i>	1
1au, 3a, 4, 5	110883	Fixed Height Rigid Backrest 13.5", 14" Wide with 0" Connector and Hardware <i>Used with 15" back height with omit push handles or 16" back height with standard or low profile push handles</i>	1
1au, 3b, 4, 5	110884	Fixed Height Rigid Backrest 13.5", 14" Wide with 1" Connector and Hardware <i>Used with 15" back height with omit push handles or 16" back height with standard or low profile push handles</i>	1
1au, 3c, 4, 5	110885	Fixed Height Rigid Backrest 13.5", 14" Wide with 2" Connector and Hardware <i>Used with 15" back height with omit push handles or 16" back height with standard or low profile push handles</i>	1
1au, 3d, 4, 5	110886	Fixed Height Rigid Backrest 13.5", 14" Wide with 3" Connector and Hardware <i>Used with 15" back height with omit push handles or 16" back height with standard or low profile push handles</i>	1
1av, 3a, 4, 5	110887	Fixed Height Rigid Backrest 14.5", 14" Wide with 0" Connector and Hardware <i>Used with 16" back height with omit push handles or 17" back height with standard or low profile push handles</i>	1
1av, 3b, 4, 5	110888	Fixed Height Rigid Backrest 14.5", 14" Wide with 1" Connector and Hardware <i>Used with 16" back height with omit push handles or 17" back height with standard or low profile push handles</i>	1
1av, 3c, 4, 5	110889	Fixed Height Rigid Backrest 14.5", 14" Wide with 2" Connector and Hardware <i>Used with 16" back height with omit push handles or 17" back height with standard or low profile push handles</i>	1
1av, 3d, 4, 5	110890	Fixed Height Rigid Backrest 14.5", 14" Wide with 3" Connector and Hardware <i>Used with 16" back height with omit push handles or 17" back height with standard or low profile push handles</i>	1
1aw, 3a, 4, 5	110891	Fixed Height Rigid Backrest 15.5", 14" Wide with 0" Connector and Hardware <i>Used with 17" back height with omit push handles or 18" back height with standard or low profile push handles</i>	1
1aw, 3b, 4, 5	110892	Fixed Height Rigid Backrest 15.5", 14" Wide with 1" Connector and Hardware <i>Used with 17" back height with omit push handles or 18" back height with standard or low profile push handles</i>	1
1aw, 3c, 4, 5	110893	Fixed Height Rigid Backrest 15.5", 14" Wide with 2" Connector and Hardware <i>Used with 17" back height with omit push handles or 18" back height with standard or low profile push handles</i>	1
1aw, 3d, 4, 5	110894	Fixed Height Rigid Backrest 15.5", 14" Wide with 3" Connector and Hardware <i>Used with 17" back height with omit push handles or 18" back height with standard or low profile push handles</i>	1



Item Number	Part Number	Description	Quantity
1ax, 3a, 4, 5	110895	Fixed Height Rigid Backrest 16.5", 14" Wide with 0" Connector and Hardware <i>Used with 18" back height with omit push handles</i>	1
1ax, 3b, 4, 5	110896	Fixed Height Rigid Backrest 16.5", 14" Wide with 1" Connector and Hardware <i>Used with 18" back height with omit push handles</i>	1
1ax, 3c, 4, 5	110897	Fixed Height Rigid Backrest 16.5", 14" Wide with 2" Connector and Hardware <i>Used with 18" back height with omit push handles</i>	1
1ax, 3d, 4, 5	110898	Fixed Height Rigid Backrest 16.5", 14" Wide with 3" Connector and Hardware <i>Used with 18" back height with omit push handles</i>	1
1ay, 3a, 4, 5	110899	Fixed Height Rigid Backrest 7.5", 15" Wide with 0" Connector and Hardware <i>Used with 9" back height with omit push handles, 13" back height with fold down push handles or 10" back height with standard or low profile push handles</i>	1
1ay, 3b, 4, 5	110900	Fixed Height Rigid Backrest 7.5", 15" Wide with 1" Connector and Hardware <i>Used with 9" back height with omit push handles, 13" back height with fold down push handles or 10" back height with standard or low profile push handles</i>	1
1ay, 3c, 4, 5	110901	Fixed Height Rigid Backrest 7.5", 15" Wide with 2" Connector and Hardware <i>Used with 9" back height with omit push handles, 13" back height with fold down push handles or 10" back height with standard or low profile push handles</i>	1
1ay, 3d, 4, 5	110902	Fixed Height Rigid Backrest 7.5", 15" Wide with 3" Connector and Hardware <i>Used with 9" back height with omit push handles, 13" back height with fold down push handles or 10" back height with standard or low profile push handles</i>	1
1az, 3a, 4, 5	110903	Fixed Height Rigid Backrest 8.5", 15" Wide with 0" Connector and Hardware <i>Used with 10" back height with omit push handles, 14" back height with fold down push handles or 11" back height with standard or low profile push handles</i>	1
1az, 3b, 4, 5	110904	Fixed Height Rigid Backrest 8.5", 15" Wide with 1" Connector and Hardware <i>Used with 10" back height with omit push handles, 14" back height with fold down push handles or 11" back height with standard or low profile push handles</i>	1
1az, 3c, 4, 5	110905	Fixed Height Rigid Backrest 8.5", 15" Wide with 2" Connector and Hardware <i>Used with 10" back height with omit push handles, 14" back height with fold down push handles or 11" back height with standard or low profile push handles</i>	1
1az, 3d, 4, 5	110906	Fixed Height Rigid Backrest 8.5", 15" Wide with 3" Connector and Hardware <i>Used with 10" back height with omit push handles, 14" back height with fold down push handles or 11" back height with standard or low profile push handles</i>	1
1ba, 3a, 4, 5	110907	Fixed Height Rigid Backrest 9.5", 15" Wide with 0" Connector and Hardware <i>Used with 11" back height with omit push handles, 15" back height with fold down push handles or 12" back height with standard or low profile push handles</i>	1
1ba, 3b, 4, 5	110908	Fixed Height Rigid Backrest 9.5", 15" Wide with 1" Connector and Hardware	1

Item Number	Part Number	Description	Quantity
		<i>Used with 11" back height with omit push handles, 15" back height with fold down push handles or 12" back height with standard or low profile push handles</i>	
1ba, 3c, 4, 5	110909	Fixed Height Rigid Backrest 9.5", 15" Wide with 2" Connector and Hardware <i>Used with 11" back height with omit push handles, 15" back height with fold down push handles or 12" back height with standard or low profile push handles</i>	1
1ba, 3d, 4, 5	110910	Fixed Height Rigid Backrest 9.5", 15" Wide with 3" Connector and Hardware <i>Used with 11" back height with omit push handles, 15" back height with fold down push handles or 12" back height with standard or low profile push handles</i>	1
1bb, 3a, 4, 5	110911	Fixed Height Rigid Backrest 10.5", 15" Wide with 0" Connector and Hardware <i>Used with 12" back height with omit push handles, 16" back height with fold down push handles or 13" back height with standard or low profile push handles</i>	1
1bb, 3b, 4, 5	110912	Fixed Height Rigid Backrest 10.5", 15" Wide with 1" Connector and Hardware <i>Used with 12" back height with omit push handles, 16" back height with fold down push handles or 13" back height with standard or low profile push handles</i>	1
1bb, 3c, 4, 5	110913	Fixed Height Rigid Backrest 10.5", 15" Wide with 2" Connector and Hardware <i>Used with 12" back height with omit push handles, 16" back height with fold down push handles or 13" back height with standard or low profile push handles</i>	1
1bb, 3d, 4, 5	110914	Fixed Height Rigid Backrest 10.5", 15" Wide with 3" Connector and Hardware <i>Used with 12" back height with omit push handles, 16" back height with fold down push handles or 13" back height with standard or low profile push handles</i>	1
1bc, 3a, 4, 5	110915	Fixed Height Rigid Backrest 11.5", 15" Wide with 0" Connector and Hardware <i>Used with 13" back height with omit push handles, 17" back height with fold down push handles or 14" back height with standard or low profile push handles</i>	1
1bc, 3b, 4, 5	110916	Fixed Height Rigid Backrest 11.5", 15" Wide with 1" Connector and Hardware <i>Used with 13" back height with omit push handles, 17" back height with fold down push handles or 14" back height with standard or low profile push handles</i>	1
1bc, 3b, 4, 5	110917	Fixed Height Rigid Backrest 11.5", 15" Wide with 2" Connector and Hardware <i>Used with 13" back height with omit push handles, 17" back height with fold down push handles or 14" back height with standard or low profile push handles</i>	1
1bc, 3b, 4, 5	110918	Fixed Height Rigid Backrest 11.5", 15" Wide with 3" Connector and Hardware <i>Used with 13" back height with omit push handles, 17" back height with fold down push handles or 14" back height with standard or low profile push handles</i>	1
1bd, 3a, 4, 5	110919	Fixed Height Rigid Backrest 12.5", 15" Wide with 0" Connector and Hardware <i>Used with 14" back height with omit push handles, 18" back height with fold down push handles or 15" back height with standard or low profile push handles</i>	1
1bd, 3b, 4, 5	110920	Fixed Height Rigid Backrest 12.5", 15" Wide with 1" Connector and Hardware	1

Item Number	Part Number	Description	Quantity
		<i>Used with 14" back height with omit push handles, 18" back height with fold down push handles or 15" back height with standard or low profile push handles</i>	
1bd, 3c, 4, 5	110921	Fixed Height Rigid Backrest 12.5", 15" Wide with 2" Connector and Hardware <i>Used with 14" back height with omit push handles, 18" back height with fold down push handles or 15" back height with standard or low profile push handles</i>	1
1bd, 3d, 4, 5	110922	Fixed Height Rigid Backrest 12.5", 15" Wide with 3" Connector and Hardware <i>Used with 14" back height with omit push handles, 18" back height with fold down push handles or 15" back height with standard or low profile push handles</i>	1
1be,3a, 4, 5	110923	Fixed Height Rigid Backrest 13.5", 15" Wide with 0" Connector and Hardware <i>Used with 15" back height with omit push handles or 16" back height with standard or low profile push handles</i>	1
1be,3b, 4, 5	110924	Fixed Height Rigid Backrest 13.5", 15" Wide with 1" Connector and Hardware <i>Used with 15" back height with omit push handles or 16" back height with standard or low profile push handles</i>	1
1be,3c, 4, 5	110925	Fixed Height Rigid Backrest 13.5", 15" Wide with 2" Connector and Hardware <i>Used with 15" back height with omit push handles or 16" back height with standard or low profile push handles</i>	1
1be,3d, 4, 5	110926	Fixed Height Rigid Backrest 13.5", 15" Wide with 3" Connector and Hardware <i>Used with 15" back height with omit push handles or 16" back height with standard or low profile push handles</i>	1
1bf, 3a, 4, 5	110927	Fixed Height Rigid Backrest 14.5", 15" Wide with 0" Connector and Hardware <i>Used with 16" back height with omit push handles or 17" back height with standard or low profile push handles</i>	1
1bf, 3b, 4, 5	110928	Fixed Height Rigid Backrest 14.5", 15" Wide with 1" Connector and Hardware <i>Used with 16" back height with omit push handles or 17" back height with standard or low profile push handles</i>	1
1bf, 3c, 4, 5	110929	Fixed Height Rigid Backrest 14.5", 15" Wide with 2" Connector and Hardware <i>Used with 16" back height with omit push handles or 17" back height with standard or low profile push handles</i>	1
1bf, 3d, 4, 5	110930	Fixed Height Rigid Backrest 14.5", 15" Wide with 3" Connector and Hardware <i>Used with 16" back height with omit push handles or 17" back height with standard or low profile push handles</i>	1
1bg, 3a, 4, 5	110931	Fixed Height Rigid Backrest 15.5", 15" Wide with 0" Connector and Hardware <i>Used with 17" back height with omit push handles or 18" back height with standard or low profile push handles</i>	1
1bg, 3b, 4, 5	110932	Fixed Height Rigid Backrest 15.5", 15" Wide with 1" Connector and Hardware <i>Used with 17" back height with omit push handles or 18" back height with standard or low profile push handles</i>	1
1bg, 3c, 4, 5	110933	Fixed Height Rigid Backrest 15.5", 15" Wide with 2" Connector and Hardware <i>Used with 17" back height with omit push handles or 18" back height with standard or low profile push handles</i>	1
1bg, 3d, 4, 5	110934	Fixed Height Rigid Backrest 15.5", 15" Wide with 3" Connector and Hardware	1

Item Number	Part Number	Description	Quantity
		<i>Used with 17" back height with omit push handles or 18" back height with standard or low profile push handles</i>	
1bh, 3a, 4, 5	110935	Fixed Height Rigid Backrest 16.5", 15" Wide with 0" Connector and Hardware <i>Used with 18" back height with omit push handles</i>	1
1bh, 3b, 4, 5	110936	Fixed Height Rigid Backrest 16.5", 15" Wide with 1" Connector and Hardware <i>Used with 18" back height with omit push handles</i>	1
1bh, 3c, 4, 5	110937	Fixed Height Rigid Backrest 16.5", 15" Wide with 2" Connector and Hardware <i>Used with 18" back height with omit push handles</i>	1
1bh, 3d, 4, 5	110938	Fixed Height Rigid Backrest 16.5", 15" Wide with 3" Connector and Hardware <i>Used with 18" back height with omit push handles</i>	1
1bi, 3a, 4, 5	110939	Fixed Height Rigid Backrest 7.5", 16" Wide with 0" Connector and Hardware <i>Used with 9" back height with omit push handles, 13" back height with fold down push handles or 10" back height with standard or low profile push handles</i>	1
1bi, 3b, 4, 5	110940	Fixed Height Rigid Backrest 7.5", 16" Wide with 1" Connector and Hardware <i>Used with 9" back height with omit push handles, 13" back height with fold down push handles or 10" back height with standard or low profile push handles</i>	1
1bi, 3c, 4, 5	110941	Fixed Height Rigid Backrest 7.5", 16" Wide with 2" Connector and Hardware <i>Used with 9" back height with omit push handles, 13" back height with fold down push handles or 10" back height with standard or low profile push handles</i>	1
1bi, 3d, 4, 5	110942	Fixed Height Rigid Backrest 7.5", 16" Wide with 3" Connector and Hardware <i>Used with 9" back height with omit push handles, 13" back height with fold down push handles or 10" back height with standard or low profile push handles</i>	1
1bj, 3a, 4, 5	110943	Fixed Height Rigid Backrest 8.5", 16" Wide with 0" Connector and Hardware <i>Used with 10" back height with omit push handles, 14" back height with fold down push handles or 11" back height with standard or low profile push handles</i>	1
1bj, 3b, 4, 5	110944	Fixed Height Rigid Backrest 8.5", 16" Wide with 1" Connector and Hardware <i>Used with 10" back height with omit push handles, 14" back height with fold down push handles or 11" back height with standard or low profile push handles</i>	1
1bj, 3c, 4, 5	110945	Fixed Height Rigid Backrest 8.5", 16" Wide with 2" Connector and Hardware <i>Used with 10" back height with omit push handles, 14" back height with fold down push handles or 11" back height with standard or low profile push handles</i>	1
1bj, 3d, 4, 5	110946	Fixed Height Rigid Backrest 8.5", 16" Wide with 3" Connector and Hardware <i>Used with 10" back height with omit push handles, 14" back height with fold down push handles or 11" back height with standard or low profile push handles</i>	1
1bk, 3a, 4, 5	110947	Fixed Height Rigid Backrest 9.5", 16" Wide with 0" Connector and Hardware <i>Used with 11" back height with omit push handles, 15" back height with fold down push handles or 12" back height with standard or low profile push handles</i>	1

Item Number	Part Number	Description	Quantity
1bk, 3b, 4, 5	110948	Fixed Height Rigid Backrest 9.5", 16" Wide with 1" Connector and Hardware <i>Used with 11" back height with omit push handles, 15" back height with fold down push handles or 12" back height with standard or low profile push handles</i>	1
1bk, 3c, 4, 5	110949	Fixed Height Rigid Backrest 9.5", 16" Wide with 2" Connector and Hardware <i>Used with 11" back height with omit push handles, 15" back height with fold down push handles or 12" back height with standard or low profile push handles</i>	1
1bk, 3d, 4, 5	110950	Fixed Height Rigid Backrest 9.5", 16" Wide with 3" Connector and Hardware <i>Used with 11" back height with omit push handles, 15" back height with fold down push handles or 12" back height with standard or low profile push handles</i>	1
1bl, 3a, 4, 5	110951	Fixed Height Rigid Backrest 10.5", 16" Wide with 0" Connector and Hardware <i>Used with 12" back height with omit push handles, 16" back height with fold down push handles or 13" back height with standard or low profile push handles</i>	1
1bl, 3b, 4, 5	110952	Fixed Height Rigid Backrest 10.5", 16" Wide with 1" Connector and Hardware <i>Used with 12" back height with omit push handles, 16" back height with fold down push handles or 13" back height with standard or low profile push handles</i>	1
1bl, 3c, 4, 5	110953	Fixed Height Rigid Backrest 10.5", 16" Wide with 2" Connector and Hardware <i>Used with 12" back height with omit push handles, 16" back height with fold down push handles or 13" back height with standard or low profile push handles</i>	1
1bl, 3d, 4, 5	110954	Fixed Height Rigid Backrest 10.5", 16" Wide with 3" Connector and Hardware <i>Used with 12" back height with omit push handles, 16" back height with fold down push handles or 13" back height with standard or low profile push handles</i>	1
1bm, 3a, 4, 5	110955	Fixed Height Rigid Backrest 11.5", 16" Wide with 0" Connector and Hardware <i>Used with 13" back height with omit push handles, 17" back height with fold down push handles or 14" back height with standard or low profile push handles</i>	1
1bm, 3b, 4, 5	110956	Fixed Height Rigid Backrest 11.5", 16" Wide with 1" Connector and Hardware <i>Used with 13" back height with omit push handles, 17" back height with fold down push handles or 14" back height with standard or low profile push handles</i>	1
1bm, 3c, 4, 5	110957	Fixed Height Rigid Backrest 11.5", 16" Wide with 2" Connector and Hardware <i>Used with 13" back height with omit push handles, 17" back height with fold down push handles or 14" back height with standard or low profile push handles</i>	1
1bm, 3d, 4, 5	110958	Fixed Height Rigid Backrest 11.5", 16" Wide with 3" Connector and Hardware <i>Used with 13" back height with omit push handles, 17" back height with fold down push handles or 14" back height with standard or low profile push handles</i>	1
1bn, 3a, 4, 5	110959	Fixed Height Rigid Backrest 12.5", 16" Wide with 0" Connector and Hardware <i>Used with 14" back height with omit push handles, 18" back height with fold down push handles or 15" back height with standard or low profile push handles</i>	1

Item Number	Part Number	Description	Quantity
1bn, 3b, 4, 5	110960	Fixed Height Rigid Backrest 12.5", 16" Wide with 1" Connector and Hardware <i>Used with 14" back height with omit push handles, 18" back height with fold down push handles or 15" back height with standard or low profile push handles</i>	1
1bn, 3c, 4, 5	110961	Fixed Height Rigid Backrest 12.5", 16" Wide with 2" Connector and Hardware <i>Used with 14" back height with omit push handles, 18" back height with fold down push handles or 15" back height with standard or low profile push handles</i>	1
1bn, 3d, 4, 5	110962	Fixed Height Rigid Backrest 12.5", 16" Wide with 3" Connector and Hardware <i>Used with 14" back height with omit push handles, 18" back height with fold down push handles or 15" back height with standard or low profile push handles</i>	1
1bo, 3a, 4, 5	110963	Fixed Height Rigid Backrest 13.5", 16" Wide with 0" Connector and Hardware <i>Used with 15" back height with omit push handles or 16" back height with standard or low profile push handles</i>	1
1bo, 3b, 4, 5	110964	Fixed Height Rigid Backrest 13.5", 16" Wide with 1" Connector and Hardware <i>Used with 15" back height with omit push handles or 16" back height with standard or low profile push handles</i>	1
1bo, 3c, 4, 5	110965	Fixed Height Rigid Backrest 13.5", 16" Wide with 2" Connector and Hardware <i>Used with 15" back height with omit push handles or 16" back height with standard or low profile push handles</i>	1
1bo, 3d, 4, 5	110966	Fixed Height Rigid Backrest 13.5", 16" Wide with 3" Connector and Hardware <i>Used with 15" back height with omit push handles or 16" back height with standard or low profile push handles</i>	1
1bp, 3a, 4, 5	110967	Fixed Height Rigid Backrest 14.5", 16" Wide with 0" Connector and Hardware <i>Used with 16" back height with omit push handles or 17" back height with standard or low profile push handles</i>	1
1bp, 3b, 4, 5	110968	Fixed Height Rigid Backrest 14.5", 16" Wide with 1" Connector and Hardware <i>Used with 16" back height with omit push handles or 17" back height with standard or low profile push handles</i>	1
1bp, 3c, 4, 5	110969	Fixed Height Rigid Backrest 14.5", 16" Wide with 2" Connector and Hardware <i>Used with 16" back height with omit push handles or 17" back height with standard or low profile push handles</i>	1
1bp, 3d, 4, 5	110970	Fixed Height Rigid Backrest 14.5", 16" Wide with 3" Connector and Hardware <i>Used with 16" back height with omit push handles or 17" back height with standard or low profile push handles</i>	1
1bq, 3a, 4, 5	110971	Fixed Height Rigid Backrest 15.5", 16" Wide with 0" Connector and Hardware <i>Used with 17" back height with omit push handles or 18" back height with standard or low profile push handles</i>	1
1bq, 3b, 4, 5	110972	Fixed Height Rigid Backrest 15.5", 16" Wide with 1" Connector and Hardware <i>Used with 17" back height with omit push handles or 18" back height with standard or low profile push handles</i>	1
1bq, 3c, 4, 5	110973	Fixed Height Rigid Backrest 15.5", 16" Wide with 2" Connector and Hardware <i>Used with 17" back height with omit push handles or 18" back height with standard or low profile push handles</i>	1

Item Number	Part Number	Description	Quantity
1bq, 3d, 4, 5	110974	Fixed Height Rigid Backrest 15.5", 16" Wide with 3" Connector and Hardware <i>Used with 17" back height with omit push handles or 18" back height with standard or low profile push handles</i>	1
1br, 3a, 4, 5	110975	Fixed Height Rigid Backrest 16.5", 16" Wide with 0" Connector and Hardware <i>Used with 18" back height with omit push handles</i>	1
1br, 3b, 4, 5	110976	Fixed Height Rigid Backrest 16.5", 16" Wide with 1" Connector and Hardware <i>Used with 18" back height with omit push handles</i>	1
1br, 3c, 4, 5	110977	Fixed Height Rigid Backrest 16.5", 16" Wide with 2" Connector and Hardware <i>Used with 18" back height with omit push handles</i>	1
1br, 3d, 4, 5	110978	Fixed Height Rigid Backrest 16.5", 16" Wide with 3" Connector and Hardware <i>Used with 18" back height with omit push handles</i>	1
2	000034	Sleeve 3/4" x 1 1/2" <i>Used with standard push handles, low profile push handles or fold down push handles</i>	2
3a	000589	Backrest Connector 0" LW/RG	1
3b	000590	Backrest Connector 1" LW/RG	1
3c	000591	Backrest Connector 2" LW/RG	1
3d	000592	Backrest Connector 3" LW/RG	1
4	101184	M6 x 10 BTN SHCS BLKZC w/Patch	4
5	000669	M6 x 20.5 Threaded Barrel	2
6	100804	Tube End Plug 1" <i>Used with omit push handles</i>	2
3b, 4, 5	502318	Backrest Connector Assembly 1" w/ Hardware, Rigid	1
3c, 4, 5	502319	Backrest Connector Assembly 2" w/ Hardware, Rigid	1
3d, 4, 5	502320	Backrest Connector Assembly 3" w/ Hardware, Rigid	1