


# Rogue ALX Fixed Height Backrest with Adjustable Height Rigidizer Bar (Formerly Tsunami)



Backrest 10"	10.3"
Backrest 11"	11.3"
Backrest 12"	12.3"
Backrest 13"	13.3"
Backrest 14"	14.3"
Backrest 15"	15.3"
Backrest 16"	16.3"
Backrest 17"	17.3"
Backrest 18"	18.3"
Backrest 19"	19.3"
Backrest 20"	20.3"
Backrest 21"	21.3"
Backrest 22"	22.3"

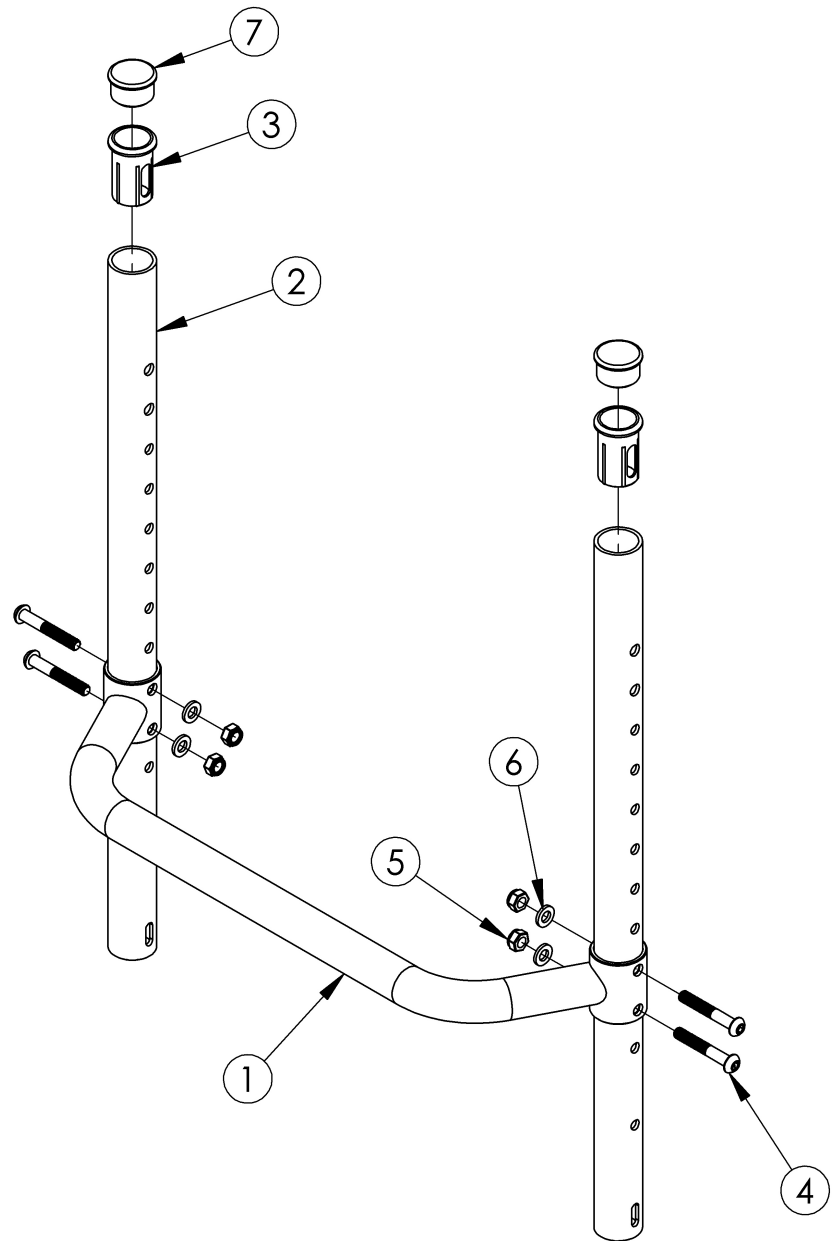

Item Number	Part Number	Description	Quantity
1a	000525	Adjustable Height Rigidizer Bar 10"	1
1b	000526	Adjustable Height Rigidizer Bar 11"	1
1c	000527	Adjustable Height Rigidizer Bar 12" Wide	1
1d	000528	Adjustable Height Rigidizer Bar 13" Wide	1

Item Number	Part Number	Description	Quantity
1e	000185	Adjustable Height Rigidizer Bar 14" Wide	1
1f	000186	Adjustable Height Rigidizer Bar 15" Wide	1
1g	000187	Adjustable Height Rigidizer Bar 16" Wide	1
1h	000188	Adjustable Height Rigidizer Bar 17" Wide	1
1i	000189	Adjustable Height Rigidizer Bar 18" Wide	1
1j	000190	Adjustable Height Rigidizer Bar 19" Wide	1
1k	000191	Adjustable Height Rigidizer Bar 20" Wide	1
1l	000529	Adjustable Height Rigidizer Bar 21"	1
1m	000530	Adjustable Height Rigidizer Bar 22"	1
2a	109096	Backrest Tube Fixed Height, 6.25" <i>Used with 12" back height with fold down push handles or 9" back height with standard or low profile push handles</i>	2
2b	109095	Backrest Tube Fixed Height, 7.25" <i>Used with 13" back height with fold down push handles or 10" back height with standard or low profile push handles</i>	2
2c	109094	Backrest Tube Fixed Height, 8.25" <i>Used with 9" back height with omit push handles, 14" back height with fold down push handles or 11" back height with standard or low profile push handles</i>	2
2d	109093	Backrest Tube Fixed Height, 9.25" <i>Used with 10" back height with omit push handles, 15" back height with fold down push handles or 12" back height with standard or low profile push handles</i>	2
2e	109092	Backrest Tube Fixed Height, 10.25" <i>Used with 11" back height with omit push handles, 16" back height with fold down push handles or 13" back height with standard or low profile push handles</i>	2
2f	109091	Backrest Tube Fixed Height, 11.25" <i>Used with 12" back height with omit push handles, 17" back height with fold down push handles or 14" back height with standard or low profile push handles</i>	2
2g	109090	Backrest Tube Fixed Height, 12.25" <i>Used with 13" back height with omit push handles, 18" back height with fold down push handles or 15" back height with standard or low profile push handles</i>	2
2h	109089	Backrest Tube Fixed Height, 13.25" <i>Used with 14" back height with omit push handles or 16" back height with standard or low profile push handles</i>	2
2i	109088	Backrest Tube Fixed Height, 14.25" <i>Used with 15" back height with omit push handles or 17" back height with standard or low profile push handles</i>	2

Item Number	Part Number	Description	Quantity
2j	109087	Backrest Tube Fixed Height, 15.25" <i>Used with 16" back height with omit push handles or 18" back height with standard or low profile push handles</i>	2
2k	109086	Backrest Tube Fixed Height, 16.25" <i>Used with 17" back height with omit push handles</i>	2
2l	000698	Backrest Tube Fixed Height, 17.25" <i>Used with 18" back height with omit push handles</i>	2
3	000034	Sleeve 3/4" x 1 1/2" <i>Used with standard push handles, low profile push handles or fold down push handles</i>	2
4	100797	M6 x 40 BHCS BLK ZC	4
5	100658	M6 Nylock Nut Blk Zn	4
6	100746	M6 Flat Washer Blk Zn	4
7	100804	Tube End Plug 1" <i>Used with omit push handles</i>	2