


Rogue2 Side Guards - Aluminum Straight

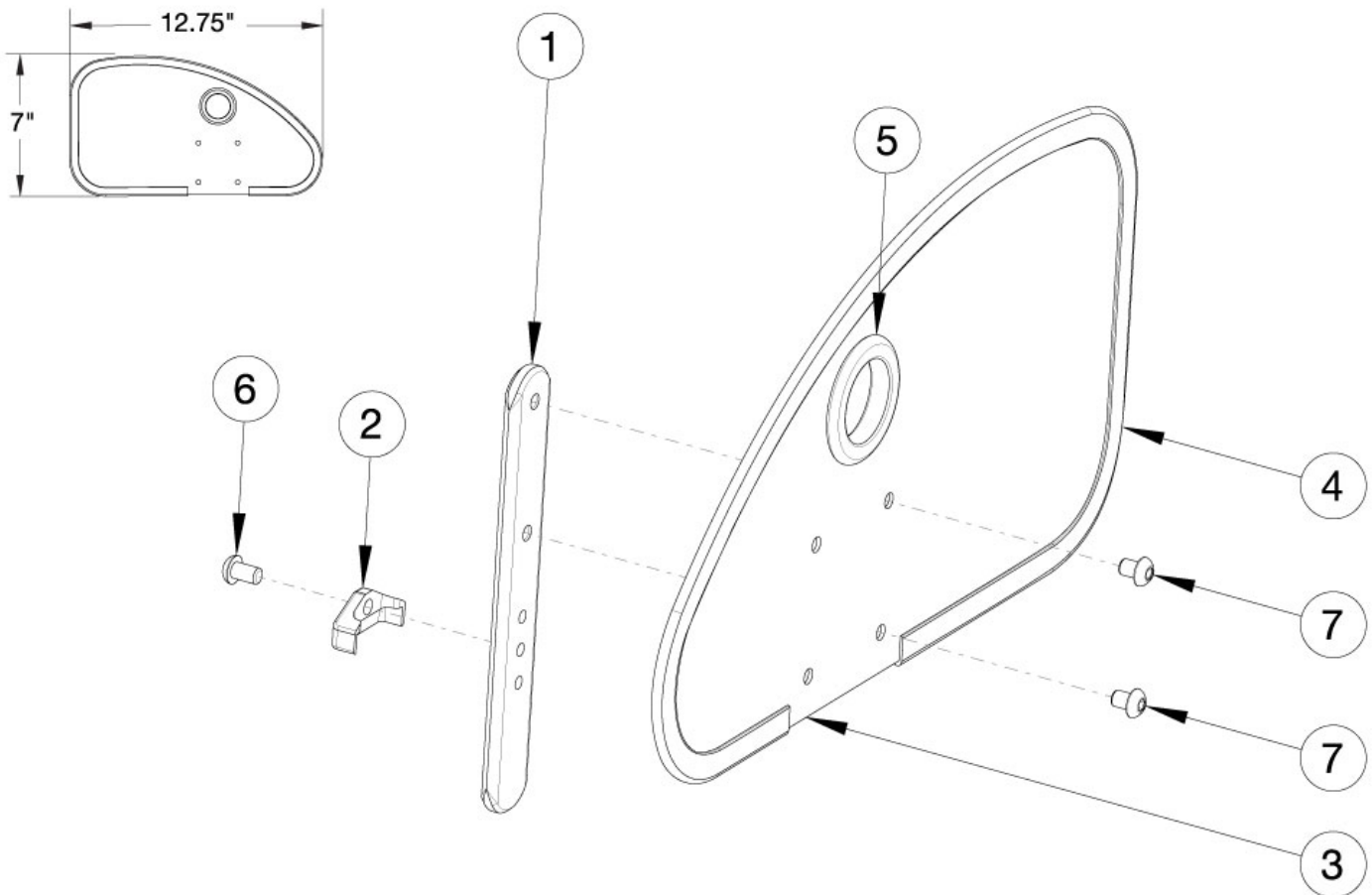

If retrofitting, side guard receivers may be required.

Item Number	Part Number	Description	Quantity
1, 2, 3a, 4-7	108346	Aluminum Straight Side Guard Assembly Left, Black Matte	1
1, 2, 3a, 4-7	108347	Aluminum Straight Side Guard Assembly Right, Black Matte	1
1, 2, 3a, 4-7	108348	Aluminum Straight Side Guard Assembly (PAIR), Black Matte	1
1, 2, 3b, 4-7	108349_ _ _	Aluminum Straight Side Guard Assembly Left, Match Frame Color	1
1, 2, 3b, 4-7	108350_ _ _	Aluminum Straight Side Guard Assembly Right, Match Frame Color	1
1, 2, 3b, 4-7	108351_ _ _	Aluminum Straight Side Guard Assembly (PAIR), Match Frame Color	1
1	000140	Side Guard Post	1
2	001342	Side Guard Post Stop	1
3a, 4, 5	108342	Aluminum Straight Side Guard Left, Black Matte	1

Item Number	Part Number	Description	Quantity
3a, 4, 5	108343	Aluminum Straight Side Guard Right, Black Matte	1
3b, 4, 5	108344_ _ _	Aluminum Straight Side Guard Left, Match Frame Color	1
3b, 4, 5	108345_ _ _	Aluminum Straight Side Guard Right, Match Frame Color	1
6	101184	M6 x 10 BTN SHCS BLKZC w/Patch	1
7	101700	M6 x 8 BTN SHCS BLK ZC	2

Matte Finish	
Code	Color Name
CGM	Charcoal Gray
EGM	Emerald Green
GGM	Graphite Gray*
IBM	Iridescent Burgundy
INM	Iridescent Navy
AGM	Military Green**
RRM	Retro Red
RBM	Royal Blue
SBM	Shadow Black
SWM	Smooth White

* Discontinued Color

** Formerly called Army Green

Gloss Finish	
Code	Color Name
BLK	Black
BLU	Blue
SKY	Blue Skies
BRO	Bright Orange*
PNK	Bubblegum Pink
BUR	Burgundy*
CBL	Candy Blue
PUR	Candy Purple
RED	Candy Red
COP	Copper
DPR	Deep Purple*
EBL	Electric Blue*
GRN	Forest Green*
FUS	Fuchsia*
GSM	Granny Smith
MTF	Metallic Fuchsia*
MOR	Metallic Orange
ORG	Orange*
BRD	Red
SAP	Sapphire*
SIL	Silver
TEA	Teal
YLW	Yellow
FLM	Graphic Image Transfer