


## Flip for x:panda Anti-Tips

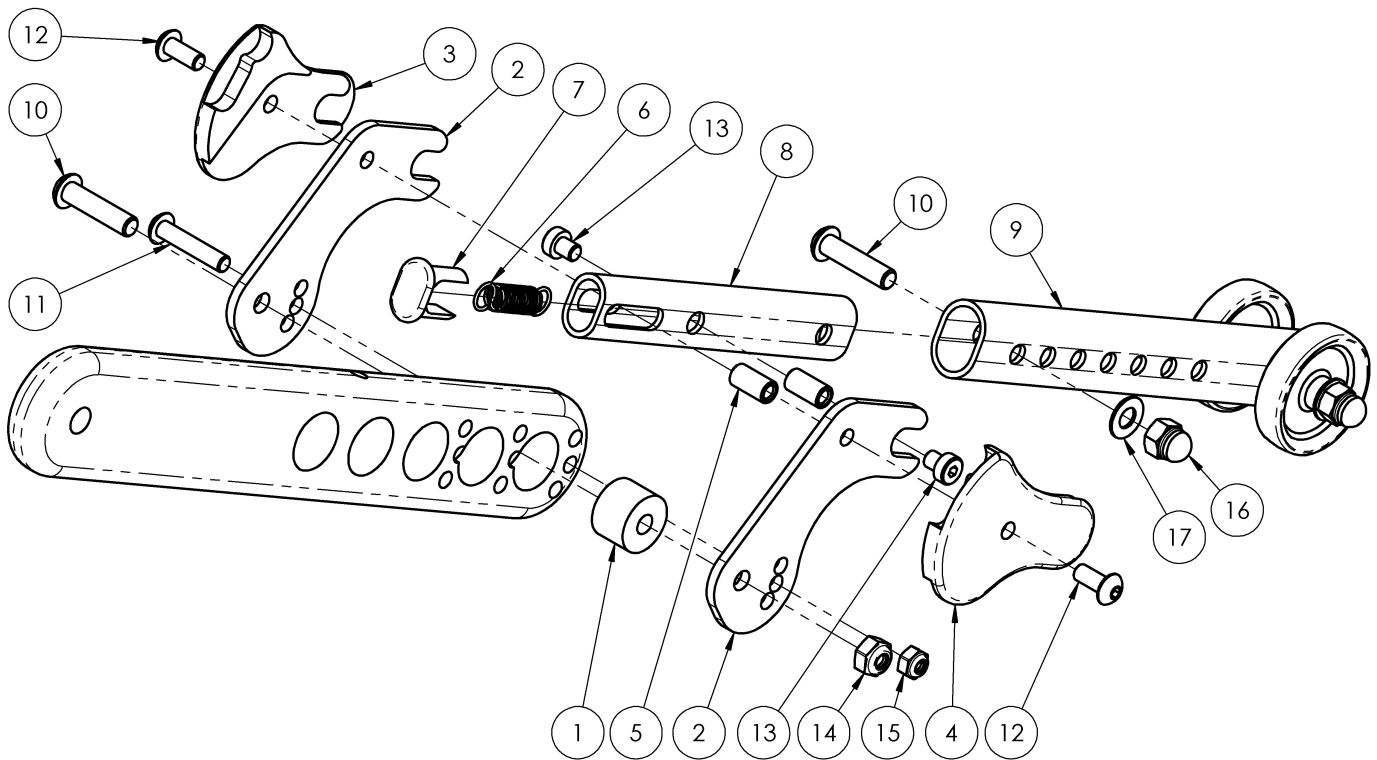

Item Number	Part Number	Description	Quantity
1-7, 8a, 9a, 10-17	505580	Anti-Tip Kit, 12" FLIP EACH	1
1-7, 8a, 9a, 10-17	502886	Anti-Tip Kit, 12" FLIP PAIR	1
1-7, 8a, 9b, 10-17	505581	Anti-Tip Kit, 16" FLIP EACH	1
1-7, 8a, 9b, 10-17	502887	Anti-Tip Kit, 16" FLIP PAIR	1
1	001906	Bushing, Anti-tip	1
2	001907	Anti-Tip Bracket	2
3	001908	Cover, Anti-Tip, R	1
4	001909	Cover, Anti-Tip, L	1
5	001910	Spring Anchor, Index, Anti-Tip	2
6	001911	Spring, Ext, .31OD x 1.16LG x .038W	1
7	001912	Cap, Tube, Anti-Tip	1
8	001913	Tube, Anti-Tip, Short	1

Item Number	Part Number	Description	Quantity
9a	001917	Extension Assembly, Anti-Tip, Short	1
9b	001918	Extension Assembly, Anti-Tip, Long	1
10	101593	M6 x 25 BTN SHCS BLKZC	2
11	102616	M5 X 25 BHCS BLK ZC	1
12	102799	M5 x 12, BHCS, BZ w/Patch	2
13	102796	M5 x 6, Low SHCS, Blk Zn	2
14	100658	M6 Nylock Nut Blk Zn	1
15	100657	M5 Nylock Nut Blk Zn	1
16	100558	M6 NYL INS Domed Cap L/N ZC	1
17	102803	Washer Flat .250 x .500 x .027 HX BLZ	1