


# Rogue XP Plastic Coated Handrim

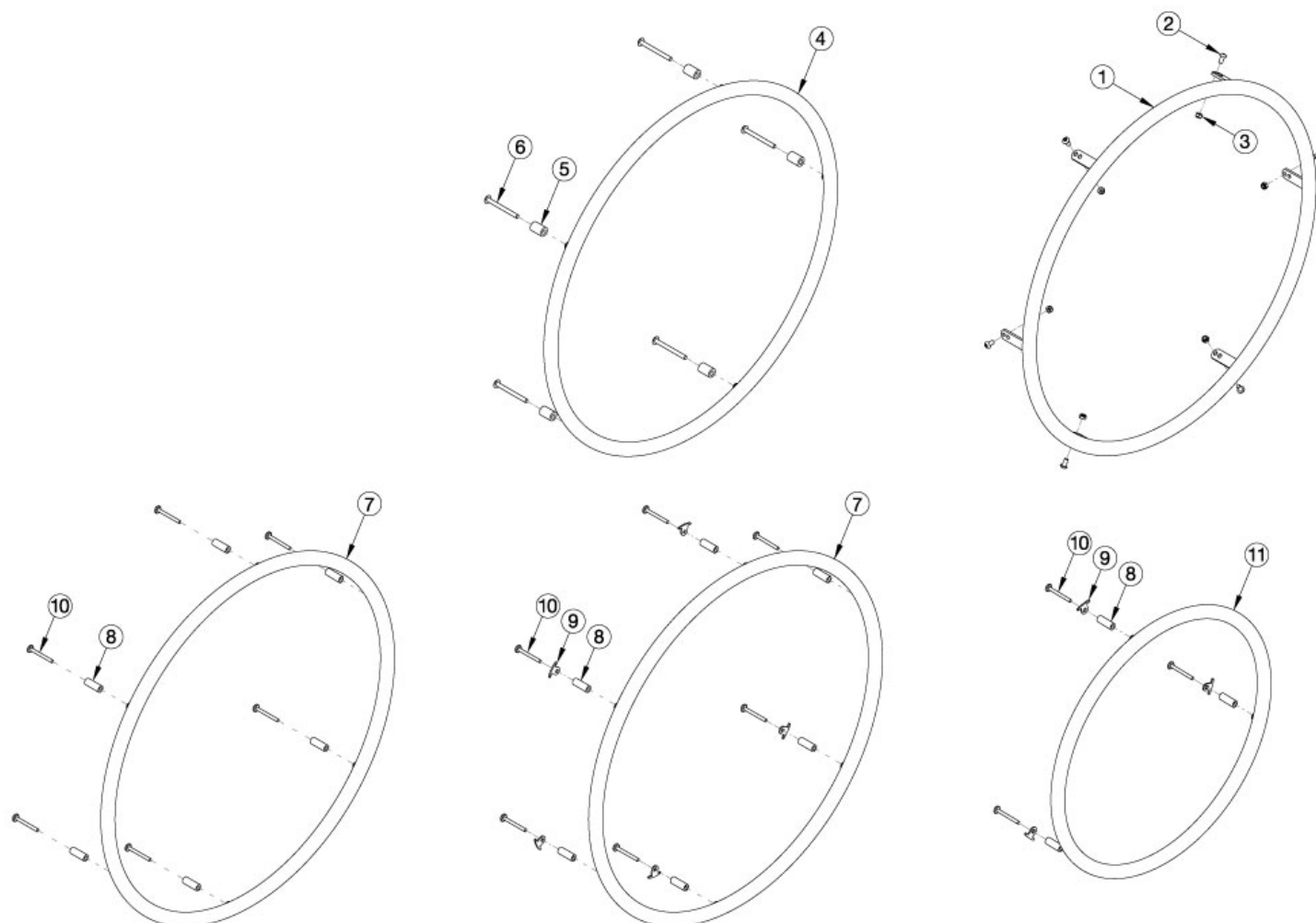

For more information see Handrim Hardware Chart. Quantity in Handrim Hardware Kits are for one wheel. Tab mounted handrims must be installed prior to the tire being installed.

| Item Number | Part Number | Description                                                                                                                                           | Quantity |
|-------------|-------------|-------------------------------------------------------------------------------------------------------------------------------------------------------|----------|
| 8a, 9, 10a  | 505263      | Handrim Hardware Kit, 3 Rivet Nut, EACH<br><i>Used with 18" Ki Spoke and 18" MAXX Spoke wheels with standard handrim mount option</i>                 | 1        |
| 8b, 9, 10b  | 116742      | Handrim Hardware Kit, Close Mount, 3 Rivet Nut, EACH<br><i>Used with 18" Ki Spoke and 18" MAXX Spoke wheels with close mount handrim mount option</i> | 1        |
| 8a, 9, 10a  | 502168      | Handrim Hardware Kit, 6 Rivet Nut, EACH<br><i>Used with Ki Spoke and MAXX Spoke wheels with standard handrim mount option</i>                         | 1        |
| 8b, 9, 10b  | 113069      | Handrim Hardware Kit, Close Mount, 6 Rivet Nut, EACH<br><i>Used with Ki Spoke and MAXX Spoke wheels with close mount handrim mount option</i>         | 1        |

| Item Number | Part Number | Description                                                                                                                                                       | Quantity |
|-------------|-------------|-------------------------------------------------------------------------------------------------------------------------------------------------------------------|----------|
| 5c, 10c     | 117377      | Handrim Hardware Kit, Standard Mount, MAXX Mag, EACH<br><i>Used with MAXX Mag wheels with standard handrim mount option</i>                                       | 1        |
| 5b, 10a     | 117378      | Handrim Hardware Kit, Close Mount, MAXX Mag, EACH<br><i>Used with MAXX Mag wheels with close mount handrim mount option</i>                                       | 1        |
| 7a          | 112819      | Handrim Plastic Coated 6 Rivet Nut 20"<br><i>Used on 20" Ki Spoke, 20" MAXX Spoke, and 20" MAXX Mag wheels. Replaced 200542.</i>                                  | 1        |
| 7b          | 112820      | Handrim Plastic Coated 6 Rivet Nut 22"<br><i>Used on 22" Ki Spoke, 22" MAXX Spoke, and 22" MAXX Mag wheels. Replaced part 100576.</i>                             | 1        |
| 7c          | 112821      | Handrim Plastic Coated 6 Rivet Nut 24"<br><i>Used on 24" Ki Spoke, 24" MAXX Spoke, and 24" MAXX Mag wheels. Replaced 100577.</i>                                  | 1        |
| 7d          | 112822      | Handrim Plastic Coated 6 Rivet Nut 25"<br><i>Used on 25" Ki Spoke and 25" MAXX Spoke wheels. Replaces Part Number 101870.</i>                                     | 1        |
| 5b, 6b      | 113070      | Handrim Hardware Kit, Close Mount, 5 Rivet Nut, EACH<br><i>Used with X Core Mag wheels with close mount handrim mount option</i>                                  | 1        |
| 4a          | 101963      | Handrim Plastic Coated 5 Rivet Nut 20"<br><i>Used on 20" 5-Spoke X Core Mag wheels</i>                                                                            | 1        |
| 2a, 3       | 502174      | Handrim Hardware Kit, 6 Tab, EACH<br><i>Used with Spinergy and Topolino wheels</i>                                                                                | 1        |
| 2b          | 502170      | Handrim Hardware Kit, Superlight, EACH<br><i>Used with Superlight Spoke wheels</i>                                                                                | 1        |
| 1a          | 112812      | Handrim Plastic Coated 6 Tab 24"<br><i>Used on 24" Superlight/Topolino wheels. Replaced 100836.</i>                                                               | 1        |
| 1b          | 112813      | Handrim Plastic Coated 6 Tab 25"<br><i>Used on 25" Spoke/Superlight/Topolino wheels</i>                                                                           | 1        |
| 1c          | 113969      | Handrim Plastic Coated Spinergy 6 Tab 20"<br><i>Replaced part number 103179. Used with Spinergy Spox 20" wheel</i>                                                | 1        |
| 1d          | 112814      | Handrim Plastic Coated Spinergy 6 Tab 22"<br><i>Replaced 100808</i>                                                                                               | 1        |
| 1e          | 112815      | Handrim Plastic Coated Spinergy 6 Tab 24"<br><i>If chair is invoiced prior to 8/14/19, quantity 2 must be ordered due to a change in finish. Replaced 100615.</i> | 1        |
| 1f          | 112816      | Handrim Plastic Coated Spinergy 6 Tab 25"<br><i>Replaced 100765</i>                                                                                               | 1        |

| Item Number | Part Number | Description                                                                                                                            | Quantity |
|-------------|-------------|----------------------------------------------------------------------------------------------------------------------------------------|----------|
| 2a          | 100669      | M5 x 10 BTN SHCS BLKZC w/Patch<br><i>Used with Spinergy and Topolino wheels</i>                                                        | 6        |
| 2b          | 100536      | M5 x 12 BTN SHCS ZC<br><i>Used with Superlight Spoke wheels</i>                                                                        | 6        |
| 3           | 100657      | M5 Nylock Nut Blk Zn<br><i>Used with Spinergy and Topolino wheels</i>                                                                  | 6        |
| 5a          | 101756      | Handrim Spacer Plastic Coated Rivet<br><i>Used with X Core Mag with standard handrim mount option</i>                                  | 5        |
| 5c          | 100629      | Handrim Spacer Alum Ano Riv<br><i>Used with MAXX Mag wheels with standard handrim mount option</i>                                     | 6        |
| 6a          | 100724      | M5 x 55 PH TRS HD M/S ZC<br><i>Used with X Core Mag with standard handrim mount option</i>                                             | 5        |
| 8a          | 100653      | Handrim Spacer Spoke<br><i>Used with Ki Spoke and MAXX Spoke with standard handrim mount option</i>                                    | 6        |
| 9           | 100698      | Handrim Tab<br><i>Used with Ki Spoke and MAXX Spoke wheels. Tabs are included / installed with MAXX Spoke wheels.</i>                  | 6        |
| 10a         | 122063      | PHMS, M5x0.8x40, CLS4.8, PHL, BLZ, NYL, DIN7985, Custom<br><i>Used with Ki Spoke and MAXX Spoke with standard handrim mount option</i> | 6        |
| 10c         | 122064      | PHMS, M5x0.8x50, CLS4.8, PHL, BLZ, NYL, DIN7985, Custom<br><i>Used with MAXX Mag wheels with standard handrim mount option</i>         | 6        |

| Handrim Hardware Chart |                |                    |                                                              |                                              |                                                                  |                   |                |                  |                  |                    |                  |                      |                  |          |  |
|------------------------|----------------|--------------------|--------------------------------------------------------------|----------------------------------------------|------------------------------------------------------------------|-------------------|----------------|------------------|------------------|--------------------|------------------|----------------------|------------------|----------|--|
| Wheel                  |                | Handrim Connection |                                                              |                                              |                                                                  | Handrim           |                |                  |                  |                    |                  |                      |                  |          |  |
| Wheel                  | Wheel Part No. | Connection Points  | Rivet/Tab                                                    | Spacer*                                      | Screw*                                                           | Aluminum Anodized | Superlight     | Plastic Coated   | Projection       | Ergonomic Standard | Ergonomic LT     | Natural Fit Standard | Natural Fit LT   | Flex Rim |  |
| 22" Spoke              | 116730         | 5 or 6             | 116732 (Used with 5 Rivet)<br><br>100698 (Used with 6 Rivet) | 100653 (Not used w/ Superlight Handrim)      | Aluminum, Plastic Ctd, Projection<br>Ergonomic: 122063           | 101898 (5 Rivet)  |                | 101964 (5 Rivet) | 112824 (6 Rivet) | 113082 (6 Rivet)   | 113085 (6 Rivet) | 200538 (6 Rivet)     | 200201 (6 Rivet) |          |  |
| 24" Spoke              | 116728         | 5 or 6             | Not used w/ Superlight Handrims                              |                                              | Natural Fit: 100835<br><br>Superlight: 100669<br><br>Nut: 100657 | 100975 (5 Rivet)  | 101161 (6 Tab) | 100976 (5 Rivet) | 112825 (6 Rivet) | 113083 (6 Rivet)   | 113086 (6 Rivet) | 100793 (6 Rivet)     | 200202 (6 Rivet) |          |  |
| 18" MAXX Spoke         | 109285         | 3                  | 100698 (Not used w/ Superlight Handrim)                      | 100653 (Not used w/ Superlight Handrim)      | Aluminum, Plastic Ctd, Projection, Ergonomic: 122063             | 100206            |                |                  |                  |                    |                  |                      |                  |          |  |
| 20" MAXX Spoke         | 108244         | 6                  |                                                              |                                              | Natural Fit: 100835                                              | 200536            |                | 112819           |                  |                    |                  |                      |                  |          |  |
| 22" MAXX Spoke         | 105135         |                    |                                                              |                                              | Superlight: 100669                                               | 100560            |                | 112820           | 112824           | 113082             | 113085           | 200538               | 200201           |          |  |
| 24" MAXX Spoke         | 105136         |                    |                                                              |                                              | Nut: 100657                                                      | 200349            | 101161         | 112821           | 112825           | 113083             | 113086           | 100793               | 200202           |          |  |
| 25" MAXX Spoke         | 107436         |                    |                                                              |                                              | 200350                                                           |                   | 112822         | 200548           | 113084           | 113087             | 200539           | 200540               |                  |          |  |
| 26" MAXX Spoke         | 107437         |                    |                                                              |                                              | 200351                                                           |                   | 112823         | 200549           |                  |                    | 100907           | 101454               |                  |          |  |
| 20" MAXX Mag           | 111853         | 6                  |                                                              | Aluminum, Plastic Coated, Projection: 100629 | Aluminum, Plastic Coated, Projection, Ergonomic: 122064          | 200536            |                | 112819           |                  |                    |                  |                      |                  |          |  |
| 22" MAXX Mag           | 111854         |                    |                                                              | Natural Fit & Ergonomic: 101756              | Natural Fit: 101893                                              | 100560            |                | 112820           | 112824           | 113082             | 113085           | 200538               | 200201           |          |  |
| 24" MAXX Mag           | 111855         |                    |                                                              |                                              |                                                                  | 105137            |                | 112821           | 112825           | 113083             | 113086           | 100793               | 200202           |          |  |
| 24" Superlight         | 101159         | 6                  |                                                              |                                              | 100536                                                           | 100754            | 101161         | 100836           |                  | 113080             | 113081           | 100830               | 100828           |          |  |
| 25" Superlight         | 101160         |                    |                                                              |                                              |                                                                  |                   |                | 101160           | 101091           |                    |                  |                      | 101464           | 101460   |  |

\* Spacer and and screw part numbers listed in the chart are for standard handrim mounting. If using close mount handrim mounting on aluminum anodized, plastic coated or projection handrims, use part number 100792 for the spacer, part number 122065 for the screw on Ki Spoke and MAXX Spoke wheels, part number 100666 for the screw on 5-Spoke X Core wheels, or part number 122063 for the screw on MAXX Mag wheels.

Effective 4/19/2024

| Handrim Hardware Chart (Continued) |                         |                    |           |         |                    |                   |            |                |            |                    |              |                      |                |                       |
|------------------------------------|-------------------------|--------------------|-----------|---------|--------------------|-------------------|------------|----------------|------------|--------------------|--------------|----------------------|----------------|-----------------------|
| Wheel                              |                         | Handrim Connection |           |         |                    | Handrim           |            |                |            |                    |              |                      |                |                       |
| Wheel                              | Wheel Part No.          | Connection Points  | Rivet/Tab | Spacer* | Screw*             | Aluminum Anodized | Superlight | Plastic Coated | Projection | Ergonomic Standard | Ergonomic LT | Natural Fit Standard | Natural Fit LT | Flex Rim              |
| 20" Spinerger Spox                 | See Spinerger Spox Page | 6                  |           |         | Ergonomic: 100536  | 103125            |            | 103179         |            |                    |              |                      |                |                       |
| 22" Spinerger Spox                 |                         |                    |           |         | All Others: 100669 | 100827            |            | 100808         |            |                    |              | 100889               | 100888         |                       |
| 24" Spinerger Spox                 |                         |                    |           |         |                    | 100766            | 101161     | 100615         |            |                    |              | 100830               | 100828         |                       |
| 25" Spinerger Spox                 |                         |                    |           |         | Nut: 100657        | 100767            | 101160     | 100765         |            |                    |              | 101464               | 101460         |                       |
| 26" Spinerger Spox                 |                         |                    |           |         |                    | 101477            |            | 101148         |            |                    |              | 200200               | 100950         |                       |
| 22" Spinerger LX                   | See Spinerger LX Page   | 6                  |           |         | Ergonomic: 100536  | 100827            |            | 100808         |            |                    |              | 100889               | 100888         |                       |
| 24" Spinerger LX                   |                         |                    |           |         | All Others: 100669 | 100766            | 101161     | 100615         |            | 113080             | 113081       | 100830               | 100828         | See Spinerger LX Page |
| 25" Spinerger LX                   |                         |                    |           |         |                    | 100767            | 101160     | 100765         |            |                    |              | 101464               | 101460         |                       |
| 26" Spinerger LX                   |                         |                    |           |         | Nut: 100657        | 101477            |            | 101148         |            |                    |              | 200200               | 100950         |                       |

| Handrim Hardware Chart (Discontinued Wheels) |                |                    |           |                                           |                                           |                   |            |                |            |                      |                |          |
|----------------------------------------------|----------------|--------------------|-----------|-------------------------------------------|-------------------------------------------|-------------------|------------|----------------|------------|----------------------|----------------|----------|
| Wheel                                        |                | Handrim Connection |           |                                           |                                           | Handrim           |            |                |            |                      |                |          |
| Wheel                                        | Wheel Part No. | Connection Points  | Rivet/Tab | Spacer*                                   | Screw*                                    | Aluminum Anodized | Superlight | Plastic Coated | Projection | Natural Fit Standard | Natural Fit LT | Flex Rim |
| 18" Ki Spoke                                 | 200529         | 3                  | 100698    | 100653                                    | Aluminum, Plastic Ctd, Projection: 122063 | 100206            |            | 101106         |            |                      |                |          |
| 20" Ki Spoke                                 | 200530         | 6                  |           |                                           |                                           | 200536            |            | 200542         |            |                      |                |          |
| 22" Ki Spoke                                 | 200531         |                    |           |                                           |                                           | 100560            |            | 100576         | 100569     | 200538               | 200201         |          |
| 24" Ki Spoke                                 | 200532         |                    |           |                                           |                                           | 200349            |            | 100577         | 200547     | 100793               | 200202         |          |
| 25" Ki Spoke                                 | 200533         |                    |           |                                           | Natural Fit: 100835                       | 200350            |            | 101870         | 200548     | 200539               | 200540         |          |
| 26" Ki Spoke                                 | 200534         |                    |           |                                           |                                           | 200351            |            | 100578         | 200549     | 100907               | 101454         |          |
| 20" 5-Spoke X Core                           | 101961         | 5                  |           | Aluminum, Projection, Natural Fit: 100629 | 100724                                    | 101897            |            | 101963         |            |                      |                |          |
| 22" 5-Spoke X Core                           | 101962         |                    |           |                                           |                                           | 101898            |            | 101964         |            |                      |                |          |
| 24" 5-Spoke X Core                           | 100960         |                    |           |                                           |                                           | 100975            |            | 100976         | 200546     |                      |                |          |
| 24" 5-Spoke X Core                           | 100960         |                    |           |                                           | Plastic Ctd: 101756                       | 101893            |            |                |            |                      | 100768         | 100769   |

\* Spacer and screw part numbers listed in the chart are for standard handrim mounting. If using close mount handrim mounting on aluminum anodized, plastic coated or projection handrims, use part number 100792 for the spacer, part number 122065 for the screw on Ki Spoke and MAXX Spoke wheels, part number 100666 for the screw on 5-Spoke X Core wheels, or part number 122063 for the screw on MAXX Mag wheels.

Effective 4/19/2024