


Ethos Swing Away Armrest

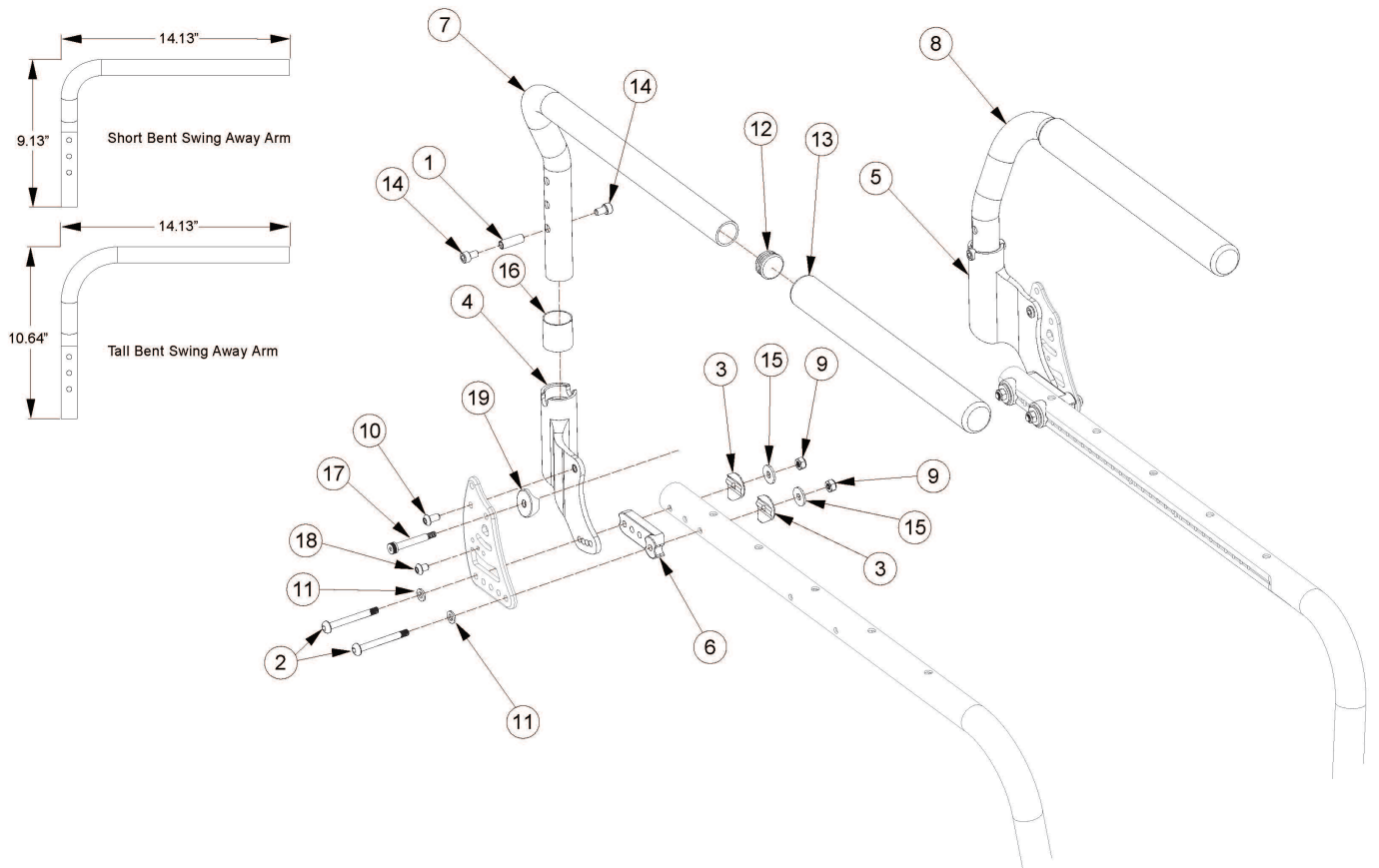

Item Number	Part Number	Description	Quantity	Retail
1-4, 6, 7a, 9-19	108172	Short Swing Away Arm with Receiver Assembly Right, Ethos	1	\$172.65
1-3, 5, 6, 8a, 9-19	108173	Short Swing Away Arm with Receiver Assembly Left, Ethos	1	\$172.65
1-6, 7a, 8a, 9-19	108174	Short Swing Away Arm with Receiver Assembly PAIR, Ethos	1	\$345.29
1-4, 6, 7b, 9-19	108175	Tall Swing Away Arm with Receiver Assembly Right, Ethos	1	\$173.11
1-3, 5, 6, 8b, 9-19	108176	Tall Swing Away Arm with Receiver Assembly Left, Ethos	1	\$173.11
1-6, 7b, 8b, 9-19	108177	Tall Swing Away Arm with Receiver Assembly PAIR, Ethos	1	\$346.23
2-4, 6, 9-11, 15-19	108164	Swing Away Arm Receiver Assembly Right, Ethos	1	\$124.93
2, 3, 5, 6, 9-11, 15-19	108165	Swing Away Arm Receiver Assembly Left, Ethos	1	\$124.93
1	000065	M6 x 23.68 Threaded Barrel	1	\$4.13
2	003232	M6 x 55 BHCS Blk Zc	2	\$8.94

Item Number	Part Number	Description	Quantity	Retail
3	003849	Saddle For Dimple 1 1/8"	2	\$0.44
4, 16	108162	Swing Away Arm Receiver with Nyliner, Right	1	\$97.14
5, 16	108163	Swing Away Arm Receiver with Nyliner, Left	1	\$97.14
6	004116	Saddle, Swing Away Arm, 1 1/8" Tube	1	\$1.13
7a, 12, 13	108166	Short Swing Away Arm Right, Ethos	1	\$39.38
8a, 12, 13	108167	Short Swing Away Arm Left, Ethos	1	\$39.38
7a, 8a, 12, 13	108168	Short Swing Away Arm PAIR, Ethos	1	\$78.76
7b, 12, 13	108169	Tall Swing Away Arm Right, Ethos	1	\$39.85
8b, 12, 13	108170	Tall Swing Away Arm Left, Ethos	1	\$39.85
7b, 8b, 12, 13	108171	Tall Swing Away Arm PAIR, Ethos	1	\$79.70
9	100658	M6 Nylock Nut Blk Zn	2	\$1.05
10	100667	M6 x 12 BTN SHCS BLKZC w/Patch	1	\$1.58
11	100746	M6 Flat Washer Blk Zn	2	\$1.58
12	100804	Tube End Plug 1"	1	\$1.58
14	110061	M6 x 10 Stainless Steel Armrest Screw with Patch	2	\$25.40
15	101853	M6 Washer Blk Zn	2	\$0.53
16	103517	Nyliner 1" ID x 1.094" OD x 1.125" Long	1	\$10.35
17	103397	M6 x 40 Shoulder Screw BLK ZC	1	\$2.55
18	102621	M5 x 8 BHCS BLK ZC	1	\$0.53
19	004117	Saddle M6 ID x 1 OD x .25 T	1	\$0.47